

# Spare Parts Catalogue

---

Version: 2017/08/07a EN



*Küschall® Compact*



**Yes, you can.®**

## **General Information**

*For any order of spare parts the following information is required:*

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

*Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.*

*The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.*



Yes, you can.<sup>®</sup>

## TABLE OF CONTENTS

---

### INFORMATION

Abbreviations .....	7
Frame colours .....	8
CPS colours .....	9

### CHASSIS

CHASSIS .....	10
Front frames for flow system .....	11
Front frames for bearing blocks .....	16
Cross brace to frame standard, SB32-40 .....	19
Cross brace to frame standard, SB42-50 .....	25
Cross brace to frame Hemi, SB32-40 .....	31
Cross brace to frame Hemi, SB42-50 .....	37
Cross brace to frame Hemi abduction, SB32-40 .....	43
Cross brace to frame Hemi abduction, SB42-50 .....	51
Rear frames .....	59
Rear wheel fixation .....	60

### SEAT

SEAT .....	61
Seat cover standard .....	62
Seat cover with 1 Velcro band (soft band) .....	63
Seat cover with 2 Velcro bands (rough band) .....	66
Seat cover standard for seat bag .....	69
Seat cover with 1 Velcro band (soft band) for seat bag .....	72
Seat cover with 2 Velcro bands (rough band) for seat bag .....	75
Seat cushion standard .....	78

### BACKREST

BACKREST .....	80
Push handles for backrest standard .....	81
Backrest tubes without push handles for standard backrest .....	82
Push handles for backrest angle adjustable .....	83
Backrest tubes without push handles for angle adjustable backrest .....	84
Push handles for backrest foldable .....	85
Push handles for backrest foldable + angle adjustable .....	86
Push handles height adjustable .....	87
Push handles height adjustable for backrest angle adjustable .....	88
Push handles height adjustable for backrest foldable .....	90
Push handles height adjustable for backrest foldable + angle adjustable .....	92
Push handles, inclined .....	93
Backrest stabilization bar .....	94
Back cover, standard .....	96
Bands to adjustable backrest .....	98
Cover to adjustable backrest .....	99

### LEGRESTS

LEGRESTS .....	101
Legrest hanger 70° .....	102



Yes, you can.®

## TABLE OF CONTENTS

---

Legrest hanger 80° .....	110
Legrest hanger height adjustable .....	118

### FOOTREST

FOOTREST .....	119
Footrest 1-piece .....	120
Footrest 1-piece, foldable .....	122
Footrest 2-piece .....	123
Footrest 2-piece, angle adjustable .....	126
Footrest high mounted .....	128
Footband .....	129

### SIDERESTS

SIDERESTS .....	130
Clothes-guards .....	131
Mudguards 01.11.2008=> .....	132
Siderest height adjustable 01.04.2008=> .....	133
Siderest fold back, height adjustable .....	134
Siderest with desk function .....	135
Tubular armrest .....	136

### FRONT WHEELS

FRONT WHEELS .....	137
Front forks for flow system .....	138
Front forks for bearing blocks .....	140
Front wheels .....	142

### REAR WHEELS

REAR WHEELS .....	144
22" Kuschall rear wheels complete .....	145
24" Kuschall rear wheels complete .....	147
26" Kuschall rear wheels complete .....	149
High Performance wheel .....	151
Mountain bike wheel .....	153
Spider wheel .....	154
Spinergy wheel .....	155
Drumbrake wheel .....	156
Handrim .....	157
Fixation handrim .....	158
Tyre .....	159
Inner tube and rim protector .....	160
Spokes guard .....	161
Quick release axle and bearing .....	162

### BRAKES

BRAKES .....	163
Brakes 15.02.2008=> .....	164
Drum brake .....	165



**Yes, you can.<sup>®</sup>**

## **TABLE OF CONTENTS**


---

### **OPTIONS & ACCESSORIES**

OPTIONS & ACCESSORIES .....	166
Tipper aid .....	167
Antitipper 12.12.2008=> .....	168
Cane holder .....	169
Transit wheels .....	170
Therapy table .....	171
Chair accessories .....	172
Fixation: screws, ... ..	173
Socket head cap screws .....	174
Countersunk screws .....	177
Button head screws .....	179
Hex-head screws, diverse, ... ..	181
Washers .....	183
Nuts .....	185
Pins, rivets, ... ..	187




## Abbreviations

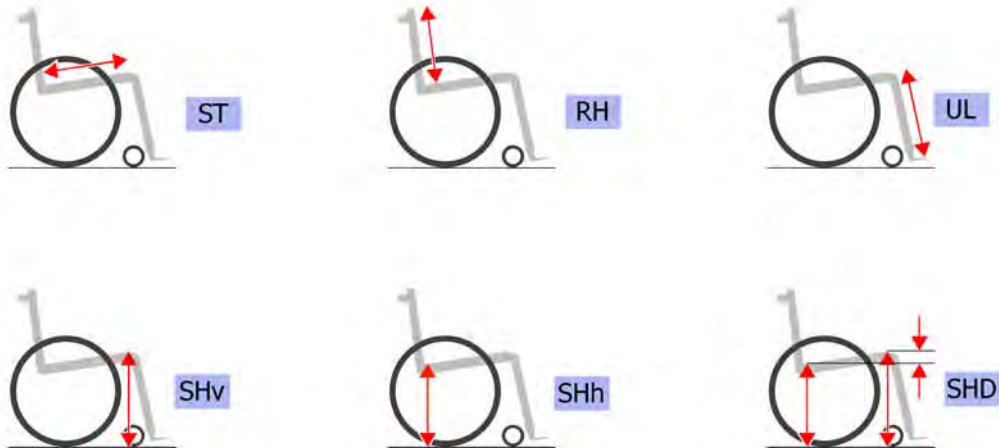


Pos.	Part number	Description	Valid from → until	Qty
L	-	Left side (sitting in the wheelchair)		0
R	-	Right side (sitting in the wheelchair)		0
L+R	-	Left and right sides (paire)		0
SB	-	Seat width (2 digits in cm or 3 digits in mm)		0
ST	-	Seat depth (2 digits in cm or 3 digits in mm)		0
RH	-	Backrest height (2 digits in cm or 3 digits in mm)		0
UL	-	Knee-to-heel length (2 digits in cm or 3 digits in mm)		0
SHv	-	Front seat-to-floor height (2 digits in cm or 3 digits in mm)		0
SHh	-	Rear seat-to-floor height (2 digits in cm or 3 digits in mm)		0
SHD	-	Front and rear seat-to-floor height difference (2 digits in cm or 3 digits in mm)		0

## Frame colours

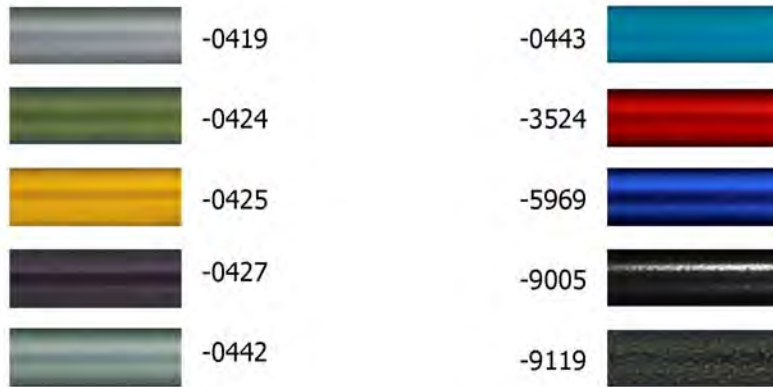


Pos.	Part number	Description	Valid from → until	Qty
-0419	-	Color code for silver grey frames		0
-0442	-	Color code for night shimmer frames	2009-01-01 →	0
-3524	-	Color code for royal red frames		0
-5969	-	Color code for royal blue frames		0
-9005	-	Color code for glossy black frames		0
-9119	-	Color code for black bronze frames		0

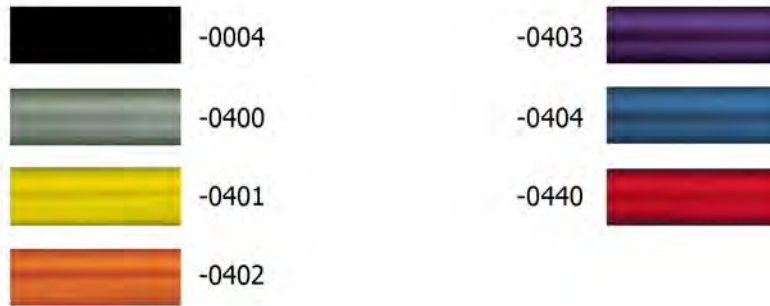

**CPS colours**


Pos.	Part number	Description	Valid from → until	Qty
-0004	-	Color code for black anodized CPS		0
-0400	-	Color code for techno-anthrazit CPS		0
-0401	-	Color code for lemon CPS		0
-0402	-	Color code for apricot CPS		0
-0404	-	Color code for b-blue CPS		0
-0440	-	Color code for comet red CPS	2007-01-01 →	0

# CHASSIS

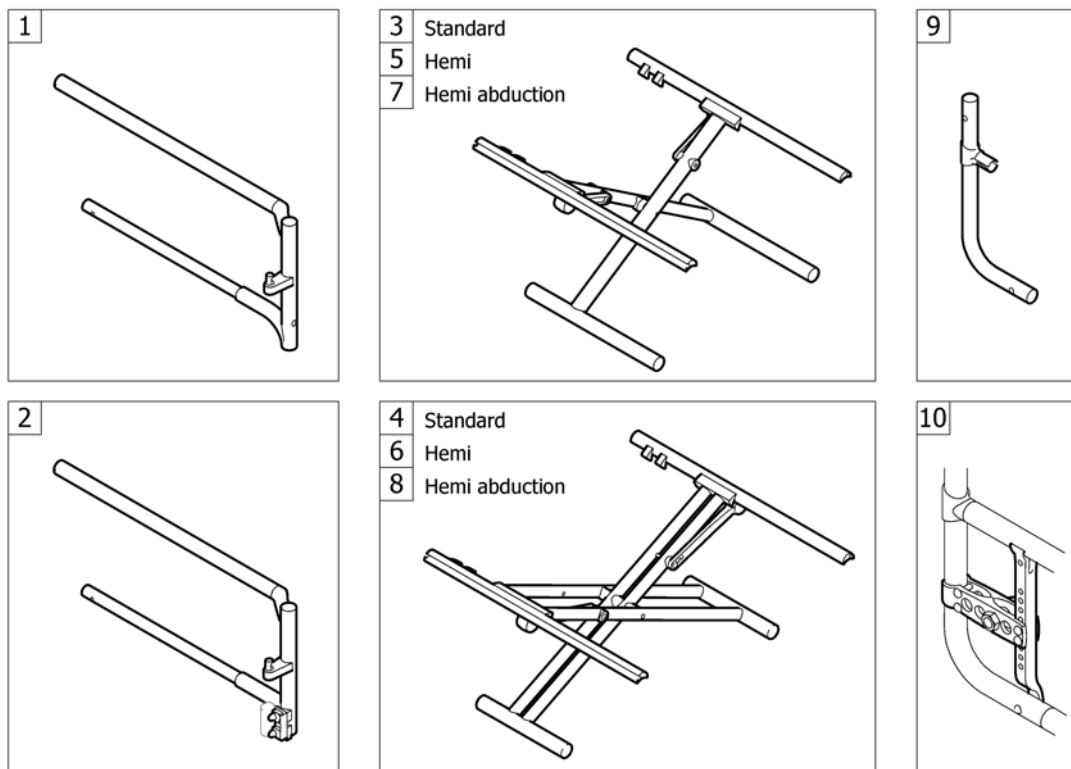

Pos.	Part number	Description	Valid from → until	Qty
1	-	Front frames for flow system	2004-01-01 → 2012-12-31	1
2	-	Front frames for bearing blocks	2006-07-01 → 2012-12-31	1
3	-	Cross brace to frame standard, SB32-40	2004-01-01 → 2012-12-31	1
4	-	Cross brace to frame standard, SB42-50	2004-01-01 → 2012-12-31	1
5	-	Cross brace to frame Hemi, SB32-40	2004-01-01 → 2012-12-31	1
6	-	Cross brace to frame Hemi, SB42-50	2004-01-01 → 2012-12-31	1
7	-	Cross brace to frame Hemi abduction, SB32-40	2004-01-01 → 2012-12-31	1
8	-	Cross brace to frame Hemi abduction, SB42-50	2004-01-01 → 2012-12-31	1
9	-	Rear frames	2004-01-01 → 2012-12-31	1
10	-	Rear wheel fixation	2004-01-01 → 2012-12-31	1

# Front frames for flow system

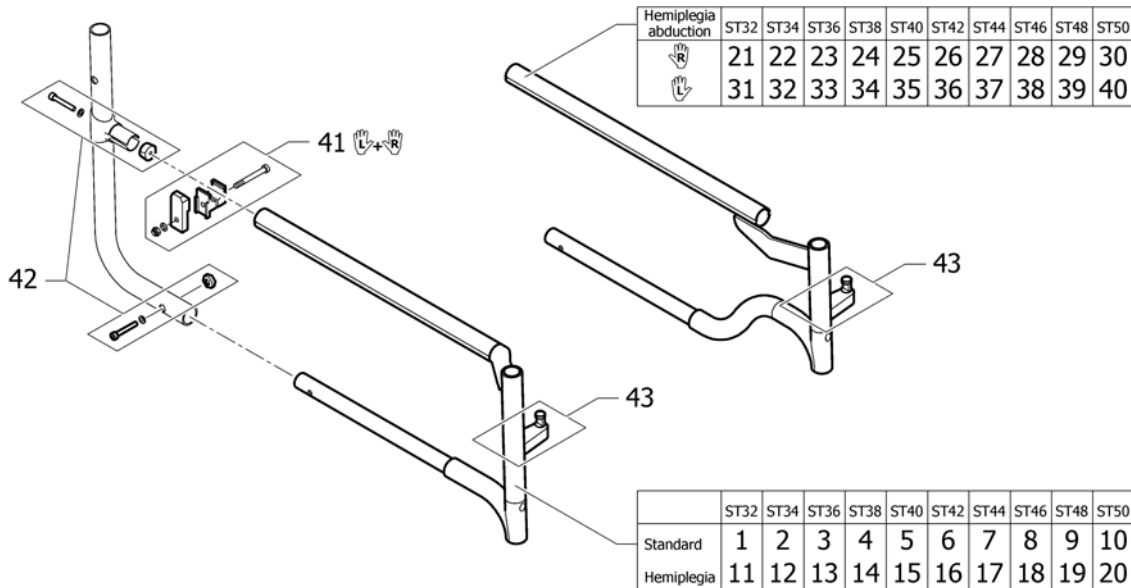

Pos.	Part number	Description	Valid from → until	Qty
1	1445134-0419	Front frame Compact ST32, silver grey		2
1	1445134-0442	Front frame Compact ST32, night shimmer	2009-01-01 →	2
1	1445134-3524	Front frame Compact ST32, royal red		2
1	1445134-5969	Front frame Compact ST32, royal blue		2
1	1445134-9005	Front frame Compact ST32, black, glossy		2
1	1445134-9119	Front frame Compact ST32, black, bronze		2
2	1445128-0419	Front frame Compact ST34, silver grey		2
2	1445128-0442	Front frame Compact ST34, night shimmer	2009-01-01 →	2
2	1445128-3524	Front frame Compact ST34, royal red		2
2	1445128-5969	Front frame Compact ST34, royal blue		2
2	1445128-9005	Front frame Compact ST34, black, glossy		2
2	1445128-9119	Front frame Compact ST34, black, bronze		2
3	1445122-0419	Front frame Compact ST36, silver grey		2
3	1445122-0442	Front frame Compact ST36, night shimmer	2009-01-01 →	2
3	1445122-3524	Front frame Compact ST36, royal red		2
3	1445122-5969	Front frame Compact ST36, royal blue		2
3	1445122-9005	Front frame Compact ST36, black, glossy		2
3	1445122-9119	Front frame Compact ST36, black, bronze		2
4	1445116-0419	Front frame Compact ST38, silver grey		2
4	1445116-0442	Front frame Compact ST38, night shimmer	2009-01-01 →	2
4	1445116-3524	Front frame Compact ST38, royal red		2
4	1445116-5969	Front frame Compact ST38, royal blue		2
4	1445116-9005	Front frame Compact ST38, black, glossy		2
4	1445116-9119	Front frame Compact ST38, black, bronze		2
5	1445110-0419	Front frame Compact ST40, silver grey		2
5	1445110-0442	Front frame Compact ST40, night shimmer	2009-01-01 →	2
5	1445110-3524	Front frame Compact ST40, royal red		2
5	1445110-5969	Front frame Compact ST40, royal blue		2
5	1445110-9005	Front frame Compact ST40, black, glossy		2
5	1445110-9119	Front frame Compact ST40, black, bronze		2
6	1445168-0419	Front frame Compact ST42, silver grey		2

## Front frames for flow system

Pos.	Part number	Description	Valid from → until	Qty
6	1445168-0442	Front frame Compact ST42, night shimmer	2009-01-01 →	2
6	1445168-3524	Front frame Compact ST42, royal red		2
6	1445168-5969	Front frame Compact ST42, royal blue		2
6	1445168-9005	Front frame Compact ST42, black, glossy		2
6	1445168-9119	Front frame Compact ST42, black, bronze		2
7	1445162-0419	Front frame Compact ST44, silver grey		2
7	1445162-0442	Front frame Compact ST44, night shimmer	2009-01-01 →	2
7	1445162-3524	Front frame Compact ST44, royal red		2
7	1445162-5969	Front frame Compact ST44, royal blue		2
7	1445162-9005	Front frame Compact ST44, black, glossy		2
7	1445162-9119	Front frame Compact ST44, black, bronze		2
8	1445156-0419	Front frame Compact ST46, silver grey		2
8	1445156-0442	Front frame Compact ST46, night shimmer	2009-01-01 →	2
8	1445156-3524	Front frame Compact ST46, royal red		2
8	1445156-5969	Front frame Compact ST46, royal blue		2
8	1445156-9005	Front frame Compact ST46, black, glossy		2
8	1445156-9119	Front frame Compact ST46, black, bronze		2
9	1445150-0419	Front frame Compact ST48, silver grey		2
9	1445150-0442	Front frame Compact ST48, night shimmer	2009-01-01 →	2
9	1445150-3524	Front frame Compact ST48, royal red		2
9	1445150-5969	Front frame Compact ST48, royal blue		2
9	1445150-9005	Front frame Compact ST48, black, glossy		2
9	1445150-9119	Front frame Compact ST48, black, bronze		2
10	1445144-0419	Front frame Compact ST50, silver grey		2
10	1445144-0442	Front frame Compact ST50, night shimmer	2009-01-01 →	2
10	1445144-3524	Front frame Compact ST50, royal red		2
10	1445144-5969	Front frame Compact ST50, royal blue		2
10	1445144-9005	Front frame Compact ST50, black, glossy		2
10	1445144-9119	Front frame Compact ST50, black, bronze		2
11	1445135-0419	Front frame Compact Hemi straight ST32, silver grey		2
11	1445135-0442	Front frame Compact Hemi straight ST32, night shimmer	2009-01-01 →	2
11	1445135-3524	Front frame Compact Hemi straight ST32, royal red		2
11	1445135-5969	Front frame Compact Hemi straight ST32, royal blue		2
11	1445135-9005	Front frame Compact Hemi straight ST32, black, glossy		2
11	1445135-9119	Front frame Compact Hemi straight ST32, black, bronze		2
12	1445129-0419	Front frame Compact Hemi straight ST34, silver grey		2
12	1445129-0442	Front frame Compact Hemi straight ST34, night shimmer	2009-01-01 →	2
12	1445129-3524	Front frame Compact Hemi straight ST34, royal red		2
12	1445129-5969	Front frame Compact Hemi straight ST34, royal blue		2
12	1445129-9005	Front frame Compact Hemi straight ST34, black, glossy		2
12	1445129-9119	Front frame Compact Hemi straight ST34, black, bronze		2
13	1445123-0419	Front frame Compact Hemi straight ST36, silver grey		2
13	1445123-0442	Front frame Compact Hemi straight ST36, night shimmer	2009-01-01 →	2
13	1445123-3524	Front frame Compact Hemi straight ST36, royal red		2
13	1445123-5969	Front frame Compact Hemi straight ST36, royal blue		2
13	1445123-9005	Front frame Compact Hemi straight ST36, black, glossy		2
13	1445123-9119	Front frame Compact Hemi straight ST36, black, bronze		2
14	1445117-0419	Front frame Compact Hemi straight ST38, silver grey		2
14	1445117-0442	Front frame Compact Hemi straight ST38, night shimmer	2009-01-01 →	2
14	1445117-3524	Front frame Compact Hemi straight ST38, royal red		2
14	1445117-5969	Front frame Compact Hemi straight ST38, royal blue		2
14	1445117-9005	Front frame Compact Hemi straight ST38, black, glossy		2
14	1445117-9119	Front frame Compact Hemi straight ST38, black, bronze		2
15	1445111-0419	Front frame Compact Hemi straight ST40, silver grey		2
15	1445111-0442	Front frame Compact Hemi straight ST40, night shimmer	2009-01-01 →	2

## Front frames for flow system

Pos.	Part number	Description	Valid from → until	Qty
15	1445111-3524	Front frame Compact Hemi straight ST40, royal red		2
15	1445111-5969	Front frame Compact Hemi straight ST40, royal blue		2
15	1445111-9005	Front frame Compact Hemi straight ST40, black, glossy		2
15	1445111-9119	Front frame Compact Hemi straight ST40, black, bronze		2
16	1445169-0419	Front frame Compact Hemi straight ST42, silver grey		2
16	1445169-0442	Front frame Compact Hemi straight ST42, night shimmer	2009-01-01 →	2
16	1445169-3524	Front frame Compact Hemi straight ST42, royal red		2
16	1445169-5969	Front frame Compact Hemi straight ST42, royal blue		2
16	1445169-9005	Front frame Compact Hemi straight ST42, black, glossy		2
16	1445169-9119	Front frame Compact Hemi straight ST42, black, bronze		2
17	1445163-0419	Front frame Compact Hemi straight ST44, silver grey		2
17	1445163-0442	Front frame Compact Hemi straight ST44, night shimmer	2009-01-01 →	2
17	1445163-3524	Front frame Compact Hemi straight ST44, royal red		2
17	1445163-5969	Front frame Compact Hemi straight ST44, royal blue		2
17	1445163-9005	Front frame Compact Hemi straight ST44, black, glossy		2
17	1445163-9119	Front frame Compact Hemi straight ST44, black, bronze		2
18	1445157-0419	Front frame Compact Hemi straight ST46, silver grey		2
18	1445157-0442	Front frame Compact Hemi straight ST46, night shimmer	2009-01-01 →	2
18	1445157-3524	Front frame Compact Hemi straight ST46, royal red		2
18	1445157-5969	Front frame Compact Hemi straight ST46, royal blue		2
18	1445157-9005	Front frame Compact Hemi straight ST46, black, glossy		2
18	1445157-9119	Front frame Compact Hemi straight ST46, black, bronze		2
19	1445151-0419	Front frame Compact Hemi straight ST48, silver grey		2
19	1445151-0442	Front frame Compact Hemi straight ST48, night shimmer	2009-01-01 →	2
19	1445151-3524	Front frame Compact Hemi straight ST48, royal red		2
19	1445151-5969	Front frame Compact Hemi straight ST48, royal blue		2
19	1445151-9005	Front frame Compact Hemi straight ST48, black, glossy		2
19	1445151-9119	Front frame Compact Hemi straight ST48, black, bronze		2
20	1445145-0419	Front frame Compact Hemi straight ST50, silver grey		2
20	1445145-0442	Front frame Compact Hemi straight ST50, night shimmer	2009-01-01 →	2
20	1445145-3524	Front frame Compact Hemi straight ST50, royal red		2
20	1445145-5969	Front frame Compact Hemi straight ST50, royal blue		2
20	1445145-9005	Front frame Compact Hemi straight ST50, black, glossy		2
20	1445145-9119	Front frame Compact Hemi straight ST50, black, bronze		2
21	1445136-0419	Front frame Compact Hemi abduction right ST32, silver grey		1
21	1445136-0442	Front frame Compact Hemi abduction right ST32, night shimmer	2009-01-01 →	1
21	1445136-3524	Front frame Compact Hemi abduction right ST32, royal red		1
21	1445136-5969	Front frame Compact Hemi abduction right ST32, royal blue		1
21	1445136-9005	Front frame Compact Hemi abduction right ST32, black, glossy		1
21	1445136-9119	Front frame Compact Hemi abduction right ST32, black, bronze		1
22	1445130-0419	Front frame Compact Hemi abduction right ST34, silver grey		1
22	1445130-0442	Front frame Compact Hemi abduction right ST34, night shimmer	2009-01-01 →	1
22	1445130-3524	Front frame Compact Hemi abduction right ST34, royal red		1
22	1445130-5969	Front frame Compact Hemi abduction right ST34, royal blue		1
22	1445130-9005	Front frame Compact Hemi abduction right ST34, black, glossy		1
22	1445130-9119	Front frame Compact Hemi abduction right ST34, black, bronze		1
23	1445124-0419	Front frame Compact Hemi abduction right ST36, silver grey		1
23	1445124-0442	Front frame Compact Hemi abduction right ST36, night shimmer	2009-01-01 →	1
23	1445124-3524	Front frame Compact Hemi abduction right ST36, royal red		1
23	1445124-5969	Front frame Compact Hemi abduction right ST36, royal blue		1
23	1445124-9005	Front frame Compact Hemi abduction right ST36, black, glossy		1
23	1445124-9119	Front frame Compact Hemi abduction right ST36, black, bronze		1
24	1445118-0419	Front frame Compact Hemi abduction right ST38, silver grey		1
24	1445118-0442	Front frame Compact Hemi abduction right ST38, night shimmer	2009-01-01 →	1
24	1445118-3524	Front frame Compact Hemi abduction right ST38, royal red		1

## Front frames for flow system


Pos.	Part number	Description	Valid from → until	Qty
24	1445118-5969	Front frame Compact Hemi abduction right ST38, royal blue		1
24	1445118-9005	Front frame Compact Hemi abduction right ST38, black, glossy		1
24	1445118-9119	Front frame Compact Hemi abduction right ST38, black, bronze		1
25	1445112-0419	Front frame Compact Hemi abduction right ST40, silver grey		1
25	1445112-0442	Front frame Compact Hemi abduction right ST40, night shimmer	2009-01-01 →	1
25	1445112-3524	Front frame Compact Hemi abduction right ST40, royal red		1
25	1445112-5969	Front frame Compact Hemi abduction right ST40, royal blue		1
25	1445112-9005	Front frame Compact Hemi abduction right ST40, black, glossy		1
25	1445112-9119	Front frame Compact Hemi abduction right ST40, black, bronze		1
26	1445170-0419	Front frame Compact Hemi abduction right ST42, silver grey		1
26	1445170-0442	Front frame Compact Hemi abduction right ST42, night shimmer	2009-01-01 →	1
26	1445170-3524	Front frame Compact Hemi abduction right ST42, royal red		1
26	1445170-5969	Front frame Compact Hemi abduction right ST42, royal blue		1
26	1445170-9005	Front frame Compact Hemi abduction right ST42, black, glossy		1
26	1445170-9119	Front frame Compact Hemi abduction right ST42, black, bronze		1
27	1445164-0419	Front frame Compact Hemi abduction right ST44, silver grey		1
27	1445164-0442	Front frame Compact Hemi abduction right ST44, night shimmer	2009-01-01 →	1
27	1445164-3524	Front frame Compact Hemi abduction right ST44, royal red		1
27	1445164-5969	Front frame Compact Hemi abduction right ST44, royal blue		1
27	1445164-9005	Front frame Compact Hemi abduction right ST44, black, glossy		1
27	1445164-9119	Front frame Compact Hemi abduction right ST44, black, bronze		1
28	1445158-0419	Front frame Compact Hemi abduction right ST46, silver grey		1
28	1445158-0442	Front frame Compact Hemi abduction right ST46, night shimmer	2009-01-01 →	1
28	1445158-3524	Front frame Compact Hemi abduction right ST46, royal red		1
28	1445158-5969	Front frame Compact Hemi abduction right ST46, royal blue		1
28	1445158-9005	Front frame Compact Hemi abduction right ST46, black, glossy		1
28	1445158-9119	Front frame Compact Hemi abduction right ST46, black, bronze		1
29	1445152-0419	Front frame Compact Hemi abduction right ST48, silver grey		1
29	1445152-0442	Front frame Compact Hemi abduction right ST48, night shimmer	2009-01-01 →	1
29	1445152-3524	Front frame Compact Hemi abduction right ST48, royal red		1
29	1445152-5969	Front frame Compact Hemi abduction right ST48, royal blue		1
29	1445152-9005	Front frame Compact Hemi abduction right ST48, black, glossy		1
29	1445152-9119	Front frame Compact Hemi abduction right ST48, black, bronze		1
30	1445146-0419	Front frame Compact Hemi abduction right ST50, silver grey		1
30	1445146-0442	Front frame Compact Hemi abduction right ST50, night shimmer	2009-01-01 →	1
30	1445146-3524	Front frame Compact Hemi abduction right ST50, royal red		1
30	1445146-5969	Front frame Compact Hemi abduction right ST50, royal blue		1
30	1445146-9005	Front frame Compact Hemi abduction right ST50, black, glossy		1
30	1445146-9119	Front frame Compact Hemi abduction right ST50, black, bronze		1
31	1445137-0419	Front frame Compact Hemi abduction left ST32, silver grey		1
31	1445137-0442	Front frame Compact Hemi abduction left ST32, night shimmer	2009-01-01 →	1
31	1445137-3524	Front frame Compact Hemi abduction left ST32, royal red		1
31	1445137-5969	Front frame Compact Hemi abduction left ST32, royal blue		1
31	1445137-9005	Front frame Compact Hemi abduction left ST32, black, glossy		1
31	1445137-9119	Front frame Compact Hemi abduction left ST32, black, bronze		1
32	1445131-0419	Front frame Compact Hemi abduction left ST34, silver grey		1
32	1445131-0442	Front frame Compact Hemi abduction left ST34, night shimmer	2009-01-01 →	1
32	1445131-3524	Front frame Compact Hemi abduction left ST34, royal red		1
32	1445131-5969	Front frame Compact Hemi abduction left ST34, royal blue		1
32	1445131-9005	Front frame Compact Hemi abduction left ST34, black, glossy		1
32	1445131-9119	Front frame Compact Hemi abduction left ST34, black, bronze		1
33	1445125-0419	Front frame Compact Hemi abduction left ST36, silver grey		1
33	1445125-0442	Front frame Compact Hemi abduction left ST36, night shimmer	2009-01-01 →	1
33	1445125-3524	Front frame Compact Hemi abduction left ST36, royal red		1
33	1445125-5969	Front frame Compact Hemi abduction left ST36, royal blue		1



## Front frames for flow system

Pos.	Part number	Description	Valid from → until	Qty
33	1445125-9005	Front frame Compact Hemi abduction left ST36, black, glossy		1
33	1445125-9119	Front frame Compact Hemi abduction left ST36, black, bronze		1
34	1445119-0419	Front frame Compact Hemi abduction left ST38, silver grey		1
34	1445119-0442	Front frame Compact Hemi abduction left ST38, night shimmer	2009-01-01 →	1
34	1445119-3524	Front frame Compact Hemi abduction left ST38, royal red		1
34	1445119-5969	Front frame Compact Hemi abduction left ST38, royal blue		1
34	1445119-9005	Front frame Compact Hemi abduction left ST38, black, glossy		1
34	1445119-9119	Front frame Compact Hemi abduction left ST38, black, bronze		1
35	1445113-0419	Front frame Compact Hemi abduction left ST40, silver grey		1
35	1445113-0442	Front frame Compact Hemi abduction left ST40, night shimmer	2009-01-01 →	1
35	1445113-3524	Front frame Compact Hemi abduction left ST40, royal red		1
35	1445113-5969	Front frame Compact Hemi abduction left ST40, royal blue		1
35	1445113-9005	Front frame Compact Hemi abduction left ST40, black, glossy		1
35	1445113-9119	Front frame Compact Hemi abduction left ST40, black, bronze		1
36	1445107-0419	Front frame Compact Hemi abduction left ST42, silver grey		1
36	1445107-0442	Front frame Compact Hemi abduction left ST42, night shimmer	2009-01-01 →	1
36	1445107-3524	Front frame Compact Hemi abduction left ST42, royal red		1
36	1445107-5969	Front frame Compact Hemi abduction left ST42, royal blue		1
36	1445107-9005	Front frame Compact Hemi abduction left ST42, black, glossy		1
36	1445107-9119	Front frame Compact Hemi abduction left ST42, black, bronze		1
37	1445165-0419	Front frame Compact Hemi abduction left ST44, silver grey		1
37	1445165-0442	Front frame Compact Hemi abduction left ST44, night shimmer	2009-01-01 →	1
37	1445165-3524	Front frame Compact Hemi abduction left ST44, royal red		1
37	1445165-5969	Front frame Compact Hemi abduction left ST44, royal blue		1
37	1445165-9005	Front frame Compact Hemi abduction left ST44, black, glossy		1
37	1445165-9119	Front frame Compact Hemi abduction left ST44, black, bronze		1
38	1445159-0419	Front frame Compact Hemi abduction left ST46, silver grey		1
38	1445159-0442	Front frame Compact Hemi abduction left ST46, night shimmer	2009-01-01 →	1
38	1445159-3524	Front frame Compact Hemi abduction left ST46, royal red		1
38	1445159-5969	Front frame Compact Hemi abduction left ST46, royal blue		1
38	1445159-9005	Front frame Compact Hemi abduction left ST46, black, glossy		1
38	1445159-9119	Front frame Compact Hemi abduction left ST46, black, bronze		1
39	1445153-0419	Front frame Compact Hemi abduction left ST48, silver grey		1
39	1445153-0442	Front frame Compact Hemi abduction left ST48, night shimmer	2009-01-01 →	1
39	1445153-3524	Front frame Compact Hemi abduction left ST48, royal red		1
39	1445153-5969	Front frame Compact Hemi abduction left ST48, royal blue		1
39	1445153-9005	Front frame Compact Hemi abduction left ST48, black, glossy		1
39	1445153-9119	Front frame Compact Hemi abduction left ST48, black, bronze		1
40	1445147-0419	Front frame Compact Hemi abduction left ST50, silver grey		1
40	1445147-0442	Front frame Compact Hemi abduction left ST50, night shimmer	2009-01-01 →	1
40	1445147-3524	Front frame Compact Hemi abduction left ST50, royal red		1
40	1445147-5969	Front frame Compact Hemi abduction left ST50, royal blue		1
40	1445147-9005	Front frame Compact Hemi abduction left ST50, black, glossy		1
40	1445147-9119	Front frame Compact Hemi abduction left ST50, black, bronze		1
41	1509133	Longitudinal locking kit, pair	2007-06-01 →	1
42	1437842	frame fixation set		1
43	1517870	Positioning bracket, Swing-away complete		2

# Front frames for bearing blocks

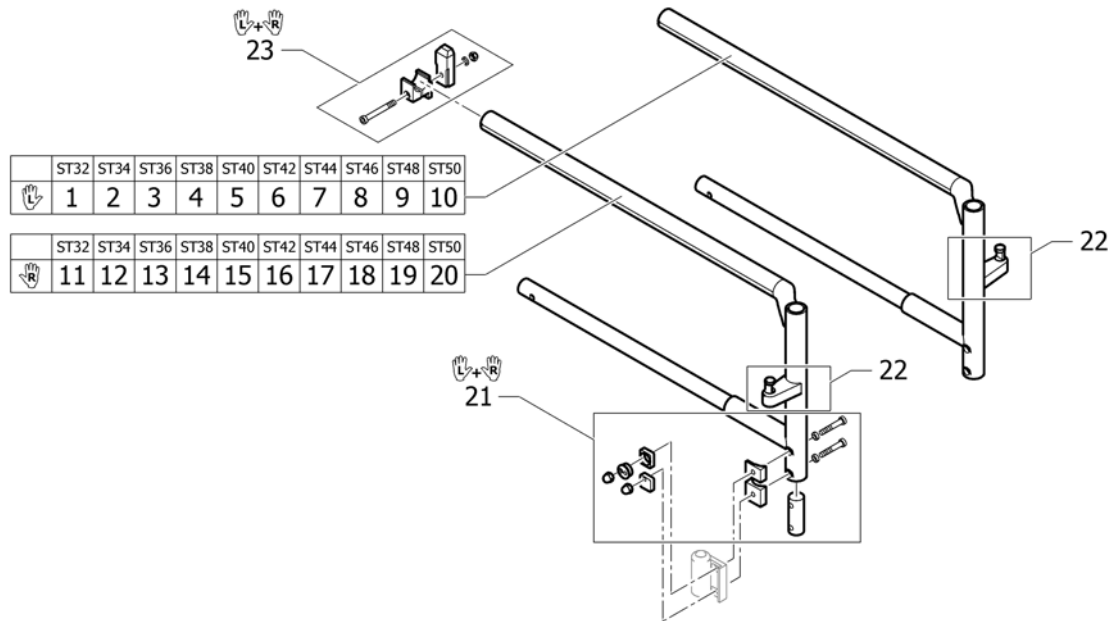

Pos.	Part number	Description	Valid from → until	Qty
1	1495560-0419	Frame Compact ST32 left ext. Silver Grey		1
1	1495560-0442	Frame Compact ST32 left ext. night shimmer	2009-01-01 →	1
1	1495560-3524	Frame Compact ST32 left ext. Royal Red		1
1	1495560-5969	Frame Compact ST32 left ext. Royal Blue		1
1	1495560-9005	Frame Compact ST32 left ext. Black glossy		1
1	1495560-9119	Frame Compact ST32 left ext. Black bronze		1
2	1495554-0419	Frame Compact ST34 left ext. Silver Grey		1
2	1495554-0442	Frame Compact ST34 left ext. night shimmer	2009-01-01 →	1
2	1495554-3524	Frame Compact ST34 left ext. Royal Red		1
2	1495554-5969	Frame Compact ST34 left ext. Royal Blue		1
2	1495554-9005	Frame Compact ST34 left ext. Black glossy		1
2	1495554-9119	Frame Compact ST34 left ext. Black bronze		1
3	1495548-0419	Frame Compact ST36 left ext. Silver Grey		1
3	1495548-0442	Frame Compact ST36 left ext. night shimmer	2009-01-01 →	1
3	1495548-3524	Frame Compact ST36 left ext. Royal Red		1
3	1495548-5969	Frame Compact ST36 left ext. Royal Blue		1
3	1495548-9005	Frame Compact ST36 left ext. Black glossy		1
3	1495548-9119	Frame Compact ST36 left ext. Black bronze		1
4	1495542-0419	Frame Compact ST38 left ext. Silver Grey		1
4	1495542-0442	Frame Compact ST38 left ext. night shimmer	2009-01-01 →	1
4	1495542-3524	Frame Compact ST38 left ext. Royal Red		1
4	1495542-5969	Frame Compact ST38 left ext. Royal Blue		1
4	1495542-9005	Frame Compact ST38 left ext. Black glossy		1
4	1495542-9119	Frame Compact ST38 left ext. Black bronze		1
5	1495536-0419	Frame Compact ST40 left ext. Silver Grey		1
5	1495536-0442	Frame Compact ST40 left ext. night shimmer	2009-01-01 →	1
5	1495536-3524	Frame Compact ST40 left ext. Royal Red		1
5	1495536-5969	Frame Compact ST40 left ext. Royal Blue		1
5	1495536-9005	Frame Compact ST40 left ext. Black glossy		1
5	1495536-9119	Frame Compact ST40 left ext. Black bronze		1
6	1495530-0419	Frame Compact ST42 left ext. Silver Grey		1




## Front frames for bearing blocks

Pos.	Part number	Description	Valid from → until	Qty
6	1495530-0442	Frame Compact ST42 left ext. night shimmer	2009-01-01 →	1
6	1495530-3524	Frame Compact ST42 left ext. Royal Red		1
6	1495530-5969	Frame Compact ST42 left ext. Royal Blue		1
6	1495530-9005	Frame Compact ST42 left ext. Black glossy		1
6	1495530-9119	Frame Compact ST42 left ext. Black bronze		1
7	1495524-0419	Frame Compact ST44 left ext. Silver Grey		1
7	1495524-0442	Frame Compact ST44 left ext. night shimmer	2009-01-01 →	1
7	1495524-3524	Frame Compact ST44 left ext. Royal Red		1
7	1495524-5969	Frame Compact ST44 left ext. Royal Blue		1
7	1495524-9005	Frame Compact ST44 left ext. Black glossy		1
7	1495524-9119	Frame Compact ST44 left ext. Black bronze		1
8	1495518-0419	Frame Compact ST46 left ext. Silver Grey		1
8	1495518-0442	Frame Compact ST46 left ext. night shimmer	2009-01-01 →	1
8	1495518-3524	Frame Compact ST46 left ext. Royal Red		1
8	1495518-5969	Frame Compact ST46 left ext. Royal Blue		1
8	1495518-9005	Frame Compact ST46 left ext. Black glossy		1
8	1495518-9119	Frame Compact ST46 left ext. Black bronze		1
9	1495512-0419	Frame Compact ST48 left ext. Silver Grey		1
9	1495512-0442	Frame Compact ST48 left ext. night shimmer	2009-01-01 →	1
9	1495512-3524	Frame Compact ST48 left ext. Royal Red		1
9	1495512-5969	Frame Compact ST48 left ext. Royal Blue		1
9	1495512-9005	Frame Compact ST48 left ext. Black glossy		1
9	1495512-9119	Frame Compact ST48 left ext. Black bronze		1
10	1495506-0419	Frame Compact ST50 left ext. Silver Grey		1
10	1495506-0442	Frame Compact ST50 left ext. night shimmer	2009-01-01 →	1
10	1495506-3524	Frame Compact ST50 left ext. Royal Red		1
10	1495506-5969	Frame Compact ST50 left ext. Royal Blue		1
10	1495506-9005	Frame Compact ST50 left ext. Black glossy		1
10	1495506-9119	Frame Compact ST50 left ext. Black bronze		1
11	1495557-0419	Frame Compact ST32 right ext. Silver Grey		1
11	1495557-0442	Frame Compact ST32 right ext. night shimmer	2009-01-01 →	1
11	1495557-3524	Frame Compact ST32 right ext. Royal Red		1
11	1495557-5969	Frame Compact ST32 right ext. Royal Blue		1
11	1495557-9005	Frame Compact ST32 right ext. Black glossy		1
11	1495557-9119	Frame Compact ST32 right ext. Black bronze		1
12	1495551-0419	Frame Compact ST34 right ext. Silver Grey		1
12	1495551-0442	Frame Compact ST34 right ext. night shimmer	2009-01-01 →	1
12	1495551-3524	Frame Compact ST34 right ext. Royal Red		1
12	1495551-5969	Frame Compact ST34 right ext. Royal Blue		1
12	1495551-9005	Frame Compact ST34 right ext. Black glossy		1
12	1495551-9119	Frame Compact ST34 right ext. Black bronze		1
13	1495545-0419	Frame Compact ST36 right ext. Silver Grey		1
13	1495545-0442	Frame Compact ST36 right ext. night shimmer	2009-01-01 →	1
13	1495545-3524	Frame Compact ST36 right ext. Royal Red		1
13	1495545-5969	Frame Compact ST36 right ext. Royal Blue		1
13	1495545-9005	Frame Compact ST36 right ext. Black glossy		1
13	1495545-9119	Frame Compact ST36 right ext. Black bronze		1
14	1495539-0419	Frame Compact ST38 right ext. Silver Grey		1
14	1495539-0442	Frame Compact ST38 right ext. night shimmer	2009-01-01 →	1
14	1495539-3524	Frame Compact ST38 right ext. Royal Red		1
14	1495539-5969	Frame Compact ST38 right ext. Royal Blue		1
14	1495539-9005	Frame Compact ST38 right ext. Black glossy		1
14	1495539-9119	Frame Compact ST38 right ext. Black bronze		1
15	1495533-0419	Frame Compact ST40 right ext. Silver Grey		1
15	1495533-0442	Frame Compact ST40 right ext. night shimmer	2009-01-01 →	1

## Front frames for bearing blocks

Pos.	Part number	Description	Valid from → until	Qty
15	1495533-3524	Frame Compact ST40 right ext. Royal Red		1
15	1495533-5969	Frame Compact ST40 right ext. Royal Blue		1
15	1495533-9005	Frame Compact ST40 right ext. Black glossy		1
15	1495533-9119	Frame Compact ST40 right ext. Black bronze		1
16	1495527-0419	Frame Compact ST42 right ext. Silver Grey		1
16	1495527-0442	Frame Compact ST42 right ext. night shimmer	2009-01-01 →	1
16	1495527-3524	Frame Compact ST42 right ext. Royal Red		1
16	1495527-5969	Frame Compact ST42 right ext. Royal Blue		1
16	1495527-9005	Frame Compact ST42 right ext. Black glossy		1
16	1495527-9119	Frame Compact ST42 right ext. Black bronze		1
17	1495521-0419	Frame Compact ST44 right ext. Silver Grey		1
17	1495521-0442	Frame Compact ST44 right ext. night shimmer	2009-01-01 →	1
17	1495521-3524	Frame Compact ST44 right ext. Royal Red		1
17	1495521-5969	Frame Compact ST44 right ext. Royal Blue		1
17	1495521-9005	Frame Compact ST44 right ext. Black glossy		1
17	1495521-9119	Frame Compact ST44 right ext. Black bronze		1
18	1495515-0419	Frame Compact ST46 right ext. Silver Grey		1
18	1495515-0442	Frame Compact ST46 right ext. night shimmer	2009-01-01 →	1
18	1495515-3524	Frame Compact ST46 right ext. Royal Red		1
18	1495515-5969	Frame Compact ST46 right ext. Royal Blue		1
18	1495515-9005	Frame Compact ST46 right ext. Black glossy		1
18	1495515-9119	Frame Compact ST46 right ext. Black bronze		1
19	1495509-0419	Frame Compact ST48 right ext. Silver Grey		1
19	1495509-0442	Frame Compact ST48 right ext. night shimmer	2009-01-01 →	1
19	1495509-3524	Frame Compact ST48 right ext. Royal Red		1
19	1495509-5969	Frame Compact ST48 right ext. Royal Blue		1
19	1495509-9005	Frame Compact ST48 right ext. Black glossy		1
19	1495509-9119	Frame Compact ST48 right ext. Black bronze		1
20	1495503-0419	Frame Compact ST50 right ext. Silver Grey		1
20	1495503-0442	Frame Compact ST50 right ext. night shimmer	2009-01-01 →	1
20	1495503-3524	Frame Compact ST50 right ext. Royal Red		1
20	1495503-5969	Frame Compact ST50 right ext. Royal Blue		1
20	1495503-9005	Frame Compact ST50 right ext. Black glossy		1
20	1495503-9119	Frame Compact ST50 right ext. Black bronze		1
21	1495570	Excenter set for front fork castor		1
22	1517870	Positioning bracket, Swing-away complete		2
23	1509133	Longitudinal locking kit, pair	2007-06-01 →	1

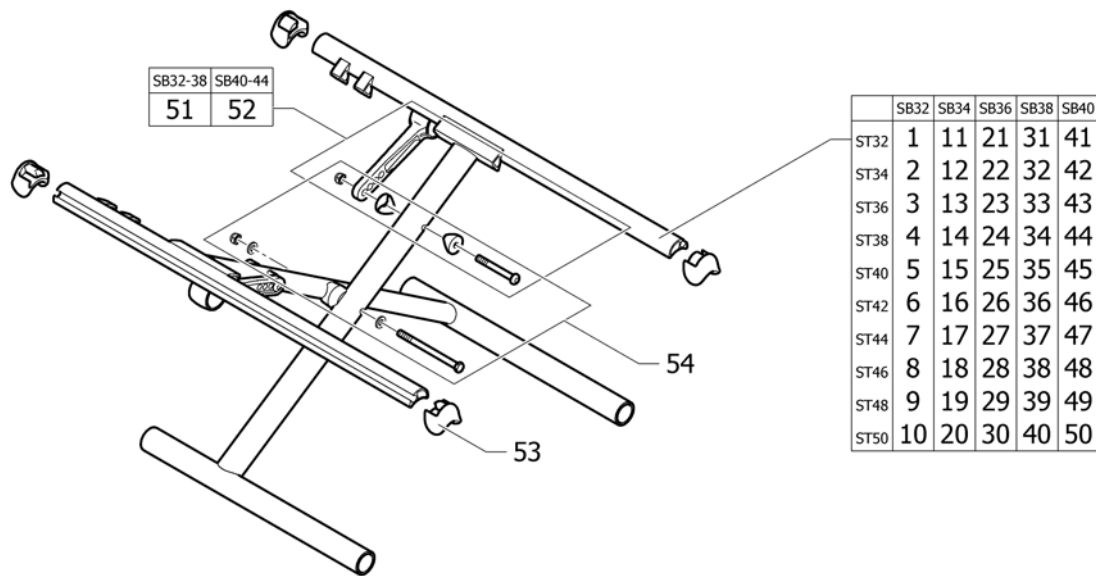**Cross brace to frame standard, SB32-40**

Pos.	Part number	Description	Valid from → until	Qty
1	1437843-0419	Cross-brace single Compact, UltraLight 70° SB32 ST32, silver grey		1
1	1437843-0442	Cross-brace single Compact, UltraLight 70° SB32 ST32, night shimmer	2009-01-01 →	1
1	1437843-3524	Cross-brace single Compact, UltraLight 70° SB32 ST32, royal red		1
1	1437843-5969	Cross-brace single Compact, UltraLight 70° SB32 ST32, royal blue		1
1	1437843-9005	Cross-brace single Compact, UltraLight 70° SB32 ST32, black, glossy		1
1	1437843-9119	Cross-brace single Compact, UltraLight 70° SB32 ST32, black, bronze		1
2	1437844-0419	Cross-brace single Compact, UltraLight 70° SB32 ST34, silver grey		1
2	1437844-0442	Cross-brace single Compact, UltraLight 70° SB32 ST34, night shimmer	2009-01-01 →	1
2	1437844-3524	Cross-brace single Compact, UltraLight 70° SB32 ST34, royal red		1
2	1437844-5969	Cross-brace single Compact, UltraLight 70° SB32 ST34, royal blue		1
2	1437844-9005	Cross-brace single Compact, UltraLight 70° SB32 ST34, black, glossy		1
2	1437844-9119	Cross-brace single Compact, UltraLight 70° SB32 ST34, black, bronze		1
3	1437845-0419	Cross-brace single Compact, UltraLight 70° SB32 ST36, silver grey		1
3	1437845-0442	Cross-brace single Compact, UltraLight 70° SB32 ST36, night shimmer	2009-01-01 →	1
3	1437845-3524	Cross-brace single Compact, UltraLight 70° SB32 ST36, royal red		1
3	1437845-5969	Cross-brace single Compact, UltraLight 70° SB32 ST36, royal blue		1
3	1437845-9005	Cross-brace single Compact, UltraLight 70° SB32 ST36, black, glossy		1
3	1437845-9119	Cross-brace single Compact, UltraLight 70° SB32 ST36, black, bronze		1
4	1437846-0419	Cross-brace single Compact, UltraLight 70° SB32 ST38, silver grey		1
4	1437846-0442	Cross-brace single Compact, UltraLight 70° SB32 ST38, night shimmer	2009-01-01 →	1
4	1437846-3524	Cross-brace single Compact, UltraLight 70° SB32 ST38, royal red		1
4	1437846-5969	Cross-brace single Compact, UltraLight 70° SB32 ST38, royal blue		1
4	1437846-9005	Cross-brace single Compact, UltraLight 70° SB32 ST38, black, glossy		1
4	1437846-9119	Cross-brace single Compact, UltraLight 70° SB32 ST38, black, bronze		1
5	1437847-0419	Cross-brace single Compact, UltraLight 70° SB32 ST40, silver grey		1
5	1437847-0442	Cross-brace single Compact, UltraLight 70° SB32 ST40, night shimmer	2009-01-01 →	1
5	1437847-3524	Cross-brace single Compact, UltraLight 70° SB32 ST40, royal red		1
5	1437847-5969	Cross-brace single Compact, UltraLight 70° SB32 ST40, royal blue		1
5	1437847-9005	Cross-brace single Compact, UltraLight 70° SB32 ST40, black, glossy		1
5	1437847-9119	Cross-brace single Compact, UltraLight 70° SB32 ST40, black, bronze		1
6	1437848-0419	Cross-brace single Compact, UltraLight 70° SB32 ST42, silver grey		1

**Cross brace to frame standard, SB32-40**

Pos.	Part number	Description	Valid from → until	Qty
6	1437848-0442	Cross-brace single Compact, UltraLight 70° SB32 ST42, night shimmer	2009-01-01 →	1
6	1437848-3524	Cross-brace single Compact, UltraLight 70° SB32 ST42, royal red		1
6	1437848-5969	Cross-brace single Compact, UltraLight 70° SB32 ST42, royal blue		1
6	1437848-9005	Cross-brace single Compact, UltraLight 70° SB32 ST42, black, glossy		1
6	1437848-9119	Cross-brace single Compact, UltraLight 70° SB32 ST42, black, bronze		1
7	1437849-0419	Cross-brace single Compact, UltraLight 70° SB32 ST44, silver grey		1
7	1437849-0442	Cross-brace single Compact, UltraLight 70° SB32 ST44, night shimmer	2009-01-01 →	1
7	1437849-3524	Cross-brace single Compact, UltraLight 70° SB32 ST44, royal red		1
7	1437849-5969	Cross-brace single Compact, UltraLight 70° SB32 ST44, royal blue		1
7	1437849-9005	Cross-brace single Compact, UltraLight 70° SB32 ST44, black, glossy		1
7	1437849-9119	Cross-brace single Compact, UltraLight 70° SB32 ST44, black, bronze		1
8	1437850-0419	Cross-brace single Compact, UltraLight 70° SB32 ST46, silver grey		1
8	1437850-0442	Cross-brace single Compact, UltraLight 70° SB32 ST46, night shimmer	2009-01-01 →	1
8	1437850-3524	Cross-brace single Compact, UltraLight 70° SB32 ST46, royal red		1
8	1437850-5969	Cross-brace single Compact, UltraLight 70° SB32 ST46, royal blue		1
8	1437850-9005	Cross-brace single Compact, UltraLight 70° SB32 ST46, black, glossy		1
8	1437850-9119	Cross-brace single Compact, UltraLight 70° SB32 ST46, black, bronze		1
9	1437851-0419	Cross-brace single Compact, UltraLight 70° SB32 ST48, silver grey		1
9	1437851-0442	Cross-brace single Compact, UltraLight 70° SB32 ST48, night shimmer	2009-01-01 →	1
9	1437851-3524	Cross-brace single Compact, UltraLight 70° SB32 ST48, royal red		1
9	1437851-5969	Cross-brace single Compact, UltraLight 70° SB32 ST48, royal blue		1
9	1437851-9005	Cross-brace single Compact, UltraLight 70° SB32 ST48, black, glossy		1
9	1437851-9119	Cross-brace single Compact, UltraLight 70° SB32 ST48, black, bronze		1
10	1437852-0419	Cross-brace single Compact, UltraLight 70° SB32 ST50, silver grey		1
10	1437852-0442	Cross-brace single Compact, UltraLight 70° SB32 ST50, night shimmer	2009-01-01 →	1
10	1437852-3524	Cross-brace single Compact, UltraLight 70° SB32 ST50, royal red		1
10	1437852-5969	Cross-brace single Compact, UltraLight 70° SB32 ST50, royal blue		1
10	1437852-9005	Cross-brace single Compact, UltraLight 70° SB32 ST50, black, glossy		1
10	1437852-9119	Cross-brace single Compact, UltraLight 70° SB32 ST50, black, bronze		1
11	1437853-0419	Cross-brace single Compact, UltraLight 70° SB34 ST32, silver grey		1
11	1437853-0442	Cross-brace single Compact, UltraLight 70° SB34 ST32, night shimmer	2009-01-01 →	1
11	1437853-3524	Cross-brace single Compact, UltraLight 70° SB34 ST32, royal red		1
11	1437853-5969	Cross-brace single Compact, UltraLight 70° SB34 ST32, royal blue		1
11	1437853-9005	Cross-brace single Compact, UltraLight 70° SB34 ST32, black, glossy		1
11	1437853-9119	Cross-brace single Compact, UltraLight 70° SB34 ST32, black, bronze		1
12	1437854-0419	Cross-brace single Compact, UltraLight 70° SB34 ST34, silver grey		1
12	1437854-0442	Cross-brace single Compact, UltraLight 70° SB34 ST34, night shimmer	2009-01-01 →	1
12	1437854-3524	Cross-brace single Compact, UltraLight 70° SB34 ST34, royal red		1
12	1437854-5969	Cross-brace single Compact, UltraLight 70° SB34 ST34, royal blue		1
12	1437854-9005	Cross-brace single Compact, UltraLight 70° SB34 ST34, black, glossy		1
12	1437854-9119	Cross-brace single Compact, UltraLight 70° SB34 ST34, black, bronze		1
13	1437855-0419	Cross-brace single Compact, UltraLight 70° SB34 ST36, silver grey		1
13	1437855-0442	Cross-brace single Compact, UltraLight 70° SB34 ST36, night shimmer	2009-01-01 →	1
13	1437855-3524	Cross-brace single Compact, UltraLight 70° SB34 ST36, royal red		1
13	1437855-5969	Cross-brace single Compact, UltraLight 70° SB34 ST36, royal blue		1
13	1437855-9005	Cross-brace single Compact, UltraLight 70° SB34 ST36, black, glossy		1
13	1437855-9119	Cross-brace single Compact, UltraLight 70° SB34 ST36, black, bronze		1
14	1437856-0419	Cross-brace single Compact, UltraLight 70° SB34 ST38, silver grey		1
14	1437856-0442	Cross-brace single Compact, UltraLight 70° SB34 ST38, night shimmer	2009-01-01 →	1
14	1437856-3524	Cross-brace single Compact, UltraLight 70° SB34 ST38, royal red		1
14	1437856-5969	Cross-brace single Compact, UltraLight 70° SB34 ST38, royal blue		1
14	1437856-9005	Cross-brace single Compact, UltraLight 70° SB34 ST38, black, glossy		1
14	1437856-9119	Cross-brace single Compact, UltraLight 70° SB34 ST38, black, bronze		1
15	1437857-0419	Cross-brace single Compact, UltraLight 70° SB34 ST40, silver grey		1
15	1437857-0442	Cross-brace single Compact, UltraLight 70° SB34 ST40, night shimmer	2009-01-01 →	1

**Cross brace to frame standard, SB32-40**

Pos.	Part number	Description	Valid from → until	Qty
15	1437857-3524	Cross-brace single Compact, UltraLight 70° SB34 ST40, royal red		1
15	1437857-5969	Cross-brace single Compact, UltraLight 70° SB34 ST40, royal blue		1
15	1437857-9005	Cross-brace single Compact, UltraLight 70° SB34 ST40, black, glossy		1
15	1437857-9119	Cross-brace single Compact, UltraLight 70° SB34 ST40, black, bronze		1
16	1437858-0419	Cross-brace single Compact, UltraLight 70° SB34 ST42, silver grey		1
16	1437858-0442	Cross-brace single Compact, UltraLight 70° SB34 ST42, night shimmer	2009-01-01 →	1
16	1437858-3524	Cross-brace single Compact, UltraLight 70° SB34 ST42, royal red		1
16	1437858-5969	Cross-brace single Compact, UltraLight 70° SB34 ST42, royal blue		1
16	1437858-9005	Cross-brace single Compact, UltraLight 70° SB34 ST42, black, glossy		1
16	1437858-9119	Cross-brace single Compact, UltraLight 70° SB34 ST42, black, bronze		1
17	1437859-0419	Cross-brace single Compact, UltraLight 70° SB34 ST44, silver grey		1
17	1437859-0442	Cross-brace single Compact, UltraLight 70° SB34 ST44, night shimmer	2009-01-01 →	1
17	1437859-3524	Cross-brace single Compact, UltraLight 70° SB34 ST44, royal red		1
17	1437859-5969	Cross-brace single Compact, UltraLight 70° SB34 ST44, royal blue		1
17	1437859-9005	Cross-brace single Compact, UltraLight 70° SB34 ST44, black, glossy		1
17	1437859-9119	Cross-brace single Compact, UltraLight 70° SB34 ST44, black, bronze		1
18	1437860-0419	Cross-brace single Compact, UltraLight 70° SB34 ST46, silver grey		1
18	1437860-0442	Cross-brace single Compact, UltraLight 70° SB34 ST46, night shimmer	2009-01-01 →	1
18	1437860-3524	Cross-brace single Compact, UltraLight 70° SB34 ST46, royal red		1
18	1437860-5969	Cross-brace single Compact, UltraLight 70° SB34 ST46, royal blue		1
18	1437860-9005	Cross-brace single Compact, UltraLight 70° SB34 ST46, black, glossy		1
18	1437860-9119	Cross-brace single Compact, UltraLight 70° SB34 ST46, black, bronze		1
19	1437861-0419	Cross-brace single Compact, UltraLight 70° SB34 ST48, silver grey		1
19	1437861-0442	Cross-brace single Compact, UltraLight 70° SB34 ST48, night shimmer	2009-01-01 →	1
19	1437861-3524	Cross-brace single Compact, UltraLight 70° SB34 ST48, royal red		1
19	1437861-5969	Cross-brace single Compact, UltraLight 70° SB34 ST48, royal blue		1
19	1437861-9005	Cross-brace single Compact, UltraLight 70° SB34 ST48, black, glossy		1
19	1437861-9119	Cross-brace single Compact, UltraLight 70° SB34 ST48, black, bronze		1
20	1437862-0419	Cross-brace single Compact, UltraLight 70° SB34 ST50, silver grey		1
20	1437862-0442	Cross-brace single Compact, UltraLight 70° SB34 ST50, night shimmer	2009-01-01 →	1
20	1437862-3524	Cross-brace single Compact, UltraLight 70° SB34 ST50, royal red		1
20	1437862-5969	Cross-brace single Compact, UltraLight 70° SB34 ST50, royal blue		1
20	1437862-9005	Cross-brace single Compact, UltraLight 70° SB34 ST50, black, glossy		1
20	1437862-9119	Cross-brace single Compact, UltraLight 70° SB34 ST50, black, bronze		1
21	1437863-0419	Cross-brace single Compact, UltraLight 70° SB36 ST32, silver grey		1
21	1437863-0442	Cross-brace single Compact, UltraLight 70° SB36 ST32, night shimmer	2009-01-01 →	1
21	1437863-3524	Cross-brace single Compact, UltraLight 70° SB36 ST32, royal red		1
21	1437863-5969	Cross-brace single Compact, UltraLight 70° SB36 ST32, royal blue		1
21	1437863-9005	Cross-brace single Compact, UltraLight 70° SB36 ST32, black, glossy		1
21	1437863-9119	Cross-brace single Compact, UltraLight 70° SB36 ST32, black, bronze		1
22	1437864-0419	Cross-brace single Compact, UltraLight 70° SB36 ST34, silver grey		1
22	1437864-0442	Cross-brace single Compact, UltraLight 70° SB36 ST34, night shimmer	2009-01-01 →	1
22	1437864-3524	Cross-brace single Compact, UltraLight 70° SB36 ST34, royal red		1
22	1437864-5969	Cross-brace single Compact, UltraLight 70° SB36 ST34, royal blue		1
22	1437864-9005	Cross-brace single Compact, UltraLight 70° SB36 ST34, black, glossy		1
22	1437864-9119	Cross-brace single Compact, UltraLight 70° SB36 ST34, black, bronze		1
23	1437865-0419	Cross-brace single Compact, UltraLight 70° SB36 ST36, silver grey		1
23	1437865-0442	Cross-brace single Compact, UltraLight 70° SB36 ST36, night shimmer	2009-01-01 →	1
23	1437865-3524	Cross-brace single Compact, UltraLight 70° SB36 ST36, royal red		1
23	1437865-5969	Cross-brace single Compact, UltraLight 70° SB36 ST36, royal blue		1
23	1437865-9005	Cross-brace single Compact, UltraLight 70° SB36 ST36, black, glossy		1
23	1437865-9119	Cross-brace single Compact, UltraLight 70° SB36 ST36, black, bronze		1
24	1437866-0419	Cross-brace single Compact, UltraLight 70° SB36 ST38, silver grey		1
24	1437866-0442	Cross-brace single Compact, UltraLight 70° SB36 ST38, night shimmer	2009-01-01 →	1
24	1437866-3524	Cross-brace single Compact, UltraLight 70° SB36 ST38, royal red		1



**Cross brace to frame standard, SB32-40**

Pos.	Part number	Description	Valid from → until	Qty
24	1437866-5969	Cross-brace single Compact, UltraLight 70° SB36 ST38, royal blue		1
24	1437866-9005	Cross-brace single Compact, UltraLight 70° SB36 ST38, black, glossy		1
24	1437866-9119	Cross-brace single Compact, UltraLight 70° SB36 ST38, black, bronze		1
25	1437867-0419	Cross-brace single Compact, UltraLight 70° SB36 ST40, silver grey		1
25	1437867-0442	Cross-brace single Compact, UltraLight 70° SB36 ST40, night shimmer	2009-01-01 →	1
25	1437867-3524	Cross-brace single Compact, UltraLight 70° SB36 ST40, royal red		1
25	1437867-5969	Cross-brace single Compact, UltraLight 70° SB36 ST40, royal blue		1
25	1437867-9005	Cross-brace single Compact, UltraLight 70° SB36 ST40, black, glossy		1
25	1437867-9119	Cross-brace single Compact, UltraLight 70° SB36 ST40, black, bronze		1
26	1437868-0419	Cross-brace single Compact, UltraLight 70° SB36 ST42, silver grey		1
26	1437868-0442	Cross-brace single Compact, UltraLight 70° SB36 ST42, night shimmer	2009-01-01 →	1
26	1437868-3524	Cross-brace single Compact, UltraLight 70° SB36 ST42, royal red		1
26	1437868-5969	Cross-brace single Compact, UltraLight 70° SB36 ST42, royal blue		1
26	1437868-9005	Cross-brace single Compact, UltraLight 70° SB36 ST42, black, glossy		1
26	1437868-9119	Cross-brace single Compact, UltraLight 70° SB36 ST42, black, bronze		1
27	1437869-0419	Cross-brace single Compact, UltraLight 70° SB36 ST44, silver grey		1
27	1437869-0442	Cross-brace single Compact, UltraLight 70° SB36 ST44, night shimmer	2009-01-01 →	1
27	1437869-3524	Cross-brace single Compact, UltraLight 70° SB36 ST44, royal red		1
27	1437869-5969	Cross-brace single Compact, UltraLight 70° SB36 ST44, royal blue		1
27	1437869-9005	Cross-brace single Compact, UltraLight 70° SB36 ST44, black, glossy		1
27	1437869-9119	Cross-brace single Compact, UltraLight 70° SB36 ST44, black, bronze		1
28	1437870-0419	Cross-brace single Compact, UltraLight 70° SB36 ST46, silver grey		1
28	1437870-0442	Cross-brace single Compact, UltraLight 70° SB36 ST46, night shimmer	2009-01-01 →	1
28	1437870-3524	Cross-brace single Compact, UltraLight 70° SB36 ST46, royal red		1
28	1437870-5969	Cross-brace single Compact, UltraLight 70° SB36 ST46, royal blue		1
28	1437870-9005	Cross-brace single Compact, UltraLight 70° SB36 ST46, black, glossy		1
28	1437870-9119	Cross-brace single Compact, UltraLight 70° SB36 ST46, black, bronze		1
29	1437871-0419	Cross-brace single Compact, UltraLight 70° SB36 ST48, silver grey		1
29	1437871-0442	Cross-brace single Compact, UltraLight 70° SB36 ST48, night shimmer	2009-01-01 →	1
29	1437871-3524	Cross-brace single Compact, UltraLight 70° SB36 ST48, royal red		1
29	1437871-5969	Cross-brace single Compact, UltraLight 70° SB36 ST48, royal blue		1
29	1437871-9005	Cross-brace single Compact, UltraLight 70° SB36 ST48, black, glossy		1
29	1437871-9119	Cross-brace single Compact, UltraLight 70° SB36 ST48, black, bronze		1
30	1437872-0419	Cross-brace single Compact, UltraLight 70° SB36 ST50, silver grey		1
30	1437872-0442	Cross-brace single Compact, UltraLight 70° SB36 ST50, night shimmer	2009-01-01 →	1
30	1437872-3524	Cross-brace single Compact, UltraLight 70° SB36 ST50, royal red		1
30	1437872-5969	Cross-brace single Compact, UltraLight 70° SB36 ST50, royal blue		1
30	1437872-9005	Cross-brace single Compact, UltraLight 70° SB36 ST50, black, glossy		1
30	1437872-9119	Cross-brace single Compact, UltraLight 70° SB36 ST50, black, bronze		1
31	1437873-0419	Cross-brace single Compact, UltraLight 70° SB38 ST32, silver grey		1
31	1437873-0442	Cross-brace single Compact, UltraLight 70° SB38 ST32, night shimmer	2009-01-01 →	1
31	1437873-3524	Cross-brace single Compact, UltraLight 70° SB38 ST32, royal red		1
31	1437873-5969	Cross-brace single Compact, UltraLight 70° SB38 ST32, royal blue		1
31	1437873-9005	Cross-brace single Compact, UltraLight 70° SB38 ST32, black, glossy		1
31	1437873-9119	Cross-brace single Compact, UltraLight 70° SB38 ST32, black, bronze		1
32	1437874-0419	Cross-brace single Compact, UltraLight 70° SB38 ST34, silver grey		1
32	1437874-0442	Cross-brace single Compact, UltraLight 70° SB38 ST34, night shimmer	2009-01-01 →	1
32	1437874-3524	Cross-brace single Compact, UltraLight 70° SB38 ST34, royal red		1
32	1437874-5969	Cross-brace single Compact, UltraLight 70° SB38 ST34, royal blue		1
32	1437874-9005	Cross-brace single Compact, UltraLight 70° SB38 ST34, black, glossy		1
32	1437874-9119	Cross-brace single Compact, UltraLight 70° SB38 ST34, black, bronze		1
33	1437875-0419	Cross-brace single Compact, UltraLight 70° SB38 ST36, silver grey		1
33	1437875-0442	Cross-brace single Compact, UltraLight 70° SB38 ST36, night shimmer	2009-01-01 →	1
33	1437875-3524	Cross-brace single Compact, UltraLight 70° SB38 ST36, royal red		1
33	1437875-5969	Cross-brace single Compact, UltraLight 70° SB38 ST36, royal blue		1

**Cross brace to frame standard, SB32-40**


Pos.	Part number	Description	Valid from → until	Qty
33	1437875-9005	Cross-brace single Compact, UltraLight 70° SB38 ST36, black, glossy		1
33	1437875-9119	Cross-brace single Compact, UltraLight 70° SB38 ST36, black, bronze		1
34	1437876-0419	Cross-brace single Compact, UltraLight 70° SB38 ST38, silver grey		1
34	1437876-0442	Cross-brace single Compact, UltraLight 70° SB38 ST38, night shimmer	2009-01-01 →	1
34	1437876-3524	Cross-brace single Compact, UltraLight 70° SB38 ST38, royal red		1
34	1437876-5969	Cross-brace single Compact, UltraLight 70° SB38 ST38, royal blue		1
34	1437876-9005	Cross-brace single Compact, UltraLight 70° SB38 ST38, black, glossy		1
34	1437876-9119	Cross-brace single Compact, UltraLight 70° SB38 ST38, black, bronze		1
35	1437877-0419	Cross-brace single Compact, UltraLight 70° SB38 ST40, silver grey		1
35	1437877-0442	Cross-brace single Compact, UltraLight 70° SB38 ST40, night shimmer	2009-01-01 →	1
35	1437877-3524	Cross-brace single Compact, UltraLight 70° SB38 ST40, royal red		1
35	1437877-5969	Cross-brace single Compact, UltraLight 70° SB38 ST40, royal blue		1
35	1437877-9005	Cross-brace single Compact, UltraLight 70° SB38 ST40, black, glossy		1
35	1437877-9119	Cross-brace single Compact, UltraLight 70° SB38 ST40, black, bronze		1
36	1437878-0419	Cross-brace single Compact, UltraLight 70° SB38 ST42, silver grey		1
36	1437878-0442	Cross-brace single Compact, UltraLight 70° SB38 ST42, night shimmer	2009-01-01 →	1
36	1437878-3524	Cross-brace single Compact, UltraLight 70° SB38 ST42, royal red		1
36	1437878-5969	Cross-brace single Compact, UltraLight 70° SB38 ST42, royal blue		1
36	1437878-9005	Cross-brace single Compact, UltraLight 70° SB38 ST42, black, glossy		1
36	1437878-9119	Cross-brace single Compact, UltraLight 70° SB38 ST42, black, bronze		1
37	1437879-0419	Cross-brace single Compact, UltraLight 70° SB38 ST44, silver grey		1
37	1437879-0442	Cross-brace single Compact, UltraLight 70° SB38 ST44, night shimmer	2009-01-01 →	1
37	1437879-3524	Cross-brace single Compact, UltraLight 70° SB38 ST44, royal red		1
37	1437879-5969	Cross-brace single Compact, UltraLight 70° SB38 ST44, royal blue		1
37	1437879-9005	Cross-brace single Compact, UltraLight 70° SB38 ST44, black, glossy		1
37	1437879-9119	Cross-brace single Compact, UltraLight 70° SB38 ST44, black, bronze		1
38	1437880-0419	Cross-brace single Compact, UltraLight 70° SB38 ST46, silver grey		1
38	1437880-0442	Cross-brace single Compact, UltraLight 70° SB38 ST46, night shimmer	2009-01-01 →	1
38	1437880-3524	Cross-brace single Compact, UltraLight 70° SB38 ST46, royal red		1
38	1437880-5969	Cross-brace single Compact, UltraLight 70° SB38 ST46, royal blue		1
38	1437880-9005	Cross-brace single Compact, UltraLight 70° SB38 ST46, black, glossy		1
38	1437880-9119	Cross-brace single Compact, UltraLight 70° SB38 ST46, black, bronze		1
39	1437881-0419	Cross-brace single Compact, UltraLight 70° SB38 ST48, silver grey		1
39	1437881-0442	Cross-brace single Compact, UltraLight 70° SB38 ST48, night shimmer	2009-01-01 →	1
39	1437881-3524	Cross-brace single Compact, UltraLight 70° SB38 ST48, royal red		1
39	1437881-5969	Cross-brace single Compact, UltraLight 70° SB38 ST48, royal blue		1
39	1437881-9005	Cross-brace single Compact, UltraLight 70° SB38 ST48, black, glossy		1
39	1437881-9119	Cross-brace single Compact, UltraLight 70° SB38 ST48, black, bronze		1
40	1437882-0419	Cross-brace single Compact, UltraLight 70° SB38 ST50, silver grey		1
40	1437882-0442	Cross-brace single Compact, UltraLight 70° SB38 ST50, night shimmer	2009-01-01 →	1
40	1437882-3524	Cross-brace single Compact, UltraLight 70° SB38 ST50, royal red		1
40	1437882-5969	Cross-brace single Compact, UltraLight 70° SB38 ST50, royal blue		1
40	1437882-9005	Cross-brace single Compact, UltraLight 70° SB38 ST50, black, glossy		1
40	1437882-9119	Cross-brace single Compact, UltraLight 70° SB38 ST50, black, bronze		1
41	1437883-0419	Cross-brace single Compact, UltraLight 70° SB40 ST32, silver grey		1
41	1437883-0442	Cross-brace single Compact, UltraLight 70° SB40 ST32, night shimmer	2009-01-01 →	1
41	1437883-3524	Cross-brace single Compact, UltraLight 70° SB40 ST32, royal red		1
41	1437883-5969	Cross-brace single Compact, UltraLight 70° SB40 ST32, royal blue		1
41	1437883-9005	Cross-brace single Compact, UltraLight 70° SB40 ST32, black, glossy		1
41	1437883-9119	Cross-brace single Compact, UltraLight 70° SB40 ST32, black, bronze		1
42	1437884-0419	Cross-brace single Compact, UltraLight 70° SB40 ST34, silver grey		1
42	1437884-0442	Cross-brace single Compact, UltraLight 70° SB40 ST34, night shimmer	2009-01-01 →	1
42	1437884-3524	Cross-brace single Compact, UltraLight 70° SB40 ST34, royal red		1
42	1437884-5969	Cross-brace single Compact, UltraLight 70° SB40 ST34, royal blue		1
42	1437884-9005	Cross-brace single Compact, UltraLight 70° SB40 ST34, black, glossy		1

**Cross brace to frame standard, SB32-40**

Pos.	Part number	Description	Valid from → until	Qty
42	1437884-9119	Cross-brace single Compact, UltraLight 70° SB40 ST34, black, bronze		1
43	1437885-0419	Cross-brace single Compact, UltraLight 70° SB40 ST36, silver grey		1
43	1437885-0442	Cross-brace single Compact, UltraLight 70° SB40 ST36, night shimmer	2009-01-01 →	1
43	1437885-3524	Cross-brace single Compact, UltraLight 70° SB40 ST36, royal red		1
43	1437885-5969	Cross-brace single Compact, UltraLight 70° SB40 ST36, royal blue		1
43	1437885-9005	Cross-brace single Compact, UltraLight 70° SB40 ST36, black, glossy		1
43	1437885-9119	Cross-brace single Compact, UltraLight 70° SB40 ST36, black, bronze		1
44	1437886-0419	Cross-brace single Compact, UltraLight 70° SB40 ST38, silver grey		1
44	1437886-0442	Cross-brace single Compact, UltraLight 70° SB40 ST38, night shimmer	2009-01-01 →	1
44	1437886-3524	Cross-brace single Compact, UltraLight 70° SB40 ST38, royal red		1
44	1437886-5969	Cross-brace single Compact, UltraLight 70° SB40 ST38, royal blue		1
44	1437886-9005	Cross-brace single Compact, UltraLight 70° SB40 ST38, black, glossy		1
44	1437886-9119	Cross-brace single Compact, UltraLight 70° SB40 ST38, black, bronze		1
45	1437887-0419	Cross-brace single Compact, UltraLight 70° SB40 ST40, silver grey		1
45	1437887-0442	Cross-brace single Compact, UltraLight 70° SB40 ST40, night shimmer	2009-01-01 →	1
45	1437887-3524	Cross-brace single Compact, UltraLight 70° SB40 ST40, royal red		1
45	1437887-5969	Cross-brace single Compact, UltraLight 70° SB40 ST40, royal blue		1
45	1437887-9005	Cross-brace single Compact, UltraLight 70° SB40 ST40, black, glossy		1
45	1437887-9119	Cross-brace single Compact, UltraLight 70° SB40 ST40, black, bronze		1
46	1437888-0419	Cross-brace single Compact, UltraLight 70° SB40 ST42, silver grey		1
46	1437888-0442	Cross-brace single Compact, UltraLight 70° SB40 ST42, night shimmer	2009-01-01 →	1
46	1437888-3524	Cross-brace single Compact, UltraLight 70° SB40 ST42, royal red		1
46	1437888-5969	Cross-brace single Compact, UltraLight 70° SB40 ST42, royal blue		1
46	1437888-9005	Cross-brace single Compact, UltraLight 70° SB40 ST42, black, glossy		1
46	1437888-9119	Cross-brace single Compact, UltraLight 70° SB40 ST42, black, bronze		1
47	1437889-0419	Cross-brace single Compact, UltraLight 70° SB40 ST44, silver grey		1
47	1437889-0442	Cross-brace single Compact, UltraLight 70° SB40 ST44, night shimmer	2009-01-01 →	1
47	1437889-3524	Cross-brace single Compact, UltraLight 70° SB40 ST44, royal red		1
47	1437889-5969	Cross-brace single Compact, UltraLight 70° SB40 ST44, royal blue		1
47	1437889-9005	Cross-brace single Compact, UltraLight 70° SB40 ST44, black, glossy		1
47	1437889-9119	Cross-brace single Compact, UltraLight 70° SB40 ST44, black, bronze		1
48	1437890-0419	Cross-brace single Compact, UltraLight 70° SB40 ST46, silver grey		1
48	1437890-0442	Cross-brace single Compact, UltraLight 70° SB40 ST46, night shimmer	2009-01-01 →	1
48	1437890-3524	Cross-brace single Compact, UltraLight 70° SB40 ST46, royal red		1
48	1437890-5969	Cross-brace single Compact, UltraLight 70° SB40 ST46, royal blue		1
48	1437890-9005	Cross-brace single Compact, UltraLight 70° SB40 ST46, black, glossy		1
48	1437890-9119	Cross-brace single Compact, UltraLight 70° SB40 ST46, black, bronze		1
49	1437891-0419	Cross-brace single Compact, UltraLight 70° SB40 ST48, silver grey		1
49	1437891-0442	Cross-brace single Compact, UltraLight 70° SB40 ST48, night shimmer	2009-01-01 →	1
49	1437891-3524	Cross-brace single Compact, UltraLight 70° SB40 ST48, royal red		1
49	1437891-5969	Cross-brace single Compact, UltraLight 70° SB40 ST48, royal blue		1
49	1437891-9005	Cross-brace single Compact, UltraLight 70° SB40 ST48, black, glossy		1
49	1437891-9119	Cross-brace single Compact, UltraLight 70° SB40 ST48, black, bronze		1
50	1437892-0419	Cross-brace single Compact, UltraLight 70° SB40 ST50, silver grey		1
50	1437892-0442	Cross-brace single Compact, UltraLight 70° SB40 ST50, night shimmer	2009-01-01 →	1
50	1437892-3524	Cross-brace single Compact, UltraLight 70° SB40 ST50, royal red		1
50	1437892-5969	Cross-brace single Compact, UltraLight 70° SB40 ST50, royal blue		1
50	1437892-9005	Cross-brace single Compact, UltraLight 70° SB40 ST50, black, glossy		1
50	1437892-9119	Cross-brace single Compact, UltraLight 70° SB40 ST50, black, bronze		1
51	1437943	Retaining lever SB32-38		1
52	1437944	Retaining lever SB40-44		1
53	1507625	Center division kit		1
54	1439479	Screw kit to Single crossbrace		1



## Cross brace to frame standard, SB42-50

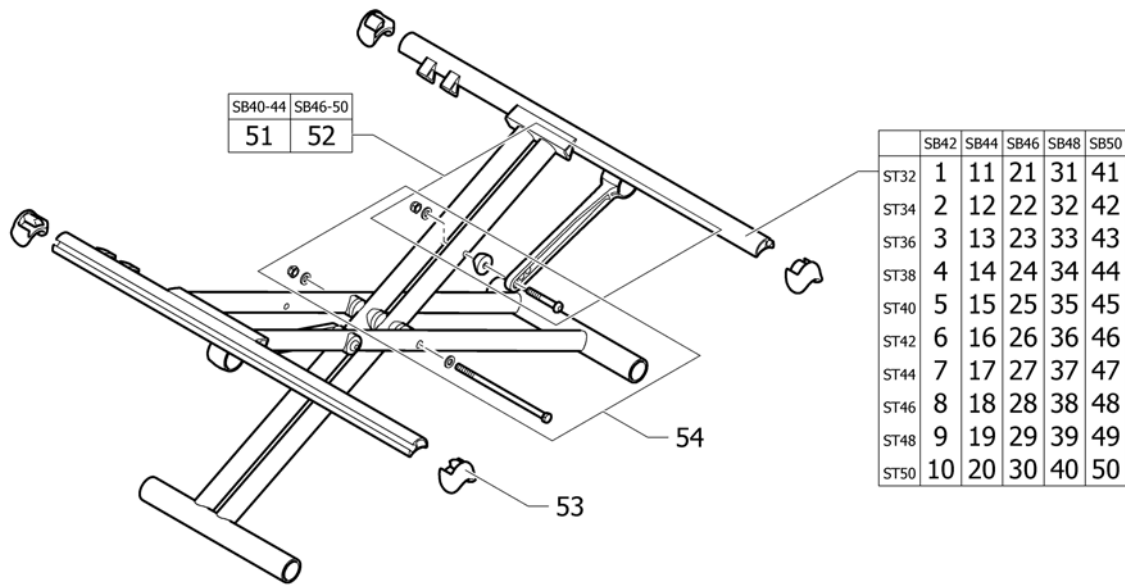

Pos.	Part number	Description	Valid from → until	Qty
1	1498789-0419	Cross-brace double Compact, UltraLight 70° SB42 ST32, silver grey		1
1	1498789-0442	Cross-brace double Compact, UltraLight 70° SB42 ST32, night shimmer	2009-01-01 →	1
1	1498789-3524	Cross-brace double Compact, UltraLight 70° SB42 ST32, royal red		1
1	1498789-5969	Cross-brace double Compact, UltraLight 70° SB42 ST32, royal blue		1
1	1498789-9005	Cross-brace double Compact, UltraLight 70° SB42 ST32, black, glossy		1
1	1498789-9119	Cross-brace double Compact, UltraLight 70° SB42 ST32, black, bronze		1
2	1498790-0419	Cross-brace double Compact, UltraLight 70° SB42 ST34, silver grey		1
2	1498790-0442	Cross-brace double Compact, UltraLight 70° SB42 ST34, night shimmer	2009-01-01 →	1
2	1498790-3524	Cross-brace double Compact, UltraLight 70° SB42 ST34, royal red		1
2	1498790-5969	Cross-brace double Compact, UltraLight 70° SB42 ST34, royal blue		1
2	1498790-9005	Cross-brace double Compact, UltraLight 70° SB42 ST34, black, glossy		1
2	1498790-9119	Cross-brace double Compact, UltraLight 70° SB42 ST34, black, bronze		1
3	1498791-0419	Cross-brace double Compact, UltraLight 70° SB42 ST36, silver grey		1
3	1498791-0442	Cross-brace double Compact, UltraLight 70° SB42 ST36, night shimmer	2009-01-01 →	1
3	1498791-3524	Cross-brace double Compact, UltraLight 70° SB42 ST36, royal red		1
3	1498791-5969	Cross-brace double Compact, UltraLight 70° SB42 ST36, royal blue		1
3	1498791-9005	Cross-brace double Compact, UltraLight 70° SB42 ST36, black, glossy		1
3	1498791-9119	Cross-brace double Compact, UltraLight 70° SB42 ST36, black, bronze		1
4	1498792-0419	Cross-brace double Compact, UltraLight 70° SB42 ST38, silver grey		1
4	1498792-0442	Cross-brace double Compact, UltraLight 70° SB42 ST38, night shimmer	2009-01-01 →	1
4	1498792-3524	Cross-brace double Compact, UltraLight 70° SB42 ST38, royal red		1
4	1498792-5969	Cross-brace double Compact, UltraLight 70° SB42 ST38, royal blue		1
4	1498792-9005	Cross-brace double Compact, UltraLight 70° SB42 ST38, black, glossy		1
4	1498792-9119	Cross-brace double Compact, UltraLight 70° SB42 ST38, black, bronze		1
5	1498793-0419	Cross-brace double Compact, UltraLight 70° SB42 ST40, silver grey		1
5	1498793-0442	Cross-brace double Compact, UltraLight 70° SB42 ST40, night shimmer	2009-01-01 →	1
5	1498793-3524	Cross-brace double Compact, UltraLight 70° SB42 ST40, royal red		1
5	1498793-5969	Cross-brace double Compact, UltraLight 70° SB42 ST40, royal blue		1
5	1498793-9005	Cross-brace double Compact, UltraLight 70° SB42 ST40, black, glossy		1
5	1498793-9119	Cross-brace double Compact, UltraLight 70° SB42 ST40, black, bronze		1
6	1498794-0419	Cross-brace double Compact, UltraLight 70° SB42 ST42, silver grey		1

**Cross brace to frame standard, SB42-50**

Pos.	Part number	Description	Valid from → until	Qty
6	1498794-0442	Cross-brace double Compact, UltraLight 70° SB42 ST42, night shimmer	2009-01-01 →	1
6	1498794-3524	Cross-brace double Compact, UltraLight 70° SB42 ST42, royal red		1
6	1498794-5969	Cross-brace double Compact, UltraLight 70° SB42 ST42, royal blue		1
6	1498794-9005	Cross-brace double Compact, UltraLight 70° SB42 ST42, black, glossy		1
6	1498794-9119	Cross-brace double Compact, UltraLight 70° SB42 ST42, black, bronze		1
7	1498795-0419	Cross-brace double Compact, UltraLight 70° SB42 ST44, silver grey		1
7	1498795-0442	Cross-brace double Compact, UltraLight 70° SB42 ST44, night shimmer	2009-01-01 →	1
7	1498795-3524	Cross-brace double Compact, UltraLight 70° SB42 ST44, royal red		1
7	1498795-5969	Cross-brace double Compact, UltraLight 70° SB42 ST44, royal blue		1
7	1498795-9005	Cross-brace double Compact, UltraLight 70° SB42 ST44, black, glossy		1
7	1498795-9119	Cross-brace double Compact, UltraLight 70° SB42 ST44, black, bronze		1
8	1498796-0419	Cross-brace double Compact, UltraLight 70° SB42 ST46, silver grey		1
8	1498796-0442	Cross-brace double Compact, UltraLight 70° SB42 ST46, night shimmer	2009-01-01 →	1
8	1498796-3524	Cross-brace double Compact, UltraLight 70° SB42 ST46, royal red		1
8	1498796-5969	Cross-brace double Compact, UltraLight 70° SB42 ST46, royal blue		1
8	1498796-9005	Cross-brace double Compact, UltraLight 70° SB42 ST46, black, glossy		1
8	1498796-9119	Cross-brace double Compact, UltraLight 70° SB42 ST46, black, bronze		1
9	1498797-0419	Cross-brace double Compact, UltraLight 70° SB42 ST48, silver grey		1
9	1498797-0442	Cross-brace double Compact, UltraLight 70° SB42 ST48, night shimmer	2009-01-01 →	1
9	1498797-3524	Cross-brace double Compact, UltraLight 70° SB42 ST48, royal red		1
9	1498797-5969	Cross-brace double Compact, UltraLight 70° SB42 ST48, royal blue		1
9	1498797-9005	Cross-brace double Compact, UltraLight 70° SB42 ST48, black, glossy		1
9	1498797-9119	Cross-brace double Compact, UltraLight 70° SB42 ST48, black, bronze		1
10	1498798-0419	Cross-brace double Compact, UltraLight 70° SB42 ST50, silver grey		1
10	1498798-0442	Cross-brace double Compact, UltraLight 70° SB42 ST50, night shimmer	2009-01-01 →	1
10	1498798-3524	Cross-brace double Compact, UltraLight 70° SB42 ST50, royal red		1
10	1498798-5969	Cross-brace double Compact, UltraLight 70° SB42 ST50, royal blue		1
10	1498798-9005	Cross-brace double Compact, UltraLight 70° SB42 ST50, black, glossy		1
10	1498798-9119	Cross-brace double Compact, UltraLight 70° SB42 ST50, black, bronze		1
11	1437903-0419	Cross-brace double Compact, UltraLight 70° SB44 ST32, silver grey		1
11	1437903-0442	Cross-brace double Compact, UltraLight 70° SB44 ST32, night shimmer	2009-01-01 →	1
11	1437903-3524	Cross-brace double Compact, UltraLight 70° SB44 ST32, royal red		1
11	1437903-5969	Cross-brace double Compact, UltraLight 70° SB44 ST32, royal blue		1
11	1437903-9005	Cross-brace double Compact, UltraLight 70° SB44 ST32, black, glossy		1
11	1437903-9119	Cross-brace double Compact, UltraLight 70° SB44 ST32, black, bronze		1
12	1437904-0419	Cross-brace double Compact, UltraLight 70° SB44 ST34, silver grey		1
12	1437904-0442	Cross-brace double Compact, UltraLight 70° SB44 ST34, night shimmer	2009-01-01 →	1
12	1437904-3524	Cross-brace double Compact, UltraLight 70° SB44 ST34, royal red		1
12	1437904-5969	Cross-brace double Compact, UltraLight 70° SB44 ST34, royal blue		1
12	1437904-9005	Cross-brace double Compact, UltraLight 70° SB44 ST34, black, glossy		1
12	1437904-9119	Cross-brace double Compact, UltraLight 70° SB44 ST34, black, bronze		1
13	1437905-0419	Cross-brace double Compact, UltraLight 70° SB44 ST36, silver grey		1
13	1437905-0442	Cross-brace double Compact, UltraLight 70° SB44 ST36, night shimmer	2009-01-01 →	1
13	1437905-3524	Cross-brace double Compact, UltraLight 70° SB44 ST36, royal red		1
13	1437905-5969	Cross-brace double Compact, UltraLight 70° SB44 ST36, royal blue		1
13	1437905-9005	Cross-brace double Compact, UltraLight 70° SB44 ST36, black, glossy		1
13	1437905-9119	Cross-brace double Compact, UltraLight 70° SB44 ST36, black, bronze		1
14	1437906-0419	Cross-brace double Compact, UltraLight 70° SB44 ST38, silver grey		1
14	1437906-0442	Cross-brace double Compact, UltraLight 70° SB44 ST38, night shimmer	2009-01-01 →	1
14	1437906-3524	Cross-brace double Compact, UltraLight 70° SB44 ST38, royal red		1
14	1437906-5969	Cross-brace double Compact, UltraLight 70° SB44 ST38, royal blue		1
14	1437906-9005	Cross-brace double Compact, UltraLight 70° SB44 ST38, black, glossy		1
14	1437906-9119	Cross-brace double Compact, UltraLight 70° SB44 ST38, black, bronze		1
15	1437907-0419	Cross-brace double Compact, UltraLight 70° SB44 ST40, silver grey		1
15	1437907-0442	Cross-brace double Compact, UltraLight 70° SB44 ST40, night shimmer	2009-01-01 →	1

**Cross brace to frame standard, SB42-50**

Pos.	Part number	Description	Valid from → until	Qty
15	1437907-3524	Cross-brace double Compact, UltraLight 70° SB44 ST40, royal red		1
15	1437907-5969	Cross-brace double Compact, UltraLight 70° SB44 ST40, royal blue		1
15	1437907-9005	Cross-brace double Compact, UltraLight 70° SB44 ST40, black, glossy		1
15	1437907-9119	Cross-brace double Compact, UltraLight 70° SB44 ST40, black, bronze		1
16	1437908-0419	Cross-brace double Compact, UltraLight 70° SB44 ST42, silver grey		1
16	1437908-0442	Cross-brace double Compact, UltraLight 70° SB44 ST42, night shimmer	2009-01-01 →	1
16	1437908-3524	Cross-brace double Compact, UltraLight 70° SB44 ST42, royal red		1
16	1437908-5969	Cross-brace double Compact, UltraLight 70° SB44 ST42, royal blue		1
16	1437908-9005	Cross-brace double Compact, UltraLight 70° SB44 ST42, black, glossy		1
16	1437908-9119	Cross-brace double Compact, UltraLight 70° SB44 ST42, black, bronze		1
17	1437909-0419	Cross-brace double Compact, UltraLight 70° SB44 ST44, silver grey		1
17	1437909-0442	Cross-brace double Compact, UltraLight 70° SB44 ST44, night shimmer	2009-01-01 →	1
17	1437909-3524	Cross-brace double Compact, UltraLight 70° SB44 ST44, royal red		1
17	1437909-5969	Cross-brace double Compact, UltraLight 70° SB44 ST44, royal blue		1
17	1437909-9005	Cross-brace double Compact, UltraLight 70° SB44 ST44, black, glossy		1
17	1437909-9119	Cross-brace double Compact, UltraLight 70° SB44 ST44, black, bronze		1
18	1437910-0419	Cross-brace double Compact, UltraLight 70° SB44 ST46, silver grey		1
18	1437910-0442	Cross-brace double Compact, UltraLight 70° SB44 ST46, night shimmer	2009-01-01 →	1
18	1437910-3524	Cross-brace double Compact, UltraLight 70° SB44 ST46, royal red		1
18	1437910-5969	Cross-brace double Compact, UltraLight 70° SB44 ST46, royal blue		1
18	1437910-9005	Cross-brace double Compact, UltraLight 70° SB44 ST46, black, glossy		1
18	1437910-9119	Cross-brace double Compact, UltraLight 70° SB44 ST46, black, bronze		1
19	1437911-0419	Cross-brace double Compact, UltraLight 70° SB44 ST48, silver grey		1
19	1437911-0442	Cross-brace double Compact, UltraLight 70° SB44 ST48, night shimmer	2009-01-01 →	1
19	1437911-3524	Cross-brace double Compact, UltraLight 70° SB44 ST48, royal red		1
19	1437911-5969	Cross-brace double Compact, UltraLight 70° SB44 ST48, royal blue		1
19	1437911-9005	Cross-brace double Compact, UltraLight 70° SB44 ST48, black, glossy		1
19	1437911-9119	Cross-brace double Compact, UltraLight 70° SB44 ST48, black, bronze		1
20	1437912-0419	Cross-brace double Compact, UltraLight 70° SB44 ST50, silver grey		1
20	1437912-0442	Cross-brace double Compact, UltraLight 70° SB44 ST50, night shimmer	2009-01-01 →	1
20	1437912-3524	Cross-brace double Compact, UltraLight 70° SB44 ST50, royal red		1
20	1437912-5969	Cross-brace double Compact, UltraLight 70° SB44 ST50, royal blue		1
20	1437912-9005	Cross-brace double Compact, UltraLight 70° SB44 ST50, black, glossy		1
20	1437912-9119	Cross-brace double Compact, UltraLight 70° SB44 ST50, black, bronze		1
21	1437913-0419	Cross-brace double Compact, UltraLight 70° SB46 ST32, silver grey		1
21	1437913-0442	Cross-brace double Compact, UltraLight 70° SB46 ST32, night shimmer	2009-01-01 →	1
21	1437913-3524	Cross-brace double Compact, UltraLight 70° SB46 ST32, royal red		1
21	1437913-5969	Cross-brace double Compact, UltraLight 70° SB46 ST32, royal blue		1
21	1437913-9005	Cross-brace double Compact, UltraLight 70° SB46 ST32, black, glossy		1
21	1437913-9119	Cross-brace double Compact, UltraLight 70° SB46 ST32, black, bronze		1
22	1437914-0419	Cross-brace double Compact, UltraLight 70° SB46 ST34, silver grey		1
22	1437914-0442	Cross-brace double Compact, UltraLight 70° SB46 ST34, night shimmer	2009-01-01 →	1
22	1437914-3524	Cross-brace double Compact, UltraLight 70° SB46 ST34, royal red		1
22	1437914-5969	Cross-brace double Compact, UltraLight 70° SB46 ST34, royal blue		1
22	1437914-9005	Cross-brace double Compact, UltraLight 70° SB46 ST34, black, glossy		1
22	1437914-9119	Cross-brace double Compact, UltraLight 70° SB46 ST34, black, bronze		1
23	1437915-0419	Cross-brace double Compact, UltraLight 70° SB46 ST36, silver grey		1
23	1437915-0442	Cross-brace double Compact, UltraLight 70° SB46 ST36, night shimmer	2009-01-01 →	1
23	1437915-3524	Cross-brace double Compact, UltraLight 70° SB46 ST36, royal red		1
23	1437915-5969	Cross-brace double Compact, UltraLight 70° SB46 ST36, royal blue		1
23	1437915-9005	Cross-brace double Compact, UltraLight 70° SB46 ST36, black, glossy		1
23	1437915-9119	Cross-brace double Compact, UltraLight 70° SB46 ST36, black, bronze		1
24	1437916-0419	Cross-brace double Compact, UltraLight 70° SB46 ST38, silver grey		1
24	1437916-0442	Cross-brace double Compact, UltraLight 70° SB46 ST38, night shimmer	2009-01-01 →	1
24	1437916-3524	Cross-brace double Compact, UltraLight 70° SB46 ST38, royal red		1

**Cross brace to frame standard, SB42-50**

Pos.	Part number	Description	Valid from → until	Qty
24	1437916-5969	Cross-brace double Compact, UltraLight 70° SB46 ST38, royal blue		1
24	1437916-9005	Cross-brace double Compact, UltraLight 70° SB46 ST38, black, glossy		1
24	1437916-9119	Cross-brace double Compact, UltraLight 70° SB46 ST38, black, bronze		1
25	1437917-0419	Cross-brace double Compact, UltraLight 70° SB46 ST40, silver grey		1
25	1437917-0442	Cross-brace double Compact, UltraLight 70° SB46 ST40, night shimmer	2009-01-01 →	1
25	1437917-3524	Cross-brace double Compact, UltraLight 70° SB46 ST40, royal red		1
25	1437917-5969	Cross-brace double Compact, UltraLight 70° SB46 ST40, royal blue		1
25	1437917-9005	Cross-brace double Compact, UltraLight 70° SB46 ST40, black, glossy		1
25	1437917-9119	Cross-brace double Compact, UltraLight 70° SB46 ST40, black, bronze		1
26	1437918-0419	Cross-brace double Compact, UltraLight 70° SB46 ST42, silver grey		1
26	1437918-0442	Cross-brace double Compact, UltraLight 70° SB46 ST42, night shimmer	2009-01-01 →	1
26	1437918-3524	Cross-brace double Compact, UltraLight 70° SB46 ST42, royal red		1
26	1437918-5969	Cross-brace double Compact, UltraLight 70° SB46 ST42, royal blue		1
26	1437918-9005	Cross-brace double Compact, UltraLight 70° SB46 ST42, black, glossy		1
26	1437918-9119	Cross-brace double Compact, UltraLight 70° SB46 ST42, black, bronze		1
27	1437919-0419	Cross-brace double Compact, UltraLight 70° SB46 ST44, silver grey		1
27	1437919-0442	Cross-brace double Compact, UltraLight 70° SB46 ST44, night shimmer	2009-01-01 →	1
27	1437919-3524	Cross-brace double Compact, UltraLight 70° SB46 ST44, royal red		1
27	1437919-5969	Cross-brace double Compact, UltraLight 70° SB46 ST44, royal blue		1
27	1437919-9005	Cross-brace double Compact, UltraLight 70° SB46 ST44, black, glossy		1
27	1437919-9119	Cross-brace double Compact, UltraLight 70° SB46 ST44, black, bronze		1
28	1437920-0419	Cross-brace double Compact, UltraLight 70° SB46 ST46, silver grey		1
28	1437920-0442	Cross-brace double Compact, UltraLight 70° SB46 ST46, night shimmer	2009-01-01 →	1
28	1437920-3524	Cross-brace double Compact, UltraLight 70° SB46 ST46, royal red		1
28	1437920-5969	Cross-brace double Compact, UltraLight 70° SB46 ST46, royal blue		1
28	1437920-9005	Cross-brace double Compact, UltraLight 70° SB46 ST46, black, glossy		1
28	1437920-9119	Cross-brace double Compact, UltraLight 70° SB46 ST46, black, bronze		1
29	1437921-0419	Cross-brace double Compact, UltraLight 70° SB46 ST48, silver grey		1
29	1437921-0442	Cross-brace double Compact, UltraLight 70° SB46 ST48, night shimmer	2009-01-01 →	1
29	1437921-3524	Cross-brace double Compact, UltraLight 70° SB46 ST48, royal red		1
29	1437921-5969	Cross-brace double Compact, UltraLight 70° SB46 ST48, royal blue		1
29	1437921-9005	Cross-brace double Compact, UltraLight 70° SB46 ST48, black, glossy		1
29	1437921-9119	Cross-brace double Compact, UltraLight 70° SB46 ST48, black, bronze		1
30	1437922-0419	Cross-brace double Compact, UltraLight 70° SB46 ST50, silver grey		1
30	1437922-0442	Cross-brace double Compact, UltraLight 70° SB46 ST50, night shimmer	2009-01-01 →	1
30	1437922-3524	Cross-brace double Compact, UltraLight 70° SB46 ST50, royal red		1
30	1437922-5969	Cross-brace double Compact, UltraLight 70° SB46 ST50, royal blue		1
30	1437922-9005	Cross-brace double Compact, UltraLight 70° SB46 ST50, black, glossy		1
30	1437922-9119	Cross-brace double Compact, UltraLight 70° SB46 ST50, black, bronze		1
31	1437923-0419	Cross-brace double Compact, UltraLight 70° SB48 ST32, silver grey		1
31	1437923-0442	Cross-brace double Compact, UltraLight 70° SB48 ST32, night shimmer	2009-01-01 →	1
31	1437923-3524	Cross-brace double Compact, UltraLight 70° SB48 ST32, royal red		1
31	1437923-5969	Cross-brace double Compact, UltraLight 70° SB48 ST32, royal blue		1
31	1437923-9005	Cross-brace double Compact, UltraLight 70° SB48 ST32, black, glossy		1
31	1437923-9119	Cross-brace double Compact, UltraLight 70° SB48 ST32, black, bronze		1
32	1437924-0419	Cross-brace double Compact, UltraLight 70° SB48 ST34, silver grey		1
32	1437924-0442	Cross-brace double Compact, UltraLight 70° SB48 ST34, night shimmer	2009-01-01 →	1
32	1437924-3524	Cross-brace double Compact, UltraLight 70° SB48 ST34, royal red		1
32	1437924-5969	Cross-brace double Compact, UltraLight 70° SB48 ST34, royal blue		1
32	1437924-9005	Cross-brace double Compact, UltraLight 70° SB48 ST34, black, glossy		1
32	1437924-9119	Cross-brace double Compact, UltraLight 70° SB48 ST34, black, bronze		1
33	1437925-0419	Cross-brace double Compact, UltraLight 70° SB48 ST36, silver grey		1
33	1437925-0442	Cross-brace double Compact, UltraLight 70° SB48 ST36, night shimmer	2009-01-01 →	1
33	1437925-3524	Cross-brace double Compact, UltraLight 70° SB48 ST36, royal red		1
33	1437925-5969	Cross-brace double Compact, UltraLight 70° SB48 ST36, royal blue		1




**Cross brace to frame standard, SB42-50**

Pos.	Part number	Description	Valid from → until	Qty
33	1437925-9005	Cross-brace double Compact, UltraLight 70° SB48 ST36, black, glossy		1
33	1437925-9119	Cross-brace double Compact, UltraLight 70° SB48 ST36, black, bronze		1
34	1437926-0419	Cross-brace double Compact, UltraLight 70° SB48 ST38, silver grey		1
34	1437926-0442	Cross-brace double Compact, UltraLight 70° SB48 ST38, night shimmer	2009-01-01 →	1
34	1437926-3524	Cross-brace double Compact, UltraLight 70° SB48 ST38, royal red		1
34	1437926-5969	Cross-brace double Compact, UltraLight 70° SB48 ST38, royal blue		1
34	1437926-9005	Cross-brace double Compact, UltraLight 70° SB48 ST38, black, glossy		1
34	1437926-9119	Cross-brace double Compact, UltraLight 70° SB48 ST38, black, bronze		1
35	1437927-0419	Cross-brace double Compact, UltraLight 70° SB48 ST40, silver grey		1
35	1437927-0442	Cross-brace double Compact, UltraLight 70° SB48 ST40, night shimmer	2009-01-01 →	1
35	1437927-3524	Cross-brace double Compact, UltraLight 70° SB48 ST40, royal red		1
35	1437927-5969	Cross-brace double Compact, UltraLight 70° SB48 ST40, royal blue		1
35	1437927-9005	Cross-brace double Compact, UltraLight 70° SB48 ST40, black, glossy		1
35	1437927-9119	Cross-brace double Compact, UltraLight 70° SB48 ST40, black, bronze		1
36	1437928-0419	Cross-brace double Compact, UltraLight 70° SB48 ST42, silver grey		1
36	1437928-0442	Cross-brace double Compact, UltraLight 70° SB48 ST42, night shimmer	2009-01-01 →	1
36	1437928-3524	Cross-brace double Compact, UltraLight 70° SB48 ST42, royal red		1
36	1437928-5969	Cross-brace double Compact, UltraLight 70° SB48 ST42, royal blue		1
36	1437928-9005	Cross-brace double Compact, UltraLight 70° SB48 ST42, black, glossy		1
36	1437928-9119	Cross-brace double Compact, UltraLight 70° SB48 ST42, black, bronze		1
37	1437929-0419	Cross-brace double Compact, UltraLight 70° SB48 ST44, silver grey		1
37	1437929-0442	Cross-brace double Compact, UltraLight 70° SB48 ST44, night shimmer	2009-01-01 →	1
37	1437929-3524	Cross-brace double Compact, UltraLight 70° SB48 ST44, royal red		1
37	1437929-5969	Cross-brace double Compact, UltraLight 70° SB48 ST44, royal blue		1
37	1437929-9005	Cross-brace double Compact, UltraLight 70° SB48 ST44, black, glossy		1
37	1437929-9119	Cross-brace double Compact, UltraLight 70° SB48 ST44, black, bronze		1
38	1437930-0419	Cross-brace double Compact, UltraLight 70° SB48 ST46, silver grey		1
38	1437930-0442	Cross-brace double Compact, UltraLight 70° SB48 ST46, night shimmer	2009-01-01 →	1
38	1437930-3524	Cross-brace double Compact, UltraLight 70° SB48 ST46, royal red		1
38	1437930-5969	Cross-brace double Compact, UltraLight 70° SB48 ST46, royal blue		1
38	1437930-9005	Cross-brace double Compact, UltraLight 70° SB48 ST46, black, glossy		1
38	1437930-9119	Cross-brace double Compact, UltraLight 70° SB48 ST46, black, bronze		1
39	1437931-0419	Cross-brace double Compact, UltraLight 70° SB48 ST48, silver grey		1
39	1437931-0442	Cross-brace double Compact, UltraLight 70° SB48 ST48, night shimmer	2009-01-01 →	1
39	1437931-3524	Cross-brace double Compact, UltraLight 70° SB48 ST48, royal red		1
39	1437931-5969	Cross-brace double Compact, UltraLight 70° SB48 ST48, royal blue		1
39	1437931-9005	Cross-brace double Compact, UltraLight 70° SB48 ST48, black, glossy		1
39	1437931-9119	Cross-brace double Compact, UltraLight 70° SB48 ST48, black, bronze		1
40	1437932-0419	Cross-brace double Compact, UltraLight 70° SB48 ST50, silver grey		1
40	1437932-0442	Cross-brace double Compact, UltraLight 70° SB48 ST50, night shimmer	2009-01-01 →	1
40	1437932-3524	Cross-brace double Compact, UltraLight 70° SB48 ST50, royal red		1
40	1437932-5969	Cross-brace double Compact, UltraLight 70° SB48 ST50, royal blue		1
40	1437932-9005	Cross-brace double Compact, UltraLight 70° SB48 ST50, black, glossy		1
40	1437932-9119	Cross-brace double Compact, UltraLight 70° SB48 ST50, black, bronze		1
41	1437933-0419	Cross-brace double Compact, UltraLight 70° SB50 ST32, silver grey		1
41	1437933-0442	Cross-brace double Compact, UltraLight 70° SB50 ST32, night shimmer	2009-01-01 →	1
41	1437933-3524	Cross-brace double Compact, UltraLight 70° SB50 ST32, royal red		1
41	1437933-5969	Cross-brace double Compact, UltraLight 70° SB50 ST32, royal blue		1
41	1437933-9005	Cross-brace double Compact, UltraLight 70° SB50 ST32, black, glossy		1
41	1437933-9119	Cross-brace double Compact, UltraLight 70° SB50 ST32, black, bronze		1
42	1437934-0419	Cross-brace double Compact, UltraLight 70° SB50 ST34, silver grey		1
42	1437934-0442	Cross-brace double Compact, UltraLight 70° SB50 ST34, night shimmer	2009-01-01 →	1
42	1437934-3524	Cross-brace double Compact, UltraLight 70° SB50 ST34, royal red		1
42	1437934-5969	Cross-brace double Compact, UltraLight 70° SB50 ST34, royal blue		1
42	1437934-9005	Cross-brace double Compact, UltraLight 70° SB50 ST34, black, glossy		1

**Cross brace to frame standard, SB42-50**

Pos.	Part number	Description	Valid from → until	Qty
42	1437934-9119	Cross-brace double Compact, UltraLight 70° SB50 ST34, black, bronze		1
43	1437935-0419	Cross-brace double Compact, UltraLight 70° SB50 ST36, silver grey		1
43	1437935-0442	Cross-brace double Compact, UltraLight 70° SB50 ST36, night shimmer	2009-01-01 →	1
43	1437935-3524	Cross-brace double Compact, UltraLight 70° SB50 ST36, royal red		1
43	1437935-5969	Cross-brace double Compact, UltraLight 70° SB50 ST36, royal blue		1
43	1437935-9005	Cross-brace double Compact, UltraLight 70° SB50 ST36, black, glossy		1
43	1437935-9119	Cross-brace double Compact, UltraLight 70° SB50 ST36, black, bronze		1
44	1437936-0419	Cross-brace double Compact, UltraLight 70° SB50 ST38, silver grey		1
44	1437936-0442	Cross-brace double Compact, UltraLight 70° SB50 ST38, night shimmer	2009-01-01 →	1
44	1437936-3524	Cross-brace double Compact, UltraLight 70° SB50 ST38, royal red		1
44	1437936-5969	Cross-brace double Compact, UltraLight 70° SB50 ST38, royal blue		1
44	1437936-9005	Cross-brace double Compact, UltraLight 70° SB50 ST38, black, glossy		1
44	1437936-9119	Cross-brace double Compact, UltraLight 70° SB50 ST38, black, bronze		1
45	1437937-0419	Cross-brace double Compact, UltraLight 70° SB50 ST40, silver grey		1
45	1437937-0442	Cross-brace double Compact, UltraLight 70° SB50 ST40, night shimmer	2009-01-01 →	1
45	1437937-3524	Cross-brace double Compact, UltraLight 70° SB50 ST40, royal red		1
45	1437937-5969	Cross-brace double Compact, UltraLight 70° SB50 ST40, royal blue		1
45	1437937-9005	Cross-brace double Compact, UltraLight 70° SB50 ST40, black, glossy		1
45	1437937-9119	Cross-brace double Compact, UltraLight 70° SB50 ST40, black, bronze		1
46	1437938-0419	Cross-brace double Compact, UltraLight 70° SB50 ST42, silver grey		1
46	1437938-0442	Cross-brace double Compact, UltraLight 70° SB50 ST42, night shimmer	2009-01-01 →	1
46	1437938-3524	Cross-brace double Compact, UltraLight 70° SB50 ST42, royal red		1
46	1437938-5969	Cross-brace double Compact, UltraLight 70° SB50 ST42, royal blue		1
46	1437938-9005	Cross-brace double Compact, UltraLight 70° SB50 ST42, black, glossy		1
46	1437938-9119	Cross-brace double Compact, UltraLight 70° SB50 ST42, black, bronze		1
47	1437939-0419	Cross-brace double Compact, UltraLight 70° SB50 ST44, silver grey		1
47	1437939-0442	Cross-brace double Compact, UltraLight 70° SB50 ST44, night shimmer	2009-01-01 →	1
47	1437939-3524	Cross-brace double Compact, UltraLight 70° SB50 ST44, royal red		1
47	1437939-5969	Cross-brace double Compact, UltraLight 70° SB50 ST44, royal blue		1
47	1437939-9005	Cross-brace double Compact, UltraLight 70° SB50 ST44, black, glossy		1
47	1437939-9119	Cross-brace double Compact, UltraLight 70° SB50 ST44, black, bronze		1
48	1437940-0419	Cross-brace double Compact, UltraLight 70° SB50 ST46, silver grey		1
48	1437940-0442	Cross-brace double Compact, UltraLight 70° SB50 ST46, night shimmer	2009-01-01 →	1
48	1437940-3524	Cross-brace double Compact, UltraLight 70° SB50 ST46, royal red		1
48	1437940-5969	Cross-brace double Compact, UltraLight 70° SB50 ST46, royal blue		1
48	1437940-9005	Cross-brace double Compact, UltraLight 70° SB50 ST46, black, glossy		1
48	1437940-9119	Cross-brace double Compact, UltraLight 70° SB50 ST46, black, bronze		1
49	1437941-0419	Cross-brace double Compact, UltraLight 70° SB50 ST48, silver grey		1
49	1437941-0442	Cross-brace double Compact, UltraLight 70° SB50 ST48, night shimmer	2009-01-01 →	1
49	1437941-3524	Cross-brace double Compact, UltraLight 70° SB50 ST48, royal red		1
49	1437941-5969	Cross-brace double Compact, UltraLight 70° SB50 ST48, royal blue		1
49	1437941-9005	Cross-brace double Compact, UltraLight 70° SB50 ST48, black, glossy		1
49	1437941-9119	Cross-brace double Compact, UltraLight 70° SB50 ST48, black, bronze		1
50	1437942-0419	Cross-brace double Compact, UltraLight 70° SB50 ST50, silver grey		1
50	1437942-0442	Cross-brace double Compact, UltraLight 70° SB50 ST50, night shimmer	2009-01-01 →	1
50	1437942-3524	Cross-brace double Compact, UltraLight 70° SB50 ST50, royal red		1
50	1437942-5969	Cross-brace double Compact, UltraLight 70° SB50 ST50, royal blue		1
50	1437942-9005	Cross-brace double Compact, UltraLight 70° SB50 ST50, black, glossy		1
50	1437942-9119	Cross-brace double Compact, UltraLight 70° SB50 ST50, black, bronze		1
51	1437944	Retaining lever SB40-44		1
52	1437945	Retaining lever SB46-50		1
53	1507625	Center division kit		1
54	1439480	Screw kit to Double crossbrace		1

## Cross brace to frame Hemi, SB32-40

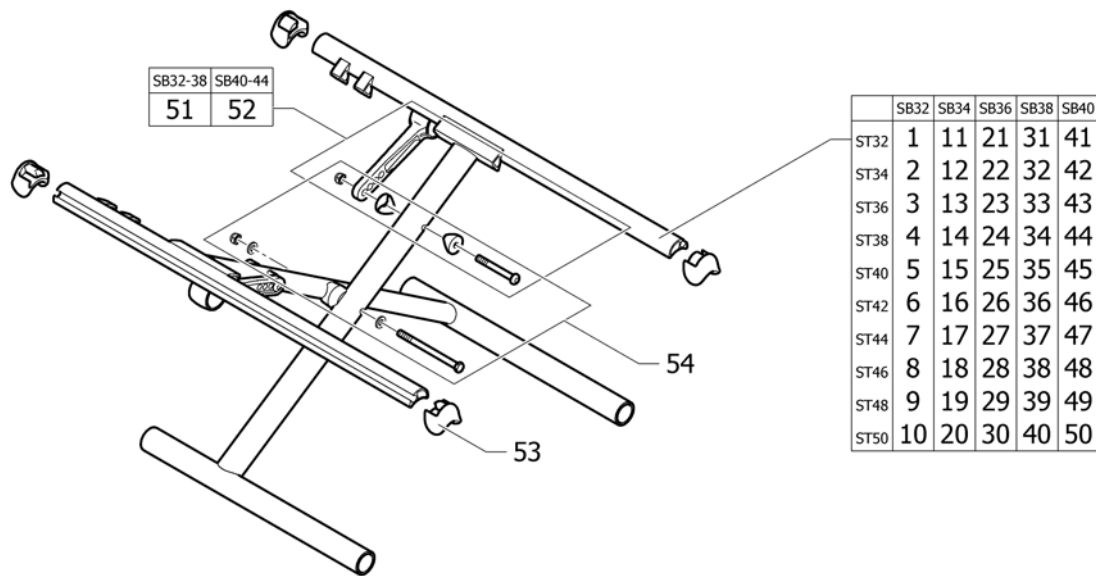

Pos.	Part number	Description	Valid from → until	Qty
1	1506782-0419	Cross-brace single Compact Hemi SB32 ST32, silver grey		1
1	1506782-0442	Cross-brace single Compact Hemi SB32 ST32, night shimmer	2009-01-01 →	1
1	1506782-3524	Cross-brace single Compact Hemi SB32 ST32, royal red		1
1	1506782-5969	Cross-brace single Compact Hemi SB32 ST32, royal blue		1
1	1506782-9005	Cross-brace single Compact Hemi SB32 ST32, black, glossy		1
1	1506782-9119	Cross-brace single Compact Hemi SB32 ST32, black, bronze		1
2	1506783-0419	Cross-brace single Compact Hemi SB32 ST34, silver grey		1
2	1506783-0442	Cross-brace single Compact Hemi SB32 ST34, night shimmer	2009-01-01 →	1
2	1506783-3524	Cross-brace single Compact Hemi SB32 ST34, royal red		1
2	1506783-5969	Cross-brace single Compact Hemi SB32 ST34, royal blue		1
2	1506783-9005	Cross-brace single Compact Hemi SB32 ST34, black, glossy		1
2	1506783-9119	Cross-brace single Compact Hemi SB32 ST34, black, bronze		1
3	1506784-0419	Cross-brace single Compact Hemi SB32 ST36, silver grey		1
3	1506784-0442	Cross-brace single Compact Hemi SB32 ST36, night shimmer	2009-01-01 →	1
3	1506784-3524	Cross-brace single Compact Hemi SB32 ST36, royal red		1
3	1506784-5969	Cross-brace single Compact Hemi SB32 ST36, royal blue		1
3	1506784-9005	Cross-brace single Compact Hemi SB32 ST36, black, glossy		1
3	1506784-9119	Cross-brace single Compact Hemi SB32 ST36, black, bronze		1
4	1506785-0419	Cross-brace single Compact Hemi SB32 ST38, silver grey		1
4	1506785-0442	Cross-brace single Compact Hemi SB32 ST38, night shimmer	2009-01-01 →	1
4	1506785-3524	Cross-brace single Compact Hemi SB32 ST38, royal red		1
4	1506785-5969	Cross-brace single Compact Hemi SB32 ST38, royal blue		1
4	1506785-9005	Cross-brace single Compact Hemi SB32 ST38, black, glossy		1
4	1506785-9119	Cross-brace single Compact Hemi SB32 ST38, black, bronze		1
5	1506786-0419	Cross-brace single Compact Hemi SB32 ST40, silver grey		1
5	1506786-0442	Cross-brace single Compact Hemi SB32 ST40, night shimmer	2009-01-01 →	1
5	1506786-3524	Cross-brace single Compact Hemi SB32 ST40, royal red		1
5	1506786-5969	Cross-brace single Compact Hemi SB32 ST40, royal blue		1
5	1506786-9005	Cross-brace single Compact Hemi SB32 ST40, black, glossy		1
5	1506786-9119	Cross-brace single Compact Hemi SB32 ST40, black, bronze		1
6	1506787-0419	Cross-brace single Compact Hemi SB32 ST42, silver grey		1

**Cross brace to frame Hemi, SB32-40**

Pos.	Part number	Description	Valid from → until	Qty
6	1506787-0442	Cross-brace single Compact Hemi SB32 ST42, night shimmer	2009-01-01 →	1
6	1506787-3524	Cross-brace single Compact Hemi SB32 ST42, royal red		1
6	1506787-5969	Cross-brace single Compact Hemi SB32 ST42, royal blue		1
6	1506787-9005	Cross-brace single Compact Hemi SB32 ST42, black, glossy		1
6	1506787-9119	Cross-brace single Compact Hemi SB32 ST42, black, bronze		1
7	1506788-0419	Cross-brace single Compact Hemi SB32 ST44, silver grey		1
7	1506788-0442	Cross-brace single Compact Hemi SB32 ST44, night shimmer	2009-01-01 →	1
7	1506788-3524	Cross-brace single Compact Hemi SB32 ST44, royal red		1
7	1506788-5969	Cross-brace single Compact Hemi SB32 ST44, royal blue		1
7	1506788-9005	Cross-brace single Compact Hemi SB32 ST44, black, glossy		1
7	1506788-9119	Cross-brace single Compact Hemi SB32 ST44, black, bronze		1
8	1506789-0419	Cross-brace single Compact Hemi SB32 ST46, silver grey		1
8	1506789-0442	Cross-brace single Compact Hemi SB32 ST46, night shimmer	2009-01-01 →	1
8	1506789-3524	Cross-brace single Compact Hemi SB32 ST46, royal red		1
8	1506789-5969	Cross-brace single Compact Hemi SB32 ST46, royal blue		1
8	1506789-9005	Cross-brace single Compact Hemi SB32 ST46, black, glossy		1
8	1506789-9119	Cross-brace single Compact Hemi SB32 ST46, black, bronze		1
9	1506790-0419	Cross-brace single Compact Hemi SB32 ST48, silver grey		1
9	1506790-0442	Cross-brace single Compact Hemi SB32 ST48, night shimmer	2009-01-01 →	1
9	1506790-3524	Cross-brace single Compact Hemi SB32 ST48, royal red		1
9	1506790-5969	Cross-brace single Compact Hemi SB32 ST48, royal blue		1
9	1506790-9005	Cross-brace single Compact Hemi SB32 ST48, black, glossy		1
9	1506790-9119	Cross-brace single Compact Hemi SB32 ST48, black, bronze		1
10	1506791-0419	Cross-brace single Compact Hemi SB32 ST50, silver grey		1
10	1506791-0442	Cross-brace single Compact Hemi SB32 ST50, night shimmer	2009-01-01 →	1
10	1506791-3524	Cross-brace single Compact Hemi SB32 ST50, royal red		1
10	1506791-5969	Cross-brace single Compact Hemi SB32 ST50, royal blue		1
10	1506791-9005	Cross-brace single Compact Hemi SB32 ST50, black, glossy		1
10	1506791-9119	Cross-brace single Compact Hemi SB32 ST50, black, bronze		1
11	1506792-0419	Cross-brace single Compact Hemi SB34 ST32, silver grey		1
11	1506792-0442	Cross-brace single Compact Hemi SB34 ST32, night shimmer	2009-01-01 →	1
11	1506792-3524	Cross-brace single Compact Hemi SB34 ST32, royal red		1
11	1506792-5969	Cross-brace single Compact Hemi SB34 ST32, royal blue		1
11	1506792-9005	Cross-brace single Compact Hemi SB34 ST32, black, glossy		1
11	1506792-9119	Cross-brace single Compact Hemi SB34 ST32, black, bronze		1
12	1506793-0419	Cross-brace single Compact Hemi SB34 ST34, silver grey		1
12	1506793-0442	Cross-brace single Compact Hemi SB34 ST34, night shimmer	2009-01-01 →	1
12	1506793-3524	Cross-brace single Compact Hemi SB34 ST34, royal red		1
12	1506793-5969	Cross-brace single Compact Hemi SB34 ST34, royal blue		1
12	1506793-9005	Cross-brace single Compact Hemi SB34 ST34, black, glossy		1
12	1506793-9119	Cross-brace single Compact Hemi SB34 ST34, black, bronze		1
13	1506794-0419	Cross-brace single Compact Hemi SB34 ST36, silver grey		1
13	1506794-0442	Cross-brace single Compact Hemi SB34 ST36, night shimmer	2009-01-01 →	1
13	1506794-3524	Cross-brace single Compact Hemi SB34 ST36, royal red		1
13	1506794-5969	Cross-brace single Compact Hemi SB34 ST36, royal blue		1
13	1506794-9005	Cross-brace single Compact Hemi SB34 ST36, black, glossy		1
13	1506794-9119	Cross-brace single Compact Hemi SB34 ST36, black, bronze		1
14	1506795-0419	Cross-brace single Compact Hemi SB34 ST38, silver grey		1
14	1506795-0442	Cross-brace single Compact Hemi SB34 ST38, night shimmer	2009-01-01 →	1
14	1506795-3524	Cross-brace single Compact Hemi SB34 ST38, royal red		1
14	1506795-5969	Cross-brace single Compact Hemi SB34 ST38, royal blue		1
14	1506795-9005	Cross-brace single Compact Hemi SB34 ST38, black, glossy		1
14	1506795-9119	Cross-brace single Compact Hemi SB34 ST38, black, bronze		1
15	1506796-0419	Cross-brace single Compact Hemi SB34 ST40, silver grey		1
15	1506796-0442	Cross-brace single Compact Hemi SB34 ST40, night shimmer	2009-01-01 →	1



**Cross brace to frame Hemi, SB32-40**

Pos.	Part number	Description	Valid from → until	Qty
15	1506796-3524	Cross-brace single Compact Hemi SB34 ST40, royal red		1
15	1506796-5969	Cross-brace single Compact Hemi SB34 ST40, royal blue		1
15	1506796-9005	Cross-brace single Compact Hemi SB34 ST40, black, glossy		1
15	1506796-9119	Cross-brace single Compact Hemi SB34 ST40, black, bronze		1
16	1506797-0419	Cross-brace single Compact Hemi SB34 ST42, silver grey		1
16	1506797-0442	Cross-brace single Compact Hemi SB34 ST42, night shimmer	2009-01-01 →	1
16	1506797-3524	Cross-brace single Compact Hemi SB34 ST42, royal red		1
16	1506797-5969	Cross-brace single Compact Hemi SB34 ST42, royal blue		1
16	1506797-9005	Cross-brace single Compact Hemi SB34 ST42, black, glossy		1
16	1506797-9119	Cross-brace single Compact Hemi SB34 ST42, black, bronze		1
17	1506798-0419	Cross-brace single Compact Hemi SB34 ST44, silver grey		1
17	1506798-0442	Cross-brace single Compact Hemi SB34 ST44, night shimmer	2009-01-01 →	1
17	1506798-3524	Cross-brace single Compact Hemi SB34 ST44, royal red		1
17	1506798-5969	Cross-brace single Compact Hemi SB34 ST44, royal blue		1
17	1506798-9005	Cross-brace single Compact Hemi SB34 ST44, black, glossy		1
17	1506798-9119	Cross-brace single Compact Hemi SB34 ST44, black, bronze		1
18	1506799-0419	Cross-brace single Compact Hemi SB34 ST46, silver grey		1
18	1506799-0442	Cross-brace single Compact Hemi SB34 ST46, night shimmer	2009-01-01 →	1
18	1506799-3524	Cross-brace single Compact Hemi SB34 ST46, royal red		1
18	1506799-5969	Cross-brace single Compact Hemi SB34 ST46, royal blue		1
18	1506799-9005	Cross-brace single Compact Hemi SB34 ST46, black, glossy		1
18	1506799-9119	Cross-brace single Compact Hemi SB34 ST46, black, bronze		1
19	1506800-0419	Cross-brace single Compact Hemi SB34 ST48, silver grey		1
19	1506800-0442	Cross-brace single Compact Hemi SB34 ST48, night shimmer	2009-01-01 →	1
19	1506800-3524	Cross-brace single Compact Hemi SB34 ST48, royal red		1
19	1506800-5969	Cross-brace single Compact Hemi SB34 ST48, royal blue		1
19	1506800-9005	Cross-brace single Compact Hemi SB34 ST48, black, glossy		1
19	1506800-9119	Cross-brace single Compact Hemi SB34 ST48, black, bronze		1
20	1506801-0419	Cross-brace single Compact Hemi SB34 ST50, silver grey		1
20	1506801-0442	Cross-brace single Compact Hemi SB34 ST50, night shimmer	2009-01-01 →	1
20	1506801-3524	Cross-brace single Compact Hemi SB34 ST50, royal red		1
20	1506801-5969	Cross-brace single Compact Hemi SB34 ST50, royal blue		1
20	1506801-9005	Cross-brace single Compact Hemi SB34 ST50, black, glossy		1
20	1506801-9119	Cross-brace single Compact Hemi SB34 ST50, black, bronze		1
21	1506802-0419	Cross-brace single Compact Hemi SB36 ST32, silver grey		1
21	1506802-0442	Cross-brace single Compact Hemi SB36 ST32, night shimmer	2009-01-01 →	1
21	1506802-3524	Cross-brace single Compact Hemi SB36 ST32, royal red		1
21	1506802-5969	Cross-brace single Compact Hemi SB36 ST32, royal blue		1
21	1506802-9005	Cross-brace single Compact Hemi SB36 ST32, black, glossy		1
21	1506802-9119	Cross-brace single Compact Hemi SB36 ST32, black, bronze		1
22	1506803-0419	Cross-brace single Compact Hemi SB36 ST34, silver grey		1
22	1506803-0442	Cross-brace single Compact Hemi SB36 ST34, night shimmer	2009-01-01 →	1
22	1506803-3524	Cross-brace single Compact Hemi SB36 ST34, royal red		1
22	1506803-5969	Cross-brace single Compact Hemi SB36 ST34, royal blue		1
22	1506803-9005	Cross-brace single Compact Hemi SB36 ST34, black, glossy		1
22	1506803-9119	Cross-brace single Compact Hemi SB36 ST34, black, bronze		1
23	1506804-0419	Cross-brace single Compact Hemi SB36 ST36, silver grey		1
23	1506804-0442	Cross-brace single Compact Hemi SB36 ST36, night shimmer	2009-01-01 →	1
23	1506804-3524	Cross-brace single Compact Hemi SB36 ST36, royal red		1
23	1506804-5969	Cross-brace single Compact Hemi SB36 ST36, royal blue		1
23	1506804-9005	Cross-brace single Compact Hemi SB36 ST36, black, glossy		1
23	1506804-9119	Cross-brace single Compact Hemi SB36 ST36, black, bronze		1
24	1506805-0419	Cross-brace single Compact Hemi SB36 ST38, silver grey		1
24	1506805-0442	Cross-brace single Compact Hemi SB36 ST38, night shimmer	2009-01-01 →	1
24	1506805-3524	Cross-brace single Compact Hemi SB36 ST38, royal red		1

**Cross brace to frame Hemi, SB32-40**

Pos.	Part number	Description	Valid from → until	Qty
24	1506805-5969	Cross-brace single Compact Hemi SB36 ST38, royal blue		1
24	1506805-9005	Cross-brace single Compact Hemi SB36 ST38, black, glossy		1
24	1506805-9119	Cross-brace single Compact Hemi SB36 ST38, black, bronze		1
25	1506806-0419	Cross-brace single Compact Hemi SB36 ST40, silver grey		1
25	1506806-0442	Cross-brace single Compact Hemi SB36 ST40, night shimmer	2009-01-01 →	1
25	1506806-3524	Cross-brace single Compact Hemi SB36 ST40, royal red		1
25	1506806-5969	Cross-brace single Compact Hemi SB36 ST40, royal blue		1
25	1506806-9005	Cross-brace single Compact Hemi SB36 ST40, black, glossy		1
25	1506806-9119	Cross-brace single Compact Hemi SB36 ST40, black, bronze		1
26	1506807-0419	Cross-brace single Compact Hemi SB36 ST42, silver grey		1
26	1506807-0442	Cross-brace single Compact Hemi SB36 ST42, night shimmer	2009-01-01 →	1
26	1506807-3524	Cross-brace single Compact Hemi SB36 ST42, royal red		1
26	1506807-5969	Cross-brace single Compact Hemi SB36 ST42, royal blue		1
26	1506807-9005	Cross-brace single Compact Hemi SB36 ST42, black, glossy		1
26	1506807-9119	Cross-brace single Compact Hemi SB36 ST42, black, bronze		1
27	1506808-0419	Cross-brace single Compact Hemi SB36 ST44, silver grey		1
27	1506808-0442	Cross-brace single Compact Hemi SB36 ST44, night shimmer	2009-01-01 →	1
27	1506808-3524	Cross-brace single Compact Hemi SB36 ST44, royal red		1
27	1506808-5969	Cross-brace single Compact Hemi SB36 ST44, royal blue		1
27	1506808-9005	Cross-brace single Compact Hemi SB36 ST44, black, glossy		1
27	1506808-9119	Cross-brace single Compact Hemi SB36 ST44, black, bronze		1
28	1506809-0419	Cross-brace single Compact Hemi SB36 ST46, silver grey		1
28	1506809-0442	Cross-brace single Compact Hemi SB36 ST46, night shimmer	2009-01-01 →	1
28	1506809-3524	Cross-brace single Compact Hemi SB36 ST46, royal red		1
28	1506809-5969	Cross-brace single Compact Hemi SB36 ST46, royal blue		1
28	1506809-9005	Cross-brace single Compact Hemi SB36 ST46, black, glossy		1
28	1506809-9119	Cross-brace single Compact Hemi SB36 ST46, black, bronze		1
29	1506810-0419	Cross-brace single Compact Hemi SB36 ST48, silver grey		1
29	1506810-0442	Cross-brace single Compact Hemi SB36 ST48, night shimmer	2009-01-01 →	1
29	1506810-3524	Cross-brace single Compact Hemi SB36 ST48, royal red		1
29	1506810-5969	Cross-brace single Compact Hemi SB36 ST48, royal blue		1
29	1506810-9005	Cross-brace single Compact Hemi SB36 ST48, black, glossy		1
29	1506810-9119	Cross-brace single Compact Hemi SB36 ST48, black, bronze		1
30	1506811-0419	Cross-brace single Compact Hemi SB36 ST50, silver grey		1
30	1506811-0442	Cross-brace single Compact Hemi SB36 ST50, night shimmer	2009-01-01 →	1
30	1506811-3524	Cross-brace single Compact Hemi SB36 ST50, royal red		1
30	1506811-5969	Cross-brace single Compact Hemi SB36 ST50, royal blue		1
30	1506811-9005	Cross-brace single Compact Hemi SB36 ST50, black, glossy		1
30	1506811-9119	Cross-brace single Compact Hemi SB36 ST50, black, bronze		1
31	1506812-0419	Cross-brace single Compact Hemi SB38 ST32, silver grey		1
31	1506812-0442	Cross-brace single Compact Hemi SB38 ST32, night shimmer	2009-01-01 →	1
31	1506812-3524	Cross-brace single Compact Hemi SB38 ST32, royal red		1
31	1506812-5969	Cross-brace single Compact Hemi SB38 ST32, royal blue		1
31	1506812-9005	Cross-brace single Compact Hemi SB38 ST32, black, glossy		1
31	1506812-9119	Cross-brace single Compact Hemi SB38 ST32, black, bronze		1
32	1506813-0419	Cross-brace single Compact Hemi SB38 ST34, silver grey		1
32	1506813-0442	Cross-brace single Compact Hemi SB38 ST34, night shimmer	2009-01-01 →	1
32	1506813-3524	Cross-brace single Compact Hemi SB38 ST34, royal red		1
32	1506813-5969	Cross-brace single Compact Hemi SB38 ST34, royal blue		1
32	1506813-9005	Cross-brace single Compact Hemi SB38 ST34, black, glossy		1
32	1506813-9119	Cross-brace single Compact Hemi SB38 ST34, black, bronze		1
33	1506814-0419	Cross-brace single Compact Hemi SB38 ST36, silver grey		1
33	1506814-0442	Cross-brace single Compact Hemi SB38 ST36, night shimmer	2009-01-01 →	1
33	1506814-3524	Cross-brace single Compact Hemi SB38 ST36, royal red		1
33	1506814-5969	Cross-brace single Compact Hemi SB38 ST36, royal blue		1

**Cross brace to frame Hemi, SB32-40**


Pos.	Part number	Description	Valid from → until	Qty
33	1506814-9005	Cross-brace single Compact Hemi SB38 ST36, black, glossy		1
33	1506814-9119	Cross-brace single Compact Hemi SB38 ST36, black, bronze		1
34	1506815-0419	Cross-brace single Compact Hemi SB38 ST38, silver grey		1
34	1506815-0442	Cross-brace single Compact Hemi SB38 ST38, night shimmer	2009-01-01 →	1
34	1506815-3524	Cross-brace single Compact Hemi SB38 ST38, royal red		1
34	1506815-5969	Cross-brace single Compact Hemi SB38 ST38, royal blue		1
34	1506815-9005	Cross-brace single Compact Hemi SB38 ST38, black, glossy		1
34	1506815-9119	Cross-brace single Compact Hemi SB38 ST38, black, bronze		1
35	1506816-0419	Cross-brace single Compact Hemi SB38 ST40, silver grey		1
35	1506816-0442	Cross-brace single Compact Hemi SB38 ST40, night shimmer	2009-01-01 →	1
35	1506816-3524	Cross-brace single Compact Hemi SB38 ST40, royal red		1
35	1506816-5969	Cross-brace single Compact Hemi SB38 ST40, royal blue		1
35	1506816-9005	Cross-brace single Compact Hemi SB38 ST40, black, glossy		1
35	1506816-9119	Cross-brace single Compact Hemi SB38 ST40, black, bronze		1
36	1506817-0419	Cross-brace single Compact Hemi SB38 ST42, silver grey		1
36	1506817-0442	Cross-brace single Compact Hemi SB38 ST42, night shimmer	2009-01-01 →	1
36	1506817-3524	Cross-brace single Compact Hemi SB38 ST42, royal red		1
36	1506817-5969	Cross-brace single Compact Hemi SB38 ST42, royal blue		1
36	1506817-9005	Cross-brace single Compact Hemi SB38 ST42, black, glossy		1
36	1506817-9119	Cross-brace single Compact Hemi SB38 ST42, black, bronze		1
37	1506818-0419	Cross-brace single Compact Hemi SB38 ST44, silver grey		1
37	1506818-0442	Cross-brace single Compact Hemi SB38 ST44, night shimmer	2009-01-01 →	1
37	1506818-3524	Cross-brace single Compact Hemi SB38 ST44, royal red		1
37	1506818-5969	Cross-brace single Compact Hemi SB38 ST44, royal blue		1
37	1506818-9005	Cross-brace single Compact Hemi SB38 ST44, black, glossy		1
37	1506818-9119	Cross-brace single Compact Hemi SB38 ST44, black, bronze		1
38	1506819-0419	Cross-brace single Compact Hemi SB38 ST46, silver grey		1
38	1506819-0442	Cross-brace single Compact Hemi SB38 ST46, night shimmer	2009-01-01 →	1
38	1506819-3524	Cross-brace single Compact Hemi SB38 ST46, royal red		1
38	1506819-5969	Cross-brace single Compact Hemi SB38 ST46, royal blue		1
38	1506819-9005	Cross-brace single Compact Hemi SB38 ST46, black, glossy		1
38	1506819-9119	Cross-brace single Compact Hemi SB38 ST46, black, bronze		1
39	1506820-0419	Cross-brace single Compact Hemi SB38 ST48, silver grey		1
39	1506820-0442	Cross-brace single Compact Hemi SB38 ST48, night shimmer	2009-01-01 →	1
39	1506820-3524	Cross-brace single Compact Hemi SB38 ST48, royal red		1
39	1506820-5969	Cross-brace single Compact Hemi SB38 ST48, royal blue		1
39	1506820-9005	Cross-brace single Compact Hemi SB38 ST48, black, glossy		1
39	1506820-9119	Cross-brace single Compact Hemi SB38 ST48, black, bronze		1
40	1506821-0419	Cross-brace single Compact Hemi SB38 ST50, silver grey		1
40	1506821-0442	Cross-brace single Compact Hemi SB38 ST50, night shimmer	2009-01-01 →	1
40	1506821-3524	Cross-brace single Compact Hemi SB38 ST50, royal red		1
40	1506821-5969	Cross-brace single Compact Hemi SB38 ST50, royal blue		1
40	1506821-9005	Cross-brace single Compact Hemi SB38 ST50, black, glossy		1
40	1506821-9119	Cross-brace single Compact Hemi SB38 ST50, black, bronze		1
41	1506822-0419	Cross-brace single Compact Hemi SB40 ST32, silver grey		1
41	1506822-0442	Cross-brace single Compact Hemi SB40 ST32, night shimmer	2009-01-01 →	1
41	1506822-3524	Cross-brace single Compact Hemi SB40 ST32, royal red		1
41	1506822-5969	Cross-brace single Compact Hemi SB40 ST32, royal blue		1
41	1506822-9005	Cross-brace single Compact Hemi SB40 ST32, black, glossy		1
41	1506822-9119	Cross-brace single Compact Hemi SB40 ST32, black, bronze		1
42	1506823-0419	Cross-brace single Compact Hemi SB40 ST34, silver grey		1
42	1506823-0442	Cross-brace single Compact Hemi SB40 ST34, night shimmer	2009-01-01 →	1
42	1506823-3524	Cross-brace single Compact Hemi SB40 ST34, royal red		1
42	1506823-5969	Cross-brace single Compact Hemi SB40 ST34, royal blue		1
42	1506823-9005	Cross-brace single Compact Hemi SB40 ST34, black, glossy		1

**Cross brace to frame Hemi, SB32-40**

Pos.	Part number	Description	Valid from → until	Qty
42	1506823-9119	Cross-brace single Compact Hemi SB40 ST34, black, bronze		1
43	1506824-0419	Cross-brace single Compact Hemi SB40 ST36, silver grey		1
43	1506824-0442	Cross-brace single Compact Hemi SB40 ST36, night shimmer	2009-01-01 →	1
43	1506824-3524	Cross-brace single Compact Hemi SB40 ST36, royal red		1
43	1506824-5969	Cross-brace single Compact Hemi SB40 ST36, royal blue		1
43	1506824-9005	Cross-brace single Compact Hemi SB40 ST36, black, glossy		1
43	1506824-9119	Cross-brace single Compact Hemi SB40 ST36, black, bronze		1
44	1506825-0419	Cross-brace single Compact Hemi SB40 ST38, silver grey		1
44	1506825-0442	Cross-brace single Compact Hemi SB40 ST38, night shimmer	2009-01-01 →	1
44	1506825-3524	Cross-brace single Compact Hemi SB40 ST38, royal red		1
44	1506825-5969	Cross-brace single Compact Hemi SB40 ST38, royal blue		1
44	1506825-9005	Cross-brace single Compact Hemi SB40 ST38, black, glossy		1
44	1506825-9119	Cross-brace single Compact Hemi SB40 ST38, black, bronze		1
45	1506826-0419	Cross-brace single Compact Hemi SB40 ST40, silver grey		1
45	1506826-0442	Cross-brace single Compact Hemi SB40 ST40, night shimmer	2009-01-01 →	1
45	1506826-3524	Cross-brace single Compact Hemi SB40 ST40, royal red		1
45	1506826-5969	Cross-brace single Compact Hemi SB40 ST40, royal blue		1
45	1506826-9005	Cross-brace single Compact Hemi SB40 ST40, black, glossy		1
45	1506826-9119	Cross-brace single Compact Hemi SB40 ST40, black, bronze		1
46	1506827-0419	Cross-brace single Compact Hemi SB40 ST42, silver grey		1
46	1506827-0442	Cross-brace single Compact Hemi SB40 ST42, night shimmer	2009-01-01 →	1
46	1506827-3524	Cross-brace single Compact Hemi SB40 ST42, royal red		1
46	1506827-5969	Cross-brace single Compact Hemi SB40 ST42, royal blue		1
46	1506827-9005	Cross-brace single Compact Hemi SB40 ST42, black, glossy		1
46	1506827-9119	Cross-brace single Compact Hemi SB40 ST42, black, bronze		1
47	1506828-0419	Cross-brace single Compact Hemi SB40 ST44, silver grey		1
47	1506828-0442	Cross-brace single Compact Hemi SB40 ST44, night shimmer	2009-01-01 →	1
47	1506828-3524	Cross-brace single Compact Hemi SB40 ST44, royal red		1
47	1506828-5969	Cross-brace single Compact Hemi SB40 ST44, royal blue		1
47	1506828-9005	Cross-brace single Compact Hemi SB40 ST44, black, glossy		1
47	1506828-9119	Cross-brace single Compact Hemi SB40 ST44, black, bronze		1
48	1506829-0419	Cross-brace single Compact Hemi SB40 ST46, silver grey		1
48	1506829-0442	Cross-brace single Compact Hemi SB40 ST46, night shimmer	2009-01-01 →	1
48	1506829-3524	Cross-brace single Compact Hemi SB40 ST46, royal red		1
48	1506829-5969	Cross-brace single Compact Hemi SB40 ST46, royal blue		1
48	1506829-9005	Cross-brace single Compact Hemi SB40 ST46, black, glossy		1
48	1506829-9119	Cross-brace single Compact Hemi SB40 ST46, black, bronze		1
49	1506830-0419	Cross-brace single Compact Hemi SB40 ST48, silver grey		1
49	1506830-0442	Cross-brace single Compact Hemi SB40 ST48, night shimmer	2009-01-01 →	1
49	1506830-3524	Cross-brace single Compact Hemi SB40 ST48, royal red		1
49	1506830-5969	Cross-brace single Compact Hemi SB40 ST48, royal blue		1
49	1506830-9005	Cross-brace single Compact Hemi SB40 ST48, black, glossy		1
49	1506830-9119	Cross-brace single Compact Hemi SB40 ST48, black, bronze		1
50	1506831-0419	Cross-brace single Compact Hemi SB40 ST50, silver grey		1
50	1506831-0442	Cross-brace single Compact Hemi SB40 ST50, night shimmer	2009-01-01 →	1
50	1506831-3524	Cross-brace single Compact Hemi SB40 ST50, royal red		1
50	1506831-5969	Cross-brace single Compact Hemi SB40 ST50, royal blue		1
50	1506831-9005	Cross-brace single Compact Hemi SB40 ST50, black, glossy		1
50	1506831-9119	Cross-brace single Compact Hemi SB40 ST50, black, bronze		1
51	1437943	Retaining lever SB32-38		1
52	1437944	Retaining lever SB40-44		1
53	1507625	Center division kit		1
54	1439479	Screw kit to Single crossbrace		1



# Cross brace to frame Hemi, SB42-50

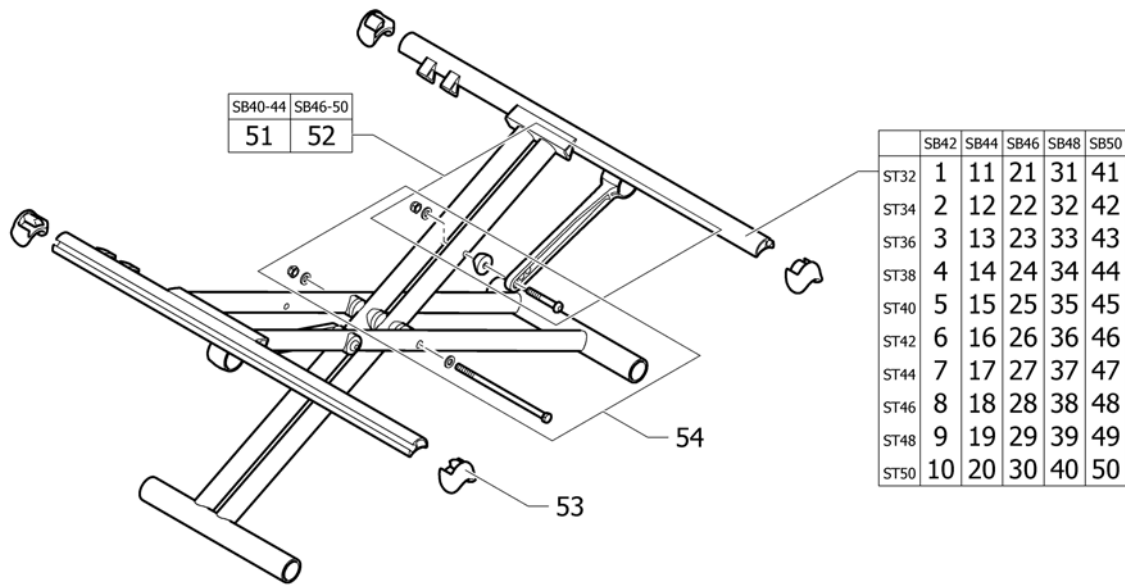

Pos.	Part number	Description	Valid from → until	Qty
1	1506832-0419	Cross-brace double Compact Hemi SB42 ST32, silver grey		1
1	1506832-0442	Cross-brace double Compact Hemi SB42 ST32, night shimmer	2009-01-01 →	1
1	1506832-3524	Cross-brace double Compact Hemi SB42 ST32, royal red		1
1	1506832-5969	Cross-brace double Compact Hemi SB42 ST32, royal blue		1
1	1506832-9005	Cross-brace double Compact Hemi SB42 ST32, black, glossy		1
1	1506832-9119	Cross-brace double Compact Hemi SB42 ST32, black, bronze		1
2	1506833-0419	Cross-brace double Compact Hemi SB42 ST34, silver grey		1
2	1506833-0442	Cross-brace double Compact Hemi SB42 ST34, night shimmer	2009-01-01 →	1
2	1506833-3524	Cross-brace double Compact Hemi SB42 ST34, royal red		1
2	1506833-5969	Cross-brace double Compact Hemi SB42 ST34, royal blue		1
2	1506833-9005	Cross-brace double Compact Hemi SB42 ST34, black, glossy		1
2	1506833-9119	Cross-brace double Compact Hemi SB42 ST34, black, bronze		1
3	1506834-0419	Cross-brace double Compact Hemi SB42 ST36, silver grey		1
3	1506834-0442	Cross-brace double Compact Hemi SB42 ST36, night shimmer	2009-01-01 →	1
3	1506834-3524	Cross-brace double Compact Hemi SB42 ST36, royal red		1
3	1506834-5969	Cross-brace double Compact Hemi SB42 ST36, royal blue		1
3	1506834-9005	Cross-brace double Compact Hemi SB42 ST36, black, glossy		1
3	1506834-9119	Cross-brace double Compact Hemi SB42 ST36, black, bronze		1
4	1506835-0419	Cross-brace double Compact Hemi SB42 ST38, silver grey		1
4	1506835-0442	Cross-brace double Compact Hemi SB42 ST38, night shimmer	2009-01-01 →	1
4	1506835-3524	Cross-brace double Compact Hemi SB42 ST38, royal red		1
4	1506835-5969	Cross-brace double Compact Hemi SB42 ST38, royal blue		1
4	1506835-9005	Cross-brace double Compact Hemi SB42 ST38, black, glossy		1
4	1506835-9119	Cross-brace double Compact Hemi SB42 ST38, black, bronze		1
5	1506836-0419	Cross-brace double Compact Hemi SB42 ST40, silver grey		1
5	1506836-0442	Cross-brace double Compact Hemi SB42 ST40, night shimmer	2009-01-01 →	1
5	1506836-3524	Cross-brace double Compact Hemi SB42 ST40, royal red		1
5	1506836-5969	Cross-brace double Compact Hemi SB42 ST40, royal blue		1
5	1506836-9005	Cross-brace double Compact Hemi SB42 ST40, black, glossy		1
5	1506836-9119	Cross-brace double Compact Hemi SB42 ST40, black, bronze		1
6	1506837-0419	Cross-brace double Compact Hemi SB42 ST42, silver grey		1

**Cross brace to frame Hemi, SB42-50**

Pos.	Part number	Description	Valid from → until	Qty
6	1506837-0442	Cross-brace double Compact Hemi SB42 ST42, night shimmer	2009-01-01 →	1
6	1506837-3524	Cross-brace double Compact Hemi SB42 ST42, royal red		1
6	1506837-5969	Cross-brace double Compact Hemi SB42 ST42, royal blue		1
6	1506837-9005	Cross-brace double Compact Hemi SB42 ST42, black, glossy		1
6	1506837-9119	Cross-brace double Compact Hemi SB42 ST42, black, bronze		1
7	1506838-0419	Cross-brace double Compact Hemi SB42 ST44, silver grey		1
7	1506838-0442	Cross-brace double Compact Hemi SB42 ST44, night shimmer	2009-01-01 →	1
7	1506838-3524	Cross-brace double Compact Hemi SB42 ST44, royal red		1
7	1506838-5969	Cross-brace double Compact Hemi SB42 ST44, royal blue		1
7	1506838-9005	Cross-brace double Compact Hemi SB42 ST44, black, glossy		1
7	1506838-9119	Cross-brace double Compact Hemi SB42 ST44, black, bronze		1
8	1506839-0419	Cross-brace double Compact Hemi SB42 ST46, silver grey		1
8	1506839-0442	Cross-brace double Compact Hemi SB42 ST46, night shimmer	2009-01-01 →	1
8	1506839-3524	Cross-brace double Compact Hemi SB42 ST46, royal red		1
8	1506839-5969	Cross-brace double Compact Hemi SB42 ST46, royal blue		1
8	1506839-9005	Cross-brace double Compact Hemi SB42 ST46, black, glossy		1
8	1506839-9119	Cross-brace double Compact Hemi SB42 ST46, black, bronze		1
9	1506840-0419	Cross-brace double Compact Hemi SB42 ST48, silver grey		1
9	1506840-0442	Cross-brace double Compact Hemi SB42 ST48, night shimmer	2009-01-01 →	1
9	1506840-3524	Cross-brace double Compact Hemi SB42 ST48, royal red		1
9	1506840-5969	Cross-brace double Compact Hemi SB42 ST48, royal blue		1
9	1506840-9005	Cross-brace double Compact Hemi SB42 ST48, black, glossy		1
9	1506840-9119	Cross-brace double Compact Hemi SB42 ST48, black, bronze		1
10	1506841-0419	Cross-brace double Compact Hemi SB42 ST50, silver grey		1
10	1506841-0442	Cross-brace double Compact Hemi SB42 ST50, night shimmer	2009-01-01 →	1
10	1506841-3524	Cross-brace double Compact Hemi SB42 ST50, royal red		1
10	1506841-5969	Cross-brace double Compact Hemi SB42 ST50, royal blue		1
10	1506841-9005	Cross-brace double Compact Hemi SB42 ST50, black, glossy		1
10	1506841-9119	Cross-brace double Compact Hemi SB42 ST50, black, bronze		1
11	1506842-0419	Cross-brace double Compact Hemi SB44 ST32, silver grey		1
11	1506842-0442	Cross-brace double Compact Hemi SB44 ST32, night shimmer	2009-01-01 →	1
11	1506842-3524	Cross-brace double Compact Hemi SB44 ST32, royal red		1
11	1506842-5969	Cross-brace double Compact Hemi SB44 ST32, royal blue		1
11	1506842-9005	Cross-brace double Compact Hemi SB44 ST32, black, glossy		1
11	1506842-9119	Cross-brace double Compact Hemi SB44 ST32, black, bronze		1
12	1506843-0419	Cross-brace double Compact Hemi SB44 ST34, silver grey		1
12	1506843-0442	Cross-brace double Compact Hemi SB44 ST34, night shimmer	2009-01-01 →	1
12	1506843-3524	Cross-brace double Compact Hemi SB44 ST34, royal red		1
12	1506843-5969	Cross-brace double Compact Hemi SB44 ST34, royal blue		1
12	1506843-9005	Cross-brace double Compact Hemi SB44 ST34, black, glossy		1
12	1506843-9119	Cross-brace double Compact Hemi SB44 ST34, black, bronze		1
13	1506844-0419	Cross-brace double Compact Hemi SB44 ST36, silver grey		1
13	1506844-0442	Cross-brace double Compact Hemi SB44 ST36, night shimmer	2009-01-01 →	1
13	1506844-3524	Cross-brace double Compact Hemi SB44 ST36, royal red		1
13	1506844-5969	Cross-brace double Compact Hemi SB44 ST36, royal blue		1
13	1506844-9005	Cross-brace double Compact Hemi SB44 ST36, black, glossy		1
13	1506844-9119	Cross-brace double Compact Hemi SB44 ST36, black, bronze		1
14	1506845-0419	Cross-brace double Compact Hemi SB44 ST38, silver grey		1
14	1506845-0442	Cross-brace double Compact Hemi SB44 ST38, night shimmer	2009-01-01 →	1
14	1506845-3524	Cross-brace double Compact Hemi SB44 ST38, royal red		1
14	1506845-5969	Cross-brace double Compact Hemi SB44 ST38, royal blue		1
14	1506845-9005	Cross-brace double Compact Hemi SB44 ST38, black, glossy		1
14	1506845-9119	Cross-brace double Compact Hemi SB44 ST38, black, bronze		1
15	1506846-0419	Cross-brace double Compact Hemi SB44 ST40, silver grey		1
15	1506846-0442	Cross-brace double Compact Hemi SB44 ST40, night shimmer	2009-01-01 →	1

**Cross brace to frame Hemi, SB42-50**

Pos.	Part number	Description	Valid from → until	Qty
15	1506846-3524	Cross-brace double Compact Hemi SB44 ST40, royal red		1
15	1506846-5969	Cross-brace double Compact Hemi SB44 ST40, royal blue		1
15	1506846-9005	Cross-brace double Compact Hemi SB44 ST40, black, glossy		1
15	1506846-9119	Cross-brace double Compact Hemi SB44 ST40, black, bronze		1
16	1506847-0419	Cross-brace double Compact Hemi SB44 ST42, silver grey		1
16	1506847-0442	Cross-brace double Compact Hemi SB44 ST42, night shimmer	2009-01-01 →	1
16	1506847-3524	Cross-brace double Compact Hemi SB44 ST42, royal red		1
16	1506847-5969	Cross-brace double Compact Hemi SB44 ST42, royal blue		1
16	1506847-9005	Cross-brace double Compact Hemi SB44 ST42, black, glossy		1
16	1506847-9119	Cross-brace double Compact Hemi SB44 ST42, black, bronze		1
17	1506848-0419	Cross-brace double Compact Hemi SB44 ST44, silver grey		1
17	1506848-0442	Cross-brace double Compact Hemi SB44 ST44, night shimmer	2009-01-01 →	1
17	1506848-3524	Cross-brace double Compact Hemi SB44 ST44, royal red		1
17	1506848-5969	Cross-brace double Compact Hemi SB44 ST44, royal blue		1
17	1506848-9005	Cross-brace double Compact Hemi SB44 ST44, black, glossy		1
17	1506848-9119	Cross-brace double Compact Hemi SB44 ST44, black, bronze		1
18	1506849-0419	Cross-brace double Compact Hemi SB44 ST46, silver grey		1
18	1506849-0442	Cross-brace double Compact Hemi SB44 ST46, night shimmer	2009-01-01 →	1
18	1506849-3524	Cross-brace double Compact Hemi SB44 ST46, royal red		1
18	1506849-5969	Cross-brace double Compact Hemi SB44 ST46, royal blue		1
18	1506849-9005	Cross-brace double Compact Hemi SB44 ST46, black, glossy		1
18	1506849-9119	Cross-brace double Compact Hemi SB44 ST46, black, bronze		1
19	1506850-0419	Cross-brace double Compact Hemi SB44 ST48, silver grey		1
19	1506850-0442	Cross-brace double Compact Hemi SB44 ST48, night shimmer	2009-01-01 →	1
19	1506850-3524	Cross-brace double Compact Hemi SB44 ST48, royal red		1
19	1506850-5969	Cross-brace double Compact Hemi SB44 ST48, royal blue		1
19	1506850-9005	Cross-brace double Compact Hemi SB44 ST48, black, glossy		1
19	1506850-9119	Cross-brace double Compact Hemi SB44 ST48, black, bronze		1
20	1506851-0419	Cross-brace double Compact Hemi SB44 ST50, silver grey		1
20	1506851-0442	Cross-brace double Compact Hemi SB44 ST50, night shimmer	2009-01-01 →	1
20	1506851-3524	Cross-brace double Compact Hemi SB44 ST50, royal red		1
20	1506851-5969	Cross-brace double Compact Hemi SB44 ST50, royal blue		1
20	1506851-9005	Cross-brace double Compact Hemi SB44 ST50, black, glossy		1
20	1506851-9119	Cross-brace double Compact Hemi SB44 ST50, black, bronze		1
21	1506852-0419	Cross-brace double Compact Hemi SB46 ST32, silver grey		1
21	1506852-0442	Cross-brace double Compact Hemi SB46 ST32, night shimmer	2009-01-01 →	1
21	1506852-3524	Cross-brace double Compact Hemi SB46 ST32, royal red		1
21	1506852-5969	Cross-brace double Compact Hemi SB46 ST32, royal blue		1
21	1506852-9005	Cross-brace double Compact Hemi SB46 ST32, black, glossy		1
21	1506852-9119	Cross-brace double Compact Hemi SB46 ST32, black, bronze		1
22	1506853-0419	Cross-brace double Compact Hemi SB46 ST34, silver grey		1
22	1506853-0442	Cross-brace double Compact Hemi SB46 ST34, night shimmer	2009-01-01 →	1
22	1506853-3524	Cross-brace double Compact Hemi SB46 ST34, royal red		1
22	1506853-5969	Cross-brace double Compact Hemi SB46 ST34, royal blue		1
22	1506853-9005	Cross-brace double Compact Hemi SB46 ST34, black, glossy		1
22	1506853-9119	Cross-brace double Compact Hemi SB46 ST34, black, bronze		1
23	1506854-0419	Cross-brace double Compact Hemi SB46 ST36, silver grey		1
23	1506854-0442	Cross-brace double Compact Hemi SB46 ST36, night shimmer	2009-01-01 →	1
23	1506854-3524	Cross-brace double Compact Hemi SB46 ST36, royal red		1
23	1506854-5969	Cross-brace double Compact Hemi SB46 ST36, royal blue		1
23	1506854-9005	Cross-brace double Compact Hemi SB46 ST36, black, glossy		1
23	1506854-9119	Cross-brace double Compact Hemi SB46 ST36, black, bronze		1
24	1506855-0419	Cross-brace double Compact Hemi SB46 ST38, silver grey		1
24	1506855-0442	Cross-brace double Compact Hemi SB46 ST38, night shimmer	2009-01-01 →	1
24	1506855-3524	Cross-brace double Compact Hemi SB46 ST38, royal red		1

**Cross brace to frame Hemi, SB42-50**

Pos.	Part number	Description	Valid from → until	Qty
24	1506855-5969	Cross-brace double Compact Hemi SB46 ST38, royal blue		1
24	1506855-9005	Cross-brace double Compact Hemi SB46 ST38, black, glossy		1
24	1506855-9119	Cross-brace double Compact Hemi SB46 ST38, black, bronze		1
25	1506856-0419	Cross-brace double Compact Hemi SB46 ST40, silver grey		1
25	1506856-0442	Cross-brace double Compact Hemi SB46 ST40, night shimmer	2009-01-01 →	1
25	1506856-3524	Cross-brace double Compact Hemi SB46 ST40, royal red		1
25	1506856-5969	Cross-brace double Compact Hemi SB46 ST40, royal blue		1
25	1506856-9005	Cross-brace double Compact Hemi SB46 ST40, black, glossy		1
25	1506856-9119	Cross-brace double Compact Hemi SB46 ST40, black, bronze		1
26	1506857-0419	Cross-brace double Compact Hemi SB46 ST42, silver grey		1
26	1506857-0442	Cross-brace double Compact Hemi SB46 ST42, night shimmer	2009-01-01 →	1
26	1506857-3524	Cross-brace double Compact Hemi SB46 ST42, royal red		1
26	1506857-5969	Cross-brace double Compact Hemi SB46 ST42, royal blue		1
26	1506857-9005	Cross-brace double Compact Hemi SB46 ST42, black, glossy		1
26	1506857-9119	Cross-brace double Compact Hemi SB46 ST42, black, bronze		1
27	1506858-0419	Cross-brace double Compact Hemi SB46 ST44, silver grey		1
27	1506858-0442	Cross-brace double Compact Hemi SB46 ST44, night shimmer	2009-01-01 →	1
27	1506858-3524	Cross-brace double Compact Hemi SB46 ST44, royal red		1
27	1506858-5969	Cross-brace double Compact Hemi SB46 ST44, royal blue		1
27	1506858-9005	Cross-brace double Compact Hemi SB46 ST44, black, glossy		1
27	1506858-9119	Cross-brace double Compact Hemi SB46 ST44, black, bronze		1
28	1506859-0419	Cross-brace double Compact Hemi SB46 ST46, silver grey		1
28	1506859-0442	Cross-brace double Compact Hemi SB46 ST46, night shimmer	2009-01-01 →	1
28	1506859-3524	Cross-brace double Compact Hemi SB46 ST46, royal red		1
28	1506859-5969	Cross-brace double Compact Hemi SB46 ST46, royal blue		1
28	1506859-9005	Cross-brace double Compact Hemi SB46 ST46, black, glossy		1
28	1506859-9119	Cross-brace double Compact Hemi SB46 ST46, black, bronze		1
29	1506860-0419	Cross-brace double Compact Hemi SB46 ST48, silver grey		1
29	1506860-0442	Cross-brace double Compact Hemi SB46 ST48, night shimmer	2009-01-01 →	1
29	1506860-3524	Cross-brace double Compact Hemi SB46 ST48, royal red		1
29	1506860-5969	Cross-brace double Compact Hemi SB46 ST48, royal blue		1
29	1506860-9005	Cross-brace double Compact Hemi SB46 ST48, black, glossy		1
29	1506860-9119	Cross-brace double Compact Hemi SB46 ST48, black, bronze		1
30	1506861-0419	Cross-brace double Compact Hemi SB46 ST50, silver grey		1
30	1506861-0442	Cross-brace double Compact Hemi SB46 ST50, night shimmer	2009-01-01 →	1
30	1506861-3524	Cross-brace double Compact Hemi SB46 ST50, royal red		1
30	1506861-5969	Cross-brace double Compact Hemi SB46 ST50, royal blue		1
30	1506861-9005	Cross-brace double Compact Hemi SB46 ST50, black, glossy		1
30	1506861-9119	Cross-brace double Compact Hemi SB46 ST50, black, bronze		1
31	1506862-0419	Cross-brace double Compact Hemi SB48 ST32, silver grey		1
31	1506862-0442	Cross-brace double Compact Hemi SB48 ST32, night shimmer	2009-01-01 →	1
31	1506862-3524	Cross-brace double Compact Hemi SB48 ST32, royal red		1
31	1506862-5969	Cross-brace double Compact Hemi SB48 ST32, royal blue		1
31	1506862-9005	Cross-brace double Compact Hemi SB48 ST32, black, glossy		1
31	1506862-9119	Cross-brace double Compact Hemi SB48 ST32, black, bronze		1
32	1506863-0419	Cross-brace double Compact Hemi SB48 ST34, silver grey		1
32	1506863-0442	Cross-brace double Compact Hemi SB48 ST34, night shimmer	2009-01-01 →	1
32	1506863-3524	Cross-brace double Compact Hemi SB48 ST34, royal red		1
32	1506863-5969	Cross-brace double Compact Hemi SB48 ST34, royal blue		1
32	1506863-9005	Cross-brace double Compact Hemi SB48 ST34, black, glossy		1
32	1506863-9119	Cross-brace double Compact Hemi SB48 ST34, black, bronze		1
33	1506864-0419	Cross-brace double Compact Hemi SB48 ST36, silver grey		1
33	1506864-0442	Cross-brace double Compact Hemi SB48 ST36, night shimmer	2009-01-01 →	1
33	1506864-3524	Cross-brace double Compact Hemi SB48 ST36, royal red		1
33	1506864-5969	Cross-brace double Compact Hemi SB48 ST36, royal blue		1




**Cross brace to frame Hemi, SB42-50**

Pos.	Part number	Description	Valid from → until	Qty
33	1506864-9005	Cross-brace double Compact Hemi SB48 ST36, black, glossy		1
33	1506864-9119	Cross-brace double Compact Hemi SB48 ST36, black, bronze		1
34	1506865-0419	Cross-brace double Compact Hemi SB48 ST38, silver grey		1
34	1506865-0442	Cross-brace double Compact Hemi SB48 ST38, night shimmer	2009-01-01 →	1
34	1506865-3524	Cross-brace double Compact Hemi SB48 ST38, royal red		1
34	1506865-5969	Cross-brace double Compact Hemi SB48 ST38, royal blue		1
34	1506865-9005	Cross-brace double Compact Hemi SB48 ST38, black, glossy		1
34	1506865-9119	Cross-brace double Compact Hemi SB48 ST38, black, bronze		1
35	1506866-0419	Cross-brace double Compact Hemi SB48 ST40, silver grey		1
35	1506866-0442	Cross-brace double Compact Hemi SB48 ST40, night shimmer	2009-01-01 →	1
35	1506866-3524	Cross-brace double Compact Hemi SB48 ST40, royal red		1
35	1506866-5969	Cross-brace double Compact Hemi SB48 ST40, royal blue		1
35	1506866-9005	Cross-brace double Compact Hemi SB48 ST40, black, glossy		1
35	1506866-9119	Cross-brace double Compact Hemi SB48 ST40, black, bronze		1
36	1506867-0419	Cross-brace double Compact Hemi SB48 ST42, silver grey		1
36	1506867-0442	Cross-brace double Compact Hemi SB48 ST42, night shimmer	2009-01-01 →	1
36	1506867-3524	Cross-brace double Compact Hemi SB48 ST42, royal red		1
36	1506867-5969	Cross-brace double Compact Hemi SB48 ST42, royal blue		1
36	1506867-9005	Cross-brace double Compact Hemi SB48 ST42, black, glossy		1
36	1506867-9119	Cross-brace double Compact Hemi SB48 ST42, black, bronze		1
37	1506868-0419	Cross-brace double Compact Hemi SB48 ST44, silver grey		1
37	1506868-0442	Cross-brace double Compact Hemi SB48 ST44, night shimmer	2009-01-01 →	1
37	1506868-3524	Cross-brace double Compact Hemi SB48 ST44, royal red		1
37	1506868-5969	Cross-brace double Compact Hemi SB48 ST44, royal blue		1
37	1506868-9005	Cross-brace double Compact Hemi SB48 ST44, black, glossy		1
37	1506868-9119	Cross-brace double Compact Hemi SB48 ST44, black, bronze		1
38	1506869-0419	Cross-brace double Compact Hemi SB48 ST46, silver grey		1
38	1506869-0442	Cross-brace double Compact Hemi SB48 ST46, night shimmer	2009-01-01 →	1
38	1506869-3524	Cross-brace double Compact Hemi SB48 ST46, royal red		1
38	1506869-5969	Cross-brace double Compact Hemi SB48 ST46, royal blue		1
38	1506869-9005	Cross-brace double Compact Hemi SB48 ST46, black, glossy		1
38	1506869-9119	Cross-brace double Compact Hemi SB48 ST46, black, bronze		1
39	1506870-0419	Cross-brace double Compact Hemi SB48 ST48, silver grey		1
39	1506870-0442	Cross-brace double Compact Hemi SB48 ST48, night shimmer	2009-01-01 →	1
39	1506870-3524	Cross-brace double Compact Hemi SB48 ST48, royal red		1
39	1506870-5969	Cross-brace double Compact Hemi SB48 ST48, royal blue		1
39	1506870-9005	Cross-brace double Compact Hemi SB48 ST48, black, glossy		1
39	1506870-9119	Cross-brace double Compact Hemi SB48 ST48, black, bronze		1
40	1506871-0419	Cross-brace double Compact Hemi SB48 ST50, silver grey		1
40	1506871-0442	Cross-brace double Compact Hemi SB48 ST50, night shimmer	2009-01-01 →	1
40	1506871-3524	Cross-brace double Compact Hemi SB48 ST50, royal red		1
40	1506871-5969	Cross-brace double Compact Hemi SB48 ST50, royal blue		1
40	1506871-9005	Cross-brace double Compact Hemi SB48 ST50, black, glossy		1
40	1506871-9119	Cross-brace double Compact Hemi SB48 ST50, black, bronze		1
41	1506872-0419	Cross-brace double Compact Hemi SB50 ST32, silver grey		1
41	1506872-0442	Cross-brace double Compact Hemi SB50 ST32, night shimmer	2009-01-01 →	1
41	1506872-3524	Cross-brace double Compact Hemi SB50 ST32, royal red		1
41	1506872-5969	Cross-brace double Compact Hemi SB50 ST32, royal blue		1
41	1506872-9005	Cross-brace double Compact Hemi SB50 ST32, black, glossy		1
41	1506872-9119	Cross-brace double Compact Hemi SB50 ST32, black, bronze		1
42	1506873-0419	Cross-brace double Compact Hemi SB50 ST34, silver grey		1
42	1506873-0442	Cross-brace double Compact Hemi SB50 ST34, night shimmer	2009-01-01 →	1
42	1506873-3524	Cross-brace double Compact Hemi SB50 ST34, royal red		1
42	1506873-5969	Cross-brace double Compact Hemi SB50 ST34, royal blue		1
42	1506873-9005	Cross-brace double Compact Hemi SB50 ST34, black, glossy		1

**Cross brace to frame Hemi, SB42-50**

Pos.	Part number	Description	Valid from → until	Qty
42	1506873-9119	Cross-brace double Compact Hemi SB50 ST34, black, bronze		1
43	1506874-0419	Cross-brace double Compact Hemi SB50 ST36, silver grey		1
43	1506874-0442	Cross-brace double Compact Hemi SB50 ST36, night shimmer	2009-01-01 →	1
43	1506874-3524	Cross-brace double Compact Hemi SB50 ST36, royal red		1
43	1506874-5969	Cross-brace double Compact Hemi SB50 ST36, royal blue		1
43	1506874-9005	Cross-brace double Compact Hemi SB50 ST36, black, glossy		1
43	1506874-9119	Cross-brace double Compact Hemi SB50 ST36, black, bronze		1
44	1506875-0419	Cross-brace double Compact Hemi SB50 ST38, silver grey		1
44	1506875-0442	Cross-brace double Compact Hemi SB50 ST38, night shimmer	2009-01-01 →	1
44	1506875-3524	Cross-brace double Compact Hemi SB50 ST38, royal red		1
44	1506875-5969	Cross-brace double Compact Hemi SB50 ST38, royal blue		1
44	1506875-9005	Cross-brace double Compact Hemi SB50 ST38, black, glossy		1
44	1506875-9119	Cross-brace double Compact Hemi SB50 ST38, black, bronze		1
45	1506876-0419	Cross-brace double Compact Hemi SB50 ST40, silver grey		1
45	1506876-0442	Cross-brace double Compact Hemi SB50 ST40, night shimmer	2009-01-01 →	1
45	1506876-3524	Cross-brace double Compact Hemi SB50 ST40, royal red		1
45	1506876-5969	Cross-brace double Compact Hemi SB50 ST40, royal blue		1
45	1506876-9005	Cross-brace double Compact Hemi SB50 ST40, black, glossy		1
45	1506876-9119	Cross-brace double Compact Hemi SB50 ST40, black, bronze		1
46	1506877-0419	Cross-brace double Compact Hemi SB50 ST42, silver grey		1
46	1506877-0442	Cross-brace double Compact Hemi SB50 ST42, night shimmer	2009-01-01 →	1
46	1506877-3524	Cross-brace double Compact Hemi SB50 ST42, royal red		1
46	1506877-5969	Cross-brace double Compact Hemi SB50 ST42, royal blue		1
46	1506877-9005	Cross-brace double Compact Hemi SB50 ST42, black, glossy		1
46	1506877-9119	Cross-brace double Compact Hemi SB50 ST42, black, bronze		1
47	1506878-0419	Cross-brace double Compact Hemi SB50 ST44, silver grey		1
47	1506878-0442	Cross-brace double Compact Hemi SB50 ST44, night shimmer	2009-01-01 →	1
47	1506878-3524	Cross-brace double Compact Hemi SB50 ST44, royal red		1
47	1506878-5969	Cross-brace double Compact Hemi SB50 ST44, royal blue		1
47	1506878-9005	Cross-brace double Compact Hemi SB50 ST44, black, glossy		1
47	1506878-9119	Cross-brace double Compact Hemi SB50 ST44, black, bronze		1
48	1506879-0419	Cross-brace double Compact Hemi SB50 ST46, silver grey		1
48	1506879-0442	Cross-brace double Compact Hemi SB50 ST46, night shimmer	2009-01-01 →	1
48	1506879-3524	Cross-brace double Compact Hemi SB50 ST46, royal red		1
48	1506879-5969	Cross-brace double Compact Hemi SB50 ST46, royal blue		1
48	1506879-9005	Cross-brace double Compact Hemi SB50 ST46, black, glossy		1
48	1506879-9119	Cross-brace double Compact Hemi SB50 ST46, black, bronze		1
49	1506880-0419	Cross-brace double Compact Hemi SB50 ST48, silver grey		1
49	1506880-0442	Cross-brace double Compact Hemi SB50 ST48, night shimmer	2009-01-01 →	1
49	1506880-3524	Cross-brace double Compact Hemi SB50 ST48, royal red		1
49	1506880-5969	Cross-brace double Compact Hemi SB50 ST48, royal blue		1
49	1506880-9005	Cross-brace double Compact Hemi SB50 ST48, black, glossy		1
49	1506880-9119	Cross-brace double Compact Hemi SB50 ST48, black, bronze		1
50	1506881-0419	Cross-brace double Compact Hemi SB50 ST50, silver grey		1
50	1506881-0442	Cross-brace double Compact Hemi SB50 ST50, night shimmer	2009-01-01 →	1
50	1506881-3524	Cross-brace double Compact Hemi SB50 ST50, royal red		1
50	1506881-5969	Cross-brace double Compact Hemi SB50 ST50, royal blue		1
50	1506881-9005	Cross-brace double Compact Hemi SB50 ST50, black, glossy		1
50	1506881-9119	Cross-brace double Compact Hemi SB50 ST50, black, bronze		1
51	1437944	Retaining lever SB40-44		1
52	1437945	Retaining lever SB46-50		1
53	1507625	Center division kit		1
54	1439480	Screw kit to Double crossbrace		1

## Cross brace to frame Hemi abduction, SB32-40

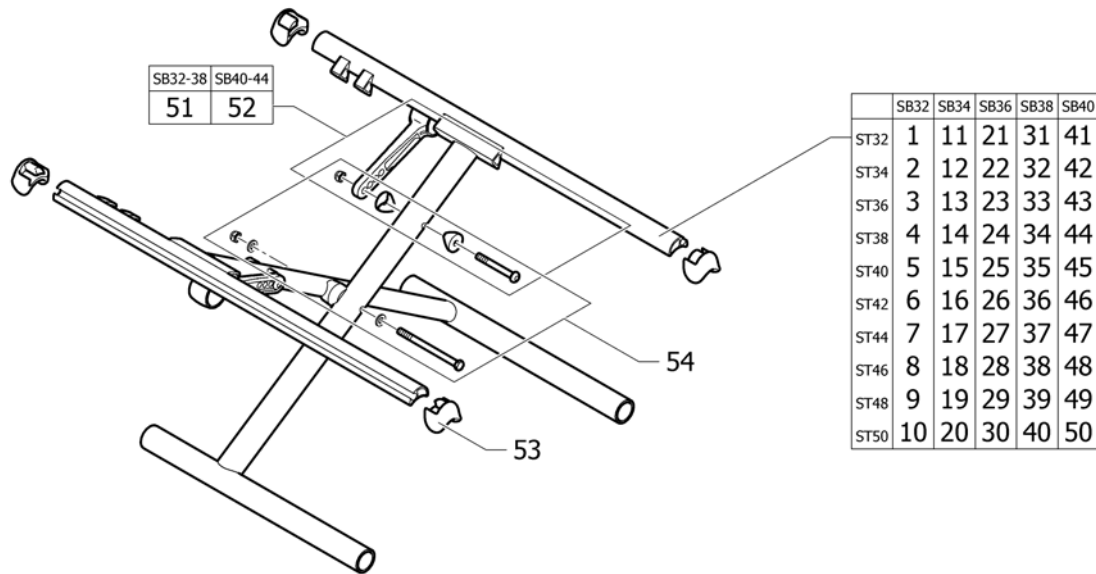

Pos.	Part number	Description	Valid from → until	Qty
1	1506667-0419	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST32, silver grey		1
1	1506667-0442	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST32, night shimmer	2009-01-01 →	1
1	1506667-3524	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST32, royal red		1
1	1506667-5969	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST32, royal blue		1
1	1506667-9005	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST32, black, glossy		1
1	1506667-9119	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST32, black, bronze		1
2	1506668-0419	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST34, silver grey		1
2	1506668-0442	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST34, night shimmer	2009-01-01 →	1
2	1506668-3524	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST34, royal red		1
2	1506668-5969	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST34, royal blue		1
2	1506668-9005	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST34, black, glossy		1
2	1506668-9119	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST34, black, bronze		1
3	1506669-0419	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST36, silver grey		1
3	1506669-0442	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST36, night shimmer	2009-01-01 →	1
3	1506669-3524	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST36, royal red		1
3	1506669-5969	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST36, royal blue		1
3	1506669-9005	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST36, black, glossy		1
3	1506669-9119	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST36, black, bronze		1
4	1506670-0419	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST38, silver grey		1
4	1506670-0442	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST38, night shimmer	2009-01-01 →	1
4	1506670-3524	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST38, royal red		1
4	1506670-5969	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST38, royal blue		1
4	1506670-9005	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST38, black, glossy		1
4	1506670-9119	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST38, black, bronze		1
5	1506671-0419	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST40, silver grey		1

**Cross brace to frame Hemi abduction, SB32-40**

Pos.	Part number	Description	Valid from → until	Qty
5	1506671-0442	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST40, night shimmer	2009-01-01 →	1
5	1506671-3524	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST40, royal red		1
5	1506671-5969	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST40, royal blue		1
5	1506671-9005	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST40, black, glossy		1
5	1506671-9119	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST40, black, bronze		1
6	1506672-0419	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST42, silver grey		1
6	1506672-0442	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST42, night shimmer	2009-01-01 →	1
6	1506672-3524	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST42, royal red		1
6	1506672-5969	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST42, royal blue		1
6	1506672-9005	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST42, black, glossy		1
6	1506672-9119	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST42, black, bronze		1
7	1506673-0419	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST44, silver grey		1
7	1506673-0442	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST44, night shimmer	2009-01-01 →	1
7	1506673-3524	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST44, royal red		1
7	1506673-5969	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST44, royal blue		1
7	1506673-9005	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST44, black, glossy		1
7	1506673-9119	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST44, black, bronze		1
8	1506674-0419	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST46, silver grey		1
8	1506674-0442	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST46, night shimmer	2009-01-01 →	1
8	1506674-3524	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST46, royal red		1
8	1506674-5969	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST46, royal blue		1
8	1506674-9005	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST46, black, glossy		1
8	1506674-9119	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST46, black, bronze		1
9	1506675-0419	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST48, silver grey		1
9	1506675-0442	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST48, night shimmer	2009-01-01 →	1
9	1506675-3524	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST48, royal red		1
9	1506675-5969	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST48, royal blue		1
9	1506675-9005	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST48, black, glossy		1
9	1506675-9119	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST48, black, bronze		1
10	1506676-0419	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST50, silver grey		1
10	1506676-0442	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST50, night shimmer	2009-01-01 →	1
10	1506676-3524	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST50, royal red		1
10	1506676-5969	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST50, royal blue		1
10	1506676-9005	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST50, black, glossy		1
10	1506676-9119	Cross-brace single Hemi abduct., UltraLight 80° SB32 ST50, black, bronze		1
11	1506677-0419	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST32, silver grey		1
11	1506677-0442	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST32, night shimmer	2009-01-01 →	1
11	1506677-3524	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST32, royal red		1
11	1506677-5969	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST32, royal blue		1
11	1506677-9005	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST32, black, glossy		1
11	1506677-9119	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST32, black, bronze		1
12	1506678-0419	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST34, silver grey		1
12	1506678-0442	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST34, night shimmer	2009-01-01 →	1
12	1506678-3524	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST34, royal red		1

**Cross brace to frame Hemi abduction, SB32-40**

Pos.	Part number	Description	Valid from → until	Qty
12	1506678-5969	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST34, royal blue		1
12	1506678-9005	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST34, black, glossy		1
12	1506678-9119	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST34, black, bronze		1
13	1506679-0419	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST36, silver grey		1
13	1506679-0442	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST36, night shimmer	2009-01-01 →	1
13	1506679-3524	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST36, royal red		1
13	1506679-5969	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST36, royal blue		1
13	1506679-9005	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST36, black, glossy		1
13	1506679-9119	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST36, black, bronze		1
14	1506680-0419	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST38, silver grey		1
14	1506680-0442	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST38, night shimmer	2009-01-01 →	1
14	1506680-3524	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST38, royal red		1
14	1506680-5969	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST38, royal blue		1
14	1506680-9005	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST38, black, glossy		1
14	1506680-9119	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST38, black, bronze		1
15	1506681-0419	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST40, silver grey		1
15	1506681-0442	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST40, night shimmer	2009-01-01 →	1
15	1506681-3524	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST40, royal red		1
15	1506681-5969	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST40, royal blue		1
15	1506681-9005	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST40, black, glossy		1
15	1506681-9119	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST40, black, bronze		1
16	1506682-0419	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST42, silver grey		1
16	1506682-0442	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST42, night shimmer	2009-01-01 →	1
16	1506682-3524	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST42, royal red		1
16	1506682-5969	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST42, royal blue		1
16	1506682-9005	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST42, black, glossy		1
16	1506682-9119	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST42, black, bronze		1
17	1506683-0419	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST44, silver grey		1
17	1506683-0442	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST44, night shimmer	2009-01-01 →	1
17	1506683-3524	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST44, royal red		1
17	1506683-5969	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST44, royal blue		1
17	1506683-9005	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST44, black, glossy		1
17	1506683-9119	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST44, black, bronze		1
18	1506684-0419	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST46, silver grey		1
18	1506684-0442	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST46, night shimmer	2009-01-01 →	1
18	1506684-3524	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST46, royal red		1
18	1506684-5969	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST46, royal blue		1
18	1506684-9005	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST46, black, glossy		1
18	1506684-9119	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST46, black, bronze		1
19	1506685-0419	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST48, silver grey		1
19	1506685-0442	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST48, night shimmer	2009-01-01 →	1
19	1506685-3524	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST48, royal red		1
19	1506685-5969	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST48, royal blue		1
19	1506685-9005	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST48, black, glossy		1



**Cross brace to frame Hemi abduction, SB32-40**

Pos.	Part number	Description	Valid from → until	Qty
19	1506685-9119	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST48, black, bronze		1
20	1506686-0419	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST50, silver grey		1
20	1506686-0442	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST50, night shimmer	2009-01-01 →	1
20	1506686-3524	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST50, royal red		1
20	1506686-5969	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST50, royal blue		1
20	1506686-9005	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST50, black, glossy		1
20	1506686-9119	Cross-brace single Hemi abduct., UltraLight 80° SB34 ST50, black, bronze		1
21	1506687-0419	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST32, silver grey		1
21	1506687-0442	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST32, night shimmer	2009-01-01 →	1
21	1506687-3524	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST32, royal red		1
21	1506687-5969	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST32, royal blue		1
21	1506687-9005	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST32, black, glossy		1
21	1506687-9119	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST32, black, bronze		1
22	1506688-0419	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST34, silver grey		1
22	1506688-0442	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST34, night shimmer	2009-01-01 →	1
22	1506688-3524	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST34, royal red		1
22	1506688-5969	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST34, royal blue		1
22	1506688-9005	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST34, black, glossy		1
22	1506688-9119	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST34, black, bronze		1
23	1506704-0419	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST36, silver grey		1
23	1506704-0442	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST36, night shimmer	2009-01-01 →	1
23	1506704-3524	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST36, royal red		1
23	1506704-5969	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST36, royal blue		1
23	1506704-9005	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST36, black, glossy		1
23	1506704-9119	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST36, black, bronze		1
24	1506705-0419	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST38, silver grey		1
24	1506705-0442	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST38, night shimmer	2009-01-01 →	1
24	1506705-3524	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST38, royal red		1
24	1506705-5969	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST38, royal blue		1
24	1506705-9005	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST38, black, glossy		1
24	1506705-9119	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST38, black, bronze		1
25	1506706-0419	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST40, silver grey		1
25	1506706-0442	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST40, night shimmer	2009-01-01 →	1
25	1506706-3524	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST40, royal red		1
25	1506706-5969	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST40, royal blue		1
25	1506706-9005	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST40, black, glossy		1
25	1506706-9119	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST40, black, bronze		1
26	1506707-0419	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST42, silver grey		1
26	1506707-0442	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST42, night shimmer	2009-01-01 →	1
26	1506707-3524	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST42, royal red		1
26	1506707-5969	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST42, royal blue		1
26	1506707-9005	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST42, black, glossy		1
26	1506707-9119	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST42, black, bronze		1
27	1506708-0419	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST44, silver grey		1



**Cross brace to frame Hemi abduction, SB32-40**

Pos.	Part number	Description	Valid from → until	Qty
27	1506708-0442	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST44, night shimmer	2009-01-01 →	1
27	1506708-3524	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST44, royal red		1
27	1506708-5969	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST44, royal blue		1
27	1506708-9005	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST44, black, glossy		1
27	1506708-9119	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST44, black, bronze		1
28	1506709-0419	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST46, silver grey		1
28	1506709-0442	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST46, night shimmer	2009-01-01 →	1
28	1506709-3524	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST46, royal red		1
28	1506709-5969	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST46, royal blue		1
28	1506709-9005	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST46, black, glossy		1
28	1506709-9119	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST46, black, bronze		1
29	1506710-0419	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST48, silver grey		1
29	1506710-0442	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST48, night shimmer	2009-01-01 →	1
29	1506710-3524	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST48, royal red		1
29	1506710-5969	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST48, royal blue		1
29	1506710-9005	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST48, black, glossy		1
29	1506710-9119	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST48, black, bronze		1
30	1506711-0419	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST50, silver grey		1
30	1506711-0442	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST50, night shimmer	2009-01-01 →	1
30	1506711-3524	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST50, royal red		1
30	1506711-5969	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST50, royal blue		1
30	1506711-9005	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST50, black, glossy		1
30	1506711-9119	Cross-brace single Hemi abduct., UltraLight 80° SB36 ST50, black, bronze		1
31	1506712-0419	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST32, silver grey		1
31	1506712-0442	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST32, night shimmer	2009-01-01 →	1
31	1506712-3524	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST32, royal red		1
31	1506712-5969	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST32, royal blue		1
31	1506712-9005	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST32, black, glossy		1
31	1506712-9119	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST32, black, bronze		1
32	1506713-0419	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST34, silver grey		1
32	1506713-0442	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST34, night shimmer	2009-01-01 →	1
32	1506713-3524	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST34, royal red		1
32	1506713-5969	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST34, royal blue		1
32	1506713-9005	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST34, black, glossy		1
32	1506713-9119	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST34, black, bronze		1
33	1506714-0419	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST36, silver grey		1
33	1506714-0442	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST36, night shimmer	2009-01-01 →	1
33	1506714-3524	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST36, royal red		1
33	1506714-5969	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST36, royal blue		1
33	1506714-9005	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST36, black, glossy		1
33	1506714-9119	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST36, black, bronze		1
34	1506715-0419	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST38, silver grey		1
34	1506715-0442	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST38, night shimmer	2009-01-01 →	1
34	1506715-3524	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST38, royal red		1

**Cross brace to frame Hemi abduction, SB32-40**

Pos.	Part number	Description	Valid from → until	Qty
34	1506715-5969	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST38, royal blue		1
34	1506715-9005	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST38, black, glossy		1
34	1506715-9119	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST38, black, bronze		1
35	1506716-0419	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST40, silver grey		1
35	1506716-0442	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST40, night shimmer	2009-01-01 →	1
35	1506716-3524	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST40, royal red		1
35	1506716-5969	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST40, royal blue		1
35	1506716-9005	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST40, black, glossy		1
35	1506716-9119	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST40, black, bronze		1
36	1506717-0419	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST42, silver grey		1
36	1506717-0442	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST42, night shimmer	2009-01-01 →	1
36	1506717-3524	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST42, royal red		1
36	1506717-5969	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST42, royal blue		1
36	1506717-9005	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST42, black, glossy		1
36	1506717-9119	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST42, black, bronze		1
37	1506718-0419	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST44, silver grey		1
37	1506718-0442	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST44, night shimmer	2009-01-01 →	1
37	1506718-3524	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST44, royal red		1
37	1506718-5969	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST44, royal blue		1
37	1506718-9005	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST44, black, glossy		1
37	1506718-9119	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST44, black, bronze		1
38	1506719-0419	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST46, silver grey		1
38	1506719-0442	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST46, night shimmer	2009-01-01 →	1
38	1506719-3524	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST46, royal red		1
38	1506719-5969	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST46, royal blue		1
38	1506719-9005	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST46, black, glossy		1
38	1506719-9119	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST46, black, bronze		1
39	1506720-0419	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST48, silver grey		1
39	1506720-0442	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST48, night shimmer	2009-01-01 →	1
39	1506720-3524	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST48, royal red		1
39	1506720-5969	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST48, royal blue		1
39	1506720-9005	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST48, black, glossy		1
39	1506720-9119	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST48, black, bronze		1
40	1506721-0419	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST50, silver grey		1
40	1506721-0442	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST50, night shimmer	2009-01-01 →	1
40	1506721-3524	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST50, royal red		1
40	1506721-5969	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST50, royal blue		1
40	1506721-9005	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST50, black, glossy		1
40	1506721-9119	Cross-brace single Hemi abduct., UltraLight 80° SB38 ST50, black, bronze		1
41	1506722-0419	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST32, silver grey		1
41	1506722-0442	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST32, night shimmer	2009-01-01 →	1
41	1506722-3524	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST32, royal red		1
41	1506722-5969	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST32, royal blue		1
41	1506722-9005	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST32, black, glossy		1


**Cross brace to frame Hemi abduction, SB32-40**

Pos.	Part number	Description	Valid from → until	Qty
41	1506722-9119	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST32, black, bronze		1
42	1506723-0419	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST34, silver grey		1
42	1506723-0442	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST34, night shimmer	2009-01-01 →	1
42	1506723-3524	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST34, royal red		1
42	1506723-5969	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST34, royal blue		1
42	1506723-9005	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST34, black, glossy		1
42	1506723-9119	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST34, black, bronze		1
43	1506724-0419	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST36, silver grey		1
43	1506724-0442	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST36, night shimmer	2009-01-01 →	1
43	1506724-3524	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST36, royal red		1
43	1506724-5969	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST36, royal blue		1
43	1506724-9005	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST36, black, glossy		1
43	1506724-9119	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST36, black, bronze		1
44	1506725-0419	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST38, silver grey		1
44	1506725-0442	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST38, night shimmer	2009-01-01 →	1
44	1506725-3524	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST38, royal red		1
44	1506725-5969	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST38, royal blue		1
44	1506725-9005	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST38, black, glossy		1
44	1506725-9119	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST38, black, bronze		1
45	1506726-0419	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST40, silver grey		1
45	1506726-0442	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST40, night shimmer	2009-01-01 →	1
45	1506726-3524	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST40, royal red		1
45	1506726-5969	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST40, royal blue		1
45	1506726-9005	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST40, black, glossy		1
45	1506726-9119	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST40, black, bronze		1
46	1506727-0419	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST42, silver grey		1
46	1506727-0442	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST42, night shimmer	2009-01-01 →	1
46	1506727-3524	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST42, royal red		1
46	1506727-5969	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST42, royal blue		1
46	1506727-9005	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST42, black, glossy		1
46	1506727-9119	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST42, black, bronze		1
47	1506728-0419	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST44, silver grey		1
47	1506728-0442	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST44, night shimmer	2009-01-01 →	1
47	1506728-3524	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST44, royal red		1
47	1506728-5969	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST44, royal blue		1
47	1506728-9005	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST44, black, glossy		1
47	1506728-9119	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST44, black, bronze		1
48	1506729-0419	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST46, silver grey		1
48	1506729-0442	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST46, night shimmer	2009-01-01 →	1
48	1506729-3524	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST46, royal red		1
48	1506729-5969	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST46, royal blue		1
48	1506729-9005	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST46, black, glossy		1
48	1506729-9119	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST46, black, bronze		1
49	1506730-0419	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST48, silver grey		1

**Cross brace to frame Hemi abduction, SB32-40**

Pos.	Part number	Description	Valid from → until	Qty
49	1506730-0442	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST48, night shimmer	2009-01-01 →	1
49	1506730-3524	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST48, royal red		1
49	1506730-5969	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST48, royal blue		1
49	1506730-9005	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST48, black, glossy		1
49	1506730-9119	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST48, black, bronze		1
50	1506731-0419	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST50, silver grey		1
50	1506731-0442	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST50, night shimmer	2009-01-01 →	1
50	1506731-3524	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST50, royal red		1
50	1506731-5969	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST50, royal blue		1
50	1506731-9005	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST50, black, glossy		1
50	1506731-9119	Cross-brace single Hemi abduct., UltraLight 80° SB40 ST50, black, bronze		1
51	1437943	Retaining lever SB32-38		1
52	1437944	Retaining lever SB40-44		1
53	1507625	Center division kit		1
54	1439479	Screw kit to Single crossbrace		1

## Cross brace to frame Hemi abduction, SB42-50

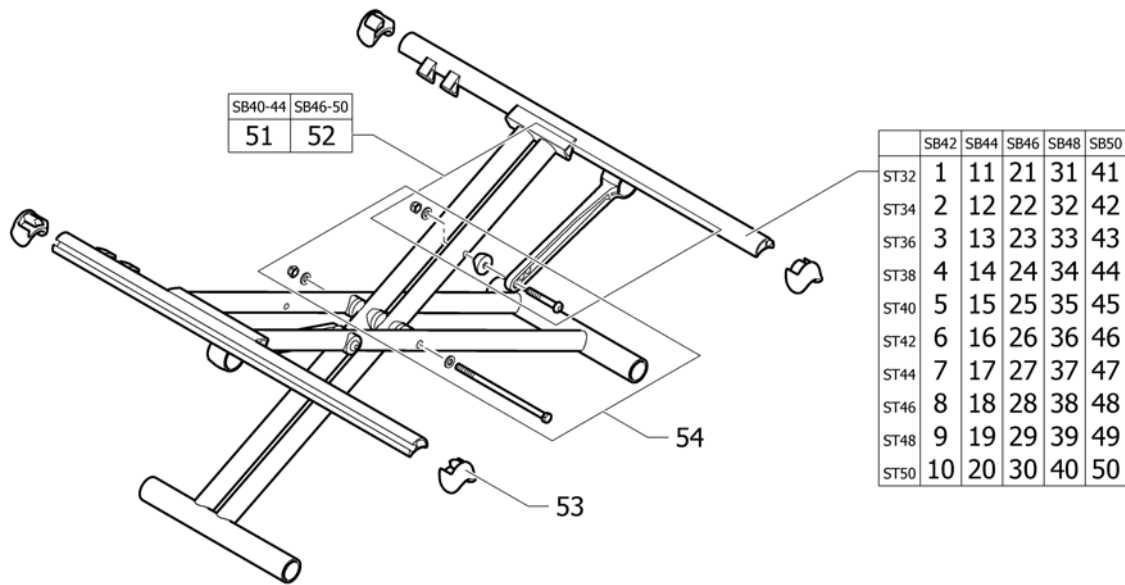

Pos.	Part number	Description	Valid from → until	Qty
1	1506732-0419	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST32, silver grey		1
1	1506732-0442	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST32, night shimmer	2009-01-01 →	1
1	1506732-3524	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST32, royal red		1
1	1506732-5969	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST32, royal blue		1
1	1506732-9005	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST32, black, glossy		1
1	1506732-9119	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST32, black, bronze		1
2	1506733-0419	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST34, silver grey		1
2	1506733-0442	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST34, night shimmer	2009-01-01 →	1
2	1506733-3524	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST34, royal red		1
2	1506733-5969	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST34, royal blue		1
2	1506733-9005	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST34, black, glossy		1
2	1506733-9119	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST34, black, bronze		1
3	1506734-0419	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST36, silver grey		1
3	1506734-0442	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST36, night shimmer	2009-01-01 →	1
3	1506734-3524	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST36, royal red		1
3	1506734-5969	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST36, royal blue		1
3	1506734-9005	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST36, black, glossy		1
3	1506734-9119	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST36, black, bronze		1
4	1506735-0419	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST38, silver grey		1
4	1506735-0442	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST38, night shimmer	2009-01-01 →	1
4	1506735-3524	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST38, royal red		1
4	1506735-5969	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST38, royal blue		1
4	1506735-9005	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST38, black, glossy		1



**Cross brace to frame Hemi abduction, SB42-50**

Pos.	Part number	Description	Valid from → until	Qty
4	1506735-9119	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST38, black, bronze		1
5	1506736-0419	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST40, silver grey		1
5	1506736-0442	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST40, night shimmer	2009-01-01 →	1
5	1506736-3524	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST40, royal red		1
5	1506736-5969	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST40, royal blue		1
5	1506736-9005	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST40, black, glossy		1
5	1506736-9119	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST40, black, bronze		1
6	1506737-0419	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST42, silver grey		1
6	1506737-0442	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST42, night shimmer	2009-01-01 →	1
6	1506737-3524	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST42, royal red		1
6	1506737-5969	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST42, royal blue		1
6	1506737-9005	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST42, black, glossy		1
6	1506737-9119	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST42, black, bronze		1
7	1506738-0419	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST44, silver grey		1
7	1506738-0442	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST44, night shimmer	2009-01-01 →	1
7	1506738-3524	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST44, royal red		1
7	1506738-5969	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST44, royal blue		1
7	1506738-9005	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST44, black, glossy		1
7	1506738-9119	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST44, black, bronze		1
8	1506739-0419	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST46, silver grey		1
8	1506739-0442	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST46, night shimmer	2009-01-01 →	1
8	1506739-3524	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST46, royal red		1
8	1506739-5969	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST46, royal blue		1
8	1506739-9005	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST46, black, glossy		1
8	1506739-9119	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST46, black, bronze		1
9	1506740-0419	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST48, silver grey		1
9	1506740-0442	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST48, night shimmer	2009-01-01 →	1
9	1506740-3524	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST48, royal red		1
9	1506740-5969	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST48, royal blue		1
9	1506740-9005	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST48, black, glossy		1
9	1506740-9119	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST48, black, bronze		1
10	1506741-0419	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST50, silver grey		1
10	1506741-0442	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST50, night shimmer	2009-01-01 →	1
10	1506741-3524	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST50, royal red		1
10	1506741-5969	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST50, royal blue		1
10	1506741-9005	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST50, black, glossy		1
10	1506741-9119	Cross-brace double Hemi abduct., UltraLight 80° SB42 ST50, black, bronze		1
11	1506742-0419	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST32, silver grey		1
11	1506742-0442	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST32, night shimmer	2009-01-01 →	1
11	1506742-3524	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST32, royal red		1



**Cross brace to frame Hemi abduction, SB42-50**

Pos.	Part number	Description	Valid from → until	Qty
11	1506742-5969	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST32, royal blue		1
11	1506742-9005	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST32, black, glossy		1
11	1506742-9119	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST32, black, bronze		1
12	1506743-0419	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST34, silver grey		1
12	1506743-0442	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST34, night shimmer	2009-01-01 →	1
12	1506743-3524	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST34, royal red		1
12	1506743-5969	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST34, royal blue		1
12	1506743-9005	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST34, black, glossy		1
12	1506743-9119	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST34, black, bronze		1
13	1506744-0419	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST36, silver grey		1
13	1506744-0442	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST36, night shimmer	2009-01-01 →	1
13	1506744-3524	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST36, royal red		1
13	1506744-5969	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST36, royal blue		1
13	1506744-9005	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST36, black, glossy		1
13	1506744-9119	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST36, black, bronze		1
14	1506745-0419	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST38, silver grey		1
14	1506745-0442	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST38, night shimmer	2009-01-01 →	1
14	1506745-3524	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST38, royal red		1
14	1506745-5969	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST38, royal blue		1
14	1506745-9005	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST38, black, glossy		1
14	1506745-9119	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST38, black, bronze		1
15	1506746-0419	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST40, silver grey		1
15	1506746-0442	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST40, night shimmer	2009-01-01 →	1
15	1506746-3524	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST40, royal red		1
15	1506746-5969	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST40, royal blue		1
15	1506746-9005	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST40, black, glossy		1
15	1506746-9119	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST40, black, bronze		1
16	1506747-0419	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST42, silver grey		1
16	1506747-0442	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST42, night shimmer	2009-01-01 →	1
16	1506747-3524	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST42, royal red		1
16	1506747-5969	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST42, royal blue		1
16	1506747-9005	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST42, black, glossy		1
16	1506747-9119	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST42, black, bronze		1
17	1506748-0419	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST44, silver grey		1
17	1506748-0442	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST44, night shimmer	2009-01-01 →	1
17	1506748-3524	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST44, royal red		1
17	1506748-5969	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST44, royal blue		1
17	1506748-9005	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST44, black, glossy		1
17	1506748-9119	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST44, black, bronze		1
18	1506749-0419	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST46, silver grey		1

**Cross brace to frame Hemi abduction, SB42-50**

Pos.	Part number	Description	Valid from → until	Qty
18	1506749-0442	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST46, night shimmer	2009-01-01 →	1
18	1506749-3524	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST46, royal red		1
18	1506749-5969	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST46, royal blue		1
18	1506749-9005	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST46, black, glossy		1
18	1506749-9119	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST46, black, bronze		1
19	1506750-0419	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST48, silver grey		1
19	1506750-0442	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST48, night shimmer	2009-01-01 →	1
19	1506750-3524	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST48, royal red		1
19	1506750-5969	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST48, royal blue		1
19	1506750-9005	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST48, black, glossy		1
19	1506750-9119	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST48, black, bronze		1
20	1506751-0419	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST50, silver grey		1
20	1506751-0442	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST50, night shimmer	2009-01-01 →	1
20	1506751-3524	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST50, royal red		1
20	1506751-5969	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST50, royal blue		1
20	1506751-9005	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST50, black, glossy		1
20	1506751-9119	Cross-brace double Hemi abduct., UltraLight 80° SB44 ST50, black, bronze		1
21	1506752-0419	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST32, silver grey		1
21	1506752-0442	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST32, night shimmer	2009-01-01 →	1
21	1506752-3524	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST32, royal red		1
21	1506752-5969	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST32, royal blue		1
21	1506752-9005	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST32, black, glossy		1
21	1506752-9119	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST32, black, bronze		1
22	1506753-0419	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST34, silver grey		1
22	1506753-0442	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST34, night shimmer	2009-01-01 →	1
22	1506753-3524	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST34, royal red		1
22	1506753-5969	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST34, royal blue		1
22	1506753-9005	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST34, black, glossy		1
22	1506753-9119	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST34, black, bronze		1
23	1506754-0419	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST36, silver grey		1
23	1506754-0442	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST36, night shimmer	2009-01-01 →	1
23	1506754-3524	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST36, royal red		1
23	1506754-5969	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST36, royal blue		1
23	1506754-9005	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST36, black, glossy		1
23	1506754-9119	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST36, black, bronze		1
24	1506755-0419	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST38, silver grey		1
24	1506755-0442	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST38, night shimmer	2009-01-01 →	1
24	1506755-3524	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST38, royal red		1
24	1506755-5969	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST38, royal blue		1
24	1506755-9005	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST38, black, glossy		1

**Cross brace to frame Hemi abduction, SB42-50**

Pos.	Part number	Description	Valid from → until	Qty
24	1506755-9119	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST38, black, bronze		1
25	1506756-0419	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST40, silver grey		1
25	1506756-0442	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST40, night shimmer	2009-01-01 →	1
25	1506756-3524	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST40, royal red		1
25	1506756-5969	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST40, royal blue		1
25	1506756-9005	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST40, black, glossy		1
25	1506756-9119	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST40, black, bronze		1
26	1506757-0419	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST42, silver grey		1
26	1506757-0442	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST42, night shimmer	2009-01-01 →	1
26	1506757-3524	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST42, royal red		1
26	1506757-5969	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST42, royal blue		1
26	1506757-9005	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST42, black, glossy		1
26	1506757-9119	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST42, black, bronze		1
27	1506758-0419	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST44, silver grey		1
27	1506758-0442	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST44, night shimmer	2009-01-01 →	1
27	1506758-3524	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST44, royal red		1
27	1506758-5969	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST44, royal blue		1
27	1506758-9005	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST44, black, glossy		1
27	1506758-9119	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST44, black, bronze		1
28	1506759-0419	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST46, silver grey		1
28	1506759-0442	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST46, night shimmer	2009-01-01 →	1
28	1506759-3524	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST46, royal red		1
28	1506759-5969	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST46, royal blue		1
28	1506759-9005	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST46, black, glossy		1
28	1506759-9119	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST46, black, bronze		1
29	1506760-0419	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST48, silver grey		1
29	1506760-0442	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST48, night shimmer	2009-01-01 →	1
29	1506760-3524	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST48, royal red		1
29	1506760-5969	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST48, royal blue		1
29	1506760-9005	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST48, black, glossy		1
29	1506760-9119	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST48, black, bronze		1
30	1506761-0419	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST50, silver grey		1
30	1506761-0442	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST50, night shimmer	2009-01-01 →	1
30	1506761-3524	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST50, royal red		1
30	1506761-5969	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST50, royal blue		1
30	1506761-9005	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST50, black, glossy		1
30	1506761-9119	Cross-brace double Hemi abduct., UltraLight 80° SB46 ST50, black, bronze		1
31	1506762-0419	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST32, silver grey		1
31	1506762-0442	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST32, night shimmer	2009-01-01 →	1
31	1506762-3524	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST32, royal red		1

**Cross brace to frame Hemi abduction, SB42-50**

Pos.	Part number	Description	Valid from → until	Qty
31	1506762-5969	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST32, royal blue		1
31	1506762-9005	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST32, black, glossy		1
31	1506762-9119	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST32, black, bronze		1
32	1506763-0419	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST34, silver grey		1
32	1506763-0442	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST34, night shimmer	2009-01-01 →	1
32	1506763-3524	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST34, royal red		1
32	1506763-5969	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST34, royal blue		1
32	1506763-9005	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST34, black, glossy		1
32	1506763-9119	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST34, black, bronze		1
33	1506764-0419	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST36, silver grey		1
33	1506764-0442	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST36, night shimmer	2009-01-01 →	1
33	1506764-3524	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST36, royal red		1
33	1506764-5969	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST36, royal blue		1
33	1506764-9005	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST36, black, glossy		1
33	1506764-9119	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST36, black, bronze		1
34	1506765-0419	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST38, silver grey		1
34	1506765-0442	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST38, night shimmer	2009-01-01 →	1
34	1506765-3524	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST38, royal red		1
34	1506765-5969	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST38, royal blue		1
34	1506765-9005	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST38, black, glossy		1
34	1506765-9119	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST38, black, bronze		1
35	1506766-0419	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST40, silver grey		1
35	1506766-0442	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST40, night shimmer	2009-01-01 →	1
35	1506766-3524	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST40, royal red		1
35	1506766-5969	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST40, royal blue		1
35	1506766-9005	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST40, black, glossy		1
35	1506766-9119	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST40, black, bronze		1
36	1506767-0419	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST42, silver grey		1
36	1506767-0442	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST42, night shimmer	2009-01-01 →	1
36	1506767-3524	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST42, royal red		1
36	1506767-5969	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST42, royal blue		1
36	1506767-9005	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST42, black, glossy		1
36	1506767-9119	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST42, black, bronze		1
37	1506768-0419	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST44, silver grey		1
37	1506768-0442	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST44, night shimmer	2009-01-01 →	1
37	1506768-3524	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST44, royal red		1
37	1506768-5969	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST44, royal blue		1
37	1506768-9005	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST44, black, glossy		1
37	1506768-9119	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST44, black, bronze		1
38	1506769-0419	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST46, silver grey		1

**Cross brace to frame Hemi abduction, SB42-50**

Pos.	Part number	Description	Valid from → until	Qty
38	1506769-0442	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST46, night shimmer	2009-01-01 →	1
38	1506769-3524	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST46, royal red		1
38	1506769-5969	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST46, royal blue		1
38	1506769-9005	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST46, black, glossy		1
38	1506769-9119	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST46, black, bronze		1
39	1506770-0419	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST48, silver grey		1
39	1506770-0442	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST48, night shimmer	2009-01-01 →	1
39	1506770-3524	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST48, royal red		1
39	1506770-5969	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST48, royal blue		1
39	1506770-9005	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST48, black, glossy		1
39	1506770-9119	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST48, black, bronze		1
40	1506771-0419	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST50, silver grey		1
40	1506771-0442	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST50, night shimmer	2009-01-01 →	1
40	1506771-3524	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST50, royal red		1
40	1506771-5969	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST50, royal blue		1
40	1506771-9005	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST50, black, glossy		1
40	1506771-9119	Cross-brace double Hemi abduct., UltraLight 80° SB48 ST50, black, bronze		1
41	1506772-0419	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST32, silver grey		1
41	1506772-0442	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST32, night shimmer	2009-01-01 →	1
41	1506772-3524	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST32, royal red		1
41	1506772-5969	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST32, royal blue		1
41	1506772-9005	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST32, black, glossy		1
41	1506772-9119	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST32, black, bronze		1
42	1506773-0419	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST34, silver grey		1
42	1506773-0442	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST34, night shimmer	2009-01-01 →	1
42	1506773-3524	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST34, royal red		1
42	1506773-5969	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST34, royal blue		1
42	1506773-9005	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST34, black, glossy		1
42	1506773-9119	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST34, black, bronze		1
43	1506774-0419	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST36, silver grey		1
43	1506774-0442	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST36, night shimmer	2009-01-01 →	1
43	1506774-3524	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST36, royal red		1
43	1506774-5969	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST36, royal blue		1
43	1506774-9005	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST36, black, glossy		1
43	1506774-9119	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST36, black, bronze		1
44	1506775-0419	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST38, silver grey		1
44	1506775-0442	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST38, night shimmer	2009-01-01 →	1
44	1506775-3524	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST38, royal red		1
44	1506775-5969	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST38, royal blue		1
44	1506775-9005	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST38, black, glossy		1




**Cross brace to frame Hemi abduction, SB42-50**

Pos.	Part number	Description	Valid from → until	Qty
44	1506775-9119	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST38, black, bronze		1
45	1506776-0419	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST40, silver grey		1
45	1506776-0442	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST40, night shimmer	2009-01-01 →	1
45	1506776-3524	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST40, royal red		1
45	1506776-5969	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST40, royal blue		1
45	1506776-9005	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST40, black, glossy		1
45	1506776-9119	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST40, black, bronze		1
46	1506777-0419	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST42, silver grey		1
46	1506777-0442	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST42, night shimmer	2009-01-01 →	1
46	1506777-3524	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST42, royal red		1
46	1506777-5969	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST42, royal blue		1
46	1506777-9005	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST42, black, glossy		1
46	1506777-9119	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST42, black, bronze		1
47	1506778-0419	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST44, silver grey		1
47	1506778-0442	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST44, night shimmer	2009-01-01 →	1
47	1506778-3524	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST44, royal red		1
47	1506778-5969	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST44, royal blue		1
47	1506778-9005	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST44, black, glossy		1
47	1506778-9119	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST44, black, bronze		1
48	1506779-0419	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST46, silver grey		1
48	1506779-0442	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST46, night shimmer	2009-01-01 →	1
48	1506779-3524	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST46, royal red		1
48	1506779-5969	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST46, royal blue		1
48	1506779-9005	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST46, black, glossy		1
48	1506779-9119	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST46, black, bronze		1
49	1506780-0419	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST48, silver grey		1
49	1506780-0442	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST48, night shimmer	2009-01-01 →	1
49	1506780-3524	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST48, royal red		1
49	1506780-5969	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST48, royal blue		1
49	1506780-9005	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST48, black, glossy		1
49	1506780-9119	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST48, black, bronze		1
50	1506781-0419	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST50, silver grey		1
50	1506781-0442	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST50, night shimmer	2009-01-01 →	1
50	1506781-3524	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST50, royal red		1
50	1506781-5969	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST50, royal blue		1
50	1506781-9005	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST50, black, glossy		1
50	1506781-9119	Cross-brace double Hemi abduct., UltraLight 80° SB50 ST50, black, bronze		1
51	1437944	Retaining lever SB40-44		1
52	1437945	Retaining lever SB46-50		1
53	1507625	Center division kit		1
54	1439480	Screw kit to Double crossbrace		1




## Rear frames

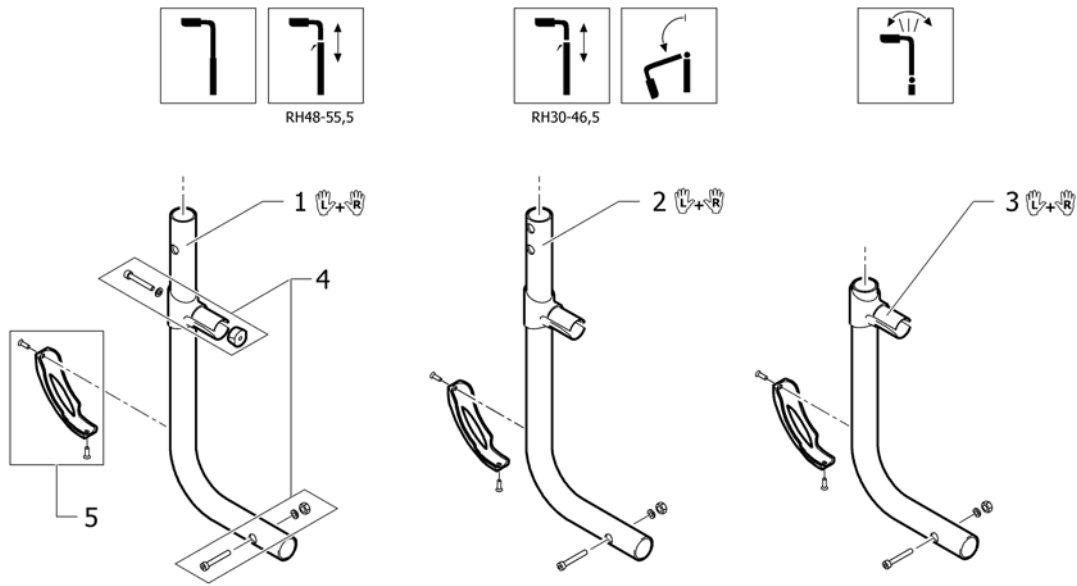


Pos.	Part number	Description	Valid from → until	Qty
1	1437840-0419	Rear frame, silver grey		1
1	1437840-0442	Rear frame, night shimmer	2009-01-01 →	1
1	1437840-3524	Rear frame, royal red		1
1	1437840-5969	Rear frame, royal blue		1
1	1437840-9005	Rear frame, black, glossy		1
1	1437840-9119	Rear frame, black, bronze		1
2	1488457-0419	Rear frame to foldable back silver grey		1
2	1488457-0442	Rear frame to foldable back night shimmer	2009-01-01 →	1
2	1488457-3524	Rear frame to foldable back royal Red		1
2	1488457-5969	Rear frame to foldable back royal Blue		1
2	1488457-9005	Rear frame to foldable back Black Glossy		1
2	1488457-9119	Rear frame to foldable back Black Bronze		1
3	1488458-0419	Rear frame to angle adjustable back Silver grey		1
3	1488458-0442	Rear frame to angle adjustable back night shimmer	2009-01-01 →	1
3	1488458-3524	Rear frame to angle adjustable back royal Red		1
3	1488458-5969	Rear frame to angle adjustable back royal Blue		1
3	1488458-9005	Rear frame to angle adjustable back Black Glossy		1
3	1488458-9119	Rear frame to angle adjustable back Black Bronze		1
4	1437842	frame fixation set		1
5	1498774	Rear frame protection, pair		1

## Rear wheel fixation

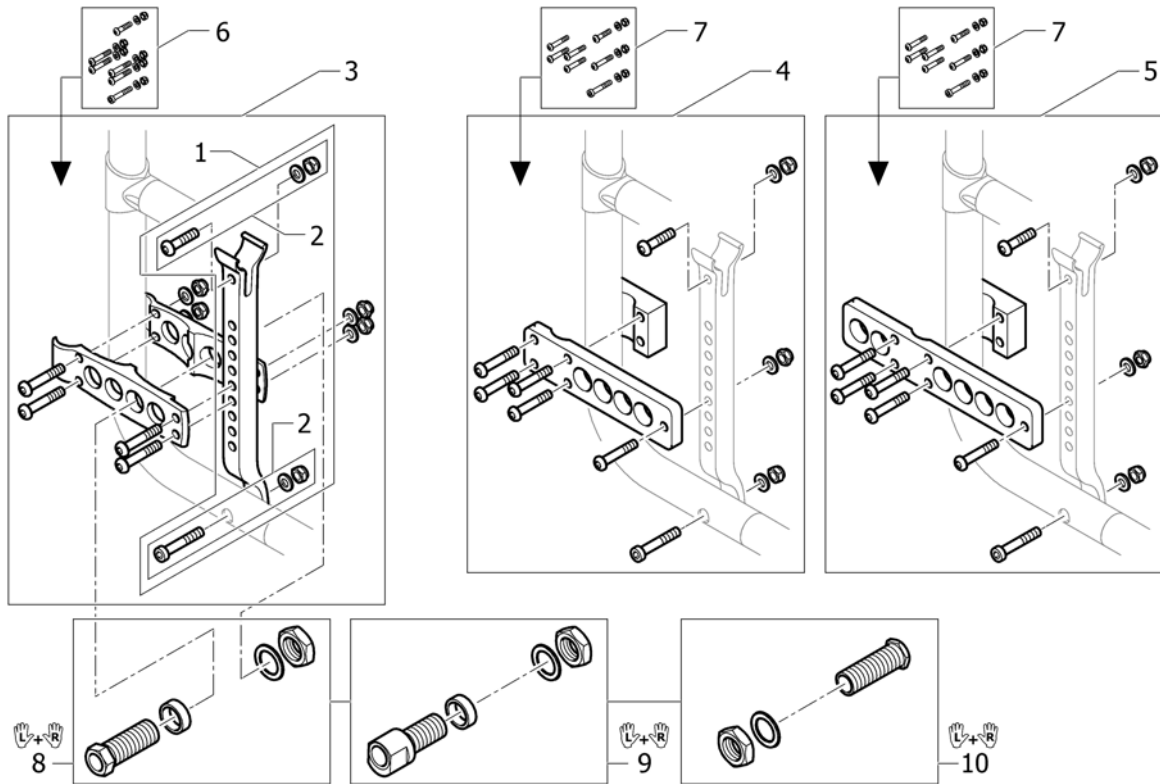


Pos.	Part number	Description	Valid from → until	Qty
1	1437977-0004	Adapterplate support kit, black anodized		2
1	1437977-0400	Adapterplate support kit, techno-anthrazit		2
1	1437977-0401	Adapterplate support kit, lemon		2
1	1437977-0402	Adapterplate support kit, apricot		2
1	1437977-0404	Adapterplate support kit, b-blue		2
1	1437977-0440	Adapterplate support kit, comet red	2007-01-01 →	2
2	1437978	fixation kit to adapter plate mounting		1
3	1445275-0004	adapterplate set black		2
3	1445275-0400	adapterplate set Tecno		2
3	1445275-0401	adapterplate set Lemon		2
3	1445275-0402	adapterplate set Apricot		2
3	1445275-0404	adapterplate set B-Blue		2
3	1445275-0440	adapterplate set comet red	2007-01-01 →	2
4	1437980-0004	Adapterplate support kit at drumbrake and 4° with siderest		2
5	1439466	Adapterplate set for Rearwhelextension		2
6	1437981	fixation kit for adapter plate supporter		1
7	1439481	fixation kit to adapter plate, drum brake		1
8	1437982	Adaptersleeve set 1°, pair		1
9	1437983	Adaptersleeve set 4°, pair		1
10	1527082	Adaptersleeve set 1° for siderest with desc function, pair	2008-12-08 →	1

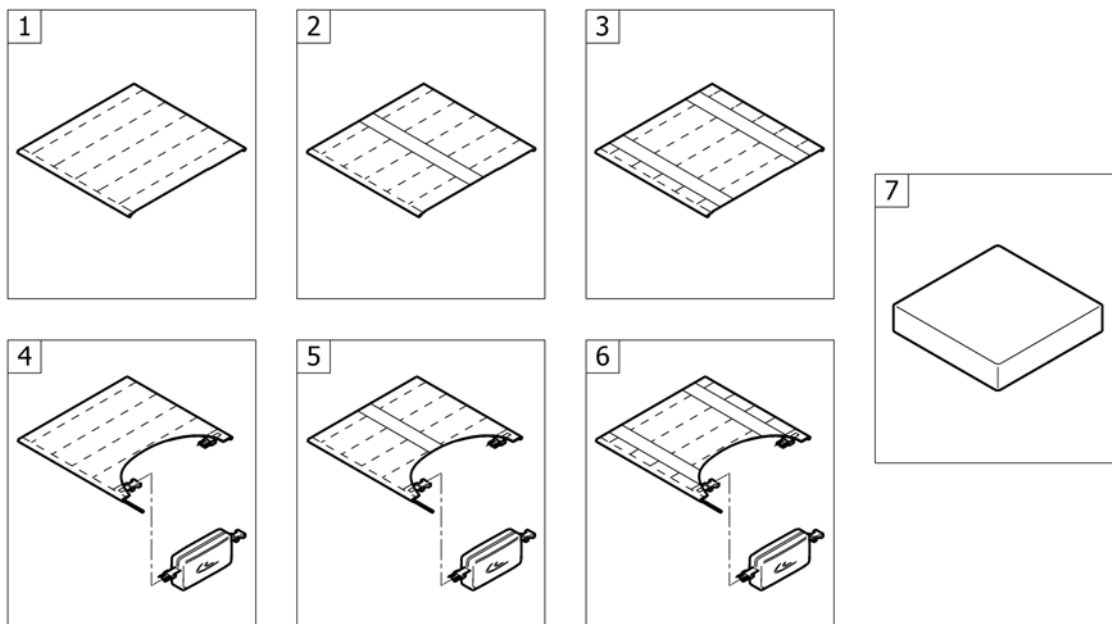


Pos.	Part number	Description	Valid from → until	Qty
1	-	Seat cover standard	2004-01-01 → 2012-12-31	1
2	-	Seat cover with 1 Velcro band (soft band)	2004-01-01 → 2012-12-31	1
3	-	Seat cover with 2 Velcro bands (rough band)	2004-01-01 → 2012-12-31	1
4	-	Seat cover standard for seat bag	2008-07-01 → 2012-12-31	1
5	-	Seat cover with 1 Velcro band (soft band) for seat bag	2008-07-01 → 2012-12-31	1
6	-	Seat cover with 2 Velcro bands (rough band) for seat bag	2008-07-01 → 2012-12-31	1
7	-	Seat cushion standard	2012-07-01 → 2012-12-31	1

# Seat cover standard

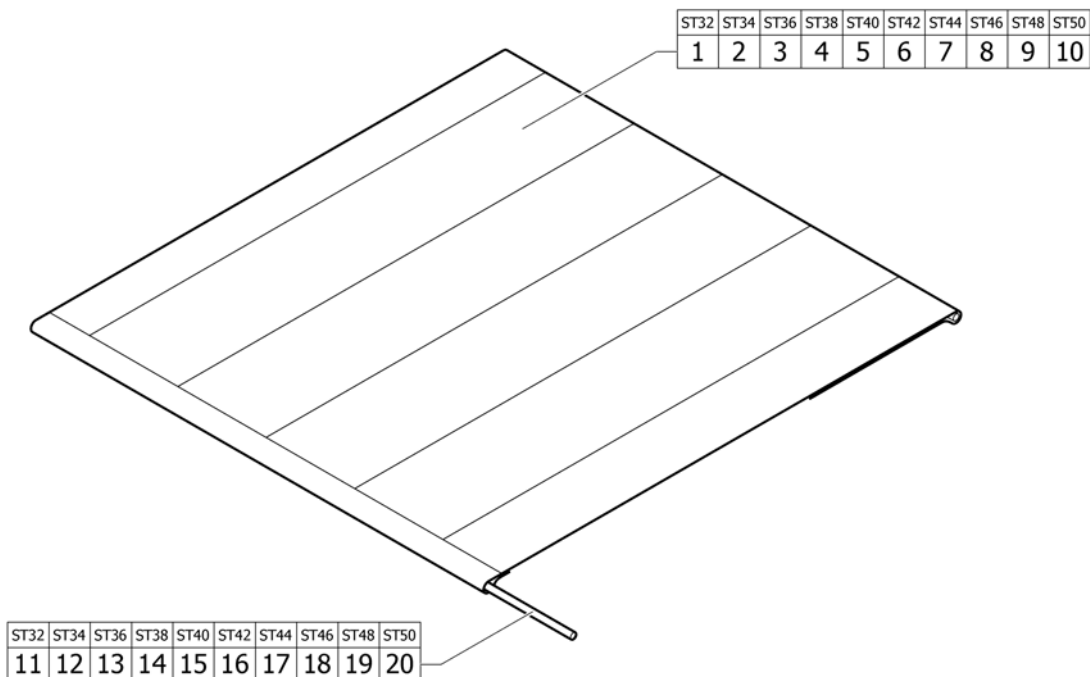

Pos.	Part number	Description	Valid from → until	Qty
1	1437946	Seat cover ST32 SB variable 32-50		1
2	1437947	Seat cover ST34 SB variable 32-50		1
3	1437948	Seat cover ST36 SB variable 32-50		1
4	1437949	Seat cover ST38 SB variable 32-50		1
5	1437950	Seat cover ST40 SB variable 32-50		1
6	1437951	Seat cover ST42 SB variable 32-50		1
7	1437952	Seat cover ST44 SB variable 32-50		1
8	1437953	Seat cover ST46 SB variable 32-50		1
9	1437954	Seat cover ST48 SB variable 32-50		1
10	1437955	Seat cover ST50 SB variable 32-50		1
11	1437966	Seat cover bar kit ST32		1
12	1437967	Seat cover bar kit ST34		1
13	1437968	Seat cover bar kit ST36		1
14	1437969	Seat cover bar kit ST38		1
15	1437970	Seat cover bar kit ST40		1
16	1437971	Seat cover bar kit ST42		1
17	1437972	Seat cover bar kit ST44		1
18	1437973	Seat cover bar kit ST46		1
19	1437974	Seat cover bar kit ST48		1
20	1437975	Seat cover bar kit ST50		1

## Seat cover with 1 Velcro band (soft band)

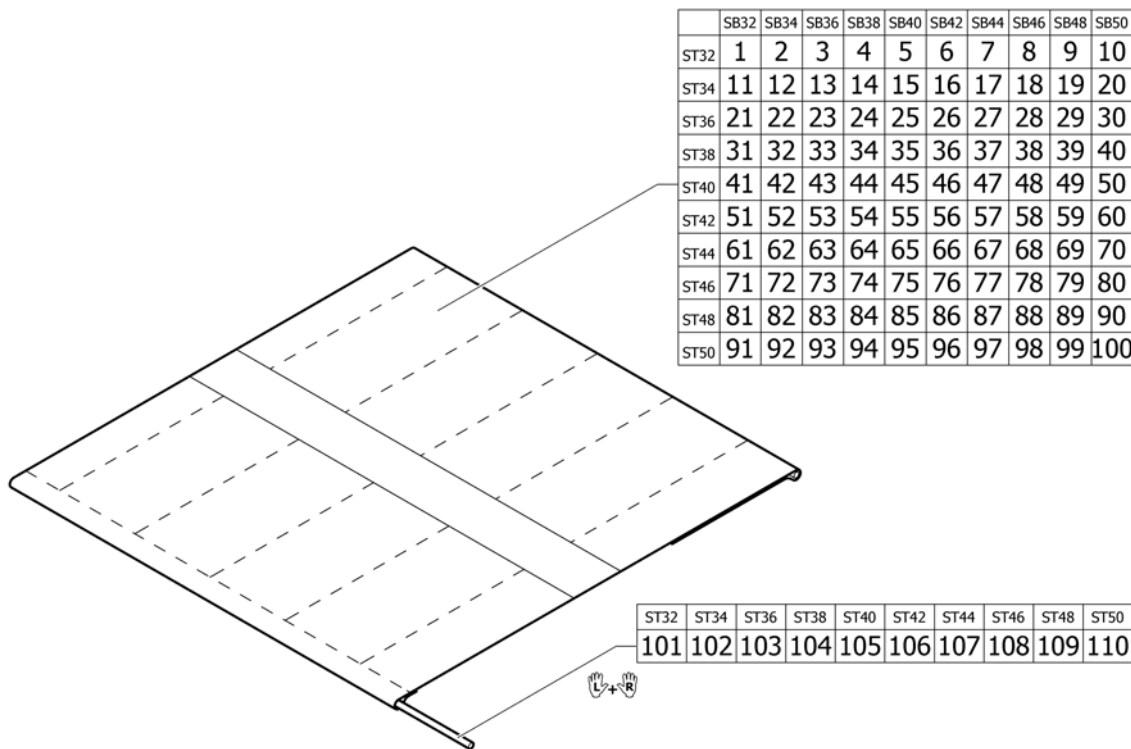

Pos.	Part number	Description	Valid from → until	Qty
1	1509138	Seat cover SB32 ST32 with 1 Velcro band (soft band)		1
2	1509139	Seat cover SB34 ST32 with 1 Velcro band (soft band)		1
3	1509140	Seat cover SB36 ST32 with 1 Velcro band (soft band)		1
4	1509141	Seat cover SB38 ST32 with 1 Velcro band (soft band)		1
5	1509142	Seat cover SB40 ST32 with 1 Velcro band (soft band)		1
6	1509143	Seat cover SB42 ST32 with 1 Velcro band (soft band)		1
7	1509144	Seat cover SB44 ST32 with 1 Velcro band (soft band)		1
8	1509145	Seat cover SB46 ST32 with 1 Velcro band (soft band)		1
9	1509146	Seat cover SB48 ST32 with 1 Velcro band (soft band)		1
10	1509147	Seat cover SB50 ST32 with 1 Velcro band (soft band)		1
11	1509148	Seat cover SB32 ST34 with 1 Velcro band (soft band)		1
12	1509149	Seat cover SB34 ST34 with 1 Velcro band (soft band)		1
13	1509150	Seat cover SB36 ST34 with 1 Velcro band (soft band)		1
14	1509152	Seat cover SB38 ST34 with 1 Velcro band (soft band)		1
15	1509153	Seat cover SB40 ST34 with 1 Velcro band (soft band)		1
16	1509154	Seat cover SB42 ST34 with 1 Velcro band (soft band)		1
17	1509155	Seat cover SB44 ST34 with 1 Velcro band (soft band)		1
18	1509156	Seat cover SB46 ST34 with 1 Velcro band (soft band)		1
19	1509157	Seat cover SB48 ST34 with 1 Velcro band (soft band)		1
20	1509158	Seat cover SB50 ST34 with 1 Velcro band (soft band)		1
21	1509160	Seat cover SB32 ST36 with 1 Velcro band (soft band)		1
22	1509161	Seat cover SB34 ST36 with 1 Velcro band (soft band)		1
23	1509162	Seat cover SB36 ST36 with 1 Velcro band (soft band)		1
24	1509163	Seat cover SB38 ST36 with 1 Velcro band (soft band)		1
25	1509164	Seat cover SB40 ST36 with 1 Velcro band (soft band)		1
26	1509165	Seat cover SB42 ST36 with 1 Velcro band (soft band)		1
27	1509166	Seat cover SB44 ST36 with 1 Velcro band (soft band)		1
28	1509167	Seat cover SB46 ST36 with 1 Velcro band (soft band)		1
29	1509168	Seat cover SB48 ST36 with 1 Velcro band (soft band)		1
30	1509169	Seat cover SB50 ST36 with 1 Velcro band (soft band)		1
31	1509170	Seat cover SB32 ST38 with 1 Velcro band (soft band)		1




**Seat cover with 1 Velcro band (soft band)**

Pos.	Part number	Description	Valid from → until	Qty
32	1509171	Seat cover SB34 ST38 with 1 Velcro band (soft band)		1
33	1509172	Seat cover SB36 ST38 with 1 Velcro band (soft band)		1
34	1509173	Seat cover SB38 ST38 with 1 Velcro band (soft band)		1
35	1509174	Seat cover SB40 ST38 with 1 Velcro band (soft band)		1
36	1509175	Seat cover SB42 ST38 with 1 Velcro band (soft band)		1
37	1509176	Seat cover SB44 ST38 with 1 Velcro band (soft band)		1
38	1509177	Seat cover SB46 ST38 with 1 Velcro band (soft band)		1
39	1509178	Seat cover SB48 ST38 with 1 Velcro band (soft band)		1
40	1509179	Seat cover SB50 ST38 with 1 Velcro band (soft band)		1
41	1509181	Seat cover SB32 ST40 with 1 Velcro band (soft band)		1
42	1509182	Seat cover SB34 ST40 with 1 Velcro band (soft band)		1
43	1509183	Seat cover SB36 ST40 with 1 Velcro band (soft band)		1
44	1509184	Seat cover SB38 ST40 with 1 Velcro band (soft band)		1
45	1509185	Seat cover SB40 ST40 with 1 Velcro band (soft band)		1
46	1509186	Seat cover SB42 ST40 with 1 Velcro band (soft band)		1
47	1509187	Seat cover SB44 ST40 with 1 Velcro band (soft band)		1
48	1509188	Seat cover SB46 ST40 with 1 Velcro band (soft band)		1
49	1509189	Seat cover SB48 ST40 with 1 Velcro band (soft band)		1
50	1509190	Seat cover SB50 ST40 with 1 Velcro band (soft band)		1
51	1509191	Seat cover SB32 ST42 with 1 Velcro band (soft band)		1
52	1509192	Seat cover SB34 ST42 with 1 Velcro band (soft band)		1
53	1509193	Seat cover SB36 ST42 with 1 Velcro band (soft band)		1
54	1509194	Seat cover SB38 ST42 with 1 Velcro band (soft band)		1
55	1509195	Seat cover SB40 ST42 with 1 Velcro band (soft band)		1
56	1509196	Seat cover SB42 ST42 with 1 Velcro band (soft band)		1
57	1509197	Seat cover SB44 ST42 with 1 Velcro band (soft band)		1
58	1509198	Seat cover SB46 ST42 with 1 Velcro band (soft band)		1
59	1509199	Seat cover SB48 ST42 with 1 Velcro band (soft band)		1
60	1509200	Seat cover SB50 ST42 with 1 Velcro band (soft band)		1
61	1509201	Seat cover SB32 ST44 with 1 Velcro band (soft band)		1
62	1509202	Seat cover SB34 ST44 with 1 Velcro band (soft band)		1
63	1509203	Seat cover SB36 ST44 with 1 Velcro band (soft band)		1
64	1509204	Seat cover SB38 ST44 with 1 Velcro band (soft band)		1
65	1509205	Seat cover SB40 ST44 with 1 Velcro band (soft band)		1
66	1509206	Seat cover SB42 ST44 with 1 Velcro band (soft band)		1
67	1509207	Seat cover SB44 ST44 with 1 Velcro band (soft band)		1
68	1509208	Seat cover SB46 ST44 with 1 Velcro band (soft band)		1
69	1509209	Seat cover SB48 ST44 with 1 Velcro band (soft band)		1
70	1509211	Seat cover SB50 ST44 with 1 Velcro band (soft band)		1
71	1509212	Seat cover SB32 ST46 with 1 Velcro band (soft band)		1
72	1509213	Seat cover SB34 ST46 with 1 Velcro band (soft band)		1
73	1509214	Seat cover SB36 ST46 with 1 Velcro band (soft band)		1
74	1509215	Seat cover SB38 ST46 with 1 Velcro band (soft band)		1
75	1509216	Seat cover SB40 ST46 with 1 Velcro band (soft band)		1
76	1509217	Seat cover SB42 ST46 with 1 Velcro band (soft band)		1
77	1509218	Seat cover SB44 ST46 with 1 Velcro band (soft band)		1
78	1509219	Seat cover SB46 ST46 with 1 Velcro band (soft band)		1
79	1509220	Seat cover SB48 ST46 with 1 Velcro band (soft band)		1
80	1509221	Seat cover SB50 ST46 with 1 Velcro band (soft band)		1
81	1509222	Seat cover SB32 ST48 with 1 Velcro band (soft band)		1
82	1509223	Seat cover SB34 ST48 with 1 Velcro band (soft band)		1
83	1509224	Seat cover SB36 ST48 with 1 Velcro band (soft band)		1
84	1509225	Seat cover SB38 ST48 with 1 Velcro band (soft band)		1
85	1509226	Seat cover SB40 ST48 with 1 Velcro band (soft band)		1
86	1509227	Seat cover SB42 ST48 with 1 Velcro band (soft band)		1

**Seat cover with 1 Velcro band (soft band)**

Pos.	Part number	Description	Valid from → until	Qty
87	1509228	Seat cover SB44 ST48 with 1 Velcro band (soft band)		1
88	1509229	Seat cover SB46 ST48 with 1 Velcro band (soft band)		1
89	1509230	Seat cover SB48 ST48 with 1 Velcro band (soft band)		1
90	1509231	Seat cover SB50 ST48 with 1 Velcro band (soft band)		1
91	1509232	Seat cover SB32 ST50 with 1 Velcro band (soft band)		1
92	1509233	Seat cover SB34 ST50 with 1 Velcro band (soft band)		1
93	1509234	Seat cover SB36 ST50 with 1 Velcro band (soft band)		1
94	1509235	Seat cover SB38 ST50 with 1 Velcro band (soft band)		1
95	1509236	Seat cover SB40 ST50 with 1 Velcro band (soft band)		1
96	1509237	Seat cover SB42 ST50 with 1 Velcro band (soft band)		1
97	1509238	Seat cover SB44 ST50 with 1 Velcro band (soft band)		1
98	1509239	Seat cover SB46 ST50 with 1 Velcro band (soft band)		1
99	1509240	Seat cover SB48 ST50 with 1 Velcro band (soft band)		1
100	1509241	Seat cover SB50 ST50 with 1 Velcro band (soft band)		1
101	1437966	Seat cover bar kit ST32		1
102	1437967	Seat cover bar kit ST34		1
103	1437968	Seat cover bar kit ST36		1
104	1437969	Seat cover bar kit ST38		1
105	1437970	Seat cover bar kit ST40		1
106	1437971	Seat cover bar kit ST42		1
107	1437972	Seat cover bar kit ST44		1
108	1437973	Seat cover bar kit ST46		1
109	1437974	Seat cover bar kit ST48		1
110	1437975	Seat cover bar kit ST50		1

## Seat cover with 2 Velcro bands (rough band)

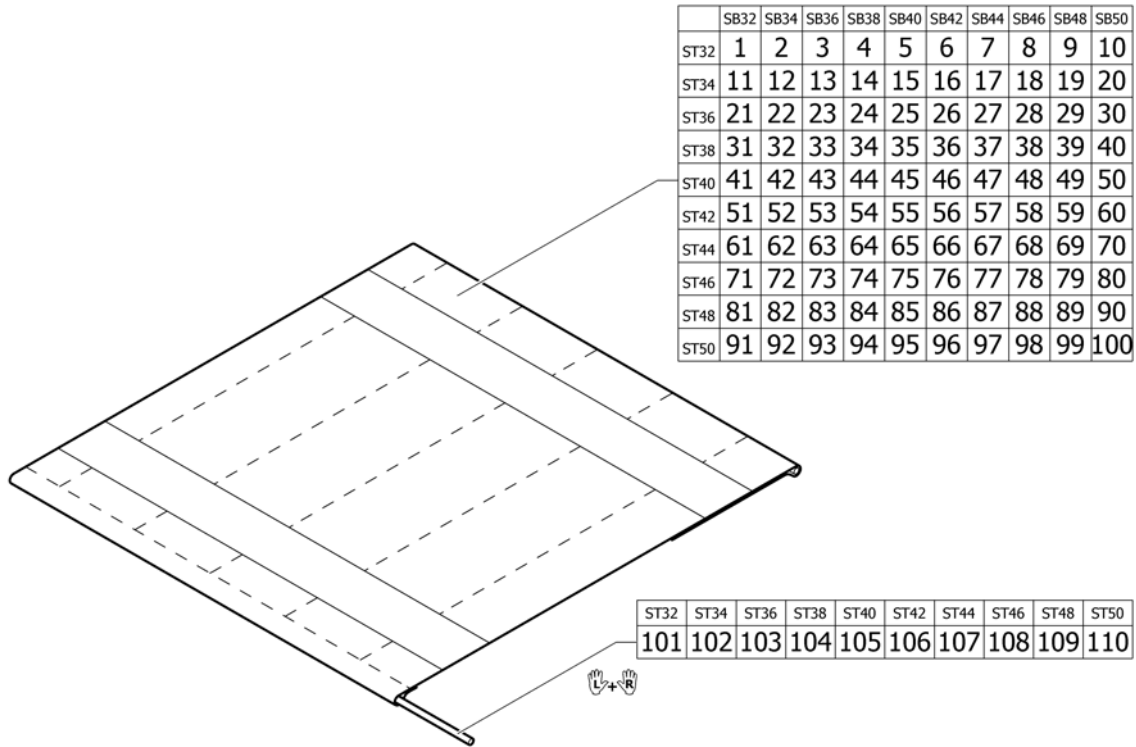

Pos.	Part number	Description	Valid from → until	Qty
1	1509242	Seat cover SB32 ST32 with 2 Velcro bands (rough band)		1
2	1509243	Seat cover SB34 ST32 with 2 Velcro bands (rough band)		1
3	1509244	Seat cover SB36 ST32 with 2 Velcro bands (rough band)		1
4	1509245	Seat cover SB38 ST32 with 2 Velcro bands (rough band)		1
5	1509246	Seat cover SB40 ST32 with 2 Velcro bands (rough band)		1
6	1509247	Seat cover SB42 ST32 with 2 Velcro bands (rough band)		1
7	1509248	Seat cover SB44 ST32 with 2 Velcro bands (rough band)		1
8	1509249	Seat cover SB46 ST32 with 2 Velcro bands (rough band)		1
9	1509250	Seat cover SB48 ST32 with 2 Velcro bands (rough band)		1
10	1509251	Seat cover SB50 ST32 with 2 Velcro bands (rough band)		1
11	1509252	Seat cover SB32 ST34 with 2 Velcro bands (rough band)		1
12	1509253	Seat cover SB34 ST34 with 2 Velcro bands (rough band)		1
13	1509254	Seat cover SB36 ST34 with 2 Velcro bands (rough band)		1
14	1509255	Seat cover SB38 ST34 with 2 Velcro bands (rough band)		1
15	1509256	Seat cover SB40 ST34 with 2 Velcro bands (rough band)		1
16	1509257	Seat cover SB42 ST34 with 2 Velcro bands (rough band)		1
17	1509258	Seat cover SB44 ST34 with 2 Velcro bands (rough band)		1
18	1509259	Seat cover SB46 ST34 with 2 Velcro bands (rough band)		1
19	1509260	Seat cover SB48 ST34 with 2 Velcro bands (rough band)		1
20	1509261	Seat cover SB50 ST34 with 2 Velcro bands (rough band)		1
21	1509262	Seat cover SB32 ST36 with 2 Velcro bands (rough band)		1
22	1509263	Seat cover SB34 ST36 with 2 Velcro bands (rough band)		1
23	1509264	Seat cover SB36 ST36 with 2 Velcro bands (rough band)		1
24	1509265	Seat cover SB38 ST36 with 2 Velcro bands (rough band)		1
25	1509266	Seat cover SB40 ST36 with 2 Velcro bands (rough band)		1
26	1509267	Seat cover SB42 ST36 with 2 Velcro bands (rough band)		1
27	1509268	Seat cover SB44 ST36 with 2 Velcro bands (rough band)		1
28	1509269	Seat cover SB46 ST36 with 2 Velcro bands (rough band)		1
29	1509270	Seat cover SB48 ST36 with 2 Velcro bands (rough band)		1
30	1509271	Seat cover SB50 ST36 with 2 Velcro bands (rough band)		1
31	1509272	Seat cover SB32 ST38 with 2 Velcro bands (rough band)		1

**Seat cover with 2 Velcro bands (rough band)**


Pos.	Part number	Description	Valid from → until	Qty
32	1509273	Seat cover SB34 ST38 with 2 Velcro bands (rough band)		1
33	1509274	Seat cover SB36 ST38 with 2 Velcro bands (rough band)		1
34	1509275	Seat cover SB38 ST38 with 2 Velcro bands (rough band)		1
35	1509276	Seat cover SB40 ST38 with 2 Velcro bands (rough band)		1
36	1509277	Seat cover SB42 ST38 with 2 Velcro bands (rough band)		1
37	1509278	Seat cover SB44 ST38 with 2 Velcro bands (rough band)		1
38	1509279	Seat cover SB46 ST38 with 2 Velcro bands (rough band)		1
39	1509280	Seat cover SB48 ST38 with 2 Velcro bands (rough band)		1
40	1509281	Seat cover SB50 ST38 with 2 Velcro bands (rough band)		1
41	1509282	Seat cover SB32 ST40 with 2 Velcro bands (rough band)		1
42	1509283	Seat cover SB34 ST40 with 2 Velcro bands (rough band)		1
43	1509284	Seat cover SB36 ST40 with 2 Velcro bands (rough band)		1
44	1509285	Seat cover SB38 ST40 with 2 Velcro bands (rough band)		1
45	1509286	Seat cover SB40 ST40 with 2 Velcro bands (rough band)		1
46	1509287	Seat cover SB42 ST40 with 2 Velcro bands (rough band)		1
47	1509288	Seat cover SB44 ST40 with 2 Velcro bands (rough band)		1
48	1509289	Seat cover SB46 ST40 with 2 Velcro bands (rough band)		1
49	1509290	Seat cover SB48 ST40 with 2 Velcro bands (rough band)		1
50	1509291	Seat cover SB50 ST40 with 2 Velcro bands (rough band)		1
51	1509292	Seat cover SB32 ST42 with 2 Velcro bands (rough band)		1
52	1509293	Seat cover SB34 ST42 with 2 Velcro bands (rough band)		1
53	1509294	Seat cover SB36 ST42 with 2 Velcro bands (rough band)		1
54	1509295	Seat cover SB38 ST42 with 2 Velcro bands (rough band)		1
55	1509296	Seat cover SB40 ST42 with 2 Velcro bands (rough band)		1
56	1509297	Seat cover SB42 ST42 with 2 Velcro bands (rough band)		1
57	1509298	Seat cover SB44 ST42 with 2 Velcro bands (rough band)		1
58	1509299	Seat cover SB46 ST42 with 2 Velcro bands (rough band)		1
59	1509300	Seat cover SB48 ST42 with 2 Velcro bands (rough band)		1
60	1509301	Seat cover SB50 ST42 with 2 Velcro bands (rough band)		1
61	1509302	Seat cover SB32 ST44 with 2 Velcro bands (rough band)		1
62	1509303	Seat cover SB34 ST44 with 2 Velcro bands (rough band)		1
63	1509304	Seat cover SB36 ST44 with 2 Velcro bands (rough band)		1
64	1509305	Seat cover SB38 ST44 with 2 Velcro bands (rough band)		1
65	1509306	Seat cover SB40 ST44 with 2 Velcro bands (rough band)		1
66	1509319	Seat cover SB42 ST44 with 2 Velcro bands (rough band)		1
67	1509320	Seat cover SB44 ST44 with 2 Velcro bands (rough band)		1
68	1509321	Seat cover SB46 ST44 with 2 Velcro bands (rough band)		1
69	1509431	Seat cover SB48 ST44 with 2 Velcro bands (rough band)		1
70	1509432	Seat cover SB50 ST44 with 2 Velcro bands (rough band)		1
71	1509433	Seat cover SB32 ST46 with 2 Velcro bands (rough band)		1
72	1509526	Seat cover SB34 ST46 with 2 Velcro bands (rough band)		1
73	1509527	Seat cover SB36 ST46 with 2 Velcro bands (rough band)		1
74	1509528	Seat cover SB38 ST46 with 2 Velcro bands (rough band)		1
75	1509529	Seat cover SB40 ST46 with 2 Velcro bands (rough band)		1
76	1509530	Seat cover SB42 ST46 with 2 Velcro bands (rough band)		1
77	1509531	Seat cover SB44 ST46 with 2 Velcro bands (rough band)		1
78	1509532	Seat cover SB46 ST46 with 2 Velcro bands (rough band)		1
79	1509533	Seat cover SB48 ST46 with 2 Velcro bands (rough band)		1
80	1509534	Seat cover SB50 ST46 with 2 Velcro bands (rough band)		1
81	1509535	Seat cover SB32 ST48 with 2 Velcro bands (rough band)		1
82	1509536	Seat cover SB34 ST48 with 2 Velcro bands (rough band)		1
83	1509537	Seat cover SB36 ST48 with 2 Velcro bands (rough band)		1
84	1509538	Seat cover SB38 ST48 with 2 Velcro bands (rough band)		1
85	1509539	Seat cover SB40 ST48 with 2 Velcro bands (rough band)		1
86	1509540	Seat cover SB42 ST48 with 2 Velcro bands (rough band)		1

**Seat cover with 2 Velcro bands (rough band)**

Pos.	Part number	Description	Valid from → until	Qty
87	1509541	Seat cover SB44 ST48 with 2 Velcro bands (rough band)		1
88	1509542	Seat cover SB46 ST48 with 2 Velcro bands (rough band)		1
89	1509543	Seat cover SB48 ST48 with 2 Velcro bands (rough band)		1
90	1509544	Seat cover SB50 ST48 with 2 Velcro bands (rough band)		1
91	1509545	Seat cover SB32 ST50 with 2 Velcro bands (rough band)		1
92	1509546	Seat cover SB34 ST50 with 2 Velcro bands (rough band)		1
93	1509547	Seat cover SB36 ST50 with 2 Velcro bands (rough band)		1
94	1509548	Seat cover SB38 ST50 with 2 Velcro bands (rough band)		1
95	1509549	Seat cover SB40 ST50 with 2 Velcro bands (rough band)		1
96	1509550	Seat cover SB42 ST50 with 2 Velcro bands (rough band)		1
97	1509551	Seat cover SB44 ST50 with 2 Velcro bands (rough band)		1
98	1509552	Seat cover SB46 ST50 with 2 Velcro bands (rough band)		1
99	1509553	Seat cover SB48 ST50 with 2 Velcro bands (rough band)		1
100	1509554	Seat cover SB50 ST50 with 2 Velcro bands (rough band)		1
101	1437966	Seat cover bar kit ST32		1
102	1437967	Seat cover bar kit ST34		1
103	1437968	Seat cover bar kit ST36		1
104	1437969	Seat cover bar kit ST38		1
105	1437970	Seat cover bar kit ST40		1
106	1437971	Seat cover bar kit ST42		1
107	1437972	Seat cover bar kit ST44		1
108	1437973	Seat cover bar kit ST46		1
109	1437974	Seat cover bar kit ST48		1
110	1437975	Seat cover bar kit ST50		1



## Seat cover standard for seat bag

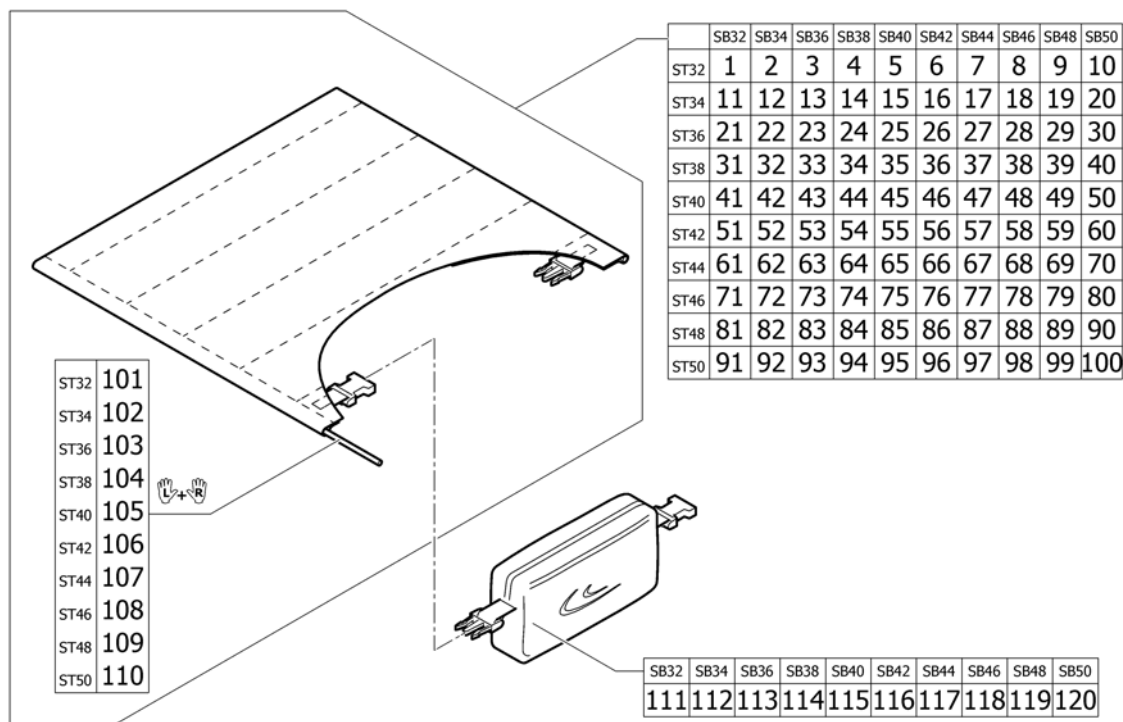

Pos.	Part number	Description	Valid from → until	Qty
1	1571358	Seat cover SB32 ST32 for seat bag		1
2	1571359	Seat cover SB34 ST32 for seat bag		1
3	1571360	Seat cover SB36 ST32 for seat bag		1
4	1571361	Seat cover SB38 ST32 for seat bag		1
5	1571362	Seat cover SB40 ST32 for seat bag		1
6	1571363	Seat cover SB42 ST32 for seat bag		1
7	1571364	Seat cover SB44 ST32 for seat bag		1
8	1571365	Seat cover SB46 ST32 for seat bag		1
9	1571366	Seat cover SB48 ST32 for seat bag		1
10	1571367	Seat cover SB50 ST32 for seat bag		1
11	1571368	Seat cover SB32 ST34 for seat bag		1
12	1571369	Seat cover SB34 ST34 for seat bag		1
13	1571370	Seat cover SB36 ST34 for seat bag		1
14	1571371	Seat cover SB38 ST34 for seat bag		1
15	1571372	Seat cover SB40 ST34 for seat bag		1
16	1571373	Seat cover SB42 ST34 for seat bag		1
17	1571374	Seat cover SB44 ST34 for seat bag		1
18	1571375	Seat cover SB46 ST34 for seat bag		1
19	1571376	Seat cover SB48 ST34 for seat bag		1
20	1571377	Seat cover SB50 ST34 for seat bag		1
21	1571378	Seat cover SB32 ST36 for seat bag		1
22	1571379	Seat cover SB34 ST36 for seat bag		1
23	1571380	Seat cover SB36 ST36 for seat bag		1
24	1571381	Seat cover SB38 ST36 for seat bag		1
25	1571382	Seat cover SB40 ST36 for seat bag		1
26	1571383	Seat cover SB42 ST36 for seat bag		1
27	1571384	Seat cover SB44 ST36 for seat bag		1
28	1571385	Seat cover SB46 ST36 for seat bag		1
29	1571386	Seat cover SB48 ST36 for seat bag		1
30	1571387	Seat cover SB50 ST36 for seat bag		1
31	1571388	Seat cover SB32 ST38 for seat bag		1


**Seat cover standard for seat bag**

Pos.	Part number	Description	Valid from → until	Qty
32	1571389	Seat cover SB34 ST38 for seat bag		1
33	1571390	Seat cover SB36 ST38 for seat bag		1
34	1571391	Seat cover SB38 ST38 for seat bag		1
35	1571411	Seat cover SB40 ST38 for seat bag		1
36	1571412	Seat cover SB42 ST38 for seat bag		1
37	1571413	Seat cover SB44 ST38 for seat bag		1
38	1571509	Seat cover SB46 ST38 for seat bag		1
39	1571581	Seat cover SB48 ST38 for seat bag		1
40	1571612	Seat cover SB50 ST38 for seat bag		1
41	1571613	Seat cover SB32 ST40 for seat bag		1
42	1571636	Seat cover SB34 ST40 for seat bag		1
43	1571637	Seat cover SB36 ST40 for seat bag		1
44	1571638	Seat cover SB38 ST40 for seat bag		1
45	1571729	Seat cover SB40 ST40 for seat bag		1
46	1571730	Seat cover SB42 ST40 for seat bag		1
47	1571731	Seat cover SB44 ST40 for seat bag		1
48	1571732	Seat cover SB46 ST40 for seat bag		1
49	1571759	Seat cover SB48 ST40 for seat bag		1
50	1571760	Seat cover SB50 ST40 for seat bag		1
51	1571765	Seat cover SB32 ST42 for seat bag		1
52	1571766	Seat cover SB34 ST42 for seat bag		1
53	1571767	Seat cover SB36 ST42 for seat bag		1
54	1572002	Seat cover SB38 ST42 for seat bag		1
55	1572003	Seat cover SB40 ST42 for seat bag		1
56	1572082	Seat cover SB42 ST42 for seat bag		1
57	1572083	Seat cover SB44 ST42 for seat bag		1
58	1572084	Seat cover SB46 ST42 for seat bag		1
59	1572085	Seat cover SB48 ST42 for seat bag		1
60	1572162	Seat cover SB50 ST42 for seat bag		1
61	1572163	Seat cover SB32 ST44 for seat bag		1
62	1572164	Seat cover SB34 ST44 for seat bag		1
63	1572194	Seat cover SB36 ST44 for seat bag		1
64	1572195	Seat cover SB38 ST44 for seat bag		1
65	1572196	Seat cover SB40 ST44 for seat bag		1
66	1572197	Seat cover SB42 ST44 for seat bag		1
67	1572354	Seat cover SB44 ST44 for seat bag		1
68	1572355	Seat cover SB46 ST44 for seat bag		1
69	1572356	Seat cover SB48 ST44 for seat bag		1
70	1572357	Seat cover SB50 ST44 for seat bag		1
71	1572358	Seat cover SB32 ST46 for seat bag		1
72	1572359	Seat cover SB34 ST46 for seat bag		1
73	1572360	Seat cover SB36 ST46 for seat bag		1
74	1572361	Seat cover SB38 ST46 for seat bag		1
75	1572362	Seat cover SB40 ST46 for seat bag		1
76	1572363	Seat cover SB42 ST46 for seat bag		1
77	1572364	Seat cover SB44 ST46 for seat bag		1
78	1572365	Seat cover SB46 ST46 for seat bag		1
79	1572406	Seat cover SB48 ST46 for seat bag		1
80	1572407	Seat cover SB50 ST46 for seat bag		1
81	1572408	Seat cover SB32 ST48 for seat bag		1
82	1572409	Seat cover SB34 ST48 for seat bag		1
83	1572410	Seat cover SB36 ST48 for seat bag		1
84	1572411	Seat cover SB38 ST48 for seat bag		1
85	1572412	Seat cover SB40 ST48 for seat bag		1
86	1572413	Seat cover SB42 ST48 for seat bag		1

**Seat cover standard for seat bag**

Pos.	Part number	Description	Valid from → until	Qty
87	1572414	Seat cover SB44 ST48 for seat bag		1
88	1572415	Seat cover SB46 ST48 for seat bag		1
89	1572416	Seat cover SB48 ST48 for seat bag		1
90	1572417	Seat cover SB50 ST48 for seat bag		1
91	1572418	Seat cover SB32 ST50 for seat bag		1
92	1572419	Seat cover SB34 ST50 for seat bag		1
93	1572420	Seat cover SB36 ST50 for seat bag		1
94	1572421	Seat cover SB38 ST50 for seat bag		1
95	1572422	Seat cover SB40 ST50 for seat bag		1
96	1572423	Seat cover SB42 ST50 for seat bag		1
97	1572424	Seat cover SB44 ST50 for seat bag		1
98	1572425	Seat cover SB46 ST50 for seat bag		1
99	1572426	Seat cover SB48 ST50 for seat bag		1
100	1572427	Seat cover SB50 ST50 for seat bag		1
101	1437966	Seat cover bar kit ST32		1
102	1437967	Seat cover bar kit ST34		1
103	1437968	Seat cover bar kit ST36		1
104	1437969	Seat cover bar kit ST38		1
105	1437970	Seat cover bar kit ST40		1
106	1437971	Seat cover bar kit ST42		1
107	1437972	Seat cover bar kit ST44		1
108	1437973	Seat cover bar kit ST46		1
109	1437974	Seat cover bar kit ST48		1
110	1437975	Seat cover bar kit ST50		1
111	1578786	Seat bag SB32		1
112	1578787	Seat bag SB34		1
113	1578788	Seat bag SB36		1
114	1578789	Seat bag SB38		1
115	1578790	Seat bag SB40		1
116	1578791	Seat bag SB42		1
117	1578792	Seat bag SB44		1
118	1578793	Seat bag SB46		1
119	1578794	Seat bag SB48		1
120	1578795	Seat bag SB50		1

## Seat cover with 1 Velcro band (soft band) for seat bag

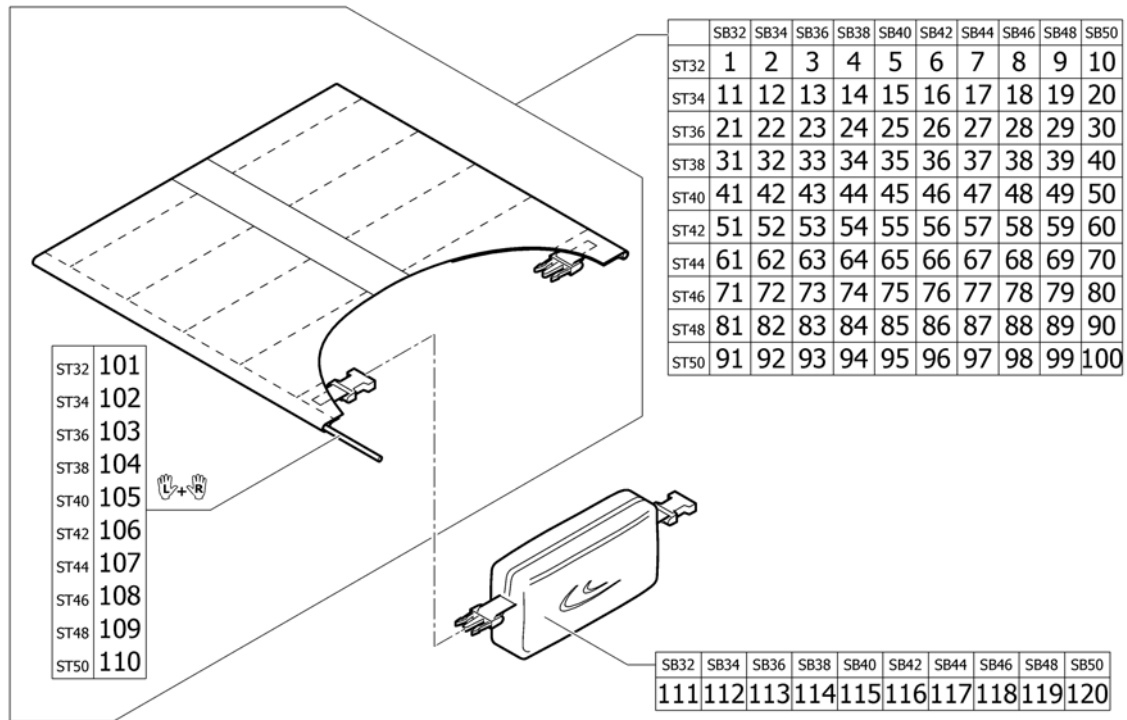

Pos.	Part number	Description	Valid from → until	Qty
1	1572428	Seat cover SB32 ST32 for seat bag with 1 Velcro band (soft band)		1
2	1572429	Seat cover SB34 ST32 for seat bag with 1 Velcro band (soft band)		1
3	1572430	Seat cover SB36 ST32 for seat bag with 1 Velcro band (soft band)		1
4	1572431	Seat cover SB38 ST32 for seat bag with 1 Velcro band (soft band)		1
5	1572432	Seat cover SB40 ST32 for seat bag with 1 Velcro band (soft band)		1
6	1572433	Seat cover SB42 ST32 for seat bag with 1 Velcro band (soft band)		1
7	1572434	Seat cover SB44 ST32 for seat bag with 1 Velcro band (soft band)		1
8	1572435	Seat cover SB46 ST32 for seat bag with 1 Velcro band (soft band)		1
9	1572436	Seat cover SB48 ST32 for seat bag with 1 Velcro band (soft band)		1
10	1572437	Seat cover SB50 ST32 for seat bag with 1 Velcro band (soft band)		1
11	1572438	Seat cover SB32 ST34 for seat bag with 1 Velcro band (soft band)		1
12	1572464	Seat cover SB34 ST34 for seat bag with 1 Velcro band (soft band)		1
13	1572465	Seat cover SB36 ST34 for seat bag with 1 Velcro band (soft band)		1
14	1572466	Seat cover SB38 ST34 for seat bag with 1 Velcro band (soft band)		1
15	1572467	Seat cover SB40 ST34 for seat bag with 1 Velcro band (soft band)		1
16	1572468	Seat cover SB42 ST34 for seat bag with 1 Velcro band (soft band)		1
17	1572469	Seat cover SB44 ST34 for seat bag with 1 Velcro band (soft band)		1
18	1572470	Seat cover SB46 ST34 for seat bag with 1 Velcro band (soft band)		1
19	1572471	Seat cover SB48 ST34 for seat bag with 1 Velcro band (soft band)		1
20	1572472	Seat cover SB50 ST34 for seat bag with 1 Velcro band (soft band)		1
21	1572473	Seat cover SB32 ST36 for seat bag with 1 Velcro band (soft band)		1
22	1572474	Seat cover SB34 ST36 for seat bag with 1 Velcro band (soft band)		1
23	1572475	Seat cover SB36 ST36 for seat bag with 1 Velcro band (soft band)		1
24	1572476	Seat cover SB38 ST36 for seat bag with 1 Velcro band (soft band)		1
25	1572477	Seat cover SB40 ST36 for seat bag with 1 Velcro band (soft band)		1
26	1572478	Seat cover SB42 ST36 for seat bag with 1 Velcro band (soft band)		1
27	1572479	Seat cover SB44 ST36 for seat bag with 1 Velcro band (soft band)		1
28	1572480	Seat cover SB46 ST36 for seat bag with 1 Velcro band (soft band)		1
29	1572481	Seat cover SB48 ST36 for seat bag with 1 Velcro band (soft band)		1
30	1572482	Seat cover SB50 ST36 for seat bag with 1 Velcro band (soft band)		1
31	1572483	Seat cover SB32 ST38 for seat bag with 1 Velcro band (soft band)		1

## Seat cover with 1 Velcro band (soft band) for seat bag


Pos.	Part number	Description	Valid from → until	Qty
32	1572484	Seat cover SB34 ST38 for seat bag with 1 Velcro band (soft band)		1
33	1572485	Seat cover SB36 ST38 for seat bag with 1 Velcro band (soft band)		1
34	1572676	Seat cover SB38 ST38 for seat bag with 1 Velcro band (soft band)		1
35	1572863	Seat cover SB40 ST38 for seat bag with 1 Velcro band (soft band)		1
36	1572864	Seat cover SB42 ST38 for seat bag with 1 Velcro band (soft band)		1
37	1572865	Seat cover SB44 ST38 for seat bag with 1 Velcro band (soft band)		1
38	1572866	Seat cover SB46 ST38 for seat bag with 1 Velcro band (soft band)		1
39	1572867	Seat cover SB48 ST38 for seat bag with 1 Velcro band (soft band)		1
40	1572868	Seat cover SB50 ST38 for seat bag with 1 Velcro band (soft band)		1
41	1572869	Seat cover SB32 ST40 for seat bag with 1 Velcro band (soft band)		1
42	1572870	Seat cover SB34 ST40 for seat bag with 1 Velcro band (soft band)		1
43	1572871	Seat cover SB36 ST40 for seat bag with 1 Velcro band (soft band)		1
44	1572872	Seat cover SB38 ST40 for seat bag with 1 Velcro band (soft band)		1
45	1572873	Seat cover SB40 ST40 for seat bag with 1 Velcro band (soft band)		1
46	1572874	Seat cover SB42 ST40 for seat bag with 1 Velcro band (soft band)		1
47	1572875	Seat cover SB44 ST40 for seat bag with 1 Velcro band (soft band)		1
48	1572876	Seat cover SB46 ST40 for seat bag with 1 Velcro band (soft band)		1
49	1572877	Seat cover SB48 ST40 for seat bag with 1 Velcro band (soft band)		1
50	1572878	Seat cover SB50 ST40 for seat bag with 1 Velcro band (soft band)		1
51	1572879	Seat cover SB32 ST42 for seat bag with 1 Velcro band (soft band)		1
52	1572880	Seat cover SB34 ST42 for seat bag with 1 Velcro band (soft band)		1
53	1572881	Seat cover SB36 ST42 for seat bag with 1 Velcro band (soft band)		1
54	1572900	Seat cover SB38 ST42 for seat bag with 1 Velcro band (soft band)		1
55	1572901	Seat cover SB40 ST42 for seat bag with 1 Velcro band (soft band)		1
56	1572902	Seat cover SB42 ST42 for seat bag with 1 Velcro band (soft band)		1
57	1572903	Seat cover SB44 ST42 for seat bag with 1 Velcro band (soft band)		1
58	1572904	Seat cover SB46 ST42 for seat bag with 1 Velcro band (soft band)		1
59	1572905	Seat cover SB48 ST42 for seat bag with 1 Velcro band (soft band)		1
60	1572906	Seat cover SB50 ST42 for seat bag with 1 Velcro band (soft band)		1
61	1572907	Seat cover SB32 ST44 for seat bag with 1 Velcro band (soft band)		1
62	1572908	Seat cover SB34 ST44 for seat bag with 1 Velcro band (soft band)		1
63	1572909	Seat cover SB36 ST44 for seat bag with 1 Velcro band (soft band)		1
64	1572921	Seat cover SB38 ST44 for seat bag with 1 Velcro band (soft band)		1
65	1572959	Seat cover SB40 ST44 for seat bag with 1 Velcro band (soft band)		1
66	1572960	Seat cover SB42 ST44 for seat bag with 1 Velcro band (soft band)		1
67	1572961	Seat cover SB44 ST44 for seat bag with 1 Velcro band (soft band)		1
68	1572962	Seat cover SB46 ST44 for seat bag with 1 Velcro band (soft band)		1
69	1572963	Seat cover SB48 ST44 for seat bag with 1 Velcro band (soft band)		1
70	1572964	Seat cover SB50 ST44 for seat bag with 1 Velcro band (soft band)		1
71	1572965	Seat cover SB32 ST46 for seat bag with 1 Velcro band (soft band)		1
72	1573142	Seat cover SB34 ST46 for seat bag with 1 Velcro band (soft band)		1
73	1573143	Seat cover SB36 ST46 for seat bag with 1 Velcro band (soft band)		1
74	1573452	Seat cover SB38 ST46 for seat bag with 1 Velcro band (soft band)		1
75	1573453	Seat cover SB40 ST46 for seat bag with 1 Velcro band (soft band)		1
76	1573454	Seat cover SB42 ST46 for seat bag with 1 Velcro band (soft band)		1
77	1573455	Seat cover SB44 ST46 for seat bag with 1 Velcro band (soft band)		1
78	1573456	Seat cover SB46 ST46 for seat bag with 1 Velcro band (soft band)		1
79	1573457	Seat cover SB48 ST46 for seat bag with 1 Velcro band (soft band)		1
80	1573458	Seat cover SB50 ST46 for seat bag with 1 Velcro band (soft band)		1
81	1573459	Seat cover SB32 ST48 for seat bag with 1 Velcro band (soft band)		1
82	1573460	Seat cover SB34 ST48 for seat bag with 1 Velcro band (soft band)		1
83	1573461	Seat cover SB36 ST48 for seat bag with 1 Velcro band (soft band)		1
84	1573462	Seat cover SB38 ST48 for seat bag with 1 Velcro band (soft band)		1
85	1573463	Seat cover SB40 ST48 for seat bag with 1 Velcro band (soft band)		1
86	1573464	Seat cover SB42 ST48 for seat bag with 1 Velcro band (soft band)		1



**Seat cover with 1 Velcro band (soft band) for seat bag**

Pos.	Part number	Description	Valid from → until	Qty
87	1573465	Seat cover SB44 ST48 for seat bag with 1 Velcro band (soft band)		1
88	1573466	Seat cover SB46 ST48 for seat bag with 1 Velcro band (soft band)		1
89	1573467	Seat cover SB48 ST48 for seat bag with 1 Velcro band (soft band)		1
90	1573468	Seat cover SB50 ST48 for seat bag with 1 Velcro band (soft band)		1
91	1573469	Seat cover SB32 ST50 for seat bag with 1 Velcro band (soft band)		1
92	1573470	Seat cover SB34 ST50 for seat bag with 1 Velcro band (soft band)		1
93	1573471	Seat cover SB36 ST50 for seat bag with 1 Velcro band (soft band)		1
94	1573496	Seat cover SB38 ST50 for seat bag with 1 Velcro band (soft band)		1
95	1573497	Seat cover SB40 ST50 for seat bag with 1 Velcro band (soft band)		1
96	1573498	Seat cover SB42 ST50 for seat bag with 1 Velcro band (soft band)		1
97	1573499	Seat cover SB44 ST50 for seat bag with 1 Velcro band (soft band)		1
98	1573500	Seat cover SB46 ST50 for seat bag with 1 Velcro band (soft band)		1
99	1573501	Seat cover SB48 ST50 for seat bag with 1 Velcro band (soft band)		1
100	1573502	Seat cover SB50 ST50 for seat bag with 1 Velcro band (soft band)		1
101	1437966	Seat cover bar kit ST32		1
102	1437967	Seat cover bar kit ST34		1
103	1437968	Seat cover bar kit ST36		1
104	1437969	Seat cover bar kit ST38		1
105	1437970	Seat cover bar kit ST40		1
106	1437971	Seat cover bar kit ST42		1
107	1437972	Seat cover bar kit ST44		1
108	1437973	Seat cover bar kit ST46		1
109	1437974	Seat cover bar kit ST48		1
110	1437975	Seat cover bar kit ST50		1
111	1578786	Seat bag SB32		1
112	1578787	Seat bag SB34		1
113	1578788	Seat bag SB36		1
114	1578789	Seat bag SB38		1
115	1578790	Seat bag SB40		1
116	1578791	Seat bag SB42		1
117	1578792	Seat bag SB44		1
118	1578793	Seat bag SB46		1
119	1578794	Seat bag SB48		1
120	1578795	Seat bag SB50		1

## Seat cover with 2 Velcro bands (rough band) for seat bag

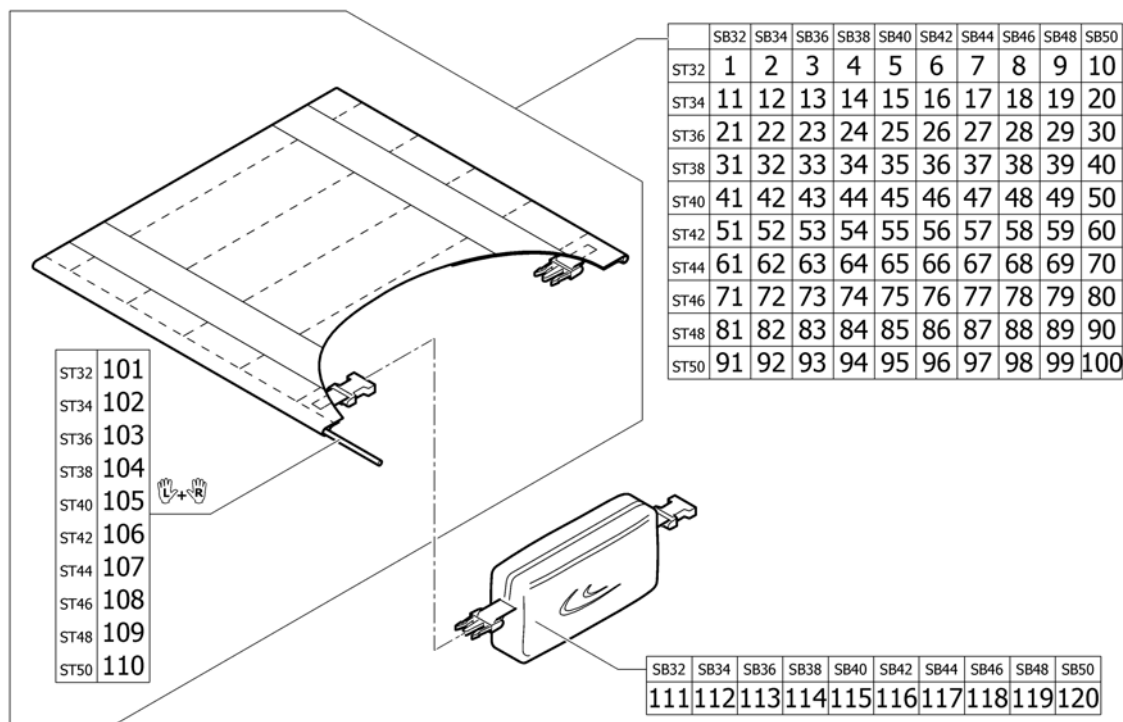

Pos.	Part number	Description	Valid from → until	Qty
1	1573503	Seat cover SB32 ST32 for seat bag with 2 Velcro bands (rough band)		1
2	1573504	Seat cover SB34 ST32 for seat bag with 2 Velcro bands (rough band)		1
3	1573505	Seat cover SB36 ST32 for seat bag with 2 Velcro bands (rough band)		1
4	1573506	Seat cover SB38 ST32 for seat bag with 2 Velcro bands (rough band)		1
5	1573507	Seat cover SB40 ST32 for seat bag with 2 Velcro bands (rough band)		1
6	1573508	Seat cover SB42 ST32 for seat bag with 2 Velcro bands (rough band)		1
7	1573509	Seat cover SB44 ST32 for seat bag with 2 Velcro bands (rough band)		1
8	1573510	Seat cover SB46 ST32 for seat bag with 2 Velcro bands (rough band)		1
9	1573511	Seat cover SB48 ST32 for seat bag with 2 Velcro bands (rough band)		1
10	1573512	Seat cover SB50 ST32 for seat bag with 2 Velcro bands (rough band)		1
11	1573513	Seat cover SB32 ST34 for seat bag with 2 Velcro bands (rough band)		1
12	1573514	Seat cover SB34 ST34 for seat bag with 2 Velcro bands (rough band)		1
13	1573515	Seat cover SB36 ST34 for seat bag with 2 Velcro bands (rough band)		1
14	1573516	Seat cover SB38 ST34 for seat bag with 2 Velcro bands (rough band)		1
15	1573517	Seat cover SB40 ST34 for seat bag with 2 Velcro bands (rough band)		1
16	1573518	Seat cover SB42 ST34 for seat bag with 2 Velcro bands (rough band)		1
17	1573519	Seat cover SB44 ST34 for seat bag with 2 Velcro bands (rough band)		1
18	1573520	Seat cover SB46 ST34 for seat bag with 2 Velcro bands (rough band)		1
19	1573521	Seat cover SB48 ST34 for seat bag with 2 Velcro bands (rough band)		1
20	1573522	Seat cover SB50 ST34 for seat bag with 2 Velcro bands (rough band)		1
21	1573523	Seat cover SB32 ST36 for seat bag with 2 Velcro bands (rough band)		1
22	1573524	Seat cover SB34 ST36 for seat bag with 2 Velcro bands (rough band)		1
23	1573525	Seat cover SB36 ST36 for seat bag with 2 Velcro bands (rough band)		1
24	1573526	Seat cover SB38 ST36 for seat bag with 2 Velcro bands (rough band)		1
25	1573527	Seat cover SB40 ST36 for seat bag with 2 Velcro bands (rough band)		1
26	1573528	Seat cover SB42 ST36 for seat bag with 2 Velcro bands (rough band)		1
27	1573529	Seat cover SB44 ST36 for seat bag with 2 Velcro bands (rough band)		1
28	1573530	Seat cover SB46 ST36 for seat bag with 2 Velcro bands (rough band)		1
29	1573531	Seat cover SB48 ST36 for seat bag with 2 Velcro bands (rough band)		1
30	1573532	Seat cover SB50 ST36 for seat bag with 2 Velcro bands (rough band)		1
31	1573533	Seat cover SB32 ST38 for seat bag with 2 Velcro bands (rough band)		1


**Seat cover with 2 Velcro bands (rough band) for seat bag**

Pos.	Part number	Description	Valid from → until	Qty
32	1573534	Seat cover SB34 ST38 for seat bag with 2 Velcro bands (rough band)		1
33	1573535	Seat cover SB36 ST38 for seat bag with 2 Velcro bands (rough band)		1
34	1573536	Seat cover SB38 ST38 for seat bag with 2 Velcro bands (rough band)		1
35	1573537	Seat cover SB40 ST38 for seat bag with 2 Velcro bands (rough band)		1
36	1573538	Seat cover SB42 ST38 for seat bag with 2 Velcro bands (rough band)		1
37	1573539	Seat cover SB44 ST38 for seat bag with 2 Velcro bands (rough band)		1
38	1573540	Seat cover SB46 ST38 for seat bag with 2 Velcro bands (rough band)		1
39	1573541	Seat cover SB48 ST38 for seat bag with 2 Velcro bands (rough band)		1
40	1573542	Seat cover SB50 ST38 for seat bag with 2 Velcro bands (rough band)		1
41	1573543	Seat cover SB32 ST40 for seat bag with 2 Velcro bands (rough band)		1
42	1573544	Seat cover SB34 ST40 for seat bag with 2 Velcro bands (rough band)		1
43	1573545	Seat cover SB36 ST40 for seat bag with 2 Velcro bands (rough band)		1
44	1573546	Seat cover SB38 ST40 for seat bag with 2 Velcro bands (rough band)		1
45	1573547	Seat cover SB40 ST40 for seat bag with 2 Velcro bands (rough band)		1
46	1573548	Seat cover SB42 ST40 for seat bag with 2 Velcro bands (rough band)		1
47	1573665	Seat cover SB44 ST40 for seat bag with 2 Velcro bands (rough band)		1
48	1573666	Seat cover SB46 ST40 for seat bag with 2 Velcro bands (rough band)		1
49	1573667	Seat cover SB48 ST40 for seat bag with 2 Velcro bands (rough band)		1
50	1573668	Seat cover SB50 ST40 for seat bag with 2 Velcro bands (rough band)		1
51	1573669	Seat cover SB32 ST42 for seat bag with 2 Velcro bands (rough band)		1
52	1573670	Seat cover SB34 ST42 for seat bag with 2 Velcro bands (rough band)		1
53	1573671	Seat cover SB36 ST42 for seat bag with 2 Velcro bands (rough band)		1
54	1573672	Seat cover SB38 ST42 for seat bag with 2 Velcro bands (rough band)		1
55	1573673	Seat cover SB40 ST42 for seat bag with 2 Velcro bands (rough band)		1
56	1573674	Seat cover SB42 ST42 for seat bag with 2 Velcro bands (rough band)		1
57	1573675	Seat cover SB44 ST42 for seat bag with 2 Velcro bands (rough band)		1
58	1573676	Seat cover SB46 ST42 for seat bag with 2 Velcro bands (rough band)		1
59	1573677	Seat cover SB48 ST42 for seat bag with 2 Velcro bands (rough band)		1
60	1573678	Seat cover SB50 ST42 for seat bag with 2 Velcro bands (rough band)		1
61	1573679	Seat cover SB32 ST44 for seat bag with 2 Velcro bands (rough band)		1
62	1573680	Seat cover SB34 ST44 for seat bag with 2 Velcro bands (rough band)		1
63	1573681	Seat cover SB36 ST44 for seat bag with 2 Velcro bands (rough band)		1
64	1573716	Seat cover SB38 ST44 for seat bag with 2 Velcro bands (rough band)		1
65	1573717	Seat cover SB40 ST44 for seat bag with 2 Velcro bands (rough band)		1
66	1573796	Seat cover SB42 ST44 for seat bag with 2 Velcro bands (rough band)		1
67	1573797	Seat cover SB44 ST44 for seat bag with 2 Velcro bands (rough band)		1
68	1573798	Seat cover SB46 ST44 for seat bag with 2 Velcro bands (rough band)		1
69	1573799	Seat cover SB48 ST44 for seat bag with 2 Velcro bands (rough band)		1
70	1573800	Seat cover SB50 ST44 for seat bag with 2 Velcro bands (rough band)		1
71	1573801	Seat cover SB32 ST46 for seat bag with 2 Velcro bands (rough band)		1
72	1573802	Seat cover SB34 ST46 for seat bag with 2 Velcro bands (rough band)		1
73	1573803	Seat cover SB36 ST46 for seat bag with 2 Velcro bands (rough band)		1
74	1573804	Seat cover SB38 ST46 for seat bag with 2 Velcro bands (rough band)		1
75	1573805	Seat cover SB40 ST46 for seat bag with 2 Velcro bands (rough band)		1
76	1573806	Seat cover SB42 ST46 for seat bag with 2 Velcro bands (rough band)		1
77	1573807	Seat cover SB44 ST46 for seat bag with 2 Velcro bands (rough band)		1
78	1573808	Seat cover SB46 ST46 for seat bag with 2 Velcro bands (rough band)		1
79	1573809	Seat cover SB48 ST46 for seat bag with 2 Velcro bands (rough band)		1
80	1573810	Seat cover SB50 ST46 for seat bag with 2 Velcro bands (rough band)		1
81	1573811	Seat cover SB32 ST48 for seat bag with 2 Velcro bands (rough band)		1
82	1573812	Seat cover SB34 ST48 for seat bag with 2 Velcro bands (rough band)		1
83	1573813	Seat cover SB36 ST48 for seat bag with 2 Velcro bands (rough band)		1
84	1573814	Seat cover SB38 ST48 for seat bag with 2 Velcro bands (rough band)		1
85	1573815	Seat cover SB40 ST48 for seat bag with 2 Velcro bands (rough band)		1
86	1573816	Seat cover SB42 ST48 for seat bag with 2 Velcro bands (rough band)		1

**Seat cover with 2 Velcro bands (rough band) for seat bag**

Pos.	Part number	Description	Valid from → until	Qty
87	1573817	Seat cover SB44 ST48 for seat bag with 2 Velcro bands (rough band)		1
88	1573818	Seat cover SB46 ST48 for seat bag with 2 Velcro bands (rough band)		1
89	1573819	Seat cover SB48 ST48 for seat bag with 2 Velcro bands (rough band)		1
90	1573820	Seat cover SB50 ST48 for seat bag with 2 Velcro bands (rough band)		1
91	1573821	Seat cover SB32 ST50 for seat bag with 2 Velcro bands (rough band)		1
92	1573822	Seat cover SB34 ST50 for seat bag with 2 Velcro bands (rough band)		1
93	1573823	Seat cover SB36 ST50 for seat bag with 2 Velcro bands (rough band)		1
94	1573824	Seat cover SB38 ST50 for seat bag with 2 Velcro bands (rough band)		1
95	1573825	Seat cover SB40 ST50 for seat bag with 2 Velcro bands (rough band)		1
96	1573826	Seat cover SB42 ST50 for seat bag with 2 Velcro bands (rough band)		1
97	1573827	Seat cover SB44 ST50 for seat bag with 2 Velcro bands (rough band)		1
98	1573828	Seat cover SB46 ST50 for seat bag with 2 Velcro bands (rough band)		1
99	1573829	Seat cover SB48 ST50 for seat bag with 2 Velcro bands (rough band)		1
100	1573830	Seat cover SB50 ST50 for seat bag with 2 Velcro bands (rough band)		1
101	1437966	Seat cover bar kit ST32		1
102	1437967	Seat cover bar kit ST34		1
103	1437968	Seat cover bar kit ST36		1
104	1437969	Seat cover bar kit ST38		1
105	1437970	Seat cover bar kit ST40		1
106	1437971	Seat cover bar kit ST42		1
107	1437972	Seat cover bar kit ST44		1
108	1437973	Seat cover bar kit ST46		1
109	1437974	Seat cover bar kit ST48		1
110	1437975	Seat cover bar kit ST50		1
111	1578786	Seat bag SB32		1
112	1578787	Seat bag SB34		1
113	1578788	Seat bag SB36		1
114	1578789	Seat bag SB38		1
115	1578790	Seat bag SB40		1
116	1578791	Seat bag SB42		1
117	1578792	Seat bag SB44		1
118	1578793	Seat bag SB46		1
119	1578794	Seat bag SB48		1
120	1578795	Seat bag SB50		1

## Seat cushion standard

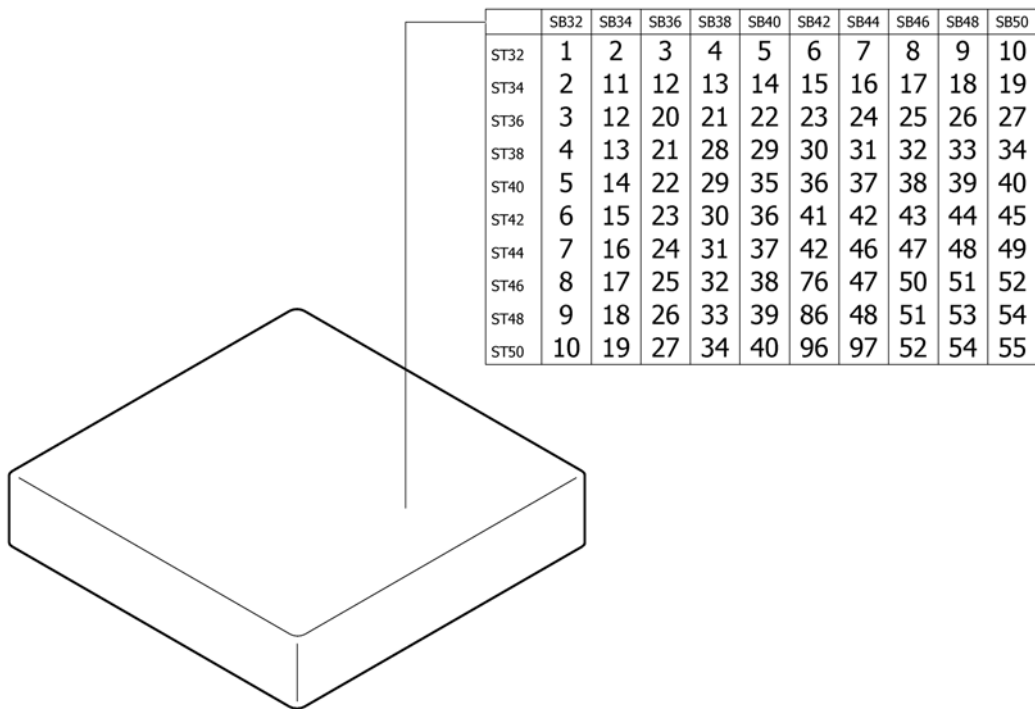


Pos.	Part number	Description	Valid from → until	Qty
1	1556235	Seat cushion 320mm x 320mm		1
2	1556236	Seat cushion 320mm x 340mm		1
3	1556316	Seat cushion 320mm x 360mm		1
4	1556317	Seat cushion 320mm x 380mm		1
5	1556318	Seat cushion 320mm x 400mm		1
6	1556319	Seat cushion 320mm x 420mm		1
7	1556320	Seat cushion 320mm x 440mm		1
8	1556321	Seat cushion 320mm x 460mm		1
9	1556322	Seat cushion 320mm x 480mm		1
10	1556323	Seat cushion 320mm x 500mm		1
11	1556325	Seat cushion 340mm x 340mm		1
12	1556326	Seat cushion 340mm x 360mm		1
13	1556327	Seat cushion 340mm x 380mm		1
14	1556328	Seat cushion 340mm x 400mm		1
15	1556329	Seat cushion 340mm x 420mm		1
16	1556330	Seat cushion 340mm x 440mm		1
17	1556331	Seat cushion 340mm x 460mm		1
18	1556332	Seat cushion 340mm x 480mm		1
19	1556333	Seat cushion 340mm x 500mm		1
20	1556335	Seat cushion 360mm x 360mm		1
21	1556336	Seat cushion 360mm x 380mm		1
22	1556337	Seat cushion 360mm x 400mm		1
23	1556338	Seat cushion 360mm x 420mm		1
24	1556339	Seat cushion 360mm x 440mm		1
25	1556340	Seat cushion 360mm x 460mm		1
26	1556341	Seat cushion 360mm x 480mm		1
27	1556342	Seat cushion 360mm x 500mm		1
28	1556344	Seat cushion 380mm x 380mm		1
29	1556345	Seat cushion 380mm x 400mm		1
30	1556346	Seat cushion 380mm x 420mm		1
31	1556347	Seat cushion 380mm x 440mm		1



**Seat cushion standard**


Pos.	Part number	Description	Valid from → until	Qty
32	1556348	Seat cushion 380mm x 460mm		1
33	1556349	Seat cushion 380mm x 480mm		1
34	1556350	Seat cushion 380mm x 500mm		1
35	1556352	Seat cushion 400mm x 400mm		1
36	1556353	Seat cushion 400mm x 420mm		1
37	1556354	Seat cushion 400mm x 440mm		1
38	1556355	Seat cushion 400mm x 460mm		1
39	1556356	Seat cushion 400mm x 480mm		1
40	1556357	Seat cushion 400mm x 500mm		1
41	1556359	Seat cushion 420mm x 420mm		1
42	1556360	Seat cushion 420mm x 440mm		1
43	1556361	Seat cushion 420mm x 460mm		1
44	1556362	Seat cushion 420mm x 480mm		1
45	1556363	Seat cushion 420mm x 500mm		1
46	1556365	Seat cushion 440mm x 440mm		1
47	1556366	Seat cushion 440mm x 460mm		1
48	1556367	Seat cushion 440mm x 480mm		1
49	1556368	Seat cushion 440mm x 500mm		1
50	1556370	Seat cushion 460mm x 460mm		1
51	1556371	Seat cushion 460mm x 480mm		1
52	1556372	Seat cushion 460mm x 500mm		1
53	1556374	Seat cushion 480mm x 480mm		1
54	1556375	Seat cushion 480mm x 500mm		1
55	1556377	Seat cushion 500mm x 500mm		1

# BACKREST

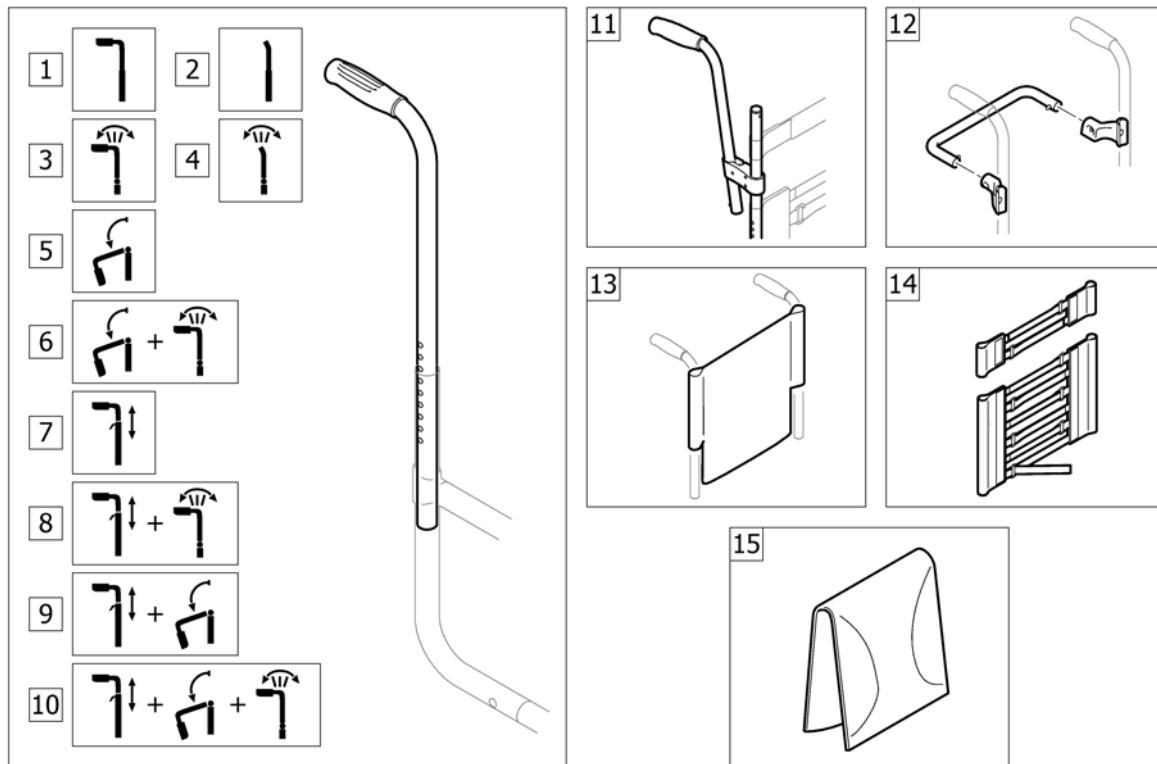


Pos.	Part number	Description	Valid from → until	Qty
1	-	Push handles for backrest standard	2004-01-01 → 2012-12-31	1
2	-	Backrest tubes without push handles for standard backrest	2008-07-01 → 2012-12-31	1
3	-	Push handles for backrest angle adjustable	2004-01-01 → 2012-12-31	1
4	-	Backrest tubes without push handles for angle adjustable backrest	2008-07-01 → 2012-12-31	1
5	-	Push handles for backrest foldable	2004-01-01 → 2012-12-31	1
6	-	Push handles for backrest foldable + angle adjustable	2004-01-01 → 2012-12-31	1
7	-	Push handles height adjustable	2004-01-01 → 2012-12-31	1
8	-	Push handles height adjustable for backrest angle adjustable	2004-01-01 → 2012-12-31	1
9	-	Push handles height adjustable for backrest foldable	2004-01-01 → 2012-12-31	1
10	-	Push handles height adjustable for backrest foldable + angle adjustable	2004-01-01 → 2012-12-31	1
11	-	Push handles, inclined	2004-01-01 → 2012-12-31	1
12	-	Backrest stabilization bar	2008-07-01 → 2012-12-31	1
13	-	Back cover, standard	2004-01-01 → 2012-12-31	1
14	-	Bands to adjustable backrest	2004-01-01 → 2012-12-31	1
15	-	Cover to adjustable backrest	2004-01-01 → 2012-12-31	1

## Push handles for backrest standard

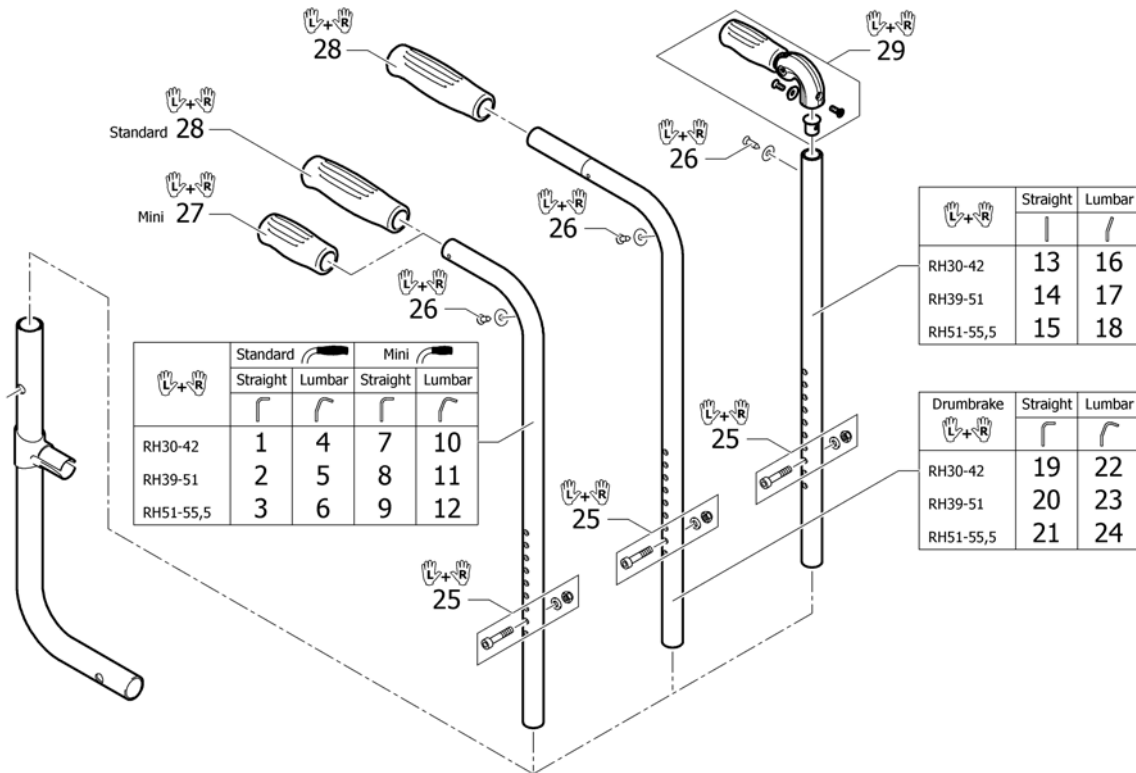


Pos.	Part number	Description	Valid from → until	Qty
1	1507721	Push handles standard, short, pair		1
2	1507722	Push handles standard, medium, pair		1
3	1507723	Push handles standard, long, pair		1
4	1507724	Push handles standard lumbar short, pair		1
5	1507725	Push handles standard lumbar medium, pair		1
6	1507726	Push handles standard lumbar long, pair		1
7	1525336	Push handles mini, short, pair		1
8	1525337	Push handles mini, medium, pair		1
9	1525338	Push handles mini, long, pair		1
10	1525339	Push handles mini lumbar short, pair		1
11	1525340	Push handles mini lumbar medium, pair		1
12	1525341	Push handles mini lumbar long, pair		1
13	1507715	Backrest tube straight short to foldable push handles, pair		1
14	1507716	Backrest tube straight medium to foldable push handles, pair		1
15	1507717	Backrest tube straight long to foldable push handles, pair		1
16	1507718	Backrest tube lumbar short to foldable push handles, pair		1
17	1507719	Backrest tube lumbar medium to foldable push handles, pair		1
18	1507720	Backrest tube lumbar long to foldable push handles, pair		1
19	1507727	Push handles short to drum brake, pair		1
20	1507728	Push handles medium to drum brake, pair		1
21	1507729	Push handles long to drum brake, pair		1
22	1507730	Push handles lumbar short to drum brake, pair		1
23	1507731	Push handles lumbar medium to drum brake, pair		1
24	1507732	Push handles lumbar long to drum brake, pair		1
25	1439279	Fixation kit to backrest tube		1
26	1432083	Fixation kit for backrest band		1
27	1525126	Handle black, mini, pair		1
28	1507734	Handle black, pair		1
29	1507733	Push handles foldable, pair		1

## Backrest tubes without push handles for standard backrest

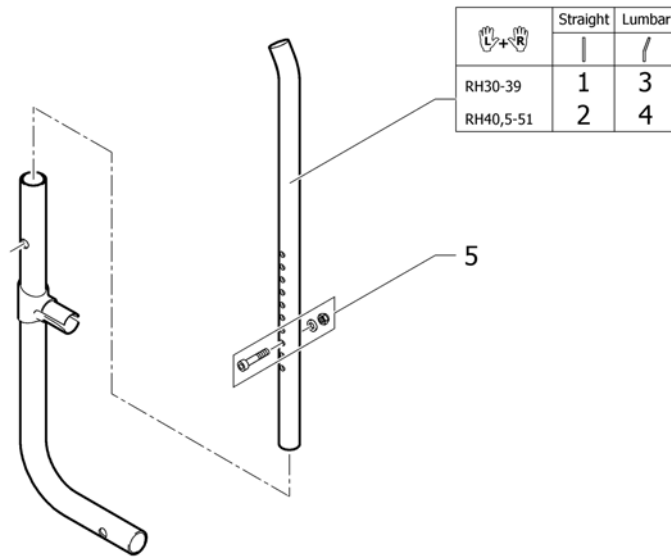

Pos.	Part number	Description	Valid from → until	Qty
1	1518292	Backrest tubes L without push handles, pair		1
2	1518293	Backrest tubes XL without push handles, pair		1
3	1518294	Backrest tubes lumbar L without push handles, pair		1
4	1518295	Backrest tubes lumbar XL without push handles, pair		1
5	1439279	Fixation kit to backrest tube		1


## Push handles for backrest angle adjustable

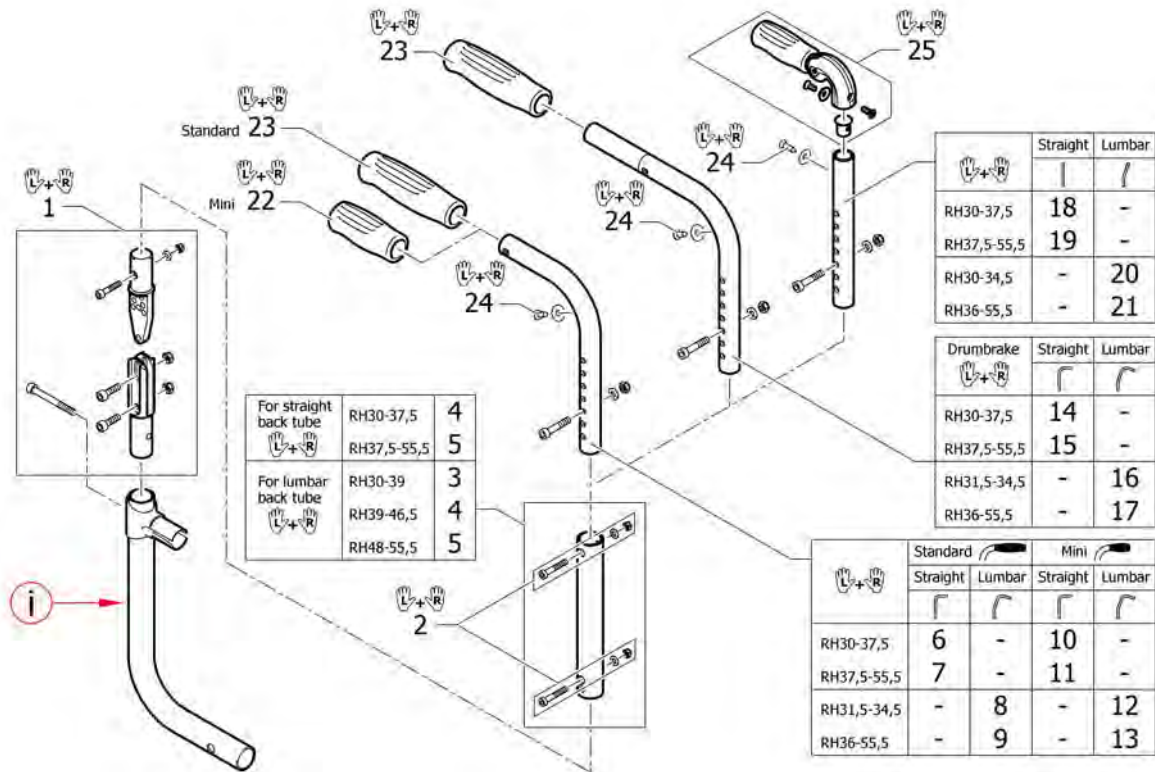

Pos.	Part number	Description	Valid from → until	Qty
i	-	A special rear frame is necessary. Please see chapter CHASSIS.		0
1	1438208	Backrest angle kit		1
2	1509134	Screw kit for telescopic tube		1
3	1439459	Tube kit short to angle adjustable back		1
4	1438209	Tube kit medium to angle adjustable back		1
5	1439460	Tube kit long to angle adjustable back		1
6	1507739	Push handles standard, XS, pair		1
7	1507740	Push handles standard, S, pair		1
8	1507741	Push handles standard lumbar XS, pair		1
9	1507742	Push handles standard lumbar S, pair		1
10	1525342	Push handles mini, XS, pair		1
11	1525343	Push handles mini, S, pair		1
12	1525344	Push handles mini lumbar XS, pair		1
13	1525345	Push handles mini lumbar S, pair		1
14	1507743	Push handles XS to drum brake, pair		1
15	1507744	Push handles S to drum brake, pair		1
16	1507745	Push handles lumbar XS to drum brake, pair		1
17	1507746	Push handles lumbar S to drum brake, pair		1
18	1507735	Backrest tube straight XS to foldable push handles, pair		1
19	1507736	Backrest tube straight S to foldable push handles, pair		1
20	1507737	Backrest tube lumbar XS to foldable push handles, pair		1
21	1507738	Backrest tube lumbar S to foldable push handles, pair		1
22	1525126	Handle black, mini, pair		1
23	1507734	Handle black, pair		1
24	1432083	Fixation kit for backrest band		1
25	1507733	Push handles foldable, pair		1




## Backrest tubes without push handles for angle adjustable backrest

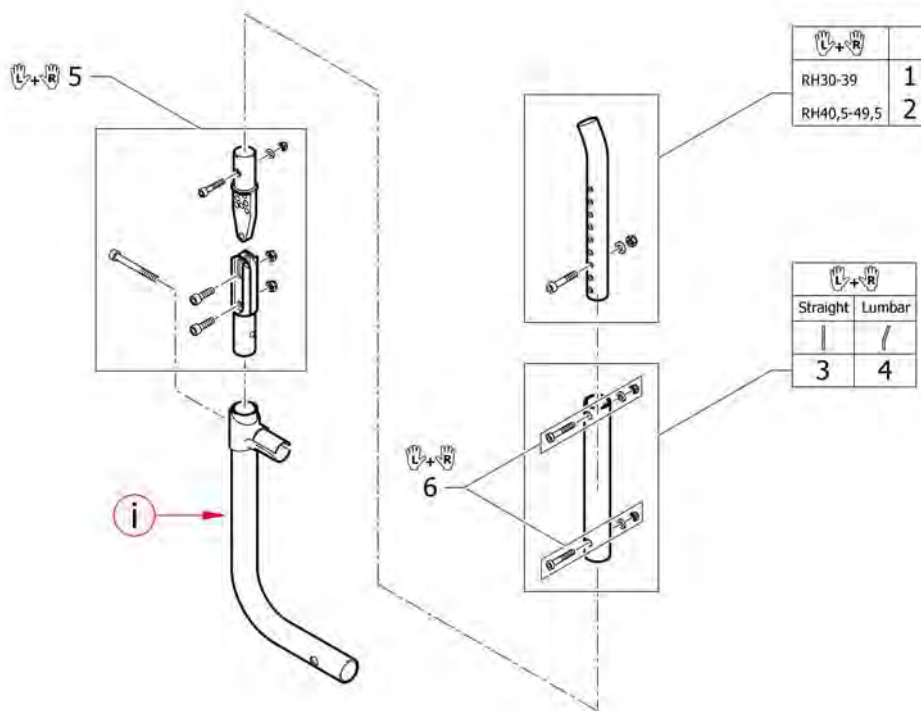


Pos.	Part number	Description	Valid from → until	Qty
i	-	A special rear frame is necessary. Please see chapter CHASSIS.		0
1	1518290	Backrest tubes S without push handles, pair		1
2	1518291	Backrest tubes M without push handles, pair		1
3	1438209	Tube kit medium to angle adjustable back		1
4	1518296	Tube kit lumbar to angle adjustable back		1
5	1438208	Backrest angle kit		1
6	1509134	Screw kit for telescopic tube		1

## Push handles for backrest foldable

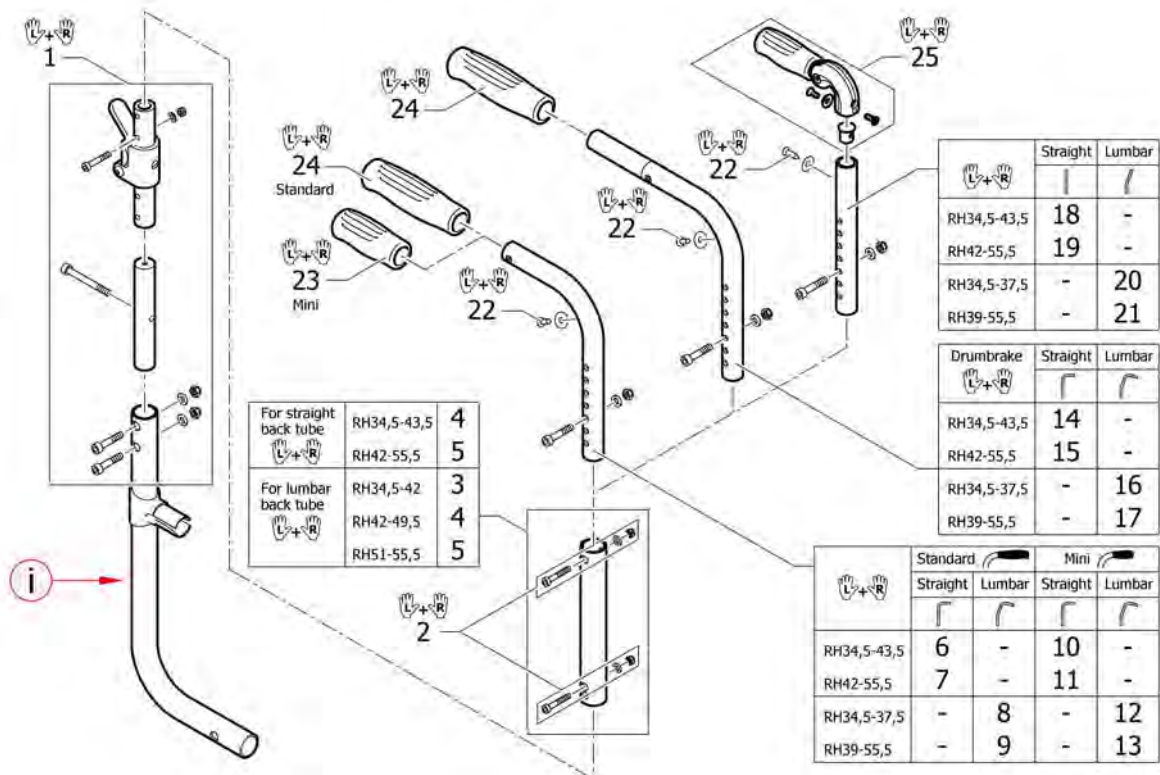


Pos.	Part number	Description	Valid from → until	Qty
i	-	A special rear frame is necessary. Please see chapter CHASSIS.		0
1	1438207	Hinge kit to backrest foldable, pair		1
2	1509134	Screw kit for telescopic tube		1
3	1439459	Tube kit short to angle adjustable back		1
4	1438209	Tube kit medium to angle adjustable back		1
5	1439460	Tube kit long to angle adjustable back		1
6	1507739	Push handles standard, XS, pair		1
7	1507740	Push handles standard, S, pair		1
8	1507741	Push handles standard lumbar XS, pair		1
9	1507742	Push handles standard lumbar S, pair		1
10	1525342	Push handles mini, XS, pair		1
11	1525343	Push handles mini, S, pair		1
12	1525344	Push handles mini lumbar XS, pair		1
13	1525345	Push handles mini lumbar S, pair		1
14	1507743	Push handles XS to drum brake, pair		1
15	1507744	Push handles S to drum brake, pair		1
16	1507745	Push handles lumbar XS to drum brake, pair		1
17	1507746	Push handles lumbar S to drum brake, pair		1
18	1507735	Backrest tube straight XS to foldable push handles, pair		1
19	1507736	Backrest tube straight S to foldable push handles, pair		1
20	1507737	Backrest tube lumbar XS to foldable push handles, pair		1
21	1507738	Backrest tube lumbar S to foldable push handles, pair		1
22	1432083	Fixation kit for backrest band		1
23	1525126	Handle black, mini, pair		1
24	1507734	Handle black, pair		1
25	1507733	Push handles foldable, pair		1

## Push handles for backrest foldable + angle adjustable

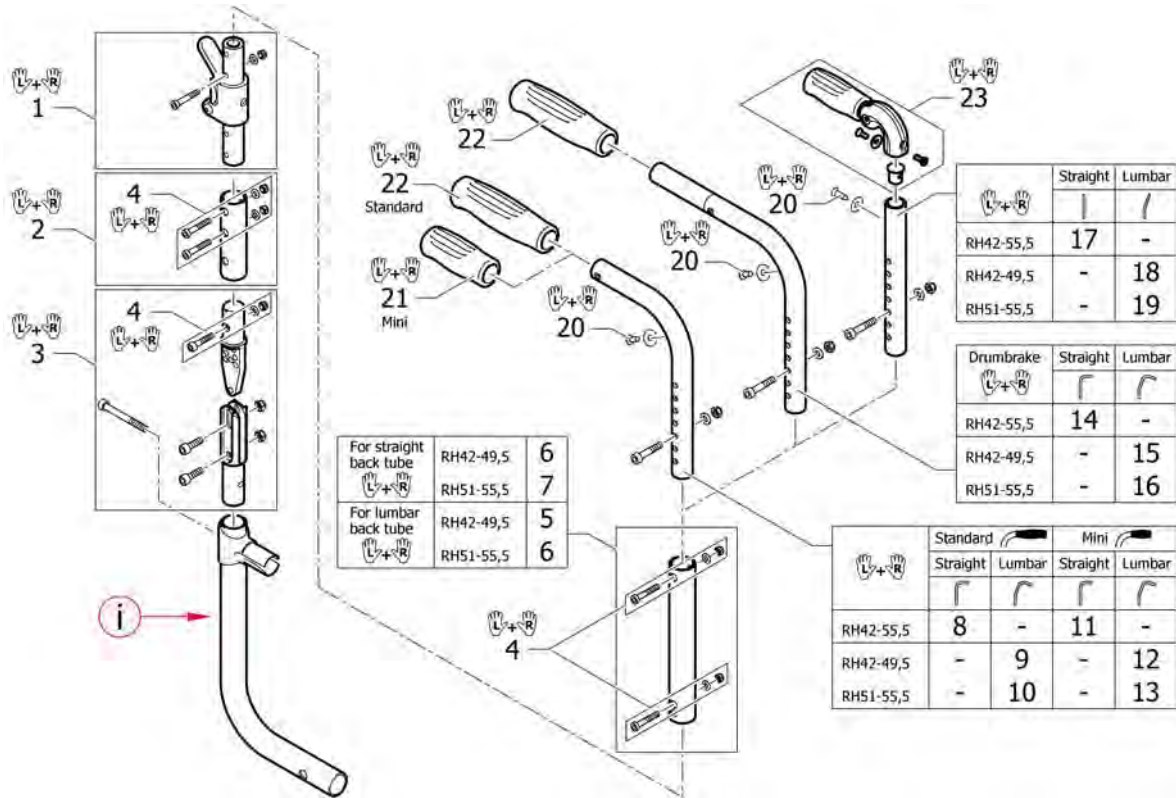


Pos.	Part number	Description	Valid from → until	Qty
i	-	A special rear frame is necessary. Please see chapter CHASSIS.		0
1	1507749	Hinge kit to backrest foldable+angle adjustable, pair		1
2	1507748	Connection tube to backrest foldable+angle adjustable, pair		1
3	1438208	Backrest angle kit		1
4	1509135	Screw kit for connection tube		1
5	1439459	Tube kit short to angle adjustable back		1
6	1438209	Tube kit medium to angle adjustable back		1
7	1439460	Tube kit long to angle adjustable back		1
8	1507739	Push handles standard, XS, pair		1
9	1507741	Push handles standard lumbar XS, pair		1
10	1507742	Push handles standard lumbar S, pair		1
11	1525342	Push handles mini, XS, pair		1
12	1525344	Push handles mini lumbar XS, pair		1
13	1525345	Push handles mini lumbar S, pair		1
14	1507743	Push handles XS to drum brake, pair		1
15	1507745	Push handles lumbar XS to drum brake, pair		1
16	1507746	Push handles lumbar S to drum brake, pair		1
17	1507735	Backrest tube straight XS to foldable push handles, pair		1
18	1507737	Backrest tube lumbar XS to foldable push handles, pair		1
19	1507738	Backrest tube lumbar S to foldable push handles, pair		1
20	1432083	Fixation kit for backrest band		1
21	1525126	Handle black, mini, pair		1
22	1507734	Handle black, pair		1
23	1507733	Push handles foldable, pair		1

## Push handles height adjustable

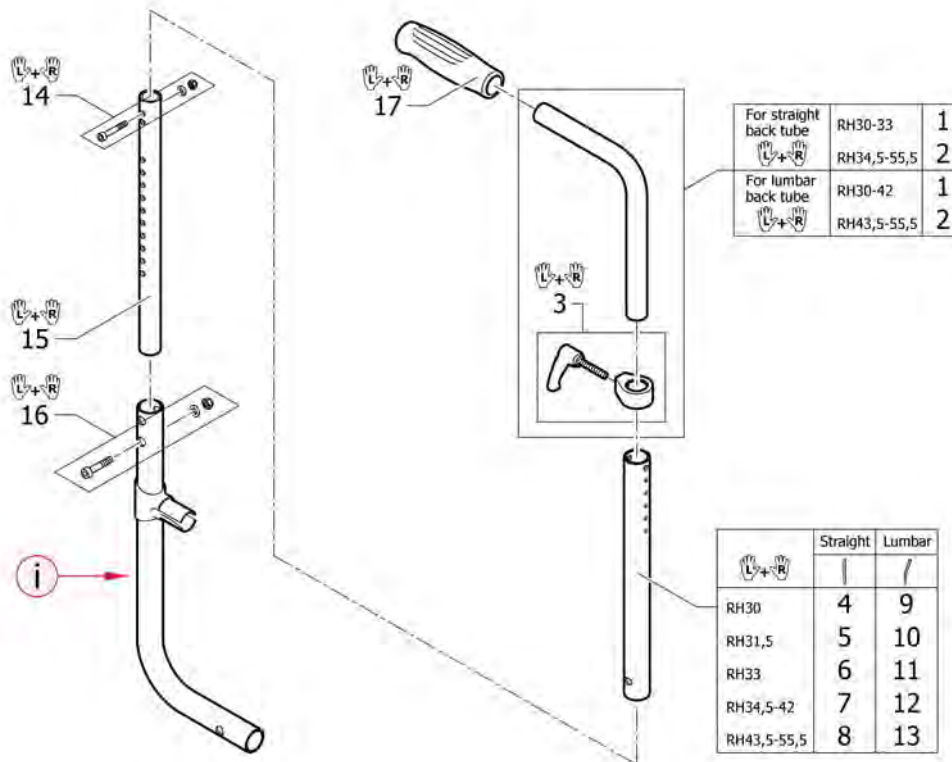

Pos.	Part number	Description	Valid from → until	Qty
i	-	A special rear frame is necessary. Please see chapter CHASSIS.		0
1	1438203	Height adjustable, integrated push handles standard, kit short UL/C04		1
2	1438204	Height adjustable, integrated push handles standard, kit long UL/C04		1
3	1549985	Adjusting lever for height adjustable, integrated push handles, pair		1
4	1507750	Backrest tube L.235mm straight to integrated push handles, pair		1
5	1507751	Backrest tube L.250mm straight to integrated push handles, pair		1
6	1507752	Backrest tube L.265mm straight to integrated push handles, pair		1
7	1507753	Backrest tube L.280mm straight to integrated push handles, pair		1
8	1509103	Backrest tube L.355mm straight to integrated push handles, pair		1
9	1509114	Backrest tube L.235mm lumbar to integrated push handles, pair		1
10	1509115	Backrest tube L.250mm lumbar to integrated push handles, pair		1
11	1509116	Backrest tube L.265mm lumbar to integrated push handles, pair		1
12	1509117	Backrest tube L.280mm lumbar to integrated push handles, pair		1
13	1509122	Backrest tube L.355mm lumbar to integrated push handles, pair		1
14	1509136	Screw kit for backrest tube		1
15	1507747	Telescopic tube frame-backrest, pair		1
16	1439279	Fixation kit to backrest tube		1
17	1507734	Handle black, pair		1

## Push handles height adjustable for backrest angle adjustable

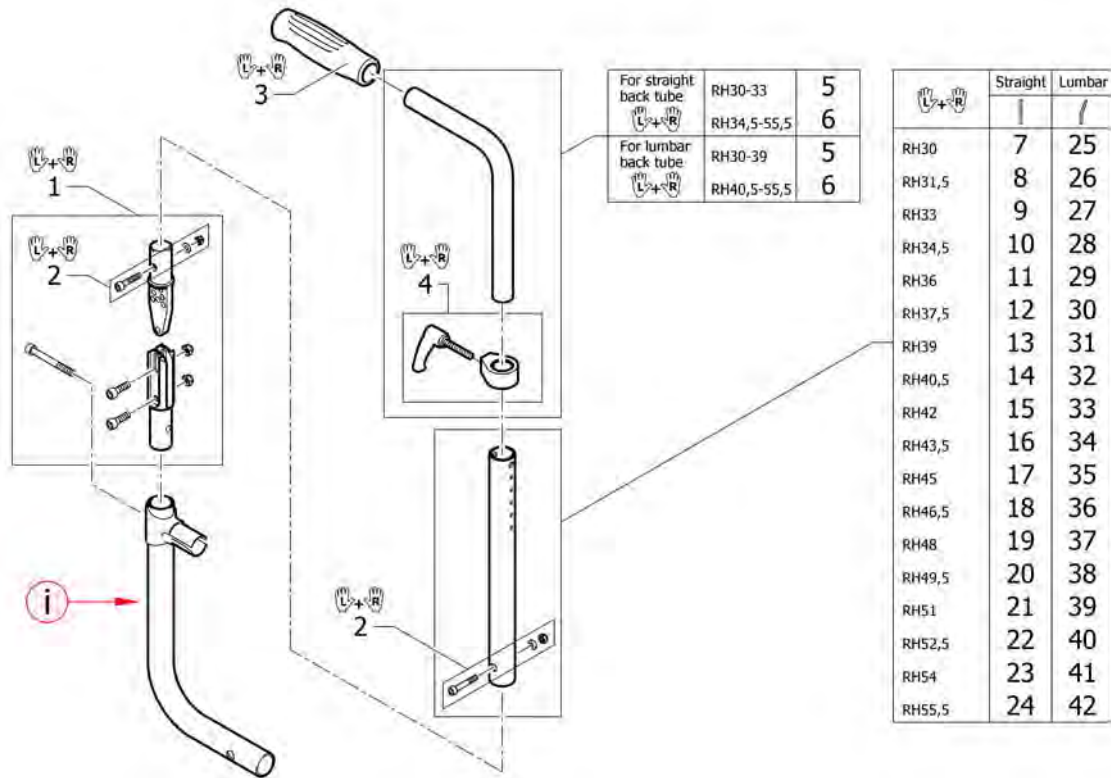


Pos.	Part number	Description	Valid from → until	Qty
i	-	A special rear frame is necessary. Please see chapter CHASSIS.		0
1	1438208	Backrest angle kit		1
2	1509136	Screw kit for backrest tube		1
3	1507734	Handle black, pair		1
4	1549985	Adjusting lever for height adjustable, integrated push handles, pair		1
5	1438203	Height adjustable, integrated push handles standard, kit short UL/C04		1
6	1438204	Height adjustable, integrated push handles standard, kit long UL/C04		1
7	1507751	Backrest tube L.250mm straight to integrated push handles, pair		1
8	1507752	Backrest tube L.265mm straight to integrated push handles, pair		1
9	1507753	Backrest tube L.280mm straight to integrated push handles, pair		1
10	1507862	Backrest tube L.295mm straight to integrated push handles, pair		1
11	1508438	Backrest tube L.310mm straight to integrated push handles, pair		1
12	1508466	Backrest tube L.325mm straight to integrated push handles, pair		1
13	1508498	Backrest tube L.340mm straight to integrated push handles, pair		1
14	1509103	Backrest tube L.355mm straight to integrated push handles, pair		1
15	1509104	Backrest tube L.370mm straight to integrated push handles, pair		1
16	1509105	Backrest tube L.385mm straight to integrated push handles, pair		1
17	1509106	Backrest tube L.400mm straight to integrated push handles, pair		1
18	1509107	Backrest tube L.415mm straight to integrated push handles, pair		1
19	1509108	Backrest tube L.430mm straight to integrated push handles, pair		1
20	1509109	Backrest tube L.445mm straight to integrated push handles, pair		1
21	1509110	Backrest tube L.460mm straight to integrated push handles, pair		1
22	1509111	Backrest tube L.475mm straight to integrated push handles, pair		1
23	1509112	Backrest tube L.490mm straight to integrated push handles, pair		1
24	1509113	Backrest tube L.505mm straight to integrated push handles, pair		1
25	1509115	Backrest tube L.250mm lumbar to integrated push handles, pair		1
26	1509116	Backrest tube L.265mm lumbar to integrated push handles, pair		1
27	1509117	Backrest tube L.280mm lumbar to integrated push handles, pair		1
28	1509118	Backrest tube L.295mm lumbar to integrated push handles, pair		1
29	1509119	Backrest tube L.310mm lumbar to integrated push handles, pair		1
30	1509120	Backrest tube L.325mm lumbar to integrated push handles, pair		1



## Push handles height adjustable for backrest angle adjustable

Pos.	Part number	Description	Valid from → until	Qty
31	1509121	Backrest tube L.340mm lumbar to integrated push handles, pair		1
32	1509122	Backrest tube L.355mm lumbar to integrated push handles, pair		1
33	1509123	Backrest tube L.370mm lumbar to integrated push handles, pair		1
34	1509124	Backrest tube L.385mm lumbar to integrated push handles, pair		1
35	1509125	Backrest tube L.400mm lumbar to integrated push handles, pair		1
36	1509126	Backrest tube L.415mm lumbar to integrated push handles, pair		1
37	1509127	Backrest tube L.430mm lumbar to integrated push handles, pair		1
38	1509128	Backrest tube L.445mm lumbar to integrated push handles, pair		1
39	1509129	Backrest tube L.460mm lumbar to integrated push handles, pair		1
40	1509130	Backrest tube L.475mm lumbar to integrated push handles, pair		1
41	1509131	Backrest tube L.490mm lumbar to integrated push handles, pair		1
42	1509132	Backrest tube L.505mm lumbar to integrated push handles, pair		1

## Push handles height adjustable for backrest foldable

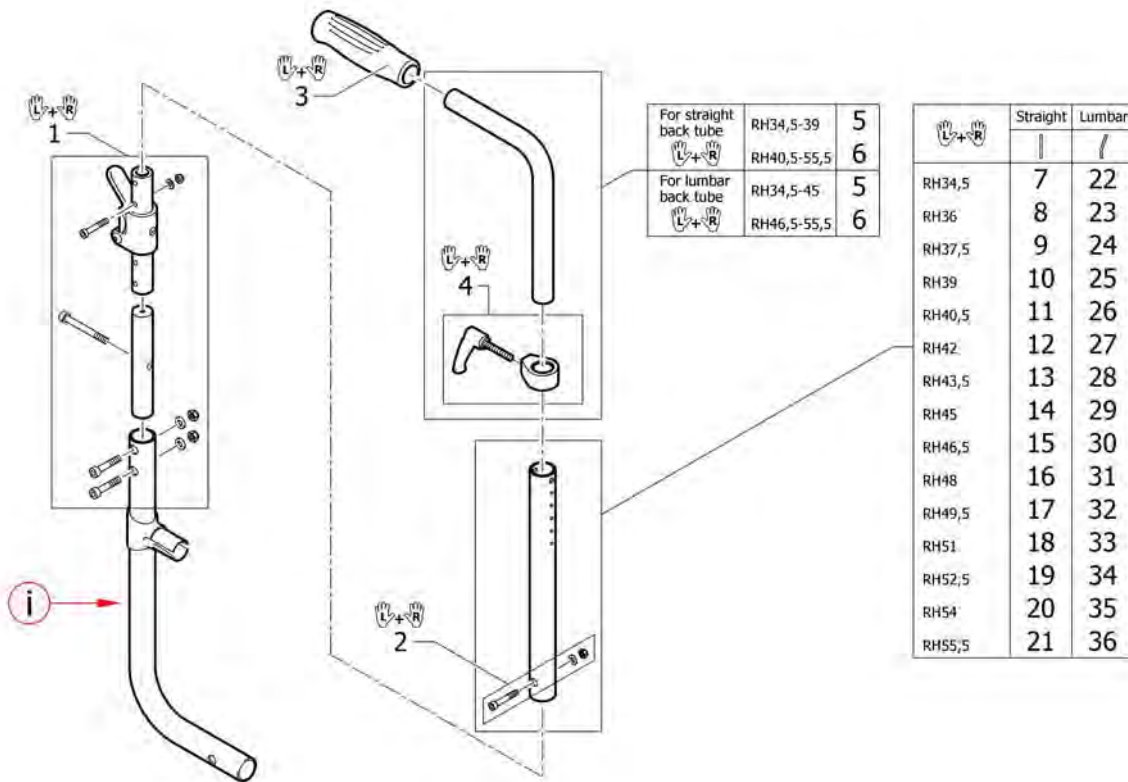


Pos.	Part number	Description	Valid from → until	Qty
i	-	A special rear frame is necessary. Please see chapter CHASSIS.		0
1	1438207	Hinge kit to backrest foldable, pair		1
2	1509136	Screw kit for backrest tube		1
3	1507734	Handle black, pair		1
4	1549985	Adjusting lever for height adjustable, integrated push handles, pair		1
5	1438203	Height adjustable, integrated push handles standard, kit short UL/C04		1
6	1438204	Height adjustable, integrated push handles standard, kit long UL/C04		1
7	1507750	Backrest tube L.235mm straight to integrated push handles, pair		1
8	1507751	Backrest tube L.250mm straight to integrated push handles, pair		1
9	1507752	Backrest tube L.265mm straight to integrated push handles, pair		1
10	1507753	Backrest tube L.280mm straight to integrated push handles, pair		1
11	1507862	Backrest tube L.295mm straight to integrated push handles, pair		1
12	1508438	Backrest tube L.310mm straight to integrated push handles, pair		1
13	1508466	Backrest tube L.325mm straight to integrated push handles, pair		1
14	1508498	Backrest tube L.340mm straight to integrated push handles, pair		1
15	1509103	Backrest tube L.355mm straight to integrated push handles, pair		1
16	1509104	Backrest tube L.370mm straight to integrated push handles, pair		1
17	1509105	Backrest tube L.385mm straight to integrated push handles, pair		1
18	1509106	Backrest tube L.400mm straight to integrated push handles, pair		1
19	1509107	Backrest tube L.415mm straight to integrated push handles, pair		1
20	1509108	Backrest tube L.430mm straight to integrated push handles, pair		1
21	1509109	Backrest tube L.445mm straight to integrated push handles, pair		1
22	1509114	Backrest tube L.235mm lumbar to integrated push handles, pair		1
23	1509115	Backrest tube L.250mm lumbar to integrated push handles, pair		1
24	1509116	Backrest tube L.265mm lumbar to integrated push handles, pair		1
25	1509117	Backrest tube L.280mm lumbar to integrated push handles, pair		1
26	1509118	Backrest tube L.295mm lumbar to integrated push handles, pair		1
27	1509119	Backrest tube L.310mm lumbar to integrated push handles, pair		1
28	1509120	Backrest tube L.325mm lumbar to integrated push handles, pair		1
29	1509121	Backrest tube L.340mm lumbar to integrated push handles, pair		1
30	1509122	Backrest tube L.355mm lumbar to integrated push handles, pair		1

**Push handles height adjustable for backrest foldable**


<b>Pos.</b>	<b>Part number</b>	<b>Description</b>	<b>Valid from → until</b>	<b>Qty</b>
31	1509123	Backrest tube L.370mm lumbar to integrated push handles, pair		1
32	1509124	Backrest tube L.385mm lumbar to integrated push handles, pair		1
33	1509125	Backrest tube L.400mm lumbar to integrated push handles, pair		1
34	1509126	Backrest tube L.415mm lumbar to integrated push handles, pair		1
35	1509127	Backrest tube L.430mm lumbar to integrated push handles, pair		1
36	1509128	Backrest tube L.445mm lumbar to integrated push handles, pair		1

## Push handles height adjustable for backrest foldable + angle adjustable

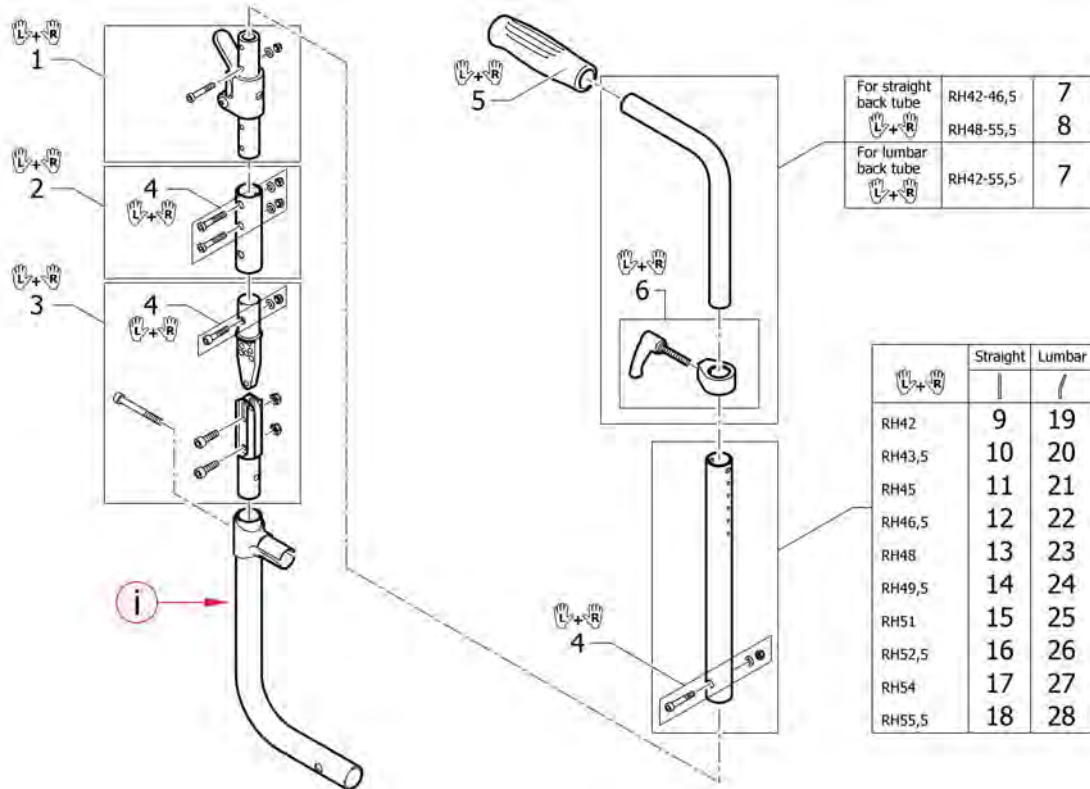

Pos.	Part number	Description	Valid from → until	Qty
i	-	A special rear frame is necessary. Please see chapter CHASSIS.		0
1	1507749	Hinge kit to backrest foldable+angle adjustable, pair		1
2	1507748	Connection tube to backrest foldable+angle adjustable, pair		1
3	1438208	Backrest angle kit		1
4	1509135	Screw kit for connection tube		1
5	1507734	Handle black, pair		1
6	1549985	Adjusting lever for height adjustable, integrated push handles, pair		1
7	1438203	Height adjustable, integrated push handles standard, kit short UL/C04		1
8	1438204	Height adjustable, integrated push handles standard, kit long UL/C04		1
9	1507750	Backrest tube L.235mm straight to integrated push handles, pair		1
10	1507751	Backrest tube L.250mm straight to integrated push handles, pair		1
11	1507752	Backrest tube L.265mm straight to integrated push handles, pair		1
12	1507753	Backrest tube L.280mm straight to integrated push handles, pair		1
13	1507862	Backrest tube L.295mm straight to integrated push handles, pair		1
14	1508438	Backrest tube L.310mm straight to integrated push handles, pair		1
15	1508466	Backrest tube L.325mm straight to integrated push handles, pair		1
16	1508498	Backrest tube L.340mm straight to integrated push handles, pair		1
17	1509103	Backrest tube L.355mm straight to integrated push handles, pair		1
18	1509104	Backrest tube L.370mm straight to integrated push handles, pair		1
19	1509114	Backrest tube L.235mm lumbar to integrated push handles, pair		1
20	1509115	Backrest tube L.250mm lumbar to integrated push handles, pair		1
21	1509116	Backrest tube L.265mm lumbar to integrated push handles, pair		1
22	1509117	Backrest tube L.280mm lumbar to integrated push handles, pair		1
23	1509118	Backrest tube L.295mm lumbar to integrated push handles, pair		1
24	1509119	Backrest tube L.310mm lumbar to integrated push handles, pair		1
25	1509120	Backrest tube L.325mm lumbar to integrated push handles, pair		1
26	1509121	Backrest tube L.340mm lumbar to integrated push handles, pair		1
27	1509122	Backrest tube L.355mm lumbar to integrated push handles, pair		1
28	1509123	Backrest tube L.370mm lumbar to integrated push handles, pair		1


## Push handles, inclined

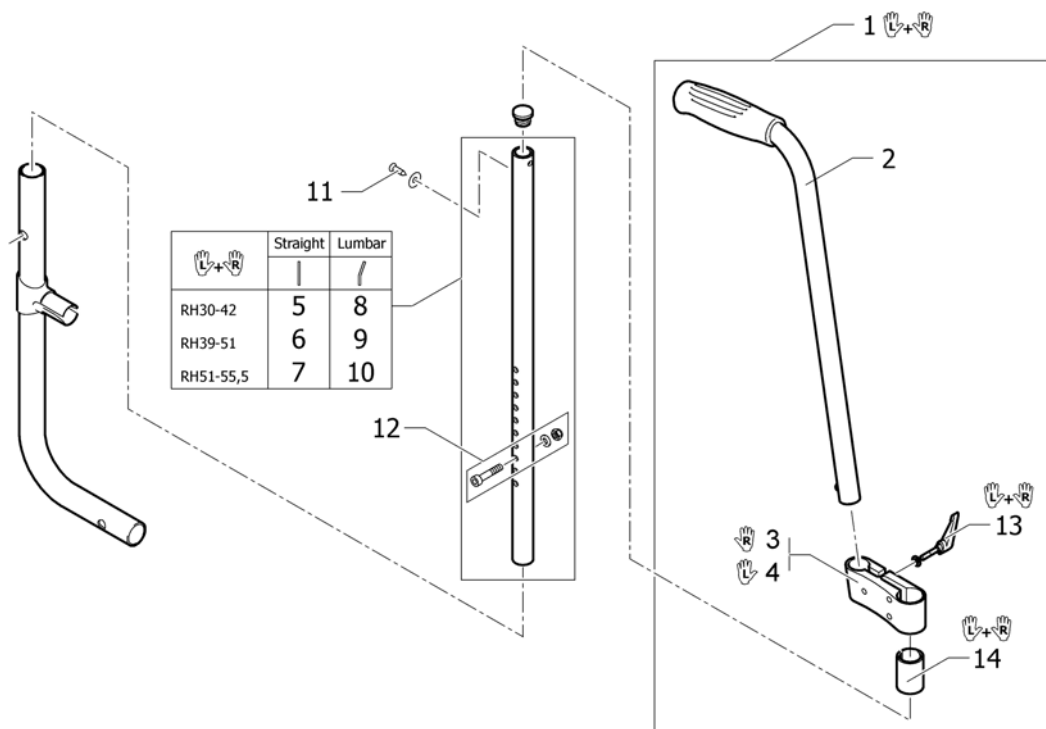

Pos.	Part number	Description	Valid from → until	Qty
1	1517876	Push handles, height adjustable, rear set, pair <i>Replaces 907.1157</i>		1
2	1438938	Push handle tube complete, rearset		1
3	1438939	Mounting fixture right complete, for rearset push handles		1
4	1438940	Mounting fixture left complete, for rearset push handles		1
5	1507715	Backrest tube straight short to foldable push handles, pair		1
6	1507716	Backrest tube straight medium to foldable push handles, pair		1
7	1507717	Backrest tube straight long to foldable push handles, pair		1
8	1507718	Backrest tube lumbar short to foldable push handles, pair		1
9	1507719	Backrest tube lumbar medium to foldable push handles, pair		1
10	1507720	Backrest tube lumbar long to foldable push handles, pair		1
11	1432083	Fixation kit for backrest band		1
12	1439279	Fixation kit to backrest tube		1
13	1529799	Adjusting lever, pair		1
14	1581293	Sleeves for mounting fixture, for rearset push handles, pair		1



## Backrest stabilization bar

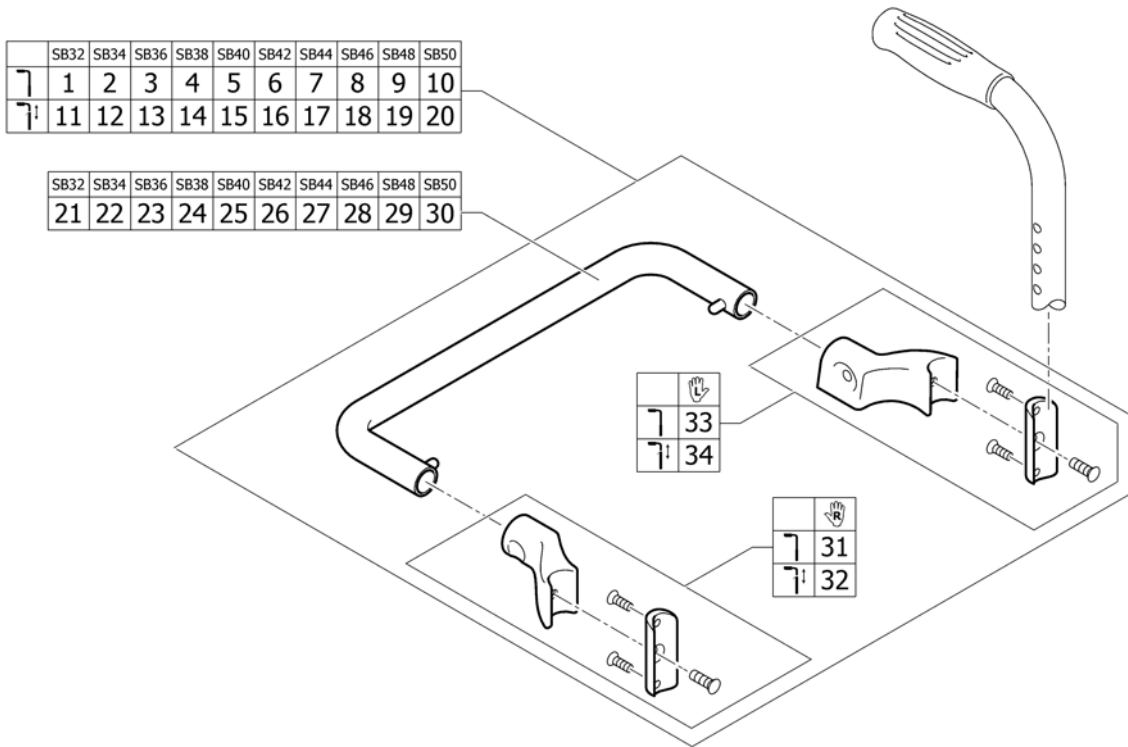


Pos.	Part number	Description	Valid from → until	Qty
1	1517978	Stabilizing bar complete SB32 for backrest tubes Ø20		1
2	1517979	Stabilizing bar complete SB34 for backrest tubes Ø20		1
3	1517980	Stabilizing bar complete SB36 for backrest tubes Ø20		1
4	1517981	Stabilizing bar complete SB38 for backrest tubes Ø20		1
5	1517982	Stabilizing bar complete SB40 for backrest tubes Ø20		1
6	1517983	Stabilizing bar complete SB42 for backrest tubes Ø20		1
7	1517984	Stabilizing bar complete SB44 for backrest tubes Ø20		1
8	1517985	Stabilizing bar complete SB46 for backrest tubes Ø20		1
9	1517986	Stabilizing bar complete SB48 for backrest tubes Ø20		1
10	1517987	Stabilizing bar complete SB50 for backrest tubes Ø20		1
11	1517988	Stabilizing bar complete SB32 for backrest tubes Ø25		1
12	1517989	Stabilizing bar complete SB34 for backrest tubes Ø25		1
13	1517990	Stabilizing bar complete SB36 for backrest tubes Ø25		1
14	1517991	Stabilizing bar complete SB38 for backrest tubes Ø25		1
15	1517992	Stabilizing bar complete SB40 for backrest tubes Ø25		1
16	1517993	Stabilizing bar complete SB42 for backrest tubes Ø25		1
17	1517994	Stabilizing bar complete SB44 for backrest tubes Ø25		1
18	1517995	Stabilizing bar complete SB46 for backrest tubes Ø25		1
19	1517996	Stabilizing bar complete SB48 for backrest tubes Ø25		1
20	1517997	Stabilizing bar complete SB50 for backrest tubes Ø25		1
21	1517998	Stabilizing bar SB32		1
22	1517999	Stabilizing bar SB34		1
23	1518000	Stabilizing bar SB36		1
24	1518001	Stabilizing bar SB38		1
25	1518002	Stabilizing bar SB40		1
26	1518003	Stabilizing bar SB42		1
27	1518004	Stabilizing bar SB44		1
28	1518005	Stabilizing bar SB46		1
29	1518006	Stabilizing bar SB48		1
30	1518007	Stabilizing bar SB50		1
31	1518009	Fixation kit to stabilizing bar, right, for backrest tubes Ø20		1

**Backrest stabilization bar**

Pos.	Part number	Description	Valid from → until	Qty
32	1518088	Fixation kit to stabilizing bar, right, for backrest tubes Ø25		1
33	1518008	Fixation kit to stabilizing bar, left, for backrest tubes Ø20		1
34	1518010	Fixation kit to stabilizing bar, left, for backrest tubes Ø25		1

## Back cover, standard

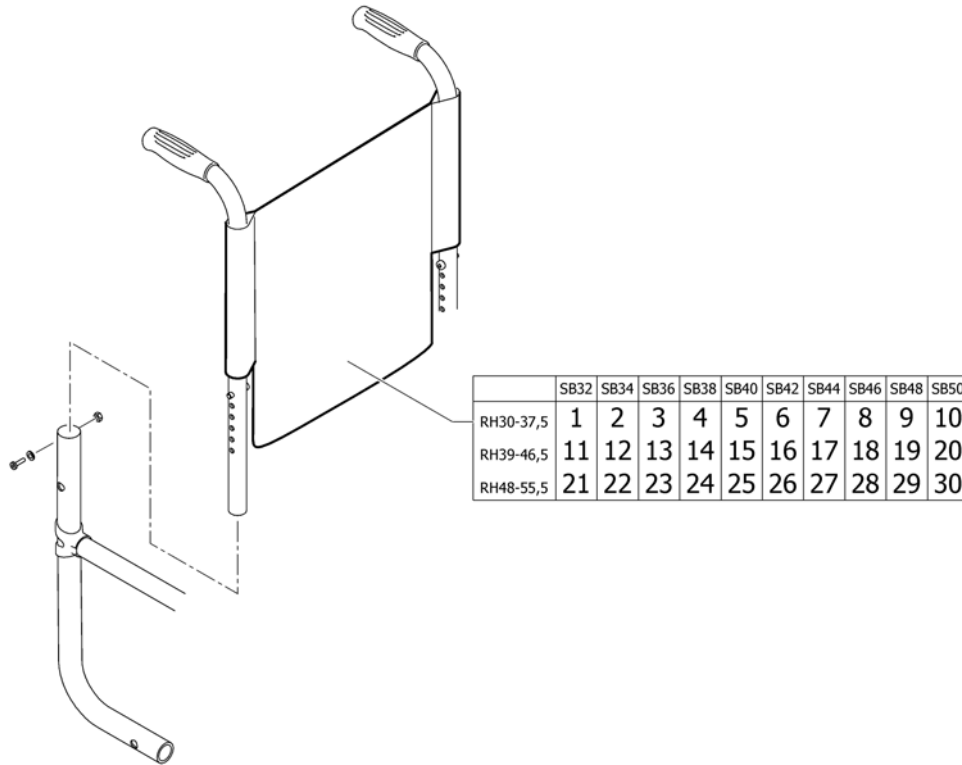


Pos.	Part number	Description	Valid from → until	Qty
1	1517552	Backrest cover SB32 short <i>Replaces 1438180</i>		1
2	1517553	Backrest cover SB34 short <i>Replaces 1438181</i>		1
3	1517554	Backrest cover SB36 short <i>Replaces 1438182</i>		1
4	1517555	Backrest cover SB38 short <i>Replaces 1438183</i>		1
5	1517556	Backrest cover SB40 short <i>Replaces 1438184</i>		1
6	1517557	Backrest cover SB42 short <i>Replaces 1438185</i>		1
7	1517558	Backrest cover SB44 short <i>Replaces 1438186</i>		1
8	1517559	Backrest cover SB46 short <i>Replaces 1438187</i>		1
9	1517560	Backrest cover SB48 short <i>Replaces 1438188</i>		1
10	1517561	Backrest cover SB50 short <i>Replaces 1438189</i>		1
11	1517562	Backrest cover SB32 medium <i>Replaces 1438418</i>		1
12	1517563	Backrest cover SB34 medium <i>Replaces 1438419</i>		1
13	1517564	Backrest cover SB36 medium <i>Replaces 1438420</i>		1
14	1517565	Backrest cover SB38 medium <i>Replaces 1438421</i>		1
15	1517566	Backrest cover SB40 medium <i>Replaces 1438422</i>		1
16	1517567	Backrest cover SB42 medium <i>Replaces 1438423</i>		1
17	1517568	Backrest cover SB44 medium <i>Replaces 1438424</i>		1
18	1517569	Backrest cover SB46 medium <i>Replaces 1438425</i>		1


**Back cover, standard**

Pos.	Part number	Description	Valid from → until	Qty
19	1517570	Backrest cover SB48 medium <i>Replaces 1438426</i>		1
20	1517571	Backrest cover SB50 medium <i>Replaces 1438427</i>		1
21	1517572	Backrest cover SB32 long <i>Replaces 1438190</i>		1
22	1517573	Backrest cover SB34 long <i>Replaces 1438191</i>		1
23	1517574	Backrest cover SB36 long <i>Replaces 1438192</i>		1
24	1517575	Backrest cover SB38 long <i>Replaces 1438193</i>		1
25	1517576	Backrest cover SB40 long <i>Replaces 1438194</i>		1
26	1517577	Backrest cover SB42 long <i>Replaces 1438195</i>		1
27	1517578	Backrest cover SB44 long <i>Replaces 1438196</i>		1
28	1517579	Backrest cover SB46 long <i>Replaces 1438197</i>		1
29	1517580	Backrest cover SB48 long <i>Replaces 1438198</i>		1
30	1517581	Backrest cover SB50 long <i>Replaces 1438199</i>		1

## Bands to adjustable backrest

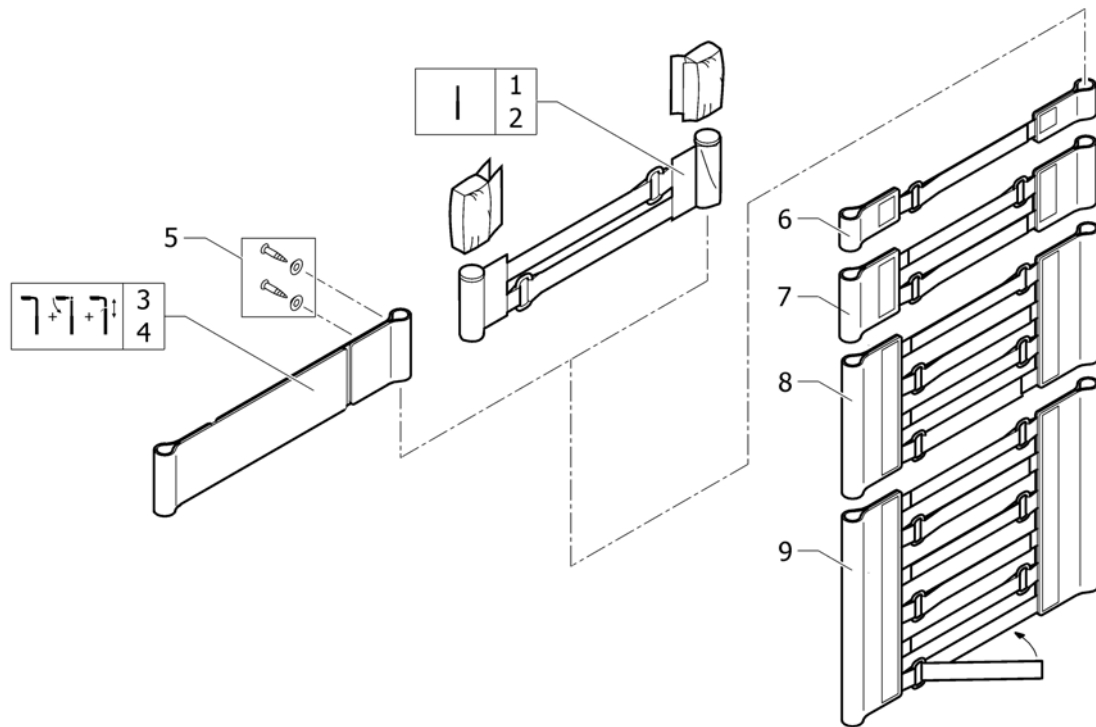

Pos.	Part number	Description	Valid from → until	Qty
1	1513172	Velcro end part, 5 cm <i>Replaces 1416975</i>		1
2	1513173	Velcro end part, 10cm <i>Replaces 1416974</i>		1
3	1513170	Backrest band 2 parts, width: 5cm <i>Replaces 709.1009</i>		1
4	1513171	Backrest band 2 parts, width: 10cm <i>Replaces 709.1018</i>		1
5	1432083	Fixation kit for backrest band		1
6	1513167	Velcro adjustment part with 1 strap, pair <i>Replaces 1439291</i>		1
7	1513168	Velcro adjustment strap with 2 straps, pair <i>Replaces 707.1088</i>		1
8	1513169	Velcro adjustment strap with 4 straps, pair <i>Replaces 707.1087</i>		1
9	1517520	Velcro adjustment strap with 6 straps, pair <i>Replaces 707.1075</i>		1

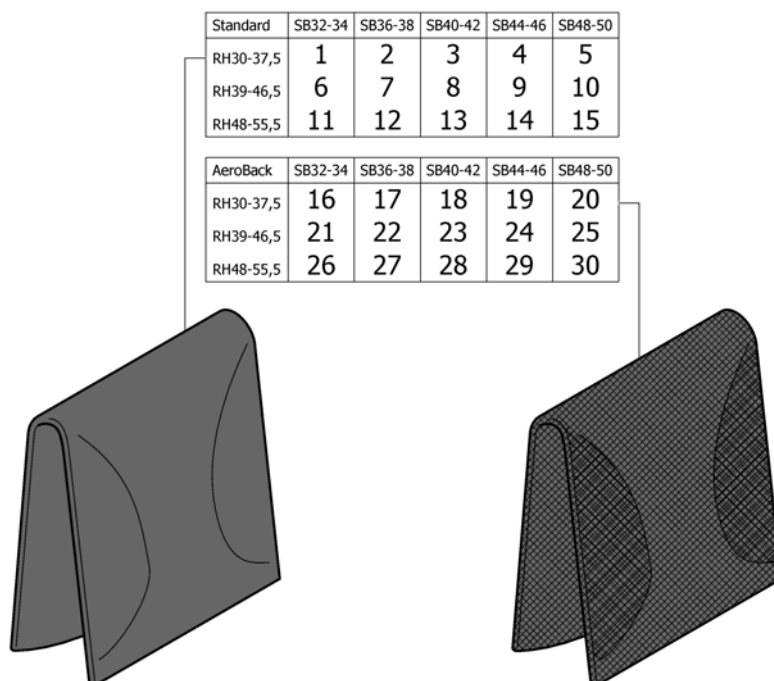
**Cover to adjustable backrest**

Pos.	Part number	Description	Valid from → until	Qty
1	1517522	Cover to adjustable backrest SB32-34 short <i>Replaces 1438161</i>		1
2	1517523	Cover to adjustable backrest SB36-38 short <i>Replaces 1438163</i>		1
3	1517524	Cover to adjustable backrest SB40-42 short <i>Replaces 1438165</i>		1
4	1517525	Cover to adjustable backrest SB44-46 short <i>Replaces 1438167</i>		1
5	1517526	Cover to adjustable backrest SB48-50 short <i>Replaces 1438169</i>		1
6	1517527	Cover to adjustable backrest SB32-34 medium <i>Replaces 1438378</i>		1
7	1517528	Cover to adjustable backrest SB36-38 medium <i>Replaces 1438380</i>		1
8	1517529	Cover to adjustable backrest SB40-42 medium <i>Replaces 1438382</i>		1
9	1517530	Cover to adjustable backrest SB44-46 medium <i>Replaces 1438384</i>		1
10	1517531	Cover to adjustable backrest SB48-50 medium <i>Replaces 1438386</i>		1
11	1517532	Cover to adjustable backrest SB32-34 long <i>Replaces 1438171</i>		1
12	1517533	Cover to adjustable backrest SB36-38 long <i>Replaces 1438173</i>		1
13	1517534	Cover to adjustable backrest SB40-42 long <i>Replaces 1438175</i>		1
14	1517535	Cover to adjustable backrest SB44-46 long <i>Replaces 1438177</i>		1
15	1517536	Cover to adjustable backrest SB48-50 long <i>Replaces 1438179</i>		1
16	1517537	Cover to adjustable backrest Aero SB32-34 short <i>Replaces 1438389</i>		1
17	1517538	Cover to adjustable backrest Aero SB36-38 short <i>Replaces 1438391</i>		1
18	1517539	Cover to adjustable backrest Aero SB40-42 short <i>Replaces 1438393</i>		1




**Cover to adjustable backrest**


Pos.	Part number	Description	Valid from → until	Qty
19	1517540	Cover to adjustable backrest Aero SB44-46 short <i>Replaces 1438395</i>		1
20	1517541	Cover to adjustable backrest Aero SB48-50 short <i>Replaces 1438397</i>		1
21	1517542	Cover to adjustable backrest Aero SB32-34 medium <i>Replaces 1438399</i>		1
22	1517543	Cover to adjustable backrest Aero SB36-38 medium <i>Replaces 1438401</i>		1
23	1517544	Cover to adjustable backrest Aero SB40-42 medium <i>Replaces 1438403</i>		1
24	1517545	Cover to adjustable backrest Aero SB44-46 medium <i>Replaces 1438405</i>		1
25	1517546	Cover to adjustable backrest Aero SB48-50 medium <i>Replaces 1438407</i>		1
26	1517547	Cover to adjustable backrest Aero SB32-34 long <i>Replaces 1438409</i>		1
27	1517548	Cover to adjustable backrest Aero SB36-38 long <i>Replaces 1438411</i>		1
28	1517549	Cover to adjustable backrest Aero SB40-42 long <i>Replaces 1438413</i>		1
29	1517550	Cover to adjustable backrest Aero SB44-46 long <i>Replaces 1438415</i>		1
30	1517551	Cover to adjustable backrest Aero SB48-50 long <i>Replaces 1438417</i>		1

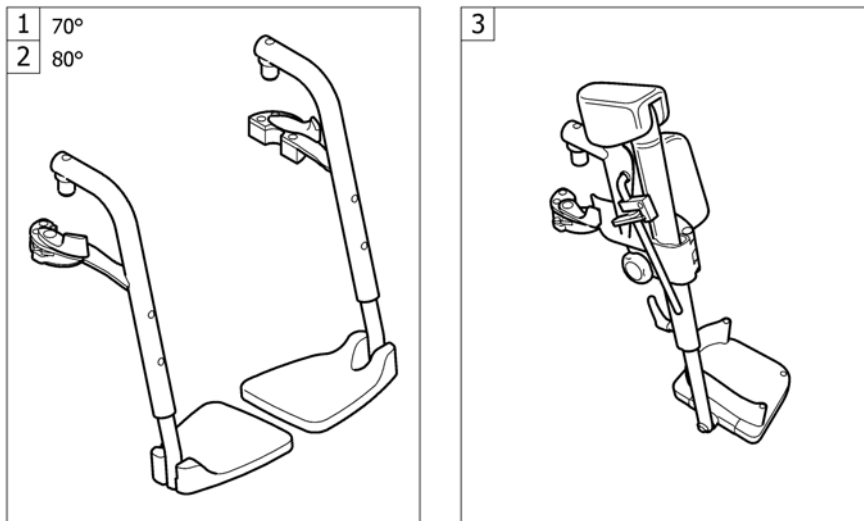


Pos.	Part number	Description	Valid from → until	Qty
1	-	Legrest hanger 70°	2004-01-01 → 2012-12-31	1
2	-	Legrest hanger 80°	2004-01-01 → 2012-12-31	1
3	-	Legrest hanger height adjustable	2004-01-01 → 2012-12-31	1

# Legrest hanger 70°

70° 	SB32	SB34	SB36	SB38	SB40	SB42	SB44	SB46	SB48	SB50
UL32-40	33	34	35	36	37	38	39	40	41	42
UL38-46	43	44	45	46	47	48	49	50	51	52
UL44-52	53	54	55	56	57	58	59	60	61	62

70° 	SB32	SB34	SB36	SB38	SB40	SB42	SB44	SB46	SB48	SB50
UL32-40	3	4	5	6	7	8	9	10	11	12
UL38-46	13	14	15	16	17	18	19	20	21	22
UL44-52	23	24	25	26	27	28	29	30	31	32

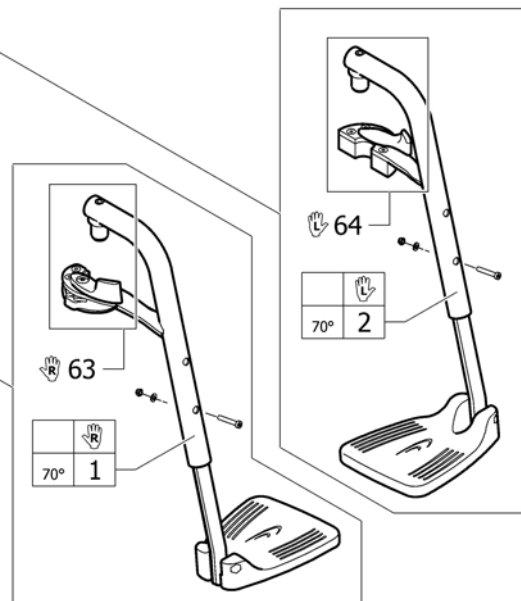

Pos.	Part number	Description	Valid from → until	Qty
1	1438226-0419	Swing Away 70°right, silver grey		1
1	1438226-0442	Swing Away 70°right, night shimmer	2009-01-01 →	1
1	1438226-3524	Swing Away 70°right, royal red		1
1	1438226-5969	Swing Away 70°right, royal blue		1
1	1438226-9005	Swing Away 70°right, black, glossy		1
1	1438226-9119	Swing Away 70°right, black, bronze		1
2	1438227-0419	Swing Away 70° left, silver grey		1
2	1438227-0442	Swing Away 70° left, night shimmer	2009-01-01 →	1
2	1438227-3524	Swing Away 70° left, royal red		1
2	1438227-5969	Swing Away 70° left, royal blue		1
2	1438227-9005	Swing Away 70° left, black, glossy		1
2	1438227-9119	Swing Away 70° left, black, bronze		1
3	1438531-0419	Swing Away complete 70° right SB32 UL short, silver grey		1
3	1438531-0442	Swing Away complete 70° right SB32 UL short, night shimmer	2009-01-01 →	1
3	1438531-3524	Swing Away complete 70° right SB32 UL short, royal red		1
3	1438531-5969	Swing Away complete 70° right SB32 UL short, royal blue		1
3	1438531-9005	Swing Away complete 70° right SB32 UL short, black, glossy		1
3	1438531-9119	Swing Away complete 70° right SB32 UL short, black, bronze		1
4	1438532-0419	Swing Away complete 70° right SB34 UL short, silver grey		1
4	1438532-0442	Swing Away complete 70° right SB34 UL short, night shimmer	2009-01-01 →	1
4	1438532-3524	Swing Away complete 70° right SB34 UL short, royal red		1
4	1438532-5969	Swing Away complete 70° right SB34 UL short, royal blue		1
4	1438532-9005	Swing Away complete 70° right SB34 UL short, black, glossy		1
4	1438532-9119	Swing Away complete 70° right SB34 UL short, black, bronze		1
5	1438533-0419	Swing Away complete 70° right SB36 UL short, silver grey		1
5	1438533-0442	Swing Away complete 70° right SB36 UL short, night shimmer	2009-01-01 →	1
5	1438533-3524	Swing Away complete 70° right SB36 UL short, royal red		1
5	1438533-5969	Swing Away complete 70° right SB36 UL short, royal blue		1
5	1438533-9005	Swing Away complete 70° right SB36 UL short, black, glossy		1
5	1438533-9119	Swing Away complete 70° right SB36 UL short, black, bronze		1
6	1438534-0419	Swing Away complete 70° right SB38 UL short, silver grey		1

## Legrest hanger 70°

Pos.	Part number	Description	Valid from → until	Qty
6	1438534-0442	Swing Away complete 70° right SB38 UL short, night shimmer	2009-01-01 →	1
6	1438534-3524	Swing Away complete 70° right SB38 UL short, royal red		1
6	1438534-5969	Swing Away complete 70° right SB38 UL short, royal blue		1
6	1438534-9005	Swing Away complete 70° right SB38 UL short, black, glossy		1
6	1438534-9119	Swing Away complete 70° right SB38 UL short, black, bronze		1
7	1438535-0419	Swing Away complete 70° right SB40 UL short, silver grey		1
7	1438535-0442	Swing Away complete 70° right SB40 UL short, night shimmer	2009-01-01 →	1
7	1438535-3524	Swing Away complete 70° right SB40 UL short, royal red		1
7	1438535-5969	Swing Away complete 70° right SB40 UL short, royal blue		1
7	1438535-9005	Swing Away complete 70° right SB40 UL short, black, glossy		1
7	1438535-9119	Swing Away complete 70° right SB40 UL short, black, bronze		1
8	1438536-0419	Swing Away complete 70° right SB42 UL short, silver grey		1
8	1438536-0442	Swing Away complete 70° right SB42 UL short, night shimmer	2009-01-01 →	1
8	1438536-3524	Swing Away complete 70° right SB42 UL short, royal red		1
8	1438536-5969	Swing Away complete 70° right SB42 UL short, royal blue		1
8	1438536-9005	Swing Away complete 70° right SB42 UL short, black, glossy		1
8	1438536-9119	Swing Away complete 70° right SB42 UL short, black, bronze		1
9	1438537-0419	Swing Away complete 70° right SB44 UL short, silver grey		1
9	1438537-0442	Swing Away complete 70° right SB44 UL short, night shimmer	2009-01-01 →	1
9	1438537-3524	Swing Away complete 70° right SB44 UL short, royal red		1
9	1438537-5969	Swing Away complete 70° right SB44 UL short, royal blue		1
9	1438537-9005	Swing Away complete 70° right SB44 UL short, black, glossy		1
9	1438537-9119	Swing Away complete 70° right SB44 UL short, black, bronze		1
10	1438538-0419	Swing Away complete 70° right SB46 UL short, silver grey		1
10	1438538-0442	Swing Away complete 70° right SB46 UL short, night shimmer	2009-01-01 →	1
10	1438538-3524	Swing Away complete 70° right SB46 UL short, royal red		1
10	1438538-5969	Swing Away complete 70° right SB46 UL short, royal blue		1
10	1438538-9005	Swing Away complete 70° right SB46 UL short, black, glossy		1
10	1438538-9119	Swing Away complete 70° right SB46 UL short, black, bronze		1
11	1438539-0419	Swing Away complete 70° right SB48 UL short, silver grey		1
11	1438539-0442	Swing Away complete 70° right SB48 UL short, night shimmer	2009-01-01 →	1
11	1438539-3524	Swing Away complete 70° right SB48 UL short, royal red		1
11	1438539-5969	Swing Away complete 70° right SB48 UL short, royal blue		1
11	1438539-9005	Swing Away complete 70° right SB48 UL short, black, glossy		1
11	1438539-9119	Swing Away complete 70° right SB48 UL short, black, bronze		1
12	1438540-0419	Swing Away complete 70° right SB50 UL short, silver grey		1
12	1438540-0442	Swing Away complete 70° right SB50 UL short, night shimmer	2009-01-01 →	1
12	1438540-3524	Swing Away complete 70° right SB50 UL short, royal red		1
12	1438540-5969	Swing Away complete 70° right SB50 UL short, royal blue		1
12	1438540-9005	Swing Away complete 70° right SB50 UL short, black, glossy		1
12	1438540-9119	Swing Away complete 70° right SB50 UL short, black, bronze		1
13	1439060-0419	Swing Away complete 70° right SB32 UL medium, silver grey		1
13	1439060-0442	Swing Away complete 70° right SB32 UL medium, night shimmer	2009-01-01 →	1
13	1439060-3524	Swing Away complete 70° right SB32 UL medium, royal red		1
13	1439060-5969	Swing Away complete 70° right SB32 UL medium, royal blue		1
13	1439060-9005	Swing Away complete 70° right SB32 UL medium, black, glossy		1
13	1439060-9119	Swing Away complete 70° right SB32 UL medium, black, bronze		1
14	1439061-0419	Swing Away complete 70° right SB34 UL medium, silver grey		1
14	1439061-0442	Swing Away complete 70° right SB34 UL medium, night shimmer	2009-01-01 →	1
14	1439061-3524	Swing Away complete 70° right SB34 UL medium, royal red		1
14	1439061-5969	Swing Away complete 70° right SB34 UL medium, royal blue		1
14	1439061-9005	Swing Away complete 70° right SB34 UL medium, black, glossy		1
14	1439061-9119	Swing Away complete 70° right SB34 UL medium, black, bronze		1
15	1439062-0419	Swing Away complete 70° right SB36 UL medium, silver grey		1
15	1439062-0442	Swing Away complete 70° right SB36 UL medium, night shimmer	2009-01-01 →	1

**Legrest hanger 70°**

Pos.	Part number	Description	Valid from → until	Qty
15	1439062-3524	Swing Away complete 70° right SB36 UL medium, royal red		1
15	1439062-5969	Swing Away complete 70° right SB36 UL medium, royal blue		1
15	1439062-9005	Swing Away complete 70° right SB36 UL medium, black, glossy		1
15	1439062-9119	Swing Away complete 70° right SB36 UL medium, black, bronze		1
16	1439063-0419	Swing Away complete 70° right SB38 UL medium, silver grey		1
16	1439063-0442	Swing Away complete 70° right SB38 UL medium, night shimmer	2009-01-01 →	1
16	1439063-3524	Swing Away complete 70° right SB38 UL medium, royal red		1
16	1439063-5969	Swing Away complete 70° right SB38 UL medium, royal blue		1
16	1439063-9005	Swing Away complete 70° right SB38 UL medium, black, glossy		1
16	1439063-9119	Swing Away complete 70° right SB38 UL medium, black, bronze		1
17	1439064-0419	Swing Away complete 70° right SB40 UL medium, silver grey		1
17	1439064-0442	Swing Away complete 70° right SB40 UL medium, night shimmer	2009-01-01 →	1
17	1439064-3524	Swing Away complete 70° right SB40 UL medium, royal red		1
17	1439064-5969	Swing Away complete 70° right SB40 UL medium, royal blue		1
17	1439064-9005	Swing Away complete 70° right SB40 UL medium, black, glossy		1
17	1439064-9119	Swing Away complete 70° right SB40 UL medium, black, bronze		1
18	1439065-0419	Swing Away complete 70° right SB42 UL medium, silver grey		1
18	1439065-0442	Swing Away complete 70° right SB42 UL medium, night shimmer	2009-01-01 →	1
18	1439065-3524	Swing Away complete 70° right SB42 UL medium, royal red		1
18	1439065-5969	Swing Away complete 70° right SB42 UL medium, royal blue		1
18	1439065-9005	Swing Away complete 70° right SB42 UL medium, black, glossy		1
18	1439065-9119	Swing Away complete 70° right SB42 UL medium, black, bronze		1
19	1439066-0419	Swing Away complete 70° right SB44 UL medium, silver grey		1
19	1439066-0442	Swing Away complete 70° right SB44 UL medium, night shimmer	2009-01-01 →	1
19	1439066-3524	Swing Away complete 70° right SB44 UL medium, royal red		1
19	1439066-5969	Swing Away complete 70° right SB44 UL medium, royal blue		1
19	1439066-9005	Swing Away complete 70° right SB44 UL medium, black, glossy		1
19	1439066-9119	Swing Away complete 70° right SB44 UL medium, black, bronze		1
20	1439067-0419	Swing Away complete 70° right SB46 UL medium, silver grey		1
20	1439067-0442	Swing Away complete 70° right SB46 UL medium, night shimmer	2009-01-01 →	1
20	1439067-3524	Swing Away complete 70° right SB46 UL medium, royal red		1
20	1439067-5969	Swing Away complete 70° right SB46 UL medium, royal blue		1
20	1439067-9005	Swing Away complete 70° right SB46 UL medium, black, glossy		1
20	1439067-9119	Swing Away complete 70° right SB46 UL medium, black, bronze		1
21	1439068-0419	Swing Away complete 70° right SB48 UL medium, silver grey		1
21	1439068-0442	Swing Away complete 70° right SB48 UL medium, night shimmer	2009-01-01 →	1
21	1439068-3524	Swing Away complete 70° right SB48 UL medium, royal red		1
21	1439068-5969	Swing Away complete 70° right SB48 UL medium, royal blue		1
21	1439068-9005	Swing Away complete 70° right SB48 UL medium, black, glossy		1
21	1439068-9119	Swing Away complete 70° right SB48 UL medium, black, bronze		1
22	1439069-0419	Swing Away complete 70° right SB50 UL medium, silver grey		1
22	1439069-0442	Swing Away complete 70° right SB50 UL medium, night shimmer	2009-01-01 →	1
22	1439069-3524	Swing Away complete 70° right SB50 UL medium, royal red		1
22	1439069-5969	Swing Away complete 70° right SB50 UL medium, royal blue		1
22	1439069-9005	Swing Away complete 70° right SB50 UL medium, black, glossy		1
22	1439069-9119	Swing Away complete 70° right SB50 UL medium, black, bronze		1
23	1438571-0419	Swing Away complete 70° right SB32 UL long, silver grey		1
23	1438571-0442	Swing Away complete 70° right SB32 UL long, night shimmer	2009-01-01 →	1
23	1438571-3524	Swing Away complete 70° right SB32 UL long, royal red		1
23	1438571-5969	Swing Away complete 70° right SB32 UL long, royal blue		1
23	1438571-9005	Swing Away complete 70° right SB32 UL long, black, glossy		1
23	1438571-9119	Swing Away complete 70° right SB32 UL long, black, bronze		1
24	1438572-0419	Swing Away complete 70° right SB34 UL long, silver grey		1
24	1438572-0442	Swing Away complete 70° right SB34 UL long, night shimmer	2009-01-01 →	1
24	1438572-3524	Swing Away complete 70° right SB34 UL long, royal red		1

## Legrest hanger 70°

Pos.	Part number	Description	Valid from → until	Qty
24	1438572-5969	Swing Away complete 70° right SB34 UL long, royal blue		1
24	1438572-9005	Swing Away complete 70° right SB34 UL long, black, glossy		1
24	1438572-9119	Swing Away complete 70° right SB34 UL long, black, bronze		1
25	1438573-0419	Swing Away complete 70° right SB36 UL long, silver grey		1
25	1438573-0442	Swing Away complete 70° right SB36 UL long, night shimmer	2009-01-01 →	1
25	1438573-3524	Swing Away complete 70° right SB36 UL long, royal red		1
25	1438573-5969	Swing Away complete 70° right SB36 UL long, royal blue		1
25	1438573-9005	Swing Away complete 70° right SB36 UL long, black, glossy		1
25	1438573-9119	Swing Away complete 70° right SB36 UL long, black, bronze		1
26	1438574-0419	Swing Away complete 70° right SB38 UL long, silver grey		1
26	1438574-0442	Swing Away complete 70° right SB38 UL long, night shimmer	2009-01-01 →	1
26	1438574-3524	Swing Away complete 70° right SB38 UL long, royal red		1
26	1438574-5969	Swing Away complete 70° right SB38 UL long, royal blue		1
26	1438574-9005	Swing Away complete 70° right SB38 UL long, black, glossy		1
26	1438574-9119	Swing Away complete 70° right SB38 UL long, black, bronze		1
27	1438575-0419	Swing Away complete 70° right SB40 UL long, silver grey		1
27	1438575-0442	Swing Away complete 70° right SB40 UL long, night shimmer	2009-01-01 →	1
27	1438575-3524	Swing Away complete 70° right SB40 UL long, royal red		1
27	1438575-5969	Swing Away complete 70° right SB40 UL long, royal blue		1
27	1438575-9005	Swing Away complete 70° right SB40 UL long, black, glossy		1
27	1438575-9119	Swing Away complete 70° right SB40 UL long, black, bronze		1
28	1438576-0419	Swing Away complete 70° right SB42 UL long, silver grey		1
28	1438576-0442	Swing Away complete 70° right SB42 UL long, night shimmer	2009-01-01 →	1
28	1438576-3524	Swing Away complete 70° right SB42 UL long, royal red		1
28	1438576-5969	Swing Away complete 70° right SB42 UL long, royal blue		1
28	1438576-9005	Swing Away complete 70° right SB42 UL long, black, glossy		1
28	1438576-9119	Swing Away complete 70° right SB42 UL long, black, bronze		1
29	1438577-0419	Swing Away complete 70° right SB44 UL long, silver grey		1
29	1438577-0442	Swing Away complete 70° right SB44 UL long, night shimmer	2009-01-01 →	1
29	1438577-3524	Swing Away complete 70° right SB44 UL long, royal red		1
29	1438577-5969	Swing Away complete 70° right SB44 UL long, royal blue		1
29	1438577-9005	Swing Away complete 70° right SB44 UL long, black, glossy		1
29	1438577-9119	Swing Away complete 70° right SB44 UL long, black, bronze		1
30	1438578-0419	Swing Away complete 70° right SB46 UL long, silver grey		1
30	1438578-0442	Swing Away complete 70° right SB46 UL long, night shimmer	2009-01-01 →	1
30	1438578-3524	Swing Away complete 70° right SB46 UL long, royal red		1
30	1438578-5969	Swing Away complete 70° right SB46 UL long, royal blue		1
30	1438578-9005	Swing Away complete 70° right SB46 UL long, black, glossy		1
30	1438578-9119	Swing Away complete 70° right SB46 UL long, black, bronze		1
31	1438579-0419	Swing Away complete 70° right SB48 UL long, silver grey		1
31	1438579-0442	Swing Away complete 70° right SB48 UL long, night shimmer	2009-01-01 →	1
31	1438579-3524	Swing Away complete 70° right SB48 UL long, royal red		1
31	1438579-5969	Swing Away complete 70° right SB48 UL long, royal blue		1
31	1438579-9005	Swing Away complete 70° right SB48 UL long, black, glossy		1
31	1438579-9119	Swing Away complete 70° right SB48 UL long, black, bronze		1
32	1438580-0419	Swing Away complete 70° right SB50 UL long, silver grey		1
32	1438580-0442	Swing Away complete 70° right SB50 UL long, night shimmer	2009-01-01 →	1
32	1438580-3524	Swing Away complete 70° right SB50 UL long, royal red		1
32	1438580-5969	Swing Away complete 70° right SB50 UL long, royal blue		1
32	1438580-9005	Swing Away complete 70° right SB50 UL long, black, glossy		1
32	1438580-9119	Swing Away complete 70° right SB50 UL long, black, bronze		1
33	1438541-0419	Swing Away complete 70° left SB32 UL short, silver grey		1
33	1438541-0442	Swing Away complete 70° left SB32 UL short, night shimmer	2009-01-01 →	1
33	1438541-3524	Swing Away complete 70° left SB32 UL short, royal red		1
33	1438541-5969	Swing Away complete 70° left SB32 UL short, royal blue		1



**Legrest hanger 70°**

Pos.	Part number	Description	Valid from → until	Qty
33	1438541-9005	Swing Away complete 70° left SB32 UL short, black, glossy		1
33	1438541-9119	Swing Away complete 70° left SB32 UL short, black, bronze		1
34	1438542-0419	Swing Away complete 70° left SB34 UL short, silver grey		1
34	1438542-0442	Swing Away complete 70° left SB34 UL short, night shimmer	2009-01-01 →	1
34	1438542-3524	Swing Away complete 70° left SB34 UL short, royal red		1
34	1438542-5969	Swing Away complete 70° left SB34 UL short, royal blue		1
34	1438542-9005	Swing Away complete 70° left SB34 UL short, black, glossy		1
34	1438542-9119	Swing Away complete 70° left SB34 UL short, black, bronze		1
35	1438543-0419	Swing Away complete 70° left SB36 UL short, silver grey		1
35	1438543-0442	Swing Away complete 70° left SB36 UL short, night shimmer	2009-01-01 →	1
35	1438543-3524	Swing Away complete 70° left SB36 UL short, royal red		1
35	1438543-5969	Swing Away complete 70° left SB36 UL short, royal blue		1
35	1438543-9005	Swing Away complete 70° left SB36 UL short, black, glossy		1
35	1438543-9119	Swing Away complete 70° left SB36 UL short, black, bronze		1
36	1438544-0419	Swing Away complete 70° left SB38 UL short, silver grey		1
36	1438544-0442	Swing Away complete 70° left SB38 UL short, night shimmer	2009-01-01 →	1
36	1438544-3524	Swing Away complete 70° left SB38 UL short, royal red		1
36	1438544-5969	Swing Away complete 70° left SB38 UL short, royal blue		1
36	1438544-9005	Swing Away complete 70° left SB38 UL short, black, glossy		1
36	1438544-9119	Swing Away complete 70° left SB38 UL short, black, bronze		1
37	1438545-0419	Swing Away complete 70° left SB40 UL short, silver grey		1
37	1438545-0442	Swing Away complete 70° left SB40 UL short, night shimmer	2009-01-01 →	1
37	1438545-3524	Swing Away complete 70° left SB40 UL short, royal red		1
37	1438545-5969	Swing Away complete 70° left SB40 UL short, royal blue		1
37	1438545-9005	Swing Away complete 70° left SB40 UL short, black, glossy		1
37	1438545-9119	Swing Away complete 70° left SB40 UL short, black, bronze		1
38	1438546-0419	Swing Away complete 70° left SB42 UL short, silver grey		1
38	1438546-0442	Swing Away complete 70° left SB42 UL short, night shimmer	2009-01-01 →	1
38	1438546-3524	Swing Away complete 70° left SB42 UL short, royal red		1
38	1438546-5969	Swing Away complete 70° left SB42 UL short, royal blue		1
38	1438546-9005	Swing Away complete 70° left SB42 UL short, black, glossy		1
38	1438546-9119	Swing Away complete 70° left SB42 UL short, black, bronze		1
39	1438547-0419	Swing Away complete 70° left SB44 UL short, silver grey		1
39	1438547-0442	Swing Away complete 70° left SB44 UL short, night shimmer	2009-01-01 →	1
39	1438547-3524	Swing Away complete 70° left SB44 UL short, royal red		1
39	1438547-5969	Swing Away complete 70° left SB44 UL short, royal blue		1
39	1438547-9005	Swing Away complete 70° left SB44 UL short, black, glossy		1
39	1438547-9119	Swing Away complete 70° left SB44 UL short, black, bronze		1
40	1438548-0419	Swing Away complete 70° left SB46 UL short, silver grey		1
40	1438548-0442	Swing Away complete 70° left SB46 UL short, night shimmer	2009-01-01 →	1
40	1438548-3524	Swing Away complete 70° left SB46 UL short, royal red		1
40	1438548-5969	Swing Away complete 70° left SB46 UL short, royal blue		1
40	1438548-9005	Swing Away complete 70° left SB46 UL short, black, glossy		1
40	1438548-9119	Swing Away complete 70° left SB46 UL short, black, bronze		1
41	1438549-0419	Swing Away complete 70° left SB48 UL short, silver grey		1
41	1438549-0442	Swing Away complete 70° left SB48 UL short, night shimmer	2009-01-01 →	1
41	1438549-3524	Swing Away complete 70° left SB48 UL short, royal red		1
41	1438549-5969	Swing Away complete 70° left SB48 UL short, royal blue		1
41	1438549-9005	Swing Away complete 70° left SB48 UL short, black, glossy		1
41	1438549-9119	Swing Away complete 70° left SB48 UL short, black, bronze		1
42	1438550-0419	Swing Away complete 70° left SB50 UL short, silver grey		1
42	1438550-0442	Swing Away complete 70° left SB50 UL short, night shimmer	2009-01-01 →	1
42	1438550-3524	Swing Away complete 70° left SB50 UL short, royal red		1
42	1438550-5969	Swing Away complete 70° left SB50 UL short, royal blue		1
42	1438550-9005	Swing Away complete 70° left SB50 UL short, black, glossy		1

## Legrest hanger 70°

Pos.	Part number	Description	Valid from → until	Qty
42	1438550-9119	Swing Away complete 70° left SB50 UL short, black, bronze		1
43	1439070-0419	Swing Away complete 70° left SB32 UL medium, silver grey		1
43	1439070-0442	Swing Away complete 70° left SB32 UL medium, night shimmer	2009-01-01 →	1
43	1439070-3524	Swing Away complete 70° left SB32 UL medium, royal red		1
43	1439070-5969	Swing Away complete 70° left SB32 UL medium, royal blue		1
43	1439070-9005	Swing Away complete 70° left SB32 UL medium, black, glossy		1
43	1439070-9119	Swing Away complete 70° left SB32 UL medium, black, bronze		1
44	1439071-0419	Swing Away complete 70° left SB34 UL medium, silver grey		1
44	1439071-0442	Swing Away complete 70° left SB34 UL medium, night shimmer	2009-01-01 →	1
44	1439071-3524	Swing Away complete 70° left SB34 UL medium, royal red		1
44	1439071-5969	Swing Away complete 70° left SB34 UL medium, royal blue		1
44	1439071-9005	Swing Away complete 70° left SB34 UL medium, black, glossy		1
44	1439071-9119	Swing Away complete 70° left SB34 UL medium, black, bronze		1
45	1439072-0419	Swing Away complete 70° left SB36 UL medium, silver grey		1
45	1439072-0442	Swing Away complete 70° left SB36 UL medium, night shimmer	2009-01-01 →	1
45	1439072-3524	Swing Away complete 70° left SB36 UL medium, royal red		1
45	1439072-5969	Swing Away complete 70° left SB36 UL medium, royal blue		1
45	1439072-9005	Swing Away complete 70° left SB36 UL medium, black, glossy		1
45	1439072-9119	Swing Away complete 70° left SB36 UL medium, black, bronze		1
46	1439073-0419	Swing Away complete 70° left SB38 UL medium, silver grey		1
46	1439073-0442	Swing Away complete 70° left SB38 UL medium, night shimmer	2009-01-01 →	1
46	1439073-3524	Swing Away complete 70° left SB38 UL medium, royal red		1
46	1439073-5969	Swing Away complete 70° left SB38 UL medium, royal blue		1
46	1439073-9005	Swing Away complete 70° left SB38 UL medium, black, glossy		1
46	1439073-9119	Swing Away complete 70° left SB38 UL medium, black, bronze		1
47	1439074-0419	Swing Away complete 70° left SB40 UL medium, silver grey		1
47	1439074-0442	Swing Away complete 70° left SB40 UL medium, night shimmer	2009-01-01 →	1
47	1439074-3524	Swing Away complete 70° left SB40 UL medium, royal red		1
47	1439074-5969	Swing Away complete 70° left SB40 UL medium, royal blue		1
47	1439074-9005	Swing Away complete 70° left SB40 UL medium, black, glossy		1
47	1439074-9119	Swing Away complete 70° left SB40 UL medium, black, bronze		1
48	1439075-0419	Swing Away complete 70° left SB42 UL medium, silver grey		1
48	1439075-0442	Swing Away complete 70° left SB42 UL medium, night shimmer	2009-01-01 →	1
48	1439075-3524	Swing Away complete 70° left SB42 UL medium, royal red		1
48	1439075-5969	Swing Away complete 70° left SB42 UL medium, royal blue		1
48	1439075-9005	Swing Away complete 70° left SB42 UL medium, black, glossy		1
48	1439075-9119	Swing Away complete 70° left SB42 UL medium, black, bronze		1
49	1439076-0419	Swing Away complete 70° left SB44 UL medium, silver grey		1
49	1439076-0442	Swing Away complete 70° left SB44 UL medium, night shimmer	2009-01-01 →	1
49	1439076-3524	Swing Away complete 70° left SB44 UL medium, royal red		1
49	1439076-5969	Swing Away complete 70° left SB44 UL medium, royal blue		1
49	1439076-9005	Swing Away complete 70° left SB44 UL medium, black, glossy		1
49	1439076-9119	Swing Away complete 70° left SB44 UL medium, black, bronze		1
50	1439077-0419	Swing Away complete 70° left SB46 UL medium, silver grey		1
50	1439077-0442	Swing Away complete 70° left SB46 UL medium, night shimmer	2009-01-01 →	1
50	1439077-3524	Swing Away complete 70° left SB46 UL medium, royal red		1
50	1439077-5969	Swing Away complete 70° left SB46 UL medium, royal blue		1
50	1439077-9005	Swing Away complete 70° left SB46 UL medium, black, glossy		1
50	1439077-9119	Swing Away complete 70° left SB46 UL medium, black, bronze		1
51	1439078-0419	Swing Away complete 70° left SB48 UL medium, silver grey		1
51	1439078-0442	Swing Away complete 70° left SB48 UL medium, night shimmer	2009-01-01 →	1
51	1439078-3524	Swing Away complete 70° left SB48 UL medium, royal red		1
51	1439078-5969	Swing Away complete 70° left SB48 UL medium, royal blue		1
51	1439078-9005	Swing Away complete 70° left SB48 UL medium, black, glossy		1
51	1439078-9119	Swing Away complete 70° left SB48 UL medium, black, bronze		1


**Legrest hanger 70°**


Pos.	Part number	Description	Valid from → until	Qty
52	1439079-0419	Swing Away complete 70° left SB50 UL medium, silver grey		1
52	1439079-0442	Swing Away complete 70° left SB50 UL medium, night shimmer	2009-01-01 →	1
52	1439079-3524	Swing Away complete 70° left SB50 UL medium, royal red		1
52	1439079-5969	Swing Away complete 70° left SB50 UL medium, royal blue		1
52	1439079-9005	Swing Away complete 70° left SB50 UL medium, black, glossy		1
52	1439079-9119	Swing Away complete 70° left SB50 UL medium, black, bronze		1
53	1438581-0419	Swing Away complete 70° left SB32 UL long, silver grey		1
53	1438581-0442	Swing Away complete 70° left SB32 UL long, night shimmer	2009-01-01 →	1
53	1438581-3524	Swing Away complete 70° left SB32 UL long, royal red		1
53	1438581-5969	Swing Away complete 70° left SB32 UL long, royal blue		1
53	1438581-9005	Swing Away complete 70° left SB32 UL long, black, glossy		1
53	1438581-9119	Swing Away complete 70° left SB32 UL long, black, bronze		1
54	1438582-0419	Swing Away complete 70° left SB34 UL long, silver grey		1
54	1438582-0442	Swing Away complete 70° left SB34 UL long, night shimmer	2009-01-01 →	1
54	1438582-3524	Swing Away complete 70° left SB34 UL long, royal red		1
54	1438582-5969	Swing Away complete 70° left SB34 UL long, royal blue		1
54	1438582-9005	Swing Away complete 70° left SB34 UL long, black, glossy		1
54	1438582-9119	Swing Away complete 70° left SB34 UL long, black, bronze		1
55	1438583-0419	Swing Away complete 70° left SB36 UL long, silver grey		1
55	1438583-0442	Swing Away complete 70° left SB36 UL long, night shimmer	2009-01-01 →	1
55	1438583-3524	Swing Away complete 70° left SB36 UL long, royal red		1
55	1438583-5969	Swing Away complete 70° left SB36 UL long, royal blue		1
55	1438583-9005	Swing Away complete 70° left SB36 UL long, black, glossy		1
55	1438583-9119	Swing Away complete 70° left SB36 UL long, black, bronze		1
56	1438584-0419	Swing Away complete 70° left SB38 UL long, silver grey		1
56	1438584-0442	Swing Away complete 70° left SB38 UL long, night shimmer	2009-01-01 →	1
56	1438584-3524	Swing Away complete 70° left SB38 UL long, royal red		1
56	1438584-5969	Swing Away complete 70° left SB38 UL long, royal blue		1
56	1438584-9005	Swing Away complete 70° left SB38 UL long, black, glossy		1
56	1438584-9119	Swing Away complete 70° left SB38 UL long, black, bronze		1
57	1438585-0419	Swing Away complete 70° left SB40 UL long, silver grey		1
57	1438585-0442	Swing Away complete 70° left SB40 UL long, night shimmer	2009-01-01 →	1
57	1438585-3524	Swing Away complete 70° left SB40 UL long, royal red		1
57	1438585-5969	Swing Away complete 70° left SB40 UL long, royal blue		1
57	1438585-9005	Swing Away complete 70° left SB40 UL long, black, glossy		1
57	1438585-9119	Swing Away complete 70° left SB40 UL long, black, bronze		1
58	1438586-0419	Swing Away complete 70° left SB42 UL long, silver grey		1
58	1438586-0442	Swing Away complete 70° left SB42 UL long, night shimmer	2009-01-01 →	1
58	1438586-3524	Swing Away complete 70° left SB42 UL long, royal red		1
58	1438586-5969	Swing Away complete 70° left SB42 UL long, royal blue		1
58	1438586-9005	Swing Away complete 70° left SB42 UL long, black, glossy		1
58	1438586-9119	Swing Away complete 70° left SB42 UL long, black, bronze		1
59	1438587-0419	Swing Away complete 70° left SB44 UL long, silver grey		1
59	1438587-0442	Swing Away complete 70° left SB44 UL long, night shimmer	2009-01-01 →	1
59	1438587-3524	Swing Away complete 70° left SB44 UL long, royal red		1
59	1438587-5969	Swing Away complete 70° left SB44 UL long, royal blue		1
59	1438587-9005	Swing Away complete 70° left SB44 UL long, black, glossy		1
59	1438587-9119	Swing Away complete 70° left SB44 UL long, black, bronze		1
60	1438588-0419	Swing Away complete 70° left SB46 UL long, silver grey		1
60	1438588-0442	Swing Away complete 70° left SB46 UL long, night shimmer	2009-01-01 →	1
60	1438588-3524	Swing Away complete 70° left SB46 UL long, royal red		1
60	1438588-5969	Swing Away complete 70° left SB46 UL long, royal blue		1
60	1438588-9005	Swing Away complete 70° left SB46 UL long, black, glossy		1
60	1438588-9119	Swing Away complete 70° left SB46 UL long, black, bronze		1
61	1438589-0419	Swing Away complete 70° left SB48 UL long, silver grey		1


**Legrest hanger 70°**

Pos.	Part number	Description	Valid from → until	Qty
61	1438589-0442	Swing Away complete 70° left SB48 UL long, night shimmer	2009-01-01 →	1
61	1438589-3524	Swing Away complete 70° left SB48 UL long, royal red		1
61	1438589-5969	Swing Away complete 70° left SB48 UL long, royal blue		1
61	1438589-9005	Swing Away complete 70° left SB48 UL long, black, glossy		1
61	1438589-9119	Swing Away complete 70° left SB48 UL long, black, bronze		1
62	1438590-0419	Swing Away complete 70° left SB50 UL long, silver grey		1
62	1438590-0442	Swing Away complete 70° left SB50 UL long, night shimmer	2009-01-01 →	1
62	1438590-3524	Swing Away complete 70° left SB50 UL long, royal red		1
62	1438590-5969	Swing Away complete 70° left SB50 UL long, royal blue		1
62	1438590-9005	Swing Away complete 70° left SB50 UL long, black, glossy		1
62	1438590-9119	Swing Away complete 70° left SB50 UL long, black, bronze		1
63	1510271	Locking mechanism for swing-away, kit right		1
64	1510272	Locking mechanism for swing-away, kit left		1

# Legrest hanger 80°

80° 	SB32	SB34	SB36	SB38	SB40	SB42	SB44	SB46	SB48	SB50
UL32-40	33	34	35	36	37	38	39	40	41	42
UL38-46	43	44	45	46	47	48	49	50	51	52
UL44-52	53	54	55	56	57	58	59	60	61	62

80° 	SB32	SB34	SB36	SB38	SB40	SB42	SB44	SB46	SB48	SB50
UL32-40	3	4	5	6	7	8	9	10	11	12
UL38-46	13	14	15	16	17	18	19	20	21	22
UL44-52	23	24	25	26	27	28	29	30	31	32

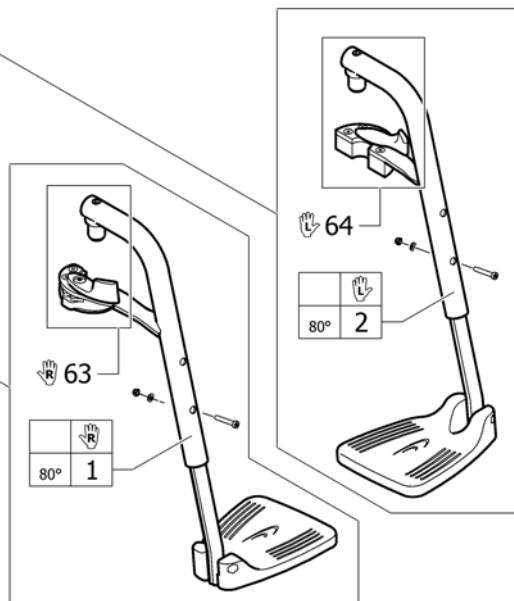

Pos.	Part number	Description	Valid from → until	Qty
1	1438228-0419	Swing away 80° right, silver grey		1
1	1438228-0442	Swing away 80° right, night shimmer	2009-01-01 →	1
1	1438228-3524	Swing away 80° right, royal red		1
1	1438228-5969	Swing away 80° right, royal blue		1
1	1438228-9005	Swing away 80° right, black, glossy		1
1	1438228-9119	Swing away 80° right, black, bronze		1
2	1438229-0419	Swing Away 80° left, silver grey		1
2	1438229-0442	Swing Away 80° left, night shimmer	2009-01-01 →	1
2	1438229-3524	Swing Away 80° left, royal red		1
2	1438229-5969	Swing Away 80° left, royal blue		1
2	1438229-9005	Swing Away 80° left, black, glossy		1
2	1438229-9119	Swing Away 80° left, black, bronze		1
3	1438551-0419	Swing Away complete 80° right SB32 UL short, silver grey		1
3	1438551-0442	Swing Away complete 80° right SB32 UL short, night shimmer	2009-01-01 →	1
3	1438551-3524	Swing Away complete 80° right SB32 UL short, royal red		1
3	1438551-5969	Swing Away complete 80° right SB32 UL short, royal blue		1
3	1438551-9005	Swing Away complete 80° right SB32 UL short, black, glossy		1
3	1438551-9119	Swing Away complete 80° right SB32 UL short, black, bronze		1
4	1438552-0419	Swing Away complete 80° right SB34 UL short, silver grey		1
4	1438552-0442	Swing Away complete 80° right SB34 UL short, night shimmer	2009-01-01 →	1
4	1438552-3524	Swing Away complete 80° right SB34 UL short, royal red		1
4	1438552-5969	Swing Away complete 80° right SB34 UL short, royal blue		1
4	1438552-9005	Swing Away complete 80° right SB34 UL short, black, glossy		1
4	1438552-9119	Swing Away complete 80° right SB34 UL short, black, bronze		1
5	1438553-0419	Swing Away complete 80° right SB36 UL short, silver grey		1
5	1438553-0442	Swing Away complete 80° right SB36 UL short, night shimmer	2009-01-01 →	1
5	1438553-3524	Swing Away complete 80° right SB36 UL short, royal red		1
5	1438553-5969	Swing Away complete 80° right SB36 UL short, royal blue		1
5	1438553-9005	Swing Away complete 80° right SB36 UL short, black, glossy		1
5	1438553-9119	Swing Away complete 80° right SB36 UL short, black, bronze		1
6	1438554-0419	Swing Away complete 80° right SB38 UL short, silver grey		1



**Legrest hanger 80°**

Pos.	Part number	Description	Valid from → until	Qty
6	1438554-0442	Swing Away complete 80° right SB38 UL short, night shimmer	2009-01-01 →	1
6	1438554-3524	Swing Away complete 80° right SB38 UL short, royal red		1
6	1438554-5969	Swing Away complete 80° right SB38 UL short, royal blue		1
6	1438554-9005	Swing Away complete 80° right SB38 UL short, black, glossy		1
6	1438554-9119	Swing Away complete 80° right SB38 UL short, black, bronze		1
7	1438555-0419	Swing Away complete 80° right SB40 UL short, silver grey		1
7	1438555-0442	Swing Away complete 80° right SB40 UL short, night shimmer	2009-01-01 →	1
7	1438555-3524	Swing Away complete 80° right SB40 UL short, royal red		1
7	1438555-5969	Swing Away complete 80° right SB40 UL short, royal blue		1
7	1438555-9005	Swing Away complete 80° right SB40 UL short, black, glossy		1
7	1438555-9119	Swing Away complete 80° right SB40 UL short, black, bronze		1
8	1438556-0419	Swing Away complete 80° right SB42 UL short, silver grey		1
8	1438556-0442	Swing Away complete 80° right SB42 UL short, night shimmer	2009-01-01 →	1
8	1438556-3524	Swing Away complete 80° right SB42 UL short, royal red		1
8	1438556-5969	Swing Away complete 80° right SB42 UL short, royal blue		1
8	1438556-9005	Swing Away complete 80° right SB42 UL short, black, glossy		1
8	1438556-9119	Swing Away complete 80° right SB42 UL short, black, bronze		1
9	1438557-0419	Swing Away complete 80° right SB44 UL short, silver grey		1
9	1438557-0442	Swing Away complete 80° right SB44 UL short, night shimmer	2009-01-01 →	1
9	1438557-3524	Swing Away complete 80° right SB44 UL short, royal red		1
9	1438557-5969	Swing Away complete 80° right SB44 UL short, royal blue		1
9	1438557-9005	Swing Away complete 80° right SB44 UL short, black, glossy		1
9	1438557-9119	Swing Away complete 80° right SB44 UL short, black, bronze		1
10	1438558-0419	Swing Away complete 80° right SB46 UL short, silver grey		1
10	1438558-0442	Swing Away complete 80° right SB46 UL short, night shimmer	2009-01-01 →	1
10	1438558-3524	Swing Away complete 80° right SB46 UL short, royal red		1
10	1438558-5969	Swing Away complete 80° right SB46 UL short, royal blue		1
10	1438558-9005	Swing Away complete 80° right SB46 UL short, black, glossy		1
10	1438558-9119	Swing Away complete 80° right SB46 UL short, black, bronze		1
11	1438559-0419	Swing Away complete 80° right SB48 UL short, silver grey		1
11	1438559-0442	Swing Away complete 80° right SB48 UL short, night shimmer	2009-01-01 →	1
11	1438559-3524	Swing Away complete 80° right SB48 UL short, royal red		1
11	1438559-5969	Swing Away complete 80° right SB48 UL short, royal blue		1
11	1438559-9005	Swing Away complete 80° right SB48 UL short, black, glossy		1
11	1438559-9119	Swing Away complete 80° right SB48 UL short, black, bronze		1
12	1438560-0419	Swing Away complete 80° right SB50 UL short, silver grey		1
12	1438560-0442	Swing Away complete 80° right SB50 UL short, night shimmer	2009-01-01 →	1
12	1438560-3524	Swing Away complete 80° right SB50 UL short, royal red		1
12	1438560-5969	Swing Away complete 80° right SB50 UL short, royal blue		1
12	1438560-9005	Swing Away complete 80° right SB50 UL short, black, glossy		1
12	1438560-9119	Swing Away complete 80° right SB50 UL short, black, bronze		1
13	1439080-0419	Swing Away complete 80° right SB32 UL medium, silver grey		1
13	1439080-0442	Swing Away complete 80° right SB32 UL medium, night shimmer	2009-01-01 →	1
13	1439080-3524	Swing Away complete 80° right SB32 UL medium, royal red		1
13	1439080-5969	Swing Away complete 80° right SB32 UL medium, royal blue		1
13	1439080-9005	Swing Away complete 80° right SB32 UL medium, black, glossy		1
13	1439080-9119	Swing Away complete 80° right SB32 UL medium, black, bronze		1
14	1439081-0419	Swing Away complete 80° right SB34 UL medium, silver grey		1
14	1439081-0442	Swing Away complete 80° right SB34 UL medium, night shimmer	2009-01-01 →	1
14	1439081-3524	Swing Away complete 80° right SB34 UL medium, royal red		1
14	1439081-5969	Swing Away complete 80° right SB34 UL medium, royal blue		1
14	1439081-9005	Swing Away complete 80° right SB34 UL medium, black, glossy		1
14	1439081-9119	Swing Away complete 80° right SB34 UL medium, black, bronze		1
15	1439082-0419	Swing Away complete 80° right SB36 UL medium, silver grey		1
15	1439082-0442	Swing Away complete 80° right SB36 UL medium, night shimmer	2009-01-01 →	1



**Legrest hanger 80°**

Pos.	Part number	Description	Valid from → until	Qty
15	1439082-3524	Swing Away complete 80° right SB36 UL medium, royal red		1
15	1439082-5969	Swing Away complete 80° right SB36 UL medium, royal blue		1
15	1439082-9005	Swing Away complete 80° right SB36 UL medium, black, glossy		1
15	1439082-9119	Swing Away complete 80° right SB36 UL medium, black, bronze		1
16	1439083-0419	Swing Away complete 80° right SB38 UL medium, silver grey		1
16	1439083-0442	Swing Away complete 80° right SB38 UL medium, night shimmer	2009-01-01 →	1
16	1439083-3524	Swing Away complete 80° right SB38 UL medium, royal red		1
16	1439083-5969	Swing Away complete 80° right SB38 UL medium, royal blue		1
16	1439083-9005	Swing Away complete 80° right SB38 UL medium, black, glossy		1
16	1439083-9119	Swing Away complete 80° right SB38 UL medium, black, bronze		1
17	1439084-0419	Swing Away complete 80° right SB40 UL medium, silver grey		1
17	1439084-0442	Swing Away complete 80° right SB40 UL medium, night shimmer	2009-01-01 →	1
17	1439084-3524	Swing Away complete 80° right SB40 UL medium, royal red		1
17	1439084-5969	Swing Away complete 80° right SB40 UL medium, royal blue		1
17	1439084-9005	Swing Away complete 80° right SB40 UL medium, black, glossy		1
17	1439084-9119	Swing Away complete 80° right SB40 UL medium, black, bronze		1
18	1439085-0419	Swing Away complete 80° right SB42 UL medium, silver grey		1
18	1439085-0442	Swing Away complete 80° right SB42 UL medium, night shimmer	2009-01-01 →	1
18	1439085-3524	Swing Away complete 80° right SB42 UL medium, royal red		1
18	1439085-5969	Swing Away complete 80° right SB42 UL medium, royal blue		1
18	1439085-9005	Swing Away complete 80° right SB42 UL medium, black, glossy		1
18	1439085-9119	Swing Away complete 80° right SB42 UL medium, black, bronze		1
19	1439086-0419	Swing Away complete 80° right SB44 UL medium, silver grey		1
19	1439086-0442	Swing Away complete 80° right SB44 UL medium, night shimmer	2009-01-01 →	1
19	1439086-3524	Swing Away complete 80° right SB44 UL medium, royal red		1
19	1439086-5969	Swing Away complete 80° right SB44 UL medium, royal blue		1
19	1439086-9005	Swing Away complete 80° right SB44 UL medium, black, glossy		1
19	1439086-9119	Swing Away complete 80° right SB44 UL medium, black, bronze		1
20	1439087-0419	Swing Away complete 80° right SB46 UL medium, silver grey		1
20	1439087-0442	Swing Away complete 80° right SB46 UL medium, night shimmer	2009-01-01 →	1
20	1439087-3524	Swing Away complete 80° right SB46 UL medium, royal red		1
20	1439087-5969	Swing Away complete 80° right SB46 UL medium, royal blue		1
20	1439087-9005	Swing Away complete 80° right SB46 UL medium, black, glossy		1
20	1439087-9119	Swing Away complete 80° right SB46 UL medium, black, bronze		1
21	1439088-0419	Swing Away complete 80° right SB48 UL medium, silver grey		1
21	1439088-0442	Swing Away complete 80° right SB48 UL medium, night shimmer	2009-01-01 →	1
21	1439088-3524	Swing Away complete 80° right SB48 UL medium, royal red		1
21	1439088-5969	Swing Away complete 80° right SB48 UL medium, royal blue		1
21	1439088-9005	Swing Away complete 80° right SB48 UL medium, black, glossy		1
21	1439088-9119	Swing Away complete 80° right SB48 UL medium, black, bronze		1
22	1439089-0419	Swing Away complete 80° right SB50 UL medium, silver grey		1
22	1439089-0442	Swing Away complete 80° right SB50 UL medium, night shimmer	2009-01-01 →	1
22	1439089-3524	Swing Away complete 80° right SB50 UL medium, royal red		1
22	1439089-5969	Swing Away complete 80° right SB50 UL medium, royal blue		1
22	1439089-9005	Swing Away complete 80° right SB50 UL medium, black, glossy		1
22	1439089-9119	Swing Away complete 80° right SB50 UL medium, black, bronze		1
23	1438591-0419	Swing Away complete 80° right SB32 UL long, silver grey		1
23	1438591-0442	Swing Away complete 80° right SB32 UL long, night shimmer	2009-01-01 →	1
23	1438591-3524	Swing Away complete 80° right SB32 UL long, royal red		1
23	1438591-5969	Swing Away complete 80° right SB32 UL long, royal blue		1
23	1438591-9005	Swing Away complete 80° right SB32 UL long, black, glossy		1
23	1438591-9119	Swing Away complete 80° right SB32 UL long, black, bronze		1
24	1438592-0419	Swing Away complete 80° right SB34 UL long, silver grey		1
24	1438592-0442	Swing Away complete 80° right SB34 UL long, night shimmer	2009-01-01 →	1
24	1438592-3524	Swing Away complete 80° right SB34 UL long, royal red		1

## Legrest hanger 80°

Pos.	Part number	Description	Valid from → until	Qty
24	1438592-5969	Swing Away complete 80° right SB34 UL long, royal blue		1
24	1438592-9005	Swing Away complete 80° right SB34 UL long, black, glossy		1
24	1438592-9119	Swing Away complete 80° right SB34 UL long, black, bronze		1
25	1438593-0419	Swing Away complete 80° right SB36 UL long, silver grey		1
25	1438593-0442	Swing Away complete 80° right SB36 UL long, night shimmer	2009-01-01 →	1
25	1438593-3524	Swing Away complete 80° right SB36 UL long, royal red		1
25	1438593-5969	Swing Away complete 80° right SB36 UL long, royal blue		1
25	1438593-9005	Swing Away complete 80° right SB36 UL long, black, glossy		1
25	1438593-9119	Swing Away complete 80° right SB36 UL long, black, bronze		1
26	1438594-0419	Swing Away complete 80° right SB38 UL long, silver grey		1
26	1438594-0442	Swing Away complete 80° right SB38 UL long, night shimmer	2009-01-01 →	1
26	1438594-3524	Swing Away complete 80° right SB38 UL long, royal red		1
26	1438594-5969	Swing Away complete 80° right SB38 UL long, royal blue		1
26	1438594-9005	Swing Away complete 80° right SB38 UL long, black, glossy		1
26	1438594-9119	Swing Away complete 80° right SB38 UL long, black, bronze		1
27	1438595-0419	Swing Away complete 80° right SB40 UL long, silver grey		1
27	1438595-0442	Swing Away complete 80° right SB40 UL long, night shimmer	2009-01-01 →	1
27	1438595-3524	Swing Away complete 80° right SB40 UL long, royal red		1
27	1438595-5969	Swing Away complete 80° right SB40 UL long, royal blue		1
27	1438595-9005	Swing Away complete 80° right SB40 UL long, black, glossy		1
27	1438595-9119	Swing Away complete 80° right SB40 UL long, black, bronze		1
28	1438596-0419	Swing Away complete 80° right SB42 UL long, silver grey		1
28	1438596-0442	Swing Away complete 80° right SB42 UL long, night shimmer	2009-01-01 →	1
28	1438596-3524	Swing Away complete 80° right SB42 UL long, royal red		1
28	1438596-5969	Swing Away complete 80° right SB42 UL long, royal blue		1
28	1438596-9005	Swing Away complete 80° right SB42 UL long, black, glossy		1
28	1438596-9119	Swing Away complete 80° right SB42 UL long, black, bronze		1
29	1438597-0419	Swing Away complete 80° right SB44 UL long, silver grey		1
29	1438597-0442	Swing Away complete 80° right SB44 UL long, night shimmer	2009-01-01 →	1
29	1438597-3524	Swing Away complete 80° right SB44 UL long, royal red		1
29	1438597-5969	Swing Away complete 80° right SB44 UL long, royal blue		1
29	1438597-9005	Swing Away complete 80° right SB44 UL long, black, glossy		1
29	1438597-9119	Swing Away complete 80° right SB44 UL long, black, bronze		1
30	1438598-0419	Swing Away complete 80° right SB46 UL long, silver grey		1
30	1438598-0442	Swing Away complete 80° right SB46 UL long, night shimmer	2009-01-01 →	1
30	1438598-3524	Swing Away complete 80° right SB46 UL long, royal red		1
30	1438598-5969	Swing Away complete 80° right SB46 UL long, royal blue		1
30	1438598-9005	Swing Away complete 80° right SB46 UL long, black, glossy		1
30	1438598-9119	Swing Away complete 80° right SB46 UL long, black, bronze		1
31	1438599-0419	Swing Away complete 80° right SB48 UL long, silver grey		1
31	1438599-0442	Swing Away complete 80° right SB48 UL long, night shimmer	2009-01-01 →	1
31	1438599-3524	Swing Away complete 80° right SB48 UL long, royal red		1
31	1438599-5969	Swing Away complete 80° right SB48 UL long, royal blue		1
31	1438599-9005	Swing Away complete 80° right SB48 UL long, black, glossy		1
31	1438599-9119	Swing Away complete 80° right SB48 UL long, black, bronze		1
32	1438600-0419	Swing Away complete 80° right SB50 UL long, silver grey		1
32	1438600-0442	Swing Away complete 80° right SB50 UL long, night shimmer	2009-01-01 →	1
32	1438600-3524	Swing Away complete 80° right SB50 UL long, royal red		1
32	1438600-5969	Swing Away complete 80° right SB50 UL long, royal blue		1
32	1438600-9005	Swing Away complete 80° right SB50 UL long, black, glossy		1
32	1438600-9119	Swing Away complete 80° right SB50 UL long, black, bronze		1
33	1438561-0419	Swing Away complete 80° left SB32 UL short, silver grey		1
33	1438561-0442	Swing Away complete 80° left SB32 UL short, night shimmer	2009-01-01 →	1
33	1438561-3524	Swing Away complete 80° left SB32 UL short, royal red		1
33	1438561-5969	Swing Away complete 80° left SB32 UL short, royal blue		1

**Legrest hanger 80°**

Pos.	Part number	Description	Valid from → until	Qty
33	1438561-9005	Swing Away complete 80° left SB32 UL short, black, glossy		1
33	1438561-9119	Swing Away complete 80° left SB32 UL short, black, bronze		1
34	1438562-0419	Swing Away complete 80° left SB34 UL short, silver grey		1
34	1438562-0442	Swing Away complete 80° left SB34 UL short, night shimmer	2009-01-01 →	1
34	1438562-3524	Swing Away complete 80° left SB34 UL short, royal red		1
34	1438562-5969	Swing Away complete 80° left SB34 UL short, royal blue		1
34	1438562-9005	Swing Away complete 80° left SB34 UL short, black, glossy		1
34	1438562-9119	Swing Away complete 80° left SB34 UL short, black, bronze		1
35	1438563-0419	Swing Away complete 80° left SB36 UL short, silver grey		1
35	1438563-0442	Swing Away complete 80° left SB36 UL short, night shimmer	2009-01-01 →	1
35	1438563-3524	Swing Away complete 80° left SB36 UL short, royal red		1
35	1438563-5969	Swing Away complete 80° left SB36 UL short, royal blue		1
35	1438563-9005	Swing Away complete 80° left SB36 UL short, black, glossy		1
35	1438563-9119	Swing Away complete 80° left SB36 UL short, black, bronze		1
36	1438564-0419	Swing Away complete 80° left SB38 UL short, silver grey		1
36	1438564-0442	Swing Away complete 80° left SB38 UL short, night shimmer	2009-01-01 →	1
36	1438564-3524	Swing Away complete 80° left SB38 UL short, royal red		1
36	1438564-5969	Swing Away complete 80° left SB38 UL short, royal blue		1
36	1438564-9005	Swing Away complete 80° left SB38 UL short, black, glossy		1
36	1438564-9119	Swing Away complete 80° left SB38 UL short, black, bronze		1
37	1438565-0419	Swing Away complete 80° left SB40 UL short, silver grey		1
37	1438565-0442	Swing Away complete 80° left SB40 UL short, night shimmer	2009-01-01 →	1
37	1438565-3524	Swing Away complete 80° left SB40 UL short, royal red		1
37	1438565-5969	Swing Away complete 80° left SB40 UL short, royal blue		1
37	1438565-9005	Swing Away complete 80° left SB40 UL short, black, glossy		1
37	1438565-9119	Swing Away complete 80° left SB40 UL short, black, bronze		1
38	1438566-0419	Swing Away complete 80° left SB42 UL short, silver grey		1
38	1438566-0442	Swing Away complete 80° left SB42 UL short, night shimmer	2009-01-01 →	1
38	1438566-3524	Swing Away complete 80° left SB42 UL short, royal red		1
38	1438566-5969	Swing Away complete 80° left SB42 UL short, royal blue		1
38	1438566-9005	Swing Away complete 80° left SB42 UL short, black, glossy		1
38	1438566-9119	Swing Away complete 80° left SB42 UL short, black, bronze		1
39	1438567-0419	Swing Away complete 80° left SB44 UL short, silver grey		1
39	1438567-0442	Swing Away complete 80° left SB44 UL short, night shimmer	2009-01-01 →	1
39	1438567-3524	Swing Away complete 80° left SB44 UL short, royal red		1
39	1438567-5969	Swing Away complete 80° left SB44 UL short, royal blue		1
39	1438567-9005	Swing Away complete 80° left SB44 UL short, black, glossy		1
39	1438567-9119	Swing Away complete 80° left SB44 UL short, black, bronze		1
40	1438568-0419	Swing Away complete 80° left SB46 UL short, silver grey		1
40	1438568-0442	Swing Away complete 80° left SB46 UL short, night shimmer	2009-01-01 →	1
40	1438568-3524	Swing Away complete 80° left SB46 UL short, royal red		1
40	1438568-5969	Swing Away complete 80° left SB46 UL short, royal blue		1
40	1438568-9005	Swing Away complete 80° left SB46 UL short, black, glossy		1
40	1438568-9119	Swing Away complete 80° left SB46 UL short, black, bronze		1
41	1438569-0419	Swing Away complete 80° left SB48 UL short, silver grey		1
41	1438569-0442	Swing Away complete 80° left SB48 UL short, night shimmer	2009-01-01 →	1
41	1438569-3524	Swing Away complete 80° left SB48 UL short, royal red		1
41	1438569-5969	Swing Away complete 80° left SB48 UL short, royal blue		1
41	1438569-9005	Swing Away complete 80° left SB48 UL short, black, glossy		1
41	1438569-9119	Swing Away complete 80° left SB48 UL short, black, bronze		1
42	1438570-0419	Swing Away complete 80° left SB50 UL short, silver grey		1
42	1438570-0442	Swing Away complete 80° left SB50 UL short, night shimmer	2009-01-01 →	1
42	1438570-3524	Swing Away complete 80° left SB50 UL short, royal red		1
42	1438570-5969	Swing Away complete 80° left SB50 UL short, royal blue		1
42	1438570-9005	Swing Away complete 80° left SB50 UL short, black, glossy		1

## Legrest hanger 80°

Pos.	Part number	Description	Valid from → until	Qty
42	1438570-9119	Swing Away complete 80° left SB50 UL short, black, bronze		1
43	1439090-0419	Swing Away complete 80° left SB32 UL medium, silver grey		1
43	1439090-0442	Swing Away complete 80° left SB32 UL medium, night shimmer	2009-01-01 →	1
43	1439090-3524	Swing Away complete 80° left SB32 UL medium, royal red		1
43	1439090-5969	Swing Away complete 80° left SB32 UL medium, royal blue		1
43	1439090-9005	Swing Away complete 80° left SB32 UL medium, black, glossy		1
43	1439090-9119	Swing Away complete 80° left SB32 UL medium, black, bronze		1
44	1439091-0419	Swing Away complete 80° left SB34 UL medium, silver grey		1
44	1439091-0442	Swing Away complete 80° left SB34 UL medium, night shimmer	2009-01-01 →	1
44	1439091-3524	Swing Away complete 80° left SB34 UL medium, royal red		1
44	1439091-5969	Swing Away complete 80° left SB34 UL medium, royal blue		1
44	1439091-9005	Swing Away complete 80° left SB34 UL medium, black, glossy		1
44	1439091-9119	Swing Away complete 80° left SB34 UL medium, black, bronze		1
45	1439092-0419	Swing Away complete 80° left SB36 UL medium, silver grey		1
45	1439092-0442	Swing Away complete 80° left SB36 UL medium, night shimmer	2009-01-01 →	1
45	1439092-3524	Swing Away complete 80° left SB36 UL medium, royal red		1
45	1439092-5969	Swing Away complete 80° left SB36 UL medium, royal blue		1
45	1439092-9005	Swing Away complete 80° left SB36 UL medium, black, glossy		1
45	1439092-9119	Swing Away complete 80° left SB36 UL medium, black, bronze		1
46	1439093-0419	Swing Away complete 80° left SB38 UL medium, silver grey		1
46	1439093-0442	Swing Away complete 80° left SB38 UL medium, night shimmer	2009-01-01 →	1
46	1439093-3524	Swing Away complete 80° left SB38 UL medium, royal red		1
46	1439093-5969	Swing Away complete 80° left SB38 UL medium, royal blue		1
46	1439093-9005	Swing Away complete 80° left SB38 UL medium, black, glossy		1
46	1439093-9119	Swing Away complete 80° left SB38 UL medium, black, bronze		1
47	1439094-0419	Swing Away complete 80° left SB40 UL medium, silver grey		1
47	1439094-0442	Swing Away complete 80° left SB40 UL medium, night shimmer	2009-01-01 →	1
47	1439094-3524	Swing Away complete 80° left SB40 UL medium, royal red		1
47	1439094-5969	Swing Away complete 80° left SB40 UL medium, royal blue		1
47	1439094-9005	Swing Away complete 80° left SB40 UL medium, black, glossy		1
47	1439094-9119	Swing Away complete 80° left SB40 UL medium, black, bronze		1
48	1439095-0419	Swing Away complete 80° left SB42 UL medium, silver grey		1
48	1439095-0442	Swing Away complete 80° left SB42 UL medium, night shimmer	2009-01-01 →	1
48	1439095-3524	Swing Away complete 80° left SB42 UL medium, royal red		1
48	1439095-5969	Swing Away complete 80° left SB42 UL medium, royal blue		1
48	1439095-9005	Swing Away complete 80° left SB42 UL medium, black, glossy		1
48	1439095-9119	Swing Away complete 80° left SB42 UL medium, black, bronze		1
49	1439096-0419	Swing Away complete 80° left SB44 UL medium, silver grey		1
49	1439096-0442	Swing Away complete 80° left SB44 UL medium, night shimmer	2009-01-01 →	1
49	1439096-3524	Swing Away complete 80° left SB44 UL medium, royal red		1
49	1439096-5969	Swing Away complete 80° left SB44 UL medium, royal blue		1
49	1439096-9005	Swing Away complete 80° left SB44 UL medium, black, glossy		1
49	1439096-9119	Swing Away complete 80° left SB44 UL medium, black, bronze		1
50	1439097-0419	Swing Away complete 80° left SB46 UL medium, silver grey		1
50	1439097-0442	Swing Away complete 80° left SB46 UL medium, night shimmer	2009-01-01 →	1
50	1439097-3524	Swing Away complete 80° left SB46 UL medium, royal red		1
50	1439097-5969	Swing Away complete 80° left SB46 UL medium, royal blue		1
50	1439097-9005	Swing Away complete 80° left SB46 UL medium, black, glossy		1
50	1439097-9119	Swing Away complete 80° left SB46 UL medium, black, bronze		1
51	1439098-0419	Swing Away complete 80° left SB48 UL medium, silver grey		1
51	1439098-0442	Swing Away complete 80° left SB48 UL medium, night shimmer	2009-01-01 →	1
51	1439098-3524	Swing Away complete 80° left SB48 UL medium, royal red		1
51	1439098-5969	Swing Away complete 80° left SB48 UL medium, royal blue		1
51	1439098-9005	Swing Away complete 80° left SB48 UL medium, black, glossy		1
51	1439098-9119	Swing Away complete 80° left SB48 UL medium, black, bronze		1



**Legrest hanger 80°**


Pos.	Part number	Description	Valid from → until	Qty
52	1439099-0419	Swing Away complete 80° left SB50 UL medium, silver grey		1
52	1439099-0442	Swing Away complete 80° left SB50 UL medium, night shimmer	2009-01-01 →	1
52	1439099-3524	Swing Away complete 80° left SB50 UL medium, royal red		1
52	1439099-5969	Swing Away complete 80° left SB50 UL medium, royal blue		1
52	1439099-9005	Swing Away complete 80° left SB50 UL medium, black, glossy		1
52	1439099-9119	Swing Away complete 80° left SB50 UL medium, black, bronze		1
53	1438601-0419	Swing Away complete 80° left SB32 UL long, silver grey		1
53	1438601-0442	Swing Away complete 80° left SB32 UL long, night shimmer	2009-01-01 →	1
53	1438601-3524	Swing Away complete 80° left SB32 UL long, royal red		1
53	1438601-5969	Swing Away complete 80° left SB32 UL long, royal blue		1
53	1438601-9005	Swing Away complete 80° left SB32 UL long, black, glossy		1
53	1438601-9119	Swing Away complete 80° left SB32 UL long, black, bronze		1
54	1438602-0419	Swing Away complete 80° left SB34 UL long, silver grey		1
54	1438602-0442	Swing Away complete 80° left SB34 UL long, night shimmer	2009-01-01 →	1
54	1438602-3524	Swing Away complete 80° left SB34 UL long, royal red		1
54	1438602-5969	Swing Away complete 80° left SB34 UL long, royal blue		1
54	1438602-9005	Swing Away complete 80° left SB34 UL long, black, glossy		1
54	1438602-9119	Swing Away complete 80° left SB34 UL long, black, bronze		1
55	1438603-0419	Swing Away complete 80° left SB36 UL long, silver grey		1
55	1438603-0442	Swing Away complete 80° left SB36 UL long, night shimmer	2009-01-01 →	1
55	1438603-3524	Swing Away complete 80° left SB36 UL long, royal red		1
55	1438603-5969	Swing Away complete 80° left SB36 UL long, royal blue		1
55	1438603-9005	Swing Away complete 80° left SB36 UL long, black, glossy		1
55	1438603-9119	Swing Away complete 80° left SB36 UL long, black, bronze		1
56	1438604-0419	Swing Away complete 80° left SB38 UL long, silver grey		1
56	1438604-0442	Swing Away complete 80° left SB38 UL long, night shimmer	2009-01-01 →	1
56	1438604-3524	Swing Away complete 80° left SB38 UL long, royal red		1
56	1438604-5969	Swing Away complete 80° left SB38 UL long, royal blue		1
56	1438604-9005	Swing Away complete 80° left SB38 UL long, black, glossy		1
56	1438604-9119	Swing Away complete 80° left SB38 UL long, black, bronze		1
57	1438605-0419	Swing Away complete 80° left SB40 UL long, silver grey		1
57	1438605-0442	Swing Away complete 80° left SB40 UL long, night shimmer	2009-01-01 →	1
57	1438605-3524	Swing Away complete 80° left SB40 UL long, royal red		1
57	1438605-5969	Swing Away complete 80° left SB40 UL long, royal blue		1
57	1438605-9005	Swing Away complete 80° left SB40 UL long, black, glossy		1
57	1438605-9119	Swing Away complete 80° left SB40 UL long, black, bronze		1
58	1438606-0419	Swing Away complete 80° left SB42 UL long, silver grey		1
58	1438606-0442	Swing Away complete 80° left SB42 UL long, night shimmer	2009-01-01 →	1
58	1438606-3524	Swing Away complete 80° left SB42 UL long, royal red		1
58	1438606-5969	Swing Away complete 80° left SB42 UL long, royal blue		1
58	1438606-9005	Swing Away complete 80° left SB42 UL long, black, glossy		1
58	1438606-9119	Swing Away complete 80° left SB42 UL long, black, bronze		1
59	1438607-0419	Swing Away complete 80° left SB44 UL long, silver grey		1
59	1438607-0442	Swing Away complete 80° left SB44 UL long, night shimmer	2009-01-01 →	1
59	1438607-3524	Swing Away complete 80° left SB44 UL long, royal red		1
59	1438607-5969	Swing Away complete 80° left SB44 UL long, royal blue		1
59	1438607-9005	Swing Away complete 80° left SB44 UL long, black, glossy		1
59	1438607-9119	Swing Away complete 80° left SB44 UL long, black, bronze		1
60	1438608-0419	Swing Away complete 80° left SB46 UL long, silver grey		1
60	1438608-0442	Swing Away complete 80° left SB46 UL long, night shimmer	2009-01-01 →	1
60	1438608-3524	Swing Away complete 80° left SB46 UL long, royal red		1
60	1438608-5969	Swing Away complete 80° left SB46 UL long, royal blue		1
60	1438608-9005	Swing Away complete 80° left SB46 UL long, black, glossy		1
60	1438608-9119	Swing Away complete 80° left SB46 UL long, black, bronze		1
61	1438609-0419	Swing Away complete 80° left SB48 UL long, silver grey		1

**Legrest hanger 80°**


Pos.	Part number	Description	Valid from → until	Qty
61	1438609-0442	Swing Away complete 80° left SB48 UL long, night shimmer	2009-01-01 →	1
61	1438609-3524	Swing Away complete 80° left SB48 UL long, royal red		1
61	1438609-5969	Swing Away complete 80° left SB48 UL long, royal blue		1
61	1438609-9005	Swing Away complete 80° left SB48 UL long, black, glossy		1
61	1438609-9119	Swing Away complete 80° left SB48 UL long, black, bronze		1
62	1438610-0419	Swing Away complete 80° left SB50 UL long, silver grey		1
62	1438610-0442	Swing Away complete 80° left SB50 UL long, night shimmer	2009-01-01 →	1
62	1438610-3524	Swing Away complete 80° left SB50 UL long, royal red		1
62	1438610-5969	Swing Away complete 80° left SB50 UL long, royal blue		1
62	1438610-9005	Swing Away complete 80° left SB50 UL long, black, glossy		1
62	1438610-9119	Swing Away complete 80° left SB50 UL long, black, bronze		1
63	1510271	Locking mechanism for swing-away, kit right		1
64	1510272	Locking mechanism for swing-away, kit left		1



# Legrest hanger height adjustable

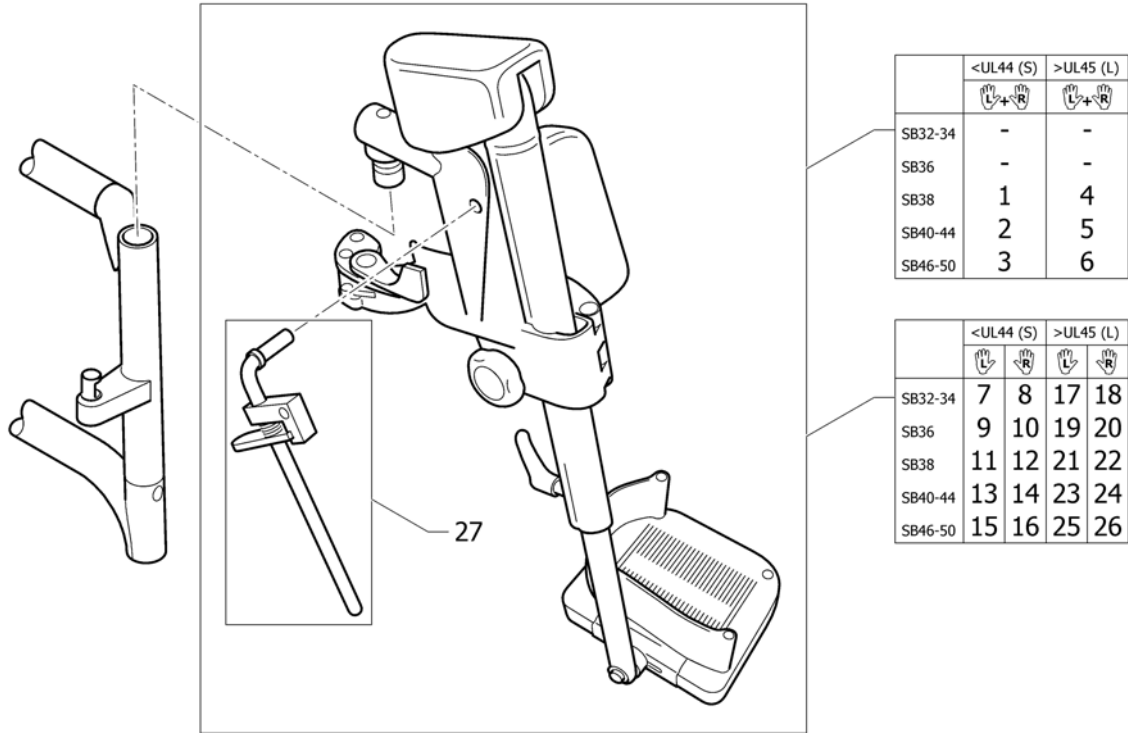


Pos.	Part number	Description	Valid from → until	Qty
1	1517819	Legrest hanger, height adjustable, pair, SB38 short		1
2	1517820	Legrest hanger, height adjustable, pair, SB40-44 short		1
3	1517821	Legrest hanger, height adjustable, pair, SB46-50 short		1
4	1517822	Legrest hanger, height adjustable, pair, SB38 long		1
5	1517823	Legrest hanger, height adjustable, pair, SB40-44 long		1
6	1517824	Legrest hanger, height adjustable, pair, SB46-50 long		1
7	1517694	Legrest hanger, height adjustable, left, SB32-34 short		1
8	1517695	Legrest hanger, height adjustable, right, SB32-34 short		1
9	1517696	Legrest hanger, height adjustable, left, SB36 short		1
10	1517697	Legrest hanger, height adjustable, right, SB36 short		1
11	1517698	Legrest hanger, height adjustable, left, SB38 short		1
12	1517699	Legrest hanger, height adjustable, right, SB38 short		1
13	1517700	Legrest hanger, height adjustable, left, SB40-44 short		1
14	1517701	Legrest hanger, height adjustable, right, SB40-44 short		1
15	1517702	Legrest hanger, height adjustable, left, SB46-50 short		1
16	1517703	Legrest hanger, height adjustable, right, SB46-50 short		1
17	1517704	Legrest hanger, height adjustable, left, SB32-34 long		1
18	1517705	Legrest hanger, height adjustable, right, SB32-34 long		1
19	1517706	Legrest hanger, height adjustable, left, SB36 long		1
20	1517707	Legrest hanger, height adjustable, right, SB36 long		1
21	1517708	Legrest hanger, height adjustable, left, SB38 long		1
22	1517709	Legrest hanger, height adjustable, right, SB38 long		1
23	1517710	Legrest hanger, height adjustable, left, SB40-44 long		1
24	1517711	Legrest hanger, height adjustable, right, SB40-44 long		1
25	1517712	Legrest hanger, height adjustable, left, SB46-50 long		1
26	1517713	Legrest hanger, height adjustable, right, SB46-50 long		1
27	1507623	Adjustment mechanism kit to leg rest hanger, height adjustable		2

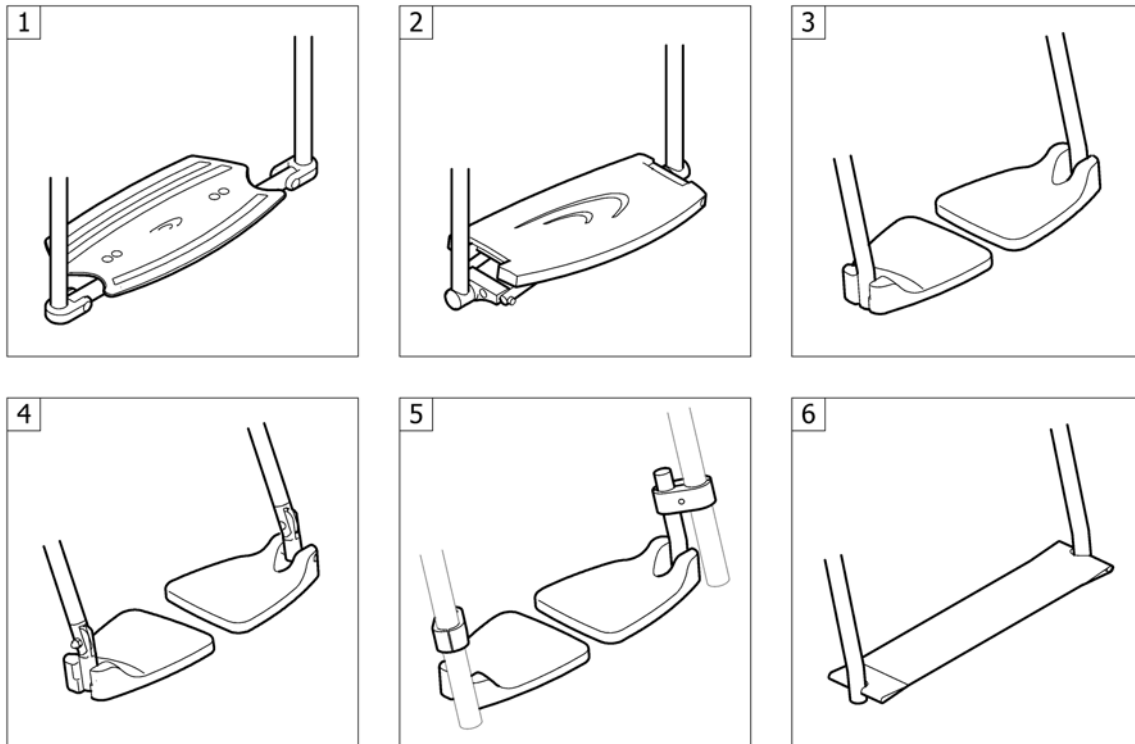**FOOTREST**

Pos.	Part number	Description	Valid from → until	Qty
1	-	Footrest 1-piece	2004-01-01 → 2012-12-31	1
2	-	Footrest 1-piece, foldable	2004-01-01 → 2012-12-31	1
3	-	Footrest 2-piece	2004-01-01 → 2012-12-31	1
4	-	Footrest 2-piece, angle adjustable	2004-01-01 → 2012-12-31	1
5	-	Footrest high mounted	2008-07-01 → 2012-12-31	1
6	-	Footband	2004-01-01 → 2012-12-31	1

# Footrest 1-piece

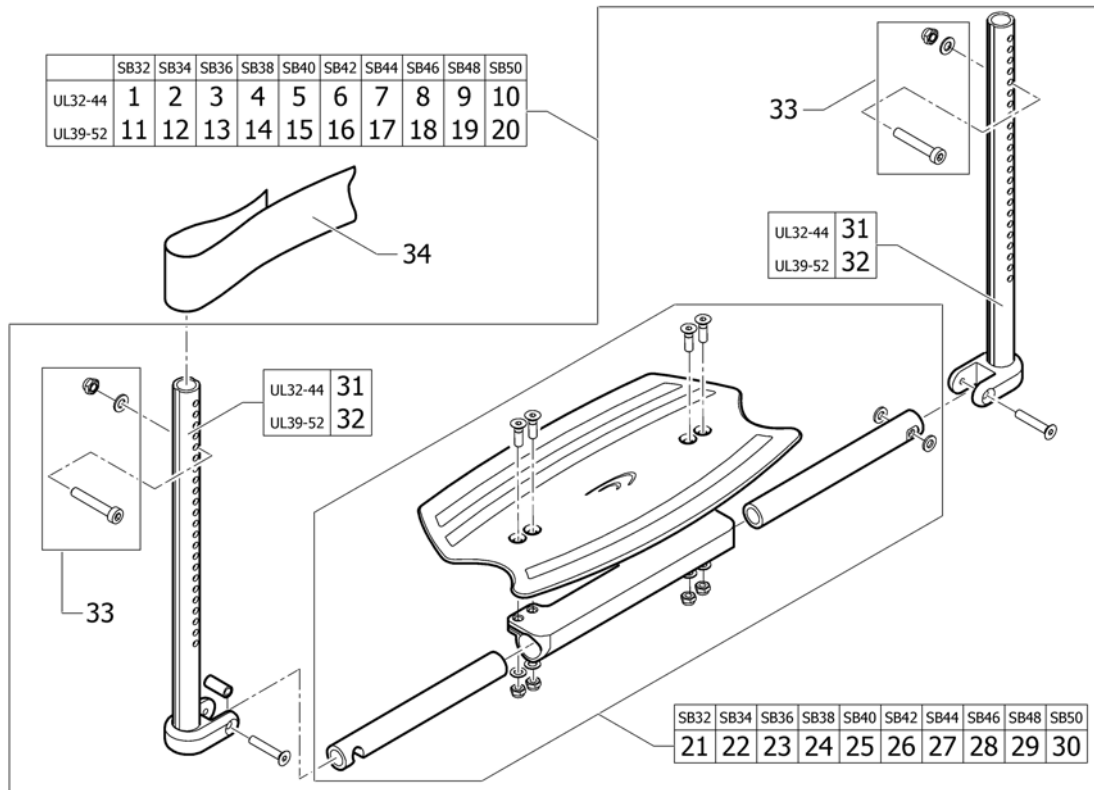


Pos.	Part number	Description	Valid from → until	Qty
1	1493772	Footrest 1-piece complete short SB32		1
2	1493773	Footrest 1-piece complete short SB34		1
3	1493774	Footrest 1-piece complete short SB36		1
4	1493775	Footrest 1-piece complete short SB38		1
5	1493776	Footrest 1-piece complete short SB40		1
6	1493777	Footrest 1-piece complete short SB42		1
7	1493778	Footrest 1-piece complete short SB44		1
8	1493779	Footrest 1-piece complete short SB46		1
9	1493780	Footrest 1-piece complete short SB48		1
10	1493781	Footrest 1-piece complete short SB50		1
11	1493782	Footrest 1-piece complete long SB32		1
12	1493783	Footrest 1-piece complete long SB34		1
13	1493784	Footrest 1-piece complete long SB36		1
14	1493785	Footrest 1-piece complete long SB38		1
15	1493786	Footrest 1-piece complete long SB40		1
16	1493787	Footrest 1-piece complete long SB42		1
17	1493788	Footrest 1-piece complete long SB44		1
18	1493789	Footrest 1-piece complete long SB46		1
19	1493790	Footrest 1-piece complete long SB48		1
20	1493791	Footrest 1-piece complete long SB50		1
21	1493758	Footrest 1-piece SB32		1
22	1493759	Footrest 1-piece SB34		1
23	1493760	Footrest 1-piece SB36		1
24	1493761	Footrest 1-piece SB38		1
25	1493762	Footrest 1-piece SB40		1
26	1493763	Footrest 1-piece SB42		1
27	1493764	Footrest 1-piece SB44		1
28	1493765	Footrest 1-piece SB46		1
29	1493766	Footrest 1-piece SB48		1
30	1493767	Footrest 1-piece SB50		1
31	1493756	Telescopic tube kit for footrest 1-piece short		1

**Footrest 1-piece**


<b>Pos.</b>	<b>Part number</b>	<b>Description</b>	<b>Valid from → until</b>	<b>Qty</b>
32	1493757	Telescopic tube kit for footrest 1-piece long		1
33	1437989	fixation kit footplate telescope		1
34	1517817	Calfstrap black		1

## Footrest 1-piece, foldable

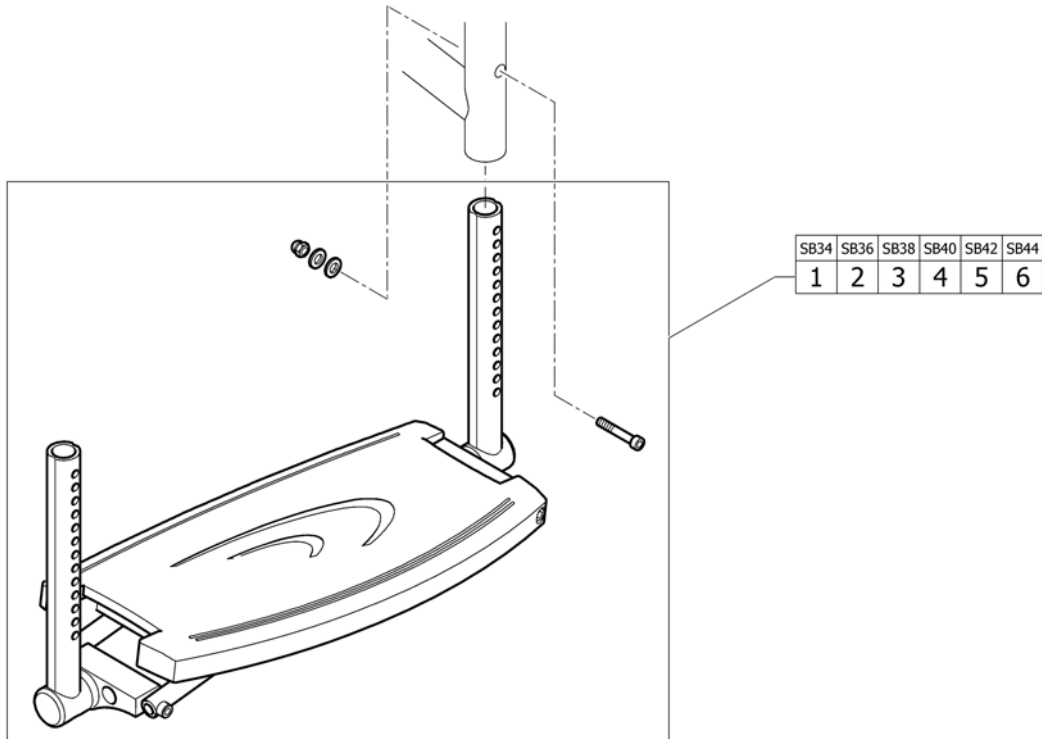

Pos.	Part number	Description	Valid from → until	Qty
1	1488733	footrest SB34 1pc. Scissor		1
2	1488734	footrest SB36 1pc. Scissor		1
3	1488735	footrest SB38 1pc. Scissor		1
4	1451756	footrest SB40 1pc. Scissor		1
5	1451757	footrest SB42 1pc. Scissor		1
6	1451758	footrest SB44 1pc. Scissor		1

# Footrest 2-piece

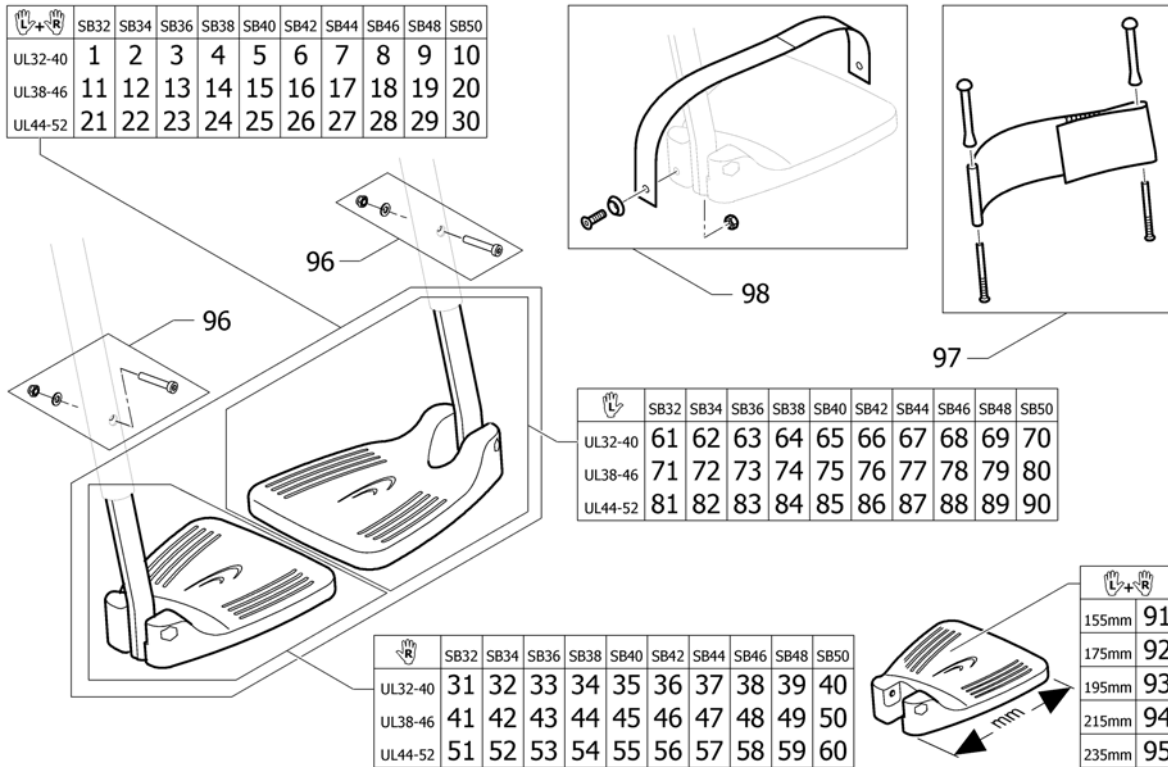

Pos.	Part number	Description	Valid from → until	Qty
1	1437990	footrest 2-pc. complete, pair SB32 short UL/C04		1
2	1437991	footrest 2-pc. complete, pair SB34 short UL/C04		1
3	1437992	footrest 2-pc. complete, pair SB36 short UL/C04		1
4	1437993	footrest 2-pc. complete, pair SB38 short UL/C04		1
5	1437994	footrest 2-pc. complete, pair SB40 short UL/C04		1
6	1437995	footrest 2-pc. complete, pair SB42 short UL/C04		1
7	1437996	footrest 2-pc. complete, pair SB44 short UL/C04		1
8	1437997	footrest 2-pc. complete, pair SB46 short UL/C04		1
9	1437998	footrest 2-pc. complete, pair SB48 short UL/C04		1
10	1437999	footrest 2-pc. complete, pair SB50 short UL/C04		1
11	1438831	pair footrest 2-pc. complete SB32 medium		1
12	1438832	pair footrest 2-pc. complete SB34 medium		1
13	1438833	pair footrest 2-pc. complete SB36 medium		1
14	1438834	pair footrest 2-pc. complete SB38 medium		1
15	1438835	pair footrest 2-pc. complete SB40 medium		1
16	1438836	pair footrest 2-pc. complete SB42 medium		1
17	1438837	pair footrest 2-pc. complete SB44 medium		1
18	1438838	pair footrest 2-pc. complete SB46 medium		1
19	1438839	pair footrest 2-pc. complete SB48 medium		1
20	1438840	pair footrest 2-pc. complete SB50 medium		1
21	1438000	footrest 2-pc. complete, pair SB32 long UL/C04		1
22	1438001	footrest 2-pc. complete, pair SB34 long UL/C04		1
23	1438002	footrest 2-pc. complete, pair SB36 long UL/C04		1
24	1438003	footrest 2-pc. complete, pair SB38 long UL/C04		1
25	1438004	footrest 2-pc. complete, pair SB40 long UL/C04		1
26	1438005	footrest 2-pc. complete, pair SB42 long UL/C04		1
27	1438006	footrest 2-pc. complete, pair SB44 long UL/C04		1
28	1438007	footrest 2-pc. complete, pair SB46 long UL/C04		1
29	1438008	footrest 2-pc. complete, pair SB48 long UL/C04		1
30	1438009	footrest 2-pc. complete, pair SB50 long UL/C04		1
31	1438010	footrest 2-pc. SB32 right short complete		1




**Footrest 2-piece**

Pos.	Part number	Description	Valid from → until	Qty
32	1438012	footrest 2-pc. SB34 right short complete		1
33	1438014	footrest 2-pc. SB36 right short complete		1
34	1438016	footrest 2-pc. SB38 right short complete		1
35	1438018	footrest 2-pc. SB40 right short complete		1
36	1438020	footrest 2-pc. SB42 right short complete		1
37	1438022	footrest 2-pc. SB44 right short complete		1
38	1438024	footrest 2-pc. SB46 right short complete		1
39	1438026	footrest 2-pc. SB48 right short complete		1
40	1438028	footrest 2-pc. SB50 right short complete		1
41	1438841	footrest 2-pc. complete SB32 right medium		1
42	1438843	footrest 2-pc. complete SB34 right medium		1
43	1438845	footrest 2-pc. complete SB36 right medium		1
44	1438847	footrest 2-pc. complete SB38 right medium		1
45	1438849	footrest 2-pc. complete SB40 right medium		1
46	1438851	footrest 2-pc. complete SB42 right medium		1
47	1438853	footrest 2-pc. complete SB44 right medium		1
48	1438855	footrest 2-pc. complete SB46 right medium		1
49	1438857	footrest 2-pc. complete SB48 right medium		1
50	1438859	footrest 2-pc. complete SB50 right medium		1
51	1438030	footrest 2-pc. SB32 right long complete		1
52	1438032	footrest 2-pc. SB34 right long complete		1
53	1438034	footrest 2-pc. SB36 right long complete		1
54	1438036	footrest 2-pc. SB38 right long complete		1
55	1438038	footrest 2-pc. SB40 right long complete		1
56	1438040	footrest 2-pc. SB42 right long complete		1
57	1438042	footrest 2-pc. SB44 right long complete		1
58	1438044	footrest 2-pc. SB46 right long complete		1
59	1438046	footrest 2-pc. SB48 right long complete		1
60	1438048	footrest 2-pc. SB50 right long complete		1
61	1438011	footrest 2-pc. SB32 left short complete		1
62	1438013	footrest 2-pc. SB34 left short complete		1
63	1438015	footrest 2-pc. SB36 left short complete		1
64	1438017	footrest 2-pc. SB38 left short complete		1
65	1438019	footrest 2-pc. SB40 left short complete		1
66	1438021	footrest 2-pc. SB42 left short complete		1
67	1438023	footrest 2-pc. SB44 left short complete		1
68	1438025	footrest 2-pc. SB46 left short complete		1
69	1438027	footrest 2-pc. SB48 left short complete		1
70	1438029	footrest 2-pc. SB50 left short complete		1
71	1438842	footrest 2-pc. complete SB32 left medium		1
72	1438844	footrest 2-pc. complete SB34 left medium		1
73	1438846	footrest 2-pc. complete SB36 left medium		1
74	1438848	footrest 2-pc. complete SB38 left medium		1
75	1438850	footrest 2-pc. complete SB40 left medium		1
76	1438852	footrest 2-pc. complete SB42 left medium		1
77	1438854	footrest 2-pc. complete SB44 left medium		1
78	1438856	footrest 2-pc. complete SB46 left medium		1
79	1438858	footrest 2-pc. complete SB48 left medium		1
80	1438860	footrest 2-pc. complete SB50 left medium		1
81	1438031	footrest 2-pc. SB32 left long complete		1
82	1438033	footrest 2-pc. SB34 left long complete		1
83	1438035	footrest 2-pc. SB36 left long complete		1
84	1438037	footrest 2-pc. SB38 left long complete		1
85	1438039	footrest 2-pc. SB40 left long complete		1
86	1438041	footrest 2-pc. SB42 left long complete		1

**Footrest 2-piece**

Pos.	Part number	Description	Valid from → until	Qty
87	1438043	footrest 2-pc. SB44 left long complete		1
88	1438045	footrest 2-pc. SB46 left long complete		1
89	1438047	footrest 2-pc. SB48 left long complete		1
90	1438049	footrest 2-pc. SB50 left long complete		1
91	1579331	Footrest 2-pc. 155mm, pair <i>Replaces 1432350 and 1432351</i>		1
92	1579332	Footrest 2-pc. 175mm, pair <i>Replaces 1432352 and 1432353</i>		1
93	1579333	Footrest 2-pc. 195mm, pair <i>Replaces 1432354 and 1432355</i>		1
94	1579334	Footrest 2-pc. 215mm, pair <i>Replaces 1432356 and 1432357</i>		1
95	1579335	Footrest 2-pc. 235mm, pair <i>Replaces 1432358 and 1432359</i>		1
96	1437989	fixation kit footplate telescope		1
97	1437834	Heel loop kit short		1
98	1438143	instep strap UL/C04		1

# Footrest 2-piece, angle adjustable

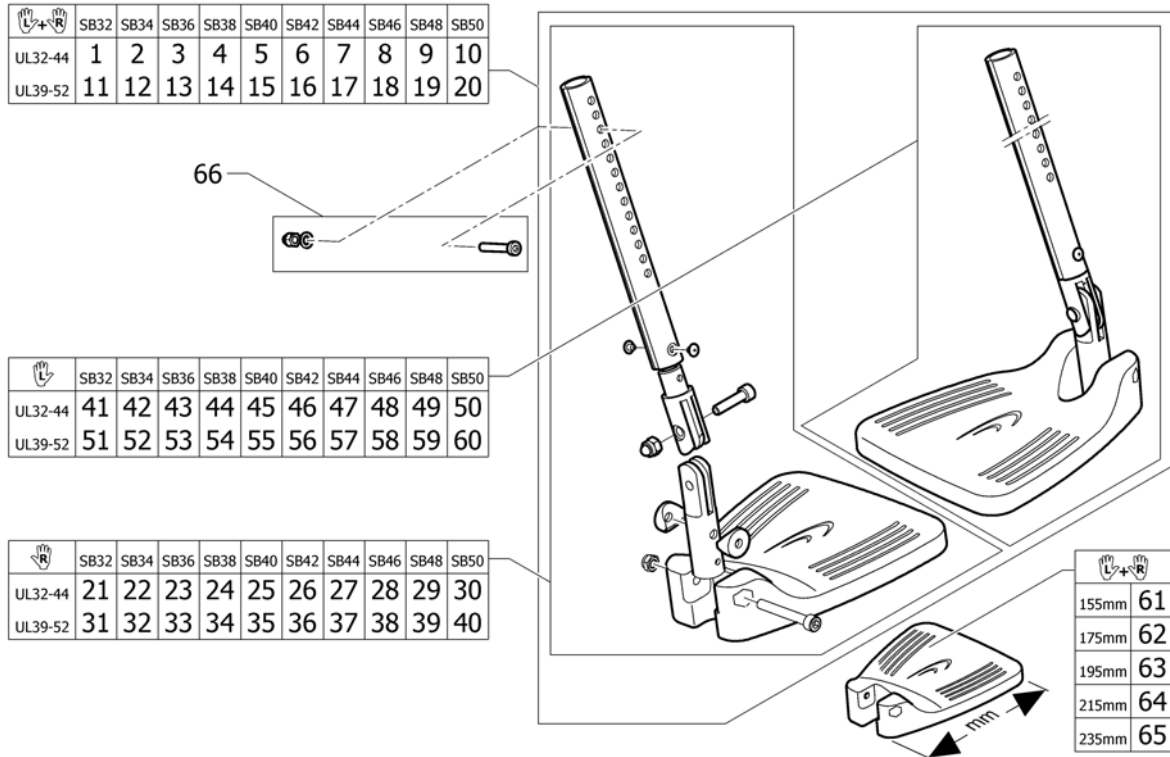


Pos.	Part number	Description	Valid from → until	Qty
1	1438050	pair footrest 2-pc. angle adjustable complete SB32 short		1
2	1438051	pair footrest 2-pc. angle adjustable complete SB34 short		1
3	1438052	pair footrest 2-pc. angle adjustable complete SB36 short		1
4	1438053	pair footrest 2-pc. angle adjustable complete SB38 short		1
5	1438054	pair footrest 2-pc. angle adjustable complete SB40 short		1
6	1438055	pair footrest 2-pc. angle adjustable complete SB42 short		1
7	1438056	pair footrest 2-pc. angle adjustable complete SB44 short		1
8	1438057	pair footrest 2-pc. angle adjustable complete SB46 short		1
9	1438058	pair footrest 2-pc. angle adjustable complete SB48 short		1
10	1438059	pair footrest 2-pc. angle adjustable complete SB50 short		1
11	1438060	pair footrest 2-pc. angle adjustable complete SB32 long		1
12	1438061	pair footrest 2-pc. angle adjustable complete SB34 long		1
13	1438062	pair footrest 2-pc. angle adjustable complete SB36 long		1
14	1438063	pair footrest 2-pc. angle adjustable complete SB38 long		1
15	1438064	pair footrest 2-pc. angle adjustable complete SB40 long		1
16	1438065	pair footrest 2-pc. angle adjustable complete SB42 long		1
17	1438066	pair footrest 2-pc. angle adjustable complete SB44 long		1
18	1438067	pair footrest 2-pc. angle adjustable complete SB46 long		1
19	1438068	pair footrest 2-pc. angle adjustable complete SB48 long		1
20	1438069	pair footrest 2-pc. angle adjustable complete SB50 long		1
21	1438070	footrest 2-pc. angle adjustable complete SB32 short right		1
22	1438072	footrest 2-pc. angle adjustable complete SB34 short right		1
23	1438074	footrest 2-pc. angle adjustable complete SB36 short right		1
24	1438076	footrest 2-pc. angle adjustable complete SB38 short right		1
25	1438078	footrest 2-pc. angle adjustable complete SB40 short right		1
26	1438080	footrest 2-pc. angle adjustable complete SB42 short right		1
27	1438082	footrest 2-pc. angle adjustable complete SB44 short right		1
28	1438084	footrest 2-pc. angle adjustable complete SB46 short right		1
29	1438086	footrest 2-pc. angle adjustable complete SB48 short right		1
30	1438088	footrest 2-pc. angle adjustable complete SB50 short right		1
31	1438090	footrest 2-pc. angle adjustable complete SB32 long right		1

**Footrest 2-piece, angle adjustable**

Pos.	Part number	Description	Valid from → until	Qty
32	1438092	footrest 2-pc. angle adjustable complete SB34 long right		1
33	1438094	footrest 2-pc. angle adjustable complete SB36 long right		1
34	1438096	footrest 2-pc. angle adjustable complete SB38 long right		1
35	1438098	footrest 2-pc. angle adjustable complete SB40 long right		1
36	1438100	footrest 2-pc. angle adjustable complete SB42 long right		1
37	1438102	footrest 2-pc. angle adjustable complete SB44 long right		1
38	1438104	footrest 2-pc. angle adjustable complete SB46 long right		1
39	1438106	footrest 2-pc. angle adjustable complete SB48 long right		1
40	1438108	footrest 2-pc. angle adjustable complete SB50 long right		1
41	1438071	footrest 2-pc. angle adjustable complete SB32 short left.		1
42	1438073	footrest 2-pc. angle adjustable complete SB34 short left.		1
43	1438075	footrest 2-pc. angle adjustable complete SB36 short left.		1
44	1438077	footrest 2-pc. angle adjustable complete SB38 short left.		1
45	1438079	footrest 2-pc. angle adjustable complete SB40 short left.		1
46	1438081	footrest 2-pc. angle adjustable complete SB42 short left.		1
47	1438083	footrest 2-pc. angle adjustable complete SB44 short left.		1
48	1438085	footrest 2-pc. angle adjustable complete SB46 short left.		1
49	1438087	footrest 2-pc. angle adjustable complete SB48 short left.		1
50	1438089	footrest 2-pc. angle adjustable complete SB50 short left.		1
51	1438091	footrest 2-pc. angle adjustable complete SB32 long left.		1
52	1438093	footrest 2-pc. angle adjustable complete SB34 long left.		1
53	1438095	footrest 2-pc. angle adjustable complete SB36 long left.		1
54	1438097	footrest 2-pc. angle adjustable complete SB38 long left.		1
55	1438099	footrest 2-pc. angle adjustable complete SB40 long left.		1
56	1438101	footrest 2-pc. angle adjustable complete SB42 long left.		1
57	1438103	footrest 2-pc. angle adjustable complete SB44 long left.		1
58	1438105	footrest 2-pc. angle adjustable complete SB46 long left.		1
59	1438107	footrest 2-pc. angle adjustable complete SB48 long left.		1
60	1438109	footrest 2-pc. angle adjustable complete SB50 long left.		1
61	1579331	Footrest 2-pc. 155mm, pair <i>Replaces 1432350 and 1432351</i>		1
62	1579332	Footrest 2-pc. 175mm, pair <i>Replaces 1432352 and 1432353</i>		1
63	1579333	Footrest 2-pc. 195mm, pair <i>Replaces 1432354 and 1432355</i>		1
64	1579334	Footrest 2-pc. 215mm, pair <i>Replaces 1432356 and 1432357</i>		1
65	1579335	Footrest 2-pc. 235mm, pair <i>Replaces 1432358 and 1432359</i>		1
66	1437989	fixation kit footplate telescope		1


# Footrest high mounted

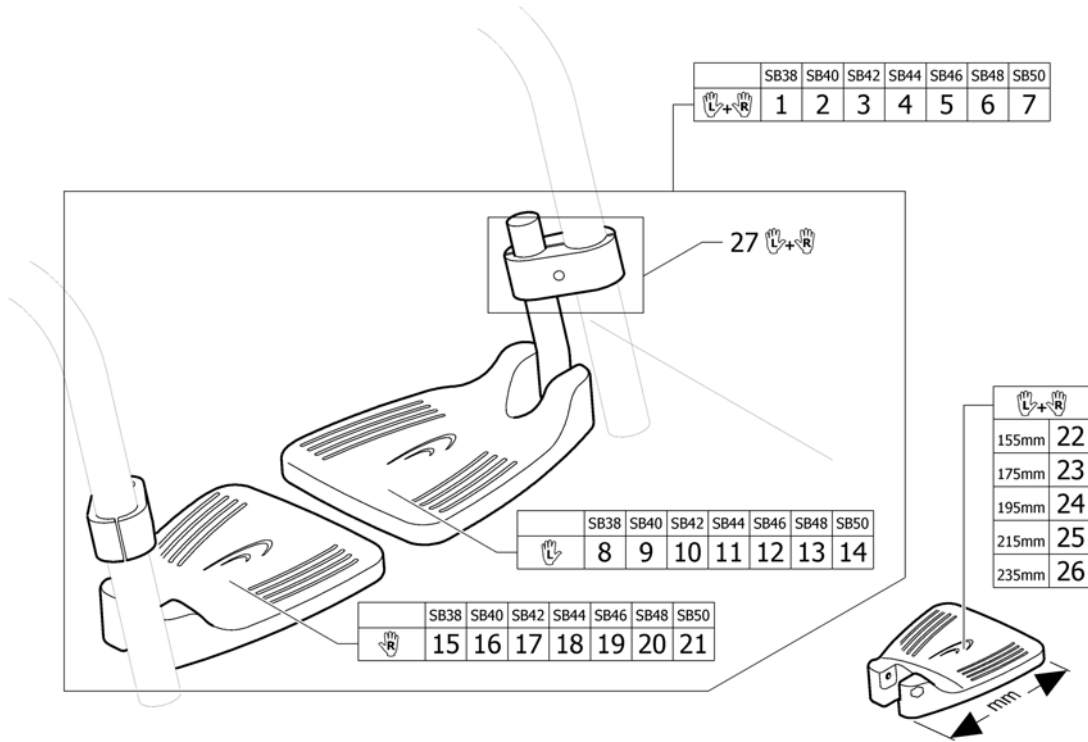

Pos.	Part number	Description	Valid from → until	Qty
1	1518152	Footrest 2-pc. high mounted SB38 complete,pair		1
2	1518153	Footrest 2-pc. high mounted SB40 complete,pair		1
3	1518154	Footrest 2-pc. high mounted SB42 complete,pair		1
4	1518155	Footrest 2-pc. high mounted SB44 complete,pair		1
5	1518156	Footrest 2-pc. high mounted SB46 complete,pair		1
6	1518157	Footrest 2-pc. high mounted SB48 complete,pair		1
7	1518158	Footrest 2-pc. high mounted SB50 complete,pair		1
8	1518174	Footrest 2-pc. high mounted SB38 left		1
9	1518175	Footrest 2-pc. high mounted SB40 left		1
10	1518176	Footrest 2-pc. high mounted SB42 left		1
11	1518177	Footrest 2-pc. high mounted SB44 left		1
12	1518236	Footrest 2-pc. high mounted SB46 left		1
13	1518282	Footrest 2-pc. high mounted SB48 left		1
14	1518283	Footrest 2-pc. high mounted SB50 left		1
15	1518159	Footrest 2-pc. high mounted SB38 right		1
16	1518160	Footrest 2-pc. high mounted SB40 right		1
17	1518161	Footrest 2-pc. high mounted SB42 right		1
18	1518162	Footrest 2-pc. high mounted SB44 right		1
19	1518163	Footrest 2-pc. high mounted SB46 right		1
20	1518164	Footrest 2-pc. high mounted SB48 right		1
21	1518165	Footrest 2-pc. high mounted SB50 right		1
22	1579331	Footrest 2-pc. 155mm, pair <i>Replaces 1432350 and 1432351</i>		1
23	1579332	Footrest 2-pc. 175mm, pair <i>Replaces 1432352 and 1432353</i>		1
24	1579333	Footrest 2-pc. 195mm, pair <i>Replaces 1432354 and 1432355</i>		1
25	1579334	Footrest 2-pc. 215mm, pair <i>Replaces 1432356 and 1432357</i>		1
26	1579335	Footrest 2-pc. 235mm, pair <i>Replaces 1432358 and 1432359</i>		1
27	1518284	Clamping kit for footrest high mounted, pair <i>Replaces 1439132</i>		1

**Footband**


	SB32-34	SB36-38	SB40-42	SB44-46	SB48-50
UL32-40	1	2	3	4	5
UL38-46	6	7	8	9	10
UL44-52	11	12	13	14	15

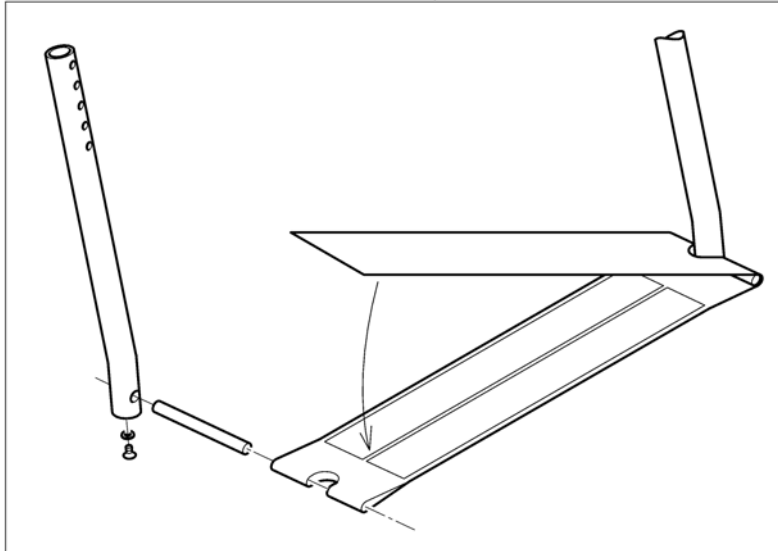

Pos.	Part number	Description	Valid from → until	Qty
1	1438127	Footstrap complete SB32-34 short		1
2	1438128	Footstrap complete SB36-38 short		1
3	1438129	Footstrap complete SB40-42 short		1
4	1438130	Footstrap complete SB44-46 short		1
5	1438131	Footstrap complete SB48-50 short		1
6	1439461	foot strap complete seat width 32-34 medium		1
7	1439462	foot strap complete seat width 36-38 medium		1
8	1439463	foot strap complete seat width 40-42 medium		1
9	1439464	foot strap complete seat width 44-46 medium		1
10	1439465	foot strap complete seat width 48-50 medium		1
11	1438122	Footstrap complete SB32-34 long		1
12	1438123	Footstrap complete SB36-38 long		1
13	1438124	Footstrap complete SB40-42 long		1
14	1438125	Footstrap complete SB44-46 long		1
15	1438126	Footstrap complete SB48-50 long		1




# SIDERESTS

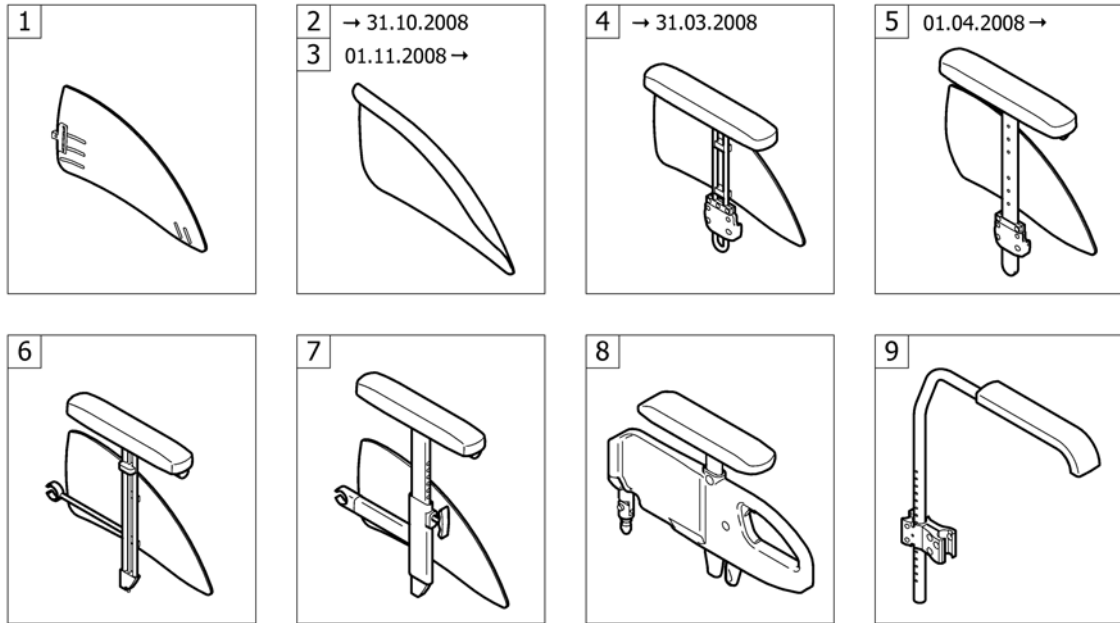


Pos.	Part number	Description	Valid from → until	Qty
1	-	Clothes-guards	2004-01-01 → 2012-12-31	1
3	-	Mudguards 01.11.2008=>	2008-11-01 → 2012-12-31	1
5	-	Siderest height adjustable 01.04.2008=>	2008-04-01 → 2012-12-31	1
7	-	Siderest fold back, height adjustable	2004-01-01 → 2012-12-31	1
8	-	Siderest with desk function	2007-02-01 → 2012-12-31	1
9	-	Tubular armrest	2004-01-01 → 2012-12-31	1

# Clothes-guards

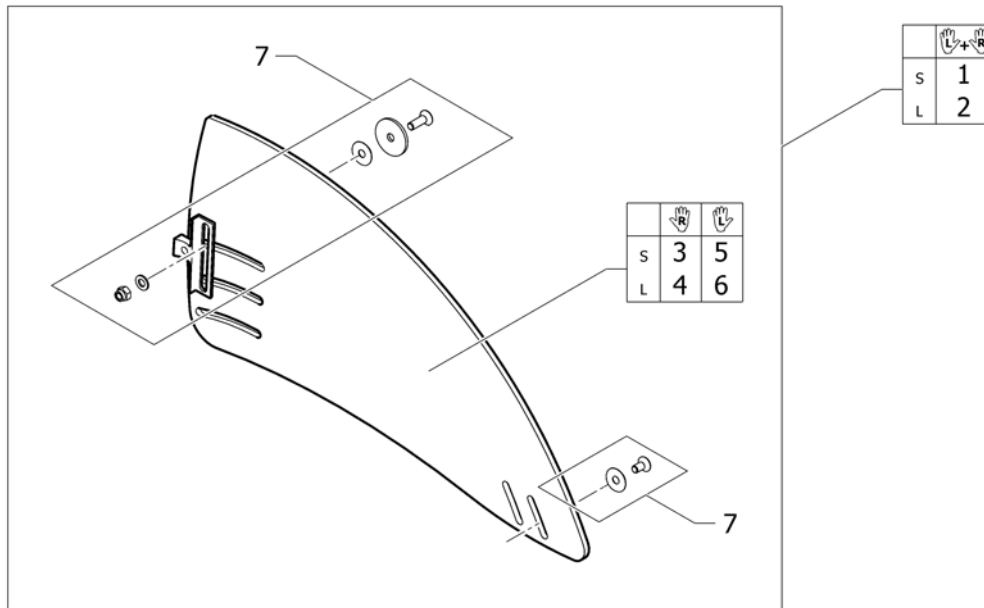


Pos.	Part number	Description	Valid from → until	Qty
1	1517825	Clothes-guards S with fixation kit, pair		1
2	1517826	Clothes-guards L with fixation kit, pair		1
3	1517828	Clothes-guard S right <i>Replaces 1432471</i>		1
4	1517830	Clothes-guard L right <i>Replaces 1432473</i>		1
5	1517827	Clothes-guard S left <i>Replaces 1432470</i>		1
6	1517829	Clothes-guard L left <i>Replaces 1432472</i>		1
7	1438144	Fixation kit for clothes-guard and mudguard		1

# Mudguards 01.11.2008=>

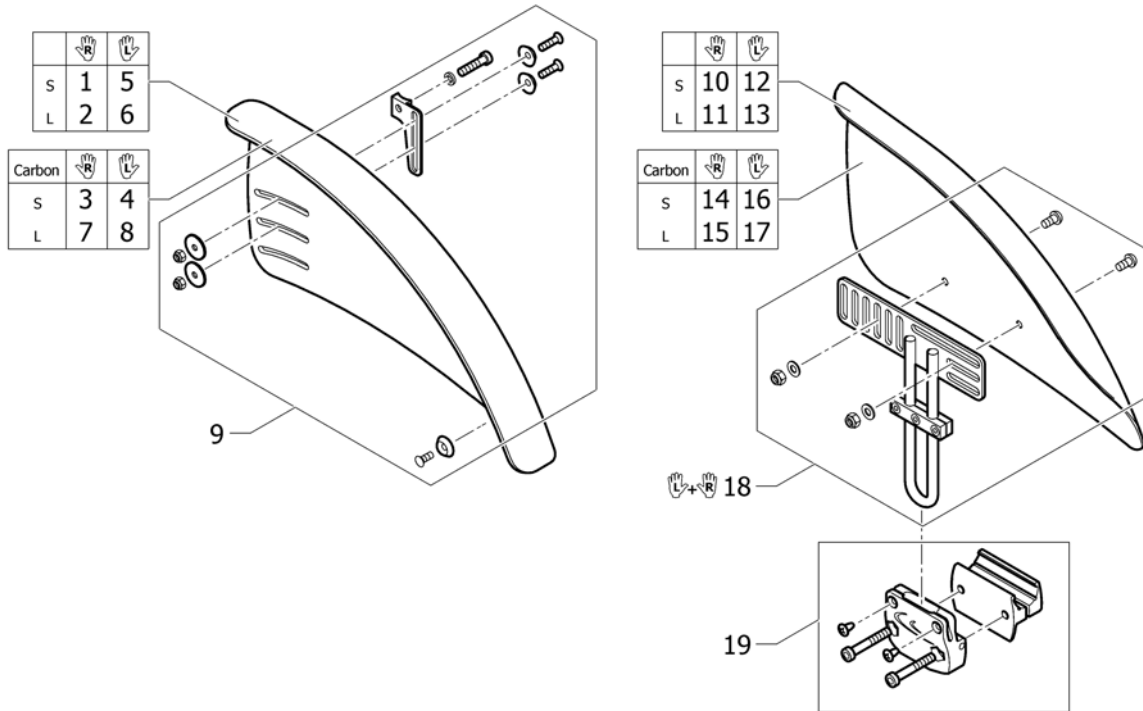


Pos.	Part number	Description	Valid from → until	Qty
1	1432475	Mudguard S right		1
2	1432477	Mudguard L right		1
3	1529782	Mudguard Carbon S short right <i>Replaces 1439160</i>		1
4	1529781	Mudguard Carbon S short left <i>Replaces 1439161</i>		1
5	1432474	Mudguard S left		1
6	1432476	Mudguard L left		1
7	1529784	Mudguard Carbon L long right <i>Replaces 1439162</i>		1
8	1529783	Mudguard Carbon L long left <i>Replaces 1439163</i>		1
9	1438144	Fixation kit for clothes-guard and mudguard		1
10	1438807	Mudguard S right removable		1
11	1438809	Mudguard L right removable		1
12	1438808	Mudguard S left removable		1
13	1438810	Mudguard L left removable		1
14	1510276	Mudguard Carbon right M	2008-11-01 →	1
15	1510278	Mudguard Carbon right L	2008-11-01 →	1
16	1510275	Mudguard Carbon left M	2008-11-01 →	1
17	1510277	Mudguard Carbon left L	2008-11-01 →	1
18	1438149	Fixation kit for mudguard removable		1
19	1439301	Sidepart Adaption complete		2

## Siderest height adjustable 01.04.2008=>

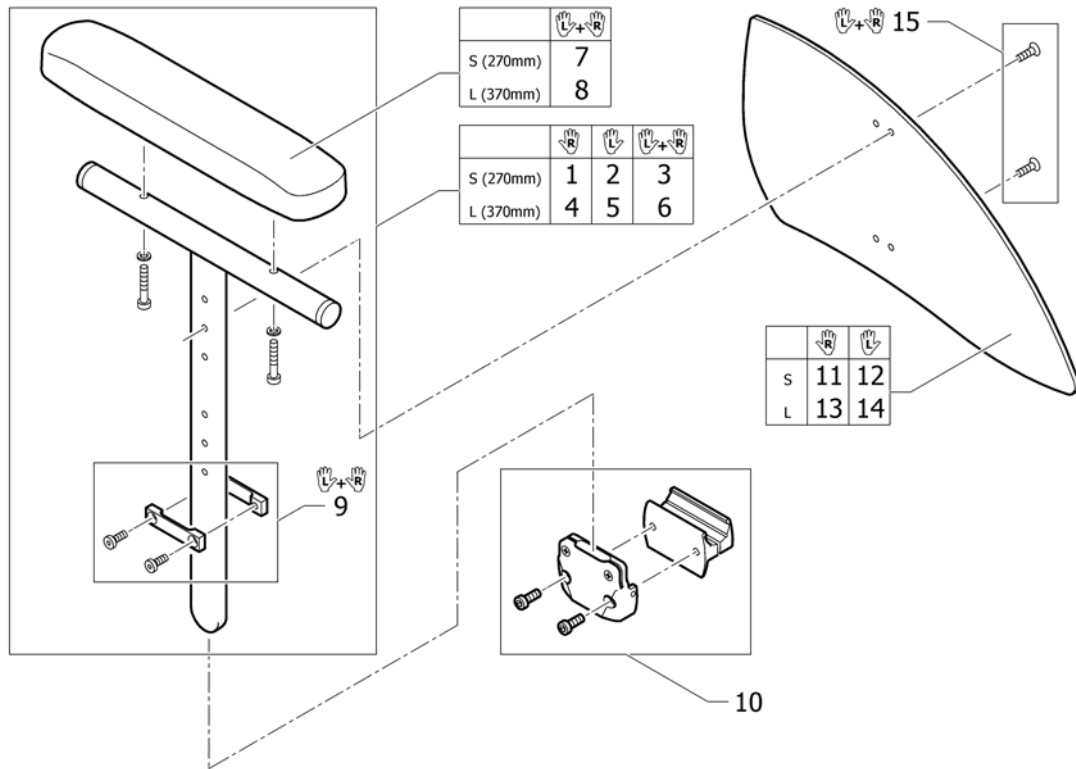


Pos.	Part number	Description	Valid from → until	Qty
1	1517677	Siderest height adjustable short, right		1
2	1517678	Siderest height adjustable short, left		1
3	1517682	Siderest height adjustable short, pair		1
4	1517679	Siderest height adjustable long, right		1
5	1517680	Siderest height adjustable long, left		1
6	1517683	Siderest height adjustable long, pair		1
7	1439497	Ampad S (270mm) incl. screws, pair		1
8	1439498	Ampad L (370mm) incl. screws, pair		1
9	1517681	Siderests height clamp kit, pair		1
10	1439301	Sidepart Adaption complete		2
11	1517690	Clothes-guard small right to siderest		1
12	1517689	Clothes-guard small left to siderest		1
13	1517692	Clothes-guard large right to siderest		1
14	1517691	Clothes-guard large left to siderest		1
15	1517693	Clothes-guard fixation kit		1

## Siderest fold back, height adjustable

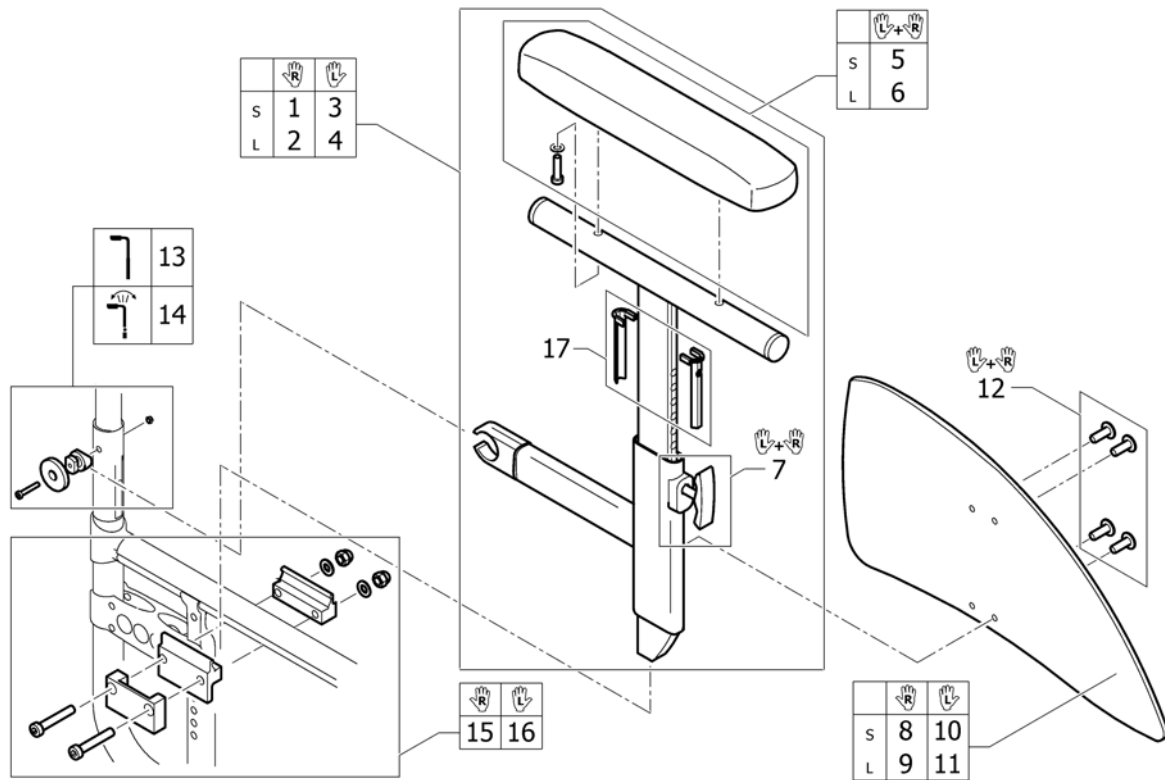

Pos.	Part number	Description	Valid from → until	Qty
1	1439577	Side rest height adjustable, flip back right short complete		1
2	1439579	Side rest height adjustable, flip back right long complete		1
3	1439578	Side rest height adjustable, flip back left short complete		1
4	1439580	Side rest height adjustable, flip back left long complete		1
5	1439497	Armpad S (270mm) incl. screws, pair		1
6	1439498	Armpad L (370mm) incl. screws, pair		1
7	1439581	Closing mechanism for Siderest height adj. Flip back		1
8	1517690	Clothes-guard small right to siderest <i>Replaces 1438732</i>		1
9	1517692	Clothes-guard large right to siderest <i>Replaces 1438734</i>		1
10	1517689	Clothes-guard small left to siderest <i>Replaces 1438731</i>		1
11	1517691	Clothes-guard large left to siderest <i>Replaces 1438733</i>		1
12	1445274	Fixation kit clothes-guard to armrest		1
13	1438159	Side rest flip-back fixation kit UL/C04		1
14	1439490	Fixation-Set for Siderest with angle adj. back		1
15	1439488	Click-In Adapterset RH to Siderest flip back		1
16	1439489	Click-In Adapterset LH to Siderest flip back		1
17	1579327	Guide part for side rest height adjustable, kit 20 Pcs. <i>Replaces 400.1084 (20x)</i>		1


## Siderest with desk function

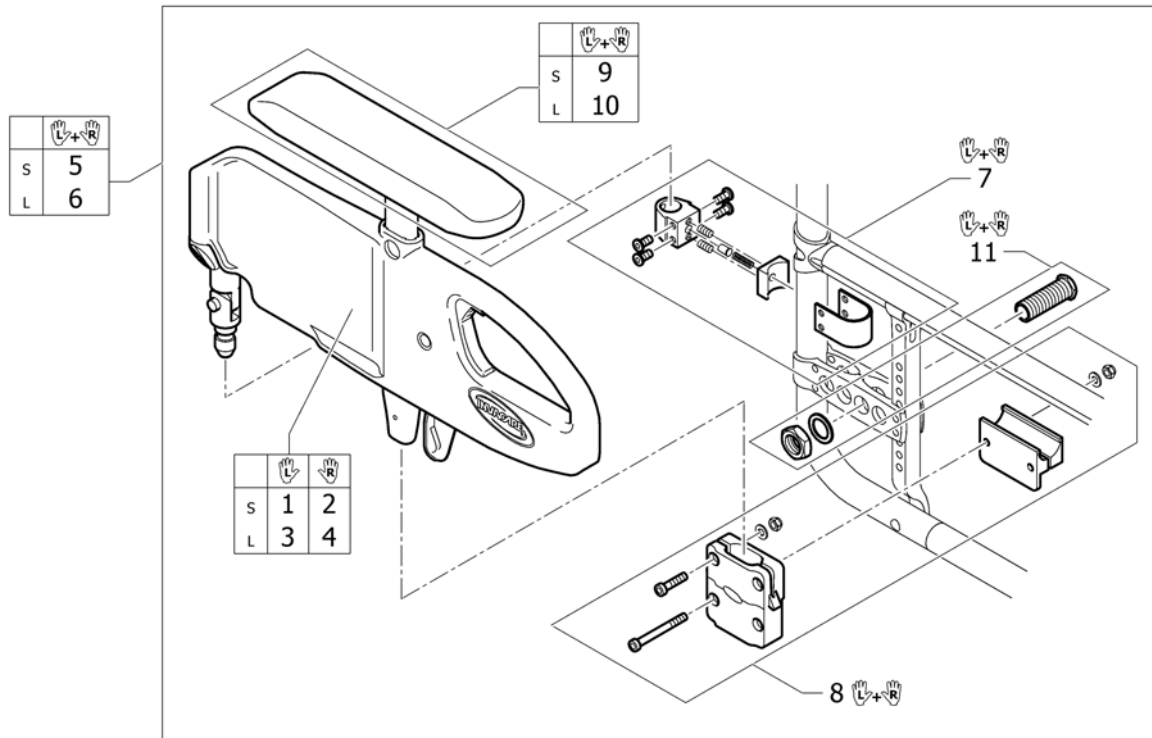




Pos.	Part number	Description	Valid from → until	Qty
1	1506096	Siderest with desk function, flip-back, height adj., removable, short, left		1
2	1506095	Siderest with desk function, flip-back, height adj., removable, short, right		1
3	1506098	Siderest with desk function, flip-back, height adj., removable, long left		1
4	1506097	Siderest with desk function, flip-back, height adj., removable, long, right		1
5	1506099	Siderest with desk function, flip-back, height adj., removable, pair short		1
6	1506100	Siderest with desk function, flip-back, height adj., removable, pair long		1
7	1506101	Fixation kit backrest for siderest with desc function, pair		1
8	1506102	Fixation kit frame for siderest with desc function, pair		1
9	1521532	Ampad short incl. Screws, pair		1
10	1521533	Ampad long incl. Screws, pair		1
11	1527082	Adaptersleeve set 1° for siderest with desc function, pair		1



# Tubular armrest

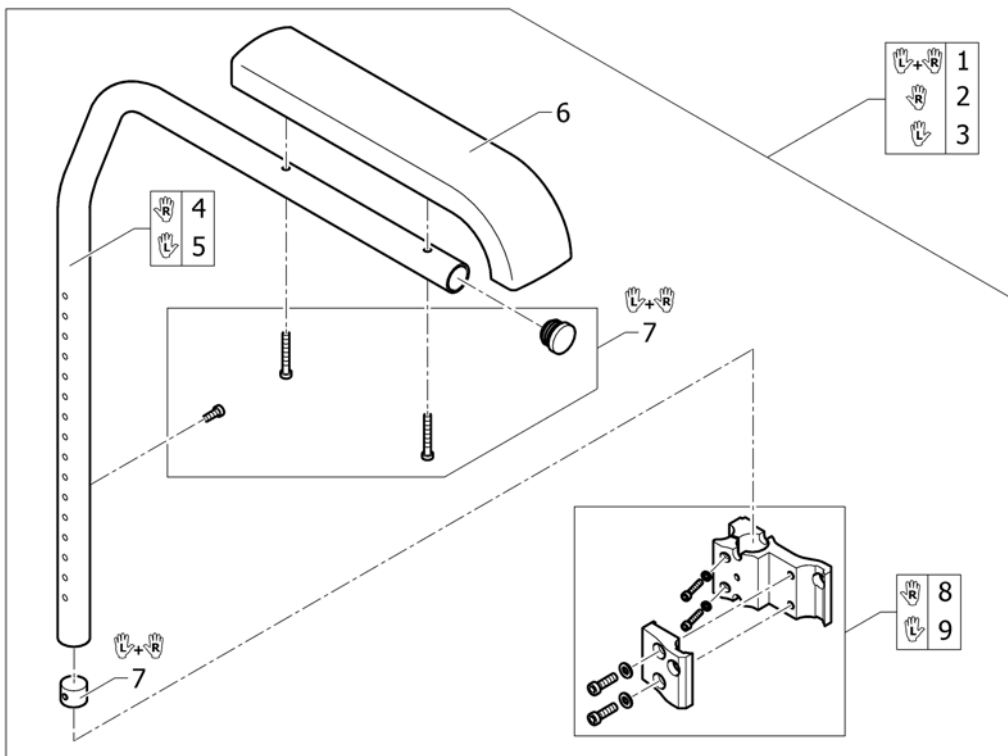


Pos.	Part number	Description	Valid from → until	Qty
1	913.1004	Tubular armrest pair, complete, with standard cushion		1
2	913.1157.000	Tubular armrest right, complete, with standard cushion		1
3	913.1158.000	Tubular armrest left, complete, with standard cushion		1
4	SP1606431	Armrest right		1
5	SP1606432	Armrest left		1
6	SP1589276	Standard pad		2
7	SP1607693	Screws kit for tubular armrest, pair		1
8	913.1103	Clamping bracket complete, right		1
9	913.1009	Clamping bracket complete, left		1

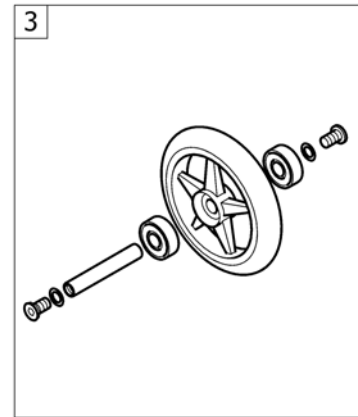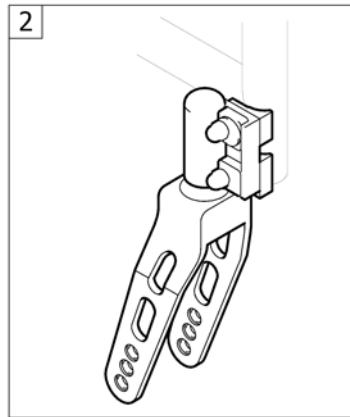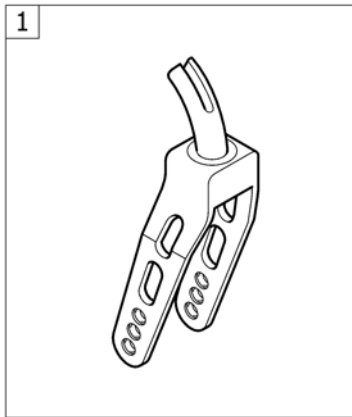**FRONT WHEELS**

Pos.	Part number	Description	Valid from → until	Qty
1	-	Front forks for flow system	2004-01-01 → 2012-12-31	1
2	-	Front forks for bearing blocks	2006-07-01 → 2012-12-31	1
3	-	Front wheels	2004-01-01 → 2012-12-31	1

# Front forks for flow system

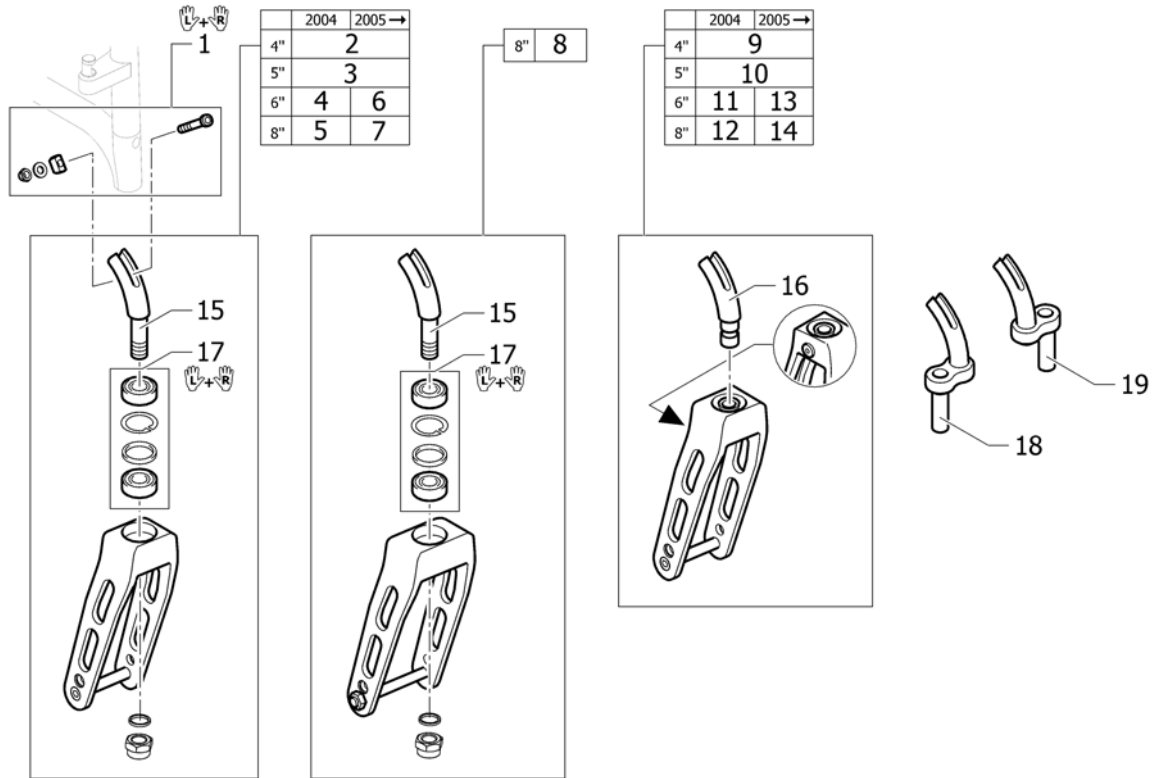


Pos.	Part number	Description	Valid from → until	Qty
1	1438230	fixation kit to front fork Compact		1
2	1437984-0004	Fork 4" Compact/Ultra-Light, black anodized		2
2	1437984-0400	Fork 4" Compact/Ultra-Light, techno-anthrazit		2
2	1437984-0401	Fork 4" Compact/Ultra-Light, lemon		2
2	1437984-0402	Fork 4" Compact/Ultra-Light, apricot		2
2	1437984-0404	Fork 4" Compact/Ultra-Light, b-blue		2
2	1437984-0440	Fork 4" Compact/Ultra-Light, comet red	2007-01-01 →	2
3	1437985-0004	Fork 5" Compact/Ultra-Light, black anodized		2
3	1437985-0400	Fork 5" Compact/Ultra-Light, techno-anthrazit		2
3	1437985-0401	Fork 5" Compact/Ultra-Light, lemon		2
3	1437985-0402	Fork 5" Compact/Ultra-Light, apricot		2
3	1437985-0404	Fork 5" Compact/Ultra-Light, b-blue		2
3	1437985-0440	Fork 5" Compact/Ultra-Light, comet red	2007-01-01 →	2
6	1439573-0004	Front fork 6", black anodized	2005-01-01 →	2
6	1439573-0400	Front fork 6", techno-anthrazit	2005-01-01 →	2
6	1439573-0401	Front fork 6", lemon	2005-01-01 →	2
6	1439573-0402	Front fork 6", apricot	2005-01-01 →	2
6	1439573-0404	Front fork 6", b-blue	2005-01-01 →	2
6	1439573-0440	Front fork 6", comet red	2007-01-01 →	2
7	1439574-0004	Front fork 8", black anodized	2005-01-01 →	2
7	1439574-0400	Front fork 8", techno-anthrazit	2005-01-01 →	2
7	1439574-0401	Front fork 8", lemon	2005-01-01 →	2
7	1439574-0402	Front fork 8", apricot	2005-01-01 →	2
7	1439574-0404	Front fork 8", b-blue	2005-01-01 →	2
7	1439574-0440	Front fork 8", comet red	2007-01-01 →	2
8	1439529-0004	Fork 8" wide Compact/Ultra-Light		2
9	1439482-0004	Fork 4" Compact/Ultra-Light quick release, black anodized		2
9	1439482-0400	Fork 4" Compact/Ultra-Light quick release, techno-anthrazit		2
9	1439482-0401	Fork 4" Compact/Ultra-Light quick release, lemon		2
9	1439482-0402	Fork 4" Compact/Ultra-Light quick release, apricot		2
9	1439482-0404	Fork 4" Compact/Ultra-Light quick release, b-blue		2

**Front forks for flow system**

Pos.	Part number	Description	Valid from → until	Qty
9	1439482-0440	Fork 4" Compact/Ultra-Light quick release, comet red	2007-01-01 →	2
10	1439483-0004	Fork 5" Compact/Ultra-Light quick release, black anodized		2
10	1439483-0400	Fork 5" Compact/Ultra-Light quick release, techno-anthrazit		2
10	1439483-0401	Fork 5" Compact/Ultra-Light quick release, lemon		2
10	1439483-0402	Fork 5" Compact/Ultra-Light quick release, apricot		2
10	1439483-0404	Fork 5" Compact/Ultra-Light quick release, b-blue		2
10	1439483-0440	Fork 5" Compact/Ultra-Light quick release, comet red	2007-01-01 →	2
13	1439575-0004	Front fork 6", quick release, black anodized	2005-01-01 →	2
13	1439575-0400	Front fork 6", quick release, techno-anthrazit	2005-01-01 →	2
13	1439575-0401	Front fork 6", quick release, lemon	2005-01-01 →	2
13	1439575-0402	Front fork 6", quick release, apricot	2005-01-01 →	2
13	1439575-0404	Front fork 6", quick release, b-blue	2005-01-01 →	2
13	1439575-0440	Front fork 6", quick release, comet red	2007-01-01 →	2
14	1439576-0004	Front fork 8", quick release, black anodized	2005-01-01 →	2
14	1439576-0400	Front fork 8", quick release, techno-anthrazit	2005-01-01 →	2
14	1439576-0401	Front fork 8", quick release, lemon	2005-01-01 →	2
14	1439576-0402	Front fork 8", quick release, apricot	2005-01-01 →	2
14	1439576-0404	Front fork 8", quick release, b-blue	2005-01-01 →	2
14	1439576-0440	Front fork 8", quick release, comet red	2007-01-01 →	2
15	1432422	Front fork pin		2
16	1439389-0003	Front fork bolt Quickrelease		2
17	1510219	Bearing kit		1
18	1438754-0003	front wheel extension right		1
19	1438753-0003	front wheel extension left		1

## Front forks for bearing blocks

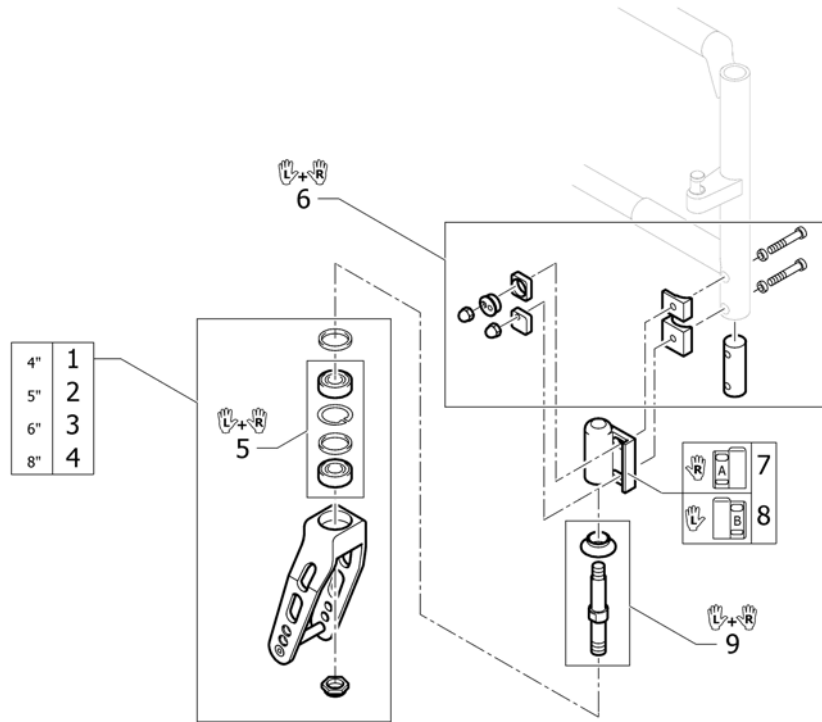


Pos.	Part number	Description	Valid from → until	Qty
1	1495565-0004	Front fork 4" to Frame ext., black anodized		2
1	1495565-0400	Front fork 4" to Frame ext., techno-anthrazit		2
1	1495565-0401	Front fork 4" to Frame ext., lemon		2
1	1495565-0402	Front fork 4" to Frame ext., apricot		2
1	1495565-0404	Front fork 5" to Frame ext., b-blue		2
1	1495565-0440	Front fork 4" to Frame ext., comet red	2007-01-01 →	2
2	1495566-0004	Front fork 5" to Frame ext., black anodized		2
2	1495566-0400	Front fork 5" to Frame ext., techno-anthrazit		2
2	1495566-0401	Front fork 5" to Frame ext., lemon		2
2	1495566-0402	Front fork 5" to Frame ext., apricot		2
2	1495566-0404	Front fork 5" to Frame ext., b-blue		2
2	1495566-0440	Front fork 5" to Frame ext., comet red	2007-01-01 →	2
3	1495567-0004	Front fork 6" to Frame ext., black anodized		2
3	1495567-0400	Front fork 6" to Frame ext., techno-anthrazit		2
3	1495567-0401	Front fork 6" to Frame ext., lemon		2
3	1495567-0402	Front fork 6" to Frame ext., apricot		2
3	1495567-0404	Front fork 6" to Frame ext., b-blue		2
3	1495567-0440	Front fork 6" to Frame ext., comet red	2007-01-01 →	2
4	1495568-0004	Front fork 8" to Frame ext., black anodized		2
4	1495568-0400	Front fork 8" to Frame ext., techno-anthrazit		2
4	1495568-0401	Front fork 8" to Frame ext., lemon		2
4	1495568-0402	Front fork 8" to Frame ext., apricot		2
4	1495568-0404	Front fork 8" to Frame ext., b-blue		2
4	1495568-0440	Front fork 8" to Frame ext., comet red	2007-01-01 →	2
5	1510219	Bearing kit		1
6	1495570	Excenter set for front fork castor		1
7	1495563-0004	Front fork castor right to Frame ext., black anodized		1
7	1495563-0400	Front fork castor right to Frame ext., techno-anthrazit		1
7	1495563-0401	Front fork castor right to Frame ext., lemon		1
7	1495563-0402	Front fork castor right to Frame ext., apricot		1
7	1495563-0404	Front fork castor right to Frame ext., b-blue		1

**Front forks for bearing blocks**

<b>Pos.</b>	<b>Part number</b>	<b>Description</b>	<b>Valid from → until</b>	<b>Qty</b>
7	1495563-0440	Front fork castor right to Frame ext., comet red	2007-01-01 →	1
8	1495564-0004	Front fork castor left to Frame ext., black anodized		1
8	1495564-0400	Front fork castor left to Frame ext., techno-anthrazit		1
8	1495564-0401	Front fork castor left to Frame ext., lemon		1
8	1495564-0402	Front fork castor left to Frame ext., apricot		1
8	1495564-0404	Front fork castor left to Frame ext., b-blue		1
8	1495564-0440	Front fork castor left to Frame ext., comet red	2007-01-01 →	1
9	1495569	Front fork pin kit to frame extension		1



# Front wheels

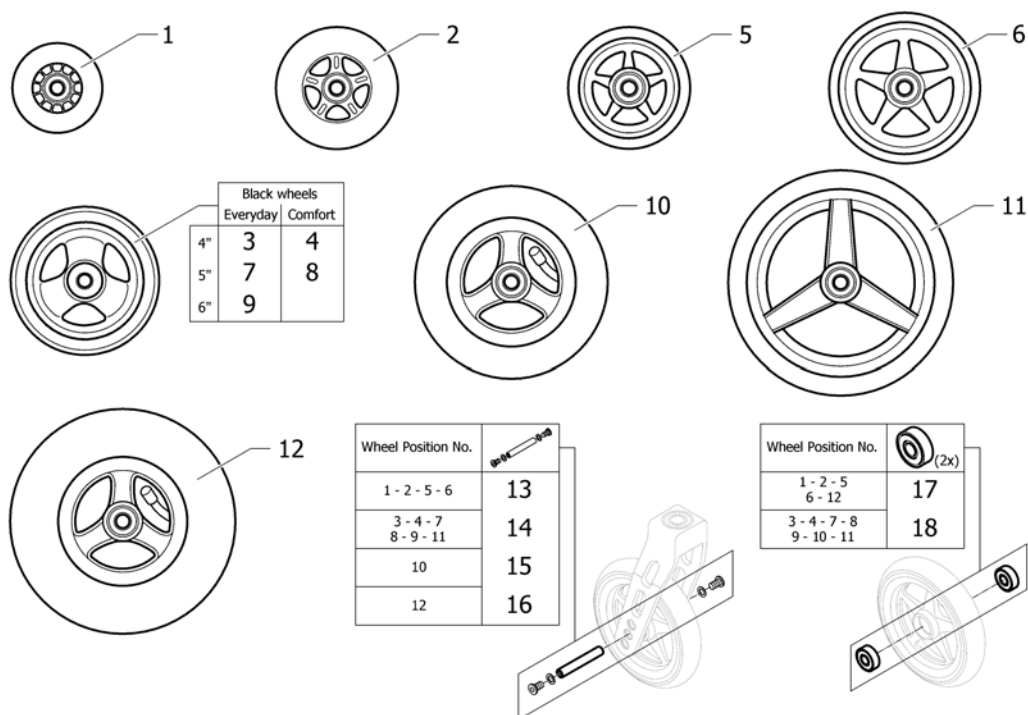


Pos.	Part number	Description	Valid from → until	Qty
1	1513180	3" Sports wheel, transparent <i>Replaces 732.1026</i>		2
2	1513181	4" Skater wheel, transparent <i>Replaces 1416805</i>		2
3	SP1638070	4" Everyday wheel, black PU, pair <i>Replaces 1562626</i>	2017-06-26 →	1
4	SP1638072	4" Comfort wheel, black PU, pair <i>Replaces 1561139</i>	2017-01-01 →	1
5	1513183-0400	4" Starec wheel, techno-anthrazit <i>Replaces 1420968-0400</i>		2
5	1513183-0401	4" Starec wheel, lemon <i>Replaces 1420968-0401</i>		2
5	1513183-0402	4" Starec wheel, apricot <i>Replaces 1420968-0402</i>		2
5	1513183-0404	4" Starec wheel, b-blue <i>Replaces 1420968-0404</i>		2
5	1513183-0440	4" Starec wheel, comet red <i>Replaces 1420968-0440</i>	2007-01-01 →	2
5	1513183-0004	4" Starec wheel, black anodized <i>Replaces 1513183-0408</i>		2
6	1513187-0400	5" Starec wheel, techno-anthrazit <i>Replaces 732.1032.950</i>		2
6	1513187-0401	5" Starec wheel, lemon <i>Replaces 732.1032.951</i>		2
6	1513187-0402	5" Starec wheel, apricot <i>Replaces 732.1032.954</i>		2
6	1513187-0404	5" Starec wheel, b-blue <i>Replaces 732.1032.957</i>		2
6	1513187-0440	5" Starec wheel, comet red <i>Replaces 732.1032-0440</i>	2007-01-01 →	2
6	1513187-0004	5" Starec wheel, black anodized <i>Replaces 732.1032.961</i>		2
7	SP1638071	5" Everyday wheel, black PU, pair <i>Replaces 1562627</i>	2017-05-01 →	1
8	SP1638073	5" Comfort wheel, black PU, pair <i>Replaces 1542640</i>	2017-01-01 →	1

**Front wheels**


Pos.	Part number	Description	Valid from → until	Qty
9	1562628	6" Everyday wheel, 3 spokes, black <i>Replaces 1513190</i>	2012-07-01 →	2
10	1513189	6" Pneumatic wheel <i>Replaces 732.1020</i>		2
11	1513192	7" Light wheel <i>Replaces 732.1014</i>		2
12	1513191	7" Pneumatic wheel <i>Only with Fork 8" wide - Replaces 732.1006</i>		2
13	1420972	Axle set complete 3" Sport / 4" and 5" Starec / 4" Skater wheel		2
14	1570108	Axle set without spacer		2
15	1570109	Axle set complete 6" pneumatic wheel		2
16	1420977	Axle set complete 7" pneumatic wheel <i>Only with Fork 8" wide</i>		2
17	1570104	Ball bearing 608-2RS 8/22*7, pair		2
18	1570105	Ball bearing 6000-2RS, pair		2

## REAR WHEELS

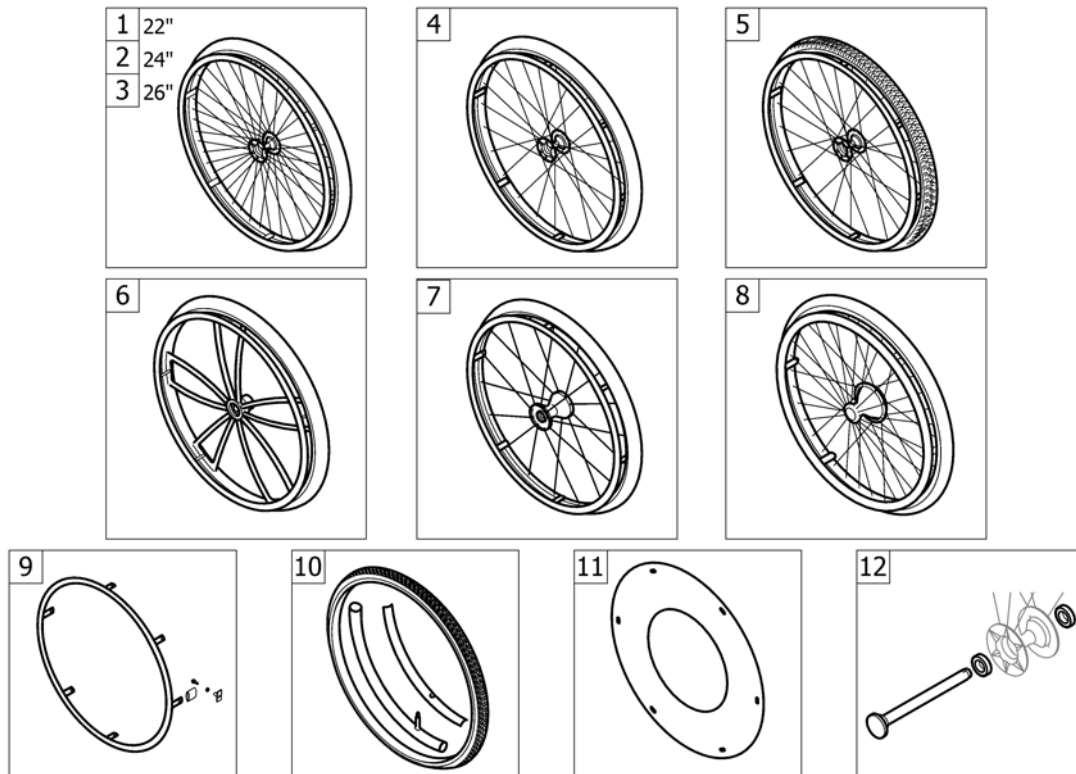

Pos.	Part number	Description	Valid from → until	Qty
1	-	22" Küschall rear wheels complete	2004-01-01 → 2012-12-31	1
2	-	24" Küschall rear wheels complete	2004-01-01 → 2012-12-31	1
3	-	26" Küschall rear wheels complete	2004-01-01 → 2012-12-31	1
4	-	High Performance wheel	2004-01-01 → 2012-12-31	1
5	-	Mountain bike wheel	2004-01-01 → 2012-12-31	1
6	-	Spider wheel	2004-01-01 → 2012-12-31	1
7	-	Spinergy wheel	2004-01-01 → 2012-12-31	1
8	-	Drumbrake wheel	2004-01-01 → 2012-12-31	1
9	-	Handrim	2004-01-01 → 2012-12-31	1
10	-	Tyre	2004-01-01 → 2012-12-31	1
11	-	Spokes guard	2004-01-01 → 2012-12-31	1
12	-	Quick release axle and bearing	2004-01-01 → 2012-12-31	1

## 22" Küschall rear wheels complete

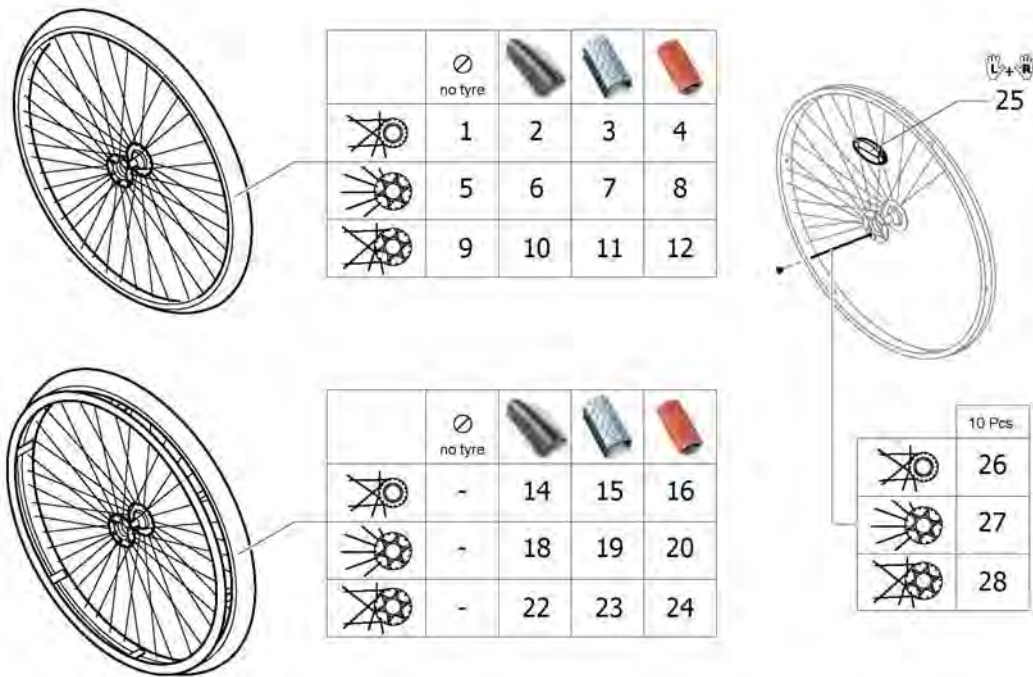


Pos.	Part number	Description	Valid from → until	Qty
5	1559082-0407	Rear wheel 22" Starec R, colourless, without handrim, without tyre <i>Replaces 904.1081</i>		2
5	1559082-0400	Rear wheel 22" Starec R, tecno, without handrim, without tyre <i>Replaces 904.1081.950</i>		2
5	1559082-0401	Rear wheel 22" Starec R, lemon, without handrim, without tyre <i>Replaces 904.1081.951</i>		2
5	1559082-0402	Rear wheel 22" Starec R, apricot, without handrim, without tyre <i>Replaces 904.1081.954</i>		2
5	1559082-0404	Rear wheel 22" Starec R, b-blue, without handrim, without tyre <i>Replaces 904.1081.957</i>		2
5	1559082-0440	Rear wheel 22" Starec R, comet red, without handrim, without tyre <i>Replaces 904.1081-0440</i>	2007-01-01 →	2
6	SP1601740	Rear wheel 22", Starec hub R, without handrim, Rightrun black/grey <i>Replaces 904.1195</i>		2
7	904.1158	Rear wheel 22", Starec hub R, without handrim, Tread		2
9	1559083-0407	Rear wheel 22", Starec X, colourless, without handrim, without tyre <i>Replaces 904.1131</i>		2
9	1559083-0400	Rear wheel 22" Starec X, Tecno, without handrim, without tyre <i>Replaces 904.1131.950</i>		2
9	1559083-0401	Rear wheel 22" Starec X, Lemon, without handrim, without tyre <i>Replaces 904.1131.951</i>		2
9	1559083-0402	Rear wheel 22" Starec X, Apricot, without handrim, without tyre <i>Replaces 904.1131.954</i>		2
9	1559083-0404	Rear wheel 22" Starec X, B-blue, without handrim, without tyre <i>Replaces 904.1131.957</i>		2
9	1559083-0440	Rear wheel 22" Starec X, Comet red, without handrim, without tyre <i>Replaces 904.1131-0440</i>	2007-01-01 →	2
10	SP1601741	Rear wheel 22", Starec hub X, without handrim, Rightrun black/grey <i>Replaces 904.1189</i>		2
11	904.1139	Rear wheel 22", Starec hub X, without handrim, Tread		2
18	SP1601742	Rear wheel 22", Starec hub R, handrim, Rightrun black/grey <i>Replaces 904.1192</i>		2
19	904.1149	Rear wheel 22", Starec hub R, handrim, Tread		2
22	SP1601743	Rear wheel 22", Starec hub X, handrim, Rightrun black/grey <i>Replaces 904.1186</i>		2
23	904.1123	Rear wheel 22", Starec hub X, handrim, Tread		2

**22" Küschall rear wheels complete**

Pos.	Part number	Description	Valid from → until	Qty
25	1571073	Spokereflection, pair <i>Replaces 220.1033</i>		1
26	1579083	Spokes 230mm complete, kit 10 Pcs.		1
27	1579081	Spokes 211mm complete, kit 10 Pcs.		1
28	1579082	Spokes 228mm complete, kit 10 Pcs.		1

## 24" Küschall rear wheels complete

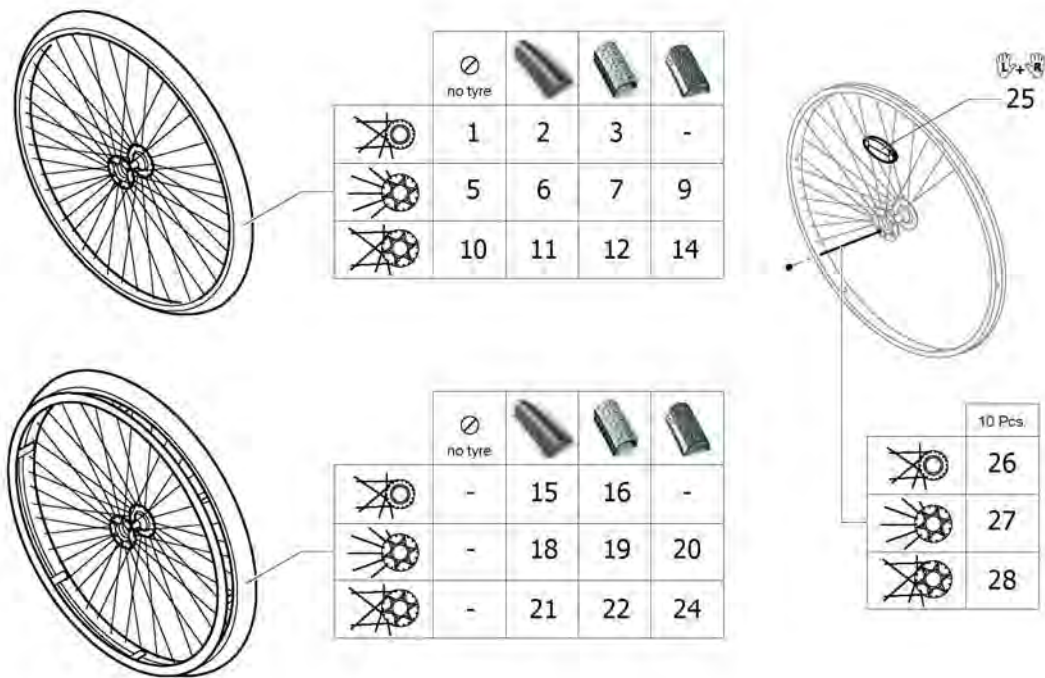


Pos.	Part number	Description	Valid from → until	Qty
5	1567103-0407	Rear wheel 24" Starec R, colourless, without handrim, without tyre <i>Replaces 904.1082</i>		2
5	1567103-0400	Rear wheel 24" Starec R, tecno, without handrim, without tyre <i>Replaces 904.1082.950</i>		2
5	1567103-0401	Rear wheel 24" Starec R, lemon, without handrim, without tyre <i>Replaces 904.1082.951</i>		2
5	1567103-0402	Rear wheel 24" Starec R, apricot, without handrim, without tyre <i>Replaces 904.1082.954</i>		2
5	1567103-0404	Rear wheel 24" Starec R, b-blue, without handrim, without tyre <i>Replaces 904.1082.957</i>		2
5	1567103-0440	Rear wheel 24" Starec R, comet red, without handrim, without tyre <i>Replaces 904.1082-0440</i>	2007-01-01 →	2
6	SP1601744	Rear wheel 24", Starec hub R, without handrim, Rightrun black/grey <i>Replaces 904.1161</i>		2
7	904.1160	Rear wheel 24", Starec hub R, without handrim, Tread		2
9	1554397-0007	Rear wheel 24", Starec R, colourless, without handrim, marathon evolution		2
9	1554397-0400	Rear wheel 24" Starec R, tecno, without handrim, marathon evolution		2
9	1554397-0401	Rear wheel 24" Starec R, lemon, without handrim, marathon evolution		2
9	1554397-0402	Rear wheel 24" Starec R, apricot, without handrim, marathon evolution		2
9	1554397-0404	Rear wheel 24" Starec R, b-blue, without handrim, marathon evolution		2
9	1554397-0440	Rear wheel 24" Starec R, comet red, without handrim, marathon evolution	2007-01-01 →	2
10	1567104-0407	Rear wheel 24", Starec X, colourless, without handrim, without tyre <i>Replaces 1416885</i>		2
10	1567104-0400	Rear wheel 24" Starec X, Tecno, without handrim, without tyre <i>Replaces 904.1132.950</i>		2
10	1567104-0401	Rear wheel 24" Starec X, Lemon, without handrim, without tyre <i>Replaces 904.1132.951</i>		2
10	1567104-0402	Rear wheel 24" Starec X, Apricot, without handrim, without tyre <i>Replaces 904.1132.954</i>		2
10	1567104-0404	Rear wheel 24" Starec X, B-blue, without handrim, without tyre <i>Replaces 904.1132.957</i>		2
10	1567104-0440	Rear wheel 24" Starec X, Comet red, without handrim, without tyre <i>Replaces 904.1132-0440</i>	2007-01-01 →	2
11	SP1601745	Rear wheel 24", Starec hub X, without handrim, Rightrun black/grey tyre <i>Replaces 1570471</i>		2



**24" Küschall rear wheels complete**

Pos.	Part number	Description	Valid from → until	Qty
12	904.1141	Rear wheel 24", Starec hub X, without handrim, Tread		2
14	1554399-0007	Rear wheel 24", Starec X, colourless, without handrim, marathon		2
14	1554399-0400	Rear wheel 24" Starec X, tecno, without handrim, marathon evolution		2
14	1554399-0401	Rear wheel 24" Starec X, lemon, without handrim, marathon evolution		2
14	1554399-0402	Rear wheel 24" Starec X, apricot, without handrim, marathon evolution		2
14	1554399-0404	Rear wheel 24" Starec X, b-blue, without handrim, marathon evolution		2
14	1554399-0440	Rear wheel 24" Starec X, comet red, without handrim, marathon evolution	2007-01-01 →	2
18	SP1601746	Rear wheel 24", Starec hub R, handrim, Rightrun black/grey <i>Replaces 904.1152</i>		2
19	904.1151	Rear wheel 24", Starec hub R, handrim, Tread		2
20	1554398-0007	Rear wheel 24", Starec R, colourless, handrim, marathon evolution		2
20	1554398-0400	Rear wheel 24" Starec R, tecno, handrim, marathon evolution		2
20	1554398-0401	Rear wheel 24" Starec R, lemon, handrim, marathon evolution		2
20	1554398-0402	Rear wheel 24" Starec R, apricot, handrim, marathon evolution		2
20	1554398-0404	Rear wheel 24" Starec R, b-blue, handrim, marathon evolution		2
20	1554398-0440	Rear wheel 24" Starec R, comet red, handrim, marathon evolution	2007-01-01 →	2
21	SP1601747	Rear wheel 24", Starec hub X, handrim, Rightrun black/grey tyre <i>Replaces 1570472</i>		2
22	904.1125	Rear wheel 24", Starec hub X, handrim, Tread		2
24	1554400-0007	Rear wheel 24", Starec X, colourless, handrim, marathon evolution		2
24	1554400-0400	Rear wheel 24" Starec X, tecno, handrim, marathon evolution		2
24	1554400-0401	Rear wheel 24" Starec X, lemon, handrim, marathon evolution		2
24	1554400-0402	Rear wheel 24" Starec X, apricot, handrim, marathon evolution		2
24	1554400-0404	Rear wheel 24" Starec X, b-blue, handrim, marathon evolution		2
24	1554400-0440	Rear wheel 24" Starec X, comet red, handrim, marathon evolution	2007-01-01 →	2
25	1571073	Spokereflection, pair <i>Replaces 220.1033</i>		1
26	1579087	Spokes 256mm complete, kit 10 Pcs.		1
27	1579084	Spokes 238mm complete, kit 10 Pcs.		1
28	1579086	Spokes 253mm complete, kit 10 Pcs.		1

## 26" Küschall rear wheels complete

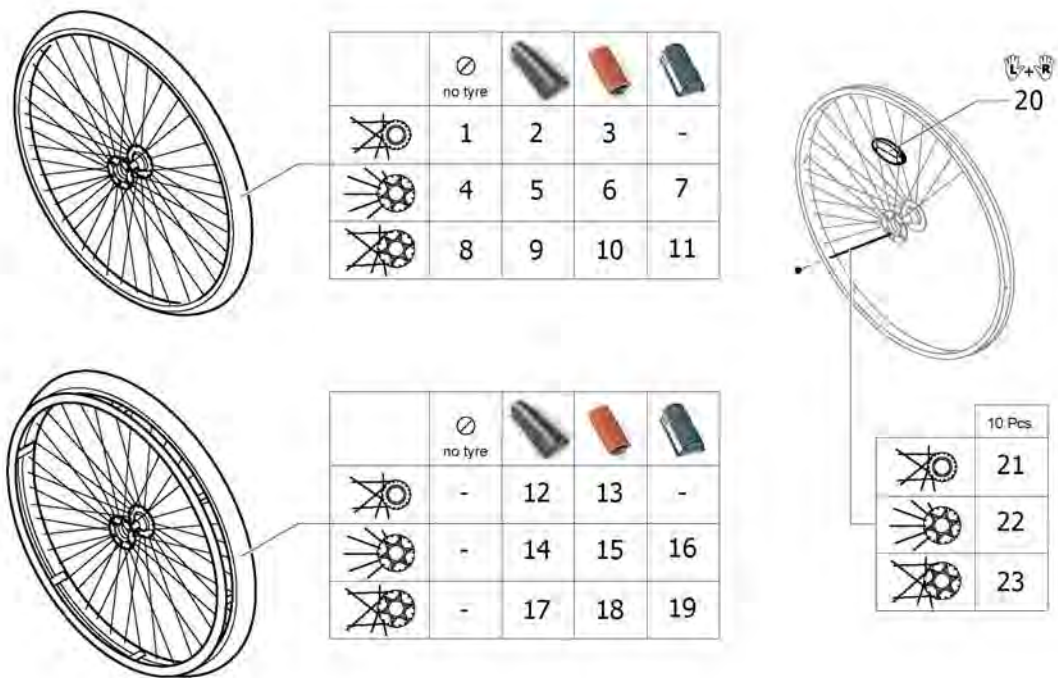


Pos.	Part number	Description	Valid from → until	Qty
4	1567098-0407	Rear wheel 26" Starec R, colourless, without handrim, without tyre <i>Replaces 904.1083</i>		2
4	1567098-0400	Rear wheel 26" Starec R, tecno, without handrim, without tyre <i>Replaces 904.1083.950</i>		2
4	1567098-0401	Rear wheel 26" Starec R, lemon, without handrim, without tyre <i>Replaces 904.1083.951</i>		2
4	1567098-0402	Rear wheel 26" Starec R, apricot, without handrim, without tyre <i>Replaces 904.1083.954</i>		2
4	1567098-0404	Rear wheel 26" Starec R, b-blue, without handrim, without tyre <i>Replaces 904.1083.957</i>		2
4	1567098-0440	Rear wheel 26" Starec R, comet red, without handrim, without tyre <i>Replaces 904.1083-0440</i>	2007-01-01 →	2
5	SP1601748	Rear wheel 26", Starec hub R, without handrim, Rightrun black/grey <i>Replaces 904.1196</i>		2
6	904.1165	Rear wheel 26", Starec hub R, without handrim, indoor sports		2
7	1554401-0007	Rear wheel 26", Starec R, colourless, without handrim, marathon evolution		2
7	1554401-0400	Rear wheel 26" Starec R, tecno, without handrim, marathon evolution		2
7	1554401-0401	Rear wheel 26" Starec R, lemon, without handrim, marathon evolution		2
7	1554401-0402	Rear wheel 26" Starec R, apricot, without handrim, marathon evolution		2
7	1554401-0404	Rear wheel 26" Starec R, b-blue, without handrim, marathon evolution		2
7	1554401-0440	Rear wheel 26" Starec R, comet red, without handrim, marathon evolution	2007-01-01 →	2
8	1567099-0407	Rear wheel 26", Starec X, colourless, without handrim, without tyre <i>Replaces 904.1133</i>		2
8	1567099-0400	Rear wheel 26" Starec X, Tecno, without handrim, without tyre <i>Replaces 904.1133.950</i>		2
8	1567099-0401	Rear wheel 26" Starec X, Lemon, without handrim, without tyre <i>Replaces 904.1133.951</i>		2
8	1567099-0402	Rear wheel 26" Starec X, Apricot, without handrim, without tyre <i>Replaces 904.1133.954</i>		2
8	1567099-0404	Rear wheel 26" Starec X, B-blue, without handrim, without tyre <i>Replaces 904.1133.957</i>		2
8	1567099-0440	Rear wheel 26" Starec X, Comet red, without handrim, without tyre <i>Replaces 904.1133-0440</i>	2007-01-01 →	2
9	SP1601749	Rear wheel 26", Starec hub X, without handrim, Rightrun black/grey <i>Replaces 904.1190</i>		2

**26" Küschall rear wheels complete**

Pos.	Part number	Description	Valid from → until	Qty
10	904.1146	Rear wheel 26", Starec hub X, without handrim, Indoor sports		2
11	1554403-0007	Rear wheel 26", Starec X, colourless, without handrim, marathon		2
11	1554403-0400	Rear wheel 26" Starec X, tecno, without handrim, marathon evolution		2
11	1554403-0401	Rear wheel 26" Starec X, lemon, without handrim, marathon evolution		2
11	1554403-0402	Rear wheel 26" Starec X, apricot, without handrim, marathon evolution		2
11	1554403-0404	Rear wheel 26" Starec X, b-blue, without handrim, marathon evolution		2
11	1554403-0440	Rear wheel 26" Starec X, comet red, without handrim, marathon evolution	2007-01-01 →	2
14	SP1601750	Rear wheel 26", Starec hub R, handrim, Rightrun black/grey <i>Replaces 904.1193</i>		2
15	904.1155	Rear wheel 26", Starec hub R, handrim, Indoor sports		2
16	1554402-0007	Rear wheel 26", Starec R, colourless, handrim, marathon evolution		2
16	1554402-0400	Rear wheel 26" Starec R, tecno, handrim, marathon evolution		2
16	1554402-0401	Rear wheel 26" Starec R, lemon, handrim, marathon evolution		2
16	1554402-0402	Rear wheel 26" Starec R, apricot, handrim, marathon evolution		2
16	1554402-0404	Rear wheel 26" Starec R, b-blue, handrim, marathon evolution		2
16	1554402-0440	Rear wheel 26" Starec R, comet red, handrim, marathon evolution	2007-01-01 →	2
17	SP1601751	Rear wheel 26", Starec hub X, handrim, Rightrun black/grey <i>Replaces 904.1187</i>		2
18	904.1129	Rear wheel 26", Starec hub X, handrim, Indoor sports		2
19	1554417-0007	Rear wheel 26", Starec X, colourless, handrim, marathon evolution		2
19	1554417-0400	Rear wheel 26" Starec X, tecno, handrim, marathon evolution		2
19	1554417-0401	Rear wheel 26" Starec X, lemon, handrim, marathon evolution		2
19	1554417-0402	Rear wheel 26" Starec X, apricot, handrim, marathon evolution		2
19	1554417-0404	Rear wheel 26" Starec X, b-blue, handrim, marathon evolution		2
19	1554417-0440	Rear wheel 26" Starec X, comet red, handrim, marathon evolution	2007-01-01 →	2
20	1571073	Spokereflection, pair <i>Replaces 220.1033</i>		1
21	1579091	Spokes 280mm complete, kit 10 Pcs.		1
22	1579089	Spokes 262mm complete, kit 10 Pcs.		1
23	1579090	Spokes 278mm complete, kit 10 Pcs.		1

## High Performance wheel

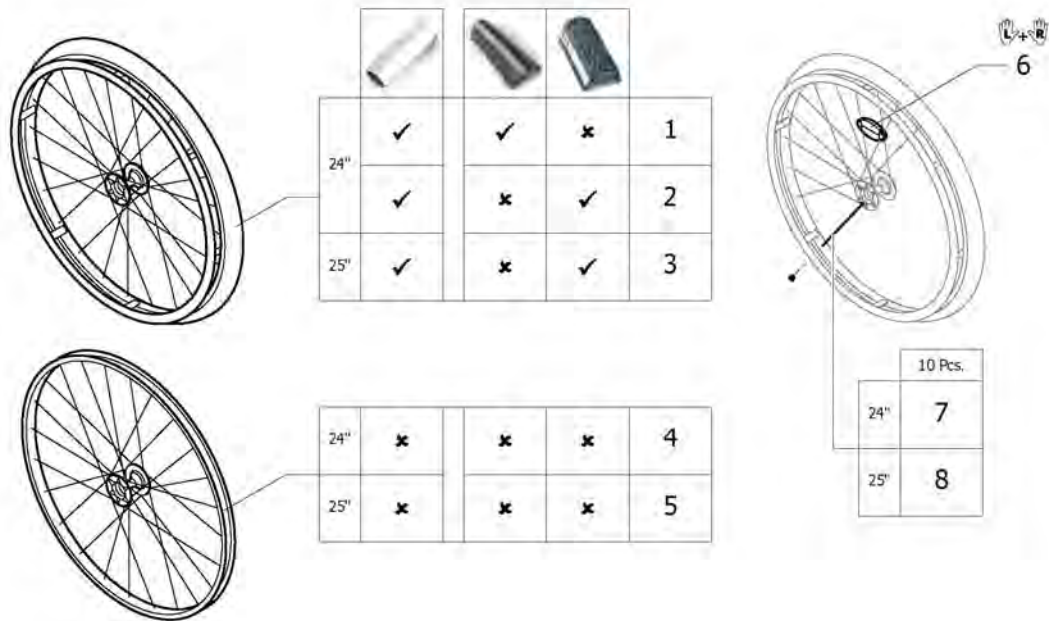


Pos.	Part number	Description	Valid from → until	Qty
1	SP1601754-0004	Rear wheel 24" High Performance black anodized, handrim, Rightrun black/ grey <i>Replaces 1420958-0004</i>	2010-01-01 →	2
1	SP1601754-0400	Rear wheel 24" High Performance techno-anthracite, handrim, Rightrun black/ grey <i>Replaces 1420958-0400</i>	2009-01-01 →	2
1	SP1601754-0401	Rear wheel 24" High Performance lemon, handrim, Rightrun black/grey <i>Replaces 1420958-0401</i>	2009-01-01 →	2
1	SP1601754-0402	Rear wheel 24" High Performance apricot, handrim, Rightrun black/grey <i>Replaces 1420958-0402</i>	2009-01-01 →	2
1	SP1601754-0404	Rear wheel 24" High Performance b-blue, handrim, Rightrun black/grey <i>Replaces 1420958-0404</i>	2009-01-01 →	2
1	SP1601754-0440	Rear wheel 24" High Performance comet red, handrim, Rightrun black/grey <i>Replaces 1420958-0440</i>	2009-01-01 →	2
2	1527046-0004	Wheel 24", High Performance, black, handrim alu, tyre Marathon Evolution	2010-01-01 →	2
2	1527046-0400	Wheel 24", High Performance, techno-anthracite, handrim alu, tyre Marathon Evolution	2009-01-01 →	2
2	1527046-0401	Wheel 24", High Performance, lemon, handrim alu, tyre Marathon Evolution	2009-01-01 →	2
2	1527046-0402	Wheel 24", High Performance, apricot, handrim alu, tyre Marathon Evolution	2009-01-01 →	2
2	1527046-0404	Wheel 24", High Performance, b-blue, handrim alu, tyre Marathon Evolution	2009-01-01 →	2
2	1527046-0440	Wheel 24", High Performance, comet red, handrim alu, tyre Marathon Evolution	2009-01-01 →	2
3	1445479-0004	Wheel 25", High Performance, black, handrim alu, tyre Marathon Evolution	2010-01-01 →	2
3	1445479-0400	Wheel 25", High Performance, techno-anthracite, handrim alu, tyre Marathon Evolution	2009-01-01 →	2
3	1445479-0401	Wheel 25", High Performance, lemon, handrim alu, tyre Marathon Evolution	2009-01-01 →	2
3	1445479-0402	Wheel 25", High Performance, apricot, handrim alu, tyre Marathon Evolution	2009-01-01 →	2

**High Performance wheel**

Pos.	Part number	Description	Valid from → until	Qty
3	1445479-0404	Wheel 25", High Performance, b-blue, handrim alu, tyre Marathon Evolution	2009-01-01 →	2
3	1445479-0440	Wheel 25", High Performance, comet red, handrim alu, tyre Marathon Evolution	2009-01-01 →	2
4	1517723-0004	Wheel 24", High Performance, black, without handrim, without tyre	2010-01-01 →	2
4	1517723-0400	Wheel 24", High Performance, techno-anthracite, without handrim, without tyre	2009-01-01 →	2
4	1517723-0401	Wheel 24", High Performance, lemon, without handrim, without tyre	2009-01-01 →	2
4	1517723-0402	Wheel 24", High Performance, apricot, without handrim, without tyre	2009-01-01 →	2
4	1517723-0404	Wheel 24", High Performance, b-blue, without handrim, without tyre	2009-01-01 →	2
4	1517723-0440	Wheel 24", High Performance, comet red, without handrim, without tyre	2009-01-01 →	2
5	1562937-0004	Wheel 25", High Performance, black, without handrim, without tyre	2010-01-01 →	2
5	1562937-0400	Wheel 25", High Performance, techno-anthrac., without handrim, without tyre	2009-01-01 →	2
5	1562937-0401	Wheel 25", High Performance, lemon, without handrim, without tyre	2009-01-01 →	2
5	1562937-0402	Wheel 25", High Performance, apricot, without handrim, without tyre	2009-01-01 →	2
5	1562937-0404	Wheel 25", High Performance, b-blue, without handrim, without tyre	2009-01-01 →	2
5	1562937-0440	Wheel 25", High Performance, comet red, without handrim, without tyre	2009-01-01 →	2
6	1571073	Spokereflection, pair <i>Replaces 220.1033</i>		1
7	1578867	Spokes kit for High Performance wheel 24", 10 Pcs.		1
8	1578868	Spokes kit for High Performance wheel 25", 10 Pcs.		1


## Mountain bike wheel

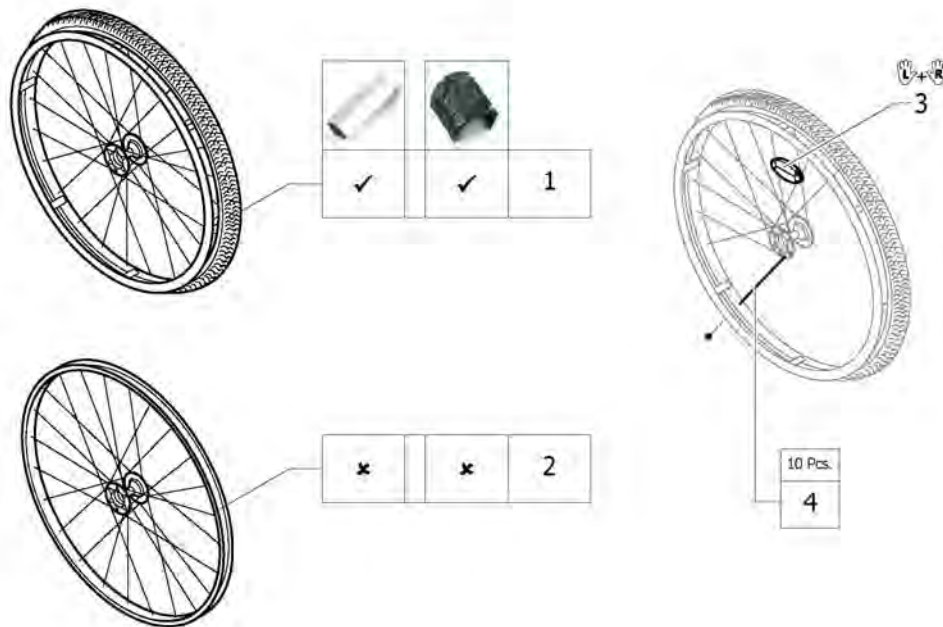

Pos.	Part number	Description	Valid from → until	Qty
1	1517875	Rear wheel 24", Mountain Bike, handrim alu, tyre MTB <i>Replaces 1420959</i>		2
2	1529327	Rear wheel Mountain Bike 24", without handrim, without tyre <i>Replaces 1439420</i>		2
3	1571073	Spokereflection, pair <i>Replaces 220.1033</i>		1
4	1579085	Spokes kit for Mountain Bike wheel, 10 Pcs.		1




## Spider wheel

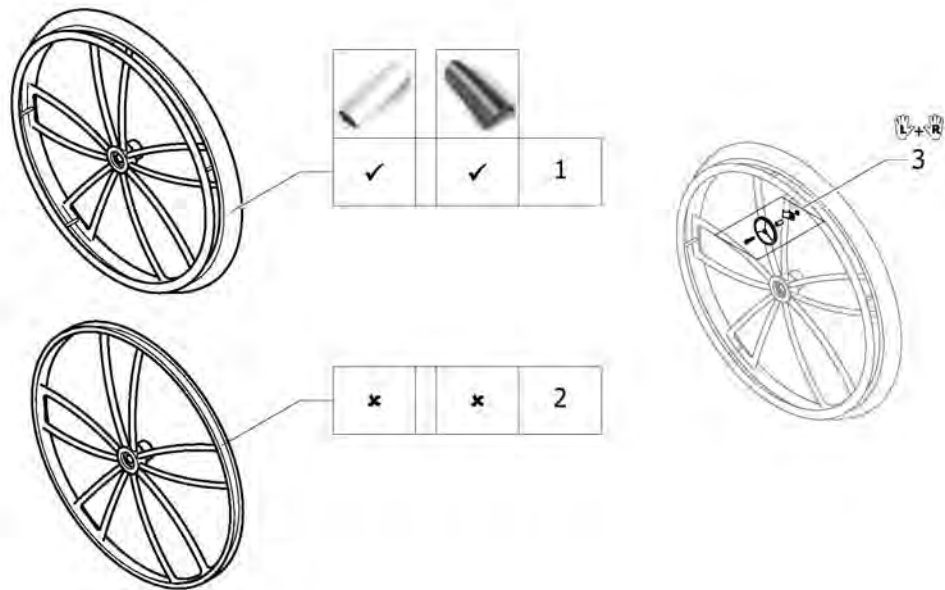


Pos.	Part number	Description	Valid from → until	Qty
1	SP1601849	Rear wheel 24" Spiderwheel, handrim, Rightrun black/grey <i>Replaces 904.1201.000</i>		2
2	1424944	Rear wheel 24", Spiderwheel without handrim, without tyre		2
3	1571074	Spoke reflection set for Spider wheel, pair <i>Replaces 1439352</i>		1

# Spinergy wheel

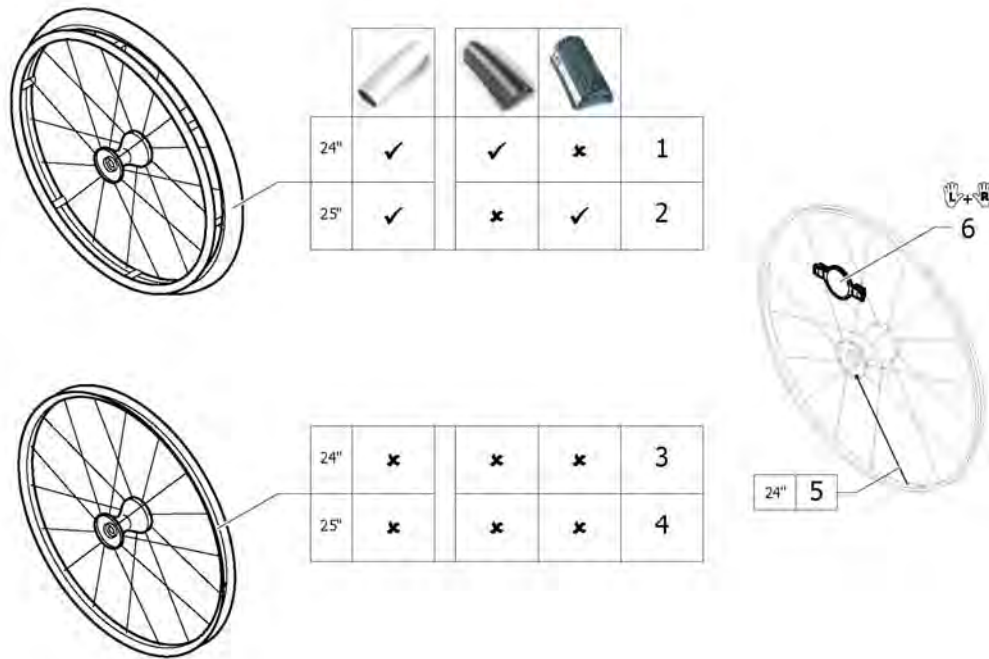

Pos.	Part number	Description	Valid from → until	Qty
1	SP1601755	Rear wheel 24" Spinergy, handrim, Rightrun black/grey <i>Replaces 1485805</i>		2
2	1525127	Rear wheel 25", Spinergy, handrim Alu, tyre Mararthon Evolution	2009-01-01 →	2
3	1570426	Rear wheel 24", Spinergy, without handrim, without tyre		2
4	1525128	Rear wheel 25", Spinergy, without handrim, without tyre	2009-01-01 →	2
5	1579092	Spoke for Spinergy wheel 24", 1 Pcs.		36
6	1571088	Spoke reflection for Spinergy wheel, pair <i>Replaces 1451931</i>		1

## Drumbrake wheel

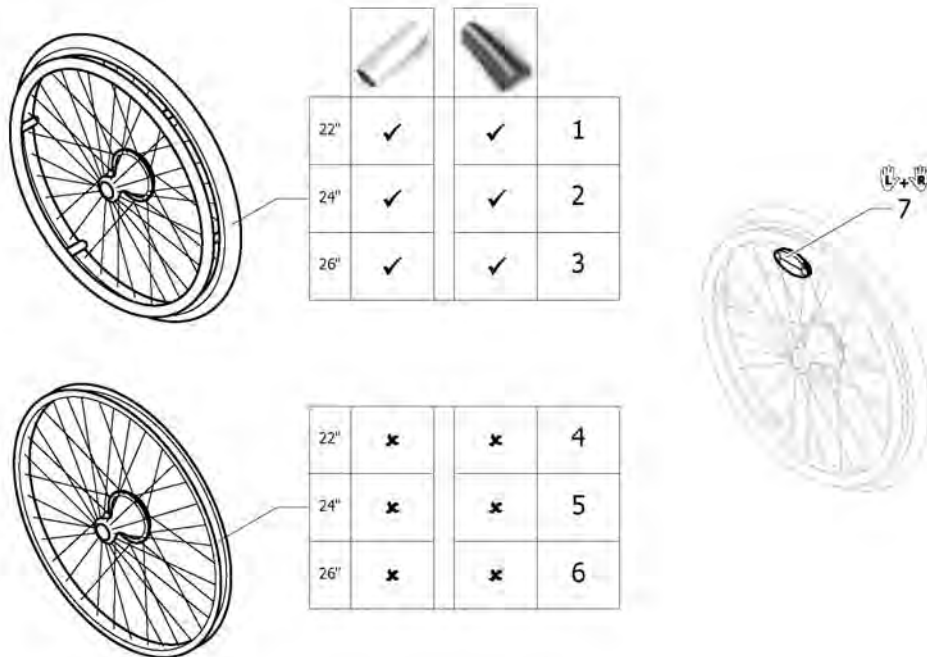

Pos.	Part number	Description	Valid from → until	Qty
1	SP1601850	Rear wheel 22" Drumbrake Octopus, handrim, Rightrun black/grey <i>Replaces 1445495</i>		2
4	1445492	Rear wheel 22" Drumbrake Octopus, without handrim, without tyre		2
2	SP1601851	Rear wheel 24" Drumbrake Octopus, handrim, Rightrun black/grey <i>Replaces 1445496</i>		2
5	1445493	Rear wheel 24" Drumbrake Octopus, without handrim, without tyre		2
3	SP1601852	Rear wheel 26" Drumbrake Octopus, handrim, Rightrun black/grey <i>Replaces 1445497</i>		2
6	1445494	Rear wheel 26" Drumbrake Octopus, without handrim, without tyre		2
7	1571073	Spokereflection, pair <i>Replaces 220.1033</i>		1


# Handrim

		3cm	4cm	22"	24"	25"	26"	24"	24"	26"	22"	24"	26"	24"	26"
	3cm	1	3	5	7	9	11	13	15	16	17	18	19	20	
	4cm	2	4	6	8	10	12	14							
	3cm	-	3	5	-	9	11	-	-	16	-	18	-	20	
	4cm	-	4	6	-	10	12	-	-						
	3cm	1	-	-	-	-	-	-	15	-	-	-	-	-	
	4cm	2	-	-	-	-	-	-							
	3cm	-	21	-	-	22	-	-	-	23	-	-	-	-	
	4cm	-													
	3cm	-	3	5	-	9	11	-	-	16	-	18	-	20	
	4cm	-	4	6	-	10	12	-	-						
	3cm	1	3	-	7	9	11	13	15	16	17	18	19	20	
	4cm	2	4	-	8	10	12	14							



Pos.	Part number	Description	Valid from → until	Qty
Fix	-	Fixation handrim		1
1	1568713-0005	Handrim 22" Aluminium, distance 3cm		2
2	1568700	Handrim 22" Alu, 4cm <i>Replaces 457.1023</i>		2
3	1568714-0005	Handrim 24" Aluminium, distance 3cm		2
4	1568702	Handrim 24" Alu light, colourless anodized, 4cm <i>Replaces 1432287-0005</i>		2
5	1568730-0005	Handrim 25" Aluminium, distance 3cm		2
6	1568708	Handrim 25" Alu light for high performance, 4cm <i>Replaces 1420890-0005</i>		2
7	1568731-0005	Handrim 26" Aluminium, distance 3cm		2
8	1568709	Handrim 26" Alu, 4cm <i>Replaces 457.1043</i>		2
9	1568716	Handrim 24" Titanium, distance 3cm		2
10	1568705	Handrim 24" Titanium, 4cm <i>Replaces 457.1051</i>		2
11	1568715	Handrim 24" steel polished, distance 3cm		2
12	1568704	Handrim 24" steel polished, 4cm <i>Replaces 457.1009</i>		2
13	1568777	Handrim 26" steel polished, distance 3cm		2
14	1568711	Handrim 26" steel polished, 4cm <i>Replaces 457.1010</i>		2
15	1568701	Handrim 22" Super-Grip <i>Replaces 457.1045.429</i>		2
16	1568703	Handrim 24" Super-Grip <i>Replaces 457.1017.429</i>		2
17	1568710	Handrim 26" Super-Grip <i>Replaces 457.1046.429</i>		2
19	1568712	Handrim 26" Tetra <i>Replaces 1436971</i>		2
21	457.1053.000	Handrim 24" Alu for Spider wheel		2
22	1416801	Handrim 24" Titan for Spider Wheel		2
23	1416802	Handrim 24" Super-Grip for Spider wheel		2

# Fixation handrim

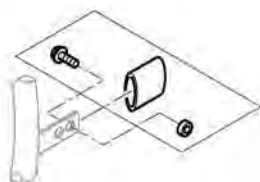


		Aluminium	Titanium	Steel	Supergrip	Tetra	Maxgrip						
	3cm												
	4cm												
	3cm	1	1	1	1	1	1	12	12	1	1		
	4cm	2	3	2	3	2	3	2	3	-	-	2	3
	3cm	4	4	4	4	4	4	4	12	12	4	4	
	4cm	5	6	5	6	5	6	5	6	-	-	5	6
	3cm	7	7	-	-	-	-	7	7	-	-	-	-
	4cm	8	9	-	-	-	-	8	9	-	-	-	-
	3cm	10	-	10	-	-	-	10	-	-	-	-	-
	4cm	11	-	11	-	-	-	11	-	-	-	-	-
	3cm	7	-	7	-	7	-	7	-	12	-	7	-
	4cm	8	-	8	-	8	-	8	-	-	-	8	-
	3cm	1	-	1	-	1	-	1	-	12	-	1	-
	4cm	2	-	2	-	2	-	2	-	-	-	2	-

Pos.	Part number	Description	Valid from → until	Qty
1	1495958	Fixation kit handrim, 3 cm		2
2	1495959	Fixation kit handrim, 4 cm		2
3	1495961	Fixation kit handrim, 4 cm with spokes guard		2
4	1496019	Fixation kit handrim for High Performance wheel, 3 cm		2
5	1496020	Fixation kit handrim for High Performance wheel, 4 cm		2
6	1497111	Fixation kit handrim for High Performance wheel, 4 cm with spokes guard		2
7	1496015	Fixation kit handrim for Mountain bike wheel, 3 cm		2
8	1496016	Fixation kit handrim for Mountain bike wheel, 4 cm		2
9	1496018	Fixation kit handrim for Mountain bike wheel, 4 cm with spokes guard		2
10	1418931	Fixation kit handrim for Spider wheel, 3 cm		2
11	1418932	Fixation kit handrim for Spider wheel, 4 cm		2
12	1570759	Fixation kit handrim Tetra, 3 cm		2

# Tyre

																
	22"	24"	25"	26"	22"	24"	24"	25"	26"	24"	22"	24"	24"	24"	22"	26"
 Küschall Räder	1	5	-	15	2	6	7	-	16	-	3	9	10	11	4	17
 High Performance	-	5	13	-	-	6	7	14	-	-	-	-	-	-	-	-
 Mountainbike	-	-	-	-	-	-	-	-	-	8	-	-	-	-	-	-
 Spiderwheel	-	5	-	-	-	6	7	-	-	-	-	9	10	11	-	-
 Spinergy	-	5	13	-	-	6	7	14	-	-	-	-	-	-	-	-
 Drumbrake	1	5	13	15	2	6	7	14	16	-	3	9	10	11	4	17

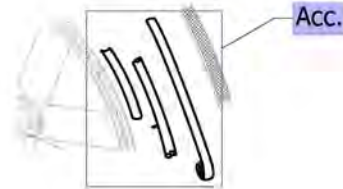

Pos.	Part number	Description	Valid from → until	Qty
Acc.	-	Inner tube and rim protector		1
1	SP1590117	Tyre Rightrun black/grey 22" <i>Replaces 1570776</i>		2
2	1570836	Treaded tyre 22" <i>Replaces 1493768</i>		2
3	1570837	Solid tyre 22", light, grey <i>Replaces 728.1031</i>		2
5	SP1590118	Tyre Rightrun black/grey 24" <i>Replaces 1570828</i>		2
6	1570829	Treaded Tyre 24" <i>Replaces 728.1002</i>		2
7	1570838	Tyre 24" Marathon Evolution <i>Replaces 1438521</i>	2006-07-01 →	2
8	1570830	Tyre Mountain bike, 24" <i>Replaces 1420887</i>		2
9	1570839	Solid tyre 24", light, grey <i>Replaces 728.1032</i>		2
10	1570831	Solid tyre 24", profile <i>Replaces 728.1029</i>		2
11	1570840	KIK-Type 24", black, fail-safe <i>Replaces 728.1015</i>		2
13	SP1590412	Tyre Rightrun black/grey 25" <i>Replaces 1570833</i>		2
14	1570841	Tyre 25" Marathon Evolution <i>Replaces 1493769</i>	2006-07-01 →	2
15	SP1590119	Tyre Rightrun black/grey 26" <i>Replaces 1570834</i>		2
16	1570842	Tyre 26" Marathon Evolution <i>Replaces 1517977</i>	2008-11-14 →	2
17	1570835	Indoor sports tyre 26" / 10 bar <i>Replaces 729.1002</i>		2



# Inner tube and rim protector

		22"	24"	25"	26"	22"	24"	24"	25"	26"	24"	22"	24"	24"	24"	22"	26"
	A	1	2	-	5	1	2	2	-	5	-	-	-	-	-	-	-
	B	6	7	-	9	6	7	7	-	9	-	6	7	7	7	6	9
	A	-	2	4	-	-	2	2	4	-	-	-	-	-	-	-	-
	B	-	8	8	-	-	8	8	8	-	-	-	-	-	-	-	-
	A	-	-	-	-	-	-	-	-	-	3	-	-	-	-	-	-
	B	-	-	-	-	-	-	-	-	-	6	-	-	-	-	-	-
	A	-	2	-	-	-	2	2	-	-	-	-	-	-	-	-	-
	B	-	7	-	-	-	7	7	-	-	-	-	7	7	7	-	-
	A	-	2	4	-	-	2	2	4	-	-	-	-	-	-	-	-
	B	-	8	8	-	-	8	8	8	-	-	-	8	8	8	-	-
	A	1	2	-	5	1	2	2	-	5	-	-	-	-	-	-	-
	B	6	7	-	9	6	7	7	-	9	-	6	7	7	7	6	9

Inner Tube (A)

Rim protector (B)

10









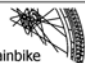
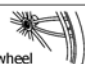


Pos.	Part number	Description	Valid from → until	Qty
1	1570848	Inner Tube 22" x 1" <i>Replaces 727.1007</i>		2
2	1570849	Inner Tube 24" x 1" <i>Replaces 727.1002</i>		2
3	1570852	Inner tube for Mountain bike <i>Replaces 1420884</i>		2
4	1570851	Inner tube 26" for High-Performance wheel <i>Replaces 1430886</i>		2
5	1570850	Inner Tube 26" x 1" <i>Replaces 727.1003</i>		2
6	1570844	Rim protection 22" <i>Replaces 726.1005</i>		2
7	1570845	Rim protection 24" Large <i>Replaces 726.1008.000</i>		2
8	1495962	Rim Protection for High Performance / Spinergy wheel		2
9	1570846	Rim protection 26" <i>Replaces 726.1003</i>		2
10	1571072	Rear wheel inset anti-flat for 20,22,24,26" tyre <i>Replaces 1424917</i>		2

# Spokes guard

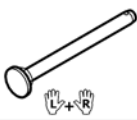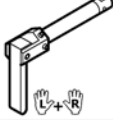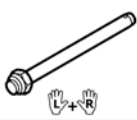


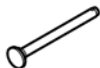

										
	22"	24"	26"	24"	24"	24"	24"	24"	22"	24"
 Küschall wheel	1	2	3	4	5	6	7	8	9	10
 High Performance	-	2	-	4	5	6	7	8	-	10
 Mountainbike	1	-	-	-	-	-	-	-	9	-
 Spiderwheel	-	-	-	-	-	-	-	-	-	-
 Spinerger	-	2	-	4	5	6	7	8	-	10
 Drumbrake	-	11	-	-	-	-	-	-	-	-

Pos.	Part number	Description	Valid from → until	Qty
1	1564453	Spokes guard Transparent 22" <i>Replaces 736.1005</i>		2
2	1564457	Spokes guard Transparent 24" <i>Replaces 736.1004</i>		2
3	1564492	Spokes guard Transparent 26" <i>Replaces 736.1003</i>		2
4	1564456	Spokes guard Küschall, black 24" <i>Replaces 736.1002</i>		2
5	1564459	Spokes guard Carbon 24" <i>Replaces 736.1014</i>	2004-01-01 → 2012-12-31	2
6	1564462	Spokes guard Switzerland 24" <i>Replaces 1497108</i>	2007-01-01 →	2
7	1564463	Spokes guard Pirates 24" <i>Replaces 1497109</i>	2007-01-01 →	2
11	1583436	Spokes guard Transparent 24" for rear wheel 24" Drumbrake Octopus	2014-08-19 →	2




# Quick release axle and bearing

					
 Küschall wheel	1	2	3	7	
 High Performance	1	2	3	7	
 Mountainbike	1	2	3	7	
 Spiderwheel	-	2	3	8	
 Spinerge	5	6	-	7	
 Drumbrake	4	-	-	8	


			
		x	✓
		✓	✓

Pos.	Part number	Description	Valid from → until	Qty
1	1509137-0004	Quick release axles long with cap, pair, black	2010-01-01 →	1
1	1509137-0400	Quick release axles long with cap, pair, tecno anthracite	2008-01-01 →	1
1	1509137-0401	Quick release axles long with cap, pair, lemon	2008-01-01 →	1
1	1509137-0404	Quick release axles long with cap, pair, b-blue	2008-01-01 →	1
1	1509137-0407	Quick release axles long with cap, pair, colourless	2008-01-01 →	1
1	1509137-0440	Quick release axles long with cap, pair, comet red	2008-01-01 →	1
2	1493755	Tetra clip with quick release axle, pair, long		1
4	1571103	Quickrelease axle for drum brake, pair <i>Replaces 1438987</i>		1
5	1507627	Quickrelease axle long with cap for Spinerge wheel, pair		1
6	1512029	Tetra clip with quick release axle, long, for Spinerge wheel, pair		1
7	1571069	Bearing R8-2RS 1/2"/1.1/8"x5/16", pair		2
8	1571070	Bearing for Spider wheel, pair		2

**BRAKES**

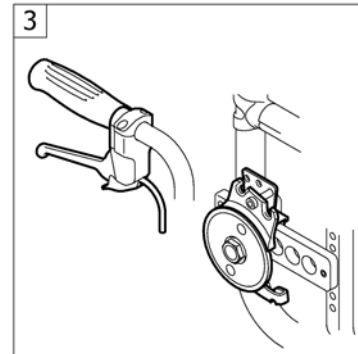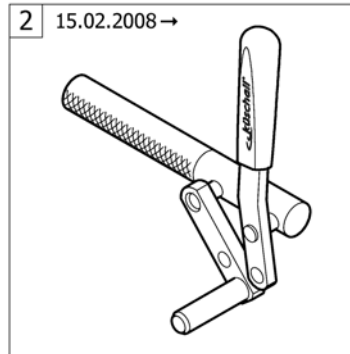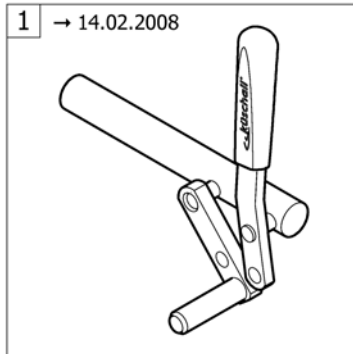
Pos.	Part number	Description	Valid from → until	Qty
2	-	Brakes 15.02.2008=>	2008-02-15 → 2012-12-31	1
3	-	Drum brake	2004-01-01 → 2012-12-31	1

# Brakes 15.02.2008=>

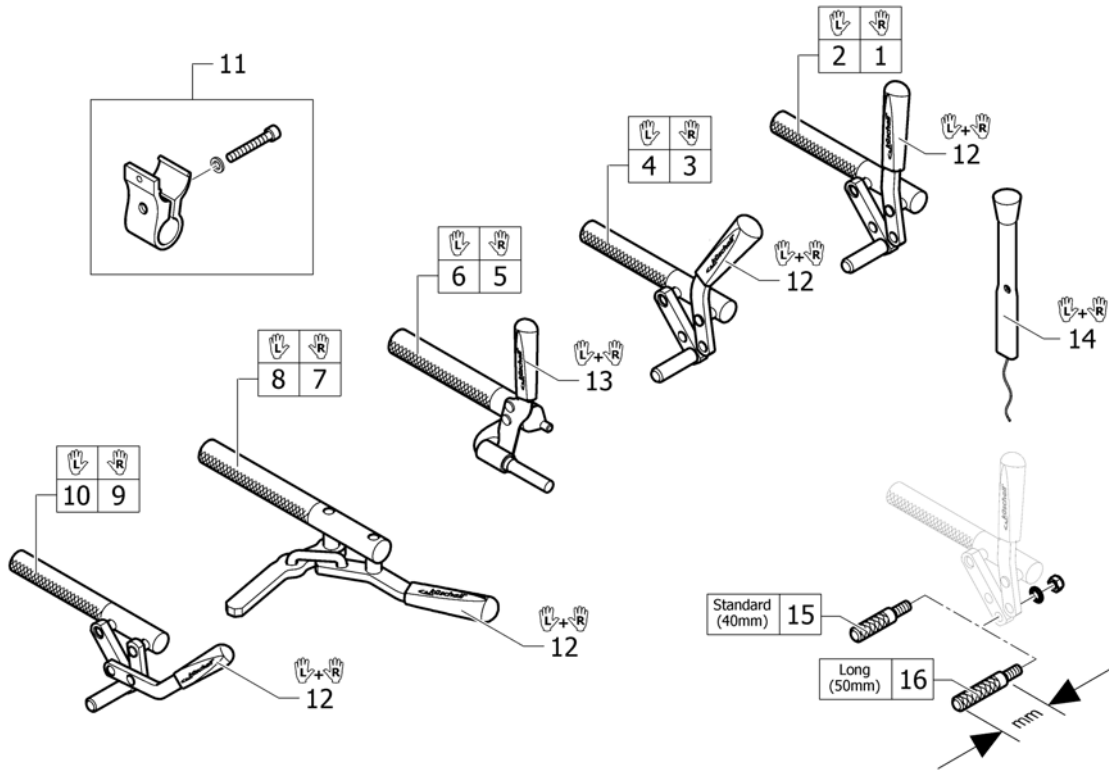

Pos.	Part number	Description	Valid from → until	Qty
1	1512128	Brake, straight lever, right		1
2	1512127	Brake, straight lever, left		1
3	1512126	Brake, bent lever, right		1
4	1512125	Brake, bent lever, left		1
5	SP1512132	Performance brake right		1
6	SP1512131	Performance brake left		1
7	SP1512134	Active brake right		1
8	SP1512133	Active brake left		1
9	1512130	Pull brake right		1
10	1512129	Pull brake left		1
11	1513363	brake clamping piece complete UL/C07		2
12	1513177	Brake handle standard, pair <i>Replaces 1432056</i>		1
13	1513178	Brake handle for performance brake, pair <i>Replaces 1432055</i>		1
14	1513179	Brake lever extension, pair <i>Replaces 901.1095</i>		1
15	1579328	Brake shoe standard (40mm), kit 4 Pcs. <i>Replaces 500.1045 (4x)</i>		1
16	1579329	Brake shoe long (50mm), kit 4 Pcs. <i>Replaces 500.1013 (4x)</i>		1


# Drum brake

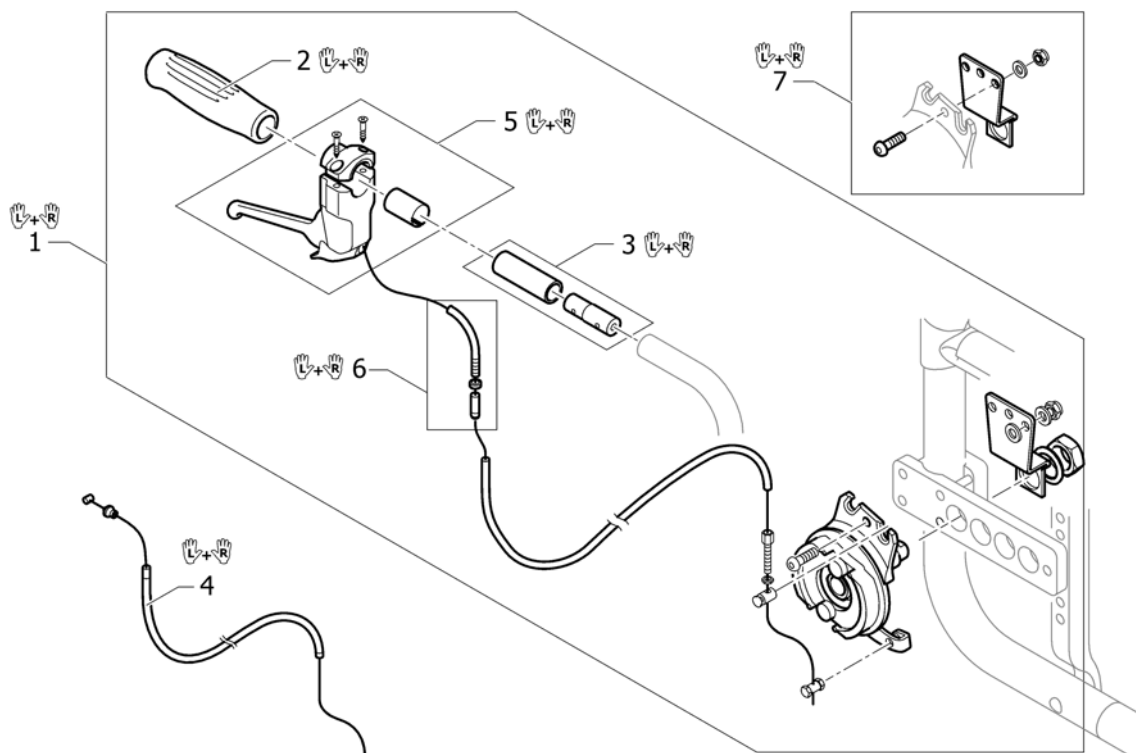


Pos.	Part number	Description	Valid from → until	Qty
1	1439456	Drumbrake set Compact / Ultra Light 04		1
2	1507734	Handle black, pair		1
3	1527108	Push handle extension for drumbrake, pair		1
4	1564310	Drumbrake cable with hose, pair		1
5	1580442	Drumbrake handle, pair		1
6	1580441	Guide tube for Drumbrake, pair		1
7	1583458	Latch pivot point for drumbrake, pair		1



## OPTIONS & ACCESSORIES

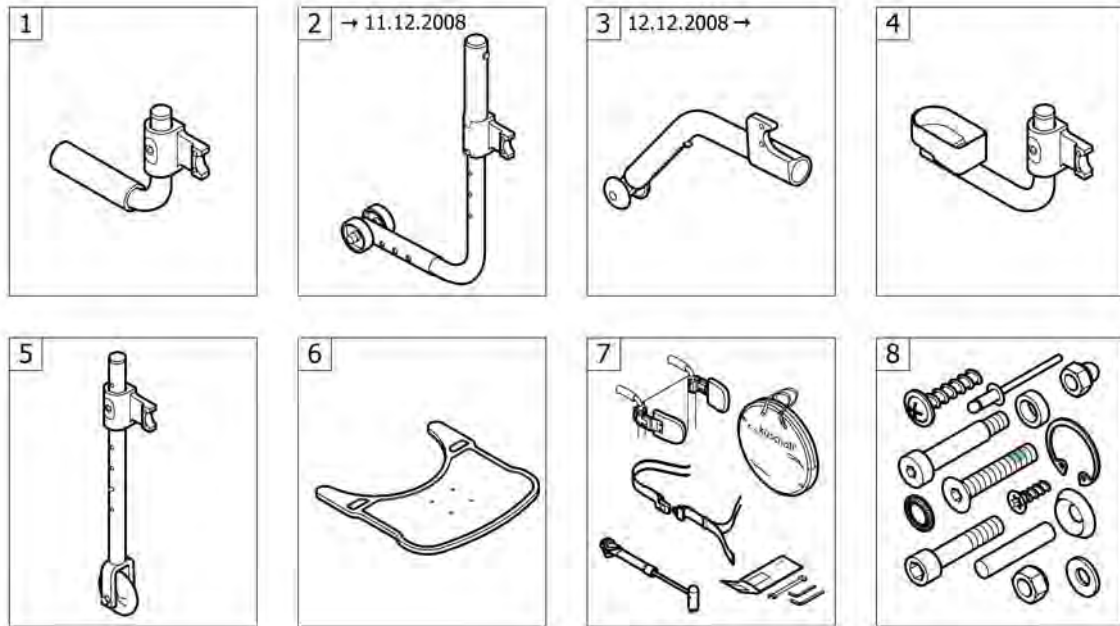


Pos.	Part number	Description	Valid from → until	Qty
1	-	Tipper aid	2004-01-01 → 2012-12-31	1
3	-	Antitipper 12.12.2008=>	2008-12-12 → 2012-12-31	1
4	-	Cane holder	2004-01-01 → 2012-12-31	1
5	-	Transit wheels	2004-01-01 → 2012-12-31	1
6	-	Therapy table	2004-01-01 → 2012-12-31	1
7	-	Chair accessories	2004-01-01 → 2012-12-31	1
8	-	Fixation: screws, ...	2004-01-01 → 2012-12-31	1

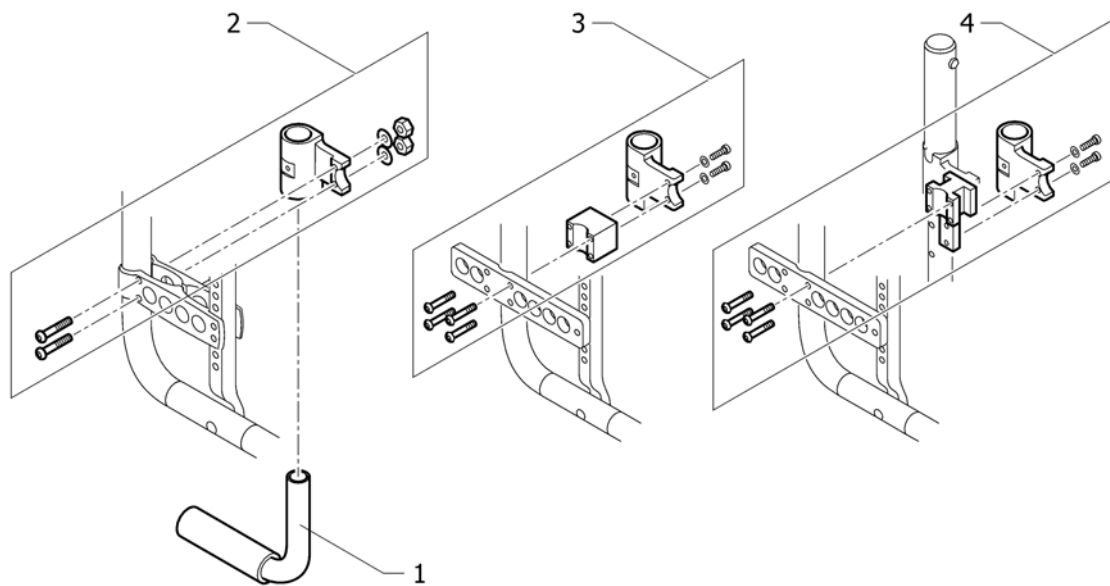
**Tipper aid**

Pos.	Part number	Description	Valid from → until	Qty
1	1438213	Tipper-Aid complete UL/C04		1
2	1438217	Adaptation holder set (standard adapter plate)		1
3	1439468	Adaptation holder kit (adapter plate for rear wheel ext. or drum brake)		2
4	1439467	Adaptation holder kit for 2nd Adaptation (adapter plate for rear wheel ext. or drum brake)		2

# Antitipper 12.12.2008=>

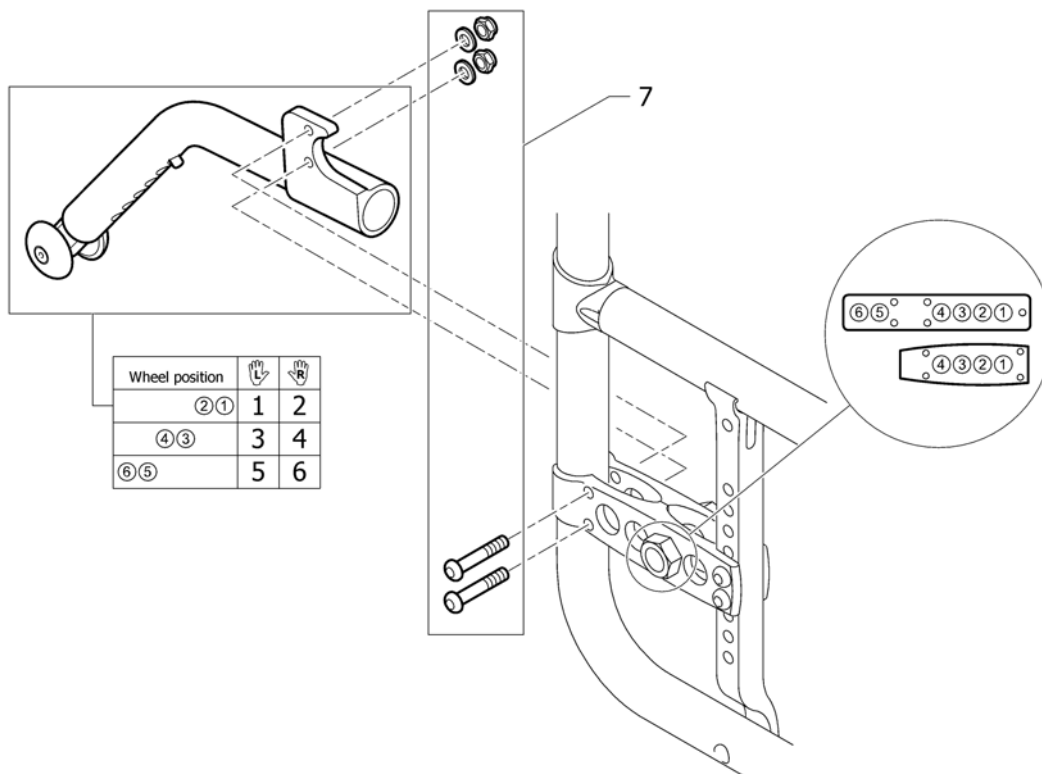


Pos.	Part number	Description	Valid from → until	Qty
1	1524910	Antitipper short left		1
2	1524909	Antitipper short right		1
3	1524912	Antitipper medium left		1
4	1524911	Antitipper medium right		1
5	1524914	Antitipper long left		1
6	1524913	Antitipper long right		1
7	1445447	Fixation kit to Antitipper (standard adapter plate)		1

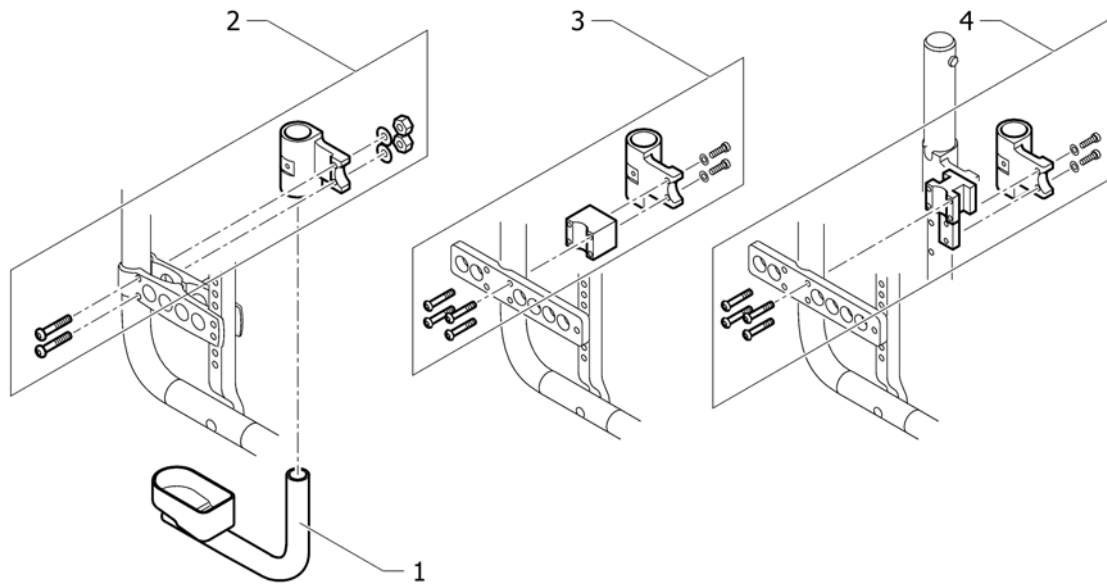**Cane holder**


Pos.	Part number	Description	Valid from → until	Qty
1	1438214	cane holder complete UL/C04		1
2	1438217	Adaptation holder set (standard adapter plate)		1
3	1439468	Adaptation holder kit (adapter plate for rear wheel ext. or drum brake)		2
4	1439467	Adaptation holder kit for 2nd Adaptation (adapter plate for rear wheel ext. or drum brake)		2

## Transit wheels

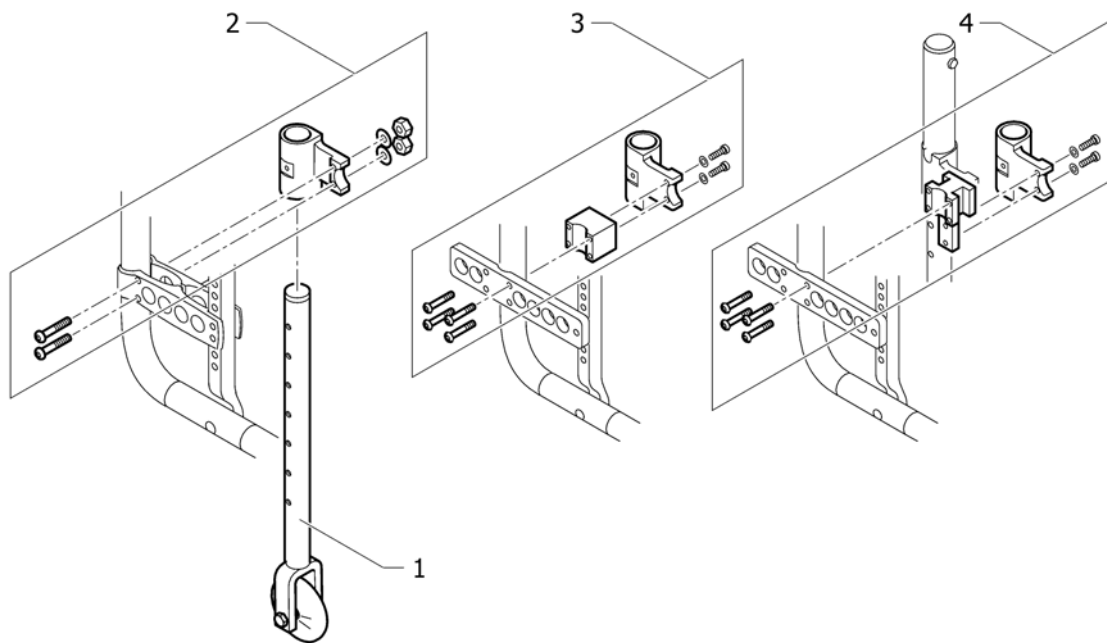

Pos.	Part number	Description	Valid from → until	Qty
1	1438215	pair transit wheels complete UL/C04		1
2	1438217	Adaptation holder set (standard adapter plate)		1
3	1439468	Adaptation holder kit (adapter plate for rear wheel ext. or drum brake)		2
4	1439467	Adaptation holder kit for 2nd Adaptation (adapter plate for rear wheel ext. or drum brake)		2


## Therapy table

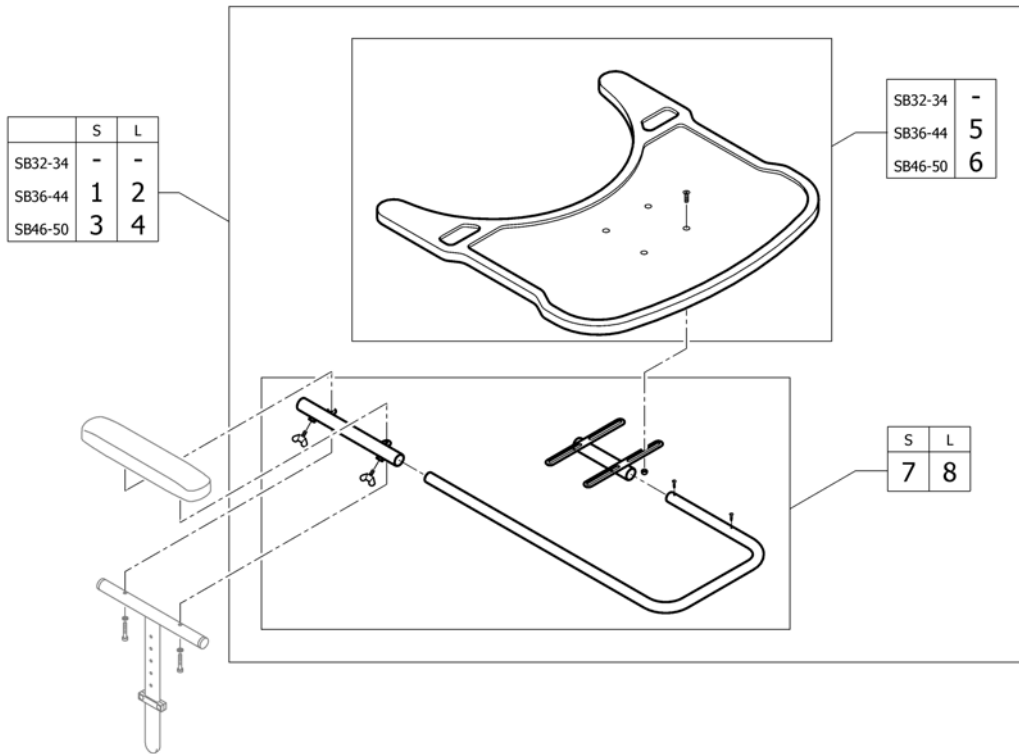


Pos.	Part number	Description	Valid from → until	Qty
1	1439492	Therapy table M complete for Armrest short		1
2	1438374	Therapy table M complete to Armrest long		1
3	1439493	Therapy table L complete for Armrest short		1
4	1438375	Therapy table L complete to Armrest long		1
5	1582670	Therapy table M		1
6	1582671	Therapy table L		1
7	1582672	Fixation kit for therapy table on armrest short		1
8	1582673	Fixation kit for therapy table on armrest long		1



## Chair accessories

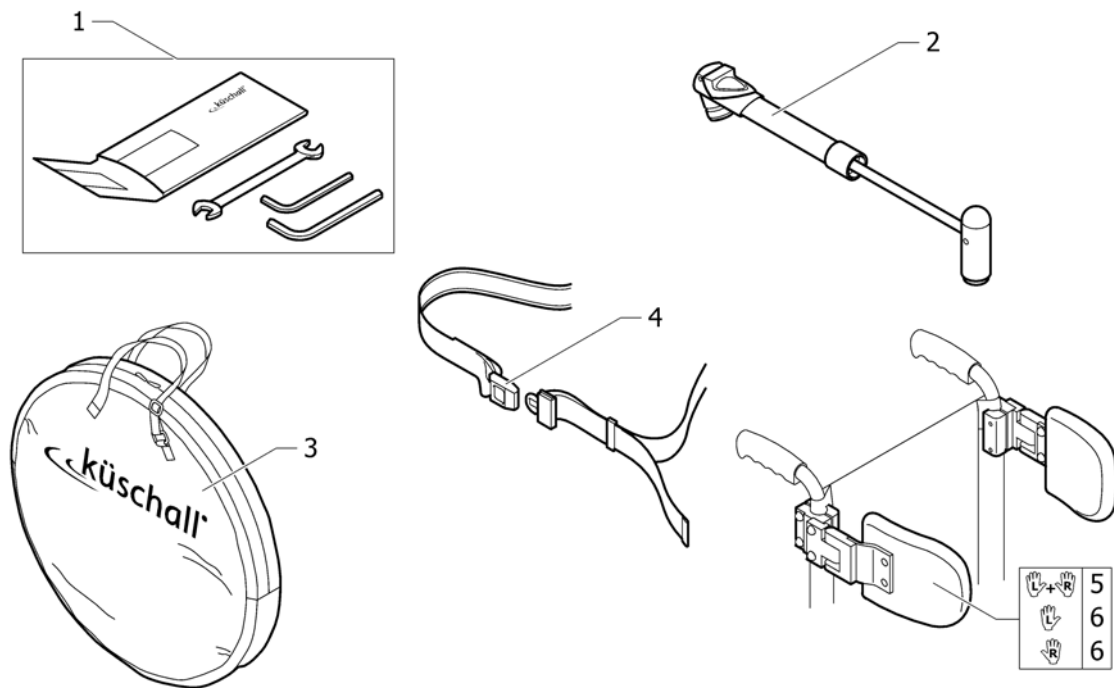


Pos.	Part number	Description	Valid from → until	Qty
1	1518285	Tool Kit <i>Replaces 800.1013</i>		1
2	1517924	Air pump <i>Replaces 800.1013</i>		1
3	1518287	Wheel bag 24" - 26" <i>Replaces 1420922</i>		1
4	1518288	Seat belt		1
5	1518289	Lateralsupport, pair		1
6	1564299	Lateralsupport		2

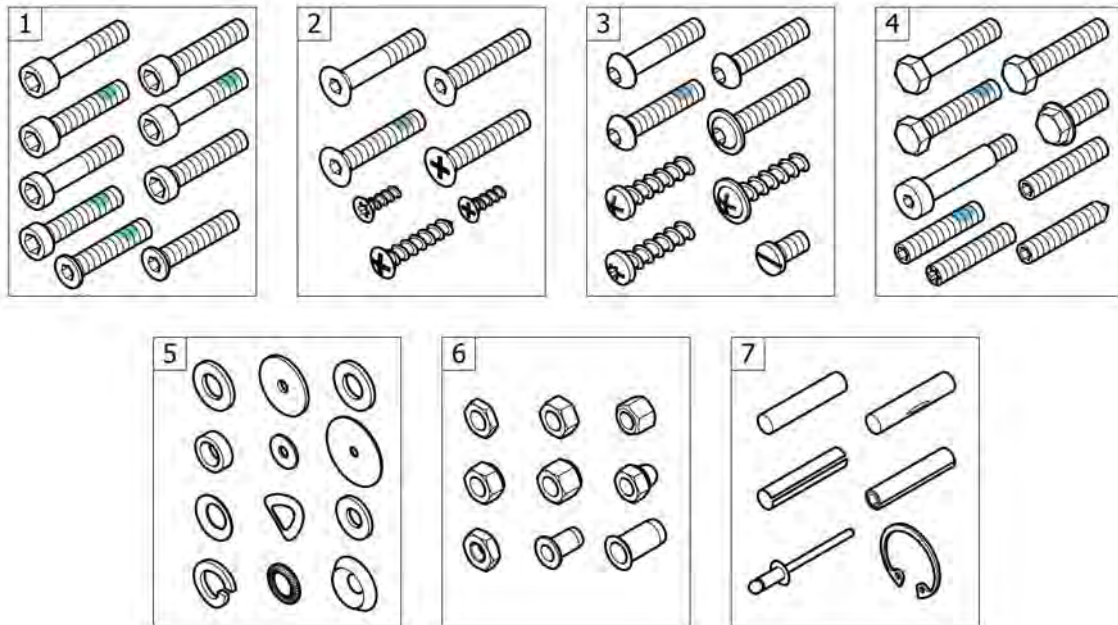**Fixation: screws, ...**

Pos.	Part number	Description	Valid from → until	Qty
1	-	Socket head cap screws		1
2	-	Countersunk screws		1
3	-	Button head screws		1
4	-	Hex-head screws, diverse, ...		1
5	-	Washers		1
6	-	Nuts		1
7	-	Pins, rivets, ...		1

# Socket head cap screws

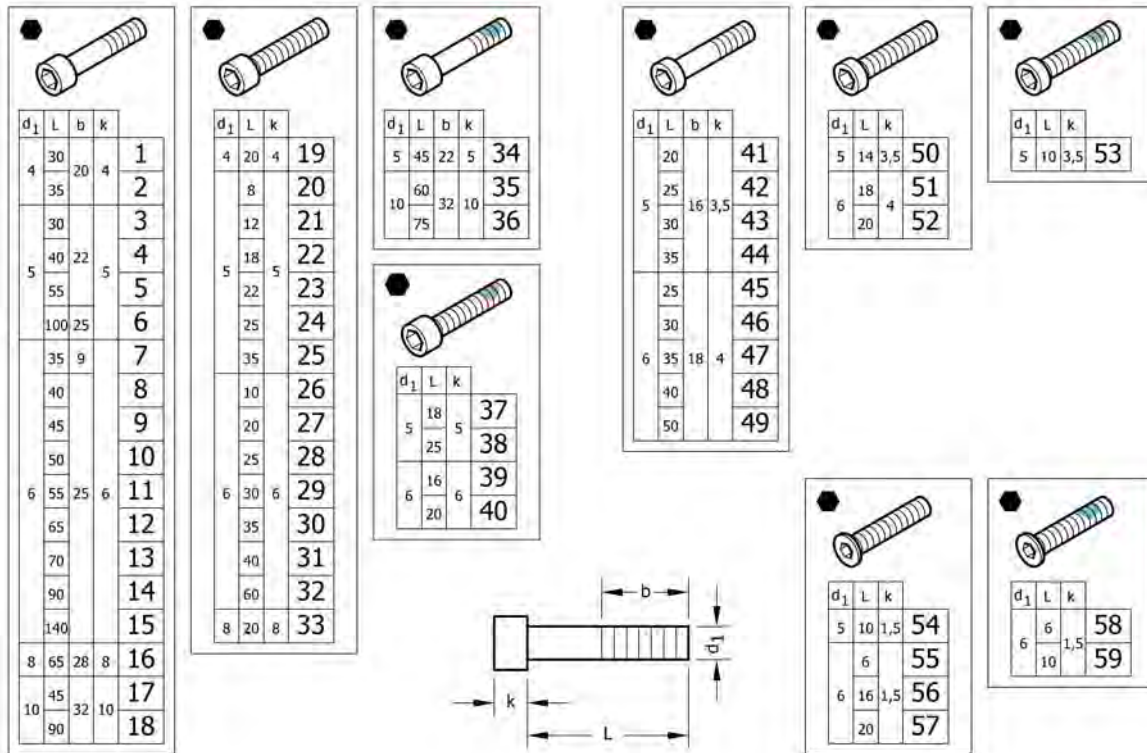

Pos.	Part number	Description	Valid from → until	Qty
1	1579115	Hex socket head cap screws partially threaded, zi.Blue, M4x30, kit 25 <i>Replaces 1560191 (25x)</i>		1
2	1579116	Hex socket head cap screws partially threaded, zi.Blue, M4x35, kit 25 <i>Replaces 1563765 (25x)</i>		1
3	1579117	Hex socket head cap screws partially threaded, black, M5x30, kit 25 Pcs. <i>Replaces 100.1091 (25x)</i>		1
4	1579118	Hex socket head cap screws partially threaded, black, M5x40, kit 25 Pcs. <i>Replaces 100.1093 (25x)</i>		1
5	1579119	Hex socket head cap screws partially threaded, black, M5x55, kit 25 Pcs. <i>Replaces 100.1096 (25x)</i>		1
6	1579120	Hex socket head cap screws partially threaded, black, M5x100, kit 10 Pcs. <i>Replaces 1425240 (10x)</i>		1
7	1579121	Hex socket head cap screws partially threaded, black, M6x35, kit 5 Pcs. <i>Replaces 107.1001 (5x)</i>		1
8	1579122	Hex socket head cap screws partially threaded, black, M6x40, kit 25 Pcs. <i>Replaces 100.1133 (25x)</i>		1
9	1579123	Hex socket head cap screws partially threaded, black, M6x45, kit 25 Pcs. <i>Replaces 100.1134 (25x)</i>		1
10	1579124	Hex socket head cap screws partially threaded, black, M6x50, kit 25 Pcs. <i>Replaces 100.1135 (25x)</i>		1
11	1579125	Hex socket head cap screws partially threaded, black, M6x55, kit 25 Pcs. <i>Replaces 100.1136 (25x)</i>		1
11	1579126	Hex socket head cap screws partially threaded, zi.Blue, M6x55, kit 25 <i>Replaces ZM006551A (25x)</i>		1
12	1579127	Hex socket head cap screws partially threaded, black, M6x65, kit 10 Pcs. <i>Replaces 100.1138 (10x)</i>		1
13	1579128	Hex socket head cap screws partially threaded, black, M6x70, kit 10 Pcs. <i>Replaces 100.1139 (10x)</i>		1
14	1579129	Hex socket head cap screws partially threaded, black, M6x90, kit 10 Pcs. <i>Replaces 1529118 (10x)</i>		1
15	1579130	Hex socket head cap screws partially threaded, black, M6x140, kit 10 Pcs. <i>Replaces 1529117 (10x)</i>		1
16	1579131	Hex socket head cap screws partially threaded, black, M8x65, kit 10 Pcs. <i>Replaces 1510316 (10x)</i>		1
17	1579132	Hex socket head cap screws partially threaded, black, M10x45, kit 2 Pcs. <i>Replaces 1496010 (2x)</i>		1


**Socket head cap screws**

Pos.	Part number	Description	Valid from → until	Qty
18	1579133	Hex socket head cap screws partially threaded, black, M10x90, kit 2 Pcs. <i>Replaces 1496013 (2x)</i>		1
19	1579134	Hex socket head cap screws fully threaded, black, M4x20, kit 25 Pcs. <i>Replaces 100.1050 (25x)</i>		1
20	1579135	Hex socket head cap screws fully threaded, black, M5x8, kit 5 Pcs. <i>Replaces 1493753 (5x)</i>		1
21	1579136	Hex socket head cap screws fully threaded, black, M5x12, kit 25 Pcs. <i>Replaces 100.1084 (25x)</i>		1
22	1579137	Hex socket head cap screws fully threaded, black, M5x18, kit 25 Pcs. <i>Replaces 100.1087 (25x)</i>		1
23	1579138	Hex socket head cap screws fully threaded, black, M5x22, kit 25 Pcs. <i>Replaces 100.1089 (25x)</i>		1
24	1579139	Hex socket head cap screws fully threaded, black, M5x25, kit 25 Pcs. <i>Replaces 100.1090 (25x)</i>		1
25	1579140	Hex socket head cap screws fully threaded, black, M5x35, kit 25 Pcs. <i>Replaces 100.1092 (25x)</i>		1
26	1579141	Hex socket head cap screws fully threaded, black, M6x10, kit 25 Pcs. <i>Replaces 100.1123 (25x)</i>		1
27	1579142	Hex socket head cap screws fully threaded, black, M6x20, kit 25 Pcs. <i>Replaces 100.1128 (25x)</i>		1
28	1579143	Hex socket head cap screws fully threaded, black, M6x25, kit 25 Pcs. <i>Replaces 100.1130 (25x)</i>		1
29	1579144	Hex socket head cap screws fully threaded, black, M6x30, kit 25 Pcs. <i>Replaces 100.1131 (25x)</i>		1
30	1579145	Hex socket head cap screws fully threaded, black, M6x35, kit 25 Pcs. <i>Replaces 100.1132 (25x)</i>		1
31	1579146	Hex socket head cap screws fully threaded, black, M6x40, kit 10 Pcs. <i>Replaces 1563349 (10x)</i>		1
32	1579147	Hex socket head cap screws fully threaded, black, M6x60, kit 10 Pcs. <i>Replaces 100.1137 (10x)</i>		1
33	1579148	Hex socket head cap screws fully threaded, black, M8x20, kit 10 Pcs. <i>Replaces 1510307 (10x)</i>		1
34	1579149	Hex sock.Head cap screws partial.Thread., black, TufLok, M5x45, kit 5 Pcs. <i>Replaces 1496014 (5x)</i>		1
35	1579150	Hex sock.Head cap screws partial.Thread., black, TufLok, M10x60, kit 2 Pcs. <i>Replaces 1496011 (2x)</i>		1
36	1579151	Hex sock.Head cap screws partial.Thread., black, TufLok, M10x75, kit 2 Pcs. <i>Replaces 1496012 (2x)</i>		1
37	1579152	Hex socket head cap screws fully threaded, black, TufLok, M5x18, kit 10 Pcs. <i>Replaces 1425241 (10x)</i>		1
38	1579153	Hex socket head cap screws fully threaded, black, TufLok, M5x25, kit 10 Pcs. <i>Replaces 1570153 (10x)</i>		1
39	1579154	Hex socket head cap screws full.Thread., zi.Blue, TufLok, M6x16, kit 10 Pcs. <i>Replaces 1560219 (10x)</i>		1
40	1579155	Hex socket head cap screws fully threaded, black, TufLok, M6x20, kit 10 Pcs. <i>Replaces 1498775 (10x)</i>		1
41	1579156	Hex sock.Head cap screws, low head, part.Thread., black, M5x20, kit 25 Pcs. <i>Replaces 1438501 (25x)</i>		1
42	1579157	Hex sock.Head cap screws, low head, part.Thread., black, M5x25, kit 25 Pcs. <i>Replaces 1438518 (25x)</i>		1
43	1579158	Hex sock.Head cap screws, low head, part.Thread., black, M5x30, kit 25 Pcs. <i>Replaces 100.5090 (25x)</i>		1
44	1579159	Hex sock.Head cap screws, low head, part.Thread., black, M5x35, kit 25 Pcs. <i>Replaces 100.5091 (25x)</i>		1

**Socket head cap screws**

Pos.	Part number	Description	Valid from → until	Qty
45	1579160	Hex sock.Head cap screws, low head, part.Thread., black, M6x25, kit 25 Pcs. <i>Replaces 1432120 (25x)</i>		1
46	1579161	Hex sock.Head cap screws, low head, part.Thread., black, M6x30, kit 25 Pcs. <i>Replaces 100.5130 (25x)</i>		1
47	1579162	Hex sock.Head cap screws, low head, part.Thread., black, M6x35, kit 25 Pcs. <i>Replaces 1438504 (25x)</i>		1
48	1579163	Hex sock.Head cap screws, low head, part.Thread., black, M6x40, kit 25 Pcs. <i>Replaces 100.5132 (25x)</i>		1
49	1579164	Hex sock.Head cap screws, low head, part.Thread., black, M6x50, kit 25 Pcs. <i>Replaces 100.5134 (25x)</i>		1
50	1579165	Hex sock.Head cap screws, low head, part.Thread., black, M5x14, kit 25 Pcs. <i>Replaces 100.5085 (25x)</i>		1
51	1579166	Hex sock.Head cap screws, low head, part.Thread., black, M6x18, kit 25 Pcs. <i>Replaces 100.5126 (25x)</i>		1
52	1579167	Hex sock.Head cap screws, low head, part.Thread., black, M6x20, kit 25 Pcs. <i>Replaces 100.5127 (25x)</i>		1
53	1579168	Hex soc.Head cap scr., low head, part.Threa., bl., TufL., M5x10, kit 10 Pcs. <i>Replaces 1496026 (10x)</i>		1
54	1579169	Hex socket head cap screws, special low head, black, M5x10, kit 25 Pcs. <i>Replaces 1437290 (25x)</i>		1
55	1579170	Hex socket head cap screws, special low head, black, M6x6, kit 25 Pcs. <i>Replaces 1425182 (25x)</i>		1
56	1579171	Hex socket head cap screws, special low head, black, M6x16, kit 25 Pcs. <i>Replaces 1541181 (25x)</i>		1
57	1579172	Hex socket head cap screws, special low head, black, M6x20, kit 25 Pcs. <i>Replaces 107.1022 (25x)</i>		1
58	1579173	Hex socket head cap screws, spec.Low head, black, TufLok, M5x6, kit 10 Pcs. <i>Replaces 1532039 (10x)</i>		1
59	1579174	Hex socket head cap screws, spec.Low head, black, TufLok, M6x10, kit 10 Pcs. <i>Replaces 1420946 (10x)</i>		1

## Countersunk screws

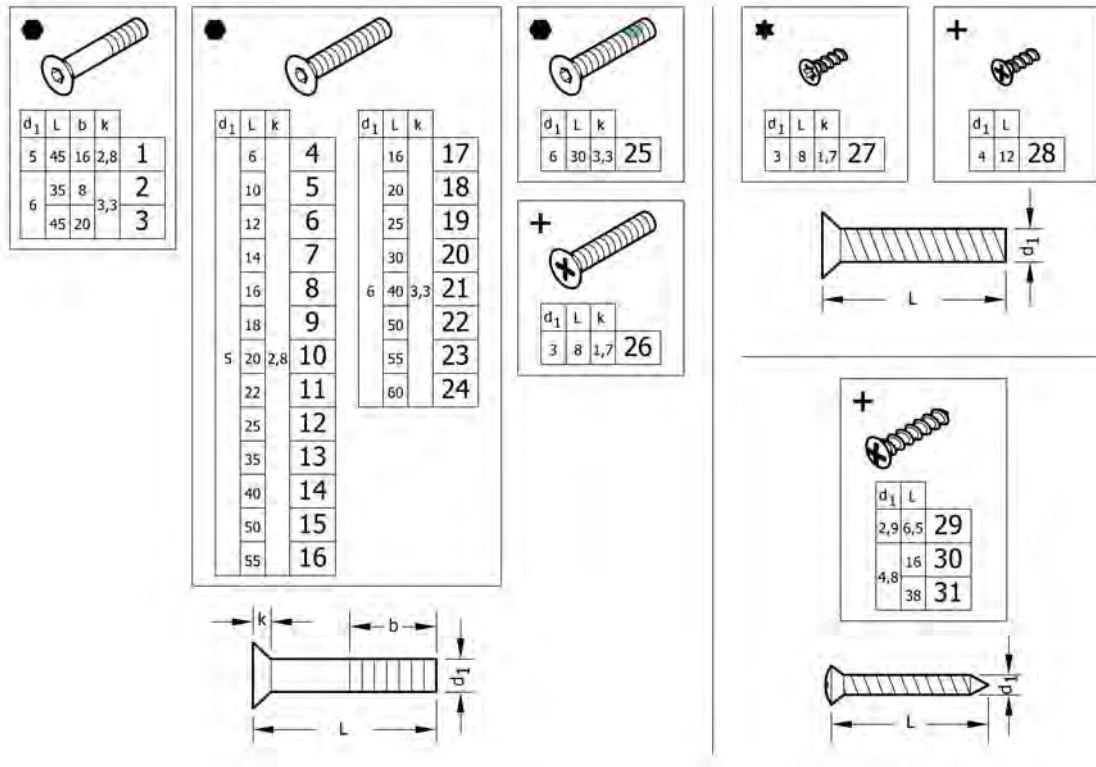


Pos.	Part number	Description	Valid from → until	Qty
1	1579175	Countersunk head screws partially threaded, Black, M5x45, kit 10 Pcs. <i>Replaces 1438773 (10x)</i>		1
2	1579176	Countersunk head screws partially threaded, Black, M6x35, kit 10 Pcs. <i>Replaces 107.1004 (10x)</i>		1
3	1579177	Countersunk head screws partially threaded, Black, M6x45, kit 10 Pcs. <i>Replaces 100.3133 (10x)</i>		1
4	1579178	Countersunk head screws fully threaded, Black, M5x6, kit 25 Pcs. <i>Replaces 1445273 (25x)</i>		1
5	1579179	Countersunk head screws fully threaded, Black, M5x10, kit 25 Pcs. <i>Replaces 100.3082 (25x)</i>		1
6	1579180	Countersunk head screws fully threaded, Black, M5x12, kit 25 Pcs. <i>Replaces 1438999 (25x)</i>		1
7	1579181	Countersunk head screws fully threaded, Black, M5x14, kit 25 Pcs. <i>Replaces 107.1015 (25x)</i>		1
8	1579182	Countersunk head screws fully threaded, Black, M5x16, kit 25 Pcs. <i>Replaces 100.3085 (25x)</i>		1
9	1579183	Countersunk head screws fully threaded, Black, M5x18, kit 25 Pcs. <i>Replaces 100.3086 (25x)</i>		1
10	1579184	Countersunk head screws fully threaded, Black, M5x20, kit 25 Pcs. <i>Replaces 100.3087 (25x)</i>		1
11	1579185	Countersunk head screws fully threaded, Black, M5x22, kit 25 Pcs. <i>Replaces 1416930 (25x)</i>		1
12	1579186	Countersunk head screws fully threaded, Black, M5x25, kit 25 Pcs. <i>Replaces 100.3089 (25x)</i>		1
13	1579187	Countersunk head screws fully threaded, Black, M5x35, kit 25 Pcs. <i>Replaces 100.3091 (25x)</i>		1
14	1579188	Countersunk head screws fully threaded, Black, M5x40, kit 25 Pcs. <i>Replaces 1432148 (25x)</i>		1
15	1579189	Countersunk head screws fully threaded, Black, M5x50, kit 25 Pcs. <i>Replaces 100.3094 (25x)</i>		1
16	1579190	Countersunk head screws fully threaded, Black, M5x55, kit 25 Pcs. <i>Replaces 100.3095 (25x)</i>		1
17	1579191	Countersunk head screws fully threaded, Black, M6x16, kit 25 Pcs. <i>Replaces 100.3125 (25x)</i>		1
18	1579192	Countersunk head screws full.Threaded, zinc flake coat., M6x20, kit 25 Pcs. <i>Replaces 100.3127 (25x)</i>		1



**Countersunk screws**

Pos.	Part number	Description	Valid from → until	Qty
18	1579193	Countersunk head screws fully threaded, Black, M6x20, kit 25 Pcs. <i>Replaces 1437809 (25x)</i>		1
19	1579194	Countersunk head screws fully threaded, Black, M6x25, kit 25 Pcs. <i>Replaces 100.3129 (25x)</i>		1
20	1579195	Countersunk head screws fully threaded, Black, M6x30, kit 25 Pcs. <i>Replaces 100.3130 (25x)</i>		1
21	1579196	Countersunk head screws fully threaded, Black, M6x40, kit 25 Pcs. <i>Replaces 100.3132 (25x)</i>		1
22	1579197	Countersunk head screws fully threaded, Black, M6x50, kit 25 Pcs. <i>Replaces 100.3134 (25x)</i>		1
23	1579198	Countersunk head screws fully threaded, Black, M6x55, kit 25 Pcs. <i>Replaces 100.3135 (25x)</i>		1
24	1579199	Countersunk head screws fully threaded, Black, M6x60, kit 25 Pcs. <i>Replaces 100.3136 (25x)</i>		1
25	1579200	Countersunk head screws fully threaded, Black, TufLok, M6x30, kit 25 Pcs. <i>Replaces 1550479 (25x)</i>		1
26	1579201	Countersunk head screws fully threaded, zi.Blue, M3x8, kit 25 Pcs. <i>Replaces 1529279 (25x)</i>		1
27	1579202	Countersunk head PT screws, Torx, stainless steel, 3x8, kit 25 Pcs. <i>Replaces 1529278 (25x)</i>		1
28	1579203	Countersunk head PT screws, Black, KA40x12, kit 25 Pcs. <i>Replaces 1438513 (25x)</i>		1
29	1579204	Countersunk head tapping screws with cone end, Black, ST2,9x6,5, kit 25 Pcs. <i>Replaces 126.5041 (25x)</i>		1
30	1579205	Countersunk head tapping screws with cone end, Black, ST4,8x16, kit 25 Pcs. <i>Replaces 126.5202 (25x)</i>		1
31	1579206	Countersunk head tapping screws with cone end, Black, ST4,8x38, kit 25 Pcs. <i>Replaces 126.5207 (25x)</i>		1

## Button head screws

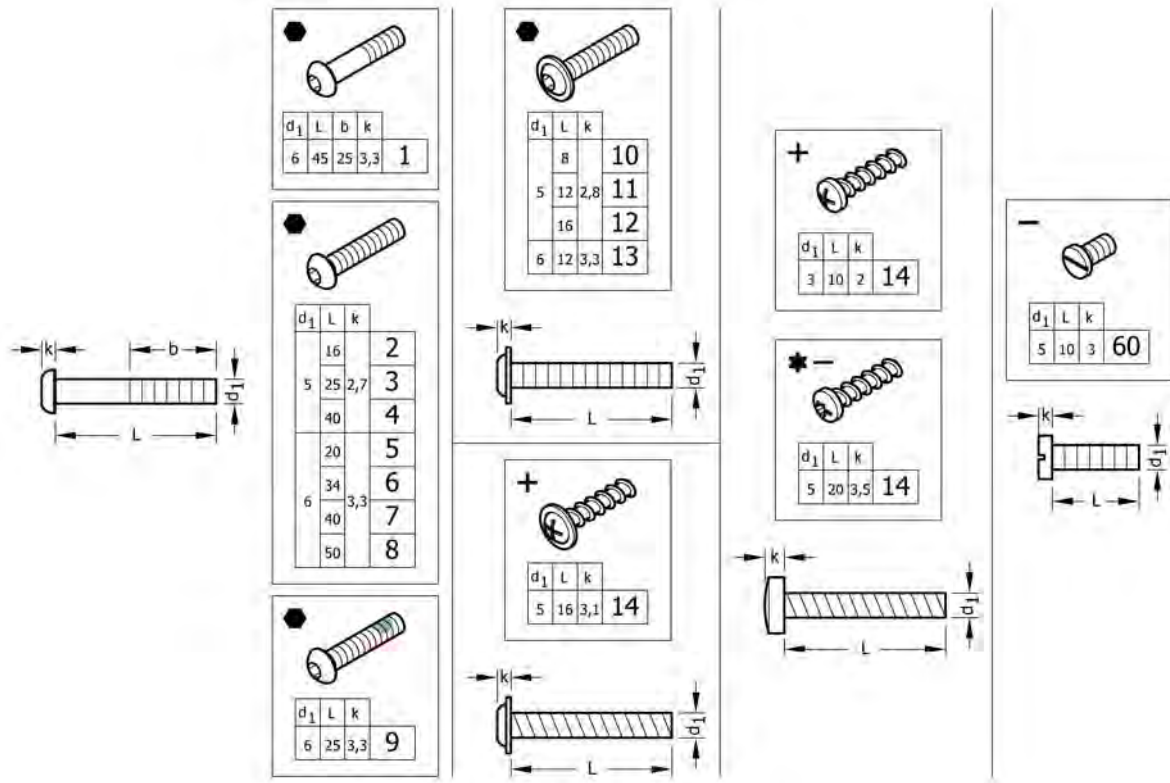


Pos.	Part number	Description	Valid from → until	Qty
1	1579259	Hex sock.Button head cap screws partially thread., Black, M6x45, kit 25 Pcs. <i>Replaces 1438507 (25x)</i>		1
2	1579260	Hex socket button head cap screws fully threaded, Black, M5x16, kit 25 Pcs. <i>Replaces 1445341-0003 (25x)</i>		1
3	1579261	Hex socket button head cap screws fully threaded, Black, M5x25, kit 25 Pcs. <i>Replaces 107.1021 (25x)</i>		1
4	1579262	Hex socket button head cap screws fully threaded, Black, M5x40, kit 25 Pcs. <i>Replaces 1445101-0003 (25x)</i>		1
5	1579263	Hex socket button head cap screws fully threaded, Black, M6x20, kit 25 Pcs. <i>Replaces 1439153 (25x)</i>		1
6	1579264	Hex socket button head cap screws fully threaded, Black, M6x34, kit 25 Pcs. <i>Replaces 1438506-0003 (25x)</i>		1
7	1579265	Hex socket button head cap screws fully threaded, Black, M6x40, kit 25 Pcs. <i>Replaces 1439530-0003 (25x)</i>		1
8	1579266	Hex socket button head cap screws fully threaded, Black, M6x50, kit 25 Pcs. <i>Replaces 1439531-0003 (25x)</i>		1
9	1579267	Hex sock.But.Head cap screws full.Thread., Black, TufLok, M6x25, kit 25 Pcs. <i>Replaces 1438505 (25x)</i>		1
10	1579268	Button head socket cap screws with flange, Black, M5x8, kit 25 Pcs. <i>Replaces 1536268 (25x)</i>		1
11	1579269	Button head socket cap screws with flange, Black, M5x12, kit 25 Pcs. <i>Replaces 1529290 (25x)</i>		1
12	1579270	Button head socket cap screws with flange, Black, M5x16, kit 25 Pcs. <i>Replaces 1445102-0003 (25x)</i>		1
13	1579271	Button head socket cap screws with flange, Black, M6x12, kit 25 Pcs. <i>Replaces 1432204 (25x)</i>		1
14	1579272	Round washer head PT® screws, form H, zinc plated blue, M5x16, kit 25 Pcs. <i>Replaces 1536057 (25x)</i>		1

## Button head screws

Pos.	Part number	Description	Valid from → until	Qty
15	1579273	Pan head PT screws, form H, zinc plated blue, M3x10, kit 25 Pcs. <i>Replaces ZM003510A (25x)</i>		1
16	1579274	Pan head PT screws, Torx, Black, M5x20, kit 25 Pcs. <i>Replaces 1424569 (25x)</i>		1
17	1579275	Slotted machine screws, Black, M5x10, kit 25 Pcs. <i>Replaces 102.2083 (25x)</i>		1

## Hex-head screws, diverse, ...

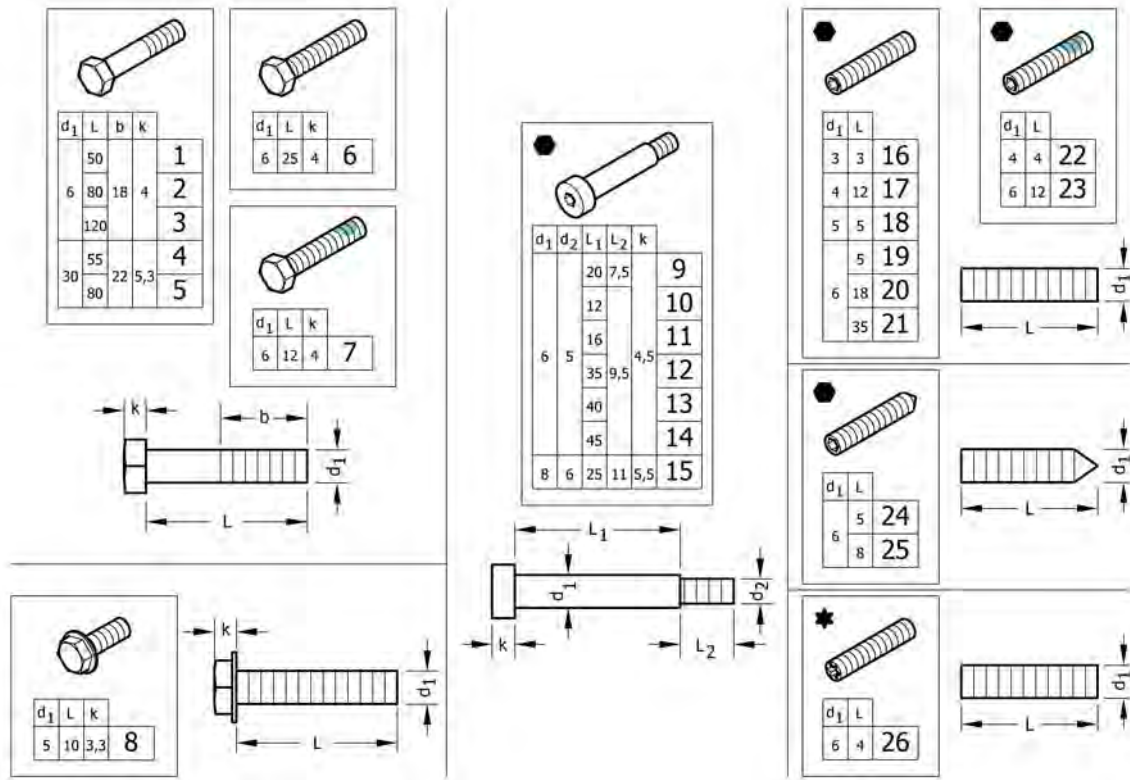


Pos.	Part number	Description	Valid from → until	Qty
1	1579276	Hex head bolts partially threaded, Black, M6x50, kit 10 Pcs. <i>Replaces 101.1134 (10x)</i>		1
2	1579277	Hex head bolts partially threaded, Black, M6x80, kit 10 Pcs. <i>Replaces 1438508 (10x)</i>		1
3	1579278	Hex head bolts partially threaded, Black, M6x120, kit 10 Pcs. <i>Replaces 1438509 (10x)</i>		1
4	1579279	Hex head bolts partially threaded, Black, M8x55, kit 10 Pcs. <i>Replaces 101.1174 (10x)</i>		1
5	1579280	Hex head bolts partially threaded, Black, M8x80, kit 10 Pcs. <i>Replaces 101.1179 (10x)</i>		1
6	1579281	Hex head screws fully threaded, Black, M6x25, kit 10 Pcs. <i>Replaces 1437299 (10x)</i>		1
7	1579282	Hex head screws fully threaded, Black, TufLok, M6x12, kit 10 Pcs. <i>Replaces 1564245 (10x)</i>		1
8	1579283	Hex flange head machine screws with flangeBlack, M5x10, kit 10 Pcs. <i>Replaces 107.1014 (10x)</i>		1
9	1579295	Hex socket head shoulder bolts, Black, Ø6x20 / M5, kit 4 Pcs. <i>Replaces 1488934-0003 (4x)</i>		1
10	1579296	Hex socket head shoulder bolts, Black, Ø6x12 / M5, kit 4 Pcs. <i>Replaces 1564246 (4x)</i>		1
11	1579297	Hex socket head shoulder bolts, Black, Ø6x16 / M5, kit 4 Pcs. <i>Replaces 1529291 (4x)</i>		1
12	1579298	Hex socket head shoulder bolts, Black, Ø6x35 / M5, kit 4 Pcs. <i>Replaces 1496034 (4x)</i>		1
13	1579299	Hex socket head shoulder bolts, Black, Ø6x40 / M5, kit 4 Pcs. <i>Replaces 107.1023 (4x)</i>		1
14	1579300	Hex socket head shoulder bolts, Black, Ø6x45 / M5, kit 4 Pcs. <i>Replaces 1496030 (4x)</i>		1
15	1579301	Hex socket head shoulder bolts, Black, Ø8x25 / M6, kit 4 Pcs. <i>Replaces ZM008252A (4x)</i>		1
16	1579284	Hex socket set screws with flat point, Black, M3x3, kit 25 Pcs. <i>Replaces 116.1001 (25x)</i>		1
17	1579285	Hex socket set screws with flat point, Black, M4x12, kit 25 Pcs. <i>Replaces 116.1046 (25x)</i>		1
18	1579286	Hex socket set screws with flat point, zinc plated blue, M5x5, kit 25 Pcs. <i>Replaces 116.1081 (25x)</i>		1

**Hex-head screws, diverse, ...**

Pos.	Part number	Description	Valid from → until	Qty
19	1579287	Hex socket set screws with flat point, Black, M6x5, kit 25 Pcs. <i>Replaces 1437272 (25x)</i>		1
20	1579288	Hex socket set screws with flat point, zinc plated blue, M6x18, kit 25 Pcs. <i>Replaces 1559106 (25x)</i>		1
21	1579289	Hex socket set screws with flat point, Black, M6x35, kit 25 Pcs. <i>Replaces 116.1133 (25x)</i>		1
22	1579290	Hex socket set screws with flat point, Black, TufLok, M4x4, kit 10 Pcs. <i>Replaces 1493792 (10x)</i>		1
23	1579291	Hex socket set screws with flat point, Z.Plat.Bl., TufLok, M6x12, kit 10Pcs. <i>Replaces 1439057 (10x)</i>		1
24	1579292	Hex socket set screws with cone point, Black, M6x5, kit 25 Pcs. <i>Replaces 116.2121 (25x)</i>		1
25	1579293	Hex socket set screws with cone point, Black, M6x8, kit 25 Pcs. <i>Replaces 116.2123 (25x)</i>		1
26	1579294	Screws with cup point, Torx, zinc plated blue, M6x4, kit 25 Pcs. <i>Replaces 1567763 (25x)</i>		1

# Washers

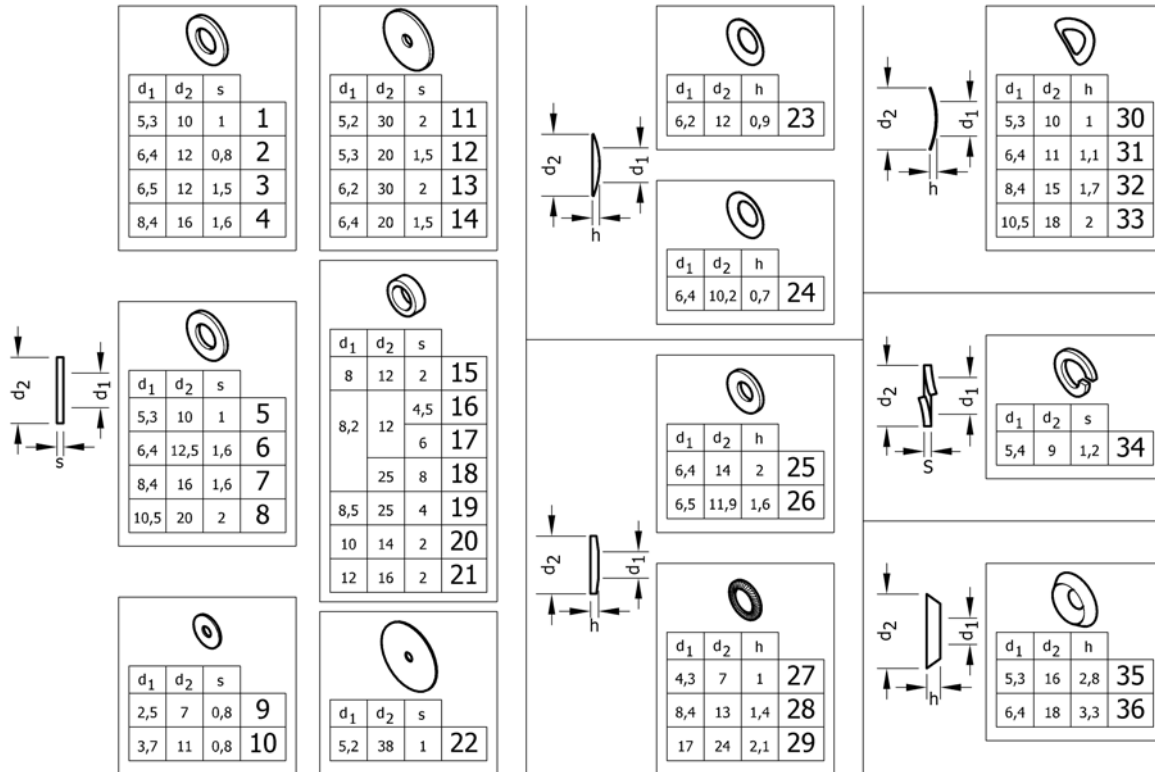


Pos.	Part number	Description	Valid from → until	Qty
1	1579225	Flat washers without chamfer, black, 5,3x10x1, kit 50 Pcs. <i>Replaces 150.1004 (50x)</i>		1
2	1579226	Flat washers without chamfer, black, 6,4x12x0,8, kit 50 Pcs. <i>Replaces 150.2001 (50x)</i>		1
3	1579227	Flat washers without chamfer, zinc plated blue, 6,5x12x1,5, kit 50 Pcs. <i>Replaces 5729091 (50x)</i>		1
4	1579228	Flat washers without chamfer, black, 8,4x16x1,6, kit 50 Pcs. <i>Replaces 150.1012 (50x)</i>		1
5	1579229	Flat washers without chamfer, polyamide, 5,3x10x1, kit 50 Pcs. <i>Replaces 150.3003 (50x)</i>		1
6	1579230	Flat washers without chamfer, polyamide, 6,4x12,5x1,6, kit 50 Pcs. <i>Replaces 150.3004 (50x)</i>		1
7	1579231	Flat washers without chamfer, polyamide, 8,4x16x1,6, kit 50 Pcs. <i>Replaces 150.3005 (50x)</i>		1
8	1579232	Flat washers without chamfer, polyamide, 10,5x20x2, kit 50 Pcs. <i>Replaces 150.3006 (50x)</i>		1
9	1579233	Flat washers without chamfer, zinc plated blue, 2,5x7x0,8, kit 50 Pcs. <i>Replaces 150.2012 (50x)</i>		1
10	1579234	Flat washers without chamfer, zinc plated blue, 3,7x11x0,8, kit 50 Pcs. <i>Replaces ZM203501A (50x)</i>		1
11	1579235	Flat washers without chamfer, black, 5,2x30x2, kit 20 Pcs. <i>Replaces 150.2006 (20x)</i>		1
12	1579236	Flat washers without chamfer, black, 5,3x20x1,5, kit 50 Pcs. <i>Replaces 150.2010 (50x)</i>		1
13	1579237	Flat washers without chamfer, black, 6,2x30x2, kit 20 Pcs. <i>Replaces 150.2009 (20x)</i>		1
14	1579238	Flat washers without chamfer, black, 6,4x25x1,5, kit 50 Pcs. <i>Replaces 1451753 (50x)</i>		1
15	1579239	Hinge rings, black, 8x12x2, kit 50 Pcs. <i>Replaces 150.5002 (50x)</i>		1
16	1579240	Distance sleeve, alu, 8,2x12x4,5, kit 20 Pcs. <i>Replaces 1420970 (20x)</i>		1
17	1579241	Distance sleeve, alu, 8,2x12x6, kit 20 Pcs. <i>Replaces 1420969 (20x)</i>		1
18	1581292	Distance sleeve, alu, 8,2x25x8, kit 2 Pcs. <i>Replaces 151.1002 (2x)</i>		1



## Washers

Pos.	Part number	Description	Valid from → until	Qty
19	1583052	Distance sleeve, alu, black, 8,5x25x4, kit 10 Pcs. <i>Replaces 151.1014 (10x)</i>		1
20	1579242	Hinge rings, zinc plated blue, 10x14x2, kit 50 Pcs. <i>Replaces 1506666 (50x)</i>		1
21	1579243	Hinge rings, black, 12x16x2, kit 50 Pcs. <i>Replaces 150.5006 (50x)</i>		1
22	1579244	Rubber washers, 5,2x38x1, kit 20 Pcs. <i>Replaces 1424891 (20x)</i>		1
23	1579245	Disc springs, black, 6,2x12x0,9, kit 50 Pcs. <i>Replaces 160.1015 (50x)</i>		1
24	1579246	Conical spring washers, zinc plated blue, 6,4x10,2x0,7, kit 50 Pcs. <i>Replaces 161.1004 (50x)</i>		1
25	1579247	Conical spring washers, black, 6,4x14x2, kit 50 Pcs. <i>Replaces 161.2004 (50x)</i>		1
26	1579248	Conical spring washers, zinc plated blue, 6,5x11,9x1,6, kit 50 Pcs. <i>Replaces ZM206001A (50x)</i>		1
27	1579249	Ribbed lock washers, black, 4,3x7x1, kit 50 Pcs. <i>Replaces 155.2002 (50x)</i>		1
28	1579250	Ribbed lock washers, zinc plated blue, 8,4x13x1,4, kit 50 Pcs. <i>Replaces 155.2005 (50x)</i>		1
29	1579251	Ribbed lock washers, zinc plated blue, 17x24x2,1, kit 50 Pcs. <i>Replaces 155.2009 (50x)</i>		1
30	1579252	Curved spring washers, black, 5,3x10x1, kit 50 Pcs. <i>Replaces 1438515 (50x)</i>		1
31	1579253	Curved spring washers, black, 6,4x11x1,1, kit 50 Pcs. <i>Replaces 1438516 (50x)</i>		1
32	1579254	Curved spring washers, zinc plated blue, 8,4x15x1,7, kit 50 Pcs. <i>Replaces ZM208003A (50x)</i>		1
33	1579255	Curved spring washers, zinc plated blue, 10,5x18x2, kit 50 Pcs. <i>Replaces ZM201581A (50x)</i>		1
34	1579256	Split spring lock washers, black, 5,4x9x1,2, kit 50 Pcs. <i>Replaces 155.1003 (50x)</i>		1
35	1579257	Finishing washers 90° countersunk head screw, black, 5,3x16x2,8, kit 50 Pcs. <i>Replaces 150.6003 (50x)</i>		1
36	1579258	Finishing washers 90° countersunk head screw, black, 6,4x18x3,3, kit 50 Pcs. <i>Replaces 150.6004 (50x)</i>		1

# Nuts

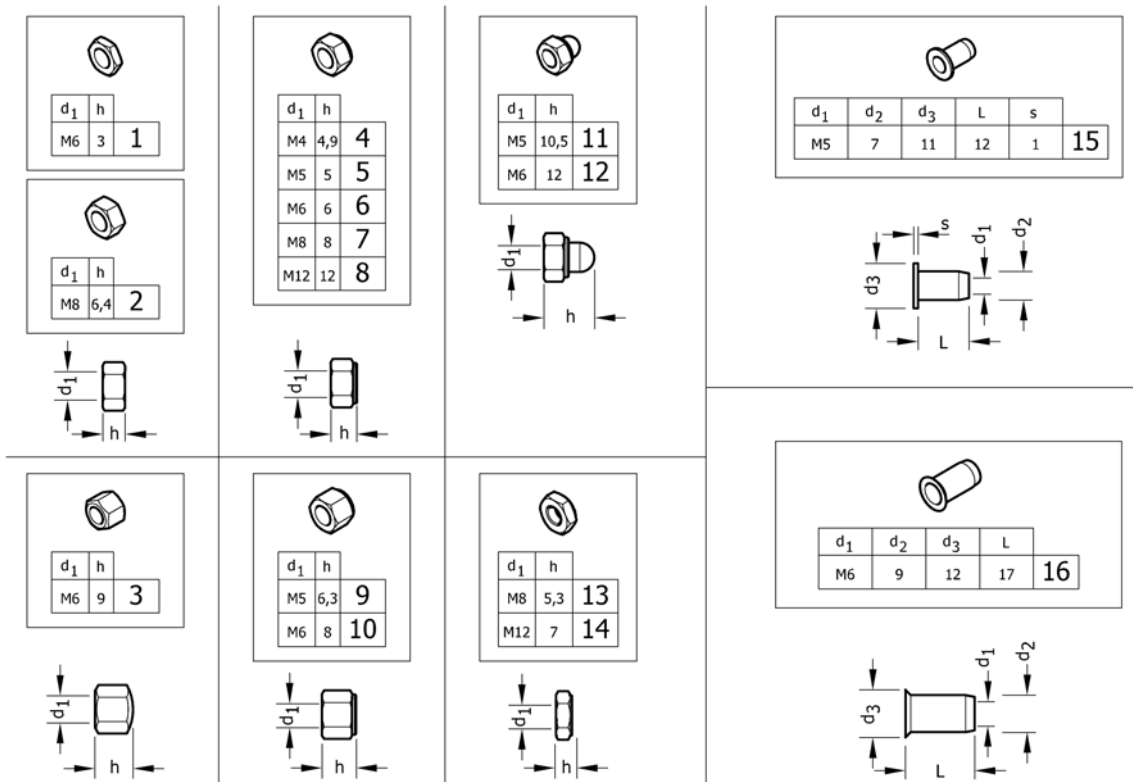


Pos.	Part number	Description	Valid from → until	Qty
1	1579207	Hex jam nuts, black, M6, kit 25 Pcs. <i>Replaces 108.3004 (25x)</i>		1
2	1579208	Hexagon nuts , black, M8, kit 25 Pcs. <i>Replaces 1420997 (25x)</i>		1
3	1579209	Hex cap nuts, low type, black, M6, kit 25 Pcs. <i>Replaces 109.5009 (25x)</i>		1
4	1579210	Prev.Torque lock nuts, thin type, with polyam.Insert, black, M4, kit 25 Pcs. <i>Replaces 109.1002 (25x)</i>		1
5	1579211	Prev.Torque lock nuts, thin type, with polyam.Insert, black, M5, kit 25 Pcs. <i>Replaces 109.1003 (25x)</i>		1
5	1579212	Prev.Torque lock nuts, thin type, with polya.Insert, zi.Bl., M5, kit 25 Pcs. <i>Replaces 109.5008 (25x)</i>		1
6	1579213	Prev.Torque lock nuts, thin type, with polyam.Insert, black, M6, kit 25 Pcs. <i>Replaces 109.1004 (25x)</i>		1
6	1579214	Prev.Torque lock nuts, thin type, with polya.Insert, zi.Bl., M6, kit 25 Pcs. <i>Replaces ZM106001A (25x)</i>		1
7	1579215	Prev.Torque lock nuts, thin type, with polyam.Insert, black, M8, kit 25 Pcs. <i>Replaces 109.1005 (25x)</i>		1
8	1579216	Prev.Torque lock nuts, thin type, with polya.Insert, black, M12, kit 4 Pcs. <i>Replaces 109.1007 (4x)</i>		1
9	1579217	Preva.Torq.Hex lock nuts, high type with poly.Insert, black, M5, kit 25 Pcs. <i>Replaces 1493771 (25x)</i>		1
10	1579218	Preva.Torq.Hex lock nuts, high type with poly.Insert, black, M6, kit 25 Pcs. <i>Replaces 109.5001 (25x)</i>		1
11	1579219	Prevail.Torque hex domed cap nuts with polyam.Insert, black, M5, kit 25 Pcs. <i>Replaces 109.5006 (25x)</i>		1
12	1579220	Prevail.Torque hex domed cap nuts with polyam.Insert, black, M6, kit 25 Pcs. <i>Replaces 109.5003 (25x)</i>		1
13	1579221	Prevailing torque hex lock nuts, all-metal, thin typeblack, M8, kit 10 Pcs. <i>Replaces 109.5005 (10x)</i>		1
14	1579222	Prevailing torque hex lock nuts, all-metal, thin typeblack, M12, kit 4 Pcs. <i>Replaces 109.5004 (4x)</i>		1
15	1579223	Blind rivet nuts, flat head, open type, black, M5, kit 10 Pcs. <i>Replaces 180.4007 (10x)</i>		1

## Nuts

Pos.	Part number	Description	Valid from → until	Qty
16	1579224	Blind rivet nut with low countersunk head, open type, black, M6, kit 10 Pcs. <i>Replaces 180.4105 (10x)</i>		1

## Pins, rivets, ...

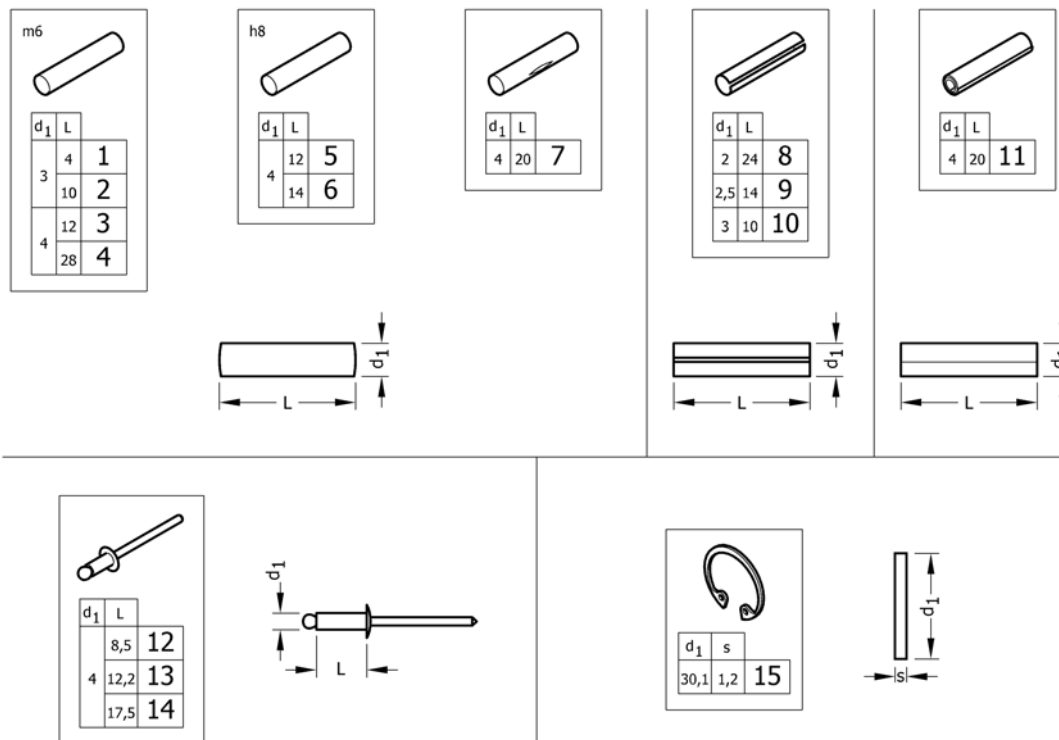

Pos.	Part number	Description	Valid from → until	Qty
1	1579302	Parallel pins, Ø3x4 m6, kit 10 Pcs. <i>Replaces 1439336 (10x)</i>		1
2	1579303	Parallel pins, Ø3x10 m6, kit 10 Pcs. <i>Replaces 1439337 (10x)</i>		1
3	1579304	Parallel pins, Ø4x12 m6, kit 20 Pcs. <i>Replaces 200.1124 (20x)</i>		1
4	1579305	Parallel pins, Ø4x28 m6, kit 20 Pcs. <i>Replaces 1437344 (20x)</i>		1
5	1579306	Parallel pins, Ø4x12 h8, kit 20 Pcs. <i>Replaces 1487663 (20x)</i>		1
6	1579307	Parallel pins, Ø4x14 h8, kit 20 Pcs. <i>Replaces 1487662 (20x)</i>		1
7	1579308	Grooved pins, center grooved, Ø4x20, kit 20 Pcs. <i>Replaces 1517718 (20x)</i>		1
8	1579309	Slotted spring pins, heavy duty, Black, Ø2x24, kit 20 Pcs. <i>Replaces 200.4011 (20x)</i>		1
9	1579310	Slotted spring pins, heavy duty, Black, Ø2,5x14, kit 20 Pcs. <i>Replaces 200.4007 (20x)</i>		1
10	1579311	Slotted spring pins, heavy duty, stainless steel, Ø3x10, kit 20 Pcs. <i>Replaces 1567997 (20x)</i>		1
11	1579312	Spiral spring pins, Black, Ø4x20, kit 20 Pcs. <i>Replaces 200.3167 (20x)</i>		1
12	1579313	Dome head blind rivets, Ø4x8,5, kit 20 Pcs. <i>Replaces 1439292 (20x)</i>		1
13	1579314	Dome head blind rivets, Ø4x12,2, kit 20 Pcs. <i>Replaces 180.5116 (20x)</i>		1
14	1579315	Dome head blind rivets, Ø4x17,5, kit 20 Pcs. <i>Replaces 180.5019 (20x)</i>		1
15	1579316	Retaining rings for bores, Std.Design, Z.plat.Blue, Ø30,1X1,2, kit 20 Pcs. <i>Replaces ZM50J281A (20x)</i>		1



Yes, you can.®



---

**Belgium & Luxemburg:**

Invacare nv, Autobaan 22, B-8210 Loppem  
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11  
belgium@invacare.com / www.invacare.be

**Denmark:**

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby  
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01  
denmark@invacare.com / www.invacare.dk

**Deutschland:**

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny  
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66  
info@invacare-aquatec.com / www.invacare-aquatec.de

**European Distributor Organisation:**

Invacare, Kleiststraße 49, D-32457 Porta Westfalica  
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541  
eda@invacare.com / www.invacare.eu.com

**España:**

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)  
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20  
contactsp@invacare.com / www.invacare.es

**France:**

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes  
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24  
contactfr@invacare.com / www.invacare.fr

**Ireland:**

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland  
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085  
ireland@invacare.com / www.invacare.ie

**Italia:**

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)  
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34  
italia@invacare.com / www.invacare.it

**Nederland:**

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede  
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758  
nederland@invacare.com, csede@invacare.com / www.invacare.nl

**Norge:**

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo  
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01  
norway@invacare.com, island@invacare.com / www.invacare.no

**Österreich:**

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee  
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4  
info-austria@invacare.com / www.invacare.at

**Portugal:**

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio  
Tel: (351) (0)225 1059 46147, Fax: (351) (0)225 1057 39  
portugal@invacare.com / www.invacare.pt

**Sverige & Suomi:**

Invacare AB, Fagerstagatan 9, S-163 91 Spånåsa  
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08  
sweden@invacare.com, finland@invacare.com / www.invacare.se

**Switzerland:**

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil  
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81  
switzerland@invacare.com / www.invacare.ch

**United Kingdom:**

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ  
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201  
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220  
uk@invacare.com / www.invacare.co.uk