

Spare Parts Catalogue

Version: 2020/06/29a EN



Küschall® K-SERIES



Yes, you can.®

General Information

For any order of spare parts the following information is required:

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.

The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.



Yes, you can.®

TABLE OF CONTENTS

INFORMATION

Abbreviations	7
---------------------	---

CHASSIS

Frame Aluminium 75° standard	8
Configuration table for frame Aluminium 75° standard	15
Frame Aluminium 75° adducted	16
Configuration table for frame Aluminium 75° adducted	19
Frame Aluminium 90° standard	20
Configuration table for frame Aluminium 90° standard	27
Frame Aluminium 90° adducted	28
Configuration table for frame Aluminium 90° adducted	31
Axle tube Aluminium	32
Axle tube Carbon	34
Vario-Ax	36
Axle holder	37
Rear wheel extension standard	38
Rear wheel extension for Vario-Ax	39

SEAT

Seat module	40
Seatcover standard	42
Seatcover adjustable	45
Seatcover light	47
Seatcover light for seat upholstery bag	48
Seat cushion standard	49

BACKREST

Backrest (rounded bar)	51
Küschall backrest parts for Matrix backs	53
Backrest cover light	54
Bands to adjustable backrest	55
Cover to adjustable backrest	56
Push handles, height adjustable, rear set	58

FOOTRESTS

Footrest standard mounted on frame aluminium short	59
Configuration table for footrest standard mounted on frame aluminium short	62
Footrest standard mounted on frame aluminium Std or +5cm	63
Configuration table for footrest standard mounted on frame aluminium Std. or +5cm	66
Footrest standard for adducted frame	67
Configuration table for footrest standard mounted on adducted frame	70
Footrest angle adjustable	71
Configuration table for footrest angle adjustable	72
Footrest angle adjustable for adducted frame	73
Configuration table for footrest angle adjust. mounted on adducted frame	74
Footrest highmounted	75
Footrest flip-to-side	76



Yes, you can.®

TABLE OF CONTENTS

Configuration table for footrest flip-to-side	77
Calf strap	78

SIDERESTS

Clothes-guards carbon	79
Configuration table clothes-guard carbon	80
Clothes-guards economy	81
Configuration table clothes-guard economy	82
Mudguards carbon	83
Configuration and size table mudguards carbon	84
Mudguards economy	85
Configuration table mudguards Economy	86
Siderest height adjustable	87
Tubular armrest	88

FRONT WHEELS

Front wheel supporter	89
Front wheels	90

REAR WHEELS

Starec rear wheels complete	91
High Performance wheel	92
Spinergy LXX wheel	93
Active wheel	94
Handrim	95
Fixation handrim	96
Tyres	97
Inner tube and Rim protection	98
Spokes guard	99
Quick release axle and bearing	100

BRAKES

Brakes	101
--------------	-----

SAFETY OPTIONS

Seatbelt for transportation	102
Bodypoint posture belt	103
Bodypoint calf strap	104

SEAT OPTIONS

Suspension	105
Seat upholstery bag	106
Seat frame bag	107

PROPULSION OPTIONS

One arm drive	108
---------------------	-----

OPTIONS & ACCESSORIES




Yes, you can.®

TABLE OF CONTENTS


Antitipper	109
Transit wheel	110
Cane holder	111
Chair accessories	112
Fixation: screws	113
Socket head cap screws	114
Countersunk screws	116
Button head screws	117
Hex-head screws, diverse	118
Washers	119
Nuts	120
Pins, rivets	121

Abbreviations

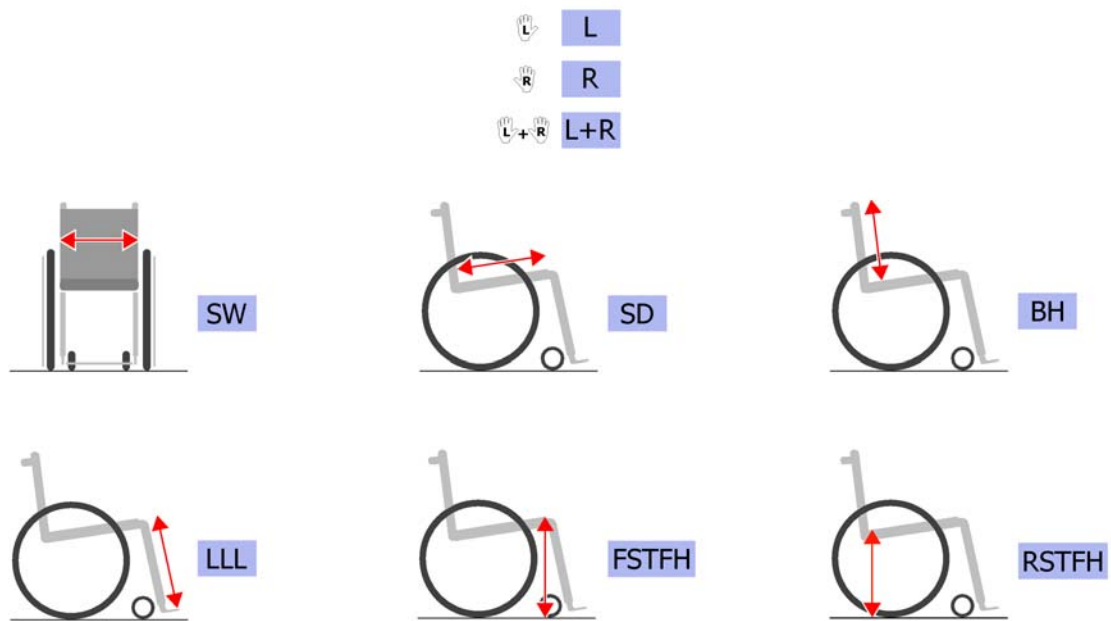

Pos.	Part number	Description	Valid from → until	Qty
L	!SPDABRC1-0012	Left side (sitting in the wheelchair)		0
R	!SPDABRC1-0013	Right side (sitting in the wheelchair)		0
L+R	!SPDABRC1-0014	Left an right sides (paire)		0
SW	!SPDABRC1-0002	Seat width (2 digits in cm or 3 digits in mm)		0
SD	!SPDABRC1-0003	Seat depth (2 digits in cm or 3 digits in mm)		0
BH	!SPDABRC1-0005	Backrest height (2 digits in cm or 3 digits in mm)		0
LLL	!SPDABRC1-0011	Knee-to-heel length (2 digits in cm or 3 digits in mm)		0
FSTF	!SPDABRC1-0007	Front seat-to-floor height (2 digits in cm or 3 digits in mm)		0
H				
RSTF	!SPDABRC1-0008	Rear seat-to-floor height (2 digits in cm or 3 digits in mm)		0
H				

Frame Aluminium 75° standard

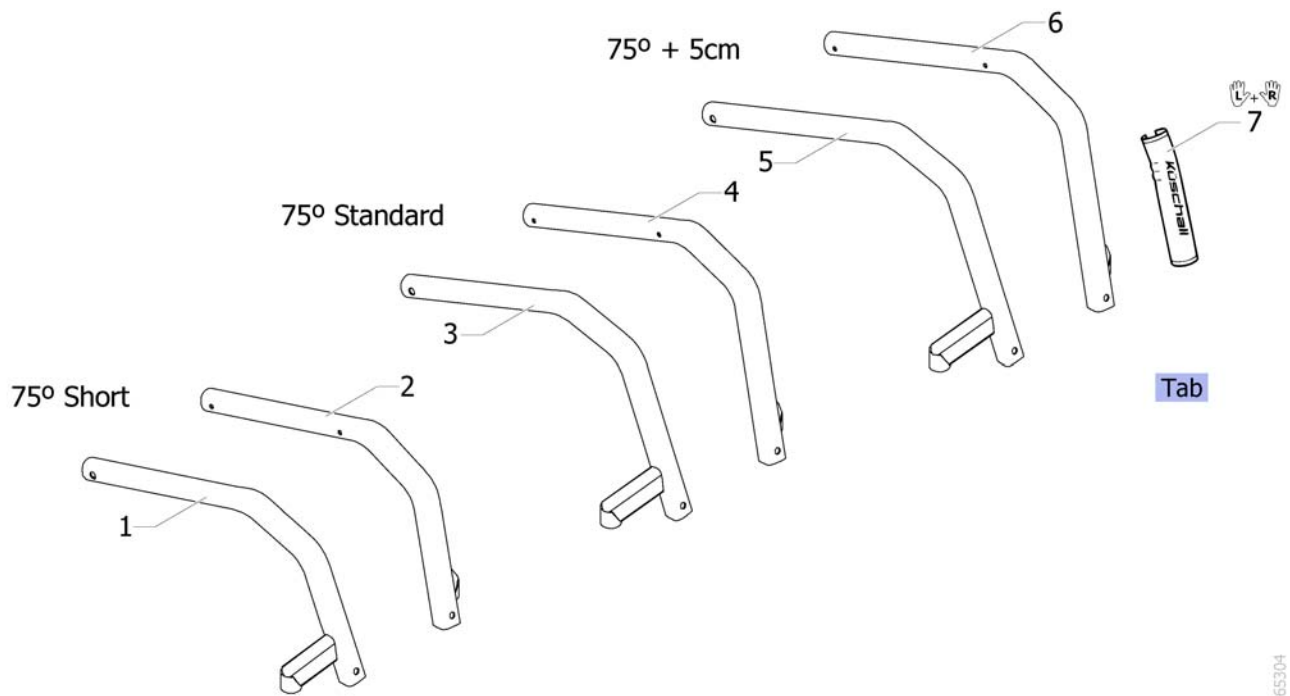

Pos.	Part number	Description	Valid from → until	Qty
Tab	-	Configuration table for frame Aluminium 75° standard		0
1	SP1654476-0464	Front frame 75° short, right, Space Black with decal Bold blue		1
1	SP1654476-0465	Front frame 75° short, right, Anthracite Grey with decal Bold blue		1
1	SP1654476-0466	Front frame 75° short, right, Pure White with decal Bold blue		1
1	SP1654476-0467	Front frame 75° short, right, Carmine Red with decal Bold blue		1
1	SP1654476-0468	Front frame 75° short, right, Azure blue with decal Bold blue		1
1	SP1654477-0464	Front frame 75° short, right, Space Black with decal Bold dark grey		1
1	SP1654477-0465	Front frame 75° short, right, Anthracite Grey with decal Bold dark grey		1
1	SP1654477-0466	Front frame 75° short, right, Pure White with decal Bold dark grey		1
1	SP1654477-0467	Front frame 75° short, right, Carmine Red with decal Bold dark grey		1
1	SP1654477-0468	Front frame 75° short, right, Azure blue with decal Bold dark grey		1
1	SP1654478-0464	Front frame 75° short, right, Space Black with decal Bold gold		1
1	SP1654478-0465	Front frame 75° short, right, Anthracite Grey with decal Bold gold		1
1	SP1654478-0466	Front frame 75° short, right, Pure White with decal Bold gold		1
1	SP1654478-0467	Front frame 75° short, right, Carmine Red with decal Bold gold		1
1	SP1654478-0468	Front frame 75° short, right, Azure blue with decal Bold gold		1
1	SP1654479-0464	Front frame 75° short, right, Space Black with decal Bold light grey		1
1	SP1654479-0465	Front frame 75° short, right, Anthracite Grey with decal Bold light grey		1
1	SP1654479-0466	Front frame 75° short, right, Pure White with decal Bold light grey		1
1	SP1654479-0467	Front frame 75° short, right, Carmine Red with decal Bold light grey		1
1	SP1654479-0468	Front frame 75° short, right, Azure blue with decal Bold light grey		1
1	SP1654480-0464	Front frame 75° short, right, Space Black with decal BOLD red		1
1	SP1654480-0465	Front frame 75° short, right, Anthracite Grey with decal BOLD red		1
1	SP1654480-0466	Front frame 75° short, right, Pure White with decal BOLD red		1
1	SP1654480-0467	Front frame 75° short, right, Carmine Red with decal BOLD red		1
1	SP1654480-0468	Front frame 75° short, right, Azure blue with decal BOLD red		1
1	SP1654481-0464	Front frame 75° short, right, Space Black with decal Bold black		1
1	SP1654481-0465	Front frame 75° short, right, Anthracite Grey with decal Bold black		1
1	SP1654481-0466	Front frame 75° short, right, Pure White with decal Bold black		1
1	SP1654481-0467	Front frame 75° short, right, Carmine Red with decal Bold black		1
1	SP1654481-0468	Front frame 75° short, right, Azure blue with decal Bold black		1

Frame Aluminium 75° standard

Pos.	Part number	Description	Valid from → until	Qty
1	SP1654482-0185	Front frame 75° short, right, dynamit orange without decal for the option TAG		1
1	SP1654482-0419	Front frame 75° short, right, Silver Grey without decal for the option TAG		1
1	SP1654482-0450	Front frame 75° short, right, Smokey Grey without decal for the option TAG		1
1	SP1654482-0464	Front frame 75° short, right, Space Black without decal for the option TAG		1
1	SP1654482-0465	Front frame 75° short, right, Anthracite Grey without decal for the option TAG		1
1	SP1654482-0466	Front frame 75° short, right, Pure White without decal for the option TAG		1
1	SP1654482-0467	Front frame 75° short, right, Carmine Red without decal for the option TAG		1
1	SP1654482-0468	Front frame 75° short, right, Azure blue without decal for the option TAG		1
1	SP1654482-0469	Front frame 75° short, right, Ruby Red without decal for the option TAG		1
1	SP1654482-0470	Front frame 75° short, right, ferrite without decal for the option TAG		1
1	SP1654482-0471	Front frame 75° short, right, Slate Metallic without decal for the option TAG		1
1	SP1654482-0472	Front frame 75° short, right, Forest Green without decal for the option TAG		1
1	SP1654482-5969	Front frame 75° short, right, Royal blue without decal for the option TAG		1
1	SP1654482-9005	Front frame 75° short, right, Glossy Black without decal for the option TAG		1
2	SP1654483-0464	Front frame 75° short, left, Space Black with decal Bold blue		1
2	SP1654483-0465	Front frame 75° short, left, Anthracite Grey with decal Bold blue		1
2	SP1654483-0466	Front frame 75° short, left, Pure White with decal Bold blue		1
2	SP1654483-0467	Front frame 75° short, left, Carmine Red with decal Bold blue		1
2	SP1654483-0468	Front frame 75° short, left, Azure blue with decal Bold blue		1
2	SP1654484-0464	Front frame 75° short, left, Space Black with decal Bold dark grey		1
2	SP1654484-0465	Front frame 75° short, left, Anthracite Grey with decal Bold dark grey		1
2	SP1654484-0466	Front frame 75° short, left, Pure White with decal Bold dark grey		1
2	SP1654484-0467	Front frame 75° short, left, Carmine Red with decal Bold dark grey		1
2	SP1654484-0468	Front frame 75° short, left, Azure blue with decal Bold dark grey		1
2	SP1654485-0464	Front frame 75° short, left, Space Black with decal Bold gold		1
2	SP1654485-0465	Front frame 75° short, left, Anthracite Grey with decal Bold gold		1
2	SP1654485-0466	Front frame 75° short, left, Pure White with decal Bold gold		1
2	SP1654485-0467	Front frame 75° short, left, Carmine Red with decal Bold gold		1
2	SP1654485-0468	Front frame 75° short, left, Azure blue with decal Bold gold		1
2	SP1654486-0464	Front frame 75° short, left, Space Black with decal Bold light grey		1
2	SP1654486-0465	Front frame 75° short, left, Anthracite Grey with decal Bold light grey		1
2	SP1654486-0466	Front frame 75° short, left, Pure White with decal Bold light grey		1
2	SP1654486-0467	Front frame 75° short, left, Carmine Red with decal Bold light grey		1
2	SP1654486-0468	Front frame 75° short, left, Azure blue with decal Bold light grey		1
2	SP1654487-0464	Front frame 75° short, left, Space Black with decal BOLD red		1
2	SP1654487-0465	Front frame 75° short, left, Anthracite Grey with decal BOLD red		1
2	SP1654487-0466	Front frame 75° short, left, Pure White with decal BOLD red		1
2	SP1654487-0467	Front frame 75° short, left, Carmine Red with decal BOLD red		1
2	SP1654487-0468	Front frame 75° short, left, Azure blue with decal BOLD red		1
2	SP1654488-0464	Front frame 75° short, left, Space Black with decal Bold black		1
2	SP1654488-0465	Front frame 75° short, left, Anthracite Grey with decal Bold black		1
2	SP1654488-0466	Front frame 75° short, left, Pure White with decal Bold black		1
2	SP1654488-0467	Front frame 75° short, left, Carmine Red with decal Bold black		1
2	SP1654488-0468	Front frame 75° short, left, Azure blue with decal Bold black		1
2	SP1654489-0185	Front frame 75° short, left, dynamit orange with decal TAG red		1
2	SP1654489-0419	Front frame 75° short, left, Silver Grey with decal TAG red		1
2	SP1654489-0450	Front frame 75° short, left, Smokey Grey with decal TAG red		1
2	SP1654489-0464	Front frame 75° short, left, Space Black with decal TAG red		1
2	SP1654489-0465	Front frame 75° short, left, Anthracite Grey with decal TAG red		1
2	SP1654489-0466	Front frame 75° short, left, Pure White with decal TAG red		1
2	SP1654489-0467	Front frame 75° short, left, Carmine Red with decal TAG red		1
2	SP1654489-0468	Front frame 75° short, left, Azure blue with decal TAG red		1

Frame Aluminium 75° standard

Pos.	Part number	Description	Valid from → until	Qty
2	SP1654489-0469	Front frame 75° short, left, Ruby Red with decal TAG red		1
2	SP1654489-0470	Front frame 75° short, left, ferrite with decal TAG red		1
2	SP1654489-0471	Front frame 75° short, left, Slate Metallic with decal TAG red		1
2	SP1654489-0472	Front frame 75° short, left, Forest Green with decal TAG red		1
2	SP1654489-5969	Front frame 75° short, left, Royal blue with decal TAG red		1
2	SP1654489-9005	Front frame 75° short, left, Glossy Black with decal TAG red		1
2	SP1654490-0185	Front frame 75° short, left, dynamit orange with decal TAG black		1
2	SP1654490-0419	Front frame 75° short, left, Silver Grey with decal TAG black		1
2	SP1654490-0450	Front frame 75° short, left, Smokey Grey with decal TAG black		1
2	SP1654490-0464	Front frame 75° short, left, Space Black with decal TAG black		1
2	SP1654490-0465	Front frame 75° short, left, Anthracite Grey with decal TAG black		1
2	SP1654490-0466	Front frame 75° short, left, Pure White with decal TAG black		1
2	SP1654490-0467	Front frame 75° short, left, Carmine Red with decal TAG black		1
2	SP1654490-0468	Front frame 75° short, left, Azure blue with decal TAG black		1
2	SP1654490-0469	Front frame 75° short, left, Ruby Red with decal TAG black		1
2	SP1654490-0470	Front frame 75° short, left, ferrite with decal TAG black		1
2	SP1654490-0471	Front frame 75° short, left, Slate Metallic with decal TAG black		1
2	SP1654490-0472	Front frame 75° short, left, Forest Green with decal TAG black		1
2	SP1654490-5969	Front frame 75° short, left, Royal blue with decal TAG black		1
2	SP1654490-9005	Front frame 75° short, left, Glossy Black with decal TAG black		1
3	SP1654491-0464	Front frame 75° standard, right, Space Black with decal Bold blue		1
3	SP1654491-0465	Front frame 75° standard, right, Anthracite Grey with decal Bold blue		1
3	SP1654491-0466	Front frame 75° standard, right, Pure White with decal Bold blue		1
3	SP1654491-0467	Front frame 75° standard, right, Carmine Red with decal Bold blue		1
3	SP1654491-0468	Front frame 75° standard, right, Azure blue with decal Bold blue		1
3	SP1654492-0464	Front frame 75° standard, right, Space Black with decal Bold dark grey		1
3	SP1654492-0465	Front frame 75° standard, right, Anthracite Grey with decal Bold dark grey		1
3	SP1654492-0466	Front frame 75° standard, right, Pure White with decal Bold dark grey		1
3	SP1654492-0467	Front frame 75° standard, right, Carmine Red with decal Bold dark grey		1
3	SP1654492-0468	Front frame 75° standard, right, Azure blue with decal Bold dark grey		1
3	SP1654493-0464	Front frame 75° standard, right, Space Black with decal Bold gold		1
3	SP1654493-0465	Front frame 75° standard, right, Anthracite Grey with decal Bold gold		1
3	SP1654493-0466	Front frame 75° standard, right, Pure White with decal Bold gold		1
3	SP1654493-0467	Front frame 75° standard, right, Carmine Red with decal Bold gold		1
3	SP1654493-0468	Front frame 75° standard, right, Azure blue with decal Bold gold		1
3	SP1654494-0464	Front frame 75° standard, right, Space Black with decal Bold light grey		1
3	SP1654494-0465	Front frame 75° standard, right, Anthracite Grey with decal Bold light grey		1
3	SP1654494-0466	Front frame 75° standard, right, Pure White with decal Bold light grey		1
3	SP1654494-0467	Front frame 75° standard, right, Carmine Red with decal Bold light grey		1
3	SP1654494-0468	Front frame 75° standard, right, Azure blue with decal Bold light grey		1
3	SP1654495-0464	Front frame 75° standard, right, Space Black with decal BOLD red		1
3	SP1654495-0465	Front frame 75° standard, right, Anthracite Grey with decal BOLD red		1
3	SP1654495-0466	Front frame 75° standard, right, Pure White with decal BOLD red		1
3	SP1654495-0467	Front frame 75° standard, right, Carmine Red with decal BOLD red		1
3	SP1654495-0468	Front frame 75° standard, right, Azure blue with decal BOLD red		1
3	SP1654496-0464	Front frame 75° standard, right, Space Black with decal Bold black		1
3	SP1654496-0465	Front frame 75° standard, right, Anthracite Grey with decal Bold black		1
3	SP1654496-0466	Front frame 75° standard, right, Pure White with decal Bold black		1
3	SP1654496-0467	Front frame 75° standard, right, Carmine Red with decal Bold black		1
3	SP1654496-0468	Front frame 75° standard, right, Azure blue with decal Bold black		1
3	SP1654497-0185	Front frame 75° standard, right, dynamit orange without decal for the option TAG		1
3	SP1654497-0419	Front frame 75° standard, right, Silver Grey without decal for the option TAG		1
3	SP1654497-0450	Front frame 75° standard, right, Smokey Grey without decal for the option TAG		1

Frame Aluminium 75° standard

Pos.	Part number	Description	Valid from → until	Qty
3	SP1654497-0464	Front frame 75° standard, right, Space Black without decal for the option TAG		1
3	SP1654497-0465	Front frame 75° standard, right, Anthracite Grey without decal for the option TAG		1
3	SP1654497-0466	Front frame 75° standard, right, Pure White without decal for the option TAG		1
3	SP1654497-0467	Front frame 75° standard, right, Carmine Red without decal for the option TAG		1
3	SP1654497-0468	Front frame 75° standard, right, Azure blue without decal for the option TAG		1
3	SP1654497-0469	Front frame 75° standard, right, Ruby Red without decal for the option TAG		1
3	SP1654497-0470	Front frame 75° standard, right, ferrite without decal for the option TAG		1
3	SP1654497-0471	Front frame 75° standard, right, Slate Metallic without decal for the option TAG		1
3	SP1654497-0472	Front frame 75° standard, right, Forest Green without decal for the option TAG		1
3	SP1654497-5969	Front frame 75° standard, right, Royal blue without decal for the option TAG		1
3	SP1654497-9005	Front frame 75° standard, right, Glossy Black without decal for the option TAG		1
4	SP1654498-0464	Front frame 75° standard, left, Space Black with decal Bold blue		1
4	SP1654498-0465	Front frame 75° standard, left, Anthracite Grey with decal Bold blue		1
4	SP1654498-0466	Front frame 75° standard, left, Pure White with decal Bold blue		1
4	SP1654498-0467	Front frame 75° standard, left, Carmine Red with decal Bold blue		1
4	SP1654498-0468	Front frame 75° standard, left, Azure blue with decal Bold blue		1
4	SP1654499-0464	Front frame 75° standard, left, Space Black with decal Bold dark grey		1
4	SP1654499-0465	Front frame 75° standard, left, Anthracite Grey with decal Bold dark grey		1
4	SP1654499-0466	Front frame 75° standard, left, Pure White with decal Bold dark grey		1
4	SP1654499-0467	Front frame 75° standard, left, Carmine Red with decal Bold dark grey		1
4	SP1654499-0468	Front frame 75° standard, left, Azure blue with decal Bold dark grey		1
4	SP1654500-0464	Front frame 75° standard, left, Space Black with decal Bold gold		1
4	SP1654500-0465	Front frame 75° standard, left, Anthracite Grey with decal Bold gold		1
4	SP1654500-0466	Front frame 75° standard, left, Pure White with decal Bold gold		1
4	SP1654500-0467	Front frame 75° standard, left, Carmine Red with decal Bold gold		1
4	SP1654500-0468	Front frame 75° standard, left, Azure blue with decal Bold gold		1
4	SP1654501-0464	Front frame 75° standard, left, Space Black with decal Bold light grey		1
4	SP1654501-0465	Front frame 75° standard, left, Anthracite Grey with decal Bold light grey		1
4	SP1654501-0466	Front frame 75° standard, left, Pure White with decal Bold light grey		1
4	SP1654501-0467	Front frame 75° standard, left, Carmine Red with decal Bold light grey		1
4	SP1654501-0468	Front frame 75° standard, left, Azure blue with decal Bold light grey		1
4	SP1654502-0464	Front frame 75° standard, left, Space Black with decal BOLD red		1
4	SP1654502-0465	Front frame 75° standard, left, Anthracite Grey with decal BOLD red		1
4	SP1654502-0466	Front frame 75° standard, left, Pure White with decal BOLD red		1
4	SP1654502-0467	Front frame 75° standard, left, Carmine Red with decal BOLD red		1
4	SP1654502-0468	Front frame 75° standard, left, Azure blue with decal BOLD red		1
4	SP1654503-0464	Front frame 75° standard, left, Space Black with decal Bold black		1
4	SP1654503-0465	Front frame 75° standard, left, Anthracite Grey with decal Bold black		1
4	SP1654503-0466	Front frame 75° standard, left, Pure White with decal Bold black		1
4	SP1654503-0467	Front frame 75° standard, left, Carmine Red with decal Bold black		1
4	SP1654503-0468	Front frame 75° standard, left, Azure blue with decal Bold black		1
4	SP1654504-0185	Front frame 75° standard, left, dynamit orange with decal TAG red		1
4	SP1654504-0419	Front frame 75° standard, left, Silver Grey with decal TAG red		1
4	SP1654504-0450	Front frame 75° standard, left, Smokey Grey with decal TAG red		1
4	SP1654504-0464	Front frame 75° standard, left, Space Black with decal TAG red		1
4	SP1654504-0465	Front frame 75° standard, left, Anthracite Grey with decal TAG red		1
4	SP1654504-0466	Front frame 75° standard, left, Pure White with decal TAG red		1
4	SP1654504-0467	Front frame 75° standard, left, Carmine Red with decal TAG red		1

Frame Aluminium 75° standard

Pos.	Part number	Description	Valid from → until	Qty
4	SP1654504-0468	Front frame 75° standard, left, Azure blue with decal TAG red		1
4	SP1654504-0469	Front frame 75° standard, left, Ruby Red with decal TAG red		1
4	SP1654504-0470	Front frame 75° standard, left, ferrite with decal TAG red		1
4	SP1654504-0471	Front frame 75° standard, left, Slate Metallic with decal TAG red		1
4	SP1654504-0472	Front frame 75° standard, left, Forest Green with decal TAG red		1
4	SP1654504-5969	Front frame 75° standard, left, Royal blue with decal TAG red		1
4	SP1654504-9005	Front frame 75° standard, left, Glossy Black with decal TAG red		1
4	SP1654505-0185	Front frame 75° standard, left, dynamit orange with decal TAG black		1
4	SP1654505-0419	Front frame 75° standard, left, Silver Grey with decal TAG black		1
4	SP1654505-0450	Front frame 75° standard, left, Smokey Grey with decal TAG black		1
4	SP1654505-0464	Front frame 75° standard, left, Space Black with decal TAG black		1
4	SP1654505-0465	Front frame 75° standard, left, Anthracite Grey with decal TAG black		1
4	SP1654505-0466	Front frame 75° standard, left, Pure White with decal TAG black		1
4	SP1654505-0467	Front frame 75° standard, left, Carmine Red with decal TAG black		1
4	SP1654505-0468	Front frame 75° standard, left, Azure blue with decal TAG black		1
4	SP1654505-0469	Front frame 75° standard, left, Ruby Red with decal TAG black		1
4	SP1654505-0470	Front frame 75° standard, left, ferrite with decal TAG black		1
4	SP1654505-0471	Front frame 75° standard, left, Slate Metallic with decal TAG black		1
4	SP1654505-0472	Front frame 75° standard, left, Forest Green with decal TAG black		1
4	SP1654505-5969	Front frame 75° standard, left, Royal blue with decal TAG black		1
4	SP1654505-9005	Front frame 75° standard, left, Glossy Black with decal TAG black		1
5	SP1654506-0464	Front frame 75° +5cm, right, Space Black with decal Bold blue		1
5	SP1654506-0465	Front frame 75° +5cm, right, Anthracite Grey with decal Bold blue		1
5	SP1654506-0466	Front frame 75° +5cm, right, Pure White with decal Bold blue		1
5	SP1654506-0467	Front frame 75° +5cm, right, Carmine Red with decal Bold blue		1
5	SP1654506-0468	Front frame 75° +5cm, right, Azure blue with decal Bold blue		1
5	SP1654507-0464	Front frame 75° +5cm, right, Space Black with decal Bold dark grey		1
5	SP1654507-0465	Front frame 75° +5cm, right, Anthracite Grey with decal Bold dark grey		1
5	SP1654507-0466	Front frame 75° +5cm, right, Pure White with decal Bold dark grey		1
5	SP1654507-0467	Front frame 75° +5cm, right, Carmine Red with decal Bold dark grey		1
5	SP1654507-0468	Front frame 75° +5cm, right, Azure blue with decal Bold dark grey		1
5	SP1654508-0464	Front frame 75° +5cm, right, Space Black with decal Bold gold		1
5	SP1654508-0465	Front frame 75° +5cm, right, Anthracite Grey with decal Bold gold		1
5	SP1654508-0466	Front frame 75° +5cm, right, Pure White with decal Bold gold		1
5	SP1654508-0467	Front frame 75° +5cm, right, Carmine Red with decal Bold gold		1
5	SP1654508-0468	Front frame 75° +5cm, right, Azure blue with decal Bold gold		1
5	SP1654509-0464	Front frame 75° +5cm, right, Space Black with decal Bold light grey		1
5	SP1654509-0465	Front frame 75° +5cm, right, Anthracite Grey with decal Bold light grey		1
5	SP1654509-0466	Front frame 75° +5cm, right, Pure White with decal Bold light grey		1
5	SP1654509-0467	Front frame 75° +5cm, right, Carmine Red with decal Bold light grey		1
5	SP1654509-0468	Front frame 75° +5cm, right, Azure blue with decal Bold light grey		1
5	SP1654510-0464	Front frame 75° +5cm, right, Space Black with decal BOLD red		1
5	SP1654510-0465	Front frame 75° +5cm, right, Anthracite Grey with decal BOLD red		1
5	SP1654510-0466	Front frame 75° +5cm, right, Pure White with decal BOLD red		1
5	SP1654510-0467	Front frame 75° +5cm, right, Carmine Red with decal BOLD red		1
5	SP1654510-0468	Front frame 75° +5cm, right, Azure blue with decal BOLD red		1
5	SP1654511-0464	Front frame 75° +5cm, right, Space Black with decal Bold black		1
5	SP1654511-0465	Front frame 75° +5cm, right, Anthracite Grey with decal Bold black		1
5	SP1654511-0466	Front frame 75° +5cm, right, Pure White with decal Bold black		1
5	SP1654511-0467	Front frame 75° +5cm, right, Carmine Red with decal Bold black		1
5	SP1654511-0468	Front frame 75° +5cm, right, Azure blue with decal Bold black		1
5	SP1654512-0185	Front frame 75° +5cm, right, dynamit orange without decal for the option TAG		1
5	SP1654512-0419	Front frame 75° +5cm, right, Silver Grey without decal for the option TAG		1
5	SP1654512-0450	Front frame 75° +5cm, right, Smokey Grey without decal for the option TAG		1

Frame Aluminium 75° standard


Pos.	Part number	Description	Valid from → until	Qty
5	SP1654512-0464	Front frame 75° +5cm, right, Space Black without decal for the option TAG		1
5	SP1654512-0465	Front frame 75° +5cm, right, Anthracite Grey without decal for the option TAG		1
5	SP1654512-0466	Front frame 75° +5cm, right, Pure White without decal for the option TAG		1
5	SP1654512-0467	Front frame 75° +5cm, right, Carmine Red without decal for the option TAG		1
5	SP1654512-0468	Front frame 75° +5cm, right, Azure blue without decal for the option TAG		1
5	SP1654512-0469	Front frame 75° +5cm, right, Ruby Red without decal for the option TAG		1
5	SP1654512-0470	Front frame 75° +5cm, right, ferrite without decal for the option TAG		1
5	SP1654512-0471	Front frame 75° +5cm, right, Slate Metallic without decal for the option TAG		1
5	SP1654512-0472	Front frame 75° +5cm, right, Forest Green without decal for the option TAG		1
5	SP1654512-5969	Front frame 75° +5cm, right, Royal blue without decal for the option TAG		1
5	SP1654512-9005	Front frame 75° +5cm, right, Glossy Black without decal for the option TAG		1
6	SP1654513-0464	Front frame 75° +5cm, left, Space Black with decal Bold blue		1
6	SP1654513-0465	Front frame 75° +5cm, left, Anthracite Grey with decal Bold blue		1
6	SP1654513-0466	Front frame 75° +5cm, left, Pure White with decal Bold blue		1
6	SP1654513-0467	Front frame 75° +5cm, left, Carmine Red with decal Bold blue		1
6	SP1654513-0468	Front frame 75° +5cm, left, Azure blue with decal Bold blue		1
6	SP1654514-0464	Front frame 75° +5cm, left, Space Black with decal Bold dark grey		1
6	SP1654514-0465	Front frame 75° +5cm, left, Anthracite Grey with decal Bold dark grey		1
6	SP1654514-0466	Front frame 75° +5cm, left, Pure White with decal Bold dark grey		1
6	SP1654514-0467	Front frame 75° +5cm, left, Carmine Red with decal Bold dark grey		1
6	SP1654514-0468	Front frame 75° +5cm, left, Azure blue with decal Bold dark grey		1
6	SP1654515-0464	Front frame 75° +5cm, left, Space Black with decal Bold gold		1
6	SP1654515-0465	Front frame 75° +5cm, left, Anthracite Grey with decal Bold gold		1
6	SP1654515-0466	Front frame 75° +5cm, left, Pure White with decal Bold gold		1
6	SP1654515-0467	Front frame 75° +5cm, left, Carmine Red with decal Bold gold		1
6	SP1654515-0468	Front frame 75° +5cm, left, Azure blue with decal Bold gold		1
6	SP1654516-0464	Front frame 75° +5cm, left, Space Black with decal Bold light grey		1
6	SP1654516-0465	Front frame 75° +5cm, left, Anthracite Grey with decal Bold light grey		1
6	SP1654516-0466	Front frame 75° +5cm, left, Pure White with decal Bold light grey		1
6	SP1654516-0467	Front frame 75° +5cm, left, Carmine Red with decal Bold light grey		1
6	SP1654516-0468	Front frame 75° +5cm, left, Azure blue with decal Bold light grey		1
6	SP1654517-0464	Front frame 75° +5cm, left, Space Black with decal BOLD red		1
6	SP1654517-0465	Front frame 75° +5cm, left, Anthracite Grey with decal BOLD red		1
6	SP1654517-0466	Front frame 75° +5cm, left, Pure White with decal BOLD red		1
6	SP1654517-0467	Front frame 75° +5cm, left, Carmine Red with decal BOLD red		1
6	SP1654517-0468	Front frame 75° +5cm, left, Azure blue with decal BOLD red		1
6	SP1654518-0464	Front frame 75° +5cm, left, Space Black with decal Bold black		1
6	SP1654518-0465	Front frame 75° +5cm, left, Anthracite Grey with decal Bold black		1
6	SP1654518-0466	Front frame 75° +5cm, left, Pure White with decal Bold black		1
6	SP1654518-0467	Front frame 75° +5cm, left, Carmine Red with decal Bold black		1
6	SP1654518-0468	Front frame 75° +5cm, left, Azure blue with decal Bold black		1
6	SP1654519-0185	Front frame 75° +5cm, left, dynamit orange with decal TAG red		1
6	SP1654519-0419	Front frame 75° +5cm, left, Silver Grey with decal TAG red		1
6	SP1654519-0450	Front frame 75° +5cm, left, Smokey Grey with decal TAG red		1
6	SP1654519-0464	Front frame 75° +5cm, left, Space Black with decal TAG red		1
6	SP1654519-0465	Front frame 75° +5cm, left, Anthracite Grey with decal TAG red		1
6	SP1654519-0466	Front frame 75° +5cm, left, Pure White with decal TAG red		1
6	SP1654519-0467	Front frame 75° +5cm, left, Carmine Red with decal TAG red		1
6	SP1654519-0468	Front frame 75° +5cm, left, Azure blue with decal TAG red		1
6	SP1654519-0469	Front frame 75° +5cm, left, Ruby Red with decal TAG red		1
6	SP1654519-0470	Front frame 75° +5cm, left, ferrite with decal TAG red		1
6	SP1654519-0471	Front frame 75° +5cm, left, Slate Metallic with decal TAG red		1

Frame Aluminium 75° standard

Pos.	Part number	Description	Valid from → until	Qty
6	SP1654519-0472	Front frame 75° +5cm, left, Forest Green with decal TAG red		1
6	SP1654519-5969	Front frame 75° +5cm, left, Royal blue with decal TAG red		1
6	SP1654519-9005	Front frame 75° +5cm, left, Glossy Black with decal TAG red		1
6	SP1654520-0185	Front frame 75° +5cm, left, dynamit orange with decal TAG black		1
6	SP1654520-0419	Front frame 75° +5cm, left, Silver Grey with decal TAG black		1
6	SP1654520-0450	Front frame 75° +5cm, left, Smokey Grey with decal TAG black		1
6	SP1654520-0464	Front frame 75° +5cm, left, Space Black with decal TAG black		1
6	SP1654520-0465	Front frame 75° +5cm, left, Anthracite Grey with decal TAG black		1
6	SP1654520-0466	Front frame 75° +5cm, left, Pure White with decal TAG black		1
6	SP1654520-0467	Front frame 75° +5cm, left, Carmine Red with decal TAG black		1
6	SP1654520-0468	Front frame 75° +5cm, left, Azure blue with decal TAG black		1
6	SP1654520-0469	Front frame 75° +5cm, left, Ruby Red with decal TAG black		1
6	SP1654520-0470	Front frame 75° +5cm, left, ferrite with decal TAG black		1
6	SP1654520-0471	Front frame 75° +5cm, left, Slate Metallic with decal TAG black		1
6	SP1654520-0472	Front frame 75° +5cm, left, Forest Green with decal TAG black		1
6	SP1654520-5969	Front frame 75° +5cm, left, Royal blue with decal TAG black		1
6	SP1654520-9005	Front frame 75° +5cm, left, Glossy Black with decal TAG black		1
7	SP1659570	Front frame protectors, pair		1

Configuration table for frame Aluminium 75° standard


i	Rear Wheel 24"					Rear Wheel 25"					Rear Wheel 26"				
	FSTF	LLL	LLL	LLL	LLL	FSTF	LLL	LLL	LLL	LLL	FSTF	LLL	LLL	LLL	LLL
Frame Aluminium 75° Short (236 mm)	FSTF 45	LLL 29 ->	LLL 32	LLL 33 ->	LLL 42	LLL 29 ->	LLL 32	LLL 33 ->	LLL 43	LLL 29 ->	LLL 32	LLL 33 ->	LLL 44	LLL 29 ->	LLL 44
	FSTF 46	LLL 29 ->	LLL 33	LLL 34 ->	LLL 43	LLL 30 ->	LLL 43	LLL 34 ->	LLL 44	LLL 30 ->	LLL 44	LLL 34 ->	LLL 45	LLL 30 ->	LLL 45
	FSTF 47	LLL 29 ->	LLL 34	LLL 35 ->	LLL 44	LLL 31 ->	LLL 44	LLL 35 ->	LLL 45	LLL 31 ->	LLL 45	LLL 35 ->	LLL 46	LLL 31 ->	LLL 46
Frame Aluminium 75° Std. (283,3 mm)	FSTF 48	LLL 29 ->	LLL 36	LLL 37 ->	LLL 45	LLL 33 ->	LLL 45	LLL 37 ->	LLL 46	LLL 33 ->	LLL 46	LLL 37 ->	LLL 47	LLL 33 ->	LLL 47
	FSTF 49	LLL 29 ->	LLL 37	LLL 38 ->	LLL 46	LLL 34 ->	LLL 46	LLL 38 ->	LLL 47	LLL 34 ->	LLL 47	LLL 38 ->	LLL 48	LLL 34 ->	LLL 48
	FSTF 50	LLL 29 ->	LLL 38	LLL 39 ->	LLL 47	LLL 35 ->	LLL 47	LLL 39 ->	LLL 48	LLL 35 ->	LLL 48	LLL 39 ->	LLL 49	LLL 35 ->	LLL 49
Frame Aluminium 75° +5cm (302 mm)	FSTF 51	LLL 29 ->	LLL 38	LLL 39 ->	LLL 47	LLL 35 ->	LLL 47	LLL 39 ->	LLL 48	LLL 35 ->	LLL 48	LLL 39 ->	LLL 49	LLL 35 ->	LLL 49
	FSTF 52	LLL 29 ->	LLL 39	LLL 40 ->	LLL 48	LLL 36 ->	LLL 48	LLL 40 ->	LLL 49	LLL 36 ->	LLL 49	LLL 40 ->	LLL 50	LLL 36 ->	LLL 50
	FSTF 53	LLL 29 ->	LLL 40	LLL 41 ->	LLL 49	LLL 37 ->	LLL 49	LLL 41 ->	LLL 50	LLL 37 ->	LLL 50	LLL 41 ->	LLL 51	LLL 37 ->	LLL 51

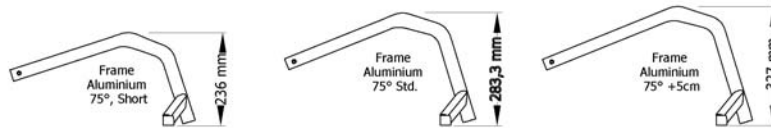

ISPS1665363

Pos.	Part number	Description	Valid from → until	Qty
i	ISPDINFC1-0037	Type of frame in relation to footrest, knee to heel length (UL), front seat height (SHv) and rear wheel size		1

Frame Aluminium 75° adducted

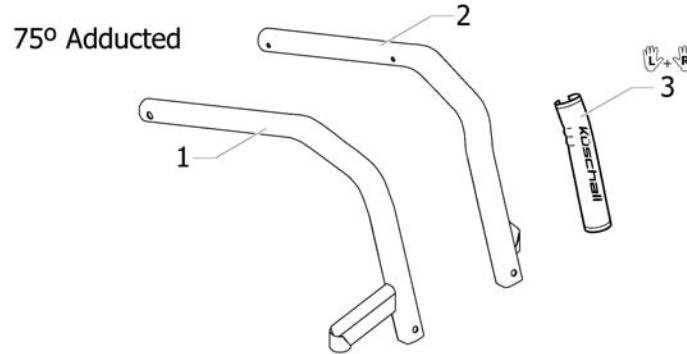

Tab

ISPRS1666305

Pos.	Part number	Description	Valid from → until	Qty
Tab	-	Configuration table for frame Aluminium 75° adducted		0
1	SP1654521-0464	Front frame 75° adducted, right, Space Black with decal Bold blue		1
1	SP1654521-0465	Front frame 75° adducted, right, Anthracite Grey with decal Bold blue		1
1	SP1654521-0466	Front frame 75° adducted, right, Pure White with decal Bold blue		1
1	SP1654521-0467	Front frame 75° adducted, right, Carmine Red with decal Bold blue		1
1	SP1654521-0468	Front frame 75° adducted, right, Azure blue with decal Bold blue		1
1	SP1654522-0464	Front frame 75° adducted, right, Space Black with decal Bold dark grey		1
1	SP1654522-0465	Front frame 75° adducted, right, Anthracite Grey with decal Bold dark grey		1
1	SP1654522-0466	Front frame 75° adducted, right, Pure White with decal Bold dark grey		1
1	SP1654522-0467	Front frame 75° adducted, right, Carmine Red with decal Bold dark grey		1
1	SP1654522-0468	Front frame 75° adducted, right, Azure blue with decal Bold dark grey		1
1	SP1654523-0464	Front frame 75° adducted, right, Space Black with decal Bold gold		1
1	SP1654523-0465	Front frame 75° adducted, right, Anthracite Grey with decal Bold gold		1
1	SP1654523-0466	Front frame 75° adducted, right, Pure White with decal Bold gold		1
1	SP1654523-0467	Front frame 75° adducted, right, Carmine Red with decal Bold gold		1
1	SP1654523-0468	Front frame 75° adducted, right, Azure blue with decal Bold gold		1
1	SP1654524-0464	Front frame 75° adducted, right, Space Black with decal Bold light grey		1
1	SP1654524-0465	Front frame 75° adducted, right, Anthracite Grey with decal Bold light grey		1
1	SP1654524-0466	Front frame 75° adducted, right, Pure White with decal Bold light grey		1
1	SP1654524-0467	Front frame 75° adducted, right, Carmine Red with decal Bold light grey		1
1	SP1654524-0468	Front frame 75° adducted, right, Azure blue with decal Bold light grey		1
1	SP1654525-0464	Front frame 75° adducted, right, Space Black with decal BOLD red		1
1	SP1654525-0465	Front frame 75° adducted, right, Anthracite Grey with decal BOLD red		1
1	SP1654525-0466	Front frame 75° adducted, right, Pure White with decal BOLD red		1
1	SP1654525-0467	Front frame 75° adducted, right, Carmine Red with decal BOLD red		1
1	SP1654525-0468	Front frame 75° adducted, right, Azure blue with decal BOLD red		1
1	SP1654526-0464	Front frame 75° adducted, right, Space Black with decal Bold black		1
1	SP1654526-0465	Front frame 75° adducted, right, Anthracite Grey with decal Bold black		1
1	SP1654526-0466	Front frame 75° adducted, right, Pure White with decal Bold black		1
1	SP1654526-0467	Front frame 75° adducted, right, Carmine Red with decal Bold black		1
1	SP1654526-0468	Front frame 75° adducted, right, Azure blue with decal Bold black		1

Frame Aluminium 75° adducted

Pos.	Part number	Description	Valid from → until	Qty
1	SP1654527-0185	Front frame 75° adducted, right, dynamit orange without decal for the option TAG		1
1	SP1654527-0419	Front frame 75° adducted, right, Silver Grey without decal for the option TAG		1
1	SP1654527-0450	Front frame 75° adducted, right, Smokey Grey without decal for the option TAG		1
1	SP1654527-0464	Front frame 75° adducted, right, Space Black without decal for the option TAG		1
1	SP1654527-0465	Front frame 75° adducted, right, Anthracite Grey without decal for the option TAG		1
1	SP1654527-0466	Front frame 75° adducted, right, Pure White without decal for the option TAG		1
1	SP1654527-0467	Front frame 75° adducted, right, Carmine Red without decal for the option TAG		1
1	SP1654527-0468	Front frame 75° adducted, right, Azure blue without decal for the option TAG		1
1	SP1654527-0469	Front frame 75° adducted, right, Ruby Red without decal for the option TAG		1
1	SP1654527-0470	Front frame 75° adducted, right, ferrite without decal for the option TAG		1
1	SP1654527-0471	Front frame 75° adducted, right, Slate Metallic without decal for the option TAG		1
1	SP1654527-0472	Front frame 75° adducted, right, Forest Green without decal for the option TAG		1
1	SP1654527-5969	Front frame 75° adducted, right, Royal blue without decal for the option TAG		1
1	SP1654527-9005	Front frame 75° adducted, right, Glossy Black without decal for the option TAG		1
2	SP1654528-0464	Front frame 75° adducted, left, Space Black with decal Bold blue		1
2	SP1654528-0465	Front frame 75° adducted, left, Anthracite Grey with decal Bold blue		1
2	SP1654528-0466	Front frame 75° adducted, left, Pure White with decal Bold blue		1
2	SP1654528-0467	Front frame 75° adducted, left, Carmine Red with decal Bold blue		1
2	SP1654528-0468	Front frame 75° adducted, left, Azure blue with decal Bold blue		1
2	SP1654529-0464	Front frame 75° adducted, left, Space Black with decal Bold dark grey		1
2	SP1654529-0465	Front frame 75° adducted, left, Anthracite Grey with decal Bold dark grey		1
2	SP1654529-0466	Front frame 75° adducted, left, Pure White with decal Bold dark grey		1
2	SP1654529-0467	Front frame 75° adducted, left, Carmine Red with decal Bold dark grey		1
2	SP1654529-0468	Front frame 75° adducted, left, Azure blue with decal Bold dark grey		1
2	SP1654530-0464	Front frame 75° adducted, left, Space Black with decal Bold gold		1
2	SP1654530-0465	Front frame 75° adducted, left, Anthracite Grey with decal Bold gold		1
2	SP1654530-0466	Front frame 75° adducted, left, Pure White with decal Bold gold		1
2	SP1654530-0467	Front frame 75° adducted, left, Carmine Red with decal Bold gold		1
2	SP1654530-0468	Front frame 75° adducted, left, Azure blue with decal Bold gold		1
2	SP1654531-0464	Front frame 75° adducted, left, Space Black with decal Bold light grey		1
2	SP1654531-0465	Front frame 75° adducted, left, Anthracite Grey with decal Bold light grey		1
2	SP1654531-0466	Front frame 75° adducted, left, Pure White with decal Bold light grey		1
2	SP1654531-0467	Front frame 75° adducted, left, Carmine Red with decal Bold light grey		1
2	SP1654531-0468	Front frame 75° adducted, left, Azure blue with decal Bold light grey		1
2	SP1654532-0464	Front frame 75° adducted, left, Space Black with decal BOLD red		1
2	SP1654532-0465	Front frame 75° adducted, left, Anthracite Grey with decal BOLD red		1
2	SP1654532-0466	Front frame 75° adducted, left, Pure White with decal BOLD red		1
2	SP1654532-0467	Front frame 75° adducted, left, Carmine Red with decal BOLD red		1
2	SP1654532-0468	Front frame 75° adducted, left, Azure blue with decal BOLD red		1
2	SP1654533-0464	Front frame 75° adducted, left, Space Black with decal Bold black		1
2	SP1654533-0465	Front frame 75° adducted, left, Anthracite Grey with decal Bold black		1
2	SP1654533-0466	Front frame 75° adducted, left, Pure White with decal Bold black		1
2	SP1654533-0467	Front frame 75° adducted, left, Carmine Red with decal Bold black		1
2	SP1654533-0468	Front frame 75° adducted, left, Azure blue with decal Bold black		1
2	SP1654534-0185	Front frame 75° adducted, left, dynamit orange with decal TAG red		1

Frame Aluminium 75° adducted

Pos.	Part number	Description	Valid from → until	Qty
2	SP1654534-0419	Front frame 75° adducted, left, Silver Grey with decal TAG red		1
2	SP1654534-0450	Front frame 75° adducted, left, Smokey Grey with decal TAG red		1
2	SP1654534-0464	Front frame 75° adducted, left, Space Black with decal TAG red		1
2	SP1654534-0465	Front frame 75° adducted, left, Anthracite Grey with decal TAG red		1
2	SP1654534-0466	Front frame 75° adducted, left, Pure White with decal TAG red		1
2	SP1654534-0467	Front frame 75° adducted, left, Carmine Red with decal TAG red		1
2	SP1654534-0468	Front frame 75° adducted, left, Azure blue with decal TAG red		1
2	SP1654534-0469	Front frame 75° adducted, left, Ruby Red with decal TAG red		1
2	SP1654534-0470	Front frame 75° adducted, left, ferrite with decal TAG red		1
2	SP1654534-0471	Front frame 75° adducted, left, Slate Metallic with decal TAG red		1
2	SP1654534-0472	Front frame 75° adducted, left, Forest Green with decal TAG red		1
2	SP1654534-5969	Front frame 75° adducted, left, Royal blue with decal TAG red		1
2	SP1654534-9005	Front frame 75° adducted, left, Glossy Black with decal TAG red		1
2	SP1654535-0185	Front frame 75° adducted, left, dynamit orange with decal TAG black		1
2	SP1654535-0419	Front frame 75° adducted, left, Silver Grey with decal TAG black		1
2	SP1654535-0450	Front frame 75° adducted, left, Smokey Grey with decal TAG black		1
2	SP1654535-0464	Front frame 75° adducted, left, Space Black with decal TAG black		1
2	SP1654535-0465	Front frame 75° adducted, left, Anthracite Grey with decal TAG black		1
2	SP1654535-0466	Front frame 75° adducted, left, Pure White with decal TAG black		1
2	SP1654535-0467	Front frame 75° adducted, left, Carmine Red with decal TAG black		1
2	SP1654535-0468	Front frame 75° adducted, left, Azure blue with decal TAG black		1
2	SP1654535-0469	Front frame 75° adducted, left, Ruby Red with decal TAG black		1
2	SP1654535-0470	Front frame 75° adducted, left, ferrite with decal TAG black		1
2	SP1654535-0471	Front frame 75° adducted, left, Slate Metallic with decal TAG black		1
2	SP1654535-0472	Front frame 75° adducted, left, Forest Green with decal TAG black		1
2	SP1654535-5969	Front frame 75° adducted, left, Royal blue with decal TAG black		1
2	SP1654535-9005	Front frame 75° adducted, left, Glossy Black with decal TAG black		1
3	SP1659570	Front frame protectors, pair		1

Configuration table for frame Aluminium 75° adducted


i	Rear Wheel 24"				Rear Wheel 25"				Rear Wheel 26"						
	Frame Aluminium Adducted 75° (203,6 mm)	FSTF 48	LLL 29 -> LLL 36	LLL 37 -> LLL 45	FSTF 49	LLL 29 -> LLL 36	LLL 37 -> LLL 46	FSTF 50	LLL 29 -> LLL 36	LLL 37 -> LLL 47	FSTF 51	LLL 29 -> LLL 37	LLL 38 -> LLL 48	FSTF 52	LLL 29 -> LLL 38
	FSTF 49	LLL 29 -> LLL 37	LLL 38 -> LLL 46	FSTF 50	LLL 29 -> LLL 37	LLL 38 -> LLL 47	FSTF 51	LLL 29 -> LLL 38	LLL 39 -> LLL 48						
	FSTF 50	LLL 29 -> LLL 38	LLL 39 -> LLL 47	FSTF 51	LLL 29 -> LLL 38	LLL 39 -> LLL 48									



ISPS1665364

Pos.	Part number	Description	Valid from → until	Qty
i	ISPDINFC1-0037	Type of frame in relation to footrest, knee to heel length (UL), front seat height (SHv) and rear wheel size		1

Frame Aluminium 90° standard

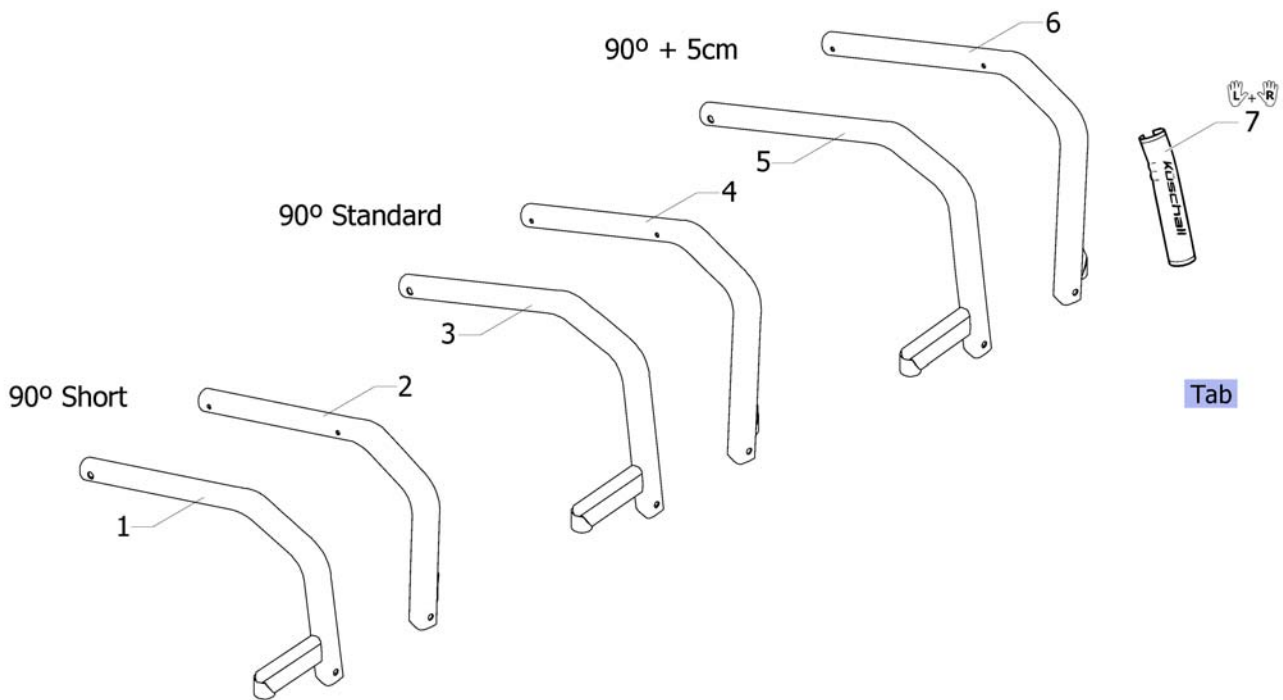

ISPS1665306

Pos.	Part number	Description	Valid from → until	Qty
Tab	-	Configuration table for frame Aluminium 90° standard		0
1	SP1654536-0464	Front frame 90° short, right, Space Black with decal Bold blue		1
1	SP1654536-0465	Front frame 90° short, right, Anthracite Grey with decal Bold blue		1
1	SP1654536-0466	Front frame 90° short, right, Pure White with decal Bold blue		1
1	SP1654536-0467	Front frame 90° short, right, Carmine Red with decal Bold blue		1
1	SP1654536-0468	Front frame 90° short, right, Azure blue with decal Bold blue		1
1	SP1654537-0464	Front frame 90° short, right, Space Black with decal Bold dark grey		1
1	SP1654537-0465	Front frame 90° short, right, Anthracite Grey with decal Bold dark grey		1
1	SP1654537-0466	Front frame 90° short, right, Pure White with decal Bold dark grey		1
1	SP1654537-0467	Front frame 90° short, right, Carmine Red with decal Bold dark grey		1
1	SP1654537-0468	Front frame 90° short, right, Azure blue with decal Bold dark grey		1
1	SP1654538-0464	Front frame 90° short, right, Space Black with decal Bold gold		1
1	SP1654538-0465	Front frame 90° short, right, Anthracite Grey with decal Bold gold		1
1	SP1654538-0466	Front frame 90° short, right, Pure White with decal Bold gold		1
1	SP1654538-0467	Front frame 90° short, right, Carmine Red with decal Bold gold		1
1	SP1654538-0468	Front frame 90° short, right, Azure blue with decal Bold gold		1
1	SP1654539-0464	Front frame 90° short, right, Space Black with decal Bold light grey		1
1	SP1654539-0465	Front frame 90° short, right, Anthracite Grey with decal Bold light grey		1
1	SP1654539-0466	Front frame 90° short, right, Pure White with decal Bold light grey		1
1	SP1654539-0467	Front frame 90° short, right, Carmine Red with decal Bold light grey		1
1	SP1654539-0468	Front frame 90° short, right, Azure blue with decal Bold light grey		1
1	SP1654540-0464	Front frame 90° short, right, Space Black with decal BOLD red		1
1	SP1654540-0465	Front frame 90° short, right, Anthracite Grey with decal BOLD red		1
1	SP1654540-0466	Front frame 90° short, right, Pure White with decal BOLD red		1
1	SP1654540-0467	Front frame 90° short, right, Carmine Red with decal BOLD red		1
1	SP1654540-0468	Front frame 90° short, right, Azure blue with decal BOLD red		1
1	SP1654541-0464	Front frame 90° short, right, Space Black with decal Bold black		1
1	SP1654541-0465	Front frame 90° short, right, Anthracite Grey with decal Bold black		1
1	SP1654541-0466	Front frame 90° short, right, Pure White with decal Bold black		1
1	SP1654541-0467	Front frame 90° short, right, Carmine Red with decal Bold black		1
1	SP1654541-0468	Front frame 90° short, right, Azure blue with decal Bold black		1

Frame Aluminium 90° standard

Pos.	Part number	Description	Valid from → until	Qty
1	SP1654542-0185	Front frame 90° short, right, dynamit orange without decal for the option TAG		1
1	SP1654542-0419	Front frame 90° short, right, Silver Grey without decal for the option TAG		1
1	SP1654542-0450	Front frame 90° short, right, Smokey Grey without decal for the option TAG		1
1	SP1654542-0464	Front frame 90° short, right, Space Black without decal for the option TAG		1
1	SP1654542-0465	Front frame 90° short, right, Anthracite Grey without decal for the option TAG		1
1	SP1654542-0466	Front frame 90° short, right, Pure White without decal for the option TAG		1
1	SP1654542-0467	Front frame 90° short, right, Carmine Red without decal for the option TAG		1
1	SP1654542-0468	Front frame 90° short, right, Azure blue without decal for the option TAG		1
1	SP1654542-0469	Front frame 90° short, right, Ruby Red without decal for the option TAG		1
1	SP1654542-0470	Front frame 90° short, right, ferrite without decal for the option TAG		1
1	SP1654542-0471	Front frame 90° short, right, Slate Metallic without decal for the option TAG		1
1	SP1654542-0472	Front frame 90° short, right, Forest Green without decal for the option TAG		1
1	SP1654542-5969	Front frame 90° short, right, Royal blue without decal for the option TAG		1
1	SP1654542-9005	Front frame 90° short, right, Glossy Black without decal for the option TAG		1
2	SP1654543-0464	Front frame 90° short, left, Space Black with decal Bold blue		1
2	SP1654543-0465	Front frame 90° short, left, Anthracite Grey with decal Bold blue		1
2	SP1654543-0466	Front frame 90° short, left, Pure White with decal Bold blue		1
2	SP1654543-0467	Front frame 90° short, left, Carmine Red with decal Bold blue		1
2	SP1654543-0468	Front frame 90° short, left, Azure blue with decal Bold blue		1
2	SP1654544-0464	Front frame 90° short, left, Space Black with decal Bold dark grey		1
2	SP1654544-0465	Front frame 90° short, left, Anthracite Grey with decal Bold dark grey		1
2	SP1654544-0466	Front frame 90° short, left, Pure White with decal Bold dark grey		1
2	SP1654544-0467	Front frame 90° short, left, Carmine Red with decal Bold dark grey		1
2	SP1654544-0468	Front frame 90° short, left, Azure blue with decal Bold dark grey		1
2	SP1654545-0464	Front frame 90° short, left, Space Black with decal Bold gold		1
2	SP1654545-0465	Front frame 90° short, left, Anthracite Grey with decal Bold gold		1
2	SP1654545-0466	Front frame 90° short, left, Pure White with decal Bold gold		1
2	SP1654545-0467	Front frame 90° short, left, Carmine Red with decal Bold gold		1
2	SP1654545-0468	Front frame 90° short, left, Azure blue with decal Bold gold		1
2	SP1654546-0464	Front frame 90° short, left, Space Black with decal Bold light grey		1
2	SP1654546-0465	Front frame 90° short, left, Anthracite Grey with decal Bold light grey		1
2	SP1654546-0466	Front frame 90° short, left, Pure White with decal Bold light grey		1
2	SP1654546-0467	Front frame 90° short, left, Carmine Red with decal Bold light grey		1
2	SP1654546-0468	Front frame 90° short, left, Azure blue with decal Bold light grey		1
2	SP1654547-0464	Front frame 90° short, left, Space Black with decal BOLD red		1
2	SP1654547-0465	Front frame 90° short, left, Anthracite Grey with decal BOLD red		1
2	SP1654547-0466	Front frame 90° short, left, Pure White with decal BOLD red		1
2	SP1654547-0467	Front frame 90° short, left, Carmine Red with decal BOLD red		1
2	SP1654547-0468	Front frame 90° short, left, Azure blue with decal BOLD red		1
2	SP1654548-0464	Front frame 90° short, left, Space Black with decal Bold black		1
2	SP1654548-0465	Front frame 90° short, left, Anthracite Grey with decal Bold black		1
2	SP1654548-0466	Front frame 90° short, left, Pure White with decal Bold black		1
2	SP1654548-0467	Front frame 90° short, left, Carmine Red with decal Bold black		1
2	SP1654548-0468	Front frame 90° short, left, Azure blue with decal Bold black		1
2	SP1654549-0185	Front frame 90° short, left, dynamit orange with decal TAG red		1
2	SP1654549-0419	Front frame 90° short, left, Silver Grey with decal TAG red		1
2	SP1654549-0450	Front frame 90° short, left, Smokey Grey with decal TAG red		1
2	SP1654549-0464	Front frame 90° short, left, Space Black with decal TAG red		1
2	SP1654549-0465	Front frame 90° short, left, Anthracite Grey with decal TAG red		1
2	SP1654549-0466	Front frame 90° short, left, Pure White with decal TAG red		1
2	SP1654549-0467	Front frame 90° short, left, Carmine Red with decal TAG red		1
2	SP1654549-0468	Front frame 90° short, left, Azure blue with decal TAG red		1

Frame Aluminium 90° standard

Pos.	Part number	Description	Valid from → until	Qty
2	SP1654549-0469	Front frame 90° short, left, Ruby Red with decal TAG red		1
2	SP1654549-0470	Front frame 90° short, left, ferrite with decal TAG red		1
2	SP1654549-0471	Front frame 90° short, left, Slate Metallic with decal TAG red		1
2	SP1654549-0472	Front frame 90° short, left, Forest Green with decal TAG red		1
2	SP1654549-5969	Front frame 90° short, left, Royal blue with decal TAG red		1
2	SP1654549-9005	Front frame 90° short, left, Glossy Black with decal TAG red		1
2	SP1654550-0185	Front frame 90° short, left, dynamit orange with decal TAG black		1
2	SP1654550-0419	Front frame 90° short, left, Silver Grey with decal TAG black		1
2	SP1654550-0450	Front frame 90° short, left, Smokey Grey with decal TAG black		1
2	SP1654550-0464	Front frame 90° short, left, Space Black with decal TAG black		1
2	SP1654550-0465	Front frame 90° short, left, Anthracite Grey with decal TAG black		1
2	SP1654550-0466	Front frame 90° short, left, Pure White with decal TAG black		1
2	SP1654550-0467	Front frame 90° short, left, Carmine Red with decal TAG black		1
2	SP1654550-0468	Front frame 90° short, left, Azure blue with decal TAG black		1
2	SP1654550-0469	Front frame 90° short, left, Ruby Red with decal TAG black		1
2	SP1654550-0470	Front frame 90° short, left, ferrite with decal TAG black		1
2	SP1654550-0471	Front frame 90° short, left, Slate Metallic with decal TAG black		1
2	SP1654550-0472	Front frame 90° short, left, Forest Green with decal TAG black		1
2	SP1654550-5969	Front frame 90° short, left, Royal blue with decal TAG black		1
2	SP1654550-9005	Front frame 90° short, left, Glossy Black with decal TAG black		1
3	SP1654551-0464	Front frame 90° standard, right, Space Black with decal Bold blue		1
3	SP1654551-0465	Front frame 90° standard, right, Anthracite Grey with decal Bold blue		1
3	SP1654551-0466	Front frame 90° standard, right, Pure White with decal Bold blue		1
3	SP1654551-0467	Front frame 90° standard, right, Carmine Red with decal Bold blue		1
3	SP1654551-0468	Front frame 90° standard, right, Azure blue with decal Bold blue		1
3	SP1654552-0464	Front frame 90° standard, right, Space Black with decal Bold dark grey		1
3	SP1654552-0465	Front frame 90° standard, right, Anthracite Grey with decal Bold dark grey		1
3	SP1654552-0466	Front frame 90° standard, right, Pure White with decal Bold dark grey		1
3	SP1654552-0467	Front frame 90° standard, right, Carmine Red with decal Bold dark grey		1
3	SP1654552-0468	Front frame 90° standard, right, Azure blue with decal Bold dark grey		1
3	SP1654553-0464	Front frame 90° standard, right, Space Black with decal Bold gold		1
3	SP1654553-0465	Front frame 90° standard, right, Anthracite Grey with decal Bold gold		1
3	SP1654553-0466	Front frame 90° standard, right, Pure White with decal Bold gold		1
3	SP1654553-0467	Front frame 90° standard, right, Carmine Red with decal Bold gold		1
3	SP1654553-0468	Front frame 90° standard, right, Azure blue with decal Bold gold		1
3	SP1654567-0464	Front frame 90° standard, right, Space Black with decal Bold light grey		1
3	SP1654567-0465	Front frame 90° standard, right, Anthracite Grey with decal Bold light grey		1
3	SP1654567-0466	Front frame 90° standard, right, Pure White with decal Bold light grey		1
3	SP1654567-0467	Front frame 90° standard, right, Carmine Red with decal Bold light grey		1
3	SP1654567-0468	Front frame 90° standard, right, Azure blue with decal Bold light grey		1
3	SP1654568-0464	Front frame 90° standard, right, Space Black with decal BOLD red		1
3	SP1654568-0465	Front frame 90° standard, right, Anthracite Grey with decal BOLD red		1
3	SP1654568-0466	Front frame 90° standard, right, Pure White with decal BOLD red		1
3	SP1654568-0467	Front frame 90° standard, right, Carmine Red with decal BOLD red		1
3	SP1654568-0468	Front frame 90° standard, right, Azure blue with decal BOLD red		1
3	SP1654569-0464	Front frame 90° standard, right, Space Black with decal Bold black		1
3	SP1654569-0465	Front frame 90° standard, right, Anthracite Grey with decal Bold black		1
3	SP1654569-0466	Front frame 90° standard, right, Pure White with decal Bold black		1
3	SP1654569-0467	Front frame 90° standard, right, Carmine Red with decal Bold black		1
3	SP1654569-0468	Front frame 90° standard, right, Azure blue with decal Bold black		1
3	SP1654570-0185	Front frame 90° standard, right, dynamit orange without decal for the option TAG		1
3	SP1654570-0419	Front frame 90° standard, right, Silver Grey without decal for the option TAG		1
3	SP1654570-0450	Front frame 90° standard, right, Smokey Grey without decal for the option TAG		1

Frame Aluminium 90° standard

Pos.	Part number	Description	Valid from → until	Qty
3	SP1654570-0464	Front frame 90° standard, right, Space Black without decal for the option TAG		1
3	SP1654570-0465	Front frame 90° standard, right, Anthracite Grey without decal for the option TAG		1
3	SP1654570-0466	Front frame 90° standard, right, Pure White without decal for the option TAG		1
3	SP1654570-0467	Front frame 90° standard, right, Carmine Red without decal for the option TAG		1
3	SP1654570-0468	Front frame 90° standard, right, Azure blue without decal for the option TAG		1
3	SP1654570-0469	Front frame 90° standard, right, Ruby Red without decal for the option TAG		1
3	SP1654570-0470	Front frame 90° standard, right, ferrite without decal for the option TAG		1
3	SP1654570-0471	Front frame 90° standard, right, Slate Metallic without decal for the option TAG		1
3	SP1654570-0472	Front frame 90° standard, right, Forest Green without decal for the option TAG		1
3	SP1654570-5969	Front frame 90° standard, right, Royal blue without decal for the option TAG		1
3	SP1654570-9005	Front frame 90° standard, right, Glossy Black without decal for the option TAG		1
4	SP1654571-0464	Front frame 90° standard, left, Space Black with decal Bold blue		1
4	SP1654571-0465	Front frame 90° standard, left, Anthracite Grey with decal Bold blue		1
4	SP1654571-0466	Front frame 90° standard, left, Pure White with decal Bold blue		1
4	SP1654571-0467	Front frame 90° standard, left, Carmine Red with decal Bold blue		1
4	SP1654571-0468	Front frame 90° standard, left, Azure blue with decal Bold blue		1
4	SP1654572-0464	Front frame 90° standard, left, Space Black with decal Bold dark grey		1
4	SP1654572-0465	Front frame 90° standard, left, Anthracite Grey with decal Bold dark grey		1
4	SP1654572-0466	Front frame 90° standard, left, Pure White with decal Bold dark grey		1
4	SP1654572-0467	Front frame 90° standard, left, Carmine Red with decal Bold dark grey		1
4	SP1654572-0468	Front frame 90° standard, left, Azure blue with decal Bold dark grey		1
4	SP1654573-0464	Front frame 90° standard, left, Space Black with decal Bold gold		1
4	SP1654573-0465	Front frame 90° standard, left, Anthracite Grey with decal Bold gold		1
4	SP1654573-0466	Front frame 90° standard, left, Pure White with decal Bold gold		1
4	SP1654573-0467	Front frame 90° standard, left, Carmine Red with decal Bold gold		1
4	SP1654573-0468	Front frame 90° standard, left, Azure blue with decal Bold gold		1
4	SP1654574-0464	Front frame 90° standard, left, Space Black with decal Bold light grey		1
4	SP1654574-0465	Front frame 90° standard, left, Anthracite Grey with decal Bold light grey		1
4	SP1654574-0466	Front frame 90° standard, left, Pure White with decal Bold light grey		1
4	SP1654574-0467	Front frame 90° standard, left, Carmine Red with decal Bold light grey		1
4	SP1654574-0468	Front frame 90° standard, left, Azure blue with decal Bold light grey		1
4	SP1654575-0464	Front frame 90° standard, left, Space Black with decal BOLD red		1
4	SP1654575-0465	Front frame 90° standard, left, Anthracite Grey with decal BOLD red		1
4	SP1654575-0466	Front frame 90° standard, left, Pure White with decal BOLD red		1
4	SP1654575-0467	Front frame 90° standard, left, Carmine Red with decal BOLD red		1
4	SP1654575-0468	Front frame 90° standard, left, Azure blue with decal BOLD red		1
4	SP1654576-0464	Front frame 90° standard, left, Space Black with decal Bold black		1
4	SP1654576-0465	Front frame 90° standard, left, Anthracite Grey with decal Bold black		1
4	SP1654576-0466	Front frame 90° standard, left, Pure White with decal Bold black		1
4	SP1654576-0467	Front frame 90° standard, left, Carmine Red with decal Bold black		1
4	SP1654576-0468	Front frame 90° standard, left, Azure blue with decal Bold black		1
4	SP1654838-0185	Front frame 90° standard, left, dynamit orange with decal TAG red		1
4	SP1654838-0419	Front frame 90° standard, left, Silver Grey with decal TAG red		1
4	SP1654838-0450	Front frame 90° standard, left, Smokey Grey with decal TAG red		1
4	SP1654838-0464	Front frame 90° standard, left, Space Black with decal TAG red		1
4	SP1654838-0465	Front frame 90° standard, left, Anthracite Grey with decal TAG red		1
4	SP1654838-0466	Front frame 90° standard, left, Pure White with decal TAG red		1
4	SP1654838-0467	Front frame 90° standard, left, Carmine Red with decal TAG red		1

Frame Aluminium 90° standard

Pos.	Part number	Description	Valid from → until	Qty
4	SP1654838-0468	Front frame 90° standard, left, Azure blue with decal TAG red		1
4	SP1654838-0469	Front frame 90° standard, left, Ruby Red with decal TAG red		1
4	SP1654838-0470	Front frame 90° standard, left, ferrite with decal TAG red		1
4	SP1654838-0471	Front frame 90° standard, left, Slate Metallic with decal TAG red		1
4	SP1654838-0472	Front frame 90° standard, left, Forest Green with decal TAG red		1
4	SP1654838-5969	Front frame 90° standard, left, Royal blue with decal TAG red		1
4	SP1654838-9005	Front frame 90° standard, left, Glossy Black with decal TAG red		1
4	SP1655123-0185	Front frame 90° standard, left, dynamit orange with decal TAG black		1
4	SP1655123-0419	Front frame 90° standard, left, Silver Grey with decal TAG black		1
4	SP1655123-0450	Front frame 90° standard, left, Smokey Grey with decal TAG black		1
4	SP1655123-0464	Front frame 90° standard, left, Space Black with decal TAG black		1
4	SP1655123-0465	Front frame 90° standard, left, Anthracite Grey with decal TAG black		1
4	SP1655123-0466	Front frame 90° standard, left, Pure White with decal TAG black		1
4	SP1655123-0467	Front frame 90° standard, left, Carmine Red with decal TAG black		1
4	SP1655123-0468	Front frame 90° standard, left, Azure blue with decal TAG black		1
4	SP1655123-0469	Front frame 90° standard, left, Ruby Red with decal TAG black		1
4	SP1655123-0470	Front frame 90° standard, left, ferrite with decal TAG black		1
4	SP1655123-0471	Front frame 90° standard, left, Slate Metallic with decal TAG black		1
4	SP1655123-0472	Front frame 90° standard, left, Forest Green with decal TAG black		1
4	SP1655123-5969	Front frame 90° standard, left, Royal blue with decal TAG black		1
4	SP1655123-9005	Front frame 90° standard, left, Glossy Black with decal TAG black		1
5	SP1655124-0464	Front frame 90° +5cm, right, Space Black with decal Bold blue		1
5	SP1655124-0465	Front frame 90° +5cm, right, Anthracite Grey with decal Bold blue		1
5	SP1655124-0466	Front frame 90° +5cm, right, Pure White with decal Bold blue		1
5	SP1655124-0467	Front frame 90° +5cm, right, Carmine Red with decal Bold blue		1
5	SP1655124-0468	Front frame 90° +5cm, right, Azure blue with decal Bold blue		1
5	SP1655238-0464	Front frame 90° +5cm, right, Space Black with decal Bold dark grey		1
5	SP1655238-0465	Front frame 90° +5cm, right, Anthracite Grey with decal Bold dark grey		1
5	SP1655238-0466	Front frame 90° +5cm, right, Pure White with decal Bold dark grey		1
5	SP1655238-0467	Front frame 90° +5cm, right, Carmine Red with decal Bold dark grey		1
5	SP1655238-0468	Front frame 90° +5cm, right, Azure blue with decal Bold dark grey		1
5	SP1655239-0464	Front frame 90° +5cm, right, Space Black with decal Bold gold		1
5	SP1655239-0465	Front frame 90° +5cm, right, Anthracite Grey with decal Bold gold		1
5	SP1655239-0466	Front frame 90° +5cm, right, Pure White with decal Bold gold		1
5	SP1655239-0467	Front frame 90° +5cm, right, Carmine Red with decal Bold gold		1
5	SP1655239-0468	Front frame 90° +5cm, right, Azure blue with decal Bold gold		1
5	SP1655240-0464	Front frame 90° +5cm, right, Space Black with decal Bold light grey		1
5	SP1655240-0465	Front frame 90° +5cm, right, Anthracite Grey with decal Bold light grey		1
5	SP1655240-0466	Front frame 90° +5cm, right, Pure White with decal Bold light grey		1
5	SP1655240-0467	Front frame 90° +5cm, right, Carmine Red with decal Bold light grey		1
5	SP1655240-0468	Front frame 90° +5cm, right, Azure blue with decal Bold light grey		1
5	SP1655241-0464	Front frame 90° +5cm, right, Space Black with decal BOLD red		1
5	SP1655241-0465	Front frame 90° +5cm, right, Anthracite Grey with decal BOLD red		1
5	SP1655241-0466	Front frame 90° +5cm, right, Pure White with decal BOLD red		1
5	SP1655241-0467	Front frame 90° +5cm, right, Carmine Red with decal BOLD red		1
5	SP1655241-0468	Front frame 90° +5cm, right, Azure blue with decal BOLD red		1
5	SP1655471-0464	Front frame 90° +5cm, right, Space Black with decal Bold black		1
5	SP1655471-0465	Front frame 90° +5cm, right, Anthracite Grey with decal Bold black		1
5	SP1655471-0466	Front frame 90° +5cm, right, Pure White with decal Bold black		1
5	SP1655471-0467	Front frame 90° +5cm, right, Carmine Red with decal Bold black		1
5	SP1655471-0468	Front frame 90° +5cm, right, Azure blue with decal Bold black		1
5	SP1655472-0185	Front frame 90° +5cm, right, dynamit orange without decal for the option TAG		1
5	SP1655472-0419	Front frame 90° +5cm, right, Silver Grey without decal for the option TAG		1
5	SP1655472-0450	Front frame 90° +5cm, right, Smokey Grey without decal for the option TAG		1

Frame Aluminium 90° standard


Pos.	Part number	Description	Valid from → until	Qty
5	SP1655472-0464	Front frame 90° +5cm, right, Space Black without decal for the option TAG		1
5	SP1655472-0465	Front frame 90° +5cm, right, Anthracite Grey without decal for the option TAG		1
5	SP1655472-0466	Front frame 90° +5cm, right, Pure White without decal for the option TAG		1
5	SP1655472-0467	Front frame 90° +5cm, right, Carmine Red without decal for the option TAG		1
5	SP1655472-0468	Front frame 90° +5cm, right, Azure blue without decal for the option TAG		1
5	SP1655472-0469	Front frame 90° +5cm, right, Ruby Red without decal for the option TAG		1
5	SP1655472-0470	Front frame 90° +5cm, right, ferrite without decal for the option TAG		1
5	SP1655472-0471	Front frame 90° +5cm, right, Slate Metallic without decal for the option TAG		1
5	SP1655472-0472	Front frame 90° +5cm, right, Forest Green without decal for the option TAG		1
5	SP1655472-5969	Front frame 90° +5cm, right, Royal blue without decal for the option TAG		1
5	SP1655472-9005	Front frame 90° +5cm, right, Glossy Black without decal for the option TAG		1
6	SP1655473-0464	Front frame 90° +5cm, left, Space Black with decal Bold blue		1
6	SP1655473-0465	Front frame 90° +5cm, left, Anthracite Grey with decal Bold blue		1
6	SP1655473-0466	Front frame 90° +5cm, left, Pure White with decal Bold blue		1
6	SP1655473-0467	Front frame 90° +5cm, left, Carmine Red with decal Bold blue		1
6	SP1655473-0468	Front frame 90° +5cm, left, Azure blue with decal Bold blue		1
6	SP1655474-0464	Front frame 90° +5cm, left, Space Black with decal Bold dark grey		1
6	SP1655474-0465	Front frame 90° +5cm, left, Anthracite Grey with decal Bold dark grey		1
6	SP1655474-0466	Front frame 90° +5cm, left, Pure White with decal Bold dark grey		1
6	SP1655474-0467	Front frame 90° +5cm, left, Carmine Red with decal Bold dark grey		1
6	SP1655474-0468	Front frame 90° +5cm, left, Azure blue with decal Bold dark grey		1
6	SP1655475-0464	Front frame 90° +5cm, left, Space Black with decal Bold gold		1
6	SP1655475-0465	Front frame 90° +5cm, left, Anthracite Grey with decal Bold gold		1
6	SP1655475-0466	Front frame 90° +5cm, left, Pure White with decal Bold gold		1
6	SP1655475-0467	Front frame 90° +5cm, left, Carmine Red with decal Bold gold		1
6	SP1655475-0468	Front frame 90° +5cm, left, Azure blue with decal Bold gold		1
6	SP1655476-0464	Front frame 90° +5cm, left, Space Black with decal Bold light grey		1
6	SP1655476-0465	Front frame 90° +5cm, left, Anthracite Grey with decal Bold light grey		1
6	SP1655476-0466	Front frame 90° +5cm, left, Pure White with decal Bold light grey		1
6	SP1655476-0467	Front frame 90° +5cm, left, Carmine Red with decal Bold light grey		1
6	SP1655476-0468	Front frame 90° +5cm, left, Azure blue with decal Bold light grey		1
6	SP1655477-0464	Front frame 90° +5cm, left, Space Black with decal BOLD red		1
6	SP1655477-0465	Front frame 90° +5cm, left, Anthracite Grey with decal BOLD red		1
6	SP1655477-0466	Front frame 90° +5cm, left, Pure White with decal BOLD red		1
6	SP1655477-0467	Front frame 90° +5cm, left, Carmine Red with decal BOLD red		1
6	SP1655477-0468	Front frame 90° +5cm, left, Azure blue with decal BOLD red		1
6	SP1655478-0464	Front frame 90° +5cm, left, Space Black with decal Bold black		1
6	SP1655478-0465	Front frame 90° +5cm, left, Anthracite Grey with decal Bold black		1
6	SP1655478-0466	Front frame 90° +5cm, left, Pure White with decal Bold black		1
6	SP1655478-0467	Front frame 90° +5cm, left, Carmine Red with decal Bold black		1
6	SP1655478-0468	Front frame 90° +5cm, left, Azure blue with decal Bold black		1
6	SP1655479-0185	Front frame 90° +5cm, left, dynamit orange with decal TAG red		1
6	SP1655479-0419	Front frame 90° +5cm, left, Silver Grey with decal TAG red		1
6	SP1655479-0450	Front frame 90° +5cm, left, Smokey Grey with decal TAG red		1
6	SP1655479-0464	Front frame 90° +5cm, left, Space Black with decal TAG red		1
6	SP1655479-0465	Front frame 90° +5cm, left, Anthracite Grey with decal TAG red		1
6	SP1655479-0466	Front frame 90° +5cm, left, Pure White with decal TAG red		1
6	SP1655479-0467	Front frame 90° +5cm, left, Carmine Red with decal TAG red		1
6	SP1655479-0468	Front frame 90° +5cm, left, Azure blue with decal TAG red		1
6	SP1655479-0469	Front frame 90° +5cm, left, Ruby Red with decal TAG red		1
6	SP1655479-0470	Front frame 90° +5cm, left, ferrite with decal TAG red		1
6	SP1655479-0471	Front frame 90° +5cm, left, Slate Metallic with decal TAG red		1

Frame Aluminium 90° standard

Pos.	Part number	Description	Valid from → until	Qty
6	SP1655479-0472	Front frame 90° +5cm, left, Forest Green with decal TAG red		1
6	SP1655479-5969	Front frame 90° +5cm, left, Royal blue with decal TAG red		1
6	SP1655479-9005	Front frame 90° +5cm, left, Glossy Black with decal TAG red		1
6	SP1655480-0185	Front frame 90° +5cm, left, dynamit orange with decal TAG black		1
6	SP1655480-0419	Front frame 90° +5cm, left, Silver Grey with decal TAG black		1
6	SP1655480-0450	Front frame 90° +5cm, left, Smokey Grey with decal TAG black		1
6	SP1655480-0464	Front frame 90° +5cm, left, Space Black with decal TAG black		1
6	SP1655480-0465	Front frame 90° +5cm, left, Anthracite Grey with decal TAG black		1
6	SP1655480-0466	Front frame 90° +5cm, left, Pure White with decal TAG black		1
6	SP1655480-0467	Front frame 90° +5cm, left, Carmine Red with decal TAG black		1
6	SP1655480-0468	Front frame 90° +5cm, left, Azure blue with decal TAG black		1
6	SP1655480-0469	Front frame 90° +5cm, left, Ruby Red with decal TAG black		1
6	SP1655480-0470	Front frame 90° +5cm, left, ferrite with decal TAG black		1
6	SP1655480-0471	Front frame 90° +5cm, left, Slate Metallic with decal TAG black		1
6	SP1655480-0472	Front frame 90° +5cm, left, Forest Green with decal TAG black		1
6	SP1655480-5969	Front frame 90° +5cm, left, Royal blue with decal TAG black		1
6	SP1655480-9005	Front frame 90° +5cm, left, Glossy Black with decal TAG black		1
7	SP1659570	Front frame protectors, pair		1

Configuration table for frame Aluminium 90° standard


i	Rear Wheel 24"						Rear Wheel 25"						Rear Wheel 26"							
	FSTF		LLL		LLL		FSTF		LLL		LLL		FSTF		LLL		LLL			
Frame Aluminium 90° Short (261,1 mm)	FSTF 45	LLL 29 -> LLL 32	LLL 33 -> LLL 41	LLL 29 -> LLL 41	FSTF 46	LLL 29 -> LLL 32	LLL 33 -> LLL 42	LLL 29 -> LLL 42	FSTF 47	LLL 29 -> LLL 32	LLL 33 -> LLL 43	LLL 29 -> LLL 43	FSTF 48	LLL 29 -> LLL 33	LLL 34 -> LLL 44	LLL 30 -> LLL 44	FSTF 49	LLL 29 -> LLL 34	LLL 35 -> LLL 45	LLL 31 -> LLL 45
	FSTF 46	LLL 29 -> LLL 33	LLL 34 -> LLL 42	LLL 30 -> LLL 42	FSTF 47	LLL 29 -> LLL 33	LLL 34 -> LLL 43	LLL 30 -> LLL 43	FSTF 48	LLL 29 -> LLL 33	LLL 34 -> LLL 44	LLL 30 -> LLL 44	FSTF 49	LLL 29 -> LLL 34	LLL 35 -> LLL 45	LLL 31 -> LLL 45	FSTF 50	LLL 29 -> LLL 36	LLL 38 -> LLL 46	LLL 34 -> LLL 46
	FSTF 47	LLL 29 -> LLL 34	LLL 35 -> LLL 43	LLL 31 -> LLL 43	FSTF 48	LLL 29 -> LLL 34	LLL 35 -> LLL 44	LLL 31 -> LLL 44	FSTF 49	LLL 29 -> LLL 34	LLL 35 -> LLL 45	LLL 31 -> LLL 45	FSTF 50	LLL 29 -> LLL 36	LLL 38 -> LLL 46	LLL 34 -> LLL 46	FSTF 51	LLL 29 -> LLL 37	LLL 39 -> LLL 47	LLL 35 -> LLL 47
Frame Aluminium 90° Std. (308,3 mm)	FSTF 48	LLL 29 -> LLL 36	LLL 38 -> LLL 44	LLL 34 -> LLL 44	FSTF 49	LLL 29 -> LLL 36	LLL 38 -> LLL 45	LLL 34 -> LLL 45	FSTF 50	LLL 29 -> LLL 37	LLL 39 -> LLL 46	LLL 35 -> LLL 46	FSTF 51	LLL 29 -> LLL 37	LLL 39 -> LLL 47	LLL 35 -> LLL 47	FSTF 52	LLL 29 -> LLL 38	LLL 40 -> LLL 48	LLL 36 -> LLL 48
	FSTF 49	LLL 29 -> LLL 37	LLL 39 -> LLL 45	LLL 35 -> LLL 45	FSTF 50	LLL 29 -> LLL 37	LLL 39 -> LLL 46	LLL 35 -> LLL 46	FSTF 51	LLL 29 -> LLL 38	LLL 40 -> LLL 47	LLL 36 -> LLL 47	FSTF 52	LLL 29 -> LLL 38	LLL 40 -> LLL 48	LLL 36 -> LLL 48	FSTF 53	LLL 29 -> LLL 39	LLL 41 -> LLL 49	LLL 37 -> LLL 49
	FSTF 50	LLL 29 -> LLL 38	LLL 40 -> LLL 46	LLL 36 -> LLL 46	FSTF 51	LLL 29 -> LLL 38	LLL 40 -> LLL 47	LLL 36 -> LLL 47	FSTF 52	LLL 29 -> LLL 39	LLL 41 -> LLL 48	LLL 37 -> LLL 48	FSTF 53	LLL 29 -> LLL 39	LLL 41 -> LLL 49	LLL 37 -> LLL 49	FSTF 54	LLL 29 -> LLL 40	LLL 42 -> LLL 50	LLL 38 -> LLL 50
Frame Aluminium 90° +5cm (327 mm)	FSTF 51	LLL 29 -> LLL 39	LLL 40 -> LLL 46	LLL 36 -> LLL 46	FSTF 52	LLL 29 -> LLL 39	LLL 41 -> LLL 48	LLL 37 -> LLL 48	FSTF 53	LLL 29 -> LLL 40	LLL 42 -> LLL 49	LLL 38 -> LLL 49	FSTF 54	LLL 29 -> LLL 40	LLL 42 -> LLL 50	LLL 38 -> LLL 50	FSTF 54	LLL 29 -> LLL 40	LLL 42 -> LLL 50	LLL 38 -> LLL 50
	FSTF 51	LLL 29 -> LLL 40	LLL 41 -> LLL 47	LLL 37 -> LLL 47	FSTF 52	LLL 29 -> LLL 39	LLL 41 -> LLL 48	LLL 37 -> LLL 48	FSTF 53	LLL 29 -> LLL 40	LLL 42 -> LLL 49	LLL 38 -> LLL 49	FSTF 54	LLL 29 -> LLL 40	LLL 42 -> LLL 50	LLL 38 -> LLL 50	FSTF 54	LLL 29 -> LLL 40	LLL 42 -> LLL 50	LLL 38 -> LLL 50
	FSTF 52	LLL 29 -> LLL 41	LLL 42 -> LLL 48	LLL 38 -> LLL 48	FSTF 53	LLL 29 -> LLL 40	LLL 42 -> LLL 49	LLL 38 -> LLL 49	FSTF 54	LLL 29 -> LLL 40	LLL 42 -> LLL 50	LLL 38 -> LLL 50	FSTF 54	LLL 29 -> LLL 40	LLL 42 -> LLL 50	LLL 38 -> LLL 50	FSTF 54	LLL 29 -> LLL 40	LLL 42 -> LLL 50	LLL 38 -> LLL 50

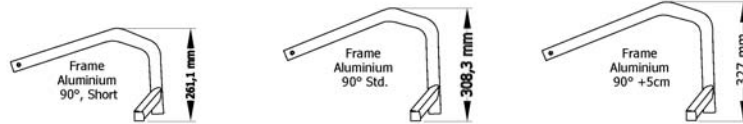

ISPS1665365

Pos.	Part number	Description	Valid from → until	Qty
i	ISPDINFC1-0037	Type of frame in relation to footrest, knee to heel length (UL), front seat height (SHv) and rear wheel size		1

Frame Aluminium 90° adducted

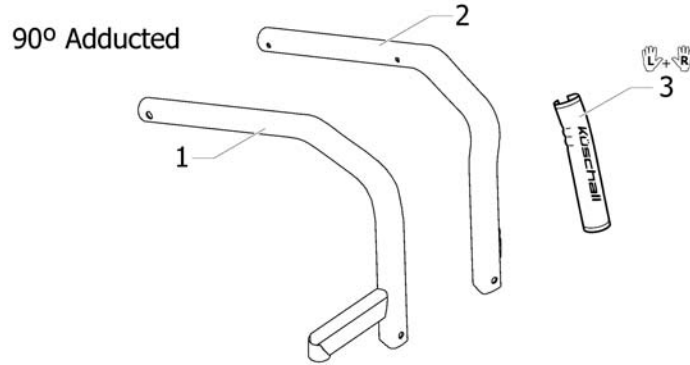

Tab

ISPS1665307

Pos.	Part number	Description	Valid from → until	Qty
Tab	-	Configuration table for frame Aluminium 90° adducted		0
1	SP1655486-0466	Front frame 90° adducted, right, Pure White with decal Bold black		1
2	SP1655490-0465	Front frame 90° adducted, left, Anthracite Grey with decal Bold gold		1
2	SP1655494-0470	Front frame 90° adducted, left, ferrite with decal TAG red		1
1	SP1655481-0464	Front frame 90° adducted, right, Space Black with decal Bold blue		1
1	SP1655481-0465	Front frame 90° adducted, right, Anthracite Grey with decal Bold blue		1
1	SP1655481-0466	Front frame 90° adducted, right, Pure White with decal Bold blue		1
1	SP1655481-0467	Front frame 90° adducted, right, Carmine Red with decal Bold blue		1
1	SP1655481-0468	Front frame 90° adducted, right, Azure blue with decal Bold blue		1
1	SP1655482-0464	Front frame 90° adducted, right, Space Black with decal Bold dark grey		1
1	SP1655482-0465	Front frame 90° adducted, right, Anthracite Grey with decal Bold dark grey		1
1	SP1655482-0466	Front frame 90° adducted, right, Pure White with decal Bold dark grey		1
1	SP1655482-0467	Front frame 90° adducted, right, Carmine Red with decal Bold dark grey		1
1	SP1655482-0468	Front frame 90° adducted, right, Azure blue with decal Bold dark grey		1
1	SP1655483-0464	Front frame 90° adducted, right, Space Black with decal Bold gold		1
1	SP1655483-0465	Front frame 90° adducted, right, Anthracite Grey with decal Bold gold		1
1	SP1655483-0466	Front frame 90° adducted, right, Pure White with decal Bold gold		1
1	SP1655483-0467	Front frame 90° adducted, right, Carmine Red with decal Bold gold		1
1	SP1655483-0468	Front frame 90° adducted, right, Azure blue with decal Bold gold		1
1	SP1655484-0464	Front frame 90° adducted, right, Space Black with decal Bold light grey		1
1	SP1655484-0465	Front frame 90° adducted, right, Anthracite Grey with decal Bold light grey		1
1	SP1655484-0466	Front frame 90° adducted, right, Pure White with decal Bold light grey		1
1	SP1655484-0467	Front frame 90° adducted, right, Carmine Red with decal Bold light grey		1
1	SP1655484-0468	Front frame 90° adducted, right, Azure blue with decal Bold light grey		1
1	SP1655485-0464	Front frame 90° adducted, right, Space Black with decal BOLD red		1
1	SP1655485-0465	Front frame 90° adducted, right, Anthracite Grey with decal BOLD red		1
1	SP1655485-0466	Front frame 90° adducted, right, Pure White with decal BOLD red		1
1	SP1655485-0467	Front frame 90° adducted, right, Carmine Red with decal BOLD red		1
1	SP1655485-0468	Front frame 90° adducted, right, Azure blue with decal BOLD red		1
1	SP1655486-0464	Front frame 90° adducted, right, Space Black with decal Bold black		1
1	SP1655486-0465	Front frame 90° adducted, right, Anthracite Grey with decal Bold black		1

Frame Aluminium 90° adducted


Pos.	Part number	Description	Valid from → until	Qty
1	SP1655486-0467	Front frame 90° adducted, right, Carmine Red with decal Bold black		1
1	SP1655486-0468	Front frame 90° adducted, right, Azure blue with decal Bold black		1
1	SP1655487-0185	Front frame 90° adducted, right, dynamit orange without decal for the option TAG		1
1	SP1655487-0419	Front frame 90° adducted, right, Silver Grey without decal for the option TAG		1
1	SP1655487-0450	Front frame 90° adducted, right, Smokey Grey without decal for the option TAG		1
1	SP1655487-0464	Front frame 90° adducted, right, Space Black without decal for the option TAG		1
1	SP1655487-0465	Front frame 90° adducted, right, Anthracite Grey without decal for the option TAG		1
1	SP1655487-0466	Front frame 90° adducted, right, Pure White without decal for the option TAG		1
1	SP1655487-0467	Front frame 90° adducted, right, Carmine Red without decal for the option TAG		1
1	SP1655487-0468	Front frame 90° adducted, right, Azure blue without decal for the option TAG		1
1	SP1655487-0469	Front frame 90° adducted, right, Ruby Red without decal for the option TAG		1
1	SP1655487-0470	Front frame 90° adducted, right, ferrite without decal for the option TAG		1
1	SP1655487-0471	Front frame 90° adducted, right, Slate Metallic without decal for the option TAG		1
1	SP1655487-0472	Front frame 90° adducted, right, Forest Green without decal for the option TAG		1
1	SP1655487-5969	Front frame 90° adducted, right, Royal blue without decal for the option TAG		1
1	SP1655487-9005	Front frame 90° adducted, right, Glossy Black without decal for the option TAG		1
2	SP1655488-0464	Front frame 90° adducted, left, Space Black with decal Bold blue		1
2	SP1655488-0465	Front frame 90° adducted, left, Anthracite Grey with decal Bold blue		1
2	SP1655488-0466	Front frame 90° adducted, left, Pure White with decal Bold blue		1
2	SP1655488-0467	Front frame 90° adducted, left, Carmine Red with decal Bold blue		1
2	SP1655488-0468	Front frame 90° adducted, left, Azure blue with decal Bold blue		1
2	SP1655489-0464	Front frame 90° adducted, left, Space Black with decal Bold dark grey		1
2	SP1655489-0465	Front frame 90° adducted, left, Anthracite Grey with decal Bold dark grey		1
2	SP1655489-0466	Front frame 90° adducted, left, Pure White with decal Bold dark grey		1
2	SP1655489-0467	Front frame 90° adducted, left, Carmine Red with decal Bold dark grey		1
2	SP1655489-0468	Front frame 90° adducted, left, Azure blue with decal Bold dark grey		1
2	SP1655490-0464	Front frame 90° adducted, left, Space Black with decal Bold gold		1
2	SP1655490-0466	Front frame 90° adducted, left, Pure White with decal Bold gold		1
2	SP1655490-0467	Front frame 90° adducted, left, Carmine Red with decal Bold gold		1
2	SP1655490-0468	Front frame 90° adducted, left, Azure blue with decal Bold gold		1
2	SP1655491-0464	Front frame 90° adducted, left, Space Black with decal Bold light grey		1
2	SP1655491-0465	Front frame 90° adducted, left, Anthracite Grey with decal Bold light grey		1
2	SP1655491-0466	Front frame 90° adducted, left, Pure White with decal Bold light grey		1
2	SP1655491-0467	Front frame 90° adducted, left, Carmine Red with decal Bold light grey		1
2	SP1655491-0468	Front frame 90° adducted, left, Azure blue with decal Bold light grey		1
2	SP1655492-0464	Front frame 90° adducted, left, Space Black with decal BOLD red		1
2	SP1655492-0465	Front frame 90° adducted, left, Anthracite Grey with decal BOLD red		1
2	SP1655492-0466	Front frame 90° adducted, left, Pure White with decal BOLD red		1
2	SP1655492-0467	Front frame 90° adducted, left, Carmine Red with decal BOLD red		1
2	SP1655492-0468	Front frame 90° adducted, left, Azure blue with decal BOLD red		1
2	SP1655493-0464	Front frame 90° adducted, left, Space Black with decal Bold black		1
2	SP1655493-0465	Front frame 90° adducted, left, Anthracite Grey with decal Bold black		1
2	SP1655493-0466	Front frame 90° adducted, left, Pure White with decal Bold black		1
2	SP1655493-0467	Front frame 90° adducted, left, Carmine Red with decal Bold black		1
2	SP1655493-0468	Front frame 90° adducted, left, Azure blue with decal Bold black		1

Frame Aluminium 90° adducted

Pos.	Part number	Description	Valid from → until	Qty
2	SP1655494-0185	Front frame 90° adducted, left, dynamit orange with decal TAG red		1
2	SP1655494-0419	Front frame 90° adducted, left, Silver Grey with decal TAG red		1
2	SP1655494-0450	Front frame 90° adducted, left, Smokey Grey with decal TAG red		1
2	SP1655494-0464	Front frame 90° adducted, left, Space Black with decal TAG red		1
2	SP1655494-0465	Front frame 90° adducted, left, Anthracite Grey with decal TAG red		1
2	SP1655494-0466	Front frame 90° adducted, left, Pure White with decal TAG red		1
2	SP1655494-0467	Front frame 90° adducted, left, Carmine Red with decal TAG red		1
2	SP1655494-0468	Front frame 90° adducted, left, Azure blue with decal TAG red		1
2	SP1655494-0469	Front frame 90° adducted, left, Ruby Red with decal TAG red		1
2	SP1655494-0471	Front frame 90° adducted, left, Slate Metallic with decal TAG red		1
2	SP1655494-0472	Front frame 90° adducted, left, Forest Green with decal TAG red		1
2	SP1655494-5969	Front frame 90° adducted, left, Royal blue with decal TAG red		1
2	SP1655494-9005	Front frame 90° adducted, left, Glossy Black with decal TAG red		1
2	SP1655495-0185	Front frame 90° adducted, left, dynamit orange with decal TAG black		1
2	SP1655495-0419	Front frame 90° adducted, left, Silver Grey with decal TAG black		1
2	SP1655495-0450	Front frame 90° adducted, left, Smokey Grey with decal TAG black		1
2	SP1655495-0464	Front frame 90° adducted, left, Space Black with decal TAG black		1
2	SP1655495-0465	Front frame 90° adducted, left, Anthracite Grey with decal TAG black		1
2	SP1655495-0466	Front frame 90° adducted, left, Pure White with decal TAG black		1
2	SP1655495-0467	Front frame 90° adducted, left, Carmine Red with decal TAG black		1
2	SP1655495-0468	Front frame 90° adducted, left, Azure blue with decal TAG black		1
2	SP1655495-0469	Front frame 90° adducted, left, Ruby Red with decal TAG black		1
2	SP1655495-0470	Front frame 90° adducted, left, ferrite with decal TAG black		1
2	SP1655495-0471	Front frame 90° adducted, left, Slate Metallic with decal TAG black		1
2	SP1655495-0472	Front frame 90° adducted, left, Forest Green with decal TAG black		1
2	SP1655495-5969	Front frame 90° adducted, left, Royal blue with decal TAG black		1
2	SP1655495-9005	Front frame 90° adducted, left, Glossy Black with decal TAG black		1
3	SP1659570	Front frame protectors, pair		1

Configuration table for frame Aluminium 90° adducted

i	Rear Wheel 24"				Rear Wheel 25"				Rear Wheel 26"						
	Frame Aluminium Adducted 90° (308,5 mm)	FSTF 48	LLL 29 -> LLL 36	LLL 39 -> LLL 44	FSTF 49	LLL 29 -> LLL 36	LLL 39 -> LLL 45	FSTF 50	LLL 29 -> LLL 36	LLL 39 -> LLL 46	FSTF 51	LLL 29 -> LLL 37	LLL 40 -> LLL 47	FSTF 52	LLL 29 -> LLL 38

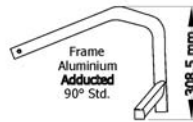

ISPS1665366

Pos.	Part number	Description	Valid from → until	Qty
i	ISPDINFC1-0037	Type of frame in relation to footrest, knee to heel length (UL), front seat height (SHv) and rear wheel size		1

Axle tube Aluminium



	In combination with:												
				SW32	SW34	SW36	SW38	SW40	SW42	SW44	SW46	SW48	SW50
0°	✓	✓	✓	1	2	3	4	5	6	7	8	9	10
3°	✓	✗	✗	12	13	14	15	16	17	18	19	20	21
	✗	✓	✓	13	14	15	16	17	18	19	20	21	22
6°	✓	✗	✗	24	25	26	27	28	29	30	31	32	33
	✗	✓	✓	25	26	27	28	29	30	31	32	33	34

	In combination with:													
					SW32	SW34	SW36	SW38	SW40	SW42	SW44	SW46	SW48	SW50
0°	✓	✓	0°/Vario		1	2	3	4	5	6	7	8	9	10
	✓	✗	3°		1	2	3	4	5	6	7	8	9	10
	✗	✓	3°		2	3	4	5	6	7	8	9	10	11
3°	✓	✗	0°/3°/Vario		12	13	14	15	16	17	18	19	20	21
	✗	✓	0°/3°/Vario		13	14	15	16	17	18	19	20	21	22
	✓	✗	6°		13	14	15	16	17	18	19	20	21	22
6°	✗	✓	6°		14	15	16	17	18	19	20	21	22	23
	✓	✗	3°/6°		24	25	26	27	28	29	30	31	32	33
	✗	✓	3°/6°		25	26	27	28	29	30	31	32	33	34

	In combination with:												
				SW32	SW34	SW36	SW38	SW40	SW42	SW44	SW46	SW48	SW50
0°	✓	✗	✗	1	2	3	4	5	6	7	8	9	10
	✗	✓	✓	2	3	4	5	6	7	8	9	10	11
3°	✓	✗	✗	13	14	15	16	17	18	19	20	21	22
	✗	✓	✓	14	15	16	17	18	19	20	21	22	23
6°	✓	✗	✗	25	26	27	28	29	30	31	32	33	34
	✗	✓	✓	26	27	28	29	30	31	32	33	34	34


ISPS1665308

Pos.	Part number	Description	Valid from → until	Qty
1	1545826	Axle tube Aluminium 0° complete, 298mm		1
2	1545827	Axle tube Aluminium 0° complete, 318mm		1
3	1545828	Axle tube Aluminium 0° complete, 338mm		1
4	1545829	Axle tube Aluminium 0° complete, 358mm		1
5	1545830	Axle tube Aluminium 0° complete, 378mm		1
6	1545831	Axle tube Aluminium 0° complete, 398mm		1
7	1545832	Axle tube Aluminium 0° complete, 418mm		1
8	1545833	Axle tube Aluminium 0° complete, 438mm		1
9	1545834	Axle tube Aluminium 0° complete, 458mm		1
10	1545835	Axle tube Aluminium 0° complete, 478mm		1
11	SP1638090	Axle tube Aluminium 0° complete, 498mm		1
12	SP1638091	Axle tube Aluminium 3° complete, 304 mm		1
13	1545836	Axle tube Aluminium 3° complete, 324mm		1
14	1545837	Axle tube Aluminium 3° complete, 344mm		1
15	1545838	Axle tube Aluminium 3° complete, 364mm		1
16	1545839	Axle tube Aluminium 3° complete, 384mm		1
17	1545840	Axle tube Aluminium 3° complete, 404mm		1
18	1545841	Axle tube Aluminium 3° complete, 424mm		1
19	1545842	Axle tube Aluminium 3° complete, 444mm		1
20	1545843	Axle tube Aluminium 3° complete, 464mm		1
21	1545844	Axle tube Aluminium 3° complete, 484mm		1
22	1545845	Axle tube Aluminium 3° complete, 504mm		1
23	SP1638092	Axle tube Aluminium 3° complete, 524 mm		1
24	SP1638093	Axle tube Aluminium 6° complete, 334 mm		1
25	1545846	Axle tube Aluminium 6° complete, 354mm		1
26	1545847	Axle tube Aluminium 6° complete, 374mm		1
27	1545848	Axle tube Aluminium 6° complete, 394mm		1
28	1545849	Axle tube Aluminium 6° complete, 414mm		1
29	1545850	Axle tube Aluminium 6° complete, 434mm		1
30	1545851	Axle tube Aluminium 6° complete, 454mm		1
31	1545852	Axle tube Aluminium 6° complete, 474mm		1

Axle tube Aluminium

Pos.	Part number	Description	Valid from → until	Qty
32	1545853	Axle tube Aluminium 6° complete, 494mm		1
33	1545854	Axle tube Aluminium 6° complete, 514mm		1
34	SP1638094	Axle tube Aluminium 6° complete, 534 mm		1

Axle tube Carbon

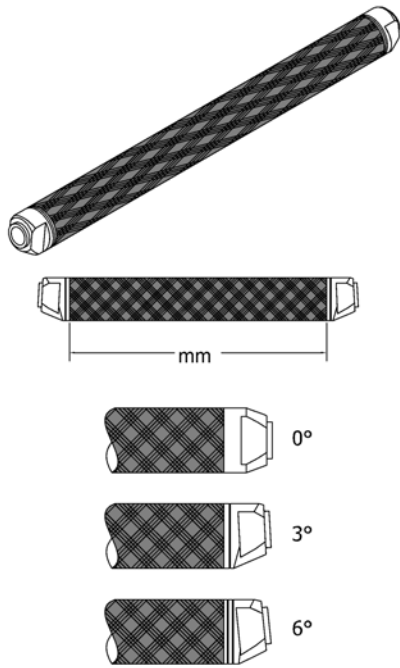

	In combination with:													
				SW32	SW34	SW36	SW38	SW40	SW42	SW44	SW46	SW48	SW50	
0°	✓	✓		1	2	3	4	5	6	7	8	9	10	
3°	✓	✗		11	12	13	14	15	16	17	18	19	20	
	✗	✓		12	13	14	15	16	17	18	19	20	21	
6°	✓	✗		22	23	24	25	26	27	28	29	30	31	
	✗	✓		23	24	25	26	27	28	29	30	31	32	

	not possible
--	--------------

	In combination with:													
				SW32	SW34	SW36	SW38	SW40	SW42	SW44	SW46	SW48	SW50	
0°	✓	✗		1	2	3	4	5	6	7	8	9	10	
	✗	✓		2	3	4	5	6	7	8	9	10	33*	
3°	✓	✗		12	13	14	15	16	17	18	19	20	21	
	✗	✓		13	14	15	16	17	18	19	20	21	34*	
6°	✓	✗		23	24	25	26	27	28	29	30	31	32	
	✗	✓		24	25	26	27	28	29	30	31	32	32	

*Aluminium


ISPS1665309

Pos.	Part number	Description	Valid from → until	Qty
1	1507564	Axle tube Carbon 0° complete, 298mm		1
2	1507565	Axle tube Carbon 0° complete, 318mm		1
3	1507566	Axle tube Carbon 0° complete, 338mm		1
4	1507567	Axle tube Carbon 0° complete, 358mm		1
5	1507568	Axle tube Carbon 0° complete, 378mm		1
6	1507569	Axle tube Carbon 0° complete, 398mm		1
7	1507570	Axle tube Carbon 0° complete, 418mm		1
8	1507571	Axle tube Carbon 0° complete, 438mm		1
9	1507572	Axle tube Carbon 0° complete, 458mm		1
10	SP1638097	Axle tube Carbon 0° complete, 478mm		1
11	SP1638111	Axle tube Carbon 3° complete, 304mm		1
12	1507575	Axle tube Carbon 3° complete, 324mm		1
13	1507576	Axle tube Carbon 3° complete, 344mm		1
14	1507577	Axle tube Carbon 3° complete, 364mm		1
15	1507578	Axle tube Carbon 3° complete, 384mm		1
16	1507579	Axle tube Carbon 3° complete, 404mm		1
17	1507580	Axle tube Carbon 3° complete, 424mm		1
18	1507581	Axle tube Carbon 3° complete, 444mm		1
19	1507582	Axle tube Carbon 3° complete, 464mm		1
20	1507583	Axle tube Carbon 3° complete, 484mm		1
21	SP1638112	Axle tube Carbon 3° complete, 504mm		1
22	SP1638113	Axle tube Carbon 6° complete, 334mm		1
23	1507585	Axle tube Carbon 6° complete, 354mm		1
24	1507586	Axle tube Carbon 6° complete, 374mm		1
25	1507587	Axle tube Carbon 6° complete, 394mm		1
26	1507588	Axle tube Carbon 6° complete, 414mm		1
27	1507589	Axle tube Carbon 6° complete, 434mm		1
28	1507590	Axle tube Carbon 6° complete, 454mm		1
29	1507591	Axle tube Carbon 6° complete, 474mm		1
30	1507592	Axle tube Carbon 6° complete, 494mm		1
31	1507593	Axle tube Carbon 6° complete, 514mm		1

Axle tube Carbon

Pos.	Part number	Description	Valid from → until	Qty
32	SP1638114	Axle tube Carbon 6° complete, 534mm		1
33	SP1638090	Axle tube Aluminium 0° complete, 498mm		1
34	SP1638092	Axle tube Aluminium 3° complete, 524 mm		1

Vario-Ax

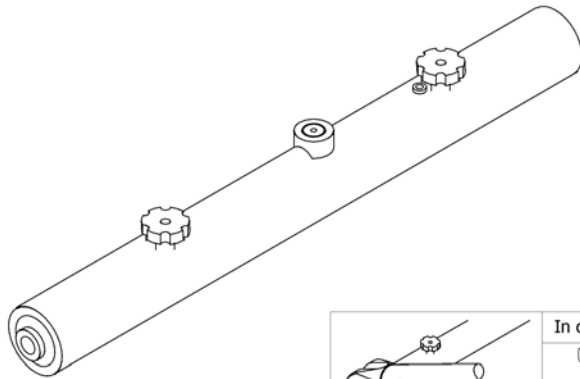


In combination with:		SW32	SW34	SW36	SW38	SW40	SW42	SW44	SW46	SW48	SW50
				1	2	3	4	5	6	7	8

In combination with:		SW32	SW34	SW36	SW38	SW40	SW42	SW44	SW46	SW48	SW50	
✓	✓	0°/Vario	1	2	3	4	5	6	7	8	9	10
✓	✗	3°	1	2	3	4	5	6	7	8	9	10
✗	✓	3°	2	3	4	5	6	7	8	9	10	11

ISPS1665310

Pos.	Part number	Description	Valid from → until	Qty
1	1507609	Vario-Ax complete 335mm		1
2	1507611	Vario-Ax complete 355mm		1
3	1507612	Vario-Ax complete 375mm		1
4	1507613	Vario-Ax complete 395mm		1
5	1507615	Vario-Ax complete 415mm		1
6	1507616	Vario-Ax complete 435mm		1
7	1507617	Vario-Ax complete 455mm		1
8	1507619	Vario-Ax complete 475mm		1
9	1507620	Vario-Ax complete 495mm		1
10	1507621	Vario-Ax complete 515mm		1
11	1507622	Vario-Ax complete 535mm		1


Axle holder

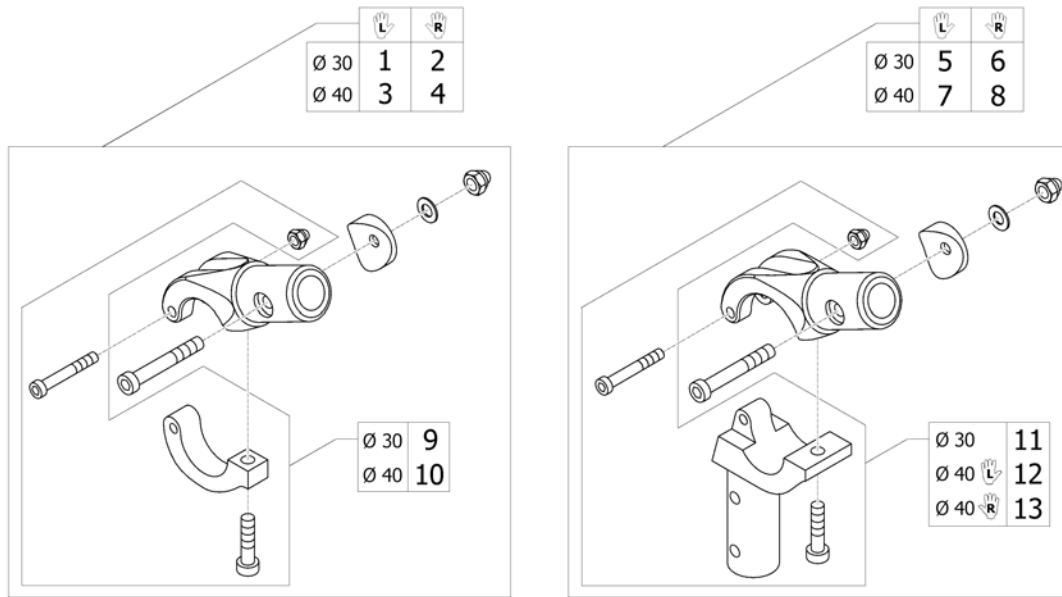

ISPS1665311

Pos.	Part number	Description	Valid from → until	Qty
1	SP1638139	Axle holder complete Ø30, left		1
2	SP1638140	Axle holder complete Ø30, right		1
3	SP1638141	Axle holder complete Ø40, left		1
4	SP1638142	Axle holder complete Ø40, right		1
5	SP1638143	Axle holder for adaptations Ø30, left		1
6	SP1638144	Axle holder for adaptations Ø30, right		1
7	SP1638145	Axle holder for adaptations Ø40, left		1
8	SP1638146	Axle holder for adaptations Ø40, right		1
9	1545821	Axle holder latch Ø30		1
10	1545822	Axle holder latch Ø40		1
11	1545823	Axle holder latch for Adaptations, Ø30		1
12	1545825	Axle holder latch for Adaptations left, Ø40		1
13	1545824	Axle holder latch for Adaptations right, Ø40		1


Rear wheel extension standard

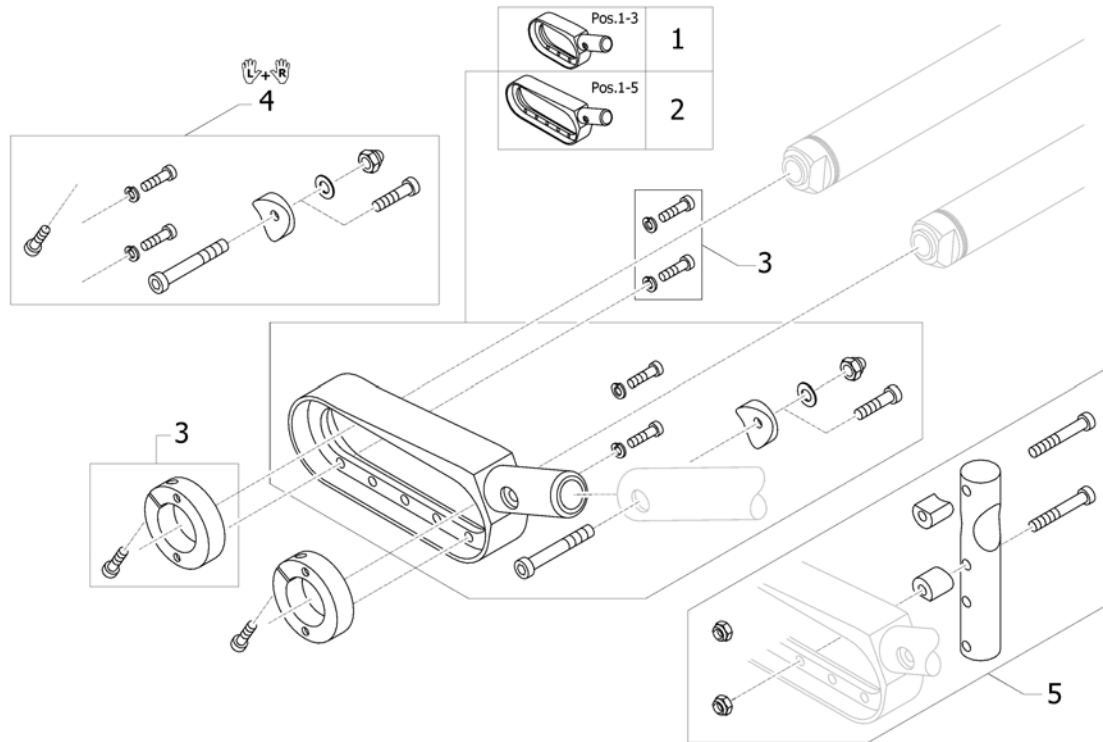

ISPS1665312

Pos.	Part number	Description	Valid from → until	Qty
1	SP1639351	Rear wheel extension short (Pos.1-3) standard complete		1
2	SP1639352	Rear wheel extension standard, black, complete		1
3	1553205	Clamping ring Ø50/30, black, incl. screw kit		1
4	1545860	Assembly kit for pair rear wheel extension		1
5	1545861	Adaptation holder, kit for rear wheel extension standard		1


Rear wheel extension for Vario-Ax

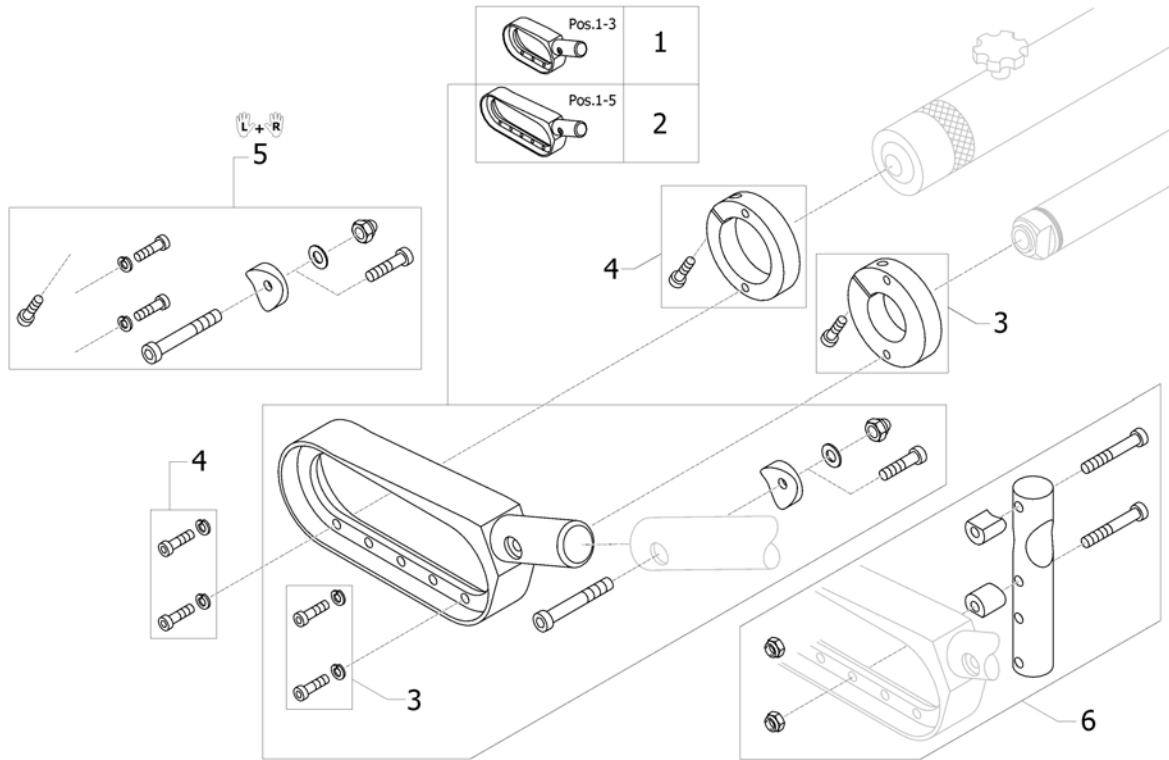

ISPS1665313

Pos.	Part number	Description	Valid from → until	Qty
1	SP1639353	Rear wheel extension (Pos.1-3) Vario-Ax complete		1
2	SP1639354	Rear wheel extension Vario-Ax, black, complete		1
3	1553207	Clamping ring Ø60/30, black, incl. screw kit		2
4	1553246	Clamping ring Ø60/40, black, incl. screw kit		2
5	1545860	Assembly kit for pair rear wheel extension		1
6	1545862	Adaptation holder, kit for rear wheel extension Vario-Ax		2

Seat module

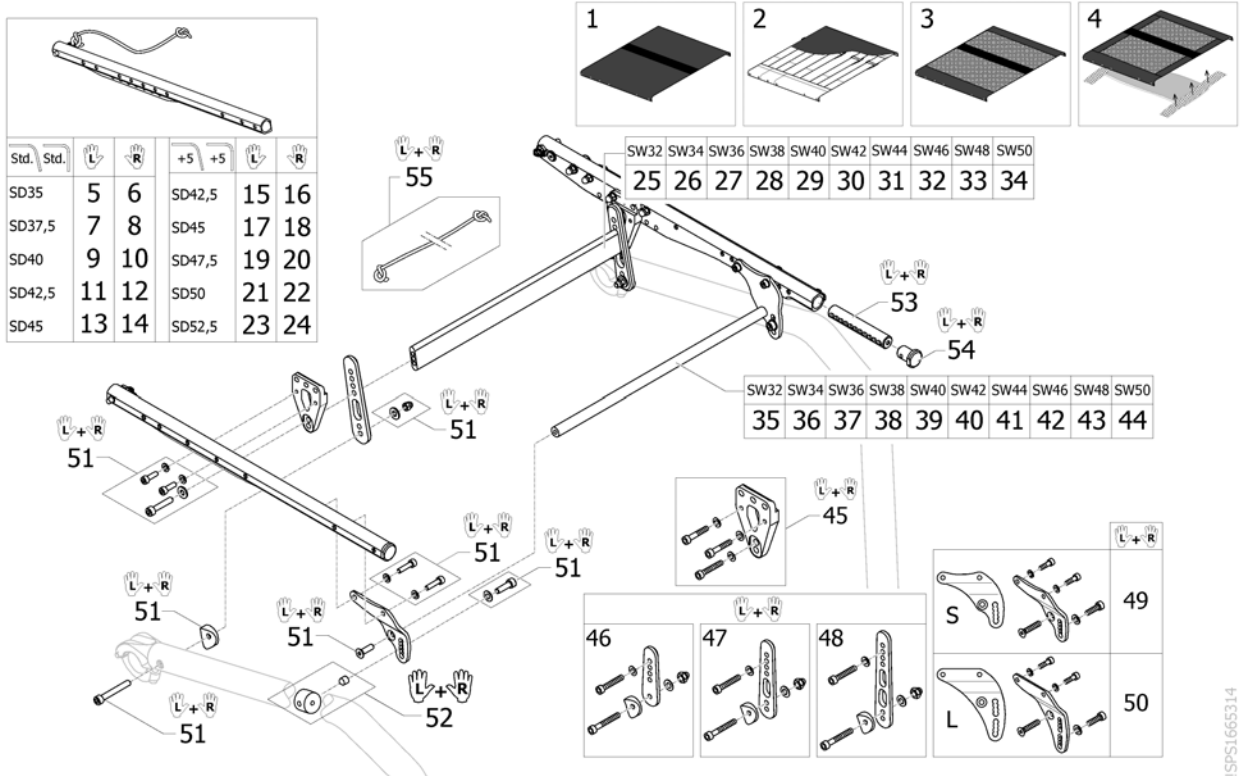


ISPS1665314

Pos.	Part number	Description	Valid from → until	Qty
1	-	Seatcover standard		1
2	-	Seatcover adjustable		1
3	-	Seatcover light		1
4	-	Seatcover light for seat upholstery bag		1
5	SP1642235	Seat frame rail ST35, left		1
6	SP1642236	Seat frame rail ST35, right		1
7	SP1642237	Seat frame rail ST37,5, left		1
8	SP1642238	Seat frame rail ST37,5, right		1
9	SP1642239	Seat frame rail ST40, left		1
10	SP1642240	Seat frame rail ST40, right		1
11	SP1642241	Seat frame rail ST42,5, left		1
12	SP1642242	Seat frame rail ST42,5, right		1
13	SP1642243	Seat frame rail ST45, left		1
14	SP1642244	Seat frame rail ST45, right		1
15	SP1642245	Seat frame rail for frame+5 ST42,5, links		1
16	SP1642246	Seat frame rail for frame+5 ST42,5, right		1
17	SP1642247	Seat frame rail for frame+5 ST45, links		1
18	SP1642248	Seat frame rail for frame+5 ST45, right		1
19	SP1642249	Seat frame rail for frame+5 ST47,5, links		1
20	SP1642250	Seat frame rail for frame+5 ST47,5, right		1
21	SP1642251	Seat frame rail for frame+5 ST50, links		1
22	SP1642252	Seat frame rail for frame+5 ST50, right		1
23	SP1642253	Seat frame rail for frame+5 ST52,5, links		1
24	SP1642254	Seat frame rail for frame+5 ST52,5, right		1
25	SP1642255	Seat brace horizontal rear, SB32		1
26	SP1642256	Seat brace horizontal rear, SB34		1
27	SP1642257	Seat brace horizontal rear, SB36		1
28	SP1642258	Seat brace horizontal rear, SB38		1
29	SP1642259	Seat brace horizontal rear, SB40		1
30	SP1642260	Seat brace horizontal rear, SB42		1
31	SP1642261	Seat brace horizontal rear, SB44		1

Seat module

Pos.	Part number	Description	Valid from → until	Qty
32	SP1642262	Seat brace horizontal rear, SB46		1
33	SP1642263	Seat brace horizontal rear, SB48		1
34	SP1642264	Seat brace horizontal rear, SB50		1
35	SP1642265	Seat brace horizontal, SB32		1
36	SP1642266	Seat brace horizontal, SB34		1
37	SP1642267	Seat brace horizontal, SB36		1
38	SP1642268	Seat brace horizontal, SB38		1
39	SP1642269	Seat brace horizontal, SB40		1
40	SP1642270	Seat brace horizontal, SB42		1
41	SP1642271	Seat brace horizontal, SB44		1
42	SP1642272	Seat brace horizontal, SB46		1
43	SP1642273	Seat brace horizontal, SB48		1
44	SP1642274	Seat brace horizontal, SB50		1
45	SP1642275	Hinge bracket-Set Seat Frame, rear, pair		1
46	SP1642276	Seat brace S set, pair		1
47	SP1642277	Seat brace M set, pair		1
48	SP1642278	Seat brace L set, pair		1
49	SP1642279	Hinge bracket S Set Front Seat Frame, pair		1
50	SP1642280	Hinge bracket L Set Front Seat Frame, pair		1
51	SP1642281	Fixation kit to hinge brackets		1
52	SP1642282	Treadet sleeve to hinge brackets, pair		1
53	SP1642283	Treadet sleeve to seat frame rail, pair		1
54	1486539-0004	Seatmodule plug, pair, black anodized		1
54	1486539-0001	Seatmodule plug, pair, chrome		1
54	1486539-0182	Seatmodule plug, pair, champagne		1
54	1486539-0183	Seatmodule plug, pair, olive		1
54	1486539-0400	Seatmodule plug, pair, techno-anthrazit		1
54	1486539-0404	Seatmodule plug, pair, b-blue		1
54	1486539-0440	Seatmodule plug, pair, comet red		1
54	1486539-0441	Seatmodule plug, pair, arctic blue		1
54	1486539-0444	Seatmodule plug, pair, copper glaze		1
55	1554427	Cord for backrest foldable		1

Seatcover standard

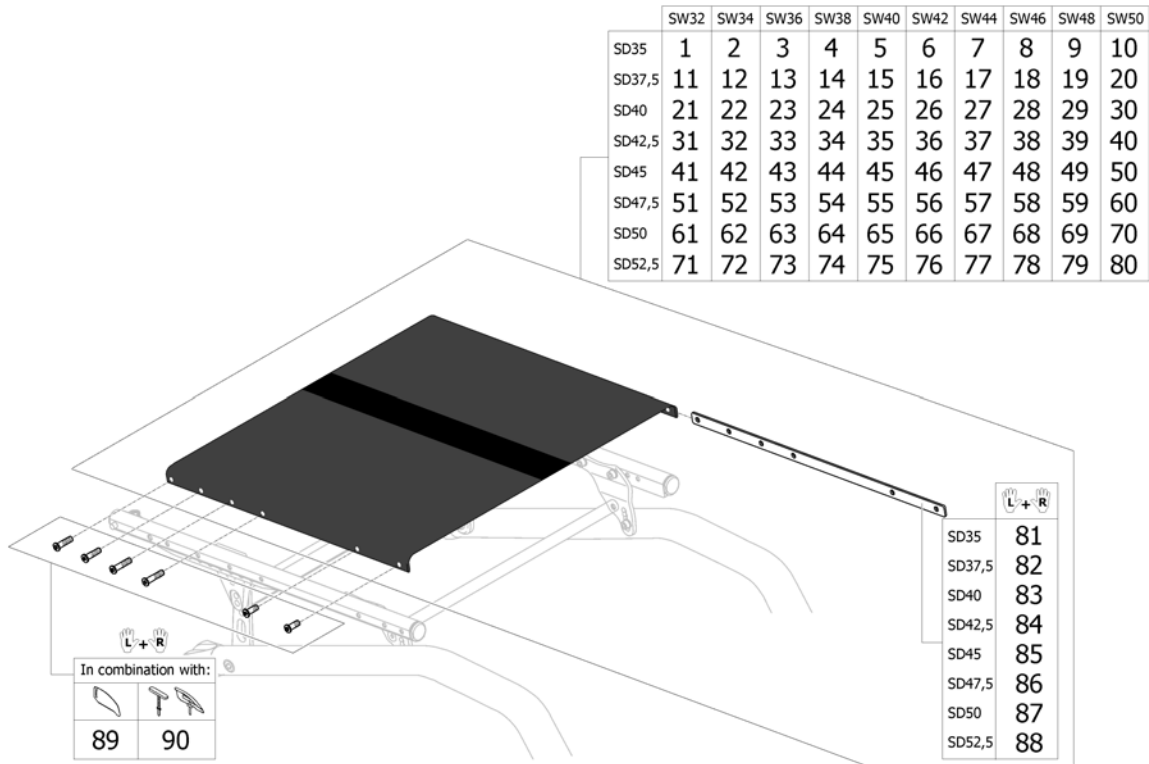

ISPS1665367

Pos.	Part number	Description	Valid from → until	Qty
1	SP1607536	Seatcover standard complete SB32 ST35 for frame standard		1
2	SP1607537	Seatcover standard complete SB34 ST35 for frame standard		1
3	SP1607538	Seatcover standard complete SB36 ST35 for frame standard		1
4	SP1607539	Seatcover standard complete SB38 ST35 for frame standard		1
5	SP1607540	Seatcover standard complete SB40 ST35 for frame standard		1
6	SP1607541	Seatcover standard complete SB42 ST35 for frame standard		1
7	SP1607542	Seatcover standard complete SB44 ST35 for frame standard		1
8	SP1607543	Seatcover standard complete SB46 ST35 for frame standard		1
9	SP1607544	Seatcover standard complete SB48 ST35 for frame standard		1
10	SP1607545	Seatcover standard complete SB50 ST35 for frame standard		1
11	SP1607639	Seatcover standard complete SB32 ST37,5 for frame standard		1
12	1546307	Seatcover standard complete SB34 ST37,5 for frame standard		1
13	1546308	Seatcover standard complete SB36 ST37,5 for frame standard		1
14	1546309	Seatcover standard complete SB38 ST37,5 for frame standard		1
15	1546310	Seatcover standard complete SB40 ST37,5 for frame standard		1
16	1546344	Seatcover standard complete SB42 ST37,5 for frame standard		1
17	1546345	Seatcover standard complete SB44 ST37,5 for frame standard		1
18	1546346	Seatcover standard complete SB46 ST37,5 for frame standard		1
19	1546347	Seatcover standard complete SB48 ST37,5 for frame standard		1
20	SP1607663	Seatcover standard complete SB50 ST37,5 for frame standard		1
21	SP1607657	Seatcover standard complete SB32 ST40 for frame standard		1
22	1546348	Seatcover standard complete SB34 ST40 for frame standard		1
23	1546349	Seatcover standard complete SB36 ST40 for frame standard		1
24	1546350	Seatcover standard complete SB38 ST40 for frame standard		1
25	1546351	Seatcover standard complete SB40 ST40 for frame standard		1
26	1546352	Seatcover standard complete SB42 ST40 for frame standard		1
27	1546353	Seatcover standard complete SB44 ST40 for frame standard		1
28	1546354	Seatcover standard complete SB46 ST40 for frame standard		1
29	1546355	Seatcover standard complete SB48 ST40 for frame standard		1
30	SP1607664	Seatcover standard complete SB50 ST40 for frame standard		1
31	SP1607658	Seatcover standard complete SB32 ST42,5 for frame standard/+5		1


Seatcover standard

Pos.	Part number	Description	Valid from → until	Qty
32	1546356	Seatcover standard complete SB34 ST42,5 for frame standard/+5		1
33	1546357	Seatcover standard complete SB36 ST42,5 for frame standard/+5		1
34	1546358	Seatcover standard complete SB38 ST42,5 for frame standard/+5		1
35	1546359	Seatcover standard complete SB40 ST42,5 for frame standard/+5		1
36	1546360	Seatcover standard complete SB42 ST42,5 for frame standard/+5		1
37	1546361	Seatcover standard complete SB44 ST42,5 for frame standard/+5		1
38	1546362	Seatcover standard complete SB46 ST42,5 for frame standard/+5		1
39	1546363	Seatcover standard complete SB48 ST42,5 for frame standard/+5		1
40	SP1607665	Seatcover standard complete SB50 ST42,5 for frame standard/+5		1
41	SP1607659	Seatcover standard complete SB32 ST45 for frame standard/+5		1
42	1546364	Seatcover standard complete SB34 ST45 for frame standard/+5		1
43	1546365	Seatcover standard complete SB36 ST45 for frame standard/+5		1
44	1546366	Seatcover standard complete SB38 ST45 for frame standard/+5		1
45	1546367	Seatcover standard complete SB40 ST45 for frame standard/+5		1
46	1546368	Seatcover standard complete SB42 ST45 for frame standard/+5		1
47	1546369	Seatcover standard complete SB44 ST45 for frame standard/+5		1
48	1546370	Seatcover standard complete SB46 ST45 for frame standard/+5		1
49	1546371	Seatcover standard complete SB48 ST45 for frame standard/+5		1
50	SP1607666	Seatcover standard complete SB50 ST45 for frame standard/+5		1
51	SP1607660	Seatcover standard complete SB32 ST47,5 for frame +5		1
52	1546372	Seatcover standard complete SB34 ST47,5 for frame +5		1
53	1546373	Seatcover standard complete SB36 ST47,5 for frame +5		1
54	1546374	Seatcover standard complete SB38 ST47,5 for frame +5		1
55	1546375	Seatcover standard complete SB40 ST47,5 for frame +5		1
56	1546376	Seatcover standard complete SB42 ST47,5 for frame +5		1
57	1546377	Seatcover standard complete SB44 ST47,5 for frame +5		1
58	1546378	Seatcover standard complete SB46 ST47,5 for frame +5		1
59	1546379	Seatcover standard complete SB48 ST47,5 for frame +5		1
60	SP1607667	Seatcover standard complete SB50 ST47,5 for frame +5		1
61	SP1607661	Seatcover standard complete SB32 ST50 for frame +5		1
62	1546380	Seatcover standard complete SB34 ST50 for frame +5		1
63	1546381	Seatcover standard complete SB36 ST50 for frame +5		1
64	1546382	Seatcover standard complete SB38 ST50 for frame +5		1
65	1546383	Seatcover standard complete SB40 ST50 for frame +5		1
66	1546384	Seatcover standard complete SB42 ST50 for frame +5		1
67	1546385	Seatcover standard complete SB44 ST50 for frame +5		1
68	1546386	Seatcover standard complete SB46 ST50 for frame +5		1
69	1546387	Seatcover standard complete SB48 ST50 for frame +5		1
70	SP1607668	Seatcover standard complete SB50 ST50 for frame +5		1
71	SP1607662	Seatcover standard complete SB32 ST52,5 for frame +5		1
72	1546388	Seatcover standard complete SB34 ST52,5 for frame +5		1
73	1546389	Seatcover standard complete SB36 ST52,5 for frame +5		1
74	1546390	Seatcover standard complete SB38 ST52,5 for frame +5		1
75	1546391	Seatcover standard complete SB40 ST52,5 for frame +5		1
76	1546392	Seatcover standard complete SB42 ST52,5 for frame +5		1
77	1546393	Seatcover standard complete SB44 ST52,5 for frame +5		1
78	1546394	Seatcover standard complete SB46 ST52,5 for frame +5		1
79	1546395	Seatcover standard complete SB48 ST52,5 for frame +5		1
80	SP1607669	Seatcover standard complete SB50 ST52,5 for frame +5		1
81	SP1642299	Seat rail kit SD35, pair		1
82	1579366	Seat rail kit SD37,5, pair		1
83	1579367	Seat rail kit SD40, pair		1
84	1579368	Seat rail kit SD42,5, pair		1
85	1579369	Seat rail kit SD45, pair		1
86	1579370	Seat rail kit SD47,5, pair		1

Seatcover standard

Pos.	Part number	Description	Valid from → until	Qty
87	1579371	Seat rail kit SD50, pair		1
88	1579372	Seat rail kit SD52,5, pair		1
89	SP1642300	Fixation kit seat cover for clothes-guard		1
90	SP1642301	Fixation kit seat cover for mudguard or siderest height adjustable		1

Seatcover adjustable

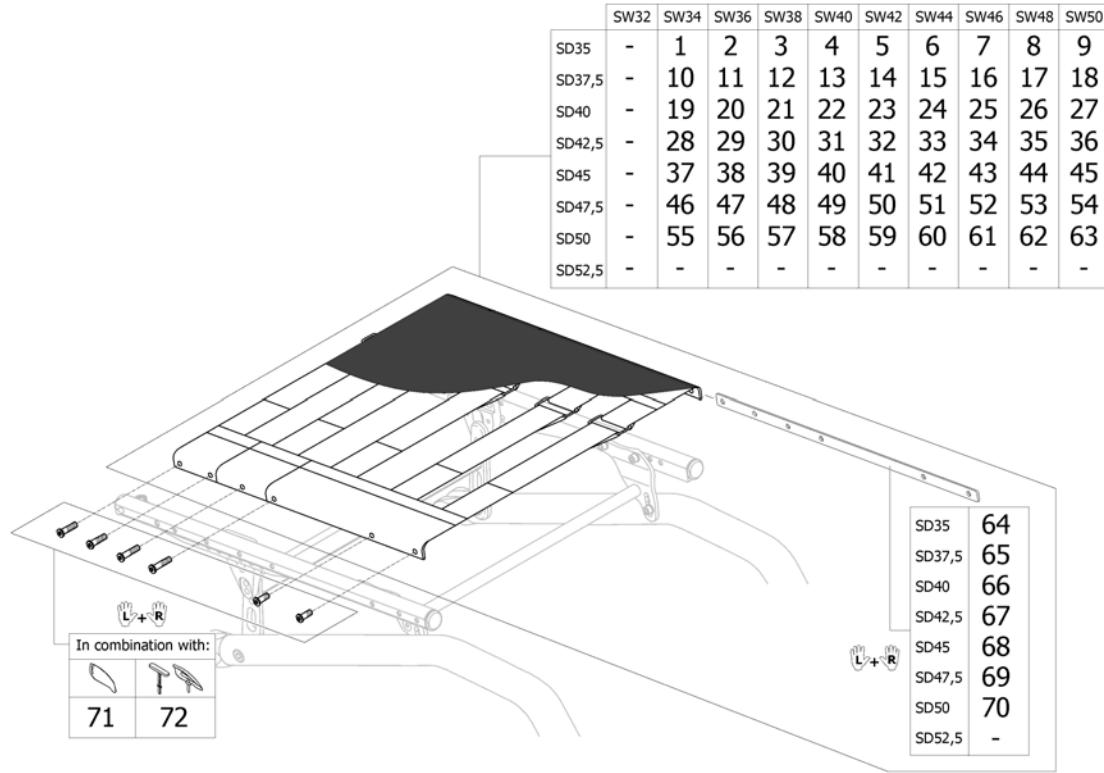


Pos.	Part number	Description	Valid from → until	Qty
1	SP1642284	Seatcover adjustable complete SB34 ST35 for frame standard		1
2	SP1642285	Seatcover adjustable complete SB36 ST35 for frame standard		1
3	SP1642286	Seatcover adjustable complete SB38 ST35 for frame standard		1
4	SP1642287	Seatcover adjustable complete SB40 ST35 for frame standard		1
5	SP1642288	Seatcover adjustable complete SB42 ST35 for frame standard		1
6	SP1642289	Seatcover adjustable complete SB44 ST35 for frame standard		1
7	SP1642290	Seatcover adjustable complete SB46 ST35 for frame standard		1
8	SP1642291	Seatcover adjustable complete SB48 ST35 for frame standard		1
9	SP1642292	Seatcover adjustable complete SB50 ST35 for frame standard		1
10	1546396	Seatcover adjustable complete SB34 ST37,5 for frame standard		1
11	1546397	Seatcover adjustable complete SB36 ST37,5 for frame standard		1
12	1546398	Seatcover adjustable complete SB38 ST37,5 for frame standard		1
13	1546399	Seatcover adjustable complete SB40 ST37,5 for frame standard		1
14	1546400	Seatcover adjustable complete SB42 ST37,5 for frame standard		1
15	1546401	Seatcover adjustable complete SB44 ST37,5 for frame standard		1
16	1546402	Seatcover adjustable complete SB46 ST37,5 for frame standard		1
17	1546403	Seatcover adjustable complete SB48 ST37,5 for frame standard		1
18	SP1642293	Seatcover adjustable complete SB50 ST37,5 for frame standard		1
19	1546455	Seatcover adjustable complete SB34 ST40 for frame standard		1
20	1546456	Seatcover adjustable complete SB36 ST40 for frame standard		1
21	1546457	Seatcover adjustable complete SB38 ST40 for frame standard		1
22	1546458	Seatcover adjustable complete SB40 ST40 for frame standard		1
23	1546459	Seatcover adjustable complete SB42 ST40 for frame standard		1
24	1546460	Seatcover adjustable complete SB44 ST40 for frame standard		1
25	1546461	Seatcover adjustable complete SB46 ST40 for frame standard		1
26	1546462	Seatcover adjustable complete SB48 ST40 for frame standard		1
27	SP1642294	Seatcover adjustable complete SB50 ST40 for frame standard		1
28	1546463	Seatcover adjustable complete SB34 ST42,5 for frame standard/+5		1
29	1546464	Seatcover adjustable complete SB36 ST42,5 for frame standard/+5		1
30	1546465	Seatcover adjustable complete SB38 ST42,5 for frame standard/+5		1
31	1546466	Seatcover adjustable complete SB40 ST42,5 for frame standard/+5		1

Seatcover adjustable

Pos.	Part number	Description	Valid from → until	Qty
32	1546467	Seatcover adjustable complete SB42 ST42,5 for frame standard/+5		1
33	1546468	Seatcover adjustable complete SB44 ST42,5 for frame standard/+5		1
34	1546469	Seatcover adjustable complete SB46 ST42,5 for frame standard/+5		1
35	1546470	Seatcover adjustable complete SB48 ST42,5 for frame standard/+5		1
36	SP1642295	Seatcover adjustable complete SB50 ST42,5 for frame standard/+5		1
37	1546471	Seatcover adjustable complete SB34 ST45 for frame standard/+5		1
38	1546472	Seatcover adjustable complete SB36 ST45 for frame standard/+5		1
39	1546473	Seatcover adjustable complete SB38 ST45 for frame standard/+5		1
40	1546474	Seatcover adjustable complete SB40 ST45 for frame standard/+5		1
41	1546475	Seatcover adjustable complete SB42 ST45 for frame standard/+5		1
42	1546476	Seatcover adjustable complete SB44 ST45 for frame standard/+5		1
43	1546477	Seatcover adjustable complete SB46 ST45 for frame standard/+5		1
44	1546478	Seatcover adjustable complete SB48 ST45 for frame standard/+5		1
45	SP1642296	Seatcover adjustable complete SB50 ST45 for frame standard/+5		1
46	1546479	Seatcover adjustable complete SB34 ST47,5 for frame +5		1
47	1546480	Seatcover adjustable complete SB36 ST47,5 for frame +5		1
48	1546481	Seatcover adjustable complete SB38 ST47,5 for frame +5		1
49	1546482	Seatcover adjustable complete SB40 ST47,5 for frame +5		1
50	1546483	Seatcover adjustable complete SB42 ST47,5 for frame +5		1
51	1546484	Seatcover adjustable complete SB44 ST47,5 for frame +5		1
52	1546485	Seatcover adjustable complete SB46 ST47,5 for frame +5		1
53	1546486	Seatcover adjustable complete SB48 ST47,5 for frame +5		1
54	SP1642297	Seatcover adjustable complete SB50 ST47,5 for frame +5		1
55	1546487	Seatcover adjustable complete SB34 ST50 for frame +5		1
56	1546488	Seatcover adjustable complete SB36 ST50 for frame +5		1
57	1546489	Seatcover adjustable complete SB38 ST50 for frame +5		1
58	1546490	Seatcover adjustable complete SB40 ST50 for frame +5		1
59	1546491	Seatcover adjustable complete SB42 ST50 for frame +5		1
60	1546492	Seatcover adjustable complete SB44 ST50 for frame +5		1
61	1546493	Seatcover adjustable complete SB46 ST50 for frame +5		1
62	1546494	Seatcover adjustable complete SB48 ST50 for frame +5		1
63	SP1642298	Seatcover adjustable complete SB50 ST50 for frame +5		1
64	SP1642299	Seat rail kit SD35, pair		1
65	1579366	Seat rail kit SD37,5, pair		1
66	1579367	Seat rail kit SD40, pair		1
67	1579368	Seat rail kit SD42,5, pair		1
68	1579369	Seat rail kit SD45, pair		1
69	1579370	Seat rail kit SD47,5, pair		1
70	1579371	Seat rail kit SD50, pair		1
71	SP1642300	Fixation kit seat cover for clothes-guard		1
72	SP1642301	Fixation kit seat cover for mudguard or siderest height adjustable		1


Seatcover light

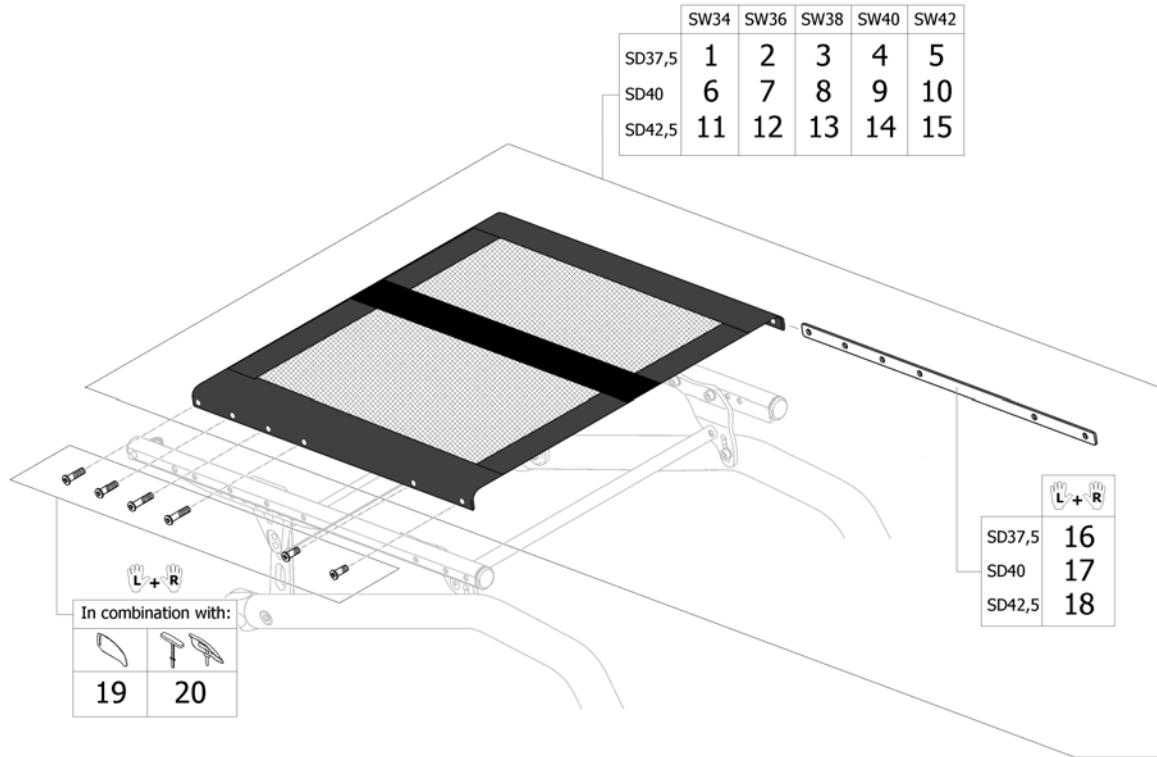

ISPS1665369

Pos.	Part number	Description	Valid from → until	Qty
1	1546503	Seatcover light complete SB34 ST37,5 for frame standard		1
2	1546504	Seatcover light complete SB36 ST37,5 for frame standard		1
3	1546505	Seatcover light complete SB38 ST37,5 for frame standard		1
4	1546506	Seatcover light complete SB40 ST37,5 for frame standard		1
5	1546507	Seatcover light complete SB42 ST37,5 for frame standard		1
6	1546511	Seatcover light complete SB34 ST40 for frame standard		1
7	1546512	Seatcover light complete SB36 ST40 for frame standard		1
8	1546513	Seatcover light complete SB38 ST40 for frame standard		1
9	1546514	Seatcover light complete SB40 ST40 for frame standard		1
10	1546515	Seatcover light complete SB42 ST40 for frame standard		1
11	1546519	Seatcover light complete SB34 ST42,5 for frame standard		1
12	1546520	Seatcover light complete SB36 ST42,5 for frame standard		1
13	1546521	Seatcover light complete SB38 ST42,5 for frame standard		1
14	1546522	Seatcover light complete SB40 ST42,5 for frame standard		1
15	1546523	Seatcover light complete SB42 ST42,5 for frame standard		1
16	1579366	Seat rail kit SD37,5, pair		1
17	1579367	Seat rail kit SD40, pair		1
18	1579368	Seat rail kit SD42,5, pair		1
19	SP1642300	Fixation kit seat cover for clothes-guard		1
20	SP1642301	Fixation kit seat cover for mudguard or siderest height adjustable		1


Seatcover light for seat upholstery bag

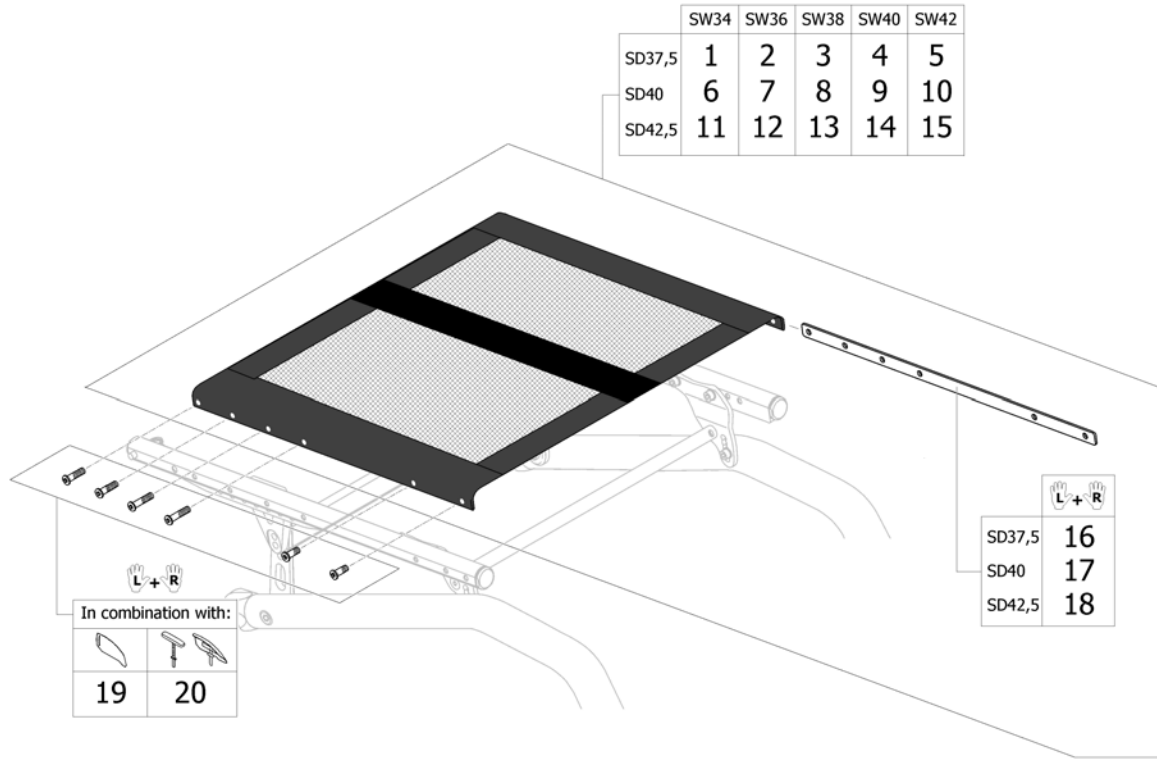

ISPS1665370

Pos.	Part number	Description	Valid from → until	Qty
1	SP1590126	Seatcover light for Seat upholstery bag complete SB34 ST37,5 for frame standard		1
2	SP1590127	Seatcover light for Seat upholstery bag complete SB36 ST37,5 for frame standard		1
3	SP1590128	Seatcover light for Seat upholstery bag complete SB38 ST37,5 for frame standard		1
4	SP1590129	Seatcover light for Seat upholstery bag complete SB40 ST37,5 for frame standard		1
5	SP1590217	Seatcover light for Seat upholstery bag complete SB42 ST37,5 for frame standard		1
6	SP1590221	Seatcover light for Seat upholstery bag complete SB34 ST40 for frame standard		1
7	SP1590222	Seatcover light for Seat upholstery bag complete SB36 ST40 for frame standard		1
8	SP1590223	Seatcover light for Seat upholstery bag complete SB38 ST40 for frame standard		1
9	SP1590224	Seatcover light for Seat upholstery bag complete SB40 ST40 for frame standard		1
10	SP1590225	Seatcover light for Seat upholstery bag complete SB42 ST40 for frame standard		1
11	SP1590269	Seatcover light for Seat upholstery bag complete SB34 ST42,5 for frame standard		1
12	SP1590270	Seatcover light for Seat upholstery bag complete SB36 ST42,5 for frame standard		1
13	SP1590271	Seatcover light for Seat upholstery bag complete SB38 ST42,5 for frame standard		1
14	SP1590304	Seatcover light for Seat upholstery bag complete SB40 ST42,5 for frame standard		1
15	SP1590305	Seatcover light for Seat upholstery bag complete SB42 ST42,5 for frame standard		1
16	1579366	Seat rail kit SD37,5, pair		1
17	1579367	Seat rail kit SD40, pair		1
18	1579368	Seat rail kit SD42,5, pair		1
19	SP1642300	Fixation kit seat cover for clothes-guard		1
20	SP1642301	Fixation kit seat cover for mudguard or siderest height adjustable		1

Seat cushion standard

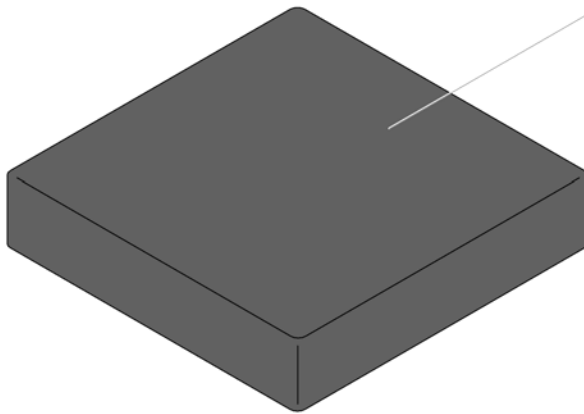

	SW32	SW34	SW36	SW38	SW40	SW42	SW44	SW46	SW48	SW50
SD35	1	2	3	4	5	6	7	8	9	10
SD37,5	11	12	4	13	14	15	16	17	18	19
SD40	20	21	5	14	22	23	24	25	26	27
SD42,5	28	29	7	16	24	30	31	32	33	34
SD45	35	36	8	17	25	37	32	38	39	40
SD47,5	41	42	9	18	26	43	33	39	44	45
SD50	46	47	10	19	27	48	34	40	45	49
SD52,5	50	51	52	53	54	55	56	57	58	59


ISPS1665315

Pos.	Part number	Description	Valid from → until	Qty
1	1556316	Seat cushion 320mm x 360mm		1
2	1556326	Seat cushion 340mm x 360mm		1
3	1556335	Seat cushion 360mm x 360mm		1
4	1556336	Seat cushion 360mm x 380mm		1
5	1556337	Seat cushion 360mm x 400mm		1
6	1556338	Seat cushion 360mm x 420mm		1
7	1556339	Seat cushion 360mm x 440mm		1
8	1556340	Seat cushion 360mm x 460mm		1
9	1556341	Seat cushion 360mm x 480mm		1
10	1556342	Seat cushion 360mm x 500mm		1
11	1556317	Seat cushion 320mm x 380mm		1
12	1556327	Seat cushion 340mm x 380mm		1
13	1556344	Seat cushion 380mm x 380mm		1
14	1556345	Seat cushion 380mm x 400mm		1
15	1556346	Seat cushion 380mm x 420mm		1
16	1556347	Seat cushion 380mm x 440mm		1
17	1556348	Seat cushion 380mm x 460mm		1
18	1556349	Seat cushion 380mm x 480mm		1
19	1556350	Seat cushion 380mm x 500mm		1
20	1556318	Seat cushion 320mm x 400mm		1
21	1556328	Seat cushion 340mm x 400mm		1
22	1556352	Seat cushion 400mm x 400mm		1
23	1556353	Seat cushion 400mm x 420mm		1
24	1556354	Seat cushion 400mm x 440mm		1
25	1556355	Seat cushion 400mm x 460mm		1
26	1556356	Seat cushion 400mm x 480mm		1
27	1556357	Seat cushion 400mm x 500mm		1
28	1556320	Seat cushion 320mm x 440mm		1
29	1556330	Seat cushion 340mm x 440mm		1
30	1556360	Seat cushion 420mm x 440mm		1
31	1556365	Seat cushion 440mm x 440mm		1

Seat cushion standard

Pos.	Part number	Description	Valid from → until	Qty
32	1556366	Seat cushion 440mm x 460mm		1
33	1556367	Seat cushion 440mm x 480mm		1
34	1556368	Seat cushion 440mm x 500mm		1
35	1556321	Seat cushion 320mm x 460mm		1
36	1556331	Seat cushion 340mm x 460mm		1
37	1556361	Seat cushion 420mm x 460mm		1
38	1556370	Seat cushion 460mm x 460mm		1
39	1556371	Seat cushion 460mm x 480mm		1
40	1556372	Seat cushion 460mm x 500mm		1
41	1556322	Seat cushion 320mm x 480mm		1
42	1556332	Seat cushion 340mm x 480mm		1
43	1556362	Seat cushion 420mm x 480mm		1
44	1556374	Seat cushion 480mm x 480mm		1
45	1556375	Seat cushion 480mm x 500mm		1
46	1556323	Seat cushion 320mm x 500mm		1
47	1556333	Seat cushion 340mm x 500mm		1
48	1556363	Seat cushion 420mm x 500mm		1
49	1556377	Seat cushion 500mm x 500mm		1
50	1556324	Seat cushion 320mm x 520mm		1
51	1556334	Seat cushion 340mm x 520mm		1
52	1556343	Seat cushion 360mm x 520mm		1
53	1556351	Seat cushion 380mm x 520mm		1
54	1556358	Seat cushion 400mm x 520mm		1
55	1556364	Seat cushion 420mm x 520mm		1
56	1556369	Seat cushion 440mm x 520mm		1
57	1556373	Seat cushion 460mm x 520mm		1
58	1556376	Seat cushion 480mm x 520mm		1
59	1556378	Seat cushion 500mm x 520mm		1

Backrest (rounded bar)

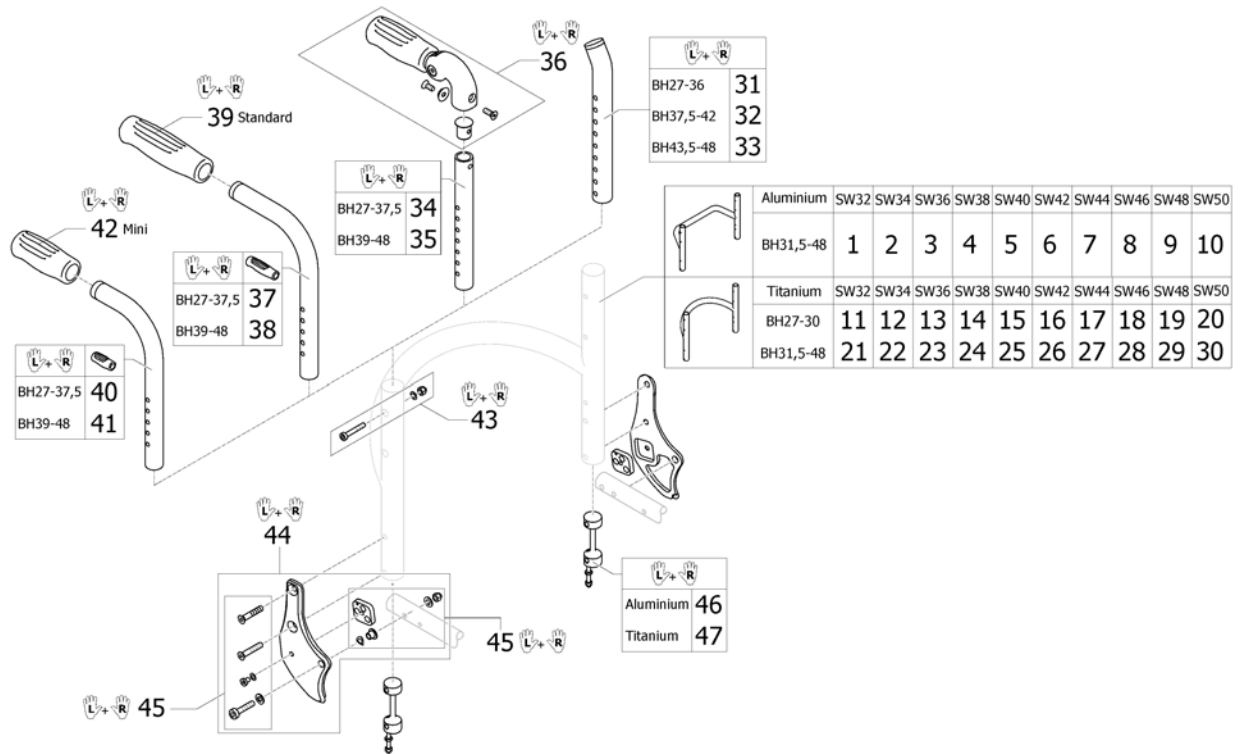


ISPS1665316

Pos.	Part number	Description	Valid from → until	Qty
1	SP1659581	Backrest tube Alu SW32 Long		1
2	SP1659582	Backrest tube Alu SW34 Long		1
3	SP1659583	Backrest tube Alu SW36 Long		1
4	SP1659584	Backrest tube Alu SW38 Long		1
5	SP1659585	Backrest tube Alu SW40 Long		1
6	SP1659586	Backrest tube Alu SW42 Long		1
7	SP1659587	Backrest tube Alu SW44 Long		1
8	SP1659588	Backrest tube Alu SW46 Long		1
9	SP1659589	Backrest tube Alu SW48 Long		1
10	SP1659590	Backrest tube Alu SW50 Long		1
11	SP1638125	Backrest tube Titan SB32 short		1
12	1546810	Backrest tube Titan SB34 short		1
13	1546811	Backrest tube Titan SB36 short		1
14	1546812	Backrest tube Titan SB38 short		1
15	1546813	Backrest tube Titan SB40 short		1
16	1546814	Backrest tube Titan SB42 short		1
17	1546815	Backrest tube Titan SB44 short		1
18	1546816	Backrest tube Titan SB46 short		1
19	1546817	Backrest tube Titan SB48 short		1
20	SP1638126	Backrest tube Titan SB50 short		1
21	SP1638127	Backrest tube Titan SB32 Long		1
22	1546818	Backrest tube Titan SB34 Long		1
23	1546819	Backrest tube Titan SB36 Long		1
24	1546820	Backrest tube Titan SB38 Long		1
25	1546821	Backrest tube Titan SB40 Long		1
26	1546822	Backrest tube Titan SB42 Long		1
27	1546823	Backrest tube Titan SB44 Long		1
28	1546824	Backrest tube Titan SB46 Long		1
29	1546825	Backrest tube Titan SB48 Long		1
30	SP1638128	Backrest tube Titan SB50 Long		1
31	1546913	Telescopic back tube standard short, pair		1

Backrest (rounded bar)

Pos.	Part number	Description	Valid from → until	Qty
32	1546914	Telescopic back tube standard medium, pair		1
33	1546946	Telescopic back tube standard long, pair		1
34	1524956	Telescopic tube short for pushhandle foldable, pair		1
35	1525121	Telescopic tube medium for pushhandle foldable, pair		1
36	1507733	Push handles foldable, pair		1
37	1525122	Backrest tube standard, short, pair		1
38	1525123	Backrest tube standard, long, pair		1
39	1507734	Handle black, pair		1
40	1525124	Backrest tube mini, short, pair		1
41	1525125	Backrest tube mini, long, pair		1
42	1525126	Handle black, mini, pair		1
43	1546947	Screw kit for backrest tube		1
44	SP1667840-0004	Kit, black connecting plate back, to adjusting the backrest angle 78/82/86/90/94° (pair)		1
45	1546950	Fixation kit to backrest joint		1
46	1546948	Sleeves for backrest tube alu, pair complete		1
47	1546949	Sleeves for backrest tube Titan, pair complete		1


Küschall backrest parts for Matrix backs

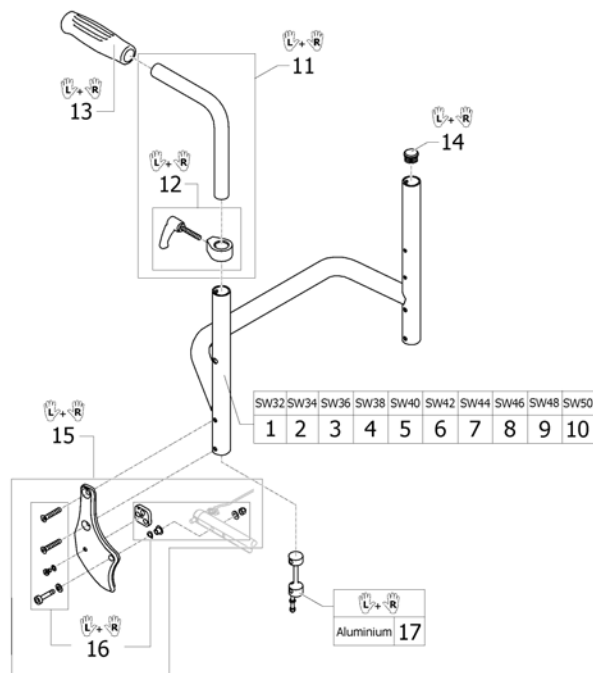

ISPS1665318

Pos.	Part number	Description	Valid from → until	Qty
1	SP1659571	Backrest tube Alu SW32 short		1
2	SP1659572	Backrest tube Alu SW34 short		1
3	SP1659573	Backrest tube Alu SW36 short		1
4	SP1659574	Backrest tube Alu SW38 short		1
5	SP1659575	Backrest tube Alu SW40 short		1
6	SP1659576	Backrest tube Alu SW42 short		1
7	SP1659577	Backrest tube Alu SW44 short		1
8	SP1659578	Backrest tube Alu SW46 short		1
9	SP1659579	Backrest tube Alu SW48 short		1
10	SP1659580	Backrest tube Alu SW50 short		1
11	1438203	Height adjustable, integrated push handles standard, kit short UL/C04		1
12	1549985	Adjusting lever for height adjustable, integrated push handles, pair		1
13	1507734	Handle black, pair		1
14	1561018	Tube plug Ø24mm, pair		1
15	SP1667840-0004	Kit, black connecting plate back, to adjusting the backrest angle 78/82/86/90/94° (pair)		1
16	1546950	Fixation kit to backrest joint		1
17	1546948	Sleeves for backrest tube alu, pair complete		1


Backrest cover light

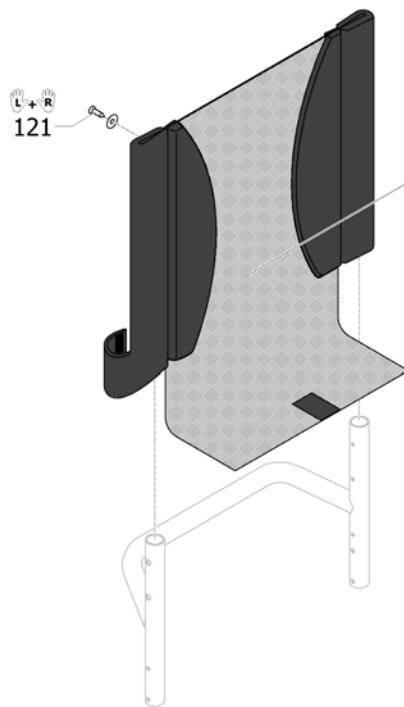

	SW34	SW36	SW38	SW40	SW42	SW44	SW46	SW48
BH27	1	2	3	4	5	6	7	8
BH28,5	9	10	11	12	13	14	15	16
BH30	17	18	19	20	21	22	23	24
BH31,5	25	26	27	28	29	30	31	32
BH33	33	34	35	36	37	38	39	40
BH34,5	41	42	43	44	45	46	47	48
BH36	49	50	51	52	53	54	55	56
BH37,5	57	58	59	60	61	62	63	64
BH39	65	66	67	68	69	70	71	72
BH40,5	73	74	75	76	77	78	79	80
BH42	81	82	83	84	85	86	87	88
BH43,5	89	90	91	92	93	94	95	96
BH45	97	98	99	100	101	102	103	104
BH46,5	105	106	107	108	109	110	111	112
BH48	113	114	115	116	117	118	119	120

ISPS1665319


Pos.	Part number	Description	Valid from → until	Qty
1	SP1659628	Backrest cover light SW34 BH27		1
2	SP1659629	Backrest cover light SW36 BH27		1
3	SP1659630	Backrest cover light SW38 BH27		1
4	SP1659631	Backrest cover light SW40 BH27		1
5	SP1659632	Backrest cover light SW42 BH27		1
6	SP1659636	Backrest cover light SW34 BH28,5		1
7	SP1659637	Backrest cover light SW36 BH28,5		1
8	SP1659638	Backrest cover light SW38 BH28,5		1
9	SP1659639	Backrest cover light SW40 BH28,5		1
10	SP1659640	Backrest cover light SW42 BH28,5		1
11	SP1659644	Backrest cover light SW34 BH30		1
12	SP1659645	Backrest cover light SW36 BH30		1
13	SP1659646	Backrest cover light SW38 BH30		1
14	SP1659647	Backrest cover light SW40 BH30		1
15	SP1659648	Backrest cover light SW42 BH30		1
16	SP1659652	Backrest cover light SW34 BH31,5		1
17	SP1659653	Backrest cover light SW36 BH31,5		1
18	SP1659654	Backrest cover light SW38 BH31,5		1
19	SP1659655	Backrest cover light SW40 BH31,5		1
20	SP1659656	Backrest cover light SW42 BH31,5		1
21	SP1659660	Backrest cover light SW34 BH33		1
22	SP1659661	Backrest cover light SW36 BH33		1
23	SP1659662	Backrest cover light SW38 BH33		1
24	SP1659663	Backrest cover light SW40 BH33		1
25	SP1659664	Backrest cover light SW42 BH33		1
26	SP1659668	Backrest cover light SW34 BH34,5		1
27	SP1659669	Backrest cover light SW36 BH34,5		1
28	SP1659670	Backrest cover light SW38 BH34,5		1
29	SP1659671	Backrest cover light SW40 BH34,5		1
30	SP1659672	Backrest cover light SW42 BH34,5		1
31	1432083	Fixation kit for backrest band		1

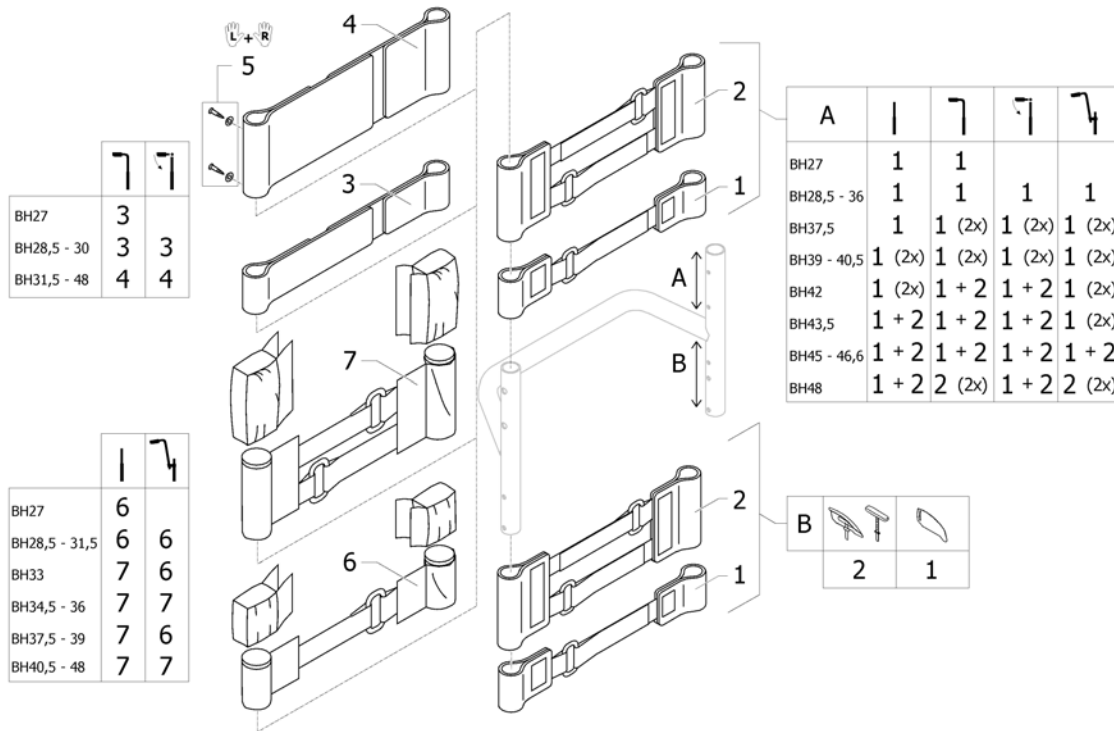


ISPS1665320

Pos.	Part number	Description	Valid from → until	Qty
1	1513167	Velcro adjustment part with 1 strap, pair		1
2	1513168	Velcro adjustment strap with 2 straps, pair		1
3	1513170	Backrest band 2 parts, width: 5cm		1
4	1513171	Backrest band 2 parts, width: 10cm		1
5	1432083	Fixation kit for backrest band		1
6	1513172	Velcro end part, 5 cm		1
7	1513173	Velcro end part, 10cm		1

Cover to adjustable backrest

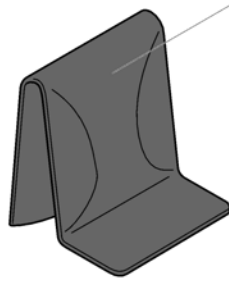

Std. (black premium)	SW32-34	SW36-38	SW40-42	SW44-46	SW48-50
BH27-30	1	2	3	4	5
BH31,5-36	6	7	8	9	10
BH37,5-42	11	12	13	14	15
BH43,5-48	16	17	18	19	20

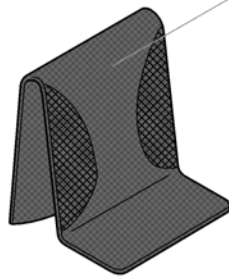

AeroBack	SW32-34	SW36-38	SW40-42	SW44-46	SW48-50
BH27-33	21	22	23	24	25
BH34,5-40,5	26	27	28	29	30
BH42-48	31	32	33	34	35


ISPS1665321

Pos.	Part number	Description	Valid from → until	Qty
1	SP1661094	Cover black premium to adjustable backrest SW32-34 Xshort		1
2	SP1661095	Cover black premium to adjustable backrest SW36-38 Xshort		1
3	SP1661096	Cover black premium to adjustable backrest SW40-42 Xshort		1
4	SP1661097	Cover black premium to adjustable backrest SW44-46 Xshort		1
5	SP1661098	Cover black premium to adjustable backrest SW48-50 Xshort		1
6	SP1661099	Cover black to adjustable backrest, SW32-34, short		1
7	SP1661100	Cover black to adjustable backrest, SW36-38, short		1
8	SP1661101	Cover black to adjustable backrest, SW40-42, short		1
9	SP1661102	Cover black to adjustable backrest, SW44-46, short		1
10	SP1661103	Cover black to adjustable backrest, SW48-50, short		1
11	SP1661104	Cover black to adjustable backrest, SW32-34, medium		1
12	SP1661105	Cover black to adjustable backrest, SW36-38, medium		1
13	SP1661106	Cover black to adjustable backrest, SW40-42, medium		1
14	SP1661107	Cover black to adjustable backrest, SW44-46, medium		1
15	SP1661108	Cover black to adjustable backrest, SW48-50, medium		1
16	SP1661109	Cover black to adjustable backrest, SW32-34, long		1
17	SP1661110	Cover black to adjustable backrest, SW36-38, long		1
18	SP1661111	Cover black to adjustable backrest, SW40-42, long		1
19	SP1661112	Cover black to adjustable backrest, SW44-46, long		1
20	SP1661113	Cover black to adjustable backrest, SW48-50, long		1
21	SP1661114	Cover 'Aero' to adjustable backrest, SW32-34, short		1
22	SP1661115	Cover 'Aero' to adjustable backrest, SW36-38, short		1
23	SP1661116	Cover 'Aero' to adjustable backrest, SW40-42, short		1
24	SP1661117	Cover 'Aero' to adjustable backrest, SW44-46, short		1
25	SP1661118	Cover 'Aero' to adjustable backrest, SW48-50, short		1
26	SP1661119	Cover 'Aero' to adjustable backrest, SW32-34, medium		1
27	SP1661120	Cover 'Aero' to adjustable backrest, SW36-38, medium		1
28	SP1661121	Cover 'Aero' to adjustable backrest, SW40-42, medium		1
29	SP1661122	Cover 'Aero' to adjustable backrest, SW44-46, medium		1
30	SP1661123	Cover 'Aero' to adjustable backrest, SW48-50, medium		1
31	SP1661124	Cover 'Aero' to adjustable backrest, SW32-34, long		1

Cover to adjustable backrest

Pos.	Part number	Description	Valid from → until	Qty
32	SP1661125	Cover 'Aero' to adjustable backrest, SW36-38, long		1
33	SP1661126	Cover 'Aero' to adjustable backrest, SW40-42, long		1
34	SP1661127	Cover 'Aero' to adjustable backrest, SW44-46, long		1
35	SP1661128	Cover 'Aero' to adjustable backrest, SW48-50, long		1

Push handles, height adjustable, rear set

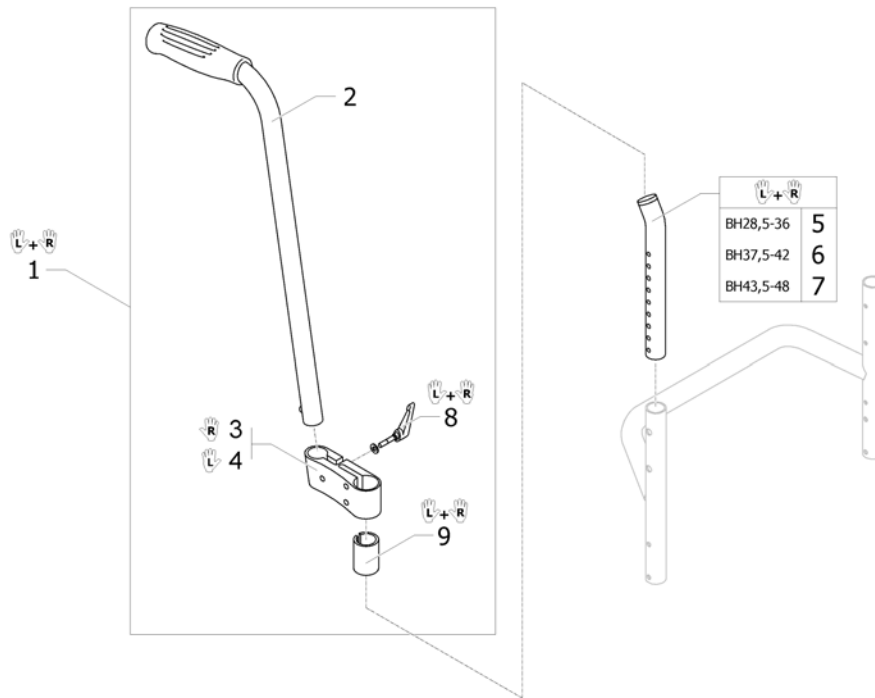

ISPS1665322

Pos.	Part number	Description	Valid from → until	Qty
1	1517876	Push handles, height adjustable, rear set, pair		1
2	1438938	Push handle tube complete, rearset		2
3	1438939	Mounting fixture right complete, for rearset push handles		1
4	1438940	Mounting fixture left complete, for rearset push handles		1
5	1546913	Telescopic back tube standard short, pair		1
6	1546914	Telescopic back tube standard medium, pair		1
7	1546946	Telescopic back tube standard long, pair		1
8	1529799	Adjusting lever, pair		1
9	1581293	Sleeves for mounting fixture, for rearset push handles, pair		1

Footrest standard mounted on frame aluminium short

Frames Aluminium short

Tab

	SW32	SW34	SW36	SW38	SW40	SW42	SW44	SW46	SW48	SW50
Aluminium	69	70	71	72	73	74	75	76	76	-
Titanium	77	78	79	80	81	82	83	84	-	-

	SW32	SW34	SW36	SW38	SW40	SW42	SW44	SW46	SW48	SW50
Aluminium	1	2	3	4	5	6	7	8	9	-
Titanium	10	11	12	13	14	15	16	17	-	-

	SW32	SW34	SW36	SW38	SW40	SW42	SW44	SW46	SW48	SW50
Aluminium	35	36	37	38	39	40	41	42	43	-
Titanium	44	45	46	47	48	49	50	51	-	-

	SW32	SW34	SW36	SW38	SW40	SW42	SW44	SW46	SW48	SW50
Aluminium	52	53	54	55	56	57	58	59	60	-
Titanium	61	62	63	64	65	66	67	68	-	-

	Aluminium	85
	Titanium	86

ISPS1665323

Pos.	Part number	Description	Valid from → until	Qty
Tab	-	Configuration table for footrest standard mounted on frame aluminium short		1
1	SP1661129	Footrest 75° Aluminium with plate complete SW32 for short frame		1
2	SP1661130	Footrest 75° Aluminium with plate complete SW34 for short frame		1
3	SP1661131	Footrest 75° Aluminium with plate complete SW36 for short frame		1
4	SP1661132	Footrest 75° Aluminium with plate complete SW38 for short frame		1
5	SP1661133	Footrest 75° Aluminium with plate complete SW40 for short frame		1
6	SP1661134	Footrest 75° Aluminium with plate complete SW42 for short frame		1
7	SP1661135	Footrest 75° Aluminium with plate complete SW44 for short frame		1
8	SP1661136	Footrest 75° Aluminium with plate complete SW46 for short frame		1
9	SP1661137	Footrest 75° Aluminium with plate complete SW48 for short frame		1
10	SP1665266	Footrest 75° Titanium with plate complete SW32 for Short frame		1
11	SP1665267	Footrest 75° Titanium with plate complete SW34 for Short frame		1
12	SP1665268	Footrest 75° Titanium with plate complete SW36 for Short frame		1
13	SP1665269	Footrest 75° Titanium with plate complete SW38 for Short frame		1
14	SP1665270	Footrest 75° Titanium with plate complete SW40 for Short frame		1
15	SP1665271	Footrest 75° Titanium with plate complete SW42 for Short frame		1
16	SP1665272	Footrest 75° Titanium with plate complete SW44 for Short frame		1
17	SP1665273	Footrest 75° Titanium with plate complete SW46 for Short frame		1
18	SP1661138	Footrest 90° Aluminium with plate complete SW32 for short frame		1
19	SP1661139	Footrest 90° Aluminium with plate complete SW34 for short frame		1
20	SP1661140	Footrest 90° Aluminium with plate complete SW36 for short frame		1
21	SP1661141	Footrest 90° Aluminium with plate complete SW38 for short frame		1
22	SP1661142	Footrest 90° Aluminium with plate complete SW40 for short frame		1
23	SP1661143	Footrest 90° Aluminium with plate complete SW42 for short frame		1
24	SP1661144	Footrest 90° Aluminium with plate complete SW44 for short frame		1
25	SP1661145	Footrest 90° Aluminium with plate complete SW46 for short frame		1
26	SP1661146	Footrest 90° Aluminium with plate complete SW48 for short frame		1
27	SP1665274	Footrest 90° Titanium with plate complete SW32 for Short frame		1
28	SP1665275	Footrest 90° Titanium with plate complete SW34 for Short frame		1
29	SP1665276	Footrest 90° Titanium with plate complete SW36 for Short frame		1
30	SP1665277	Footrest 90° Titanium with plate complete SW38 for Short frame		1

Footrest standard mounted on frame aluminium short


Pos.	Part number	Description	Valid from → until	Qty
31	SP1665278	Footrest 90° Titanium with plate complete SW40 for Short frame		1
32	SP1665279	Footrest 90° Titanium with plate complete SW42 for Short frame		1
33	SP1665280	Footrest 90° Titanium with plate complete SW44 for Short frame		1
34	SP1665281	Footrest 90° Titanium with plate complete SW46 for Short frame		1
35	SP1661147	Footrest 75° Aluminium complete SW32 for short frame		1
36	SP1661148	Footrest 75° Aluminium complete SW34 for short frame		1
37	SP1661149	Footrest 75° Aluminium complete SW36 for short frame		1
38	SP1661150	Footrest 75° Aluminium complete SW38 for short frame		1
39	SP1661151	Footrest 75° Aluminium complete SW40 for short frame		1
40	SP1661152	Footrest 75° Aluminium complete SW42 for short frame		1
41	SP1661153	Footrest 75° Aluminium complete SW44 for short frame		1
42	SP1661154	Footrest 75° Aluminium complete SW46 for short frame		1
43	SP1661155	Footrest 75° Aluminium complete SW48 for short frame		1
44	SP1665282	Footrest 75° Titanium complete SW32 short frame		1
45	SP1665283	Footrest 75° Titanium complete SW34 short frame		1
46	SP1665284	Footrest 75° Titanium complete SW36 short frame		1
47	SP1665285	Footrest 75° Titanium complete SW38 short frame		1
48	SP1665286	Footrest 75° Titanium complete SW40 short frame		1
49	SP1665287	Footrest 75° Titanium complete SW42 short frame		1
50	SP1665288	Footrest 75° Titanium complete SW44 short frame		1
51	SP1665289	Footrest 75° Titanium complete SW46 short frame		1
52	SP1661156	Footrest 90° Aluminium complete SW32 for short frame		1
53	SP1661157	Footrest 90° Aluminium complete SW34 for short frame		1
54	SP1661158	Footrest 90° Aluminium complete SW36 for short frame		1
55	SP1661159	Footrest 90° Aluminium complete SW38 for short frame		1
56	SP1661160	Footrest 90° Aluminium complete SW40 for short frame		1
57	SP1661161	Footrest 90° Aluminium complete SW42 for short frame		1
58	SP1661162	Footrest 90° Aluminium complete SW44 for short frame		1
59	SP1661163	Footrest 90° Aluminium complete SW46 for short frame		1
60	SP1661164	Footrest 90° Aluminium complete SW48 for short frame		1
61	SP1665290	Footrest 90° Titanium complete SW32 short frame		1
62	SP1665291	Footrest 90° Titanium complete SW34 short frame		1
63	SP1665292	Footrest 90° Titanium complete SW36 short frame		1
64	SP1665293	Footrest 90° Titanium complete SW38 short frame		1
65	SP1665294	Footrest 90° Titanium complete SW40 short frame		1
66	SP1665295	Footrest 90° Titanium complete SW42 short frame		1
67	SP1665296	Footrest 90° Titanium complete SW44 short frame		1
68	SP1665297	Footrest 90° Titanium complete SW46 short frame		1
69	SP1661407	Footplate 154mm		1
70	SP1661408	Footplate 174mm		1
71	SP1661409	Footplate 194mm		1
72	SP1661410	Footplate 214mm		1
73	SP1661478	Footplate 234mm		1
74	SP1661489	Footplate 254mm		1
75	SP1661490	Footplate 274mm		1
76	SP1661491	Footplate 294mm		1
77	SP1589855	Footplate Titanium 164mm		1
78	SP1589856	Footplate Titanium 184mm		1
79	SP1589857	Footplate Titanium 204mm		1
80	SP1589858	Footplate Titanium 224mm		1
81	SP1589859	Footplate Titanium 244mm		1
82	SP1589860	Footplate Titanium 264mm		1
83	SP1589861	Footplate Titanium 284mm		1
84	SP1589862	Footplate Titanium 304mm		1
85	SP1661492	Footplate sleeve Alu		1

Footrest standard mounted on frame aluminium short

Pos.	Part number	Description	Valid from → until	Qty
86	SP1661493	Footplate sleeve Titanium		1

Configuration table for footrest standard mounted on frame aluminium short


i	Rear Wheel 24"				Rear Wheel 25"				Rear Wheel 26"			
Frame Aluminium 75° Short (236 mm)	FSTF 45	LLL 29 -> LLL 32	LLL 33 -> LLL 42		FSTF 46	LLL 29 -> LLL 32	LLL 33 -> LLL 43		FSTF 47	LLL 29 -> LLL 32	LLL 33 -> LLL 44	
	FSTF 46	LLL 29 -> LLL 33	LLL 34 -> LLL 43		FSTF 47	LLL 29 -> LLL 33	LLL 34 -> LLL 44		FSTF 48	LLL 29 -> LLL 33	LLL 34 -> LLL 45	
	FSTF 47	LLL 29 -> LLL 34	LLL 35 -> LLL 44		FSTF 48	LLL 29 -> LLL 34	LLL 35 -> LLL 45		FSTF 49	LLL 29 -> LLL 34	LLL 35 -> LLL 46	
Frame Aluminium 90° Short (261,1 mm)	FSTF 45	LLL 29 -> LLL 32	LLL 33 -> LLL 41		FSTF 46	LLL 29 -> LLL 32	LLL 33 -> LLL 42		FSTF 47	LLL 29 -> LLL 32	LLL 33 -> LLL 43	
	FSTF 46	LLL 29 -> LLL 33	LLL 34 -> LLL 42		FSTF 47	LLL 29 -> LLL 33	LLL 34 -> LLL 43		FSTF 48	LLL 29 -> LLL 33	LLL 34 -> LLL 44	
	FSTF 47	LLL 29 -> LLL 34	LLL 35 -> LLL 43		FSTF 48	LLL 29 -> LLL 34	LLL 35 -> LLL 44		FSTF 49	LLL 29 -> LLL 34	LLL 35 -> LLL 45	

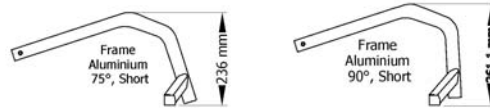

ISPS1665371

Pos.	Part number	Description	Valid from → until	Qty
i	ISPDINFC1-0036	Knee to heel length (UL) in relationship to frame type, front seat height (SHv) and rear wheel size		1

Footrest standard mounted on frame aluminium Std or +5cm

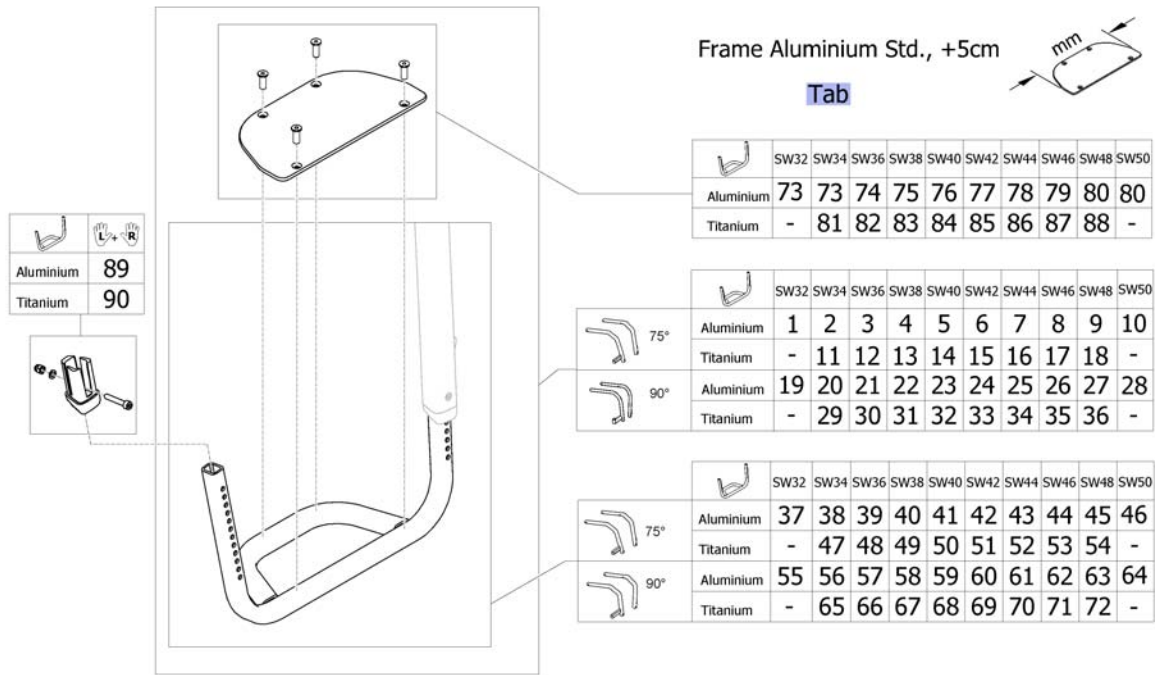

ISPS1665324

Pos.	Part number	Description	Valid from → until	Qty
Tab	-	Configuration table for footrest standard mounted on frame aluminium Std. or +5cm		1
1	SP1661229	Footrest 75° Aluminium with plate complete SW32		1
2	SP1661241	Footrest 75° Aluminium with plate complete SW34		1
3	SP1661242	Footrest 75° Aluminium with plate complete SW36		1
4	SP1661243	Footrest 75° Aluminium with plate complete SW38		1
5	SP1661244	Footrest 75° Aluminium with plate complete SW40		1
6	SP1661245	Footrest 75° Aluminium with plate complete SW42		1
7	SP1661246	Footrest 75° Aluminium with plate complete SW44		1
8	SP1661261	Footrest 75° Aluminium with plate complete SW46		1
9	SP1661262	Footrest 75° Aluminium with plate complete SW48		1
10	SP1661296	Footrest 75° Aluminium with plate complete SW50		1
11	SP1665214	Footrest 75° Titanium with plate complete SW34		1
12	SP1665215	Footrest 75° Titanium with plate complete SW36		1
13	SP1665216	Footrest 75° Titanium with plate complete SW38		1
14	SP1665217	Footrest 75° Titanium with plate complete SW40		1
15	SP1665218	Footrest 75° Titanium with plate complete SW42		1
16	SP1665219	Footrest 75° Titanium with plate complete SW44		1
17	SP1665220	Footrest 75° Titanium with plate complete SW46		1
18	SP1665221	Footrest 75° Titanium with plate complete SW48		1
19	SP1661325	Footrest 90° Aluminium with plate complete SW32		1
20	SP1661326	Footrest 90° Aluminium with plate complete SW34		1
21	SP1661327	Footrest 90° Aluminium with plate complete SW36		1
22	SP1661328	Footrest 90° Aluminium with plate complete SW38		1
23	SP1661329	Footrest 90° Aluminium with plate complete SW40		1
24	SP1661330	Footrest 90° Aluminium with plate complete SW42		1
25	SP1661331	Footrest 90° Aluminium with plate complete SW44		1
26	SP1661332	Footrest 90° Aluminium with plate complete SW46		1
27	SP1661333	Footrest 90° Aluminium with plate complete SW48		1
28	SP1661334	Footrest 90° Aluminium with plate complete SW50		1
29	SP1665222	Footrest 90° Titanium with plate complete SW34		1
30	SP1665223	Footrest 90° Titanium with plate complete SW36		1


Footrest standard mounted on frame aluminium Std or +5cm


Pos.	Part number	Description	Valid from → until	Qty
31	SP1665224	Footrest 90° Titanium with plate complete SW38		1
32	SP1665225	Footrest 90° Titanium with plate complete SW40		1
33	SP1665226	Footrest 90° Titanium with plate complete SW42		1
34	SP1665227	Footrest 90° Titanium with plate complete SW44		1
35	SP1665228	Footrest 90° Titanium with plate complete SW46		1
36	SP1665229	Footrest 90° Titanium with plate complete SW48		1
37	SP1661335	Footrest 75° Aluminium complete SW32		1
38	SP1661336	Footrest 75° Aluminium complete SW34		1
39	SP1661337	Footrest 75° Aluminium complete SW36		1
40	SP1661338	Footrest 75° Aluminium complete SW38		1
41	SP1661339	Footrest 75° Aluminium complete SW40		1
42	SP1661340	Footrest 75° Aluminium complete SW42		1
43	SP1661341	Footrest 75° Aluminium complete SW44		1
44	SP1661342	Footrest 75° Aluminium complete SW46		1
45	SP1661343	Footrest 75° Aluminium complete SW48		1
46	SP1661344	Footrest 75° Aluminium complete SW50		1
47	SP1665173	Footrest 75° Titanium complete SW34		1
48	SP1665174	Footrest 75° Titanium complete SW36		1
49	SP1665175	Footrest 75° Titanium complete SW38		1
50	SP1665176	Footrest 75° Titanium complete SW40		1
51	SP1665177	Footrest 75° Titanium complete SW42		1
52	SP1665178	Footrest 75° Titanium complete SW44		1
53	SP1665179	Footrest 75° Titanium complete SW46		1
54	SP1665197	Footrest 75° Titanium complete SW48		1
55	SP1661345	Footrest 90° Aluminium complete SW32		1
56	SP1661346	Footrest 90° Aluminium complete SW34		1
57	SP1661347	Footrest 90° Aluminium complete SW36		1
58	SP1661348	Footrest 90° Aluminium complete SW38		1
59	SP1661349	Footrest 90° Aluminium complete SW40		1
60	SP1661350	Footrest 90° Aluminium complete SW42		1
61	SP1661351	Footrest 90° Aluminium complete SW44		1
62	SP1661352	Footrest 90° Aluminium complete SW46		1
63	SP1661405	Footrest 90° Aluminium complete SW48		1
64	SP1661406	Footrest 90° Aluminium complete SW50		1
65	SP1665206	Footrest 90° Titanium complete SW34		1
66	SP1665207	Footrest 90° Titanium complete SW36		1
67	SP1665208	Footrest 90° Titanium complete SW38		1
68	SP1665209	Footrest 90° Titanium complete SW40		1
69	SP1665210	Footrest 90° Titanium complete SW42		1
70	SP1665211	Footrest 90° Titanium complete SW44		1
71	SP1665212	Footrest 90° Titanium complete SW46		1
72	SP1665213	Footrest 90° Titanium complete SW48		1
73	SP1661407	Footplate 154mm		1
74	SP1661408	Footplate 174mm		1
75	SP1661409	Footplate 194mm		1
76	SP1661410	Footplate 214mm		1
77	SP1661478	Footplate 234mm		1
78	SP1661489	Footplate 254mm		1
79	SP1661490	Footplate 274mm		1
80	SP1661491	Footplate 294mm		1
81	SP1589855	Footplate Titanium 164mm		1
82	SP1589856	Footplate Titanium 184mm		1
83	SP1589857	Footplate Titanium 204mm		1
84	SP1589858	Footplate Titanium 224mm		1
85	SP1589859	Footplate Titanium 244mm		1

Footrest standard mounted on frame aluminium Std or +5cm

Pos.	Part number	Description	Valid from → until	Qty
86	SP1589860	Footplate Titanium 264mm		1
87	SP1589861	Footplate Titanium 284mm		1
88	SP1589862	Footplate Titanium 304mm		1
89	SP1661492	Footplate sleeve Alu		1
90	SP1661493	Footplate sleeve Titanium		1

Configuration table for footrest standard mounted on frame aluminium Std. or +5cm

		Rear Wheel 24"		Rear Wheel 25"		Rear Wheel 26"			
 Frame Aluminium 75° Std. (283,3 mm)	FSTF 48	LLL 29 -> LLL 36	LLL 37 -> LLL 45	FSTF 49	LLL 29 -> LLL 36	LLL 37 -> LLL 46	FSTF 50	LLL 29 -> LLL 36	LLL 37 -> LLL 47
	FSTF 49	LLL 29 -> LLL 37	LLL 38 -> LLL 46	FSTF 50	LLL 29 -> LLL 37	LLL 38 -> LLL 47	FSTF 51	LLL 29 -> LLL 37	LLL 38 -> LLL 48
	FSTF 50	LLL 29 -> LLL 38	LLL 39 -> LLL 47	FSTF 51	LLL 29 -> LLL 38	LLL 39 -> LLL 48	FSTF 52	LLL 29 -> LLL 38	LLL 39 -> LLL 49
Frame Aluminium 75° +5cm (302 mm)	FSTF 50	LLL 29 -> LLL 38	LLL 39 -> LLL 47	FSTF 51	LLL 29 -> LLL 38	LLL 39 -> LLL 48	FSTF 52	LLL 29 -> LLL 38	LLL 39 -> LLL 49
	FSTF 51	LLL 29 -> LLL 39	LLL 40 -> LLL 48	FSTF 52	LLL 29 -> LLL 39	LLL 40 -> LLL 49	FSTF 53	LLL 29 -> LLL 39	LLL 40 -> LLL 50
	FSTF 52	LLL 29 -> LLL 40	LLL 41 -> LLL 49	FSTF 53	LLL 29 -> LLL 40	LLL 41 -> LLL 50	FSTF 54	LLL 29 -> LLL 40	LLL 41 -> LLL 51
Frame Aluminium 90° Std. (308,3 mm)	FSTF 48	LLL 29 -> LLL 36	LLL 38 -> LLL 44	FSTF 49	LLL 29 -> LLL 36	LLL 38 -> LLL 45	FSTF 50	LLL 29 -> LLL 36	LLL 38 -> LLL 46
	FSTF 49	LLL 29 -> LLL 37	LLL 39 -> LLL 45	FSTF 50	LLL 29 -> LLL 37	LLL 39 -> LLL 46	FSTF 51	LLL 29 -> LLL 37	LLL 39 -> LLL 47
	FSTF 50	LLL 29 -> LLL 38	LLL 40 -> LLL 46	FSTF 51	LLL 29 -> LLL 38	LLL 40 -> LLL 47	FSTF 52	LLL 29 -> LLL 38	LLL 40 -> LLL 48
Frame Aluminium 90° +5cm (327 mm)	FSTF 50	LLL 29 -> LLL 39	LLL 40 -> LLL 46	FSTF 51	LLL 29 -> LLL 38	LLL 40 -> LLL 47	FSTF 52	LLL 29 -> LLL 38	LLL 40 -> LLL 48
	FSTF 51	LLL 29 -> LLL 40	LLL 41 -> LLL 47	FSTF 52	LLL 29 -> LLL 39	LLL 41 -> LLL 48	FSTF 53	LLL 29 -> LLL 39	LLL 41 -> LLL 49
	FSTF 52	LLL 29 -> LLL 41	LLL 42 -> LLL 48	FSTF 53	LLL 29 -> LLL 40	LLL 42 -> LLL 49	FSTF 54	LLL 29 -> LLL 40	LLL 42 -> LLL 50

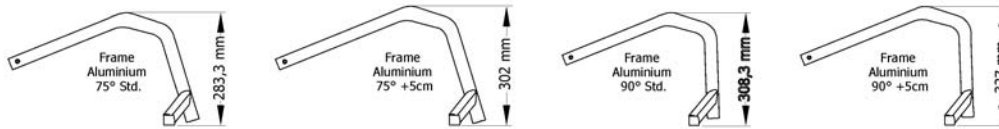

ISPS1665372

Pos.	Part number	Description	Valid from → until	Qty
i	ISPDINFC1-0036	Knee to heel length (UL) in relationship to frame type, front seat height (SHv) and rear wheel size		1

Footrest standard for adducted frame

Tab

	SW32	SW34	SW36	SW38	SW40	SW42	SW44	SW46	SW48	SW50
Aluminium	77	78	79	80	81	82	83	84	85	86
Titanium	87	88	89	90	91	92	93	94	95	-

Frame adducted		SW32	SW34	SW36	SW38	SW40	SW42	SW44	SW46	SW48	SW50
75°	Aluminium	1	2	3	4	5	6	7	8	9	10
	Titanium	11	12	13	14	15	16	17	18	19	-
90°	Aluminium	20	21	22	23	24	25	26	27	28	29
	Titanium	30	31	32	33	34	35	36	37	38	-

Frame adducted		SW32	SW34	SW36	SW38	SW40	SW42	SW44	SW46	SW48	SW50
75°	Aluminium	39	40	41	42	43	44	45	46	47	48
	Titanium	49	50	51	52	53	54	55	56	57	-
90°	Aluminium	58	59	60	61	62	63	64	65	66	67
	Titanium	68	69	70	71	72	73	74	75	76	-

If you use a highmounted Footrest take always a SW smaller for adducted frame.

ISPS1665325

Pos.	Part number	Description	Valid from → until	Qty
Tab	-	Configuration table for footrest standard mounted on adducted frame		1
1	SP1661494	Footrest 75° Aluminium with plate complete SW32 for adducted frame		1
2	SP1661495	Footrest 75° Aluminium with plate complete SW34 for adducted frame		1
3	SP1661496	Footrest 75° Aluminium with plate complete SW36 for adducted frame		1
4	SP1661497	Footrest 75° Aluminium with plate complete SW38 for adducted frame		1
5	SP1661498	Footrest 75° Aluminium with plate complete SW40 for adducted frame		1
6	SP1661499	Footrest 75° Aluminium with plate complete SW42 for adducted frame		1
7	SP1661500	Footrest 75° Aluminium with plate complete SW44 for adducted frame		1
8	SP1661501	Footrest 75° Aluminium with plate complete SW46 for adducted frame		1
9	SP1661502	Footrest 75° Aluminium with plate complete SW48 for adducted frame		1
10	SP1661503	Footrest 75° Aluminium with plate complete SW50 for adducted frame		1
11	SP1665230	Footrest 75° Titanium with plate complete SW32 for adducted frame		1
12	SP1665231	Footrest 75° Titanium with plate complete SW34 for adducted frame		1
13	SP1665232	Footrest 75° Titanium with plate complete SW36 for adducted frame		1
14	SP1665233	Footrest 75° Titanium with plate complete SW38 for adducted frame		1
15	SP1665234	Footrest 75° Titanium with plate complete SW40 for adducted frame		1
16	SP1665235	Footrest 75° Titanium with plate complete SW42 for adducted frame		1
17	SP1665236	Footrest 75° Titanium with plate complete SW44 for adducted frame		1
18	SP1665237	Footrest 75° Titanium with plate complete SW46 for adducted frame		1
19	SP1665238	Footrest 75° Titanium with plate complete SW48 for adducted frame		1
20	SP1661504	Footrest 90° Aluminium with plate complete SW32 for adducted frame		1
21	SP1661679	Footrest 90° Aluminium with plate complete SW34 for adducted frame		1
22	SP1661922	Footrest 90° Aluminium with plate complete SW36 for adducted frame		1
23	SP1661923	Footrest 90° Aluminium with plate complete SW38 for adducted frame		1
24	SP1661924	Footrest 90° Aluminium with plate complete SW40 for adducted frame		1
25	SP1661925	Footrest 90° Aluminium with plate complete SW42 for adducted frame		1
26	SP1661926	Footrest 90° Aluminium with plate complete SW44 for adducted frame		1
27	SP1661927	Footrest 90° Aluminium with plate complete SW46 for adducted frame		1
28	SP1661928	Footrest 90° Aluminium with plate complete SW48 for adducted frame		1
29	SP1661929	Footrest 90° Aluminium with plate complete SW50 for adducted frame		1
30	SP1665239	Footrest 90° Titanium with plate complete SW32 for adducted frame		1

Footrest standard for adducted frame


Pos.	Part number	Description	Valid from → until	Qty
31	SP1665240	Footrest 90° Titanium with plate complete SW34 for adducted frame		1
32	SP1665241	Footrest 90° Titanium with plate complete SW36 for adducted frame		1
33	SP1665242	Footrest 90° Titanium with plate complete SW38 for adducted frame		1
34	SP1665243	Footrest 90° Titanium with plate complete SW40 for adducted frame		1
35	SP1665244	Footrest 90° Titanium with plate complete SW42 for adducted frame		1
36	SP1665245	Footrest 90° Titanium with plate complete SW44 for adducted frame		1
37	SP1665246	Footrest 90° Titanium with plate complete SW46 for adducted frame		1
38	SP1665247	Footrest 90° Titanium with plate complete SW48 for adducted frame		1
39	SP1661930	Footrest 75° Aluminium complete SW32 for adducted frame		1
40	SP1661931	Footrest 75° Aluminium complete SW34 for adducted frame		1
41	SP1661932	Footrest 75° Aluminium complete SW36 for adducted frame		1
42	SP1661933	Footrest 75° Aluminium complete SW38 for adducted frame		1
43	SP1661934	Footrest 75° Aluminium complete SW40 for adducted frame		1
44	SP1661935	Footrest 75° Aluminium complete SW42 for adducted frame		1
45	SP1661936	Footrest 75° Aluminium complete SW44 for adducted frame		1
46	SP1661937	Footrest 75° Aluminium complete SW46 for adducted frame		1
47	SP1661938	Footrest 75° Aluminium complete SW48 for adducted frame		1
48	SP1661939	Footrest 75° Aluminium complete SW50 for adducted frame		1
49	SP1665248	Footrest 75° Titanium complete SW32 for adducted frame		1
50	SP1665249	Footrest 75° Titanium complete SW34 for adducted frame		1
51	SP1665250	Footrest 75° Titanium complete SW36 for adducted frame		1
52	SP1665251	Footrest 75° Titanium complete SW38 for adducted frame		1
53	SP1665252	Footrest 75° Titanium complete SW40 for adducted frame		1
54	SP1665253	Footrest 75° Titanium complete SW42 for adducted frame		1
55	SP1665254	Footrest 75° Titanium complete SW44 for adducted frame		1
56	SP1665255	Footrest 75° Titanium complete SW46 for adducted frame		1
57	SP1665256	Footrest 75° Titanium complete SW48 for adducted frame		1
58	SP1662077	Footrest 90° Aluminium complete SW32 for adducted frame		1
59	SP1662078	Footrest 90° Aluminium complete SW34 for adducted frame		1
60	SP1662094	Footrest 90° Aluminium complete SW36 for adducted frame		1
61	SP1662095	Footrest 90° Aluminium complete SW38 for adducted frame		1
62	SP1662096	Footrest 90° Aluminium complete SW40 for adducted frame		1
63	SP1662097	Footrest 90° Aluminium complete SW42 for adducted frame		1
64	SP1662098	Footrest 90° Aluminium complete SW44 for adducted frame		1
65	SP1662099	Footrest 90° Aluminium complete SW46 for adducted frame		1
66	SP1662100	Footrest 90° Aluminium complete SW48 for adducted frame		1
67	SP1662101	Footrest 90° Aluminium complete SW50 for adducted frame		1
68	SP1665257	Footrest 90° Titanium complete SW32 for adducted frame		1
69	SP1665258	Footrest 90° Titanium complete SW34 for adducted frame		1
70	SP1665259	Footrest 90° Titanium complete SW36 for adducted frame		1
71	SP1665260	Footrest 90° Titanium complete SW38 for adducted frame		1
72	SP1665261	Footrest 90° Titanium complete SW40 for adducted frame		1
73	SP1665262	Footrest 90° Titanium complete SW42 for adducted frame		1
74	SP1665263	Footrest 90° Titanium complete SW44 for adducted frame		1
75	SP1665264	Footrest 90° Titanium complete SW46 for adducted frame		1
76	SP1665265	Footrest 90° Titanium complete SW48 for adducted frame		1
77	SP1662102	Footplate Aluminium SW32 for adducted frame		1
78	SP1662103	Footplate Aluminium SW34 for adducted frame		1
79	SP1662104	Footplate Aluminium SW36 for adducted frame		1
80	SP1662105	Footplate Aluminium SW38 for adducted frame		1
81	SP1662106	Footplate Aluminium SW40 for adducted frame		1
82	SP1662107	Footplate Aluminium SW42 for adducted frame		1
83	SP1662108	Footplate Aluminium SW44 for adducted frame		1
84	SP1662109	Footplate Aluminium SW46 for adducted frame		1
85	SP1662110	Footplate Aluminium SW48 for adducted frame		1

Footrest standard for adducted frame

Pos.	Part number	Description	Valid from → until	Qty
86	SP1662111	Footplate Aluminium SW50 for adducted frame		1
87	SP1642211	Footplate Titanium SB32 for adducted frame		1
88	1547404	Footplate Titanium SB34 for adducted frame		1
89	1547405	Footplate Titanium SB36 for adducted frame		1
90	1547406	Footplate Titanium SB38 for adducted frame		1
91	1547407	Footplate Titanium SB40 for adducted frame		1
92	1547408	Footplate Titanium SB42 for adducted frame		1
93	1547409	Footplate Titanium SB44 for adducted frame		1
94	1547410	Footplate Titanium SB46 for adducted frame		1
95	1547411	Footplate Titanium SB48 for adducted frame		1
96	SP1661492	Footplate sleeve Alu		1
97	SP1661493	Footplate sleeve Titanium		1

Configuration table for footrest standard mounted on adducted frame


i	Rear Wheel 24"				Rear Wheel 25"				Rear Wheel 26"			
Frame Aluminium Adducted 75° (203,6 mm)	FSTF 48	LLL 29 -> LLL 36	LLL 37 -> LLL 45		FSTF 49	LLL 29 -> LLL 36	LLL 37 -> LLL 46		FSTF 50	LLL 29 -> LLL 36	LLL 37 -> LLL 47	
	FSTF 49	LLL 29 -> LLL 37	LLL 38 -> LLL 46		FSTF 50	LLL 29 -> LLL 37	LLL 38 -> LLL 47		FSTF 51	LLL 29 -> LLL 37	LLL 38 -> LLL 48	
	FSTF 50	LLL 29 -> LLL 38	LLL 39 -> LLL 47		FSTF 51	LLL 29 -> LLL 38	LLL 39 -> LLL 48		FSTF 52	LLL 29 -> LLL 38	LLL 39 -> LLL 49	
Frame Aluminium Adducted 90° (308,5 mm)	FSTF 48	LLL 29 -> LLL 36	LLL 39 -> LLL 44		FSTF 49	LLL 29 -> LLL 36	LLL 39 -> LLL 45		FSTF 50	LLL 29 -> LLL 36	LLL 39 -> LLL 46	
	FSTF 49	LLL 29 -> LLL 37	LLL 40 -> LLL 45		FSTF 50	LLL 29 -> LLL 37	LLL 40 -> LLL 46		FSTF 51	LLL 29 -> LLL 37	LLL 40 -> LLL 47	
	FSTF 50	LLL 29 -> LLL 38	LLL 41 -> LLL 46		FSTF 51	LLL 29 -> LLL 38	LLL 41 -> LLL 47		FSTF 52	LLL 29 -> LLL 38	LLL 41 -> LLL 48	

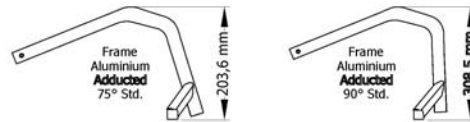

ISPS1665373

Pos.	Part number	Description	Valid from → until	Qty
i	ISPDINFC1-0036	Knee to heel length (UL) in relationship to frame type, front seat height (SHv) and rear wheel size		1


Footrest angle adjustable

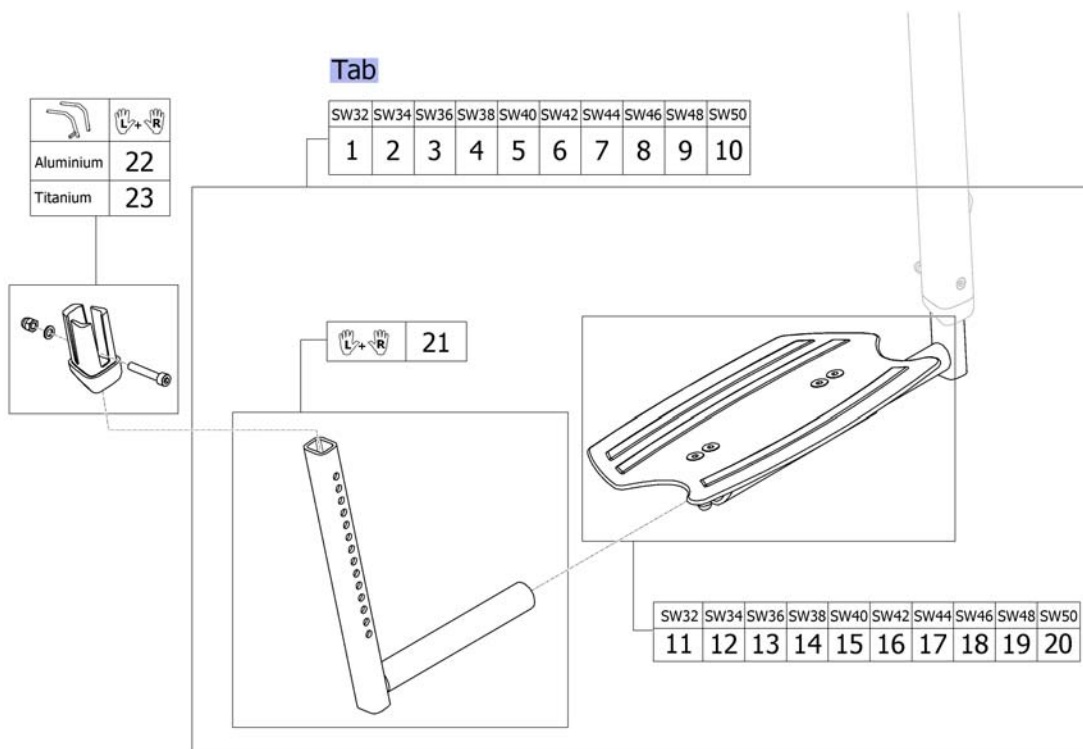


ISPS1665326

Pos.	Part number	Description	Valid from → until	Qty
Tab	-	Configuration table for footrest angle adjustable		1
1	SP1662114	Footrest angle adjustable complete SW32		1
2	SP1662115	Footrest angle adjustable complete SW34		1
3	SP1662116	Footrest angle adjustable complete SW36		1
4	SP1662117	Footrest angle adjustable complete SW38		1
5	SP1662118	Footrest angle adjustable complete SW40		1
6	SP1662119	Footrest angle adjustable complete SW42		1
7	SP1662120	Footrest angle adjustable complete SW44		1
8	SP1662121	Footrest angle adjustable complete SW46		1
9	SP1662122	Footrest angle adjustable complete SW48		1
10	SP1662146	Footrest angle adjustable complete SW50		1
11	SP1662147	footplate with clamping part SW32		1
12	SP1662148	footplate with clamping part SW34		1
13	SP1662149	footplate with clamping part SW36		1
14	SP1662150	footplate with clamping part SW38		1
15	SP1662176	footplate with clamping part SW40		1
16	SP1662177	footplate with clamping part SW42		1
17	SP1662178	footplate with clamping part SW44		1
18	SP1662179	footplate with clamping part SW46		1
19	SP1662180	footplate with clamping part SW48		1
20	SP1662181	footplate with clamping part SW50		1
21	SP1662182	Telescopic tube kit to footrest angle adjustable footrest angle adjustable		1
22	SP1661492	Footplate sleeve Alu		1
23	SP1661493	Footplate sleeve Titanium		1

Configuration table for footrest angle adjustable

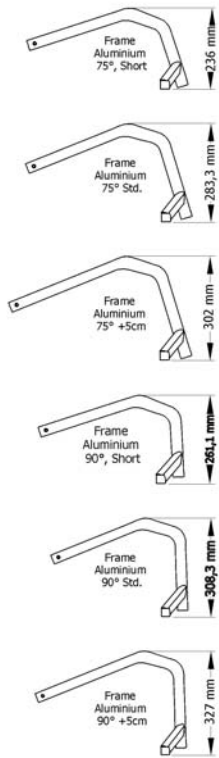


	Rear Wheel 24"			Rear Wheel 25"			Rear Wheel 26"			
	Frame Aluminium 75° Short (236 mm)	FSTF 45	LLL 29 -> LLL 32	LLL 33 -> LLL 42	FSTF 46	LLL 29 -> LLL 32	LLL 33 -> LLL 43	FSTF 47	LLL 29 -> LLL 32	LLL 33 -> LLL 44
		FSTF 46	LLL 29 -> LLL 33	LLL 34 -> LLL 43	FSTF 47	LLL 29 -> LLL 33	LLL 34 -> LLL 44	FSTF 48	LLL 29 -> LLL 33	LLL 34 -> LLL 45
		FSTF 47	LLL 29 -> LLL 34	LLL 35 -> LLL 44	FSTF 48	LLL 29 -> LLL 34	LLL 35 -> LLL 45	FSTF 49	LLL 29 -> LLL 34	LLL 35 -> LLL 46
Frame Aluminium 75° Std. (283,3 mm)	FSTF 48	LLL 29 -> LLL 36	LLL 37 -> LLL 45	FSTF 49	LLL 29 -> LLL 36	LLL 37 -> LLL 46	FSTF 50	LLL 29 -> LLL 36	LLL 37 -> LLL 47	
	FSTF 49	LLL 29 -> LLL 37	LLL 38 -> LLL 46	FSTF 50	LLL 29 -> LLL 37	LLL 38 -> LLL 47	FSTF 51	LLL 29 -> LLL 37	LLL 38 -> LLL 48	
	FSTF 50	LLL 29 -> LLL 38	LLL 39 -> LLL 47	FSTF 51	LLL 29 -> LLL 38	LLL 39 -> LLL 48	FSTF 52	LLL 29 -> LLL 38	LLL 39 -> LLL 49	
Frame Aluminium 75° +5cm (283,3 mm)	FSTF 50	LLL 29 -> LLL 38	LLL 39 -> LLL 47	FSTF 51	LLL 29 -> LLL 38	LLL 39 -> LLL 48	FSTF 52	LLL 29 -> LLL 38	LLL 39 -> LLL 49	
	FSTF 51	LLL 29 -> LLL 39	LLL 40 -> LLL 48	FSTF 52	LLL 29 -> LLL 39	LLL 40 -> LLL 49	FSTF 53	LLL 29 -> LLL 39	LLL 40 -> LLL 50	
	FSTF 52	LLL 29 -> LLL 40	LLL 41 -> LLL 49	FSTF 53	LLL 29 -> LLL 40	LLL 41 -> LLL 50	FSTF 54	LLL 29 -> LLL 40	LLL 41 -> LLL 51	
Frame Aluminium 90° Short (261,1 mm)	FSTF 45	LLL 29 -> LLL 32	LLL 33 -> LLL 41	FSTF 46	LLL 29 -> LLL 32	LLL 33 -> LLL 42	FSTF 47	LLL 29 -> LLL 32	LLL 33 -> LLL 43	
	FSTF 46	LLL 29 -> LLL 33	LLL 34 -> LLL 42	FSTF 47	LLL 29 -> LLL 33	LLL 34 -> LLL 43	FSTF 48	LLL 29 -> LLL 33	LLL 34 -> LLL 44	
	FSTF 47	LLL 29 -> LLL 34	LLL 35 -> LLL 43	FSTF 48	LLL 29 -> LLL 34	LLL 35 -> LLL 44	FSTF 49	LLL 29 -> LLL 34	LLL 35 -> LLL 45	
Frame Aluminium 90° Std. (308,3 mm)	FSTF 48	LLL 29 -> LLL 36	LLL 38 -> LLL 44	FSTF 49	LLL 29 -> LLL 36	LLL 38 -> LLL 45	FSTF 50	LLL 29 -> LLL 36	LLL 38 -> LLL 46	
	FSTF 49	LLL 29 -> LLL 37	LLL 39 -> LLL 45	FSTF 50	LLL 29 -> LLL 37	LLL 39 -> LLL 46	FSTF 51	LLL 29 -> LLL 37	LLL 39 -> LLL 47	
	FSTF 50	LLL 29 -> LLL 38	LLL 40 -> LLL 46	FSTF 51	LLL 29 -> LLL 38	LLL 40 -> LLL 47	FSTF 52	LLL 29 -> LLL 38	LLL 40 -> LLL 48	
Frame Aluminium 90° +5cm (327 mm)	FSTF 50	LLL 29 -> LLL 39	LLL 40 -> LLL 46	FSTF 51	LLL 29 -> LLL 38	LLL 40 -> LLL 47	FSTF 52	LLL 29 -> LLL 38	LLL 40 -> LLL 48	
	FSTF 51	LLL 29 -> LLL 40	LLL 41 -> LLL 47	FSTF 52	LLL 29 -> LLL 39	LLL 41 -> LLL 48	FSTF 53	LLL 29 -> LLL 39	LLL 41 -> LLL 49	
	FSTF 52	LLL 29 -> LLL 41	LLL 42 -> LLL 48	FSTF 53	LLL 29 -> LLL 40	LLL 42 -> LLL 49	FSTF 54	LLL 29 -> LLL 40	LLL 42 -> LLL 50	

ISPS1665374

Pos.	Part number	Description	Valid from → until	Qty
i	ISPDINFC1-0036	Knee to heel length (UL) in relationship to frame type, front seat height (SHv) and rear wheel size		1

Footrest angle adjustable for adducted frame

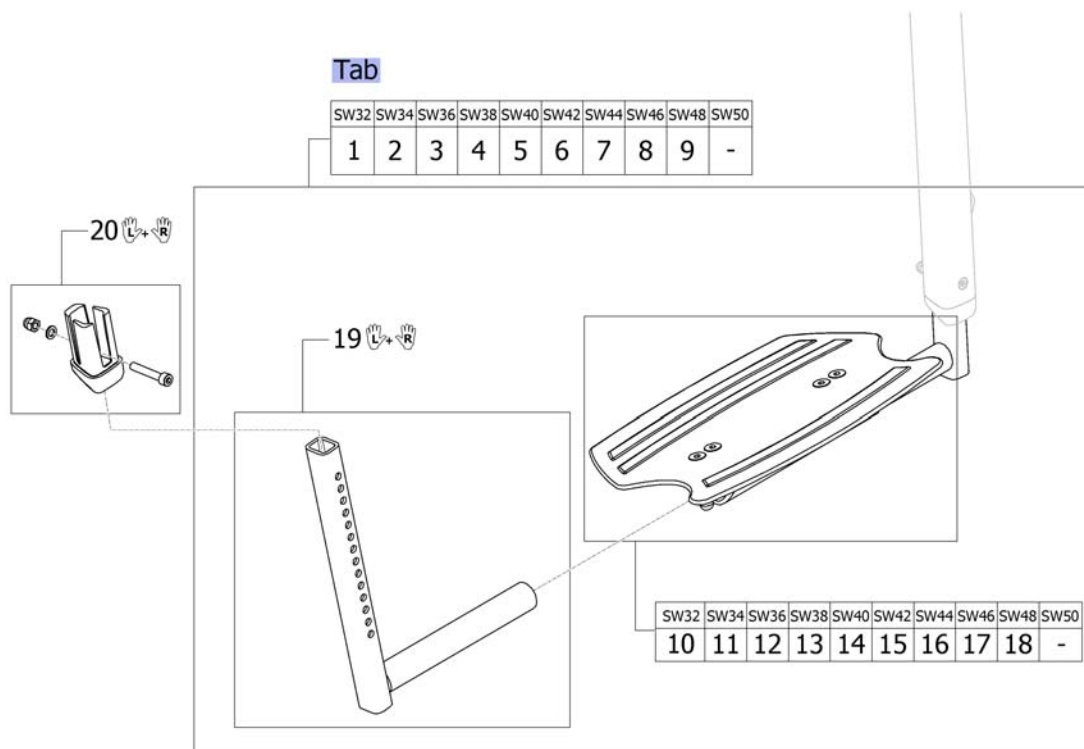


ISPS1665327

Pos.	Part number	Description	Valid from → until	Qty
Tab	-	Configuration table for footrest angle adjust. mounted on adducted frame		1
1	SP1662186	Footrest angle adjustable complete SW32 for adducted frame		1
2	SP1662187	Footrest angle adjustable complete SW34 for adducted frame		1
3	SP1662188	Footrest angle adjustable complete SW36 for adducted frame		1
4	SP1662218	Footrest angle adjustable complete SW38 for adducted frame		1
5	SP1662219	Footrest angle adjustable complete SW40 for adducted frame		1
6	SP1662220	Footrest angle adjustable complete SW42 for adducted frame		1
7	SP1662221	Footrest angle adjustable complete SW44 for adducted frame		1
8	SP1662222	Footrest angle adjustable complete SW46 for adducted frame		1
9	SP1662223	Footrest angle adjustable complete SW48 for adducted frame		1
10	SP1662224	Footplate with clamping part SW32 for adducted frame		1
11	SP1662225	footplate with clamping part SW34 for adducted frame		1
12	SP1662522	footplate with clamping part SW36 for adducted frame		1
13	SP1662523	footplate with clamping part SW38 for adducted frame		1
14	SP1662524	footplate with clamping part SW40 for adducted frame		1
15	SP1662525	footplate with clamping part SW42 for adducted frame		1
16	SP1662526	footplate with clamping part SW44 for adducted frame		1
17	SP1662527	footplate with clamping part SW46 for adducted frame		1
18	SP1662528	footplate with clamping part SW48 for adducted frame		1
19	SP1662530	Telescopic tube kit to footrest angle adjustable to adducted frame		1
20	SP1661492	Footplate sleeve Alu		1

Configuration table for footrest angle adjust. mounted on adducted frame


i	Rear Wheel 24"				Rear Wheel 25"				Rear Wheel 26"			
	FSTF	LLL	LLL	LLL	FSTF	LLL	LLL	LLL	FSTF	LLL	LLL	LLL
Frame Aluminium Adducted 75° (203,6 mm)	FSTF 48	LLL 29 -> LLL 36	LLL 37 -> LLL 45		FSTF 49	LLL 29 -> LLL 36	LLL 37 -> LLL 46		FSTF 50	LLL 29 -> LLL 36	LLL 37 -> LLL 47	
	FSTF 49	LLL 29 -> LLL 37	LLL 38 -> LLL 46		FSTF 50	LLL 29 -> LLL 37	LLL 38 -> LLL 47		FSTF 51	LLL 29 -> LLL 37	LLL 38 -> LLL 48	
	FSTF 50	LLL 29 -> LLL 38	LLL 39 -> LLL 47		FSTF 51	LLL 29 -> LLL 38	LLL 39 -> LLL 48		FSTF 52	LLL 29 -> LLL 38	LLL 39 -> LLL 49	
Frame Aluminium Adducted 90° (308,5 mm)	FSTF 48	LLL 29 -> LLL 36	LLL 39 -> LLL 44		FSTF 49	LLL 29 -> LLL 36	LLL 39 -> LLL 45		FSTF 50	LLL 29 -> LLL 36	LLL 39 -> LLL 46	
	FSTF 49	LLL 29 -> LLL 37	LLL 40 -> LLL 45		FSTF 50	LLL 29 -> LLL 37	LLL 40 -> LLL 46		FSTF 51	LLL 29 -> LLL 37	LLL 40 -> LLL 47	
	FSTF 50	LLL 29 -> LLL 38	LLL 41 -> LLL 46		FSTF 51	LLL 29 -> LLL 38	LLL 41 -> LLL 47		FSTF 52	LLL 29 -> LLL 38	LLL 41 -> LLL 48	

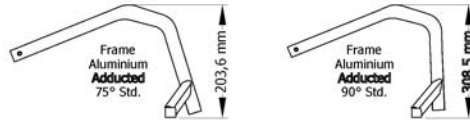

ISPS1665375

Pos.	Part number	Description	Valid from → until	Qty
i	ISPDINFC1-0036	Knee to heel length (UL) in relationship to frame type, front seat height (SHv) and rear wheel size		1


Footrest highmounted

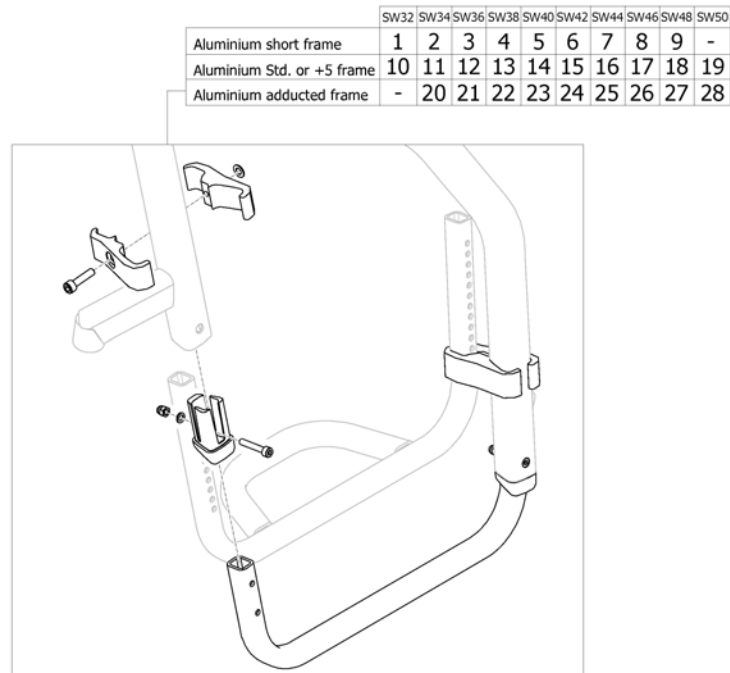

ISPS1665328

Pos.	Part number	Description	Valid from → until	Qty
1	SP1662531	Tube set for footrest, mounted in high position, SW32 for short frame		1
2	SP1662532	Tube set for footrest, mounted in high position, SW34 for short frame		1
3	SP1662533	Tube set for footrest, mounted in high position, SW36 for short frame		1
4	SP1662563	Tube set for footrest, mounted in high position, SW38 for short frame		1
5	SP1662564	Tube set for footrest, mounted in high position, SW40 for short frame		1
6	SP1662565	Tube set for footrest, mounted in high position, SW42 for short frame		1
7	SP1662566	Tube set for footrest, mounted in high position, SW44 for short frame		1
8	SP1662567	Tube set for footrest, mounted in high position, SW46 for short frame		1
9	SP1662568	Tube set for footrest, mounted in high position, SW48 for short frame		1
10	SP1662569	Tube set for footrest, mounted in high position, SW32 for Std. or +5 frame		1
11	SP1662570	Tube set for footrest, mounted in high position, SW34 for Std. or +5 frame		1
12	SP1662571	Tube set for footrest, mounted in high position, SW36 for Std. or +5 frame		1
13	SP1662572	Tube set for footrest, mounted in high position, SW38 for Std. or +5 frame		1
14	SP1662573	Tube set for footrest, mounted in high position, SW40 for Std. or +5 frame		1
15	SP1662574	Tube set for footrest, mounted in high position, SW42 for Std. or +5 frame		1
16	SP1662575	Tube set for footrest, mounted in high position, SW44 for Std. or +5 frame		1
17	SP1662576	Tube set for footrest, mounted in high position, SW46 for Std. or +5 frame		1
18	SP1662577	Tube set for footrest, mounted in high position, SW48 for Std. or +5 frame		1
19	SP1662578	Tube set for footrest, mounted in high position, SW50 for Std. or +5 frame		1
20	SP1662579	Tube set for footrest, mounted in high position, SW34 for adducted frame		1
21	SP1662580	Tube set for footrest, mounted in high position, SW36 for adducted frame		1
22	SP1662581	Tube set for footrest, mounted in high position, SW38 for adducted frame		1
23	SP1662582	Tube set for footrest, mounted in high position, SW40 for adducted frame		1
24	SP1662619	Tube set for footrest, mounted in high position, SW42 for adducted frame		1
25	SP1662620	Tube set for footrest, mounted in high position, SW44 for adducted frame		1
26	SP1662621	Tube set for footrest, mounted in high position, SW46 for adducted frame		1
27	SP1662622	Tube set for footrest, mounted in high position, SW48 for adducted frame		1
28	SP1662623	Tube set for footrest, mounted in high position, SW50 for adducted frame		1



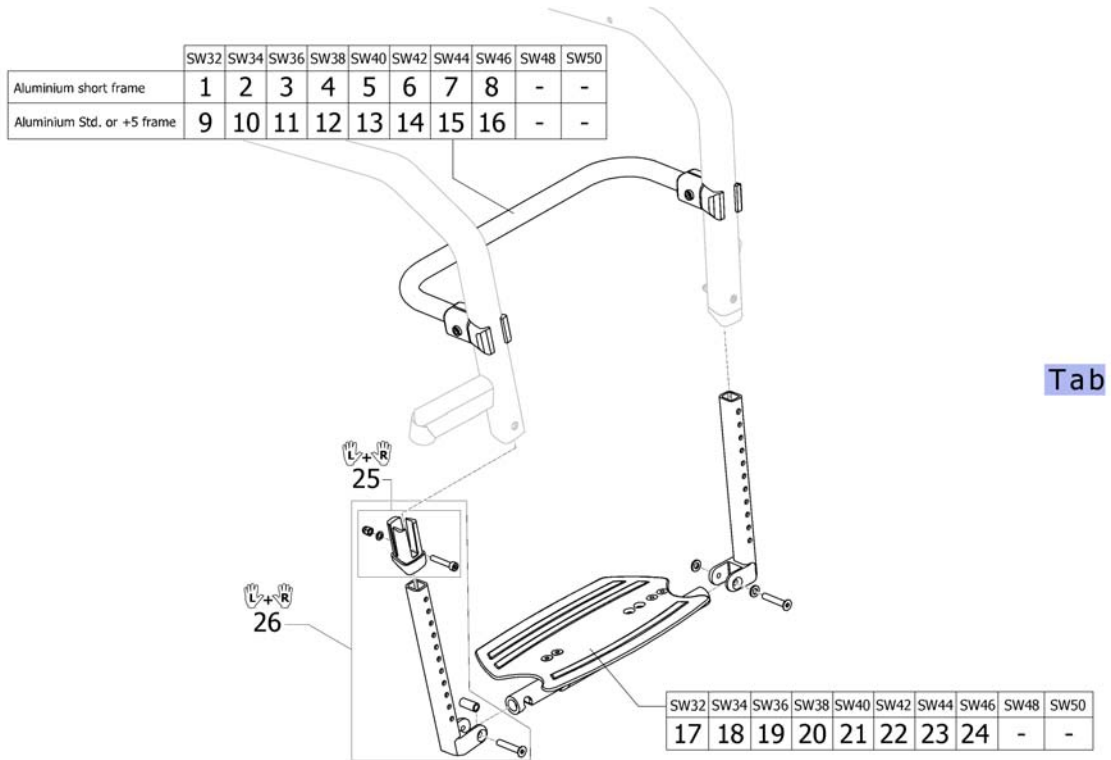




Footrest flip-to-side



11SPS1665329

Pos.	Part number	Description	Valid from → until	Qty
Tab	-	Configuration table for footrest flip-to-side		1
1	SP1662624	Reinforcement bar SW32 to Footrest flip to side for short frame		1
2	SP1662625	Reinforcement bar SW34 to Footrest flip to side for short frame		1
3	SP1662626	Reinforcement bar SW36 to Footrest flip to side for short frame		1
4	SP1662627	Reinforcement bar SW38 to Footrest flip to side for short frame		1
5	SP1662628	Reinforcement bar SW40 to Footrest flip to side for short frame		1
6	SP1662644	Reinforcement bar SW42 to Footrest flip to side for short frame		1
7	SP1662645	Reinforcement bar SW44 to Footrest flip to side for short frame		1
8	SP1662710	Reinforcement bar SW46 to Footrest flip to side for short frame		1
9	SP1662711	Reinforcement bar SW32 to Footrest flip to side		1
10	SP1662712	Reinforcement bar SW34 to Footrest flip to side		1
11	SP1662713	Reinforcement bar SW36 to Footrest flip to side		1
12	SP1662714	Reinforcement bar SW38 to Footrest flip to side		1
13	SP1662715	Reinforcement bar SW40 to Footrest flip to side		1
14	SP1662716	Reinforcement bar SW42 to Footrest flip to side		1
15	SP1662717	Reinforcement bar SW44 to Footrest flip to side		1
16	SP1662718	Reinforcement bar SW46 to Footrest flip to side		1
17	SP1642229	Footplate to footrest flip-to-side SB32		1
18	1547336	Footplate to footrest flip-to-side SB34		1
19	1547337	Footplate to footrest flip-to-side SB36		1
20	1547338	Footplate to footrest flip-to-side SB38		1
21	1547339	Footplate to footrest flip-to-side SB40		1
22	1547340	Footplate to footrest flip-to-side SB42		1
23	1547341	Footplate to footrest flip-to-side SB44		1
24	1547342	Footplate to footrest flip-to-side SB46		1
25	SP1661492	Footplate sleeve Alu		1
26	SP1662720	Telescopic tube kit to footrest flip-to-side		1


Configuration table for footrest flip-to-side

		Rear Wheel 24"	Rear Wheel 25"	Rear Wheel 26"				
 <p>Frame Aluminium 75° Short 236 mm</p>								
					Frame Aluminium 75° Std. 283,3 mm	FSTF 45 LLL 29 -> LLL 42 FSTF 46 LLL 30 -> LLL 43 FSTF 47 LLL 31 -> LLL 44	FSTF 46 LLL 29 -> LLL 43 FSTF 47 LLL 30 -> LLL 44 FSTF 48 LLL 31 -> LLL 45	FSTF 47 LLL 29 -> LLL 44 FSTF 48 LLL 30 -> LLL 45 FSTF 49 LLL 31 -> LLL 46
					Frame Aluminium 75° +5cm 302 mm	FSTF 48 LLL 33 -> LLL 45 FSTF 49 LLL 34 -> LLL 46 FSTF 50 LLL 35 -> LLL 47	FSTF 49 LLL 33 -> LLL 46 FSTF 50 LLL 34 -> LLL 47 FSTF 51 LLL 35 -> LLL 48	FSTF 50 LLL 33 -> LLL 47 FSTF 51 LLL 34 -> LLL 48 FSTF 52 LLL 35 -> LLL 49
 <p>Frame Aluminium 90° Short 261,1 mm</p>	Frame Aluminium 75° Std. (283,3 mm)	FSTF 50 LLL 35 -> LLL 47 FSTF 51 LLL 36 -> LLL 48 FSTF 52 LLL 37 -> LLL 49	FSTF 51 LLL 35 -> LLL 48 FSTF 52 LLL 36 -> LLL 49 FSTF 53 LLL 37 -> LLL 50	FSTF 52 LLL 35 -> LLL 49 FSTF 53 LLL 36 -> LLL 50 FSTF 54 LLL 37 -> LLL 51				
					Frame Aluminium 90° Std. 308,3 mm	FSTF 45 LLL 29 -> LLL 41 FSTF 46 LLL 30 -> LLL 42 FSTF 47 LLL 31 -> LLL 43	FSTF 46 LLL 29 -> LLL 42 FSTF 47 LLL 30 -> LLL 43 FSTF 48 LLL 31 -> LLL 44	FSTF 47 LLL 29 -> LLL 43 FSTF 48 LLL 30 -> LLL 44 FSTF 49 LLL 31 -> LLL 45
					Frame Aluminium 90° +5cm 327 mm	FSTF 48 LLL 34 -> LLL 44 FSTF 49 LLL 35 -> LLL 45 FSTF 50 LLL 36 -> LLL 46	FSTF 49 LLL 34 -> LLL 45 FSTF 50 LLL 35 -> LLL 46 FSTF 51 LLL 36 -> LLL 47	FSTF 50 LLL 34 -> LLL 46 FSTF 51 LLL 35 -> LLL 47 FSTF 52 LLL 36 -> LLL 48
 <p>Frame Aluminium 90° +5cm 327 mm</p>	Frame Aluminium 90° Std. (308,3 mm)	FSTF 51 LLL 36 -> LLL 47 FSTF 52 LLL 37 -> LLL 48 FSTF 53 LLL 38 -> LLL 49	FSTF 52 LLL 36 -> LLL 48 FSTF 53 LLL 37 -> LLL 49 FSTF 54 LLL 38 -> LLL 50	FSTF 53 LLL 36 -> LLL 48 FSTF 54 LLL 37 -> LLL 49 FSTF 54 LLL 38 -> LLL 50				
					Frame Aluminium 90° Std. 308,3 mm	FSTF 48 LLL 34 -> LLL 44 FSTF 49 LLL 35 -> LLL 45 FSTF 50 LLL 36 -> LLL 46	FSTF 49 LLL 34 -> LLL 45 FSTF 50 LLL 35 -> LLL 46 FSTF 51 LLL 36 -> LLL 47	FSTF 50 LLL 34 -> LLL 46 FSTF 51 LLL 35 -> LLL 47 FSTF 52 LLL 36 -> LLL 48
					Frame Aluminium 90° +5cm 327 mm	FSTF 50 LLL 36 -> LLL 46 FSTF 51 LLL 37 -> LLL 47 FSTF 52 LLL 38 -> LLL 48	FSTF 51 LLL 36 -> LLL 47 FSTF 52 LLL 37 -> LLL 48 FSTF 53 LLL 38 -> LLL 49	FSTF 52 LLL 36 -> LLL 48 FSTF 53 LLL 37 -> LLL 49 FSTF 54 LLL 38 -> LLL 50

ISPS1665376

Pos.	Part number	Description	Valid from → until	Qty
i	ISPDINFC1-0036	Knee to heel length (UL) in relationship to frame type, front seat height (SHv) and rear wheel size		1

Calf strap

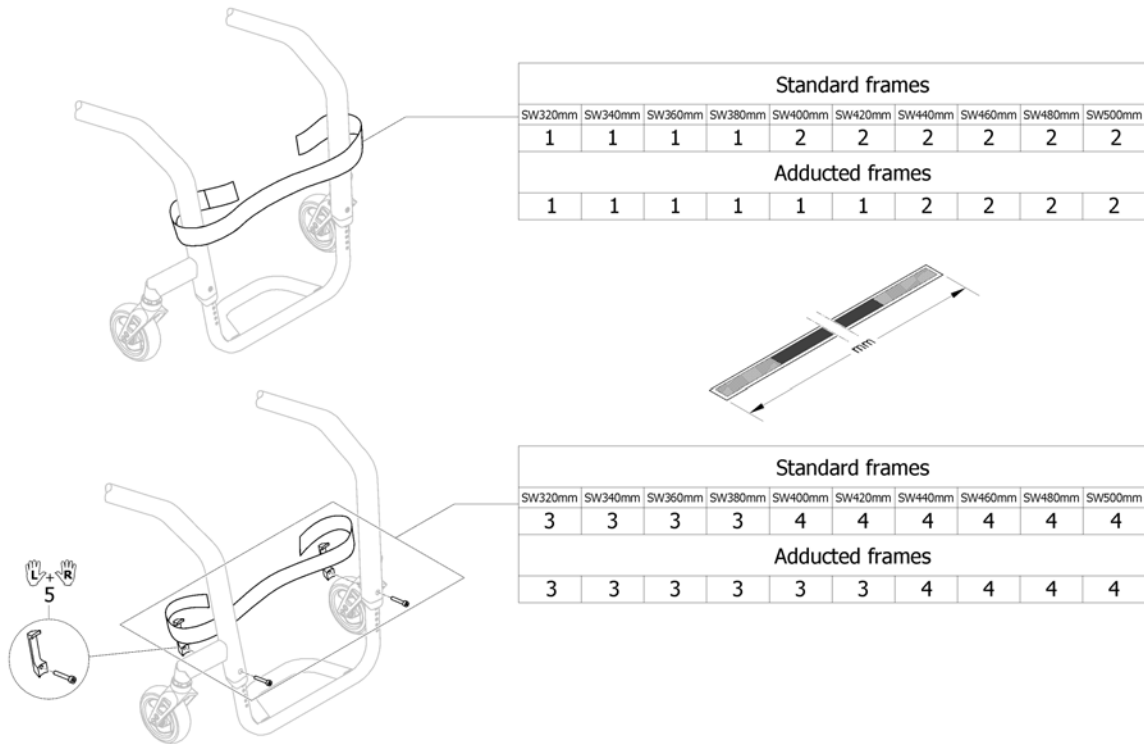


Standard frames									
SW320mm	SW340mm	SW360mm	SW380mm	SW400mm	SW420mm	SW440mm	SW460mm	SW480mm	SW500mm
1	1	1	1	2	2	2	2	2	2
Adducted frames									
1	1	1	1	1	1	2	2	2	2

Standard frames									
SW320mm	SW340mm	SW360mm	SW380mm	SW400mm	SW420mm	SW440mm	SW460mm	SW480mm	SW500mm
3	3	3	3	4	4	4	4	4	4
Adducted frames									
3	3	3	3	3	3	4	4	4	4

!!SPS1665230

Pos.	Part number	Description	Valid from → until	Qty
1	1529338	Calf strap medium complete		1
2	1561529	Calf strap complete, 940mm		1
3	SP1665833	Calf strap assembled to the supporter, size S		1
4	SP1665834	Calf strap assembled to the supporter, size M		1
5	SP1665086	Calfstrap holder, pair		1


Clothes-guards carbon

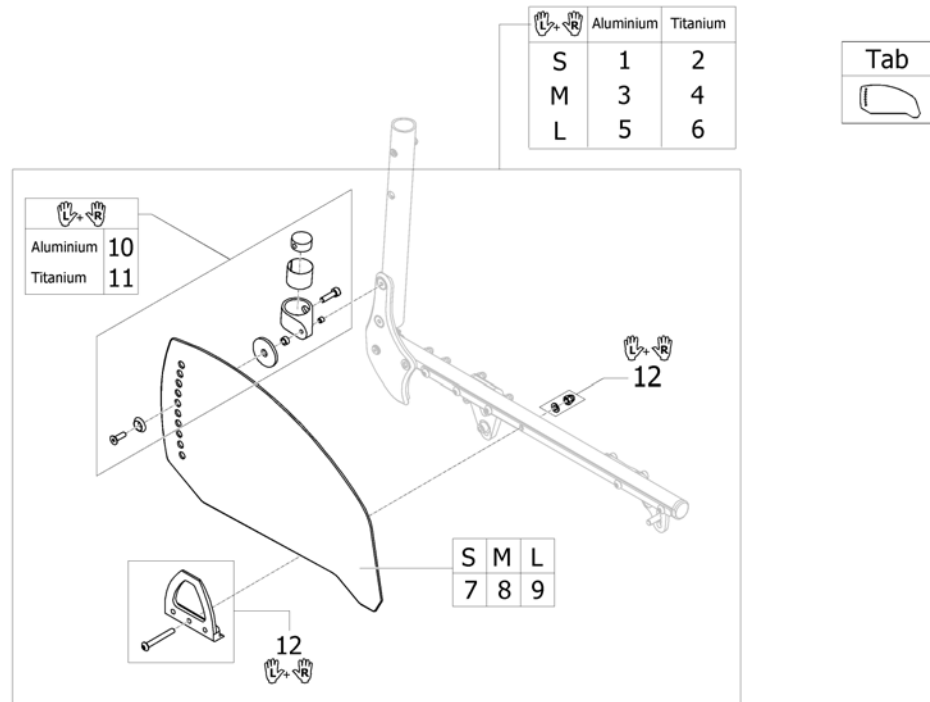

ISPS1665331

Pos.	Part number	Description	Valid from → until	Qty
Tab	-	Configuration table clothes-guard carbon		1
1	SP1662722	Clothes-guards carbon M with fixation kit to backrest Aluminium, pair		1
2	SP1662723	Clothes-guards carbon M with fixation kit to backrest Titan, pair		1
3	SP1662724	Clothes-guards carbon L with fixation kit to backrest Aluminium, pair		1
4	SP1662725	Clothes-guards carbon L with fixation kit to backrest Titan, pair		1
5	SP1664865	Clothes-guards carbon XL with fixation kit to backrest Aluminium, pair		1
6	SP1664866	Clothes-guards carbon XL with fixation kit to backrest Titan, pair		1
7	SP1662733	Clothes-guard carbon M		1
8	SP1662734	Clothes-guard carbon L		1
9	SP1664867	Clothes-guard carbon XL		1
10	1547452	Fixation kit clothes-guard carbon to backrest Aluminium		1
11	1547453	Fixation kit clothes-guard carbon to backrest Titanium		1
12	1547456	Fixation clothes-guard to frame, pair		1

Configuration table clothes-guard carbon

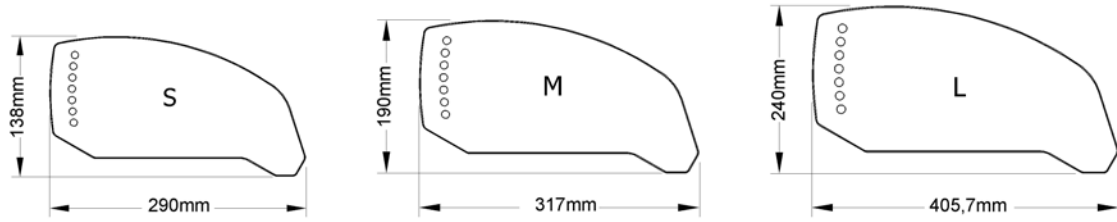


Position	24" Rear wheel					25" Rear wheel					26" Rear wheel				
	1	2	3	4	5	1	2	3	4	5	1	2	3	4	5
RSTF 380	L	M	M	M	M	---	---	---	---	---	---	---	---	---	---
RSTF 390	M	M	M	M	M	L	L	L	M	M	---	---	---	---	---
RSTF 400	M	M	M	M	M	L	M	M	M	M	L	L	L	L	L
RSTF 410	M	M	M	M	M	M	M	M	M	M	L	L	L	L	L
RSTF 420	M	M	M	M	M	M	M	M	M	M	L	L	L	M	M
RSTF 430	M	M	M	M	M	M	M	M	M	M	L	M	M	M	M
RSTF 440	M	M	M	S	S	M	M	M	M	M	M	M	M	M	M
RSTF 450	M	S	S	S	S	M	M	M	S	S	M	M	M	M	M
RSTF 460	S	S	S	S	S	M	S	S	S	S	M	M	M	M	M
RSTF 470	S	S	S	S	S	S	S	S	S	S	M	M	M	M	M
RSTF 480	S	S	S	S	S	S	S	S	S	S	M	M	M	M	M
RSTF 490	---	---	---	---	---	S	S	S	S	S	M	M	M	S	S
RSTF 500	---	---	---	---	---	---	---	---	---	---	M	S	S	S	S

SPS1665331

Pos.	Part number	Description	Valid from → until	Qty
M	!SPDINFC1-0022	Clothes-guard Size M		1
L	!SPDINFC1-0024	Clothes-guard Size L		1
XL	!SPDINFC1-0023	Clothes-guard Size M		1


Clothes-guards economy

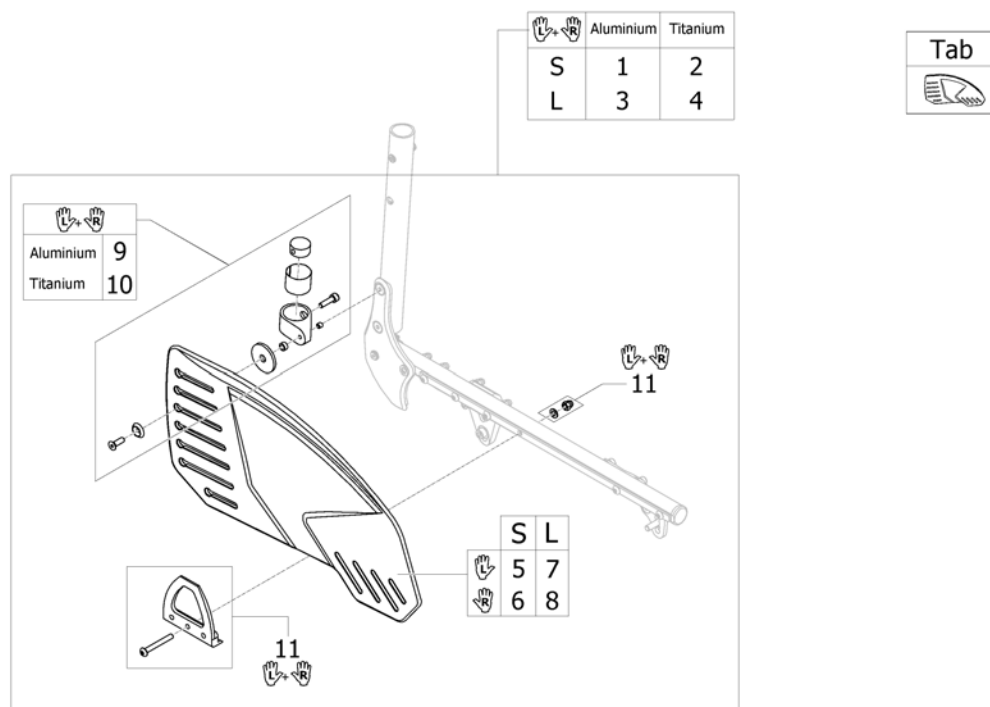

ISPS1665332

Pos.	Part number	Description	Valid from → until	Qty
Tab	-	Configuration table clothes-guard economy		1
1	SP1662735	Clothes-guards economy M with fixation kit to backrest Aluminium, pair		1
2	SP1662736	Clothes-guards economy M with fixation kit to backrest Titan, pair		1
3	SP1662737	Clothes-guards economy L with fixation kit to backrest Aluminium, pair		1
4	SP1662738	Clothes-guards economy L with fixation kit to backrest Titan, pair		1
5	SP1662739	Clothes-guard economy M left		1
6	SP1662740	Clothes-guard economy M right		1
7	SP1662775	Clothes-guard economy L left		1
8	SP1662776	Clothes-guard economy L right		1
9	1547454	Fixation kit clothes-guard economy to backrest Aluminium		1
10	1547455	Fixation kit clothes-guard economy to backrest Titanium		1
11	1547456	Fixation clothes-guard to frame, pair		1

Configuration table clothes-guard economy

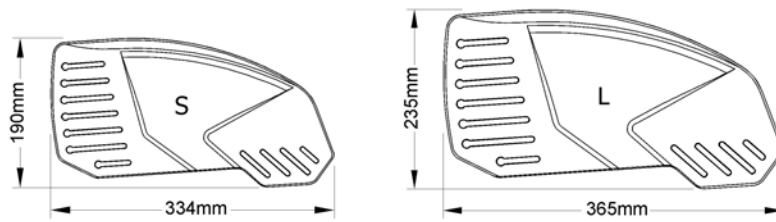


Position	24" Rear wheel					25" Rear wheel					26" Rear wheel				
	1	2	3	4	5	1	2	3	4	5	1	2	3	4	5
RSTF 380	L	L	L	L	L	---	---	---	---	---	---	---	---	---	---
RSTF 390	L	L	L	L	L	L	L	L	L	L	---	---	---	---	---
RSTF 400	L	L	L	L	L	L	L	L	L	L	L	L	L	L	L
RSTF 410	L	L	L	S	S	L	L	L	L	L	L	L	L	L	L
RSTF 420	L	S	S	S	S	L	L	L	L	L	L	L	L	L	L
RSTF 430	S	S	S	S	S	L	L	L	S	S	L	L	L	L	L
RSTF 440	S	S	S	S	S	L	S	S	S	S	L	L	L	L	L
RSTF 450	S	S	S	S	S	S	S	S	S	S	L	L	L	S	S
RSTF 460	S	S	S	S	S	S	S	S	S	S	L	S	S	S	S
RSTF 470	S	S	S	S	S	S	S	S	S	S	S	S	S	S	S
RSTF 480	S	S	S	S	S	S	S	S	S	S	S	S	S	S	S
RSTF 490	---	---	---	---	---	S	S	S	S	S	S	S	S	S	S
RSTF 500	---	---	---	---	---	---	---	---	---	---	S	S	S	S	S

SPS1.665378

Pos.	Part number	Description	Valid from → until	Qty
M	!SPDINFC1-0022	Clothes-guard Size M		1
L	!SPDINFC1-0024	Clothes-guard Size L		1


Mudguards carbon

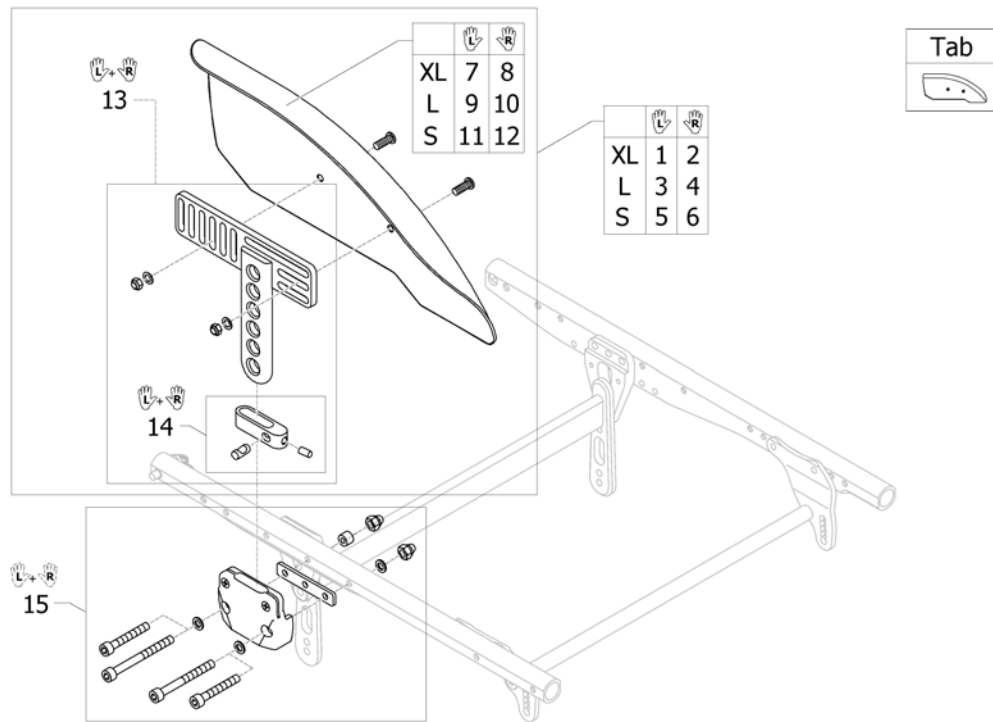

ISPS1665333

Pos.	Part number	Description	Valid from → until	Qty
Tab	-	Configuration and size table mudguards carbon		1
1	1553158	Mudguard carbon left XL short		1
2	1553203	Mudguard carbon right XL short		1
3	1547480	Mudguard carbon left L short		1
4	1547484	Mudguard carbon right L short		1
5	1547479	Mudguard carbon left M short		1
6	1547483	Mudguard carbon right M short		1
7	1553137	Mudguard carbon left XL		1
8	1553138	Mudguard carbon right XL		1
9	1547534	Mudguard carbon left L		1
10	1547536	Mudguard carbon right L		1
11	1547533	Mudguard carbon left M		1
12	1547535	Mudguard carbon right M		1
13	1547488	Mudguard carbon holder short, pair		1
14	1547537	Siderests height clamp kit, Pair		1
15	1547487	Fixationset mudguard frame		1

Configuration and size table mudguards carbon

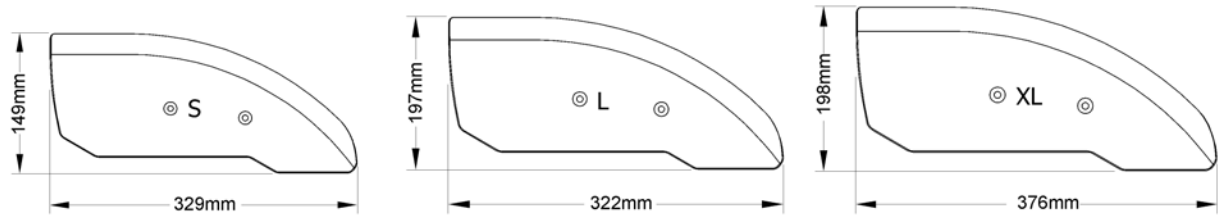


Position	24" Rear wheel					25" Rear wheel					26" Rear wheel				
	1	2	3	4	5	1	2	3	4	5	1	2	3	4	5
RSTF 380	XL	XL	XL	L	L	---	---	---	---	---	---	---	---	---	---
RSTF 390	XL	L	L	L	L	XL	XL	XL	XL	XL	---	---	---	---	---
RSTF 400	L	L	L	L	L	XL	XL	XL	XL	XL	XL	XL	XL	XL	XL
RSTF 410	L	L	L	L	L	XL	XL	XL	L	L	XL	XL	XL	XL	XL
RSTF 420	L	L	L	L	L	XL	L	L	L	L	XL	XL	XL	XL	XL
RSTF 430	L	L	L	S	S	L	L	L	L	L	XL	XL	XL	XL	XL
RSTF 440	L	S	S	S	S	L	L	L	L	L	XL	XL	XL	L	L
RSTF 450	S	S	S	S	S	L	L	L	S	S	XL	L	L	L	L
RSTF 460	S	S	S	S	S	L	S	S	S	S	L	L	L	L	L
RSTF 470	S	S	S	S	S	S	S	S	S	S	L	L	L	L	L
RSTF 480	S	S	S	S	S	S	S	S	S	S	L	L	L	L	L
RSTF 490	---	---	---	---	---	S	S	S	S	S	L	L	L	S	S
RSTF 500	---	---	---	---	---	---	---	---	---	---	L	S	S	S	S

SPS1.665379

Pos.	Part number	Description	Valid from → until	Qty
M	!SPDINFC1-0016	Mudguard carbon Size M		1
L	!SPDINFC1-0018	Mudguard carbon Size L		1
XL	!SPDINFC1-0019	Mudguard carbon Size XL		1


Mudguards economy

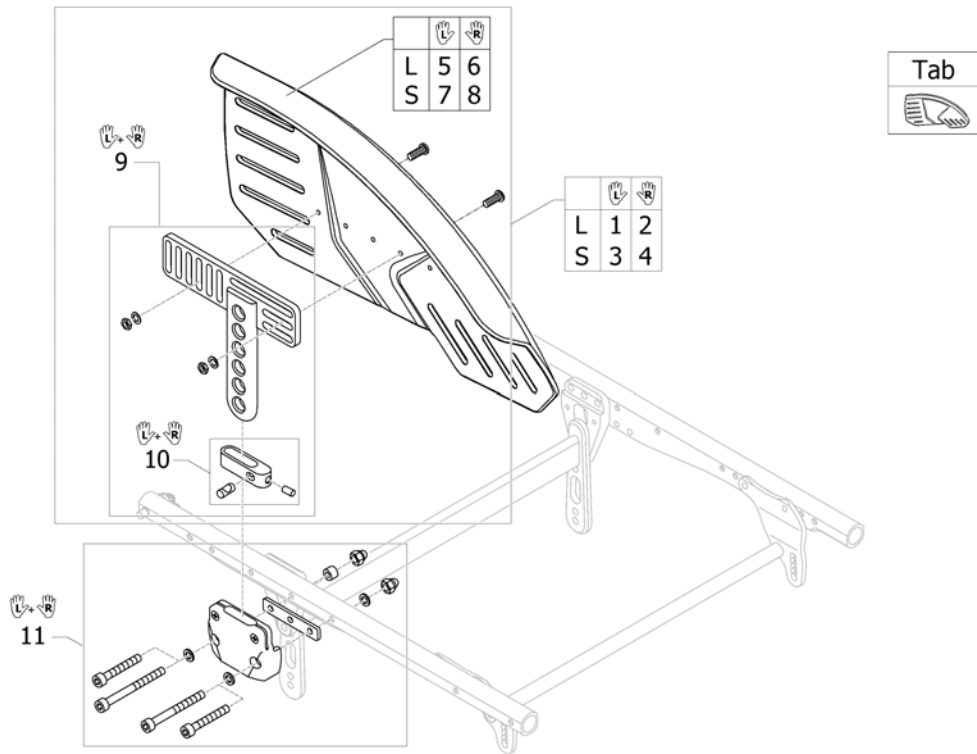

ISPS1665334

Pos.	Part number	Description	Valid from → until	Qty
1	-	Configuration table mudguards Economy		1
2	1547547	Mudguard Economy left L short		1
3	1547551	Mudguard Economy right L short		1
4	1547546	Mudguard Economy left M short		1
5	1547550	Mudguard Economy right M short		1
6	1547578	Mudguard Economy left L		1
7	1547580	Mudguard Economy right L		1
8	1547556	Mudguard Economy left M		1
9	1547579	Mudguard Economy right M		1
9	1547554	Mudguard Economy holder short, pair		1
10	1547537	Siderests height clamp kit, Pair		1
11	1547487	Fixationset mudguard frame		1

Configuration table mudguards Economy

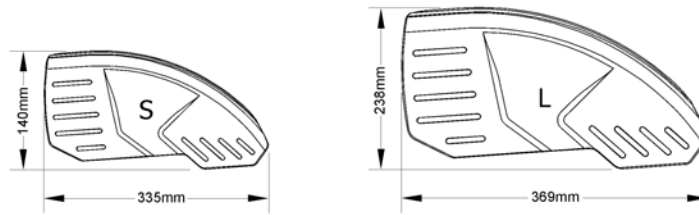


	24" Rear wheel					25" Rear wheel					26" Rear wheel				
Position	1	2	3	4	5	1	2	3	4	5	1	2	3	4	5
RSTF 380	L	L	L	L	L	---	---	---	---	---	---	---	---	---	---
RSTF 390	L	L	L	L	L	L	L	L	L	L	---	---	---	---	---
RSTF 400	L	L	S	S	S	L	L	L	L	L	L	L	L	L	L
RSTF 410	S	S	S	S	S	L	L	L	L	L	L	L	L	L	L
RSTF 420	S	S	S	S	S	L	L	S	S	S	L	L	L	L	L
RSTF 430	S	S	S	S	S	S	S	S	S	S	L	L	L	L	L
RSTF 440	S	S	S	S	S	S	S	S	S	S	L	L	S	S	S
RSTF 450	S	S	S	S	S	S	S	S	S	S	S	S	S	S	S
RSTF 460	S	S	S	S	S	S	S	S	S	S	S	S	S	S	S
RSTF 470	S	S	S	S	S	S	S	S	S	S	S	S	S	S	S
RSTF 480	S	S	S	S	S	S	S	S	S	S	S	S	S	S	S
RSTF 490	---	---	---	---	---	S	S	S	S	S	S	S	S	S	S
RSTF 500	---	---	---	---	---	---	---	---	---	---	S	S	S	S	S

SPS1.665380

Pos.	Part number	Description	Valid from → until	Qty
M	!SPDINFC1-0016	Mudguard carbon Size M		1
L	!SPDINFC1-0018	Mudguard carbon Size L		1


Siderest height adjustable

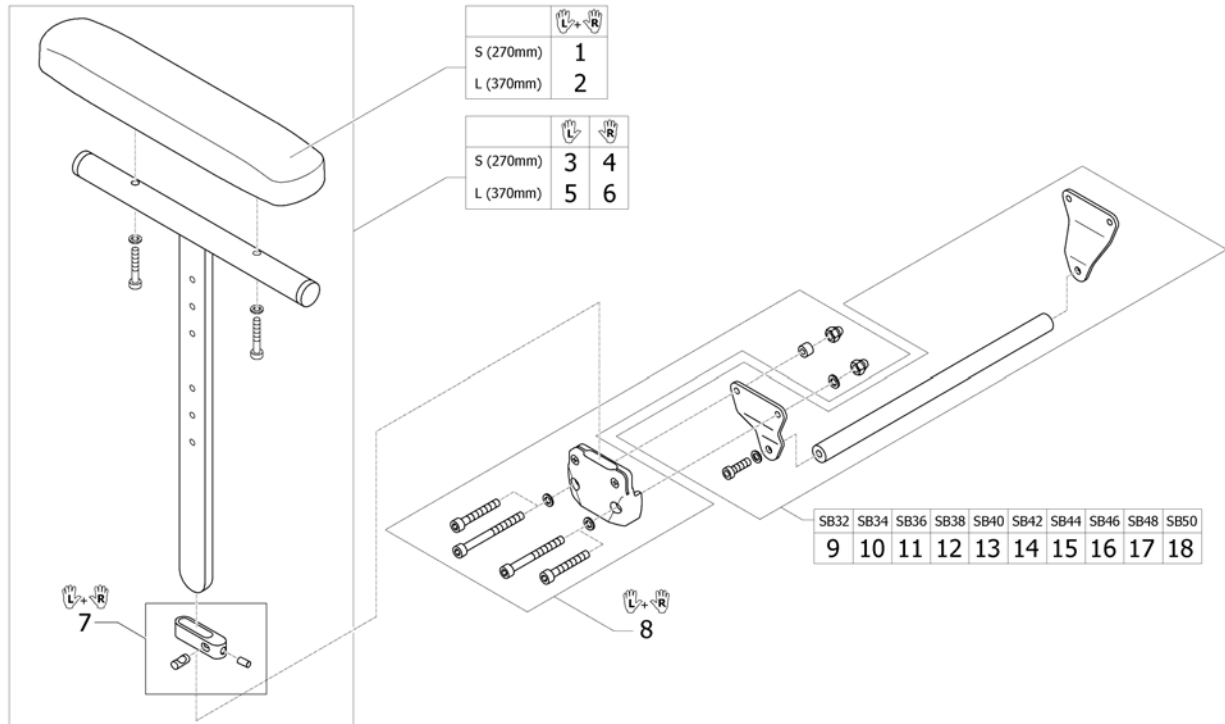

ISPS1665335

Pos.	Part number	Description	Valid from → until	Qty
1	1439497	Armpad S (270mm) incl. screws, pair		1
2	1439498	Armpad L (370mm) incl. screws, pair		1
3	1517678	Siderest height adjustable short, left		1
4	1517677	Siderest height adjustable short, right		1
5	1517680	Siderest height adjustable long, left		1
6	1517679	Siderest height adjustable long, right		1
7	1547537	Siderests height clamp kit, Pair		1
8	1505391	Fixation Siderest-to-Frame, pair		1
9	SP1642471	Bracket kit for siderest SB32		1
10	1547602	Bracket kit for siderest SB34		1
11	1547603	Bracket kit for siderest SB36		1
12	1547604	Bracket kit for siderest SB38		1
13	1547605	Bracket kit for siderest SB40		1
14	1547606	Bracket kit for siderest SB42		1
15	1547607	Bracket kit for siderest SB44		1
16	1547608	Bracket kit for siderest SB46		1
17	1547609	Bracket kit for siderest SB48		1
18	SP1642604	Bracket kit for siderest SB50		1


Tubular armrest

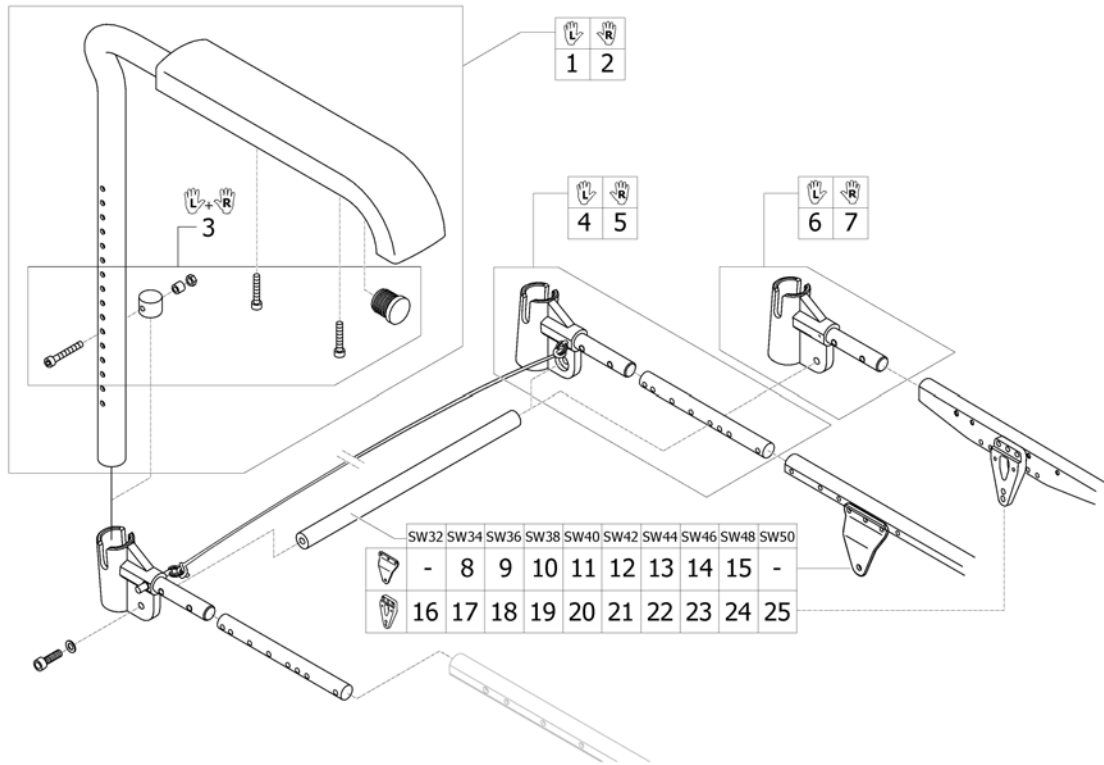

ISPS1665336

Pos.	Part number	Description	Valid from → until	Qty
1	913.1013	Armrest, with standard cushion, left		1
2	913.1012	Armrest with standard cushion, right		1
3	1547610	Boltset Armrest		1
4	1547612	Armrest adapter left incl. Fixation and cord		1
5	1547611	Armrest adapter right incl. Fixation and cord		1
6	SP1642302	Armrest adapter left incl. Fixation and cord		1
7	SP1642303	Armrest adapter right incl. Fixation and cord		1
8	1547613	Connection tube for armrest SB34		1
9	1547614	Connection tube for armrest SB36		1
10	1547615	Connection tube for armrest SB38		1
11	1547616	Connection tube for armrest SB40		1
12	1547617	Connection tube for armrest SB42		1
13	1547618	Connection tube for armrest SB44		1
14	1547619	Connection tube for armrest SB46		1
15	1547620	Connection tube for armrest SB48		1
16	SP1642304	Connection tube for armrest SB32		1
17	SP1642305	Connection tube for armrest SB34		1
18	SP1642306	Connection tube for armrest SB36		1
19	SP1642307	Connection tube for armrest SB38		1
20	SP1642308	Connection tube for armrest SB40		1
21	SP1642309	Connection tube for armrest SB42		1
22	SP1642310	Connection tube for armrest SB44		1
23	SP1642311	Connection tube for armrest SB46		1
24	SP1642312	Connection tube for armrest SB48		1
25	SP1642313	Connection tube for armrest SB50		1


Front wheel supporter

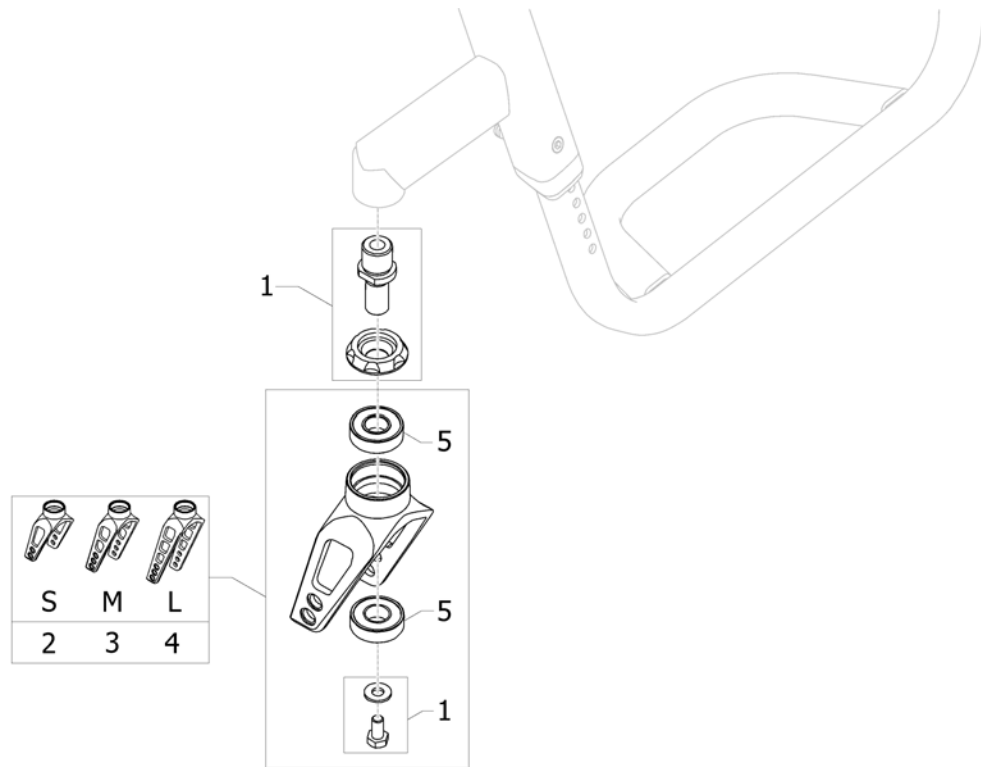

ISPS1665337

Pos.	Part number	Description	Valid from → until	Qty
1	SP1665835	Front fork pin kit K2.0, pair		1
2	SP1659452-0004	Front fork S, Black anodized, Pair		1
3	SP1659453-0004	Front fork M, Black anodized, Pair		1
4	SP1659454-0004	Fourches avant L, noir anodisé, paire		1
5	SP1659456	Bearing kit for front fork, pair		1


Front wheels

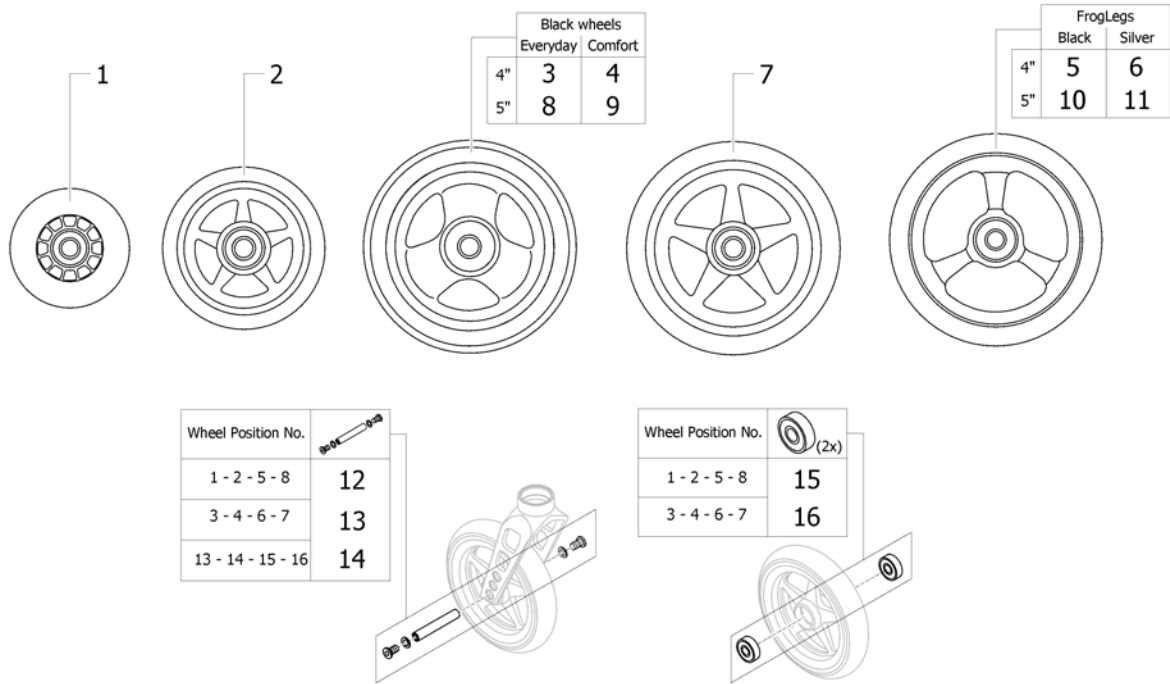

ISPS1665338

Pos.	Part number	Description	Valid from → until	Qty
1	1513180	3" Sports wheel, transparent		1
2	1513183-0004	4" Starec wheel, black anodized		1
3	SP1638070	4" Everyday wheel, black PU, pair		1
4	SP1638072	4" Comfort wheel, black PU, pair		1
5	SP1638082	4" x1.42 (36mm) FrogLegs EPIC Aluminum Wide Softroll Caster - Black hub		1
6	SP1638083	4" x1.42 (36mm) FrogLegs EPIC Aluminum Wide Softroll Caster - Silver hub		1
7	1513187-0004	5" Starec wheel, black anodized		1
8	SP1638071	5" Everyday wheel, black PU, pair		1
9	SP1638073	5" Comfort wheel, black PU, pair		1
10	SP1638084	5" x1.42 (36mm) FrogLegs EPIC Aluminum Wide Softroll Caster - Black hub		1
11	SP1638085	5" x1.42 (36mm) FrogLegs EPIC Aluminum Wide Softroll Caster - Silver hub		1
12	1420972	Axle set complete 3" Sport / 4" and 5" Starec / 4" Skater wheel		1
13	1570108	Axle set without spacer		1
14	SP1638076	Axle set complete 4" and 5" FrogLegs wheels		1
15	1570104	Ball bearing 608-2RS 8/22*7, pair		1
16	1570105	Ball bearing 6000-2RS, pair		1


Starec rear wheels complete

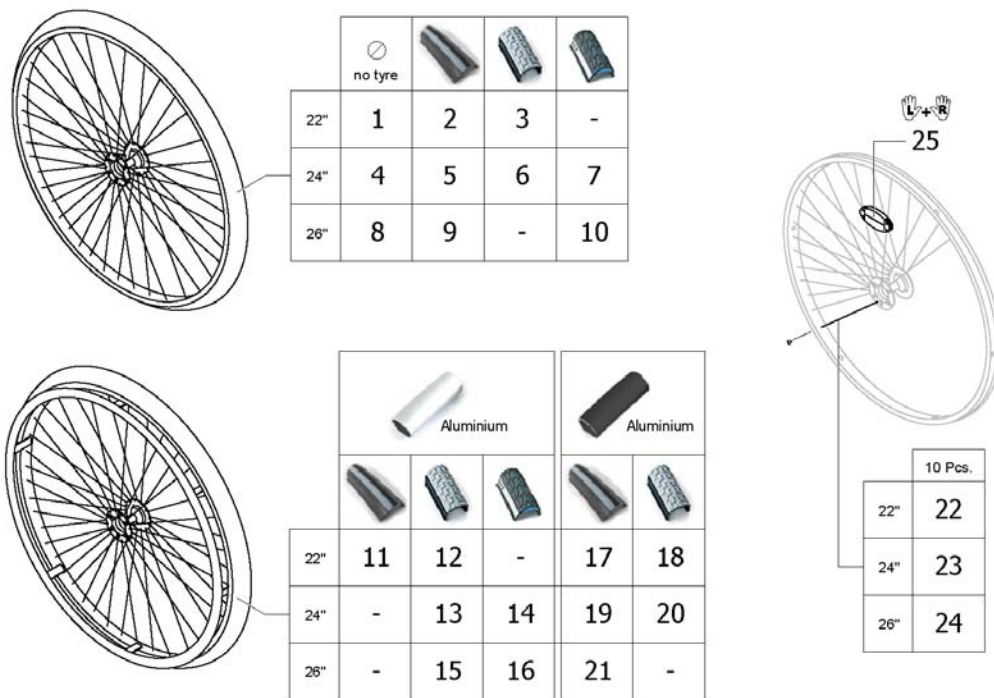

151951665339

Pos.	Part number	Description	Valid from → until	Qty
1	1559083-0407	Rear wheel 22", Starec X, colourless, without handrim, without tyre		2
2	SP1601741	Rear wheel 22", Starec hub X, without handrim, Rightrun black/grey		2
3	904.1139	Rear wheel 22", Starec hub X, without handrim, Tread		2
4	1567104-0407	Rear wheel 24", Starec X, colourless, without handrim, without tyre		2
5	SP1601745	Rear wheel 24", Starec hub X, without handrim, Rightrun black/grey tyre		2
6	904.1141	Rear wheel 24", Starec hub X, without handrim, Tread		2
7	1554399-0407	Rear wheel 24", Starec X, colourless, without handrim, marathon		2
8	1567099-0407	Rear wheel 26", Starec X, colourless, without handrim, without tyre		2
9	SP1601749	Rear wheel 26", Starec hub X, without handrim, Rightrun black/grey		2
10	1554403-0407	Rear wheel 26", Starec X, colourless, without handrim, marathon		2
11	SP1601743	Rear wheel 22", Starec hub X, handrim, Rightrun black/grey		2
12	904.1123	Rear wheel 22", Starec hub X, handrim, Tread		2
13	904.1125	Rear wheel 24", Starec hub X, handrim, Tread		2
14	1554400-0407	Rear wheel 24", Starec X, colourless, handrim, marathon evolution		2
15	SP1601751	Rear wheel 26", Starec hub X, handrim, Rightrun black/grey		2
16	1554417-0407	Rear wheel 26", Starec X, colourless, handrim, marathon evolution		2
17	SP1601860	Rear wheel 22", Starec hub X, handrim aluminium black, Rightrun black/grey		2
18	1562656	Rear wheel 22", Starec hub X, handrim aluminium black, Tread		2
19	SP1601864	Rear wheel 24", Starec hub X, handrim aluminium black, Rightrun black/grey		2
20	1562660	Rear wheel 24", Starec hub X, handrim aluminium black, Tread		2
21	SP1601866	Rear wheel 26", Starec hub X, handrim aluminium black, Rightrun black/grey		2
22	1579082	Spokes 228mm complete, kit 10 Pcs.		1
23	1579086	Spokes 253mm complete, kit 10 Pcs.		1
24	1579090	Spokes 278mm complete, kit 10 Pcs.		1
25	1571073	Spokereflection, pair		1


High Performance wheel

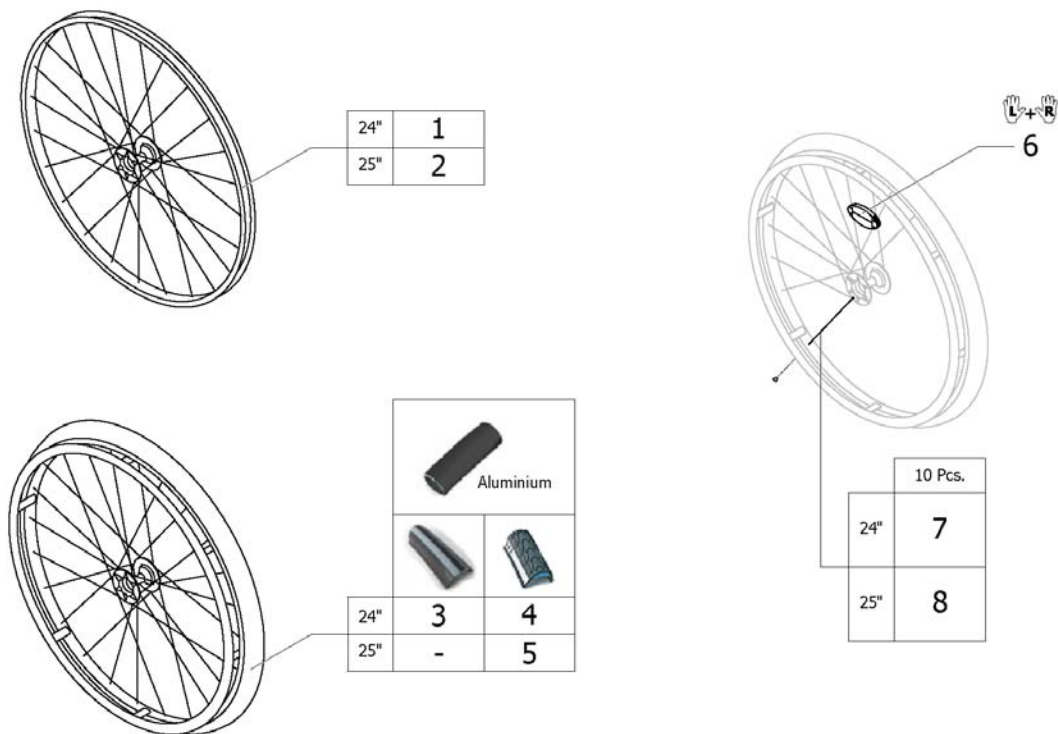

151951665341

Pos.	Part number	Description	Valid from → until	Qty
1	1517723-0004	Wheel 24", High Performance, black, without handrim, without tyre		2
2	1562937-0004	Wheel 25", High Performance, black, without handrim, without tyre		2
3	SP1601752-0004	Rear wheel 24" High Performance black anodized, handrim alu black, Rightrun black/grey		2
4	1562670-0004	Wheel 24", High Performance, black, handrim aluminium black, tyre Marathon Evolution		2
5	1562669-0004	Wheel 25", High Performance, black, handrim aluminium black, tyre Marathon Evolution		2
6	1571073	Spokereflection, pair		1
7	1578867	Spokes kit for High Performance wheel 24", 10 Pcs.		1
8	1578868	Spokes kit for High Performance wheel 25", 10 Pcs.		1


Spinergy LXX wheel

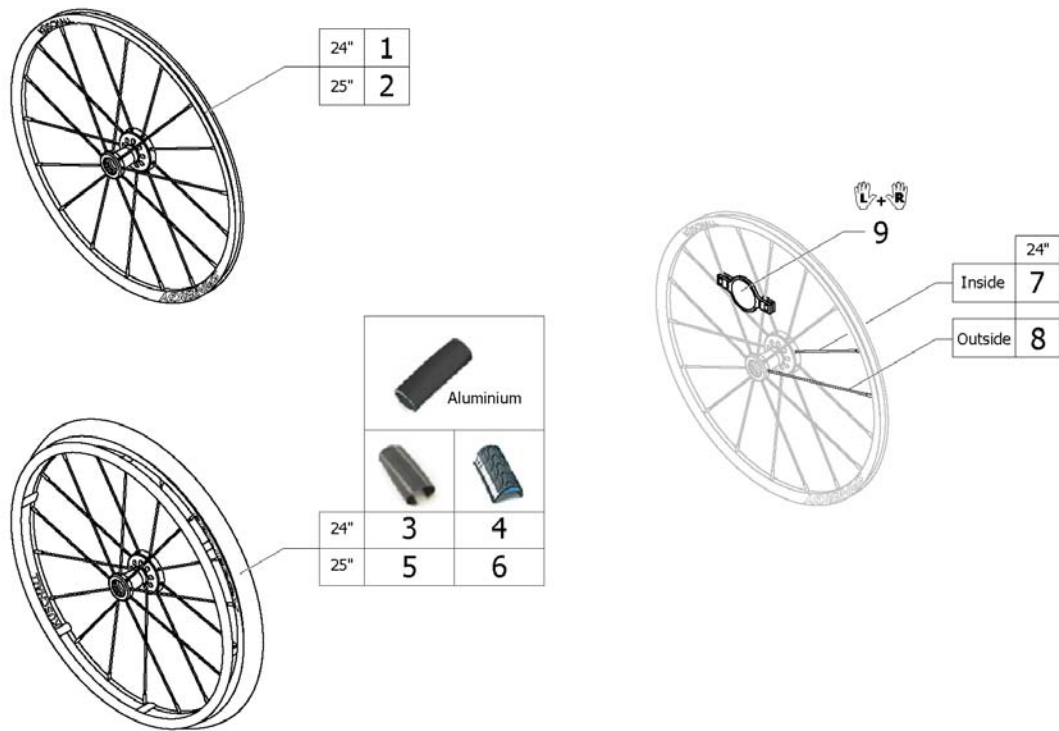

151951665343

Pos.	Part number	Description	Valid from → until	Qty
1	1579102	Rear wheel 24", Spinergy LXX, without handrim, without tyre		2
2	SP1638079	Rear wheel 25" Spinergy LXX, without handrim, without tyre		2
3	1579101	Rear wheel 24", Spinergy LXX, handrim aluminium black, tyre Schwalbe One		2
4	1579105	Rear wheel 24", Spinergy LXX, handrim aluminium black, tyre Marathon Evolution		2
5	SP1638077	Rear wheel 25" Spinergy LXX, handrim aluminium black, tyre Schwalbe One		2
6	SP1638078	Rear wheel 25" Spinergy LXX, handrim aluminium black, tyre Marathon Evolution		2
7	1579103	Spoke inside for Spinergy LXX wheel 24", 236mm., 1 Pcs.		9
8	1579104	Spoke outside for Spinergy LXX wheel 24", 249mm., 1 Pcs.		9
9	1571088	Spoke reflection for Spinergy wheel, pair		1



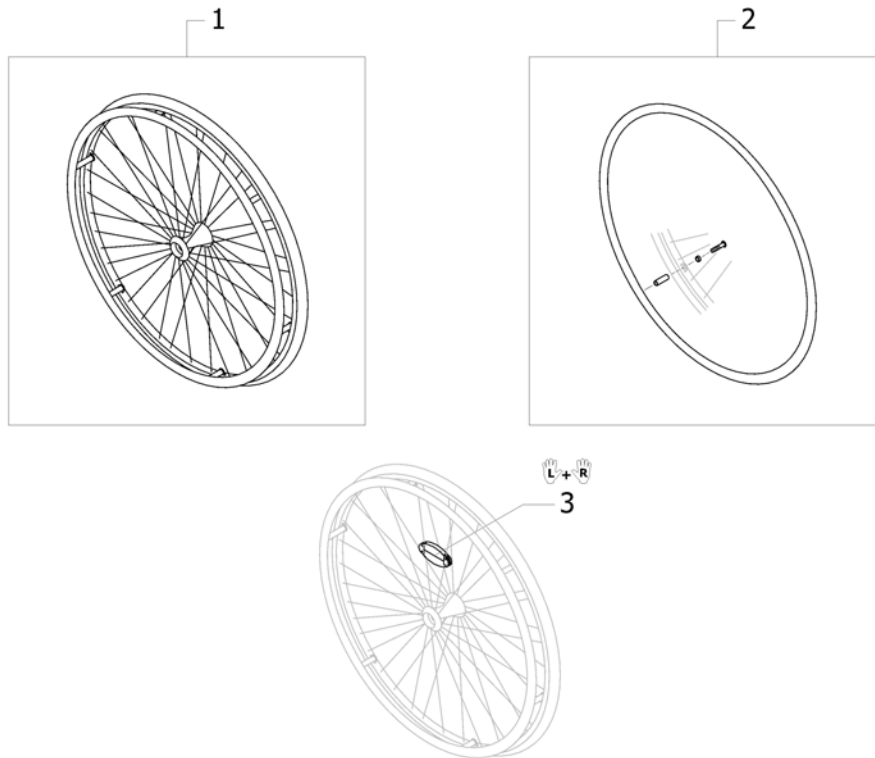












Active wheel

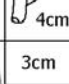

ISPS1665344

Pos.	Part number	Description	Valid from → until	Qty
1	1583058	Rear wheel 24" active		2
2	1583059	Handrim 24" Alu incl. Fixation kit for rear wheel 24" Active		2
3	1571073	Spokereflexion, pair		1


Handrim

																			
		22\" data-bbox="278 168 338 175"/>	24\" data-bbox="398 168 458 175"/>	25\" data-bbox="488 168 548 175"/>	26\" data-bbox="588 168 648 175"/>	24\" data-bbox="688 168 748 175"/>	24\" data-bbox="788 168 848 175"/>	24\" data-bbox="888 168 948 175"/>	24\" data-bbox="988 168 1048 175"/>	24\" data-bbox="1088 168 1148 175"/>	24\" data-bbox="1208 168 1268 175"/>	24\" data-bbox="1308 168 1368 175"/>	24\" data-bbox="1408 168 1468 175"/>	24\" data-bbox="1508 168 1568 175"/>	24\" data-bbox="1608 168 1668 175"/>	24\" data-bbox="1708 168 1768 175"/>	24\" data-bbox="1808 168 1868 175"/>		
	3cm	1	3	6	8	10	11	12	13	14	16	17	18	19	20	21	22	23	24
	4cm	2	4	7	9	-	-	-	-	15	-	-	18	19	20	21	22	23	24
	3cm	-	5	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
	4cm	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
	3cm	-	3	6	-	-	11	12	-	14	16	-	-	19	-	21	22	23	24
	4cm	-	4	7	-	-	-	-	-	15	-	-	-	19	-	21	22	23	24
	3cm	-	3	6	-	-	11	-	-	14	16	-	-	19	-	21	22	23	24
	4cm	-	4	7	-	-	-	-	-	15	-	-	-	19	-	21	22	23	24
																			
																			
25																			
26																			

ISPS1660545

Pos.	Part number	Description	Valid from → until	Qty
Fix	-	Fixation handrim		1
1	1568713-0005	Handrim 22" Aluminium, distance 3cm		2
2	1568700	Handrim 22" Alu, 4cm		2
3	1568714-0005	Handrim 24" Aluminium, distance 3cm		2
4	1568702	Handrim 24" Alu light, colourless anodized, 4cm		2
5	1583059	Handrim 24" Alu incl. Fixation kit for rear wheel 24" Active		2
6	1568730-0005	Handrim 25" Aluminium, distance 3cm		2
7	1568708	Handrim 25" Alu light for high performance, 4cm		2
8	1568731-0005	Handrim 26" Aluminium, distance 3cm		2
9	1568709	Handrim 26" Alu, 4cm		2
10	1568713-0411	Handrim 22" Aluminium, black, distance 3cm		2
11	1568714-0411	Handrim 24" Aluminium, black, distance 3cm		2
12	1568730-0411	Handrim 25" Aluminium, black, distance 3cm		2
13	1568731-0411	Handrim 26" Aluminium, black, distance 3cm		2
14	1568716	Handrim 24" Titanium, distance 3cm		2
15	1568705	Handrim 24" Titanium, 4cm		2
16	1568715	Handrim 24" steel polished, distance 3cm		2
17	1568777	Handrim 26" steel polished, distance 3cm		2
18	1568701	Handrim 22" Super-Grip		2
19	1568703	Handrim 24" Super-Grip		2
20	1568710	Handrim 26" Super-Grip		2
21	SP60122340	Handrim 24" Carbolife Quadro		2
22	SP1638081	Handrim 24" Carbolife Curve		2
23	SP1638080	Handrim 24" Carbolife Gekko		2
24	SP60122339	Handrim 24" Carbolife Gekko S		2
25	SP1666055	Silicon strap for Gekko handrim		2
26	SP1666056	Silicon strap for Gekko S handrim		2

Fixation handrim

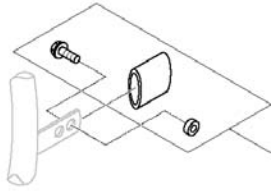


		Aluminium		Titanium		Steel		Supergrip		Gekko		Curve		Gekko S		Quadro	
	3cm	1	1	1	1	1	1	1	1	1	1	1	1	1	1	1	1
	4cm	2	3	-	-	2	3	2	3	2	3	-	-	-	-	-	-
	3cm	4	4	4	4	4	4	4	4	4	4	4	4	4	4	4	4
	4cm	5	6	-	-	5	6	5	6	5	6	5	6	5	6	5	6
	3cm	4	-	4	-	4	-	4	-	4	-	4	-	4	-	4	-
	4cm	5	-	-	-	5	-	5	-	5	-	5	-	5	-	5	-

15195.1.6605.382Z

Pos.	Part number	Description	Valid from → until	Qty
1	1495958	Fixation kit handrim, 3 cm		2
2	1495959	Fixation kit handrim, 4 cm		2
3	1495961	Fixation kit handrim, 4 cm with spokes guard		2
4	1496019	Fixation kit handrim for High Performance wheel, 3 cm		2
5	1496020	Fixation kit handrim for High Performance wheel, 4 cm		2
6	1497111	Fixation kit handrim for High Performance wheel, 4 cm with spokes guard		2

Tyres

													
	22"	24"	25"	26"	22"	24"	24"	25"	26"	24"	25"	22"	24"
 Küschall Räder	1	4	-	12	2	5	6	-	13	7	-	3	8
 Active Wheels	-	4	-	-	-	5	6	-	-	7	-	-	8
 High Performance	-	4	9	-	-	5	6	10	-	7	11	-	-
 Spinegy LXX	-	4	-	-	-	5	6	-	-	7	-	-	-

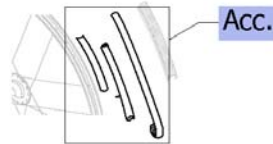

15195SPS1665346

Pos.	Part number	Description	Valid from → until	Qty
Acc.	-	Inner tube and Rim protection		1
1	SP1590117	Tyre Rightrun black/grey 22"		2
2	1570836	Treaded tyre 22"		2
3	1570837	Solid tyre 22", light, grey		2
4	SP1590118	Tyre Rightrun black/grey 24"		2
5	1570829	Treaded Tyre 24"		2
6	1570838	Tyre 24" Marathon Evolution		2
7	1578802	Tyre 24" Schwalbe One		2
8	1570839	Solid tyre 24", light, grey		2
9	SP1590412	Tyre Rightrun black/grey 25"		2
10	1570841	Tyre 25" Marathon Evolution		2
11	1578803	Tyre 25" Schwalbe One		2
12	SP1590119	Tyre Rightrun black/grey 26"		2
13	1570842	Tyre 26" Marathon Evolution		2

Inner tube and Rim protection







		22"	24"	25"	26"	22"	24"	24"	25"	26"	24"	25"	22"	24"	
	Küschall Räder	A	1	2	-	4	1	2	2	-	4	2	-	-	-
	B	5	6	-	8	5	6	6	-	8	6	-	5	6	
	Active Wheels	A	-	2	-	-	-	2	2	-	-	2	-	-	-
	B	-	6	-	-	-	6	6	-	-	6	-	-	6	
	High Performance	A	-	2	3	-	-	2	2	3	-	2	3	-	-
	B	-	7	7	-	-	7	7	7	-	7	7	-	-	
	Spinergy LXX	A	-	2	-	-	-	2	2	-	-	2	-	-	-
	B	-	7	-	-	-	7	7	-	-	7	-	-	-	

Inner Tube (A)
Rim protector (B)

15195.1.665.383

Pos.	Part number	Description	Valid from → until	Qty
1	1570848	Inner Tube 22" x 1"		2
2	1570849	Inner Tube 24" x 1"		2
3	1570851	Inner tube 26" for High-Performance wheel		2
4	1570850	Inner Tube 26" x 1"		2
5	1570844	Rim protection 22"		2
6	1570845	Rim protection 24" Large		2
7	1495962	Rim Protection for High Performance / Spinergy wheel		2
8	1570846	Rim protection 26"		2
9	1571072	Rear wheel inset anti-flat for 20,22,24,26" tyre		2








Spokes guard

						
	22"	24"	26"	22"	24"	26"
 Küschall wheel	1	2	3	4	5	6
 Active Wheels	-	-	-	-	-	-
 High Performance	-	2	-	-	5	-
 Spinerger LXX	-	-	-	-	-	-

ISPS1665347

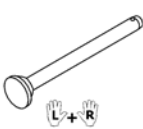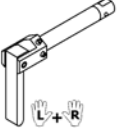Pos.	Part number	Description	Valid from → until	Qty
1	1564453	Spokes guard Transparent 22"		2
2	1564457	Spokes guard Transparent 24"		2
3	1564492	Spokes guard Transparent 26"		2
4	SP60122058	Spoke guard black 22"		2
5	SP60122057	Spoke guard black 24"		2
6	SP60122059	Spoke guard black 26"		2


Quick release axle and bearing

			
	L+R	L+R	L+R
 Küschall wheel	1	4	5
 Active Wheels	2	4	-
 High Performance	2	4	5
 Spinergy LXX	3	-	5

ISPS1665348

Pos.	Part number	Description	Valid from → until	Qty
1	1509137-0407	Quick release axles long with cap, pair, colourless		1
2	1509137-0004	Quick release axles long with cap, pair, black		1
3	1507627	Quickrelease axle long with cap for Spinergy wheel, pair		1
4	1493755	Tetra clip with quick release axle, pair, long		1
5	1571069	Bearing R8-2RS 1/2"/1.1/8"x5/16", pair		2


Brakes

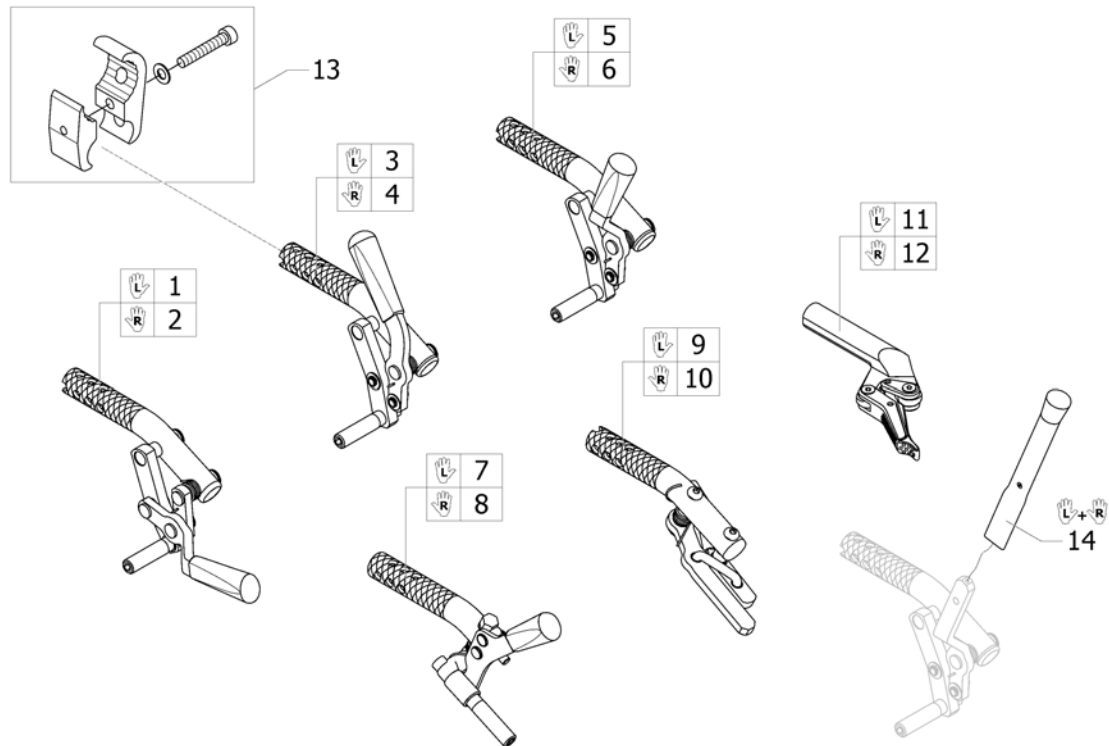

ISPS1665349


Pos.	Part number	Description	Valid from → until	Qty
1	SP1652930	Pull brake left		1
2	SP1652931	Pull brake right		1
3	SP1652934	Brake, bent lever, left		1
4	SP1652935	Brake, bent lever, right		1
5	SP1652932	Brake, straight lever, left		1
6	SP1652933	Brake, straight lever, right		1
7	SP1652938	Performance brake, left		1
8	SP1652939	Performance brake, right		1
9	SP1652926	Active brake, left		1
10	SP1652927	Active brake, right		1
11	SP1652750	Active brake light, left		1
12	SP1652751	Active brake light, right		1
13	SP1642358	Clamping piece to brake set		2
14	1513179	Brake lever extension, pair		1

Seatbelt for transportation

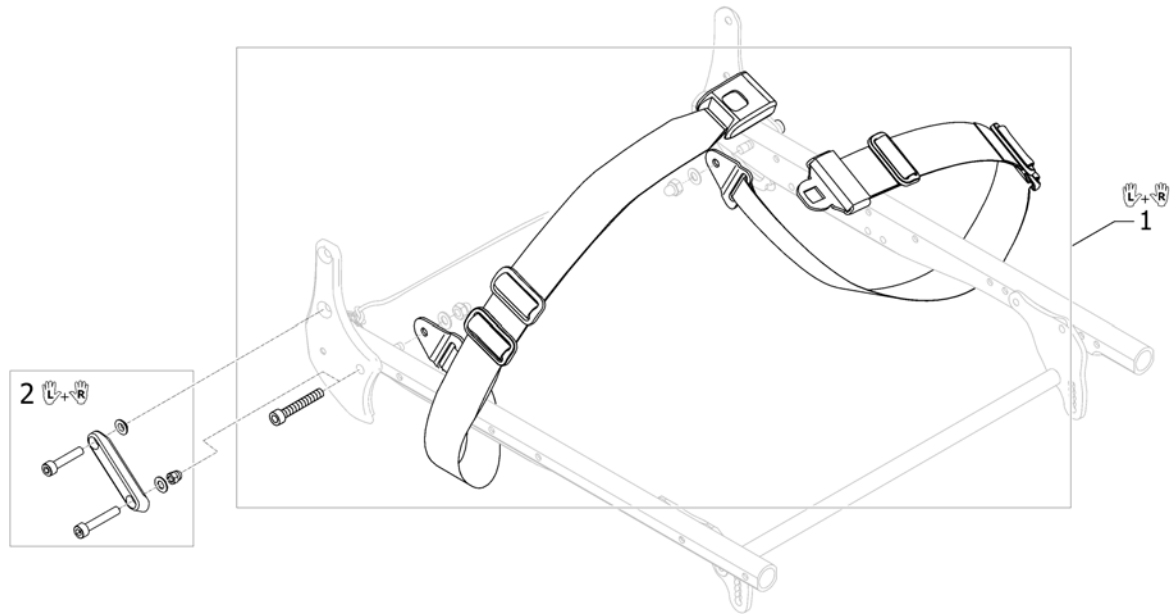

ISPS1665350


Pos.	Part number	Description	Valid from → until	Qty
1	1561025	Seatbelt for transportation		1
2	SP1665836	Seatbelt slider transportation kit, pair		1

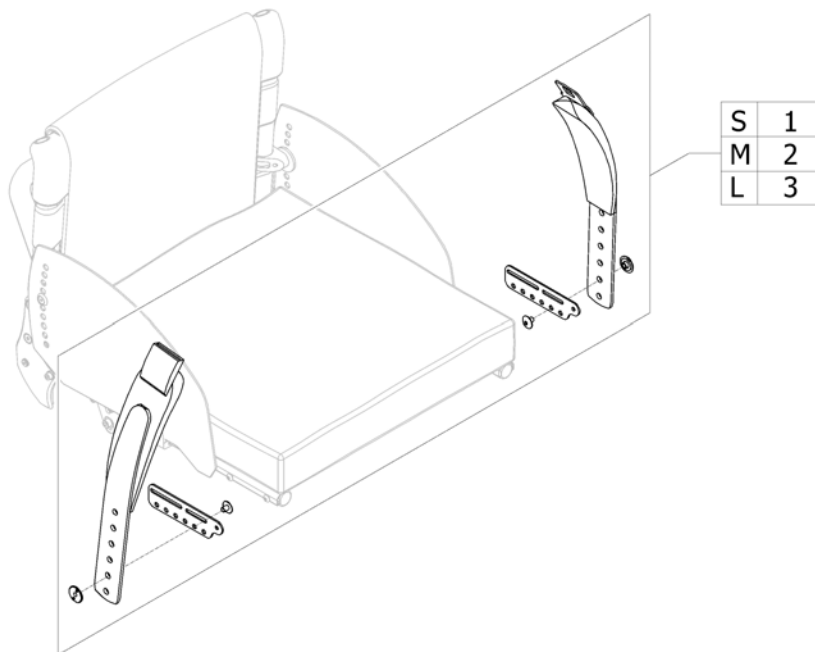Bodypoint posture belt

ISPS1666029

Pos.	Part number	Description	Valid from → until	Qty
1	SP60110787	Bodypoint posture belt "S"		1
2	SP60110788	Bodypoint posture belt "M"		1
3	SP60110789	Bodypoint posture belt "L"		1
4	SP1667826	Fixation kit "S", Bodypoint posture belt		1
5	SP1667827	Fixation kit "L", Bodypoint posture belt		1


Bodypoint calf strap

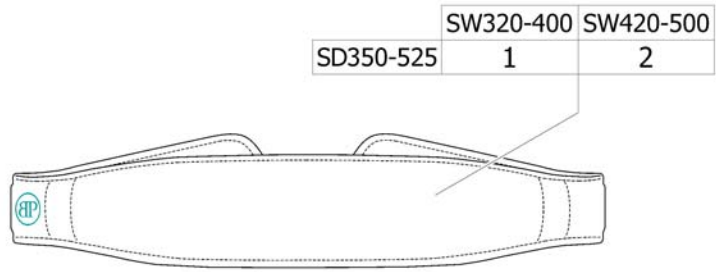

ISPS1666041

Pos.	Part number	Description	Valid from → until	Qty
1	SP60121236	Calf strap Bodypoint "S"		1
2	SP60121239	Calf strap Bodypoint "M"		1


Suspension

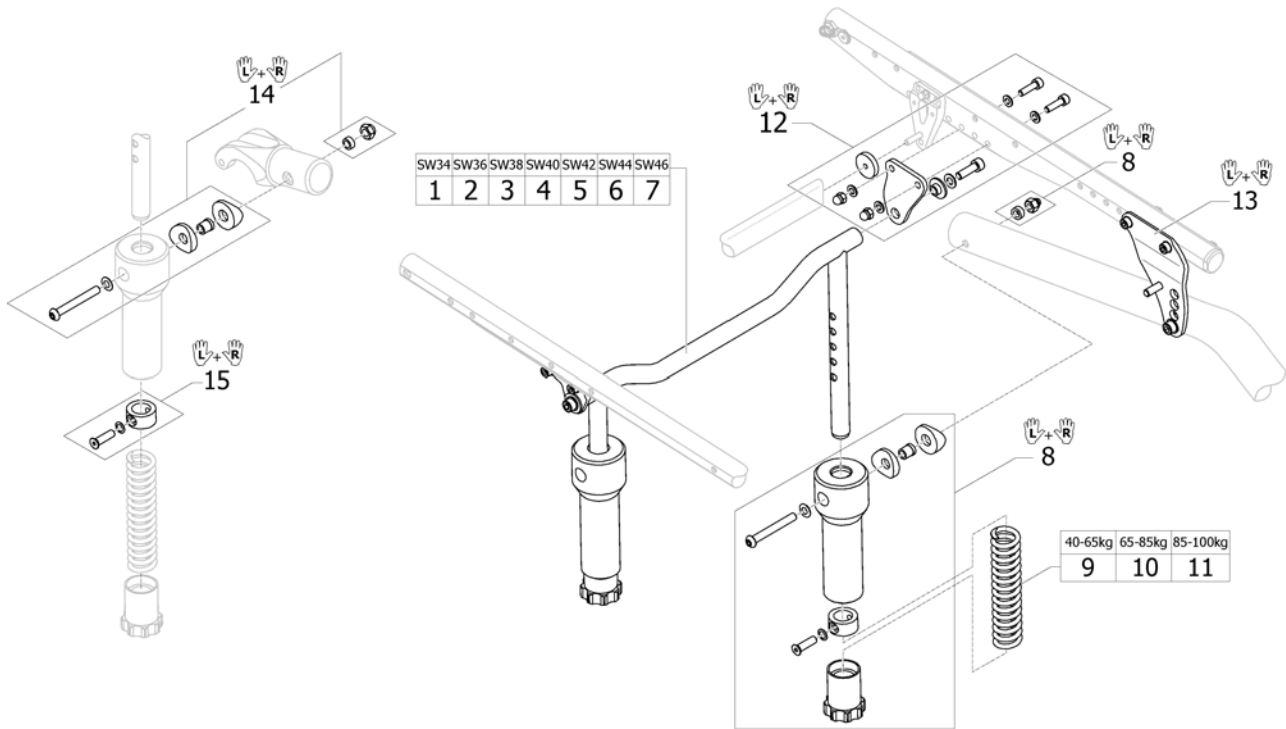

ISPS1665351

Pos.	Part number	Description	Valid from → until	Qty
1	SP1664238	Seat brace suspension SW34 complete		1
2	SP1664239	Seat brace suspension SW36 complete		1
3	SP1664240	Seat brace suspension SW38 complete		1
4	SP1664241	Seat brace suspension SW40 complete		1
5	SP1664242	Seat brace suspension SW42 complete		1
6	SP1664243	Seat brace suspension SW44 complete		1
7	SP1664244	Seat brace suspension SW46 complete		1
8	SP1664245	Spring case, pair complete		1
9	SP1664246	Pressure spring soft, kit (blue), pair		1
10	SP1664247	Pressure spring medium, kit (yellow), pair		1
11	SP1664248	Pressure spring hard, kit (red), pair		1
12	SP1664249	Hinge bracket-Set Suspension Seat Frame, rear		1
13	SP1664251	Hinge bracket-Set Suspension Seat Frame, front		1
14	SP1664252	Fixation Spring case top, pair		1
15	SP1664253	Driver bushing suspension kit, pair		1


Seat upholstery bag

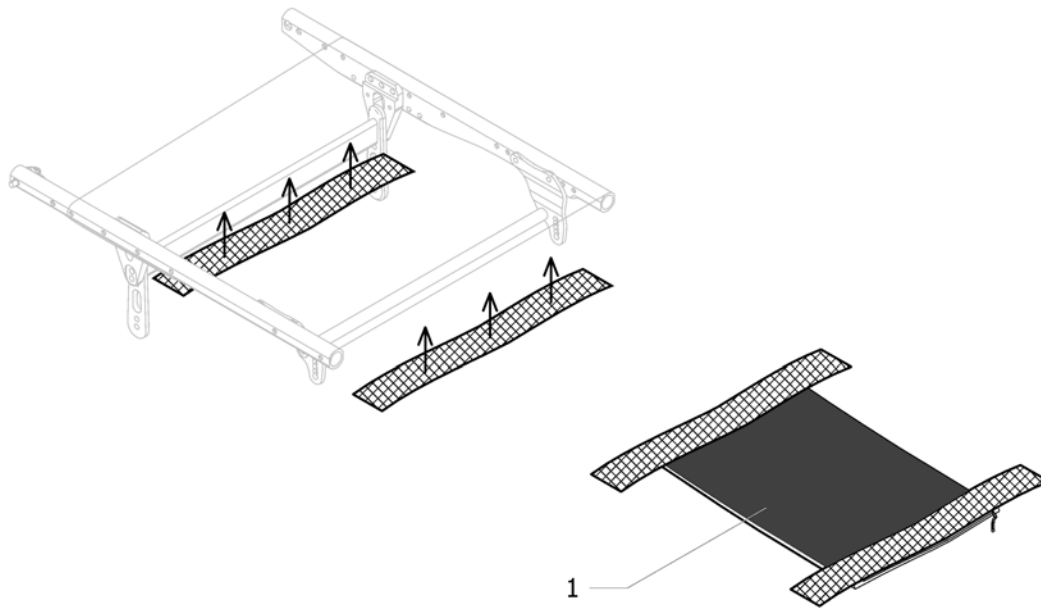

SPS1.66535Z

Pos.	Part number	Description	Valid from → until	Qty
1	1547624	Seat upholstery bag, big		1


Seat frame bag

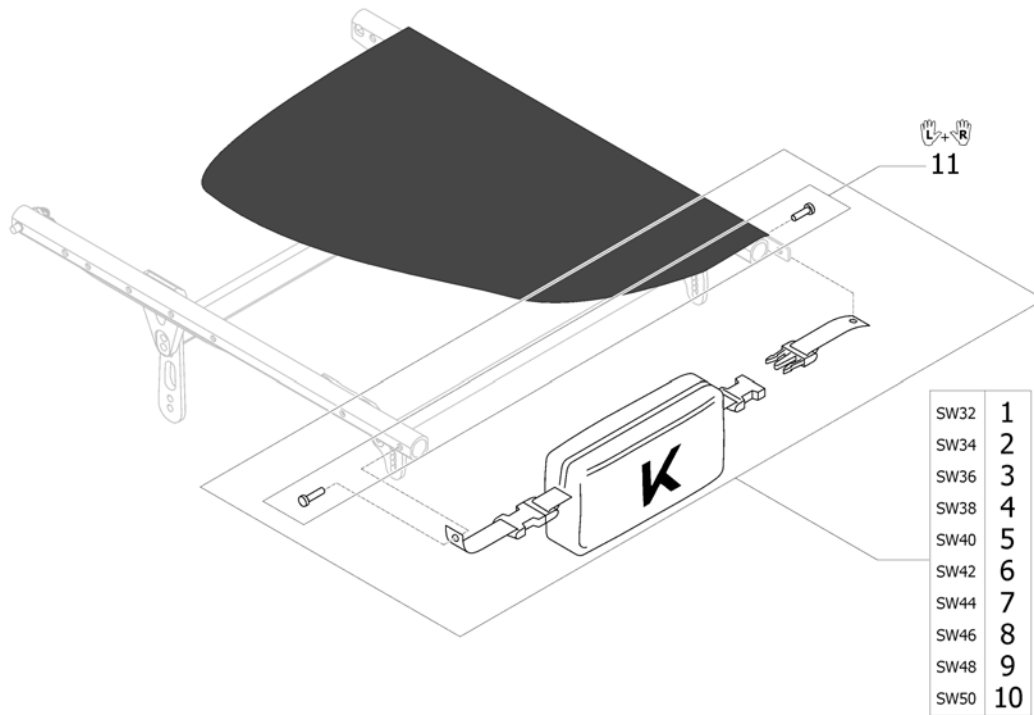

ISPS1665353


Pos.	Part number	Description	Valid from → until	Qty
1	SP1665837	Seat bag SW320		1
2	SP1665838	Seat bag SW340		1
3	SP1665839	Seat bag SW360		1
4	SP1665840	Seat bag SW380		1
5	SP1665841	Seat bag SW400		1
6	SP1665842	Seat bag SW420		1
7	SP1665843	Seat bag SW440		1
8	SP1665849	Seat bag SW460		1
9	SP1665860	Seat bag SW480		1
10	SP1665861	Seat bag SW500		1
11	SP1665862	Screw kit for seat bag		1

One arm drive

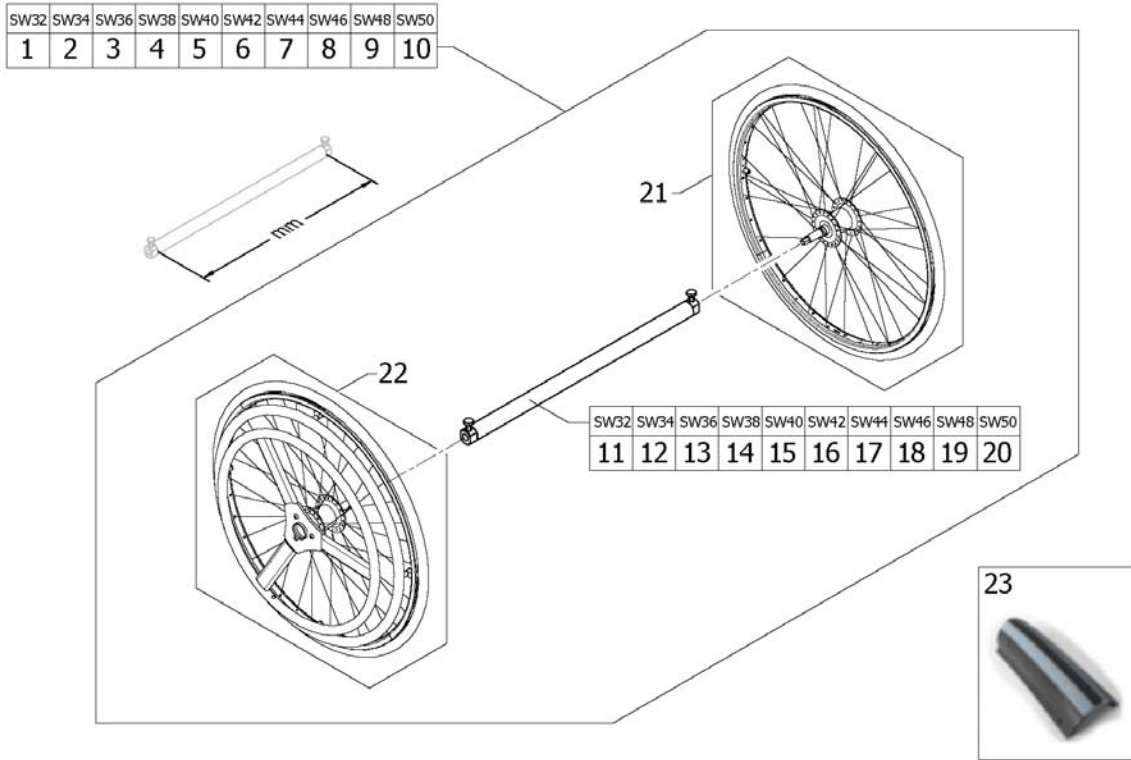

Pos.	Part number	Description	Valid from → until	Qty
1	SP1642605	One hand drive 24" wheels with Axle tube 298mm, complete		1
2	1567081	One hand drive 24" wheels with Axle tube 318mm, complete		1
3	1564370	One hand drive 24" wheels with Axle tube 338mm, complete		1
4	1564371	One hand drive 24" wheels with Axle tube 358mm, complete		1
5	1564372	One hand drive 24" wheels with Axle tube 378mm, complete		1
6	1564373	One hand drive 24" wheels with Axle tube 398mm, complete		1
7	1564374	One hand drive 24" wheels with Axle tube 418mm, complete		1
8	1564375	One hand drive 24" wheels with Axle tube 438mm, complete		1
9	1564376	One hand drive 24" wheels with Axle tube 458mm, complete		1
10	1564377	One hand drive 24" wheels with Axle tube 478mm, complete		1
11	SP1642606	Axle tube Aluminium, 298mm for one hand drive wheels		1
12	1567080	Axle tube Aluminium, 318mm for one hand drive wheels		1
13	1564359	Axle tube Aluminium, 338mm for one hand drive wheels		1
14	1564360	Axle tube Aluminium, 358mm for one hand drive wheels		1
15	1564361	Axle tube Aluminium, 378mm for one hand drive wheels		1
16	1564362	Axle tube Aluminium, 398mm for one hand drive wheels		1
17	1564363	Axle tube Aluminium, 418mm for one hand drive wheels		1
18	1564364	Axle tube Aluminium, 438mm for one hand drive wheels		1
19	1564365	Axle tube Aluminium, 458mm for one hand drive wheels		1
20	1564366	Axle tube Aluminium, 478mm for one hand drive wheels		1
21	1554421	One hand drive 24" wheel passive (without handrim)		1
22	1554422	One hand drive 24" wheel active (with handrim)		1
23	SP1590118	Tyre Rightrun black/grey 24"		1


Antitipper

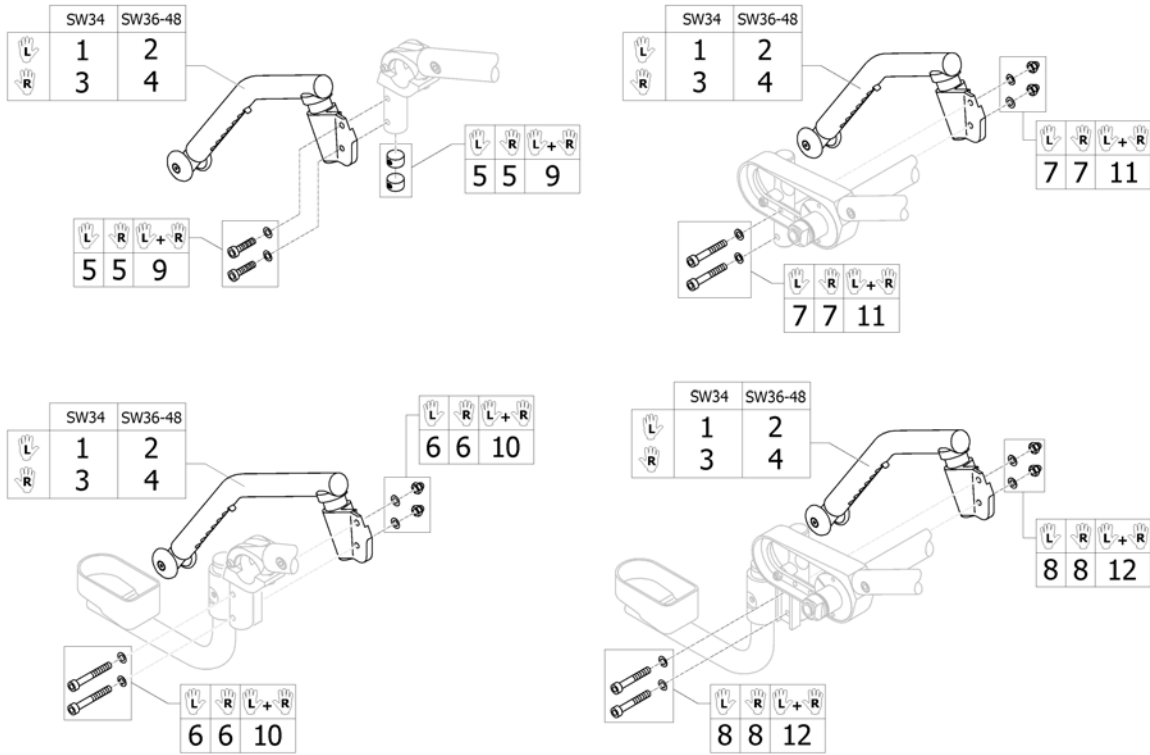

ISPS1665355

Pos.	Part number	Description	Valid from → until	Qty
1	1488927	Antitipper short left complete		1
2	1488929	Antitipper long left complete		1
3	1488928	Antitipper short right complete		1
4	1488930	Antitipper long right complete		1
5	1547621	Fixation kit for adaptations		1


Transit wheel

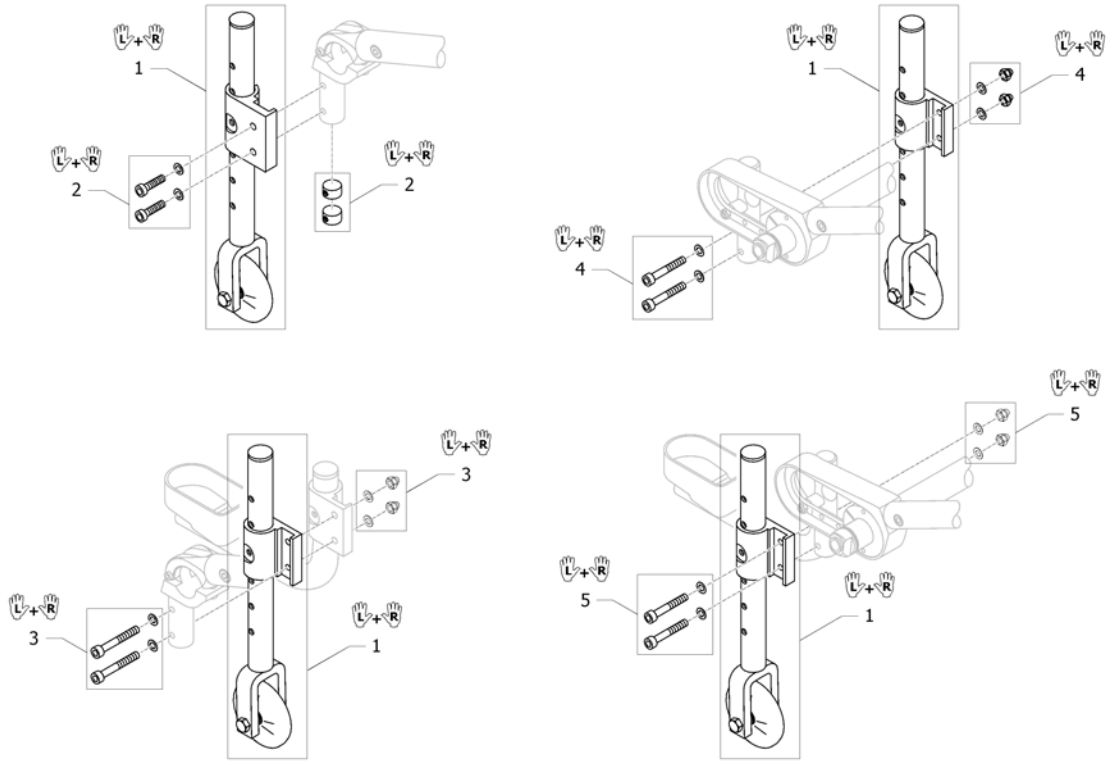

ISPS1665356

Pos.	Part number	Description	Valid from → until	Qty
1	1451791	Transit wheels, pair		1
2	1547621	Fixation kit for adaptions		1


Cane holder

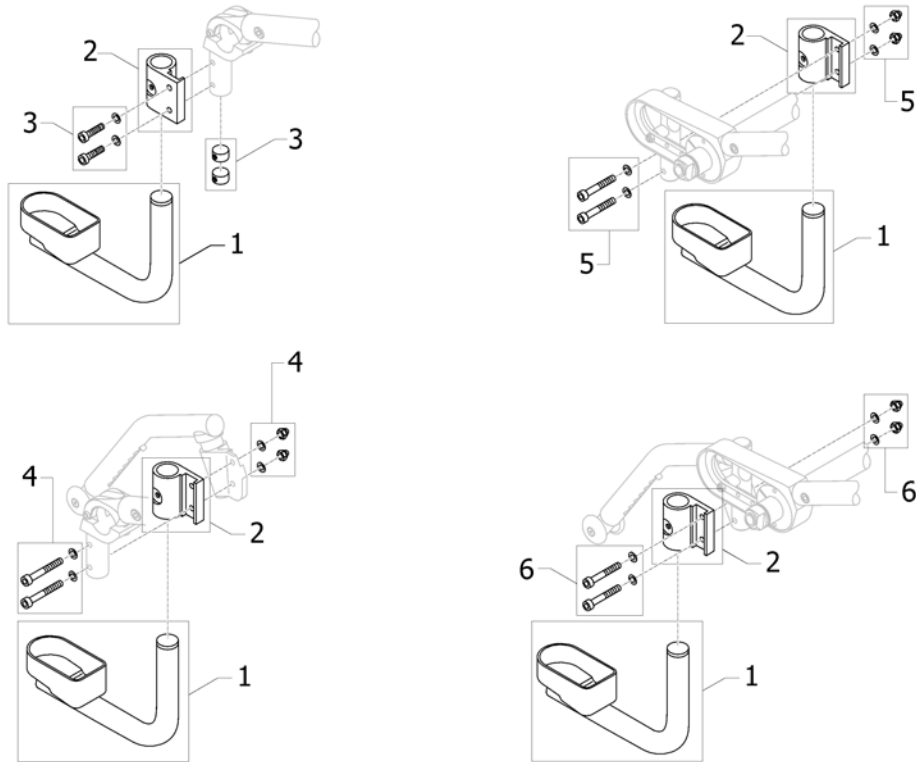

ISPS1665357

Pos.	Part number	Description	Valid from → until	Qty
1	1438214	cane holder complete UL/C04		0
2	1585084	Adaption holder		0
3	SP1643123	Fixation kit for one adaption mounted on axle holder standard		0
4	SP1643124	Fixation kit for two adaptions mounted on axle holder standard		0
5	SP1643125	Fixation kit for one adaption mounted on rear wheel extension		0
6	SP1643126	Fixation kit for two adaptions mounted on rear wheel extension		0


Chair accessories

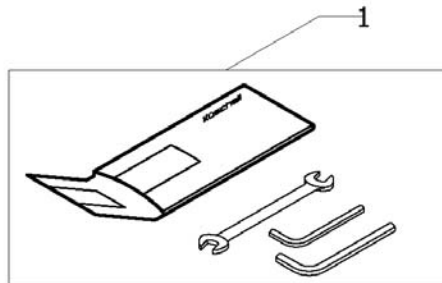

151951665358


Pos.	Part number	Description	Valid from → until	Qty
1	SP1664254	Tool Kit		1
2	1517924	Air pump		1

Fixation: screws

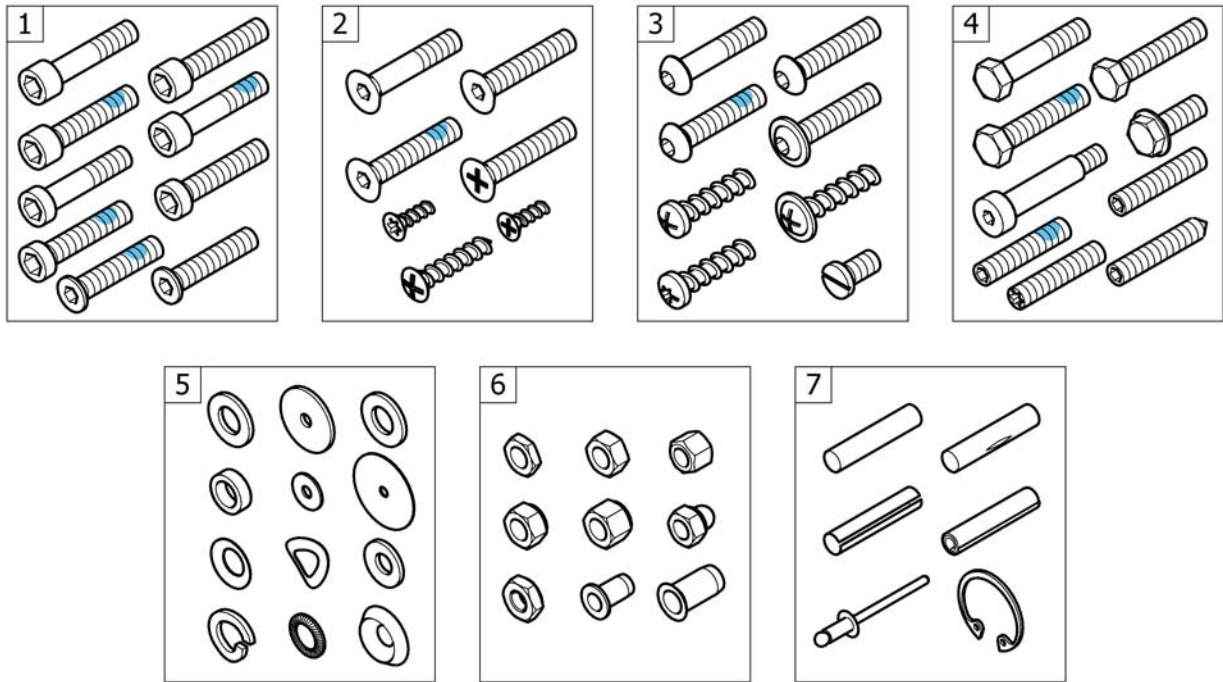

Pos.	Part number	Description	Valid from → until	Qty
1	-	Socket head cap screws		1
2	-	Countersunk screws		1
3	-	Button head screws		1
4	-	Hex-head screws, diverse		1
5	-	Washers		1
6	-	Nuts		1
7	-	Pins, rivets		1

Socket head cap screws

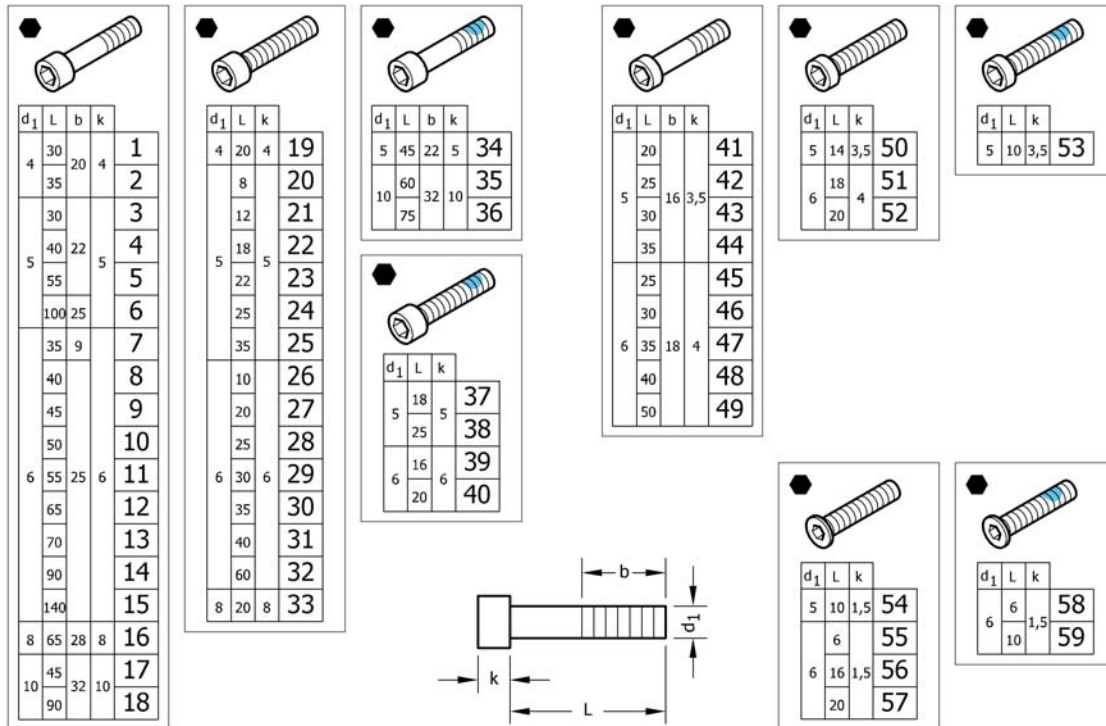


Pos.	Part number	Description	Valid from → until	Qty
1	1579115	Hex socket head cap screws partially threaded, zi.Blue, M4x30, kit 25		1
2	1579116	Hex socket head cap screws partially threaded, zi.Blue, M4x35, kit 25		1
3	1579117	Hex socket head cap screws partially threaded, black, M5x30, kit 25 Pcs.		1
4	1579118	Hex socket head cap screws partially threaded, black, M5x40, kit 25 Pcs.		1
5	1579119	Hex socket head cap screws partially threaded, black, M5x55, kit 25 Pcs.		1
6	1579120	Hex socket head cap screws partially threaded, black, M5x100, kit 10 Pcs.		1
8	1579122	Hex socket head cap screws partially threaded, black, M6x40, kit 25 Pcs.		1
9	1579123	Hex socket head cap screws partially threaded, black, M6x45, kit 25 Pcs.		1
10	1579124	Hex socket head cap screws partially threaded, black, M6x50, kit 25 Pcs.		1
11	1579125	Hex socket head cap screws partially threaded, black, M6x55, kit 25 Pcs.		1
12	1579127	Hex socket head cap screws partially threaded, black, M6x65, kit 10 Pcs.		1
13	1579128	Hex socket head cap screws partially threaded, black, M6x70, kit 10 Pcs.		1
14	1579129	Hex socket head cap screws partially threaded, black, M6x90, kit 10 Pcs.		1
15	1579130	Hex socket head cap screws partially threaded, black, M6x140, kit 10 Pcs.		1
16	1579131	Hex socket head cap screws partially threaded, black, M8x65, kit 10 Pcs.		1
17	1579132	Hex socket head cap screws partially threaded, black, M10x45, kit 2 Pcs.		1
18	1579133	Hex socket head cap screws partially threaded, black, M10x90, kit 2 Pcs.		1
19	1579134	Hex socket head cap screws fully threaded, black, M4x20, kit 25 Pcs.		1
20	1579135	Hex socket head cap screws fully threaded, black, M5x8, kit 5 Pcs.		1
21	1579136	Hex socket head cap screws fully threaded, black, M5x12, kit 25 Pcs.		1
22	1579137	Hex socket head cap screws fully threaded, black, M5x18, kit 25 Pcs.		1
23	1579138	Hex socket head cap screws fully threaded, black, M5x22, kit 25 Pcs.		1
24	1579139	Hex socket head cap screws fully threaded, black, M5x25, kit 25 Pcs.		1
25	1579140	Hex socket head cap screws fully threaded, black, M5x35, kit 25 Pcs.		1
26	1579141	Hex socket head cap screws fully threaded, black, M6x10, kit 25 Pcs.		1
27	1579142	Hex socket head cap screws fully threaded, black, M6x20, kit 25 Pcs.		1
28	1579143	Hex socket head cap screws fully threaded, black, M6x25, kit 25 Pcs.		1
29	1579144	Hex socket head cap screws fully threaded, black, M6x30, kit 25 Pcs.		1
30	1579145	Hex socket head cap screws fully threaded, black, M6x35, kit 25 Pcs.		1
31	1579146	Hex socket head cap screws fully threaded, black, M6x40, kit 10 Pcs.		1
32	1579147	Hex socket head cap screws fully threaded, black, M6x60, kit 10 Pcs.		1
33				
34				
35				
36				
37				
38				
39				
40				
41				
42				
43				
44				
45				
46				
47				
48				
49				
50				
51				
52				
53				
54				
55				
56				
57				
58				
59				

Socket head cap screws


Pos.	Part number	Description	Valid from → until	Qty
33	1579148	Hex socket head cap screws fully threaded, black, M8x20, kit 10 Pcs.		1
34	1579149	Hex sock.Head cap screws partial.Thread., black, TufLok, M5x45, kit 5 Pcs.		1
35	1579150	Hex sock.Head cap screws partial.Thread., black, TufLok, M10x60, kit 2 Pcs.		1
36	1579151	Hex sock.Head cap screws partial.Thread., black, TufLok, M10x75, kit 2 Pcs.		1
37	1579152	Hex socket head cap screws fully threaded, black, TufLok, M5x18, kit 10 Pcs.		1
38	1579153	Hex socket head cap screws fully threaded, black, TufLok, M5x25, kit 10 Pcs.		1
39	1579154	Hex socket head cap screws full.Thread., zi.Blue, TufLok, M6x16, kit 10 Pcs.		1
40	1579155	Hex socket head cap screws fully threaded, black, TufLok, M6x20, kit 10 Pcs.		1
41	1579156	Hex sock.Head cap screws, low head, part.Thread., black, M5x20, kit 25 Pcs.		1
42	1579157	Hex sock.Head cap screws, low head, part.Thread., black, M5x25, kit 25 Pcs.		1
43	1579158	Hex sock.Head cap screws, low head, part.Thread., black, M5x30, kit 25 Pcs.		1
44	1579159	Hex sock.Head cap screws, low head, part.Thread., black, M5x35, kit 25 Pcs.		1
45	1579160	Hex sock.Head cap screws, low head, part.Thread., black, M6x25, kit 25 Pcs.		1
46	1579161	Hex sock.Head cap screws, low head, part.Thread., black, M6x30, kit 25 Pcs.		1
47	1579162	Hex sock.Head cap screws, low head, part.Thread., black, M6x35, kit 25 Pcs.		1
49	1579164	Hex sock.Head cap screws, low head, part.Thread., black, M6x50, kit 25 Pcs.		1
50	1579165	Hex sock.Head cap screws, low head, part.Thread., black, M5x14, kit 25 Pcs.		1
51	1579166	Hex sock.Head cap screws, low head, part.Thread., black, M6x18, kit 25 Pcs.		1
52	1579167	Hex sock.Head cap screws, low head, part.Thread., black, M6x20, kit 25 Pcs.		1
54	1579169	Hex socket head cap screws, special low head, black, M5x10, kit 25 Pcs.		1
55	1579170	Hex socket head cap screws, special low head, black, M6x6, kit 25 Pcs.		1
56	1579171	Hex socket head cap screws, special low head, black, M6x16, kit 25 Pcs.		1
57	1579172	Hex socket head cap screws, special low head, black, M6x20, kit 25 Pcs.		1
58	1579173	Hex socket head cap screws, spec.Low head, black, TufLok, M5x6, kit 10 Pcs.		1
59	1579174	Hex socket head cap screws, spec.Low head, black, TufLok, M6x10, kit 10 Pcs.		1

Countersunk screws

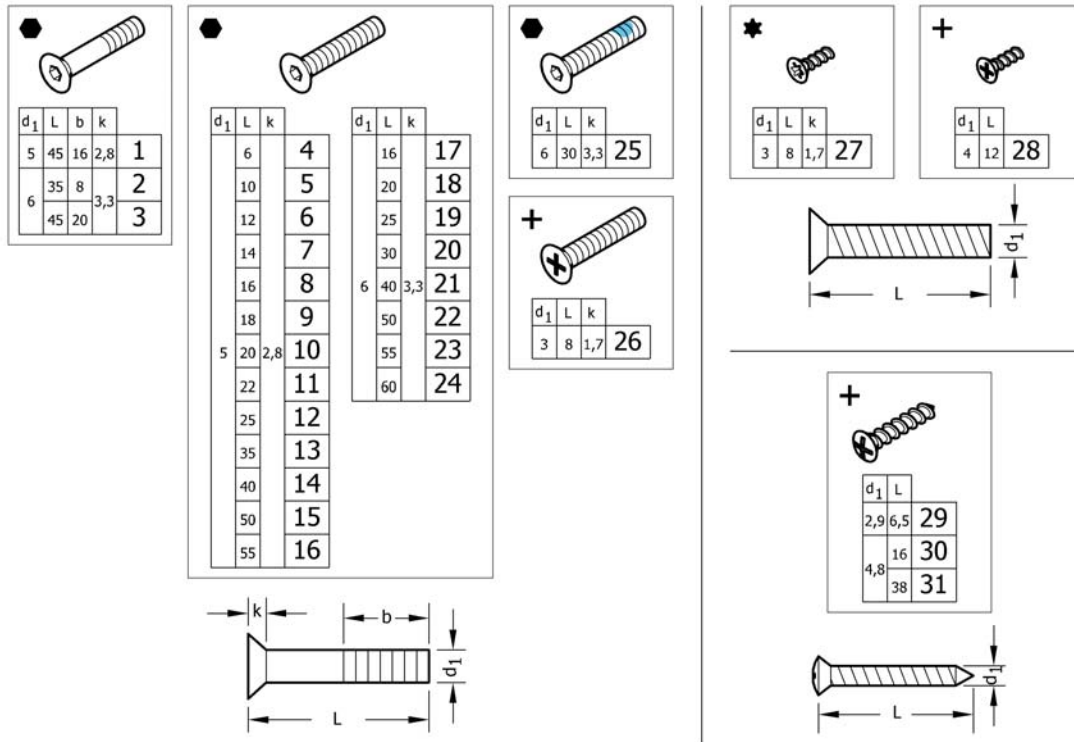


Pos.	Part number	Description	Valid from → until	Qty
1	1579175	Countersunk head screws partially threaded, Black,M5x45, kit 10 Pcs.		1
2	1579176	Countersunk head screws partially threaded, Black,M6x35, kit 10 Pcs.		1
3	1579177	Countersunk head screws partially threaded, Black,M6x45, kit 10 Pcs.		1
5	1579179	Countersunk head screws fully threaded, Black,M5x10, kit 25 Pcs.		1
6	1579180	Countersunk head screws fully threaded, Black,M5x12, kit 25 Pcs.		1
7	1579181	Countersunk head screws fully threaded, Black,M5x14, kit 25 Pcs.		1
8	1579182	Countersunk head screws fully threaded, Black,M5x16, kit 25 Pcs.		1
9	1579183	Countersunk head screws fully threaded, Black,M5x18, kit 25 Pcs.		1
10	1579184	Countersunk head screws fully threaded, Black,M5x20, kit 25 Pcs.		1
11	1579185	Countersunk head screws fully threaded, Black,M5x22, kit 25 Pcs.		1
12	1579186	Countersunk head screws fully threaded, Black,M5x25, kit 25 Pcs.		1
13	1579187	Countersunk head screws fully threaded, Black,M5x35, kit 25 Pcs.		1
14	1579188	Countersunk head screws fully threaded, Black,M5x40, kit 25 Pcs.		1
16	1579190	Countersunk head screws fully threaded, Black,M5x55, kit 25 Pcs.		1
18	1579192	Countersunk head screws full.Threaded, zinc flake coat.,M6x20, kit 25 Pcs.		1
18	1579193	Countersunk head screws fully threaded, Black,M6x20, kit 25 Pcs.		1
19	1579194	Countersunk head screws fully threaded, Black,M6x25, kit 25 Pcs.		1
20	1579195	Countersunk head screws fully threaded, Black,M6x30, kit 25 Pcs.		1
21	1579196	Countersunk head screws fully threaded, Black,M6x40, kit 25 Pcs.		1
22	1579197	Countersunk head screws fully threaded, Black,M6x50, kit 25 Pcs.		1
23	1579198	Countersunk head screws fully threaded, Black,M6x55, kit 25 Pcs.		1
24	1579199	Countersunk head screws fully threaded, Black,M6x60, kit 25 Pcs.		1
25	1579200	Countersunk head screws fully threaded, Black, TufLok,M6x30, kit 25 Pcs.		1
26	1579201	Countersunk head screws fully threaded, zi.Blue,M3x8, kit 25 Pcs.		1
27	1579202	Countersunk head PT screws, Torx, stainless steel, 3x8, kit 25 Pcs.		1
28	1579203	Countersunk head PT screws, Black, KA40x12, kit 25 Pcs.		1
30	1579205	Countersunk head tapping screws with cone end, Black, ST4,8x16, kit 25 Pcs.		1

Button head screws

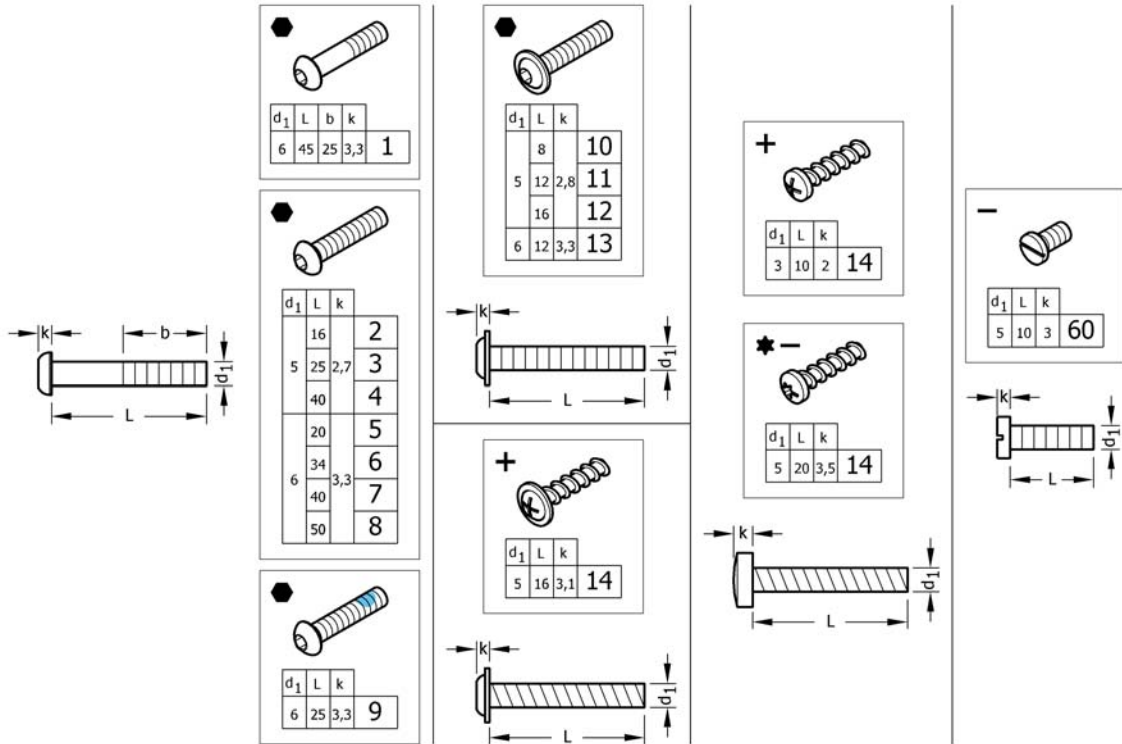


Pos.	Part number	Description	Valid from → until	Qty
1	1579259	Hex sock.Button head cap screws partially thread., Black, M6x45, kit 25 Pcs.		1
4	1579262	Hex socket button head cap screws fully threaded, Black, M5x40, kit 25 Pcs.		1
6	1579264	Hex socket button head cap screws fully threaded, Black, M6x34, kit 25 Pcs.		1
7	1579265	Hex socket button head cap screws fully threaded, Black, M6x40, kit 25 Pcs.		1
9	1579267	Hex sock.But.Head cap screws full.Thread., Black, TufLok, M6x25, kit 25 Pcs.		1
10	1579268	Button head socket cap screws with flange, Black, M5x8, kit 25 Pcs.		1
11	1579269	Button head socket cap screws with flange, Black, M5x12, kit 25 Pcs.		1
12	1579270	Button head socket cap screws with flange, Black, M5x16, kit 25 Pcs.		1
13	1579271	Button head socket cap screws with flange, Black, M6x12, kit 25 Pcs.		1
14	1579272	Round washer head PT® screws, form H, zinc plated blue, M5x16, kit 25 Pcs.		1
15	1579273	Pan head PT screws, form H, zinc plated blue, M3x10, kit 25 Pcs.		1
16	1579274	Pan head PT screws, Torx, Black, M5x20, kit 25 Pcs.		1
17	1579275	Slotted machine screws, Black, M5x10, kit 25 Pcs.		1

Hex-head screws, diverse

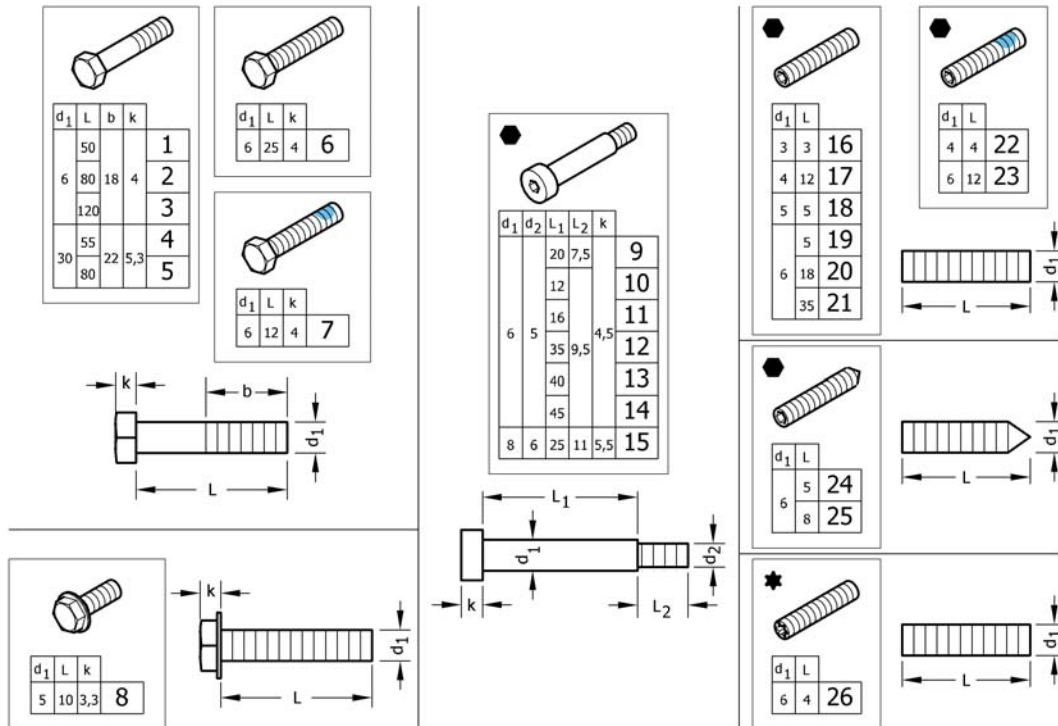


Pos.	Part number	Description	Valid from → until	Qty
4	1579279	Hex head bolts partially threaded, Black, M8x55, kit 10 Pcs.		1
5	1579280	Hex head bolts partially threaded, Black, M8x80, kit 10 Pcs.		1
7	1579282	Hex head screws fully threaded, Black, TufLok, M6x12, kit 10 Pcs.		1
8	1579283	Hex flange head machine screws with flange, Black, M5x10, kit 10 Pcs.		1
10	1579296	Hex socket head shoulder bolts, Black, Ø6x12 /M5, kit 4 Pcs.		1
11	1579297	Hex socket head shoulder bolts, Black, Ø6x16 /M5, kit 4 Pcs.		1
16	1579284	Hex socket set screws with flat point, Black, M3x3, kit 25 Pcs.		1
17	1579285	Hex socket set screws with flat point, Black, M4x12, kit 25 Pcs.		1
18	1579286	Hex socket set screws with flat point, zinc plated blue, M5x5, kit 25 Pcs.		1
19	1579287	Hex socket set screws with flat point, Black, M6x5, kit 25 Pcs.		1
20	1579288	Hex socket set screws with flat point, zinc plated blue, M6x18, kit 25 Pcs.		1
21	1579289	Hex socket set screws with flat point, Black, M6x35, kit 25 Pcs.		1
22	1579290	Hex socket set screws with flat point, Black, TufLok, M4x4, kit 10 Pcs.		1
23	1579291	Hex socket set screws with flat point, Z.Plat.Bl., TufLok, M6x12, kit 10Pcs.		1
24	1579292	Hex socket set screws with cone point, Black, M6x5, kit 25 Pcs.		1
25	1579293	Hex socket set screws with cone point, Black, M6x8, kit 25 Pcs.		1
26	1579294	Screws with cup point, Torx, zinc plated blue, M6x4, kit 25 Pcs.		1

Washers

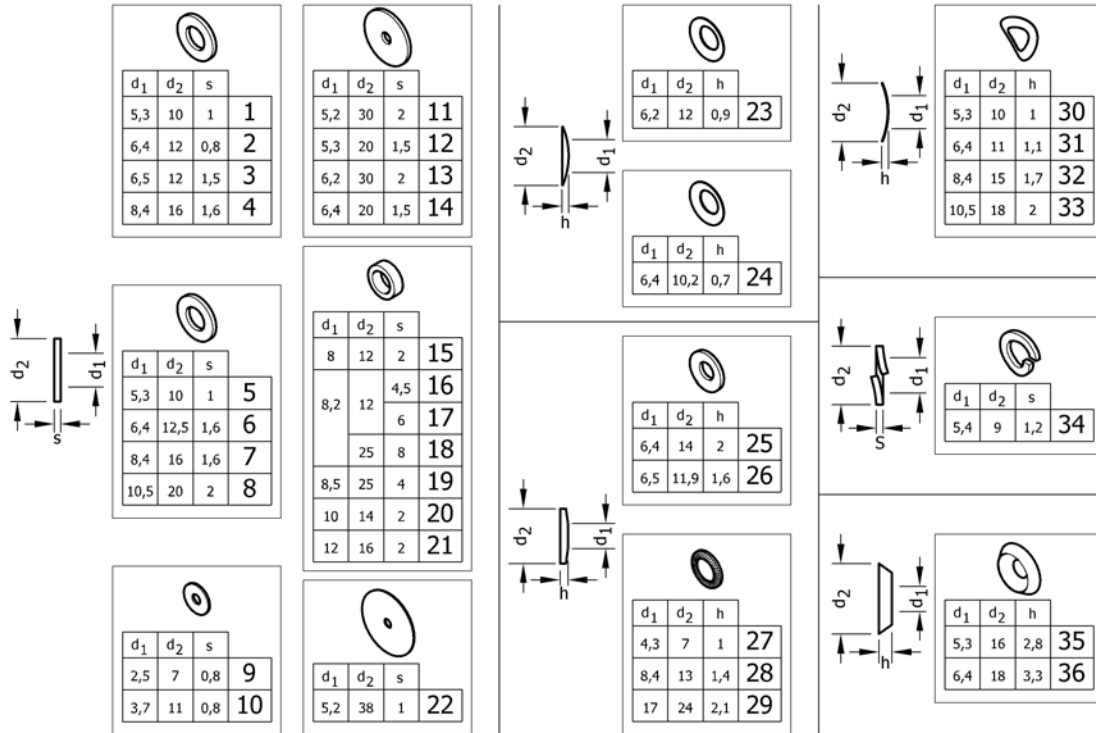


Pos.	Part number	Description	Valid from → until	Qty
1	1579225	Flat washers without chamfer, black, 5,3x10x1, kit 50 Pcs.		1
2	1579226	Flat washers without chamfer, black, 6,4x12x0,8, kit 50 Pcs.		1
3	1579227	Flat washers without chamfer, zinc plated blue, 6,5x12x1,5, kit 50 Pcs.		1
4	1579228	Flat washers without chamfer, black, 8,4x16x1,6, kit 50 Pcs.		1
5	1579229	Flat washers without chamfer, polyamide, 5,3x10x1, kit 50 Pcs.		1
6	1579230	Flat washers without chamfer, polyamide, 6,4x12,5x1,6, kit 50 Pcs.		1
8	1579232	Flat washers without chamfer, polyamide, 10,5x20x2, kit 50 Pcs.		1
9	1579233	Flat washers without chamfer, zinc plated blue, 2,5x7x0,8, kit 50 Pcs.		1
11	1579235	Flat washers without chamfer, black, 5,2x30x2, kit 20 Pcs.		1
12	1579236	Flat washers without chamfer, black, 5,3x20x1,5, kit 50 Pcs.		1
13	1579237	Flat washers without chamfer, black, 6,2x30x2, kit 20 Pcs.		1
15	1579239	Hinge rings, black, 8x12x2, kit 50 Pcs.		1
16	1579240	Distance sleeve, alu, 8,2x12x4,5, kit 20 Pcs.		1
17	1579241	Distance sleeve, alu, 8,2x12x6, kit 20 Pcs.		1
18	1581292	Distance sleeve, alu, 8,2x25x8, kit 2 Pcs.		1
19	1583052	Distance sleeve, alu, black, 8,5x25x4, kit 10 Pcs.		1
20	1579242	Hinge rings, zinc plated blue, 10x14x2, kit 50 Pcs.		1
21	1579243	Hinge rings, black, 12x16x2, kit 50 Pcs.		1
23	1579245	Disc springs, black, 6,2x12x0,9, kit 50 Pcs.		1
24	1579246	Conical spring washers, zinc plated blue, 6,4x10,2x0,7, kit 50 Pcs.		1
25	1579247	Conical spring washers, black, 6,4x14x2, kit 50 Pcs.		1
27	1579249	Ribbed lock washers, black, 4,3x7x1, kit 50 Pcs.		1
29	1579251	Ribbed lock washers, zinc plated blue, 17x24x2,1, kit 50 Pcs.		1
32	1579254	Curved spring washers, zinc plated blue, 8,4x15x1,7, kit 50 Pcs.		1
33	1579255	Curved spring washers, zinc plated blue, 10,5x18x2, kit 50 Pcs.		1
34	1579256	Split spring lock washers, black, 5,4x9x1,2, kit 50 Pcs.		1
35	1579257	Finishing washers 90° countersunk head screw, black, 5,3x16x2,8, kit 50 Pcs.		1
36	1579258	Finishing washers 90° countersunk head screw, black, 6,4x18x3,3, kit 50 Pcs.		1

Nuts

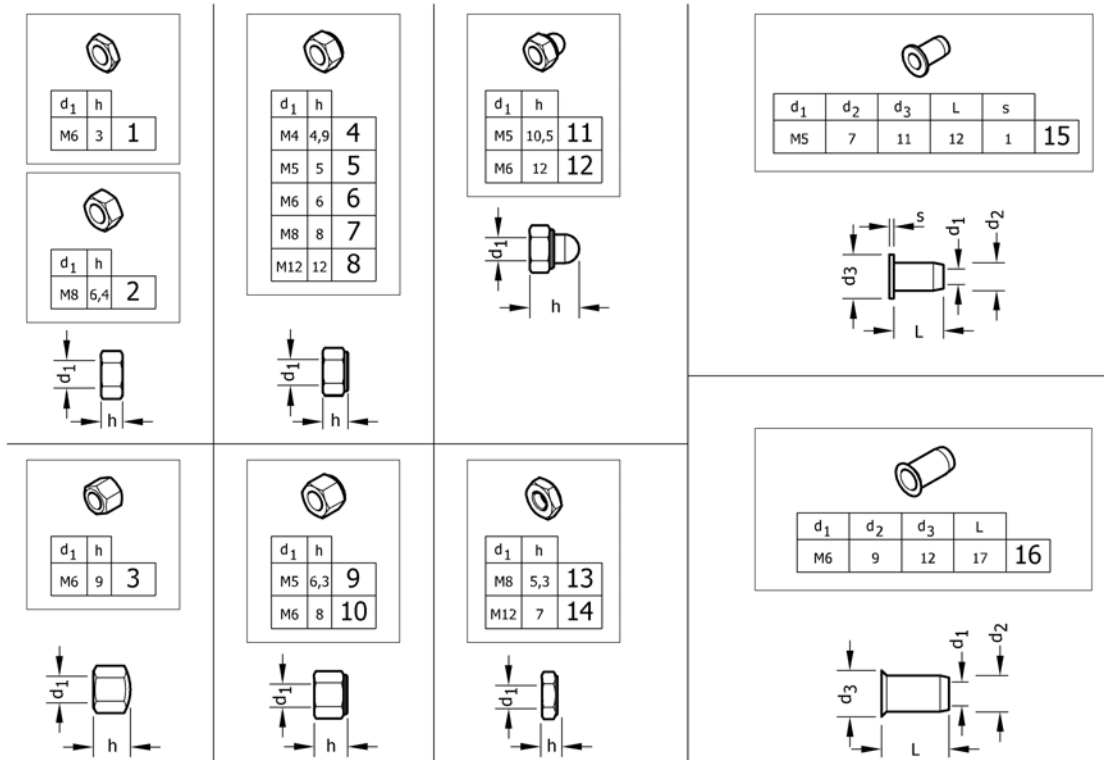

Pos.	Part number	Description	Valid from → until	Qty
1	1579207	Hex jam nuts, black, M6, kit 25 Pcs.		1
2	1579208	Hexagon nuts , black, M8, kit 25 Pcs.		1
3	1579209	Hex cap nuts, low type, black, M6, kit 25 Pcs.		1
5	1579211	Prev.Torque lock nuts, thin type, with polyam.Insert, black, M5, kit 25 Pcs.		1
5	1579212	Prev.Torque lock nuts, thin type, with poly.Insert, zi.Bl., M5, kit 25 Pcs.		1
6	1579213	Prev.Torque lock nuts, thin type, with polyam.Insert, black, M6, kit 25 Pcs.		1
6	1579214	Prev.Torque lock nuts, thin type, with poly.Insert, zi.Bl., M6, kit 25 Pcs.		1
7	1579215	Prev.Torque lock nuts, thin type, with polyam.Insert, black, M8, kit 25 Pcs.		1
9	1579217	Preva.Torq.Hex lock nuts, high type with poly.Insert, black, M5, kit 25 Pcs.		1
10	1579218	Preva.Torq.Hex lock nuts, high type with poly.Insert, black, M6, kit 25 Pcs.		1
11	1579219	Prevail.Torque hex domed cap nuts with polyam.Insert, black, M5, kit 25 Pcs.		1
12	1579220	Prevail.Torque hex domed cap nuts with polyam.Insert, black, M6, kit 25 Pcs.		1
13	1579221	Prevailing torque hex lock nuts, all-metal, thin typeblack, M8, kit 10 Pcs.		1
14	1579222	Prevailing torque hex lock nuts, all-metal, thin typeblack, M12, kit 4 Pcs.		1
15	1579223	Blind rivet nuts, flat head, open type, black, M5, kit 10 Pcs.		1
16	1579224	Blind rivet nut with low countersunk head, open type, black, M6, kit 10 Pcs.		1

Pins, rivets

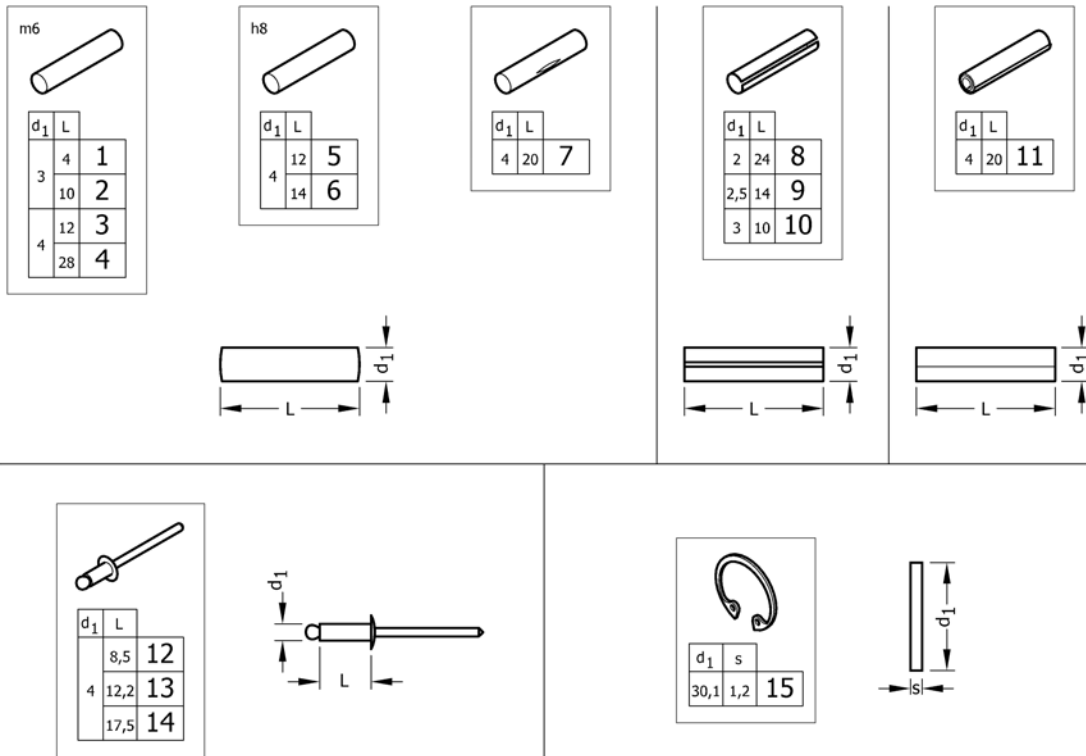

Pos.	Part number	Description	Valid from → until	Qty
4	1579305	Parallel pins, Ø4x28 m6, kit 20 Pcs.		1
7	1579308	Grooved pins, center grooved, Ø4x20, kit 20 Pcs.		1
9	1579310	Slotted spring pins, heavy duty, Black, Ø2,5x14, kit 20 Pcs.		1
10	1579311	Slotted spring pins, heavy duty, stainless steel, Ø3x10, kit 20 Pcs.		1
11	1579312	Spiral spring pins, Black, Ø4x20, kit 20 Pcs.		1
12	1579313	Dome head blind rivets, Ø4x8,5, kit 20 Pcs.		1
13	1579314	Dome head blind rivets, Ø4x12,2, kit 20 Pcs.		1
14	1579315	Dome head blind rivets, Ø4x17,5, kit 20 Pcs.		1
15	1579316	Retaining rings for bores, Std.Design, Z.plat.Blue,Ø30,1X1,2, kit 20 Pcs.		1



Yes, you can.®



Belgium & Luxemburg:

Invacare nv, Autobaan 22, B-8210 Loppem
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11
belgium@invacare.com / www.invacare.be

Denmark:

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01
denmark@invacare.com / www.invacare.dk

Deutschland:

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66
info@invacare-aquatec.com / www.invacare-aquatec.de

European Distributor Organisation:

Invacare, Kleiststraße 49, D-32457 Porta Westfalica
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541
edo@invacare.com / www.invacare.eu.com

España:

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20
contactsp@invacare.com / www.invacare.es

France:

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com / www.invacare.fr

Ireland:

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085
ireland@invacare.com / www.invacare.ie

Italia:

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34
italia@invacare.com / www.invacare.it

Nederland:

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758
nederland@invacare.com, csede@invacare.com / www.invacare.nl

Norge:

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01
norway@invacare.com, island@invacare.com / www.invacare.no

Österreich:

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4
info-austria@invacare.com / www.invacare.at

Portugal:

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39
portugal@invacare.com / www.invacare.pt

Sverige & Suomi:

Invacare AB, Fagerstagatan 9, S-163 91 Spånga
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08
sweden@invacare.com, finland@invacare.com / www.invacare.se

Switzerland:

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81
switzerland@invacare.com / www.invacare.ch

United Kingdom:

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220
uk@invacare.com / www.invacare.co.uk