

# Action 3 Jnr

CREATIVE THINKING



Invacare's Action<sup>3</sup> Junior is a lightweight, foldable paediatric wheelchair designed for children aged between 3 and 15 years. Action<sup>3</sup> Junior has been developed to match the individual needs of the child and can grow as they grow. The Action<sup>3</sup> Junior comes in attractive, modern styles, designed to appeal to kids.



Yes, you can.®

Making Life's  
Experiences Possible



# Action 3 Jnr

As children grow and develop, a key feature of a children's wheelchair is that it can be adjusted to accommodate the child's individual growth and needs. With the Invacare Action<sup>3</sup> Junior, the width of the seat and backrest can be adjusted, as can the height of the arm and footrests.

A wide range of options are available on the Action<sup>3</sup> Junior to meet every child's needs in terms of comfort and positioning.

## Features and Options




### Upholstery adjustment

As the chassis is made wider, so the upholstery on the back and on the seat of the chair can be made wider with Velcro flaps.



### Front stabilizer kit

A front stabilizer kit is available to increase forward stability for active children who are likely to climb all over the wheelchair and be especially energetic when propelling the wheelchair.

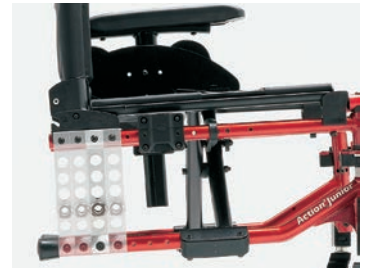

### Adjustable seat height and balance

For improved driving performance and different height options, the rear wheel can be mounted in 20 different positions.

## Tech Specs



**Seat width**  
20.5 - 38cm



**Seat depth**  
20 - 37.5cm



**Seat angle**  
1.5° / 5° / 8.5°



**Seat height**  
41 / 43.5 / 46cm



**Backrest angle**  
0° / 7° / 0° - 30°



**Backrest height**  
35 - 43cm



**Front frame angle**  
80° / 90° /  
0° - 80°



**Length of footrests**  
12.5 - 45cm



**Armrest height**  
15-25 / 18-28cm



**Driving unit width**  
Seat width +  
19/17cm



**Total folded width**  
31.5/27cm



**Total length including footrests**  
74 - 100cm



**Total length excluding footrests**  
51 - 77cm



**Total height**  
87 - 113cm



**Max. safe slope**  
12°



**Turning radius**  
75cm



**Max. user weight**  
80 kg



**Total weight from**  
15 / 14.5 kg



**Mass of the heaviest part, from**  
8.5 / 12 kg



1. Transit version



### FOR FURTHER INFORMATION:

**New Zealand:** Freephone 0800 468 222, email [sales@invacare.co.nz](mailto:sales@invacare.co.nz) or visit [www.invacare.co.nz](http://www.invacare.co.nz)  
**Australia:** Freephone 1800 460 460, email [orders@invacare.com.au](mailto:orders@invacare.com.au) or visit [www.invacare.com.au](http://www.invacare.com.au)

All rights reserved. All information quoted is believed to be correct at time of publication. Invacare reserves the right to alter product specifications without prior consent. 05/2024



Yes, you can.®