

Seating Series



SEATING AND POSITIONING
CUSHIONS | BACKS | HARDWARE



Invacare® Matrix® Seating and Positioning is an established global leader, committed to product innovation, high quality standards and superior product performance. As the world of seating evolves, the Invacare Matrix Seating Series continues to exceed customer expectations.

Contents

Cushions

Libra	3
Vi	4
PSP	5
PS	6

Backs

E2 Back Series.....	7
Elite.....	13
Elite TR.....	14
Elite Deep.....	15
PB.....	16
PB Deep.....	17
Mounting Hardware Options	18
Mini	19
MX2.....	20
Dipped Back Shell Options	21

Elan Headrests	22
-----------------------------	----

Accessories	23
--------------------------	----

Libra Cushion

THE COMPLETE SOLUTION

The Invacare® Matrix® Libra Cushion is designed with the highest level of skin protection, positioning and adjustability for unique individual needs. Specially contoured to provide balance and stability to the pelvis, optimising body function. Recommended for all users including those at highest risk of skin breakdown.



Fluid Sac:

- Dual layer, sealed compartments for maximum protection around bony prominences.
- Non-temperature sensitive, no kneading or maintenance required.

Foam base:

- High resilience foam made with Ultra-Fresh™ anti-microbial and odour protection.
- Ab/adductor contour promotes comfortable midline leg positioning
- Pre-ischial contour prevents sliding to reduce shear
- Trochanteric shelf promotes level pelvis and lateral stability
- Waterfall front edge and extended rear cushion radius

Cover:



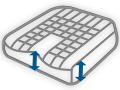



- Moisture resistant, high stretch, breathable Startex™. Hook and loop fastener strips, lifting strap and convenient zippered side pocket.
- Inner platilon liner provides extra lay of moisture protection.

Fit Kit Accessories:

Designed to enhance immersion, correct or accommodate postural issues.

Pelvic Obliquity kit and cushion modifications also available.

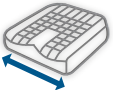Tech Specs

	 Width*	 Depth**	 Height	 Product weight	 Weight Capacity	 Limited Warranty
Libra	14 - 20"	12 - 22"	3.35" Adductor/ Abductor 2.35" Leg Trough	from 1.8kg (4lb)	136kg (300lb)	3 years
Libra Heavy Duty	21 - 24"	12 - 22"	3.6" Abductor/ Adductor 2.6" Leg Trough	from 3.6kg (8lb)	226kg (500lb)	3 years

* Actual width is 1/2" narrower than stated width

** Actual depth is 1 1/2" longer than stated depth



Yes, you can.®

Vi Cushion

STABILITY, SUPPORT AND COMFORT

The Invacare® Matrix® Vi cushion is designed to provide skin protection and positioning for individuals at moderate to high risk of skin breakdown. The unique three part foam construction targets skin protection where it is needed most, and features excellent stability, support and comfort.



Top Layer:

- Soft High Resilience foam for optimal comfort and support
- Pre-ischial contour provides pelvic stability and abductor/adductor contouring promotes midline leg positioning.
- Made with Ultra-Fresh(TM) for antibacterial and odour protection

Visco foam insert:

- Ischial relief insert made from moulded visco-elastic foam
- Provides immersion to sensitive bony prominences

Bottom Layer:

- Firm High Resilience foam for support and durability
- Trochanteric shelf provides stability to hips
- Waterfall front edge and rear radius for sacral support

Cover:

- Moisture resistant and breathable outer cover including slip resistant base, hook and loop fasteners and lifting strap
- Kid*ab*ra print standard in paediatric sizes
- Inner cover is moisture resistant and easily wiped clean

Options



Cushion and Cover Modifications:

Standard modifications include seat width and depth reductions, as well as leg length discrepancies



Pelvic Obliquity Kit:







Designed to accommodate up to 1" obliquities.



Cushion Rigidisers:

Lightweight plastic seat insert designed to fit to the bottom of the cushion inside the cover. Sits between seat rails for level support and eliminates hammock effect of sling seat upholstery.

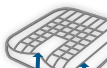Tech Specs

	 Width*	 Depth**	 Height	 Product weight	 Weight Capacity	 Limited Warranty
Paediatric	12" - 14"	12" - 14"	2.85" Adductor 2.75" Abductor 2" Leg Trough	From 680g (1.5lb)	68kg (150lb)	2 years
Standard	14", 16", 17", 18", 20", 22"	15" - 22"	3.75" Adductor 3.25" Abductor 2.5" Leg Trough	from 1.3kg (3lb)	136kg (300lb)	2 years
Heavy Duty	20 - 30"	18" - 26"	3.75" Adductor 3.25" Abductor 2.5" Leg Trough	from 1.8kg (4lb)	272kg (600lb)	2 years

* Actual width is 1/2" narrower than stated width

** Actual depth is 1 1/2" longer than stated depth

PSP Cushion

POSTURE SEAT POLYMER CUSHION

The Invacare® Matrix® PSP (Posture Seat Polymer) Cushion combines the stability and support of the PS foam base with a polymer overlay for extra protection and shear reduction. An ultra-lightweight cushion providing worry free skin protection and positioning.



Outer cover:

- Reversible Startex™ outer cover can be used smooth or fabric side out
- Moisture resistant and breathable to keep cool and dry
- Includes hook and loop strips both sides

Inner cover:

- Inner cover for extra moisture protection

Foam:

- Molded high resilient polyurethane foam made with Ultra-fresh™ anti-microbial
- Rear cushion radius and waterfall front edge
- Molded polymer insert for extra protection and shear reduction
- Unique waffled relief zone allows immersion in ischial area designed to provide off-loading to hips and thighs for support and stability

Options



Cushion and Cover Modifications:

Standard modifications include seat width and depth reductions, as well as leg length discrepancies



Pelvic Obliquity Kit:



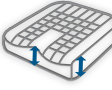



Designed to accommodate up to 1" obliquities.



Cushion Rigidisers:

Lightweight plastic seat insert designed to fit to the bottom of the cushion inside the cover. Sits between seat rails for level support and eliminates hammock effect of sling seat upholstery.

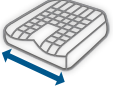Tech Specs

	 Width*	 Depth**	 Height Adductor/Abductor	 Product weight	 Weight Capacity	 Limited Warranty
Paediatric	14"	12, 14, 16"	3.5" Adductor 3.5" Abductor 2.75" Leg Trough	From 900g (2lb)	68kg (150lb)	2 years
Standard	14, 16, 17, 18, 20"	16", 18", 20", 22"	3.5" Adductor 3.5" Abductor 2.75" Leg Trough	from 1kg (2.2lb)	136kg (300lb)	2 years
Heavy Duty	21" - 30"	18" - 30" 40.5 - 76cm	4"/3.5"	From 3kg (7lb)	272kg (600lb)	2 years

* Actual width is 1/2" narrower than stated width

** Actual depth is 1 1/2" longer than stated depth



Yes, you can.®

PS Cushion

SUPERIOR POSITIONING

The Invacare® MatrX® PS Cushion design features an anatomically contoured shape with a unique waffled ischial relief recess and an innovative reversible outer cover. It is a great choice for a very wide range of individual needs, including pediatric, geriatric, bariatric and highly active.



Outer cover:

- Innovative reversible Startex™ outer cover may be used smooth or fabric side out. Hook and loop fastener strips on both sides.
- Startex™ fabric is moisture resistant and breathable

Inner cover:

- Moisture resistant
- Easily wiped clean

Cushion:

- Durable high resilient foam prevents “bottoming out”
- Anatomically molded shape for optimal positioning and comfort
- Waterfall front edge
- Made with Ultra-Fresh™ for antibacterial and odor protection
- Waffled recess provides immersion of body prominences, helps maintain pelvic position and prevents sliding

Options



Cushion and Cover Modifications:

Standard modifications include seat width and depth reductions, as well as leg length discrepancies



Pelvic Obliquity Kit:







Designed to accommodate up to 1° obliquities.



Cushion Rigidisers:

Lightweight plastic seat insert designed to fit to the bottom of the cushion inside the cover. Sits between seat rails for level support and eliminates hammock effect of sling seat upholstery.

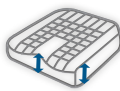Tech Specs

	 Width*	 Depth**	 Height	 Product weight	 Weight Capacity	 Limited Warranty
Paediatric	8-14"	10", 12", 14", 16"	3.75" Adductor 3.5" Abductor 2.5" Leg trough	From 450g (1lb)	68kg (150lb)	2 years
Standard	14, 16, 17, 18, 20"	16, 18, 20, 22"	4" Adductor 3.5" Abductor 2.5" Leg trough	From 900g (2lb)	136kg (300lb)	2 years
Heavy Duty	21" - 30"	18" - 30" 40.5 - 76cm	4"/3.5"	From 3kg (7lb)	272kg (600lb)	2 years

* Actual width is 1/2" narrower than stated width

** Actual depth is 1 1/2" longer than stated depth

E2 Back Series

POSTURAL SUPPORT TO ENABLE INDEPENDENT MOBILITY

Postural Support to enable independent mobility

The new **E2 Back Support** is our latest innovation in seating, offering maximum postural support for those with reduced mobility. Offering effortless mounting, the **E2 Back Support** is available in a wide range of widths and heights to ensure maximum fit. The new leading edge Ripple Foam technology provides temperature control and enhanced immersion for the most comfortable support ever.

Ergonomic Design

The new ergonomic design increases support and pressure distribution to ensure a more precise and comfortable fit. The design has been contoured to support the pelvis and allow clearance of the hips to increase stability and balance.



Back Shell

- ▶ Cutouts for airflow
- ▶ Vertical slots for Hardware attachment
- ▶ Coccyx notch
- ▶ Pre-drilled for PostureFlex Abdominal support



Yes, you can.®

Ripple Foam Technology

The rippled foam surface increases airflow for breathability, temperature control and moisture reduction. The high resilience foam provides enhanced immersion and shear reduction.



Cover

- ▶ Moisture resistant and breathable Meshtex™ outer cover
- ▶ Foam and cover overlap the shell for protection and comfort



Mounting Pin Assembly

The back supports are mounted onto the chair and secured with an easily adjustable mounting pin assembly. The flexible hardware is designed to match the needs of a range of body shapes and sizes, and can even be adjusted when the user is seated. It can also be positioned to avoid frame obstacles - such as arm supports and rigidizer bars etc.



Mounting Pins

- ▶ Height and width adjustable
- ▶ Fits chairs same as model width or 1" wider / narrower
- ▶ Back height can be adjusted even with user seated in back
- ▶ Hardware can be positioned to avoid frame obstacles like arm supports and rigidizer bars
- ▶ MiniSet interface standard on back heights up to 16", EasySet interface standard on 18"-20" back heights

The Matrix E2 Back Support

is available in two contour depths



Product Specifications

- ▶ Width: 14"-20"
- ▶ Height: 6"-20"
- ▶ Weight Capacity: 136kg.

▲ **E2 Back**
E2 Standard - 3" Contour

▲ **E2 Deep Back**
E2 Deep - 6" Contour



Easy Fix Laterals


- ▶ Fixed laterals can be installed inside or outside of the back shell
- ▶ Height adjustable in vertical slot, can be offset
- ▶ Compatible with E2 standard contour backs 26cm and higher



Swing Away Laterals

- ▶ Compatible with E2 backs 10" and higher, offset with backs 31cm and higher
- ▶ Available in 4 pad sizes: XS / S / M / L

Inside Mount Outside Mount

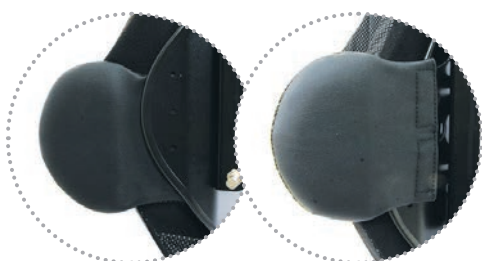

Yes, you can.®

Complete range of accessories ▾



▲ BackBone

Sacro lumbar support targets pelvis to promote an upright posture.

- Contoured anatomical PSIS/Sacrolumbar Support.
- Can be trimmed to desired size.
- Molded in loop fastener for attachment to back shell and ease of adjustment.



▲ Matrx Protect Slip Cover

To protect the backrest. Can be wiped clean with approved disinfectant wipes for greater hygiene. Easy to apply and remove for laundering.



▲ PostureFlex Abdominal Support

Designed to increase intra-abdominal pressure for enhanced voice and breath control. Provides comfortable upright postural support with flexible stretch for functional activities.

Easy keyhole attachment

- 3 mounting heights available in pre-drilled holes on back shell.
- Available in 2 sizes: M / L
- Optional quad release tab for easy independent application and removal of PostureFlex



▲ Headrest Interface Plate

Lightweight and low profile headrest attachment plate can be installed without cover removal. Included with all backs 14" and higher. Optional extra strong steel headrest mount for extreme tone or heavy use applications.



▲ Shoulder Strap Interface Plate

Interface plate for precise placement of Strap Guides.

Available in 2 sizes



▲ Matrx Privacy Flaps

Can be attached to the backrest with a pre-sewn Velcro loop strip located across the bottom rear of cushion cover.

Available in 4 sizes: XS / S / M / L

Full range of hardware mounts ▾

for different applications and greater compatibility with our wheelchair range

Standard Hardware



MiniSet:

- ▶ Standard on all E2/E2D backs up to 16" high



EasySet:

- ▶ Standard on all E2/E2D backs 18-20" high

Optional Hardware



Fixed MiniSet:

- ▶ Permanent mount requires tools for removal, recommended for tilt/recline applications
- ▶ Optional for E2/E2D backs up to 16" high



Fixed EasySet:

- ▶ Permanent mount requires tools for removal, recommended for tilt/recline applications
- ▶ Optional for E2/E2D Deep backs with Easy Set Mounting Pin Interface only (18-20" high)



Heavy-Duty MiniSet:

- ▶ Recommended for users with high tone or heavy use
- ▶ Quick release



Split:

- ▶ Allows clearance of frame obstacles, quick release
- ▶ Optional for E2/E2D backs with Easy Set Mounting Pin Interface only, not compatible with MiniSet



Heavy-Duty EasySet:

- ▶ Recommended for users with high tone or heavy intended use

Adjustments: Angle (40°), depth (2" in front or behind back canes), can be offset to accommodate rotation.

Fits: 1" back canes, inserts for 3/4" and 7/8" back canes included

Build a family of seating products

Invacare also offers a complete range of seating products to complement your E2 back support.



Invacare Matrix cushions

matrx
SEATING SERIES



Invacare Matrix Elan headrests

matrx
SEATING SERIES



Invacare Matrix Accessories

matrx
SEATING SERIES



Yes, you can.®

Elite Back

DESIGNED WITH ACTIVE LIFESTYLES IN MIND

The Invacare® Matrix® Elite Back is designed to provide precise orientation within the wheelchair for optimal postural support. The attractive, lightweight design appeals to individuals with active lifestyles.



Outer cover:

- Oversized cover/foam for extra comfort and protection
- Moisture resistant and breathable
- Includes removable lumbar positioning pad
- Optional privacy flap available

Back shell:

- Stylish, lightweight and breathable aluminium back shell with 3" contour depth.








Hardware:

- "Easy Set" hardware installed and adjusted in minutes – adjustments can be made while user is seated in the wheelchair

Accessories

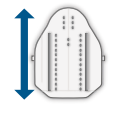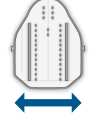

Tech Specs

Elite	 Height 10", 12", 14", 16", 18", 20"	 Width 12" - 20"	 Depth adjustment 2" forwards, 2" backwards	 Angle adjustment 40° range***	 Product weight from 800g (1.7lb)	 Weight Capacity 136kg (300lb)	 Limited Warranty 2 years
HD Elite Heavy Duty	Height 16" - 20"	Width 21" - 24"	Depth adjustment 2" forwards, 2" backwards	Angle adjustment 40° range***	Product weight From 2.7kg (6lb)	Weight Capacity 227kg (500lb)	Limited Warranty 2 years

* 14" height only available up to 18" wide

*** Limitation of depth/angle adjustment if used on chairs narrower than model width



Yes, you can.®

Elite TR Back

IMPROVED IMMERSION AND COMFORT

The Invacare® Matrix® Elite TR is specially designed for use in manual or power tilt only or tilt/recline applications. It includes an extra layer of SuperSoft foam for improved immersion and comfort, and wider support across the shoulder area.



Outer cover:

- Oversized cover/foam featuring SuperSoft HR foam layered over HR foam for comfort and protection.
- Moisture resistant and breathable
- Includes removable lumbar positioning pad
- Optional privacy flap available

Back shell:

- Stylish, lightweight and breathable aluminium back shell with 3" contour depth
- Wider support in shoulder area, and slotted for lateral support hardware attachment.

Hardware:

- "Easy Set" hardware installed and adjusted in minutes - adjustments can be made while user is seated in the wheelchair

Accessories



Tech Specs

Elite TR	Height 16", 18", 20"	Width 16" - 20"	Depth adjustment 2" forwards, 2" backwards	Angle adjustment 40°	Product weight from 1.6kg (3.5lb)	Weight Capacity 136kg (300lb)	Limited Warranty 2 years
Elite TR Heavy Duty	Height 20"	Width 21 - 24"	Depth adjustment 2" forwards, 2" backwards	Angle adjustment 40°	Product weight from 3.9kg (8.5lb)	Weight Capacity 227kg (500lb)	Limited Warranty 2 years

Elite Deep Back

DESIGNED WITH ACTIVE LIFESTYLES IN MIND

The Invacare® MatrX® Elite Deep Back is designed with integrated lateral supports to provide optimal trunk support while allowing clearance for the shoulders and hips. The attractive, light-weight design appeals to individuals with active lifestyles.



Outer cover:

- Oversized cover/foam for extra comfort and protection
- Moisture resistant and breathable
- Includes removable lumbar positioning pad
- Optional privacy flap available

Back shell:

- Stylish, lightweight and breathable aluminium back shell with 6" contour depth

Hardware:

- "Easy Set" hardware installed and adjusted in minutes - adjustments can be made while user is seated in the wheelchair

Accessories



Tech Specs

Elite Deep Back	Height 12", 16", 18", 20"	Width 14" - 20"	Depth adjustment 2" forwards, 2" backwards	Angle adjustment 40° range***	Product weight from 1.3kg (3lb)	Weight Capacity 136kg (300lb)	Limited Warranty 2 years
Elite Deep Back Heavy Duty	Height 16, 20"	Width 21 - 24"	Depth adjustment 2" forwards, 2" backwards	Angle adjustment 40° range***	Product weight From 2.7kg (6lb)	Weight Capacity 227kg (500lb)	Limited Warranty 2 years

* 14" height only available up to 18" wide

*** Limitation of depth/angle adjustment if used on chairs narrower than model width



Yes, you can.®

PB Back

EXCEPTIONAL SUPPORT AND COMFORT

The Invacare® Matrix® PB Posture Backs are contoured to provide midline support and positioning. The extensive range of sizes provides exceptional support and comfort for a variety of customers.



Foam:

- Molded foam wraps around shell for maximum comfort and protection
- Includes removable positioning sacral/lumbar support pad

Outer cover:

- Moisture resistant and breathable
- Optional privacy flap is available

Back shell:

- Contoured shell provides exceptional midline support and positioning
- Durable, lightweight ABS back shell includes 5" contour depth and built-in plate for headrest attachment

Hardware:

- "Easy Set" hardware installed and adjusted in minutes - adjustments can be made while user is seated in the wheelchair

Accessories



Tech Specs

	Height	Width	Depth adjustment	Angle adjustment	Product weight	Weight Capacity	Limited Warranty
PB	12", 16", 20"	14", 16", 17", 18", 20"	2" forwards, 2" backwards	40° range**	From 1.2kg (2.8lb)	136kg (300lb)	2 years
PB Heavy Duty	16", 20"	21", 23", 25"-30"	4"*** 2" forwards, 2" backwards	40° range**	from 2.7kg (6lb)	227kg (500lb)	2 years

** Limitation of depth/angle adjustment if used on chairs narrower than model width

PB Deep Back

EXCEPTIONAL SUPPORT AND COMFORT

The Invacare® Matrix® PB Deep Back is contoured to provide increased midline support and positioning. The extensive range of sizes provides exceptional comfort and support for a wide range of individuals with moderate to high lateral trunk positioning needs.



Foam:

- Molded foam wraps around shell for maximum comfort and protection
- Includes removable positioning sacral/lumbar support pad

Outer cover:

- Moisture resistant and breathable
- Optional privacy flap is available

Back shell:

- Contoured shell provides exceptional midline support and positioning
- Durable, lightweight ABS back shell includes 7" contour depth and built-in plate for headrest attachment

Hardware:

- "Easy Set" hardware installed and adjusted in minutes - adjustments can be made while user is seated in the wheelchair

Accessories



Tech Specs


	Height	Width	Depth adjustment	Angle adjustment	Product weight	Weight Capacity	Limited Warranty
PB Deep	12", 16", 20"	14", 16", 17", 18", 20"	2" forwards, 2" backwards	40° range**	From 1.3kg (2.9lb)	136kg (300lb)	2 years
PB Deep Heavy Duty	16", 20"	20, 22, 24, 26, 28, 30"	2" forwards, 2" backwards	40° range**	from 2.7kg (6lb)	226kg (500lb)	2 years

** Limitation of depth/angle adjustment if used on chairs narrower than model width



Yes, you can.®

Mounting Hardware Options

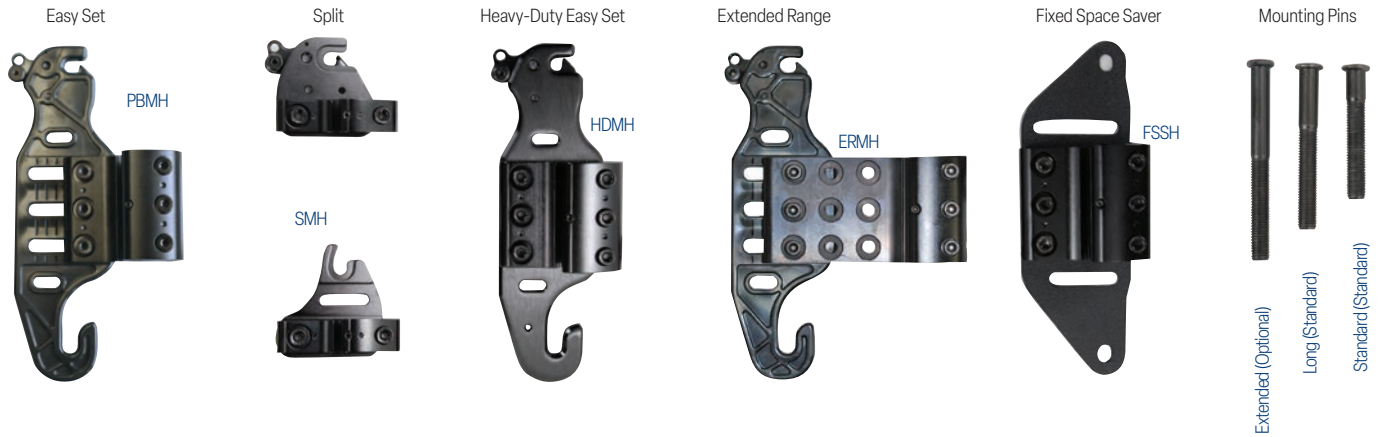

Easy Set (PBMH)

300 lb. weight capacity. Quick release design for easy removal/portability; Allows back angle/depth adjustment with client seated in wheelchair; Compatible with all models of Elite & PB backs.

Split (SMH)

500 lb. weight capacity. Allows clearance of back cane rigidizer bars or other obstacles.

Heavy-Duty Easy Set (HDMH)

500 lb. weight capacity. Included with all Heavy-Duty Back models; Recommended for users with high tone or heavy intended use.

Cane Clamps

The cane clamps are designed to allow installation onto 1" (25mm) 7/8" (22mm) and 3/4" (19mm) back cane diameters, with plastic inserts provided to accommodate the smaller diameter back canes

Fixed Space Saver (FSSH)

500 lb. weight capacity. Recommended for power tilt/recline or tighter width applications. Permanent mount requires tool for removal.

Mounting Pins

Standard/Long Mounting Pins: Included with PB and Elite Backs; allows PB Back to fit 1" narrower to 1" wider and Elite backs the same width or 1" wider wheelchairs.

Optional Extended Mounting Pins (EMP):

Allow interface to wheelchairs up to 2" wider. 300 lb. weight capacity maximum.

Mini Back

DESIGNED WITH KIDS IN MIND

The Invacare® Matrix® Mini Back features Mini Set hardware, a miniature version of the popular Easy Set hardware, designed with kids in mind. The Mini back is contoured to fit kids of all shapes and sizes – combining exceptional support in an appealing low profile package.



Cover:

- Foam and cover overlap the shell for comfort and protection.
- Water resistant and breathable, with additional options of bright pink and blue
- Includes removable lumbar positioning pad

Back shell:

- Light weight, durable aluminum shell, designed for easy attachment of optional lateral supports and headrest.

Hardware:

- “Mini Set” hardware installs and adjusts in minutes. Small attachment point for minimal interference on back canes.

Accessories

Fixed Lateral pads



Mini Swing-away Laterals



Mini-Set Mounting Hardware



Chest Harness Interface



Cover Colours Available




Pink Lollipop Startex



Blizzard Blue Startex

Tech Specs

Mini Back

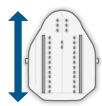

Height
7", 10", 12", 14"*



Width
10" - 14"



Depth adjustment
42" forwards,
2" backwards



Angle adjustment
40° range



Product weight
from 600g (1.3lb)



Weight Capacity
68kg (150lb)



Limited Warranty
2 years



Yes, you can.®

MX2 Back

PEAK PERFORMANCE AND LOOKS GREAT

The Invacare® Matrx® MX2 Back is the latest in back support technology. Designed for active users who demand the best, it combines aesthetic appeal with superior support and function. Lightweight carbon fibre is specially engineered to provide unmatched stability for peak performance.



Cover:

- Breathable Spacer Mesh fabric, 9" and higher backs include leatherette side panels
- Cover and foam overlap back shell for privacy, protection and comfort
- Includes removable lumbar positioning pad
- Optional privacy flap available

Carbon Fiber Back Shell:

- Exceptional durability and aesthetic appeal
- Very light weight
- 3" Contour to provide stability to hips/trunk without restricting upper body mobility








Hardware:

- Choice of Fixed Mount or Quick Release Hardware. Hardware does not come standard with the MX2 back, this will need to be selected.

Accessories

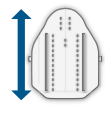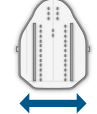

Tech Specs


<p>MX2 Back</p>	 <p>Height 6, 9, 12, 16"</p>	 <p>Width 14 - 18"</p>	 <p>Depth adjustment 2" Fixed HW 4" Quick-Release HW</p>	 <p>Angle adjustment 30°</p>	 <p>Product weight from 680g (1.5lb)</p>	 <p>Weight Capacity 114kg (250lb)</p>	 <p>Limited Warranty 2 years</p>
------------------------	----------------------------------------------------------------------------------------------------------------------------	----------------------------------------------------------------------------------------------------------------------	------------------------------------------------------------------------------------------------------------------------------------------------------------	----------------------------------------------------------------------------------------------------------------------------	------------------------------------------------------------------------------------------------------------------------------------------	---------------------------------------------------------------------------------------------------------------------------------------	----------------------------------------------------------------------------------------------------------------------------------

Dipped Back Shell Options

Now Available
in a variety of
designs to suit your
lifestyle!



Here's just a few
examples:

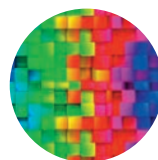

See the order form for the full range of options.

Available on standard elite and mini backs only.



Yes, you can.®


Elan Headrest

TAKES HEAD SUPPORT TO THE NEXT LEVEL

The Invacare® Matrix® Elan® Headrest range takes head support to the next level. A new elegance in head support and comfort, the Elan Headrest range is compact, lightweight and highly adjustable.



- **Elegant** – Clean, compact, streamlined aesthetics.
- **Adjustable** – Elan headrest hardware has six points of adjustment – infinite positioning opportunities to meet user needs.
- **Easy** – Single tool set up. Optional tool-less adjustment levers provided for ease of position changes.
- **Reliable** – Multi-disk locking provides superior holding strength.

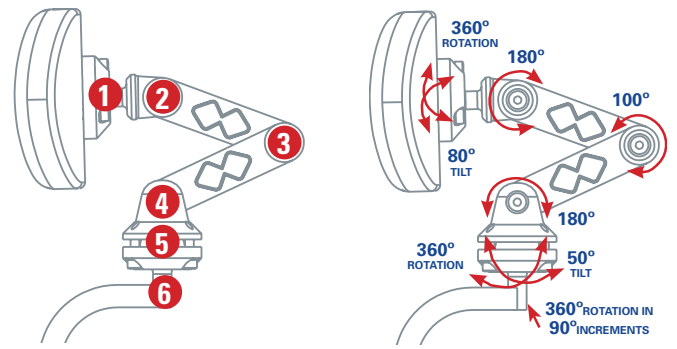

Pad Features



- **Comfortable** – Dual density foam provides immersion and prevents bottoming out.
- **Durable** – Foam pad surrounds the backing plate and can be easily removed for modification. Pads are made with our Ultra-Fresh anti-microbial additive.
- **Lightweight** – 45% Lighter than competitive pads.
- **Versatile**
 - Standard, occipital and multi-adjustable 4-point versions available.
 - Aluminum plates can be formed to desired shape.
- **Covers**
 - Zippered, moisture-resistant, breathable Startex fabric
 - Infection Control covers available as an accessory.

Tech Specs

Model Description	Product	Dimensions Weight x Height	Limited Warranty
ESP6	6" Standard Pad	6" x 3.5"	2 Years
ESP10	10" Standard Pad	10" x 5"	2 Years
ESP14	14" Standard Pad	14" x 5"	2 Years
EOP9	9" Occipital Pad	9" x 7"	2 Years
EOP12	12" Occipital Pad	12" x 8"	2 Years
E4PTSM	4 Point Pad (Small)	9" x 7"	2 Years
E4POINT	4 Point Pad	11" x 10"	2 Years
MEHW	Elan Standard Hardware	N/A	2 Years
LXSHW	LOXX Multi-Axis Hardware	N/A	2 Years

Accessories



Fixed Laterals

Designed for Invacare Matrix Elite & Elite Heavy-Duty Backs. Mounting: Installs inside back shell; Does not compromise depth/angle adjustment.

Options: Standard/ Pediatric/ Offset (Left or Right)

Sizes:

Standard 7.5" L x 5" H
Pediatric 5.5" L x 3" H
Offset 7.5" L x 5" H



Swing Away Laterals

Designed for Invacare Matrix Elite and Elite Heavy-Duty Backs. Mounting: Installs inside back shell; Does not compromise depth/angle adjustment. Lock/release latch allows pad to swing away for side transfers.

Options: Standard or Offset (Left or Right)

Pad Sizes:

4.25" L x 3.25" H
5.25" L x 3.5" H
5.75" L x 3.75" H
6.5" L x 4.25" H



Privacy Flaps

Available for all models of Invacare Matrix PB, Elite (shown), and MX2 backs.

Standard Sizes: XS up to XXL, (fit back widths 13" up to 24")

Note: Flaps for back widths greater than 24" wide are available as special order only



Chest Harness Interface for Elite Backs

Width, height and angle adjustable chest harness interface for precise orientation of shoulder straps.



Chest Harness Interface for PB Backs

Width adjustable chest harness interface for precise orientation of shoulder straps.



Bodypoint Padded Lap Belts

Available in 2 and 4 Point; Central pull, push release bottom; fits hip widths 13" - 19"



Bodypoint Aeromesh Calf Straps/Panels

Available in 4 sizes:

3.5" H and fits chairs 15" - 18" W
3.5" H and fits chairs 18" - 22" W
9" H and fits chairs 15" - 18" W
9" H and fits chairs 18" - 22" W



Ergonomic Arm Trough

Molded, ergonomic shape fits the natural arm curve; provides users flexibility of various arm positions Available for left or right side and is 13" x 4.5"; hand pad is optional add-on and is also available for left or right side and is 5" x 4.5"



Gel Arm Pads

Available for left or right side and in 3 sizes:
Narrow: 12" x 2"
Wide: 12" x 3"
Long: 14" x 3.5"



Multi-Axis Armrest Mount

Allows infinite adjustability of angle and rotation of arm pad or Multi-position arm pads or Arm Trough.



Modular Arm Pads

Durable composite frame features 3 slots for infinite fore and aft adjustability as well as offset mounting. Available for left or right side and in 2 sizes:
Full length: 14" x 4"
Desk length: 10" x 4"
Available in 2 interchangeable foam inserts: Visco Foam/Star-tex and Integral Foam



Yes, you can.®

FOR FURTHER INFORMATION:

New Zealand: Freephone 0800 468 222, email sales@invacare.co.nz or visit www.invacare.co.nz

Australia: Freephone 1800 460 460, email orders@invacare.com.au or visit www.invacare.com.au

All rights reserved. All information quoted is believed to be correct at time of publication.
Invacare reserves the right to alter product specifications without prior consent. **05/2024**



Yes, you can.[®]